Aquatec®

Orca / Orca F / Orca XL

EN	Bathlift	<i>User Manual</i>
DE	Badewannenlifter	<i>Gebrauchsanweisung</i>
FR	Elévateur de bain	<i>Manuel d'utilisation</i>
IT	Sollevatore da vasca	<i>Manuale d'uso</i>
ES	Aparato elevador de bañera	<i>Manual del usuario</i>
NL	Badlift	<i>Gebruiksaanwijzing</i>
CS	Zvedák do vany	<i>Návod k použití</i>
PT	Elevador de banho	<i>Manual de instruções</i>
SV	Badkarslyft	<i>Bruksanvisning</i>
NO	Badekarløfter	<i>Brukerveiledning</i>
DA	Badekarslift	<i>Brugsanvisning</i>

This manual must be given to the user of the product. Before using the product, read this manual and save it for future reference!

AQUATEC®

Yes, you can.

© *Aquatec* Operations GmbH

Every effort has been made to ensure that the contents of this publication are fully up-to-date at the time of going to print.

As part of its ongoing improvement of products, *Aquatec* Operations GmbH reserves the right to modify existing models at any time.

Aquatec dealers will be notified of any such modifications. Any use of this publication, or of parts thereof, as well as any reproduction of images, must have the written consent of *Aquatec* Operations GmbH.

1	General information	4	6	Transport	12
1.1	General information	4	6.1	Safety information	12
1.2	Symbols	4	7	Maintenance	12
1.3	Warranty	4	7.1	Maintenance and servicing	12
1.4	Intended use	4	7.2	Cleaning and disinfection	12
1.5	Product service life	4	8	After use	13
2	Safety	5	8.1	Storage	13
2.1	Safety information	5	8.2	Re-use	13
2.2	Safety information on electromagnetic compatibility	5	8.3	Disposal	13
3	Description	5	9	Troubleshooting	14
3.1	Seat	5	9.1	Identifying and repairing faults	14
3.2	Backrest	6	10	Technical data	15
3.3	Hand control and charger	6	10.1	Dimensions and weight	15
3.4	Identification label	6	10.2	Electronic data	15
4	Set up and installation	7	10.3	Materials	15
4.1	Safety information	7			
4.2	Setting up the bath lift	7			
4.3	Removing the bath lift	8			
4.4	Charging the battery	9			
5	Operation	10			
5.1	Safety information	10			
5.2	Controlling the bath lift	10			
5.3	Bathing	11			

I General information

I.1 General information

This user manual contains information and notes on the proper and safe use of the bath lift.

Please read this user manual carefully before using the product. Please observe all the notes, particularly the safety information, and act accordingly.

The position specifications (A, B, C, etc.) in the handling instructions always refer to the preceding figure.

I.2 Symbols

Symbols used in this user manual

In this user manual warnings are indicated by symbols. The warning symbols are shown next to a title which indicates the level of danger.

WARNING

Indicates a dangerous situation which could lead to severe injury or even death if not heeded.

CAUTION

Indicates a dangerous situation which could lead to minor injuries if not heeded.

IMPORTANT

Indicates a dangerous situation which could lead to damage if not heeded.

Useful tips, recommendations and information to ensure efficient and smooth operation.

Complies with Directive 93/42/EEC concerning medical devices. The market launch date for this product is indicated in the EU declaration of conformity.

I.3 Warranty

We provide a manufacturer's warranty for the product in accordance with our General Terms and Conditions of Business. Warranty claims can only be made through the relevant dealer.

Covers and suction cups are excluded from this.

Quality and electromagnetic compatibility

Quality is of the utmost importance for our company; all processes are based on the ISO 9001 and ISO 13485 standards.

The product meets the requirements of the standards, DIN EN 10535 (lifts), DIN EN 60601-1 (electrical safety) and DIN EN 60601-1-2 (electromagnetic compatibility).

It has a lift capacity greater than 1.5 times the nominal load.

Should you require further information, please contact the relevant Invacare® subsidiary in your country (contact addresses are provided on the back of this user manual).

I.4 Intended use

The bath lift is intended exclusively as an aid for bathing people within a bathtub. Do not use for any other purpose.

WARNING Risk of injury

▶ Never stand on the bath lift to get into or out of the bathtub, do not use it for climbing or descending, as a lifting platform or for any similar purpose.

Operating temperature: 10 - 40 °C

I.5 Product service life

If the product is used as intended and in accordance with the safety regulations, the expected product life is 5 years. The effective service life can vary, however, depending on the frequency and intensity of use.

2 Safety

2.1 Safety information

WARNING Risk of injury

- ▶ Do not use a faulty product.
- ▶ In the event of a malfunction, contact a dealer immediately.
- ▶ Do not make any unauthorized changes or alterations to the product.
- ▶ Only use accessories intended for the Aquatec® Orca bath lift.

CAUTION Danger of pinching fingers

- ▶ Do not insert fingers into the guide between the seat plate and the backrest.

2.2 Safety information on electromagnetic compatibility

This product was successfully tested to international standards for its electromagnetic compatibility. However, electromagnetic fields such as those generated by radio and television transmitters, radio equipment and mobile telephones, can influence the operation of products with electrical drives. The electronics used in this product can also cause weak electromagnetic interference but this lies below the statutory limits. You should therefore observe the following instructions:

CAUTION Electromagnetic interference

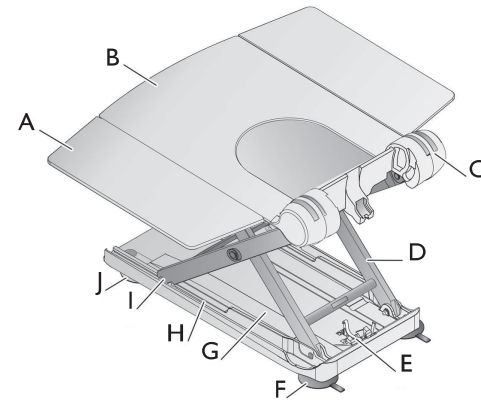
Electromagnetic interference could cause the bath lift to stop suddenly or (though this is extremely unlikely) to start moving of its own accord.

- ▶ Switch off the external device and/or your bath lift if you notice such behavior.
- ▶ If other electrically operated medical products cause a interference or are themselves disrupted, switch off the bath lift and contact your retailer.

3 Description


The following components are included within the scope of delivery of the Aquatec® Orca/Orca F/Orca XL:

3.1 Seat


A	Side flap
B	Seat plate
C	Hinge for backrest
D	Lifting scissors
E	Spindle foot retainer
F	Rear suction cups (two tabs with thread)
G	Base plate
H	Guide for the sliding shoe
I	Sliding shoe
J	Front suction cups (one tab with head)

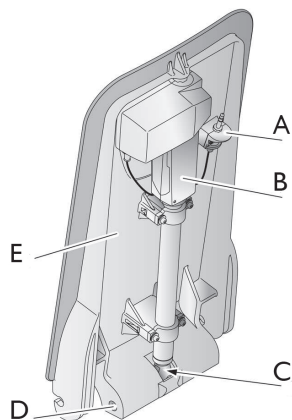3.2 Backrest

A	Jack plug
B	Drive
C	Spindle foot
D	Locking pin
E	Plate, backrest

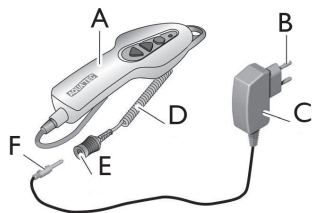
3.3 Hand control and charger

A	Hand control with battery
B	Mains plug
C	Charger
D	Hand control spiral cable
E	Jack socket of hand control
F	Jack plug of charger

3.4 Identification label

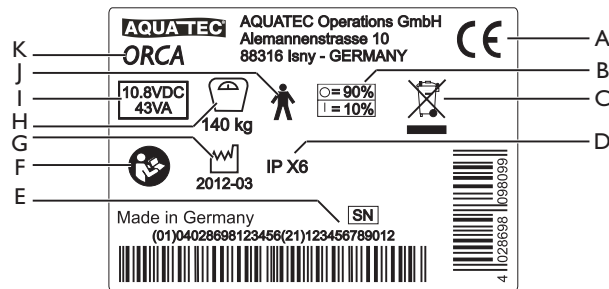The identification label also contains important information:

A	Conformity mark
B	Duty cycle (max. 10%, max. 2 min/18 min)
C	Note (disposal)
D	Type of protection
E	Serial device number
F	Note (observe accompanying documents)
G	Production timeframe
H	Load capacity
I	Nominal voltage/ rated power
J	Type B applied part
K	Device designation

The identification label is applied to the back of the backrest.

4 Set up and installation

4.1 Safety information

! IMPORTANT

- ▶ Check the parts for transport damage before commissioning and contact the dealer if required.
- ▶ Ensure during assembly that the parts are positioned correctly with respect to one another.

4.2 Setting up the bath lift

WARNING Risk of slipping!

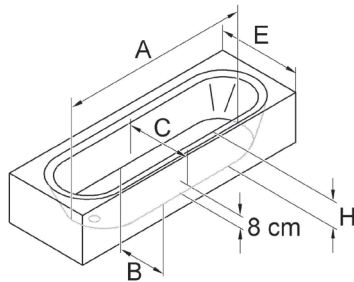
The bath lift can slip if the suction cups do not have a good hold.

- ▶ Clean the bathtub and the suction cups before setting up the bath lift.
- ▶ Do not change the position of the bath lift in the bathtub once it is set up.

WARNING Risk of injury and damage!

- ▶ Only set up the bath lift in bathtubs that comply with the specifications of the following table.


A	Tub length, bottom	≥ 800 mm
B	Tub width, bottom (inside)	280 - 600 mm
C	Tub width, at 80 mm height (inside)	≥ 430 mm
H	Tub height (inside)	≤ 420 mm (with height adapter, max. + 60 mm)
E	Tub width (outside)	580 - 690 mm (with special side flaps) 700 - 740 mm (with standard side flaps) 750 - 1000 mm (with special side flaps)


IMPORTANT

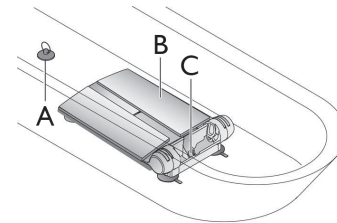
- ▶ Ensure that the suction cups are mounted.
- ▶ Ensure that the battery in the hand control is charged.
- ▶ Ensure that the spindle foot is fully inserted.
- ▶ When lifting, please remember: the seat weighs 7.5 kg, the backrest 4.6 kg (without the hand control).
- ▶ If the bathtub has a handrail, use the side flap diverters.
- ▶ Fold the side flaps outwards before raising the backrest.

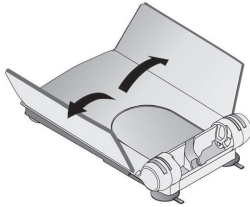1. Place the seat (B) on the bottom of the empty bathtub. Ensure that the spindle foot retainer (C) is pointing towards the end of the bathtub opposite the drain (A).

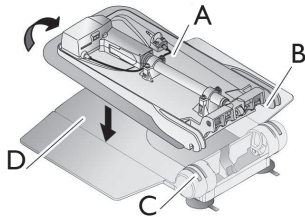2. Fold the side flaps of the seat outwards.

3. Place the backrest (A) (without the hand control) onto the seat so that the hooks (B) of the backrest fit into the hinges (C) of the seat plate (D).

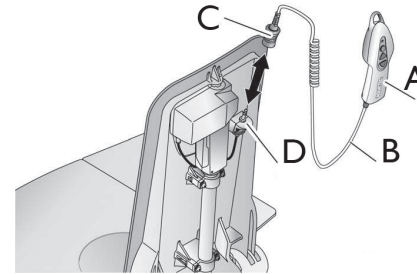
IMPORTANT Risk of damage!

Damage to the product caused by the spindle foot slipping out.

- Ensure that the spindle foot is fully retracted, before folding up the backrest.
- Once the backrest is folded up, ensure that the spindle foot is in the spindle foot retainer.

4. Hold the backrest at the center of the head end and carefully lift it into the end position.

5. Plug the jack socket with the unlocking sleeve (C) on the spiral cable (B) of the hand control (A) to the jack plug (D) until the locking mechanism engages noticeably.

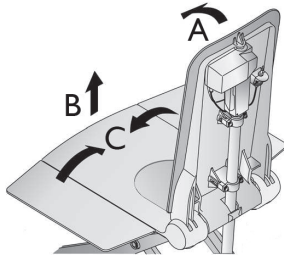
4.3 Removing the bath lift

1. Let the water out of the bathtub.
2. Lower the bath lift using the Down button until the backrest stops in the lowered end position. See Section 5.2 Controlling the bath lift.
3. Unlock the jack socket on the spiral cable of the hand control by pushing up the unlocking sleeve and pull it off the jack plug.

IMPORTANT Risk of damage!


- When folding the backrest down, ensure that spiral cable of the hand control does not become trapped.
- Do not fold the backrest down if there is water in the bathtub.

4. Hold the backrest at the head end with one hand and fold it forwards to the end position (A).
5. Pull the backrest up off the seat and out of the bathtub (B).
6. Fold the side flaps of the seat inwards (C).

! **IMPORTANT**

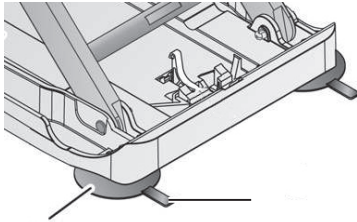- Release both rear suction cups simultaneously.

7. Release the rear suction cup (A) by pulling up on its tab (B).
8. Lift the rear of the seat further. This releases the front suction cups.

! **IMPORTANT**

- If the height adapters have been installed (optional), you also have to pull the tabs of the front suction cups to release them.

9. Lift the seat out of the bathtub.
10. Carefully store all components of the bath lift.

4.4 Charging the battery

The bath lift cannot be used when the battery is charging.

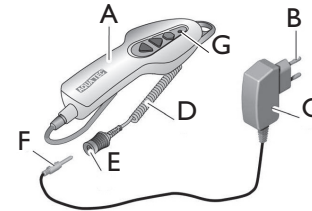
CAUTION Risk of electric shock!

- Only recharge the battery in dry rooms and never in the bathroom.

IMPORTANT

- Only use the charger supplied (Aquatec® Orca/Orca F/Orca XL) to charge the battery in the hand control.
- Only connect the charger to the mains when the hand control is connected to the charger.
- Make sure that the voltage specifications shown on the charger correspond to those of the mains supply.

1. Insert the jack plug (F) of the charger (C) into the jack socket (E) on the spiral cable (D) of the hand control (A) until it engages.
2. Insert the mains plug (B) of the charger into a power socket.

The red indicator lamp (G) on the hand control extinguishes when the battery is completely charged. It is not possible to „overcharge“ the battery.

3. Disconnect the charger from the power socket.
4. Pull the jack plug of the charger out of the jack socket of the hand control.

5 Operation

5.1 Safety information

CAUTION

Overloading the bath lift can lead to damage to the device or prevent lifting.

- Do not overload the bath lift and observe the specifications on the identification label and in Section 10 Technical data with regard to its load capacity.

IMPORTANT

- Always perform a visual inspection of the product for external damage before each use.
- Recharge the battery completely before each use.

5.2 Controlling the bath lift

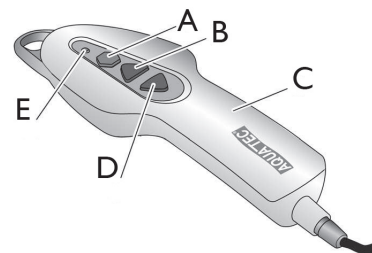
The bath lift can be moved in the following directions:

The bath lift is controlled via the hand control.

With a fully charged battery, the ORCA and ORCA F can perform 5 bathing operations with a load of 140 kg. The ORCA XL can perform 4 bathing operations with a load of 170 kg.

IMPORTANT Risk of damage

- The hand control may only be used when the backrest is properly installed and in the upright position.

IMPORTANT

If the battery in the hand control (C) is not sufficiently charged, the red indicator lamp (E) lights up red when the Down button (D) is pressed. In this case, the lowering function of the bath lift is disabled. You can however still raise the lift.

- Recharge the battery immediately after raising it.

Raising the bath lift

IMPORTANT

If the backrest is lowered, it will move into the upright position first (not on model ORCA F).

1. Press the Up button (B) on the hand control and keep it pressed.

The backrest moves into the upright position or the seat moves upwards.

2. Release the Up button on the hand control.

The bath lift stops in the current position.

Lowering the bath lift

IMPORTANT

If the seat is already lowered, the backrest lowers towards the rear (not on model ORCA F).

3. Press the Down button (D) on the hand control and keep it pressed.

The seat moves downwards or the backrest swivels to the rear.

4. Release the Down button on the hand control.

The bath lift stops in the current position.

Emergency off switch

! IMPORTANT

Only press the Emergency Off button (A) if the device does not stop immediately when the Up or Down button is released (e.g. if a button jams). The bath lift stops moving immediately and the red light integrated in the hand control lights up.

If the red indicator light goes out after the emergency off button is released, the fault has been rectified (e.g. button is released again). The device is ready to use again.

If the red indicator light stays on after the emergency off button is released, the fault has not yet been rectified. The device is not ready for use and must be repaired, e.g. in the event of damage to the electronics.

1. Please contact your dealer, if necessary.

5.3 Bathing

WARNING Risk of slipping!

The bath lift can slip if the suction cups do not have a good hold.

- ▶ Before using the bath lift ensure that it is securely positioned and cannot tip over.
- ▶ Ensure that the bath lift is set up in the bathtub as described in Section 4.2.

WARNING

- ▶ If your condition means that you cannot take a bath on your own, only use the device with supervision.

CAUTION

Bath oils or salts can impair the performance of the lifting scissors.

- ▶ If using such bath additives, do not exceed the recommended dose.

IMPORTANT

- ▶ Run the unloaded lift once to the top and then back to the bottom (function test).
- ▶ Before using, check the temperature of the bath water.

1. Press and hold the Up button (B) until the seat is level with the edge of the bathtub.
2. Make sure that the side flaps are level with the seat and rest flat on the edge of the bathtub.
3. Fill the bath with water.
4. Sit on the side flap nearest to you as you would on a chair.
5. Slide backwards until you are sitting in the middle of the seat.
6. Turn yourself into the bathing position, by lifting one leg and then the other over the edge of the bathtub.
7. Once you are sitting correctly, press the Down button (D) and hold it down to lower the seat into the bathtub.

Once the seat has reached the lowest position, it is possible to recline the backrest by continuing to hold the Down button (D). This produces a more comfortable bathing position and allows you to immerse yourself deeper in the bathwater (not on model ORCA F).

8. Keep the Down button pressed until the backrest has reached the desired position.
- To get out of the bathtub, following these steps in reverse order.
9. To raise the backrest or the bath lift, press the Up button (B) and keep it pressed until it has reached the desired position.

6 Transport

6.1 Safety information

CAUTION

The backrest is not locked on the seat when it is folded together and can come apart.

- Only transport the product at temperatures between 0°C and 40°C, atmospheric pressure of 860 - 1060 hPa and a humidity of 30 - 75%.
- Do not remove the suction cups when transporting the bath lift.
- Remove the hand control from the backrest for transport.

1. Remove the hand control.
2. Disassemble the bath lift into two sections for transport (seat and backrest). See Section 4.3 Disassembling the bath lift.

7 Maintenance

7.1 Maintenance and servicing

The UK Health and Safety Executive's Lifting Operations and Lifting Equipment Regulations 1998 (LOLER), require any equipment that is used in the workplace (e.g. by a paid carer) to lift a load be subject to safety inspection on a six monthly basis. Please refer to the HSE web site for guidance www.hse.gov.uk. Link to HSE information document on LOLER in Health and Social Care: <http://www.hse.gov.uk/pubns/hsis4.pdf>

If the above cleaning and safety notices are observed, the product will be maintenance free.

IMPORTANT

The hand control may only be opened by the manufacturer of the bath lift.

- Check the product regularly for damage and to ensure that it is securely assembled.
- In the event of damage and/or necessary repairs, contact the dealer immediately.

- The bath lift must be checked by an authorized dealer before every re-use or after 24 months at the latest (see Manufacturer's recommendation for rehab products of the SPECTARIS industrial association).
- If the bath lift is not used for a prolonged period, charge the battery completely and then raise and lower the bath lift 3 times without load. Repeat this process every 12 months. This helps to maintain the service life of the battery.

7.2 Cleaning and disinfection

The product can be cleaned and disinfected using commercially available agents.

- Disinfect the product by wiping down all generally accessible surfaces with disinfectant (after disassembling the product, if possible).

IMPORTANT

All cleaning agents and disinfectants used must be effective, compatible with one another and must protect the materials they are used to clean.

- For further information on decontamination in Healthcare Environments, please refer to 'The National Institute for Clinical Excellence' guidelines on Infection Control www.nice.org.uk/CG002 and your local infection control policy.
- Clean the product regularly by hand.
- Do not use abrasive cleaning agents.

Cleaning the seat and backrest

CAUTION Danger of pinching fingers

- Be especially careful when cleaning in the area of the lifting scissors, as they may move and cause injury.

IMPORTANT Risk of damage

- Never use a high-pressure cleaner to clean the backrest.

1. Wash the product using a cloth or a brush.
2. Rinse the product with warm water.
3. Dry the product with a cloth.
4. Lightly grease the guide for the sliding shoe (Fig. Sec. 3.1, H) using Vaseline.

Cleaning the covers (optional)

1. Unbutton the covers from the seat and backrest.
2. Wash the covers in a washing machine at max. 60°C using a mild detergent.

! **IMPORTANT** Risk of damage

- Washing at high temperatures can cause shrinkage.
- Do not dry covers in the tumble dryer.

Cleaning the charger and hand control

WARNING Risk of electric shock!

- Always disconnect the plug from the power socket before cleaning the charger.
- Never insert objects made of conductive material (e.g. knitting needles, metal pins) into the jack socket of the hand control.
- Never use a wet or damp cloth to clean the charger.

1. Clean the charger with a dry cloth.
2. Wipe the hand control with a damp cloth and dry it with a dry cloth.

8 After use

8.1 Storage

! **IMPORTANT** Risk of damage

- Store the product in a dry area at a temperature of between 0°C and 40°C, atmospheric pressure of 860 - 1060 hPa and a humidity of 30 - 75%.
- Do not store the product near heat sources.
- Do not expose the product to direct sunlight.

8.2 Re-use

The product is suitable for re-use. The number of re-uses depends on the frequency of use and the manner in which the product was used. Before re-use, the product should undergo hygienic preparation and be checked to ensure it is in proper and safe working order. Repair if necessary.

8.3 Disposal

The disposal and recycling of used products and packaging must be conducted in accordance with the applicable statutory regulations.

WARNING

- Never throw the hand control into a fire or store it near naked flames.

Duty to inform in accordance with German Battery Directive, BattG:

This product contains a rechargeable lithium-ion battery. Discharged batteries or those which can no longer be recharged must not be disposed of in domestic waste. Old batteries may contain hazardous substances which could be a danger to the environment and public health.

Please return such batteries to your retailer or to your municipal collection point. Returning them is free of charge and is required by law.

Please only throw discharged batteries into the containers provided and mask the poles of lithium batteries.

9 Troubleshooting

9.1 Identifying and repairing faults

The following table provides information on malfunctions and their possible causes.

If you cannot correct the malfunction using the remedies named, please contact your dealer directly.

Fault	Possible cause	Remedy
No motor noise audible/lift does not move.	Jack socket of the hand control is not locked correctly.	Lock the jack socket, ▸ see Section 4.2, Step 5.
	Battery is flat.	Recharge the battery completely, ▸ see Section 4.4.
	Contacts are dirty.	Clean the contacts, ▸ contact your dealer, if necessary.
	Cable is crushed/damaged.	Replace the hand control, ▸ contact your dealer, if necessary.
The lift can be raised but not lowered.	Low-voltage protection is activated	Recharge the battery completely, ▸ see Section 4.4.
The lift stops while being raised.	Excessive load.	Observe maximum load, ▸ see Section 10.1.
The bath lift is not adhering securely to the bathtub.	Suction cups are old or damaged.	Replace the suction cups, ▸ contact your dealer, if necessary.
Battery is not charging and the red indicator light flashes during charging	Faulty electronics.	▸ Have your authorized dealer check the hand control.
	Faulty battery.	▸ Replace the hand control.
	Faulty charger.	▸ Have your authorized dealer check the charger.
	Battery is too hot	▸ Do not charge the hand control near heat sources, e.g. heating elements, in direct sunlight. Charging continues automatically once the temperature drops to within permissible limits.

10 Technical data

10.1 Dimensions and weight

Total length at lowest position	920 mm
Total length at highest position	650 mm
Total height at highest position	1075 mm
Seat length	590 mm
Seat width with side flaps folded out	715 mm
Seat width with side flaps folded in	420 mm
Seat height at lowest position	60 mm
Seat height at highest position	420 mm
Backrest width	360 mm
Minimum angle of backrest	8°
Base plate width	295 mm
Max. load (ORCA/ORCA F)	140 kg
Max. load (ORCA XL)	170 kg
Total weight without hand control	12.1 kg
Seat weight	7.5 kg
Backrest weight	4.6 kg
Hand control weight	0.50 kg
Charger weight	0.1 kg

10.2 Electronic data

Activation force for the buttons of the hand control	< 5 N
Hand control operating voltage	10.8 V DC
Rated input power	43 VA
Rated current	4 A
Battery type	Lithium-ion
Charger input	100 - 240 V AC, 50 - 60 Hz
Charger output	24 V DC, 135 mA, 3 VA
Charge time from low-voltage warning	approx. 4.5 h
Type of protection, drive	IP X6 W ¹⁾
Type of protection, hand control	IP 67 ²⁾
Type of protection, entire device	IP X6 ³⁾
Sound power level	52 dB(A)

¹⁾ Protected against the penetration of water in the event of strong jets of water, tested under stipulated weather conditions.

²⁾ Dust-proof and protected against the penetration of water during temporary immersion.

³⁾ Protected against the penetration of water in the event of strong jets of water.

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Invacare Australia Pty Limited
AQUATEC Operations GmbH