

# Spare Parts Catalogue

Version: 2014/10/22b NO





**Yes, you can.®**

## **General Information**

*For any order of spare parts the following information is required:*

- Product type*
- Product name & model*
- Serial Number*
- Part-Number*
- Part-Description*

*Exploded view drawings and code numbers are not contractual and subject to modification, in relation to Invacare® continuous product improvement policy to meet end users needs.*

*The reproduction or use of this work, or of any substantial part of this work, in any form or by any electronic, mechanical or other means, including photocopying, scanning and recording, and in any information storage and retrieval system, including electronic transmission, is forbidden without the prior written permission of Invacare® International Sarl.*



**Yes, you can.®**

## **TABLE OF CONTENTS**

---

### **(Invacare® SoftAIR Excellence Complete Mattress & Pump)**

(Invacare® SoftAIR Excellence Complete Mattress & Pump) .....	5
(Invacare® SoftAIR Excellence Complete Mattress & Pump) .....	6
(Invacare® SoftAIR Excellence Complete Mattress & Pump EU Part Numbers) .....	7

### **(Invacare® SoftAIR Excellence Mattress Spares)**

(Invacare® SoftAIR Excellence Mattress Spares) .....	8
(Invacare® SoftAIR Excellence Mattress Spares) .....	9
(Invacare® SoftAIR Excellence Mattress Spares EU Part Numbers) .....	10

### **(Invacare® SoftAIR Pump Spares)**

(Invacare® SoftAIR Pump Spares) .....	11
(Invacare® SoftAIR Pump Spares) .....	12



(Invacare® SoftAIR Excellence) | (Invacare® SoftAIR Excellence Complete Mattress & Pump)  
**(Invacare® SoftAIR Excellence Complete Mattress & Pump)**



---

1	-	(Invacare® SoftAIR Excellence Complete Mattress & Pump)	1
2	-	(Invacare® SoftAIR Excellence Complete Mattress & Pump EU Part Numbers)	1

## (Invacare® SoftAIR Excellence Complete Mattress & Pump)



---

1	1539024	(Complete SoftAIR Excellence Mattress and Control Unit System. Width 830mm)	1
2	1539023	(Complete SoftAIR Excellence Mattress and Control Unit System. Width 880mm)	1
3	1539025	(Control unit SoftAIR with power cord and user manual.)	1
4	1546982	(Mattress SoftAIR Excellence width 830mm in tote bag without control unit.)	1
5	1546983	(Mattress SoftAIR Excellence width 880mm in tote bag without control unit.)	1

## (Invacare® SoftAIR Excellence Complete Mattress & Pump EU Part Numbers)



---

1	1532813	(Complete SoftAIR Excellence Mattress and Control Unit System. Width 830mm)	1
2	1532814	(Complete SoftAIR Excellence Mattress and Control Unit System. Width 880mm)	1
3	1532858	(Control Unit SoftAIR with power cord and user manual.)	1
4	1532861	(Mattress SoftAIR Excellence width 830mm in tote bag without control unit.)	1
5	1532862	(Mattress SoftAIR Excellence width 880mm in tote bag without control unit.)	1

**(Invacare® SoftAIR Excellence Mattress Spares)**



---

1	-	(Invacare® SoftAIR Excellence Mattress Spares)	1
2	-	(Invacare® SoftAIR Excellence Mattress Spares EU Part Numbers)	1



## (Invacare® SoftAIR Excellence Mattress Spares)



---

1	1547076	(Cushion SoftAIR Excellence foot width 830mm.)	1
2	1532868	(Cushion SoftAIR Excellence body width 830mm.)	1
3	1532947	(Cushion SoftAIR Excellence head width 830mm.)	1
4	1547077	(Cushion SoftAIR Excellence foot width 880mm.)	1
5	1532949	(Cushion SoftAIR Excellence body width 880mm.)	1
6	1532950	(Cushion SoftAIR Excellence head width 880mm.)	1
7	1550156	(Handle Kit Complete, 4 Hole with connector plate, Grey.)	1
8	1532822	(Coversheet, Width 830mm.)	1
9	1532857	(Coversheet, Width 880mm.)	1
10	1547078	(Side Bolster, Left)	1
11	1547079	(Side Bolster, Right.)	1
12	1533365	(Tote Bag Invacare SoftAIR)	1
13	1554742	(CPR Release)	1
14	1557976	(Base Cover 830mm.)	1
15	1557975	(Base Cover 880mm.)	1
16	1576874	(Connector 3way T-type POM 3/8" _1/2" _1/2")	1
17	1576875	(Connector 3way Y-type PP 3/8")	1

## (Invacare® SoftAIR Excellence Mattress Spares EU Part Numbers)




---

1	1532867	(Cushion SoftAIR Excellence foot width 830mm.)	1
2	1532868	(Cushion SoftAIR Excellence body width 830mm.)	1
3	1532947	(Cushion SoftAIR Excellence head width 830mm.)	1
4	1532948	(Cushion SoftAIR Excellence foot width 880mm.)	1
5	1532949	(Cushion SoftAIR Excellence body width 880mm.)	1
6	1532950	(Cushion SoftAIR Excellence head width 880mm.)	1
7	1550156	(Handle Kit Complete, 4 Hole with connector plate, Grey.)	1
8	1532822	(Coversheet, Width 830mm.)	1
9	1532857	(Coversheet, Width 880mm.)	1
10	1532951	(Side Bolster, Left.)	1
11	1532952	(Side Bolster, Right.)	1
12	1533365	(Tote Bag Invacare SoftAIR)	1
13	1554742	(CPR Release)	1
14	1557976	(Base Cover 830mm.)	1
15	1557975	(Base Cover 880mm.)	1

## (Invacare® SoftAIR Pump Spares)



## (Invacare® SoftAIR Pump Spares)

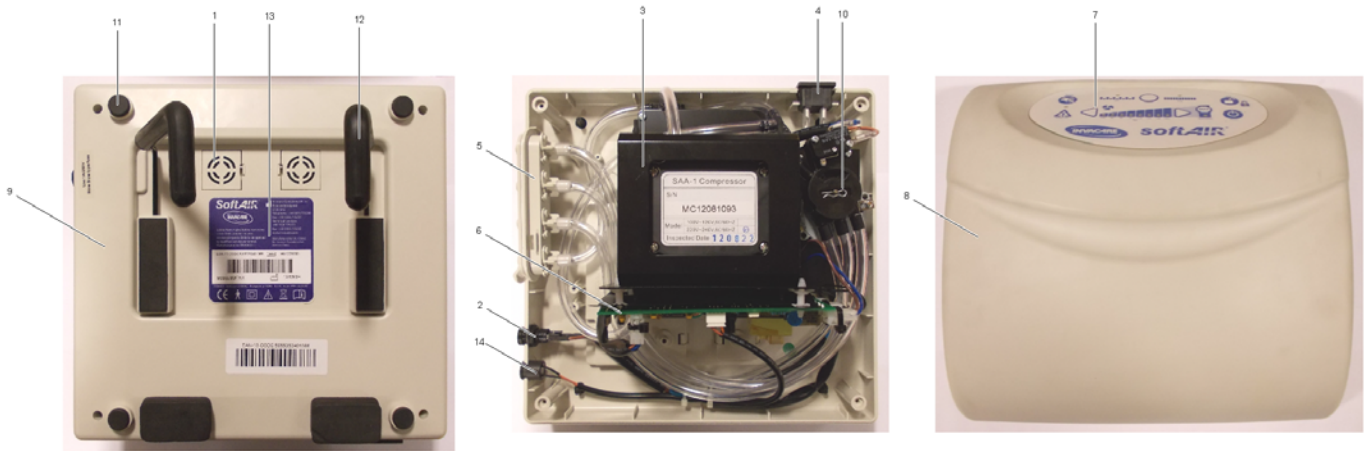

1	1532953	(Air Filter - Control Unit)	1
2	1532990	(Fuse 1A 250V, Control Unit.)	1
3	1533044	(Compressor, Control Unit.)	1
4	03111555M054001	(AC power Inlet/ IEC Socket)	1
5	1532818	(Housing Connector For G Handle Inc "O" Rings.)	1
6	1557147	(Print Circuit Board (PCB) Control Unit SoftAIR Excellence)	1
7	1533197	(Membrane, Control Unit.)	1
8	1533360	(Front Shell, Top Housing For Invacare SoftAIR.)	1
9	1533361	(Rear Shell, Bottom Housing For Invacare SoftAIR.)	1
10	1533362	(Timing Motor (Rotary Valve Motor, Synchronous Motor 220VAM-1 0.1rpm).)	1
11	1533363	(Kit Of Four (4) Rubber Feet To Rear Shell.)	1
12	1533364	(Hook Repair Kit With Cushions (35mm).)	1
12	1554898	(Hook Repair Kit With Cushions (80mm).)	1
13	1561809	(Blue compliance label SoftAIR.) <i>Please note serial number label not included.</i>	1
14	1574540	(Buzzer Kit Assembly SoftAIR.)	1
15	1547080	(UK Power Cord, Control Unit.)	1
16	1533195	(User Manual SoftAIR.)	1
17	1564030	(Softair "O" ring. Each.) <i>Qty 1 supplied.</i>	1
18	1576874	(Connector 3way T-type POM 3/8" _1/2" _1/2")	1
19	1576875	(Connector 3way Y-type PP 3/8")	1





Yes, you can.®



---

**Belgium & Luxemburg:**

Invacare nv, Autobaan 22, B-8210 Loppem  
Tel: (32) (0)50 83 10 10, Fax: (32) (0)50 83 10 11  
belgium@invacare.com / www.invacare.be

**Denmark:**

Invacare A/S, Sdr. Ringvej 37, DK-2605 Brøndby  
Tel: (45) (0)36 90 00 00, Fax: (45) (0)36 90 00 01  
denmark@invacare.com / www.invacare.dk

**Deutschland:**

Invacare Aquatec GmbH, Alemannenstraße 10, D-88316 Isny  
Tel: (49) (0)75 62 7 00 0, Fax: (49) (0)75 62 7 00 66  
info@invacare-aquatec.com / www.invacare-aquatec.de

**European Distributor Organisation:**

Invacare, Kleiststraße 49, D-32457 Porta Westfalica  
Tel: (49) (0)57 31 754 540, Fax: (49) (0)57 31 754 541  
edo@invacare.com / www.invacare.eu.com

**España:**

Invacare SA, c/Areny s/n, Polígon Industrial de Celrà, E-17460 Celrà (Girona)  
Tel: (34) (0)972 49 32 00, Fax: (34) (0)972 49 32 20  
contactsp@invacare.com / www.invacare.es

**France:**

Invacare Poirier SAS, Route de St Roch, F-37230 Fondettes  
Tel: (33) (0)2 47 62 64 66, Fax: (33) (0)2 47 42 12 24  
contactfr@invacare.com / www.invacare.fr

**Ireland:**

Invacare Ireland Ltd, Unit 5 Seatown Business Campus, Seatown Road, Swords, County Dublin - Ireland  
Tel: (353) 1 810 7084, Fax: (353) 1 810 7085  
ireland@invacare.com / www.invacare.ie

**Italia:**

Invacare Mecc San s.r.l., Via dei Pini 62, I-36016 Thiene (VI)  
Tel: (39) 0445 38 00 59, Fax: (39) 0445 38 00 34  
italia@invacare.com / www.invacare.it

**Nederland:**

Invacare BV, Celsiusstraat 46, NL-6716 BZ Ede  
Tel: (31) (0)318 695 757, Fax: (31) (0)318 695 758  
nederland@invacare.com, csede@invacare.com / www.invacare.nl

**Norge:**

Invacare AS, Grensesvingen 9, Postboks 6230, Etterstad, N-0603 Oslo  
Tel: (47) (0)22 57 95 00, Fax: (47) (0)22 57 95 01  
norway@invacare.com, island@invacare.com / www.invacare.no

**Österreich:**

Invacare Austria GmbH, Herzog Odilostrasse 101, A-5310 Mondsee  
Tel: (43) 6232 5535 0, Fax: (43) 6232 5535 4  
info-austria@invacare.com / www.invacare.at

**Portugal:**

Invacare Lda, Rua Estrada Velha, 949, P-4465-784 Leça do Balio  
Tel: (351) (0)225 1059 46/47, Fax: (351) (0)225 1057 39  
portugal@invacare.com / www.invacare.pt

**Sverige & Suomi:**

Invacare AB, Fagerstagatan 9, S-163 91 Spånga  
Tel: (46) (0)8 761 70 90, Fax: (46) (0)8 761 81 08  
sweden@invacare.com, finland@invacare.com / www.invacare.se

**Switzerland:**

Invacare AG, Benkenstrasse 260, CH-4108 Witterswil  
Tel: (41) (0)61 487 70 80, Fax: (41) (0)61 487 70 81  
switzerland@invacare.com / www.invacare.ch

**United Kingdom:**

Invacare Limited, Pencoed Technology Park, Pencoed, Bridgend CF35 5AQ  
Switchboard Tel: (44) (0)1656 776200, Fax: (44) (0)1656 776201  
Customer services Tel: (44) (0)1656 776222, Fax: (44) (0)1656 776220  
uk@invacare.com / www.invacare.co.uk