

Spare Parts Catalogue

Version: 2015/07/01b NO





Yes, you can.®

General Information

For any order of spare parts the following information is required:

- Product type*
- Product name & model*
- Serial Number*
- Part-Number*
- Part-Description*

Exploded view drawings and code numbers are not contractual and subject to modification, in relation to Invacare® continuous product improvement policy to meet end users needs.

The reproduction or use of this work, or of any substantial part of this work, in any form or by any electronic, mechanical or other means, including photocopying, scanning and recording, and in any information storage and retrieval system, including electronic transmission, is forbidden without the prior written permission of Invacare® International Sarl.



Yes, you can.®

TABLE OF CONTENTS

Informasjon & brukermanual

(User Manuals Storm4Max)	7
(User Manuals REM550)	8
(User Manuals REM24SD)	9
(User Manuals REM A & REM B)	10
(User Manuals G90)	11
(User Manuals G91)	12
(User Manuals REM34 & REM41 Tray/Centre Control)	13
(User Manuals IDC Dual Control)	14
(User Manuals Compact Dual Control)	15
(User Manuals ACS Handprogrammer)	16
(User Manual Proximity Head Remote DSL-105)	17

Chassis

(Shrouds)	18
(Shroud spring bracket/rear light)	19
(Transport fixation)	20
(Suspension)	21
(Front swingarm)	22

Sete

(Seat column)	23
(Powered seat tilt)	24
(Manual seat tilt)	25
(Seat)	26

Rygg

(Powered backrest adjustment)	27
(Manual backrest adjustment)	28
(Seat depth adapter)	29
(Backrest)	30

Benstøtte/overdel

(Calf plates)	31
(Legrest Support, Vari series, Flex2-/Max-Seat)	32

Fotplater

(Footboard)	33
(Fixed footboard)	34

Armlen

(Armrests for Max Seat)	35
-------------------------------	----

Svinghjul

(Front wheels)	36
----------------------	----

Drivhjul



Yes, you can.®

TABLE OF CONTENTS

(Drive Wheel 14")	37
Fremdrift	
(Motor with swingarm)	38
(Decoupling)	39
Elektronikk	
Elektronikk	40
(ACS Electronic Modules)	41
(Power modules ACS 2)	42
(Power module holder / Gyro)	43
(ACT for ACS 2)	44
(G-Trac-sensor)	45
(Light board)	46
(ACS Remote & Brackets)	47
(REM 24 SD with brackets)	48
(Remote REM 24SD)	49
(Remote Bracket Standard)	50
(Remote Bracket Swing away & height adjust.)	51
(Remote Bracket Attendant)	52
(REM A & B with brackets)	53
(Remote Bracket Standard)	50
(Remote Bracket Swing away & height adjust.)	54
(Remote Bracket Attendant)	52
(REM 550 with brackets)	55
(Remote Bracket Standard)	50
(Remote Bracket Swing away & height adjust.)	51
(Remote Bracket Attendant)	52
(REM 34 with brackets)	56
(Remote Bracket Standard)	57
(Remote Bracket Swing away & height adjust.)	58
(G90A with brackets)	59
(Remote Bracket Standard)	60
(Remote Bracket Swing away & height adjust.)	61
(Remote Bracket Attendant)	62
(G91S with brackets)	63
(Remote Bracket Standard)	64
(Small Chin Control with brackets)	65
(Small Chincontrol)	66
(Framework for chincontrol)	67
(Dual Control with brackets)	68
(Remote Bracket Dual Control)	69
(Compact Dual Control with brackets)	70
(Compact Dual Control)	71
(Bracket, Compact Dual Control - short)	72
(Bracket, Compact Dual Control - long)	73
(Intuitive Dual Control with brackets)	74
(ACS Cables & Accessories)	75
(Bus-Cables ACS)	76
(T-Adapter ACS)	77
(Slow-stop-sensor & Holder)	78



Yes, you can.®

TABLE OF CONTENTS

(Programming Kits)	79
(Magnetkey for Remotes)	80

Batterier, kabler og ladere

(Batteries)	81
(Chargers)	82
(Battery support)	83
(Bus-Cables ACS)	76

Sikkerhedsudstyr og tilbehør

(LED-Frontlight)	84
(LED-Rearlight)	85
(Lap Belts / H-Belt)	86
(Rearview Mirror)	87

Setetilbehør

(Headrest & Bracket)	88
(Headrest with cheek support)	89
(Headrest, sidewise adjustable)	90

Tilbehør

(Cane holder (MAX Seat))	91
(Fender for Drive Wheel)	92
(km-Counter & Bumper)	93
(Mounting-kit km-Counter)	94
(Hour Counter)	95
(KLICKfix® bag)	96
(KLICKfix® bracket for Fixed Armrest)	97
(Bag)	98
(Toolkit)	99

(User Manuals Storm4Max)

languages.io

BG	Български	FR	Français	NL	Nederlands	SV	Svenska
CS	Čeština	HE	עברית	NO	Norsk	UK	Українська
DA	Dansk	HR	Hrvatski	PL	Polski		
DE	Deutsch	HU	Magyar	PT	Português		
EN	English	IT	Italiano	RU	Русский		
ES	Español	JA	日本語	RO	Română		
ET	Eesti	LV	Latviešu	SK	Slovenčina		
FI	Suomi	LT	Lietuvių	SL	Slovenščina		

DA	SP1547066	(User Manual Storm4Max Danish)	1
DE	SP1547065	(User Manual Storm4Max German)	1
EN	SP1547067	(User Manual Storm4Max English)	1
ES	SP1547068	(User Manual Storm4Max Spanish)	1
FR	SP1547069	(User Manual Storm4Max French)	1
IT	SP1547070	(User Manual Storm4Max Italian)	1
NL	SP1547071	(User Manual Storm4Max Dutch)	1
NO	SP1581785	(User Manual Storm4Max Norwegian)	1
PT	SP1572569	(User Manual Storm4 Max Portuguese)	1
SV	SP1547072	(User Manual Storm4Max Swedish)	1

(User Manuals REM550)

languages.io

BG	Български	FR	Français	NL	Nederlands	SV	Svenska
CS	Čeština	HE	עברית	NO	Norsk	UK	Українська
DA	Dansk	HR	Hrvatski	PL	Polski		
DE	Deutsch	HU	Magyar	PT	Português		
EN	English	IT	Italiano	RU	Русский		
ES	Español	JA	日本語	RO	Română		
ET	Eesti	LV	Latviešu	SK	Slovenčina		
FI	Suomi	LT	Lietuvių	SL	Slovenščina		

DA	1528088	(User Manual REM550 Danish)	1
DE	1528089	(User Manual REM550 German)	1
EN	1528090	BRUKSANVISNING DX-REM550 MANUAL	1
ES	1528091	(User Manual REM550 Spanish)	1
FI	1528092	(User Manual REM550 Finsk)	1
FR	1528093	(User Manual REM550 French)	1
IT	1528094	(User Manual REM550 Italiano)	1
NL	1528095	(User Manual REM550 Dutch)	1
NO	1528096	BRUKSANVISNING DX REM550 MANUAL NORSK	1
PT	1528097	(User Manual REM550 Portuguese)	1
SV	1528098	(User Manual REM550 Swedish)	1
HR	SP1533279	(User Manual REM550 Croatian)	1
CS	SP1533280	(User Manual REM551 Czech)	1
ET	SP1533281	(User Manual REM552 Estonian)	1
HU	SP1533282	(User Manual REM553 Hungarian)	1
LT	SP1533283	(User Manual REM554 Lithuanian)	1
PL	SP1533284	(User Manual REM555 Polish)	1
SL	SP1533285	(User Manual REM556 Slovenian)	1

(User Manuals REM24SD)

languages.io

BG	Български	FR	Français	NL	Nederlands	SV	Svenska
CS	Čeština	HE	עברית	NO	Norsk	UK	Українська
DA	Dansk	HR	Hrvatski	PL	Polski		
DE	Deutsch	HU	Magyar	PT	Português		
EN	English	IT	Italiano	RU	Русский		
ES	Español	JA	日本語	RO	Română		
ET	Eesti	LV	Latviešu	SK	Slovenčina		
FI	Suomi	LT	Lietuvių	SL	Slovenščina		

DE	1436142	(User Manual REM24SD German)	1
FR	1436143	(User Manual REM24SD French)	1
EN	1436144	(User Manual REM24SD English)	1
SV	1436145	(User Manual REM24SD Swedish)	1
DA	1436146	(User Manual REM24SD Danish)	1
NO	1436147	(User Manual REM24SD Norwegian)	1
NL	1436148	(User Manual REM24SD Dutch)	1
ES	1436149	(User Manual REM24SD Spanish)	1
PT	1436150	(User Manual REM24SD Portuguese)	1
IT	1436385	(User Manual REM24SD Italian)	1
FI	1436386	(User Manual REM24SD Finnish)	1
HR	SP1533251	(User Manual REM24SD Croatian)	1
CS	SP1533252	(User Manual REM24SD Czech)	1
ET	SP1533253	(User Manual REM24SD Estonian)	1
HU	SP1533254	(User Manual REM24SD Hungarian)	1
LT	SP1533255	(User Manual REM24SD Lithuanian)	1
PL	SP1533256	(User Manual REM24SD Polish)	1
SL	SP1533257	(User Manual REM24SD Slovenian)	1
LV	SP1543152	(User Manual REM24SD Latvian)	1

BG	Български	FR	Français	NL	Nederlands	SV	Svenska
CS	Čeština	HE	עברית	NO	Norsk	UK	Українська
DA	Dansk	HR	Hrvatski	PL	Polski		
DE	Deutsch	HU	Magyar	PT	Português		
EN	English	IT	Italiano	RU	Русский		
ES	Español	JA	日本語	RO	Română		
ET	Eesti	LV	Latviešu	SK	Slovenčina		
FI	Suomi	LT	Lietuvių	SL	Slovenščina		

DA	1529171	(User Manual REM A & B Dansk)	1
DE	1529172	(User Manual REM A & B German)	1
EN	1529173	(User Manual REM A & B English)	1
ES	1529174	(User Manual REM A & B Spanish)	1
FI	1529228	(User Manual REM A & B Finsk)	1
FR	1529229	(User Manual REM A & B French)	1
IT	1529230	(User Manual REM A & B Italian)	1
NL	1529231	(User Manual REM A & B Dutch)	1
NO	1529232	(User Manual REM A & B Norsk)	1
PT	1529233	(User Manual REM A & B Portuguese)	1
SV	1529234	(User Manual REM A & B Swedish)	1
HR	SP1533258	(User Manual REM A & B Croatian)	1
CS	SP1533259	(User Manual REM A & B Czech)	1
ET	SP1533265	(User Manual REM A & B Estonian)	1
HU	SP1533266	(User Manual REM A & B Hungarian)	1
LT	SP1533267	(User Manual REM A & B Lithuanian)	1
PL	SP1533268	(User Manual REM A & B Polish)	1
SL	SP1533269	(User Manual REM A & B Slovenian)	1
RO	SP1533270	(User Manual REM A & B Romanian)	1

BG	Български	FR	Français	NL	Nederlands	SV	Svenska
CS	Čeština	HE	עברית	NO	Norsk	UK	Українська
DA	Dansk	HR	Hrvatski	PL	Polski		
DE	Deutsch	HU	Magyar	PT	Português		
EN	English	IT	Italiano	RU	Русский		
ES	Español	JA	日本語	RO	Română		
ET	Eesti	LV	Latviešu	SK	Slovenčina		
FI	Suomi	LT	Lietuvių	SL	Slovenščina		

(User Manuals G91)

languages.io

BG	Български	FR	Français	NL	Nederlands	SV	Svenska
CS	Čeština	HE	עברית	NO	Norsk	UK	Українська
DA	Dansk	HR	Hrvatski	PL	Polski		
DE	Deutsch	HU	Magyar	PT	Português		
EN	English	IT	Italiano	RU	Русский		
ES	Español	JA	日本語	RO	Română		
ET	Eesti	LV	Latviešu	SK	Slovenčina		
FI	Suomi	LT	Lietuvių	SL	Slovenščina		

DE	1536441	(User Manual G91 German)	1
DA	1537929	(User Manual G91 Danish)	1
NL	1537930	(User Manual G91 Dutch)	1
EN	1537931	(User Manual G91 English)	1
FI	1537932	(User Manual G91 Finnish)	1
FR	1537933	(User Manual G91 French)	1
IT	1537934	(User Manual G91 Italian)	1
NO	1537935	(User Manual G91 Norwegian)	1
PT	1537936	(User Manual G91 Portuguese)	1
ES	1537937	(User Manual G91 Spanish)	1
SV	1537938	(User Manual G91 Swedish)	1

(User Manuals REM34 & REM41 Tray/Centre Control)

languages.io

BG	Български	FR	Français	NL	Nederlands	SV	Svenska
CS	Čeština	HE	עברית	NO	Norsk	UK	Українська
DA	Dansk	HR	Hrvatski	PL	Polski		
DE	Deutsch	HU	Magyar	PT	Português		
EN	English	IT	Italiano	RU	Русский		
ES	Español	JA	日本語	RO	Română		
ET	Eesti	LV	Latviešu	SK	Slovenčina		
FI	Suomi	LT	Lietuvių	SL	Slovenščina		

DA	SP1538022	(User Manual REM41 Tray Control Danish)	1
EN	SP1538023	(User Manual REM41 Tray Control English)	1
ES	SP1538024	(User Manual REM41 Tray Control Spanish)	1
FR	SP1538025	(User Manual REM41 Tray Control French)	1
IT	SP1538026	(User Manual REM41 Tray Control Italian)	1
NL	SP1538027	(User Manual REM41 Tray Control Dutch)	1
NO	SP1538028	(User Manual REM41 Tray Control Norwegian)	1
PT	SP1538029	(User Manual REM41 Tray Control Portuguese)	1
SV	SP1538030	(User Manual REM41 Tray Control Swedish)	1
CS	SP1538031	(User Manual REM41 Tray Control Czech)	1
SL	SP1538032	(User Manual REM41 Tray Control Slovenian)	1
DE	SP1541402	(User Manual REM41 Tray Control German)	1

BG	Български	FR	Français	NL	Nederlands	SV	Svenska
CS	Čeština	HE	עברית	NO	Norsk	UK	Українська
DA	Dansk	HR	Hrvatski	PL	Polski		
DE	Deutsch	HU	Magyar	PT	Português		
EN	English	IT	Italiano	RU	Русский		
ES	Español	JA	日本語	RO	Română		
ET	Eesti	LV	Latviešu	SK	Slovenčina		
FI	Suomi	LT	Lietuvių	SL	Slovenščina		

DE	1521480	(User Manual Dual-Control German)	1
EN	1521481	(User Manual Dual-Control English)	1
FR	1521482	(User Manual Dual-Control French)	1
NL	1521483	(User Manual Dual-Control Dutch)	1
NO	1521484	BRUKSANVISNING INTUITIV LEDSAGERSTYRING	1
SV	1521485	(User Manual Dual-Control Swedish)	1
DA	1521486	(User Manual Dual-Control Danish)	1
IT	1522252	(User Manual Dual-Control Italian)	1
PT	1522254	(User Manual Dual-Control Portuguese)	1
ES	1522255	(User Manual Dual-Control Spanish)	1

BG	Български	FR	Français	NL	Nederlands	SV	Svenska
CS	Čeština	HE	עברית	NO	Norsk	UK	Українська
DA	Dansk	HR	Hrvatski	PL	Polski		
DE	Deutsch	HU	Magyar	PT	Português		
EN	English	IT	Italiano	RU	Русский		
ES	Español	JA	日本語	RO	Română		
ET	Eesti	LV	Latviešu	SK	Slovenčina		
FI	Suomi	LT	Lietuvių	SL	Slovenščina		

DA	SP1588488	(User Manual Compact Dual Control - Danish)	1
DE	SP1588489	(User Manual Compact Dual Control - German)	1
EN	SP1588462	(User Manual Compact Dual Control - English)	1
ES	SP1588496	(User Manual Compact Dual Control - Spanish)	1
FI	SP1588490	(User Manual Compact Dual Control - Finnish)	1
FR	SP1588491	(User Manual Compact Dual Control - French)	1
IT	SP1588492	(User Manual Compact Dual Control - Italian)	1
NL	SP1588493	(User Manual Compact Dual Control - Dutch)	1
NO	SP1588494	(User Manual Compact Dual Control - Norwegian)	1
PT	SP1588495	(User Manual Compact Dual Control - Portuguese)	1
SV	SP1588497	(User Manual Compact Dual Control - Swedish)	1

(User Manuals ACS Handprogrammer)

languages.io

BG	Български	FR	Français	NL	Nederlands	SV	Svenska
CS	Čeština	HE	עברית	NO	Norsk	UK	Українська
DA	Dansk	HR	Hrvatski	PL	Polski		
DE	Deutsch	HU	Magyar	PT	Português		
EN	English	IT	Italiano	RU	Русский		
ES	Español	JA	日本語	RO	Română		
ET	Eesti	LV	Latviešu	SK	Slovenčina		
FI	Suomi	LT	Lietuvių	SL	Slovenščina		

DE	1419316	(User Manual Handheld Programmer German)	1
EN	1419317	(User Manual Handheld Programmer English)	1

(User Manual Proximity Head Remote DSL-105)


languages.io

BG	Български	FR	Français	NL	Nederlands	SV	Svenska
CS	Čeština	HE	עברית	NO	Norsk	UK	Українська
DA	Dansk	HR	Hrvatski	PL	Polski		
DE	Deutsch	HU	Magyar	PT	Português		
EN	English	IT	Italiano	RU	Русский		
ES	Español	JA	日本語	RO	Română		
ET	Eesti	LV	Latviešu	SK	Slovenčina		
FI	Suomi	LT	Lietuvių	SL	Slovenščina		

EN	SP1555731	(User Manual Proximity Head Remote DSL-105 English)	1
DE	SP1555732	(User Manual Proximity Head Remote DSL-105 German)	1
FR	SP1555733	(User Manual Proximity Head Remote DSL-105 French)	1
DA	SP1555839	(User Manual Proximity Head Remote DSL-105 Danish)	1
ES	SP1555840	(User Manual Proximity Head Remote DSL-105 Spanish)	1
FI	SP1555841	(User Manual Proximity Head Remote DSL-105 Finnish)	1
IT	SP1555842	(User Manual Proximity Head Remote DSL-105 Italian)	1
NL	SP1555843	(User Manual Proximity Head Remote DSL-105 Dutch)	1
NO	SP1555844	(User Manual Proximity Head Remote DSL-105 Norwegian)	1
PT	SP1555845	(User Manual Proximity Head Remote DSL-105 Portuguese)	1
SV	SP1555846	(User Manual Proximity Head Remote DSL-105 Swedish)	1

(Shrouds)


1544691X.dwg

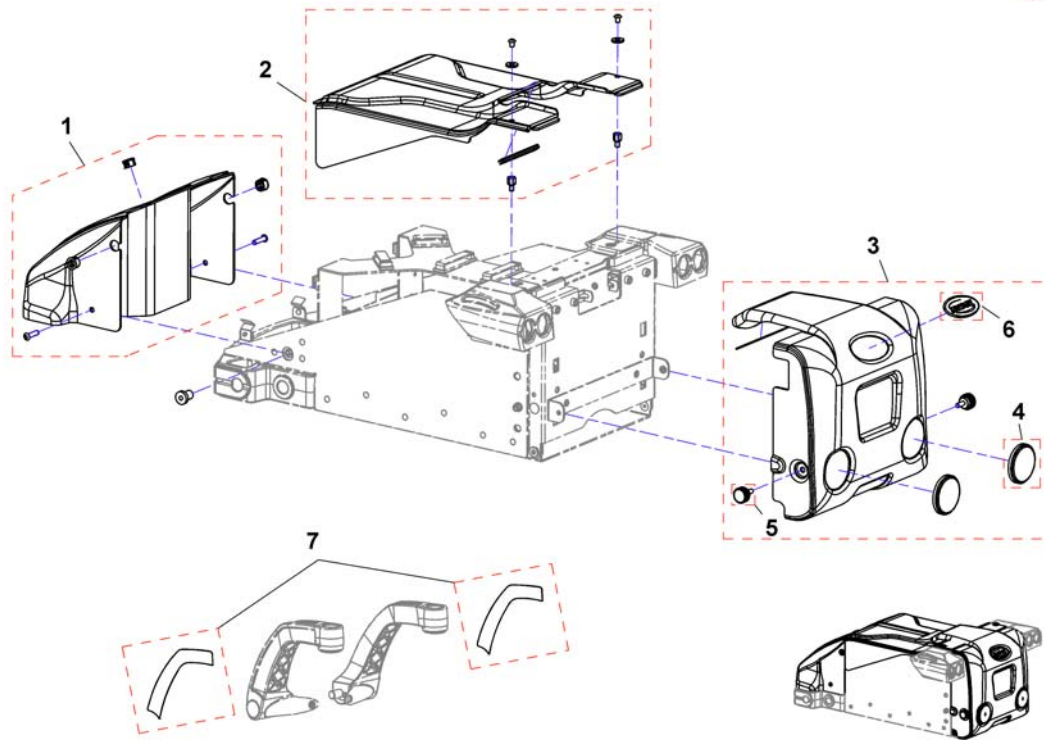


	SP1553166	LAKKSTIFT BLÅ STORM4 STORM4	1
	SP1553148	(Touch-up paint, RAL 9005 Black)	1
	SP1553167	(Touch-up paint, Champagne)	1
	SP1553169	(Touch-up paint, Red)	1
	SP1553168	(Touch-up paint, RAL 9016 White)	1
	SP1553210	(Touch-up paint, RAL 6003 Green)	1
	SP1553211	LAKKSTIFT SØLV STORM4 STORM4	1
1	SP1530339	FRONTKÅPE BLÅ STORM4	1
1	SP1530340	(Front shroud, cpl., Black (RAL 9005))	1
1	SP1530341	(Front shroud, cpl., Champagne)	1
1	SP1530342	(Front shroud, cpl., Red)	1
1	SP1530343	(Front shroud, cpl., White (RAL 9016))	1
1	SP1530344	(Front shroud, cpl., Green (RAL 6003))	1
2	SP1546754	(Middle shroud, White (RAL 9006))	1
3	SP1530346	DEKSEL BAK BLÅ STORM4	1
3	SP1530347	DEKSEL BAK SORT STORM4	1
3	SP1530348	(Rear shroud, cpl., Champagne)	1
3	SP1530427	(Rear shroud, cpl., Red)	1
3	SP1530428	(Rear shroud, cpl., White (RAL 9016))	1
3	SP1530429	(Rear shroud, cpl., Green (RAL 6003))	1
4	1419274	REFLEKTOR RØD TYPHOON	1
5	1440769	(Hand wheel)	1
6	0572-06266	INVACARE LOGO STORM4	1
7	SP1522428	DECAL KIT STORM4 STORM4	1
7	SP1523651	DECAL KIT STORM4 SORT STORM4	1
7	SP1529145	(Decal-Kit Champagne)	1
7	SP1529393	(Decal-Kit Red (RAL 3007))	1
7	SP1529395	(Decal-Kit White (RAL 9016))	1

(Shroud spring bracket/rear light)


1528810X.iso

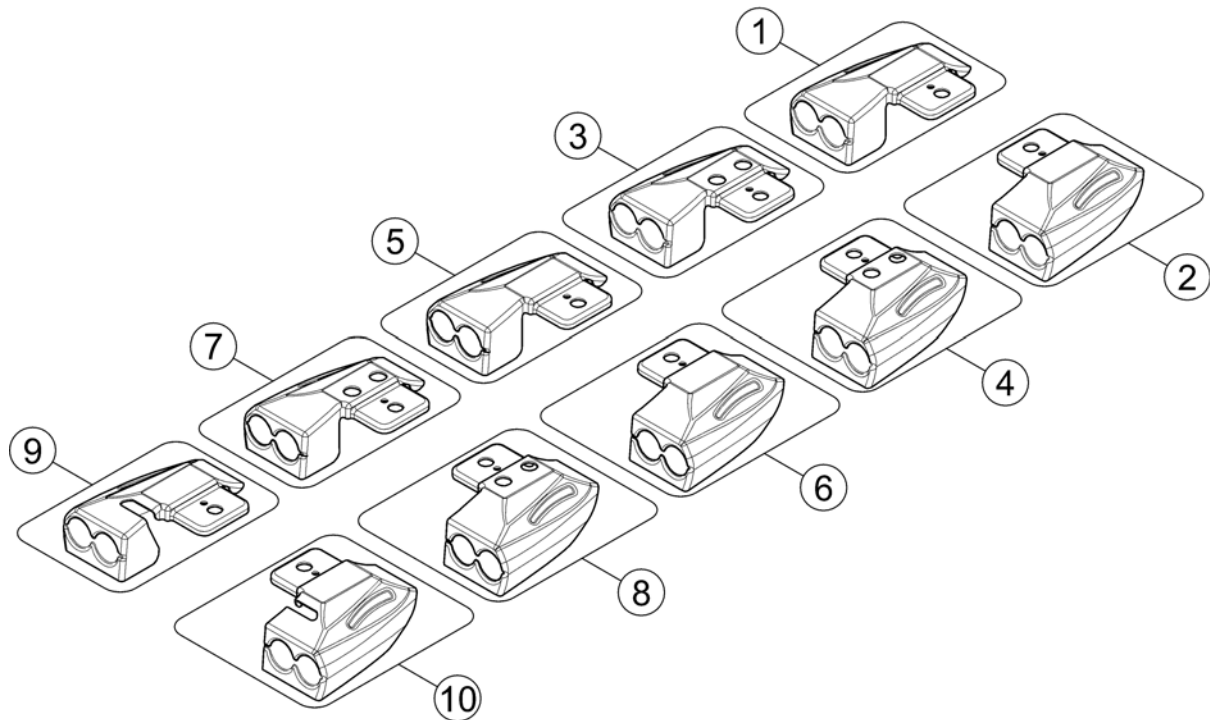


1	SP1528810	(Shroud for spring bracket without light LH)	1
2	SP1528811	(Shroud for spring bracket without light RH)	1
3	SP1552927	(Shroud for spring bracket, with baggage carrier, without light, LH)	1
4	SP1552928	(Shroud for spring bracket, with baggage carrier, without light, RH)	1
5	SP1527836	DEKSEL BAK LED LYKT VE STORM4	1
6	SP1527835	DEKSEL BAK LED LYKT HØ STORM4	1
7	SP1552929	(Shroud for spring bracket, with baggage carrier, for LED-rearlight, LH)	1
8	SP1552930	(Shroud for spring bracket, with baggage carrier, for LED-rearlight, RH)	1
9	SP1528627	(Shroud for spring bracket for rearlight Germany, LH)	1
10	SP1528626	(Shroud for spring bracket for rearlight Germany, RH)	1

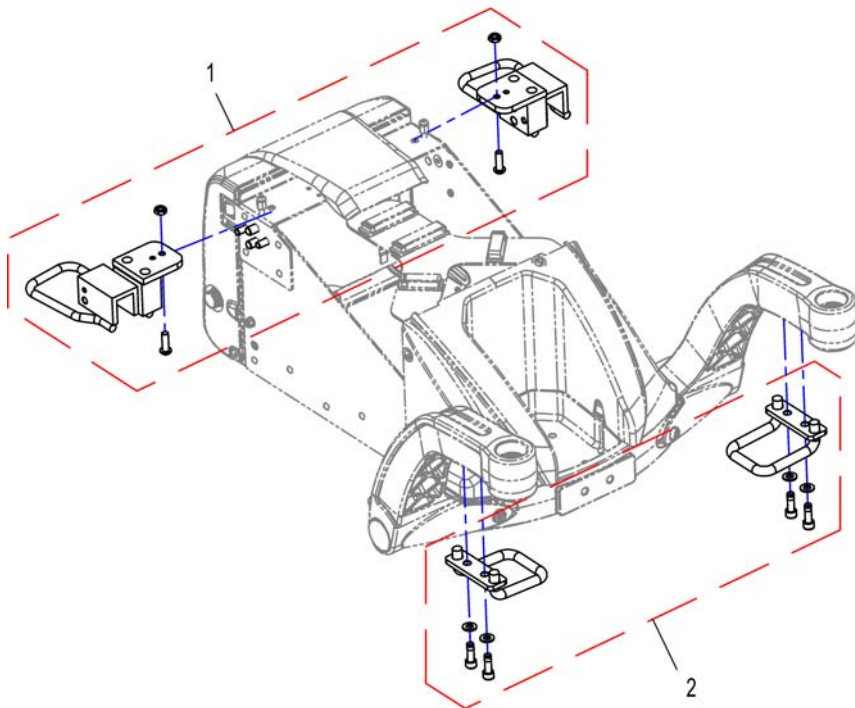

1	SP1563104	TRANSPORTFESTER BAK PAR STORM4 <i>not for passenger transportation</i>	1
2	SP1528974	TRANSPORTFESTE FORAN PAR STORM4 <i>not for passenger transportation</i>	1

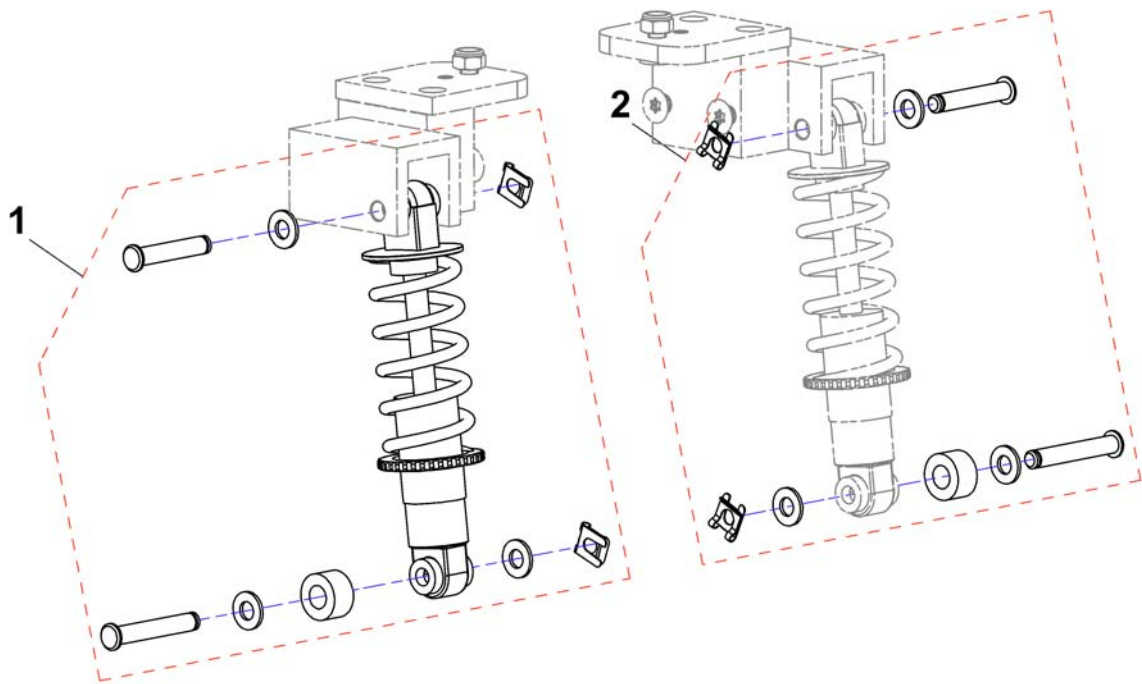


1	SP1544714	(Shock absorber, cpl.)	1
2	SP1530444	FESTEKIT STØTDEMPERE STORM4	1

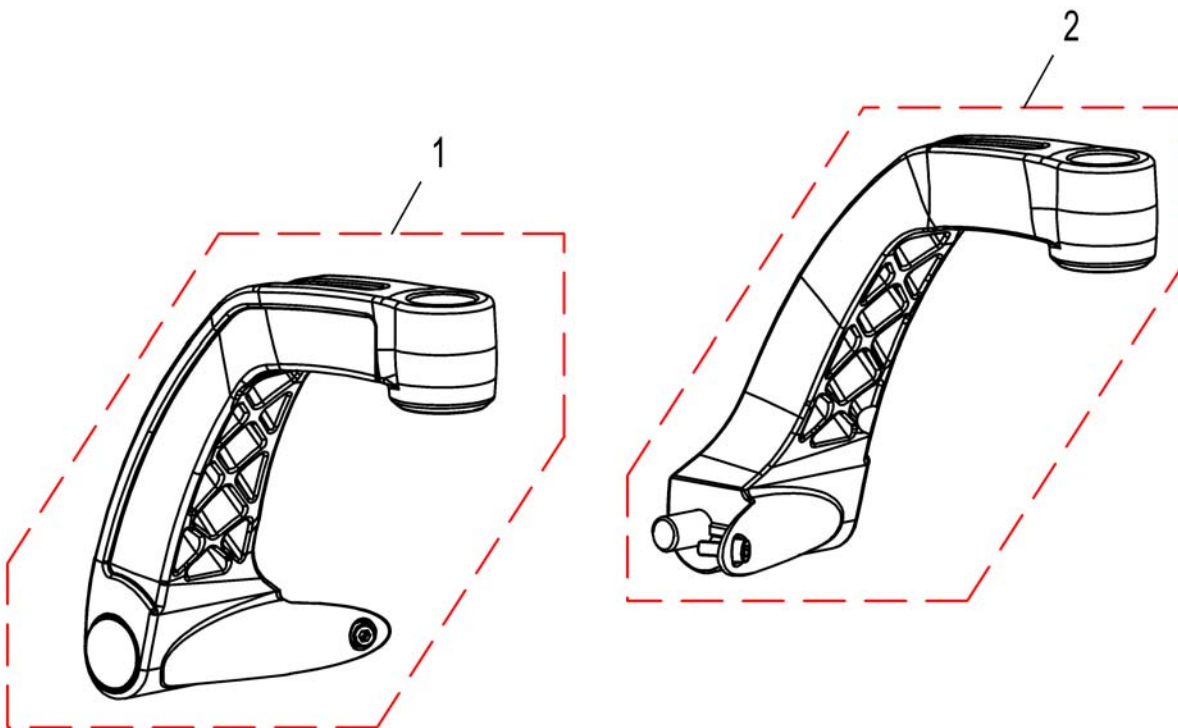

1	SP1527905	SVINGARM HØ STORM4	1
2	SP1527904	SVINGARM VE STORM4	1

1542985.dwg

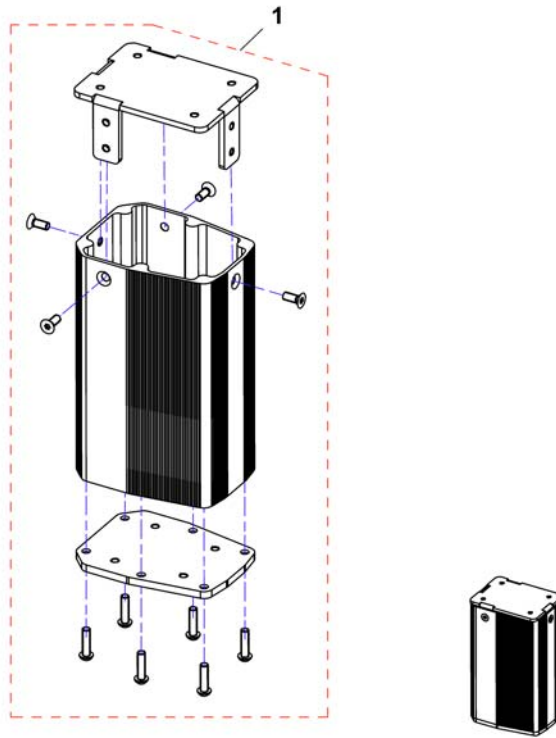


1 SP1542985

(Seat column)


1

Invacare® Storm4Max | Sete
(Powered seat tilt)

1543092.dwg

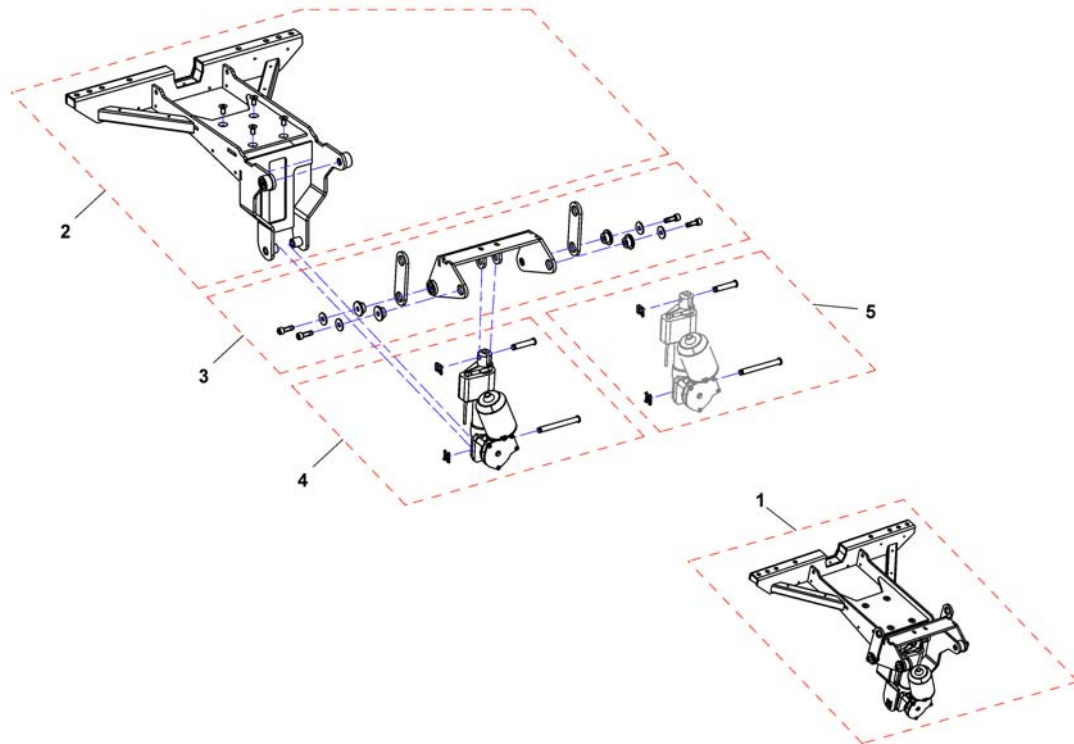


1	SP1543092	(Powered seat tilt)	1
2	SP1536630	(Tilt support)	1
3	SP1536646	(Tilt carrier)	1
4	SP1563175	(Motor for powered tilt, cpl.)	1
5	SP1532481	MONTERINGSKIT TILT STORM4	1

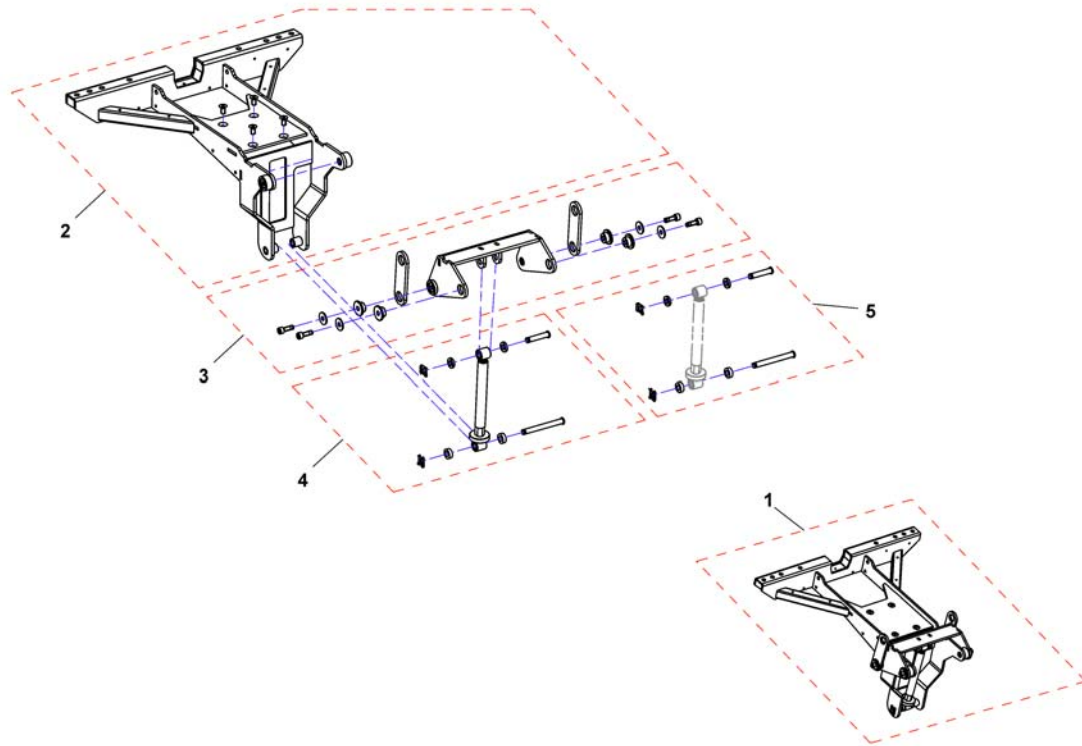


1	SP1544715	(Manual seat tilt)	1
2	SP1536630	(Tilt support)	1
3	SP1536646	(Tilt carrier)	1
4	SP1551010	(Tilt spindle, cpl.)	1
5	SP1551009	(Mounting-Kit)	1

(Seat)

1543301.dwg

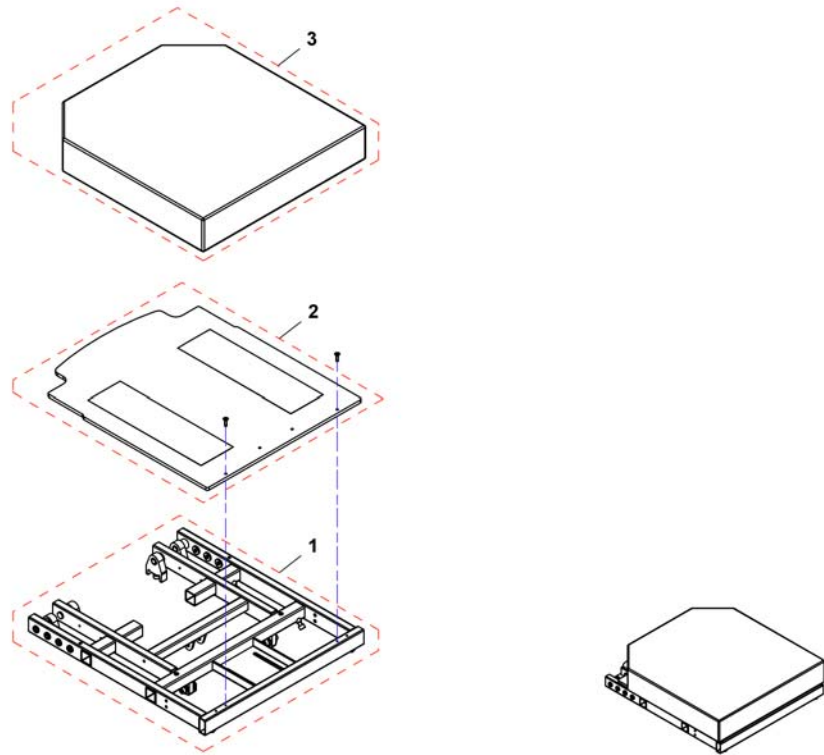

1	SP1543098	(Seat assembly, SW49)	1
1	SP1543099	(Seat assembly, SW55)	1
1	SP1543100	(Seat assembly, SW61)	1
2	SP1508137	(Seat plate SW 490mm)	1
2	SP1525083	(Seat plate SW 550mm)	1
2	SP1525082	(Seat plate SW 610mm)	1
3	SP1508059	SETEPUTE FLOSHAPE SB50 STORM4 MAX DARTEX SD53	1
3	SP1536595	SETEPUTE SB55 SD65 MAX STORM4 MAX	1
3	SP1536596	(Seat cushion 610x635, TR23)	1


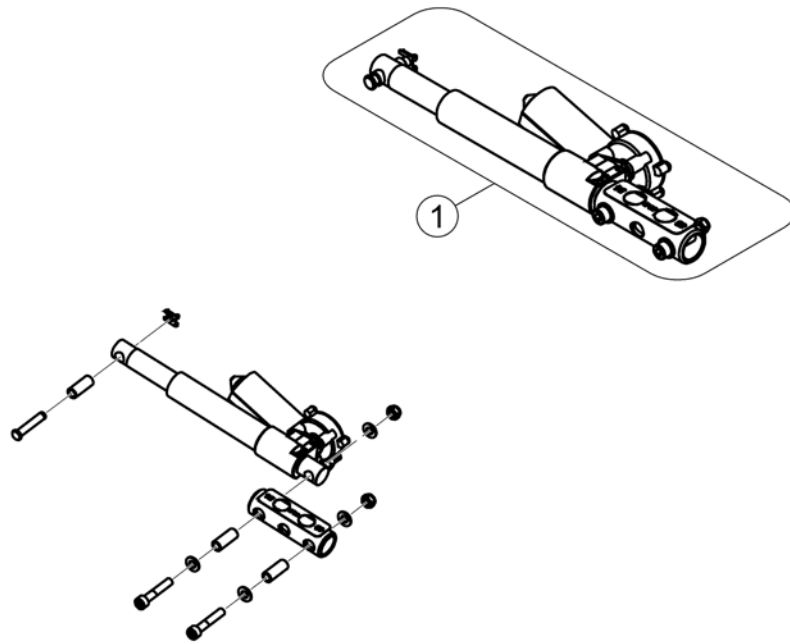


1 SP1544669 (Actuator, cpl.)

1

(Manual backrest adjustment)


1545185.jpg

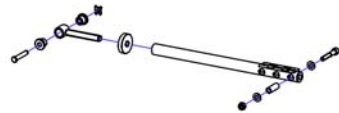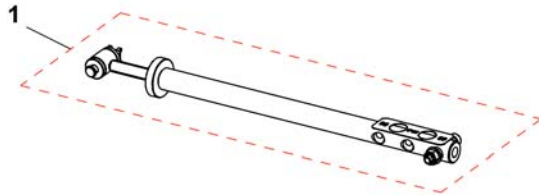

1 SP1545185 (Spindle, backrest adjustment, cpl.)

1

1551724K.dwg

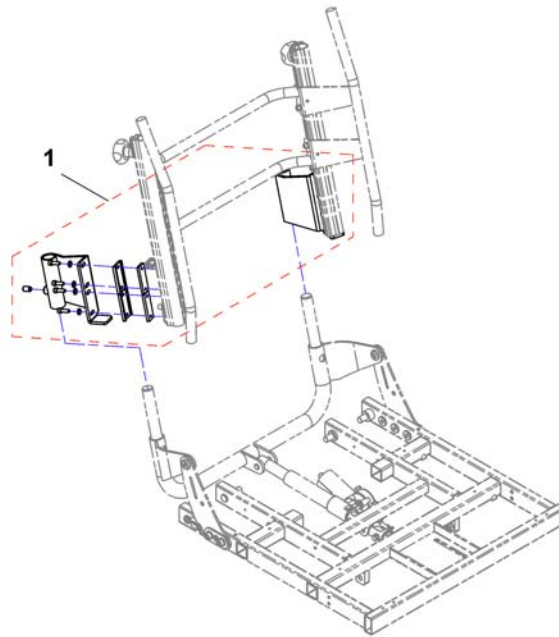


1 SP1551724


(Seat depth adapter, cpl.)

1

(Backrest)

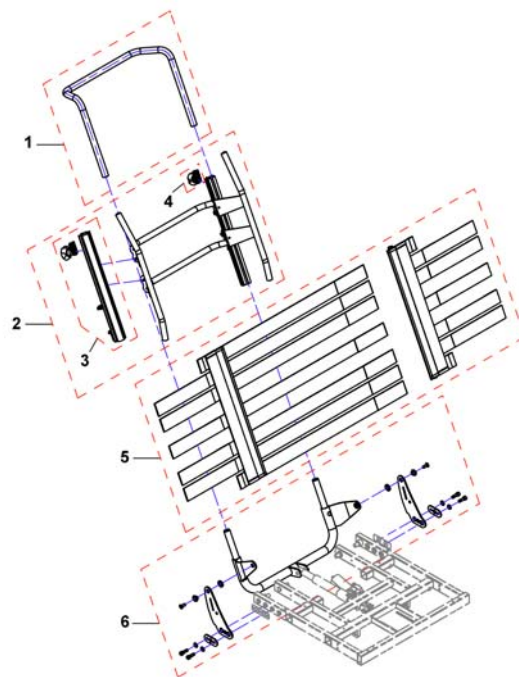


1	SP1543082	(Support tube, SW49)	1
1	SP1543081	(Support tube, SW55)	1
1	SP1543080	(Support tube, SW61)	1
2	SP1551199	(Backrest frame, cpl., SW49)	1
2	SP1526824	(Backrest frame, cpl., SW55)	1
2	SP1526823	(Backrest frame, cpl., SW61)	1
3	1445848	RYGGRØR TIL VELCRO KPL FLEX 3	1
4	5253088	LÅSKLOSS KPL F RYGGRØR REA COMFORT	1
-	1508867	(Cover SW49, dartex grey) <i>not shown</i>	1
-	SP1526459	(Cover, Lateral 640X520, Dartex Grey) <i>not shown</i>	1
-	1526424	(Cover, Lateral 700X500, Dartex Grey) <i>not shown</i>	1
5	1445694	RYGGTREKK BORRELÅS STILL BAR RYGG BELLIS AZALEA	1
5	SP1526308	(Backrest, tension adjustable, SW 55 - 61cm)	1
6	SP1544310	(Backrest bracket, cpl., SW49)	1
6	SP1544311	(Backrest bracket, cpl., SW55)	1
6	SP1544312	(Backrest bracket, cpl., SW61)	1

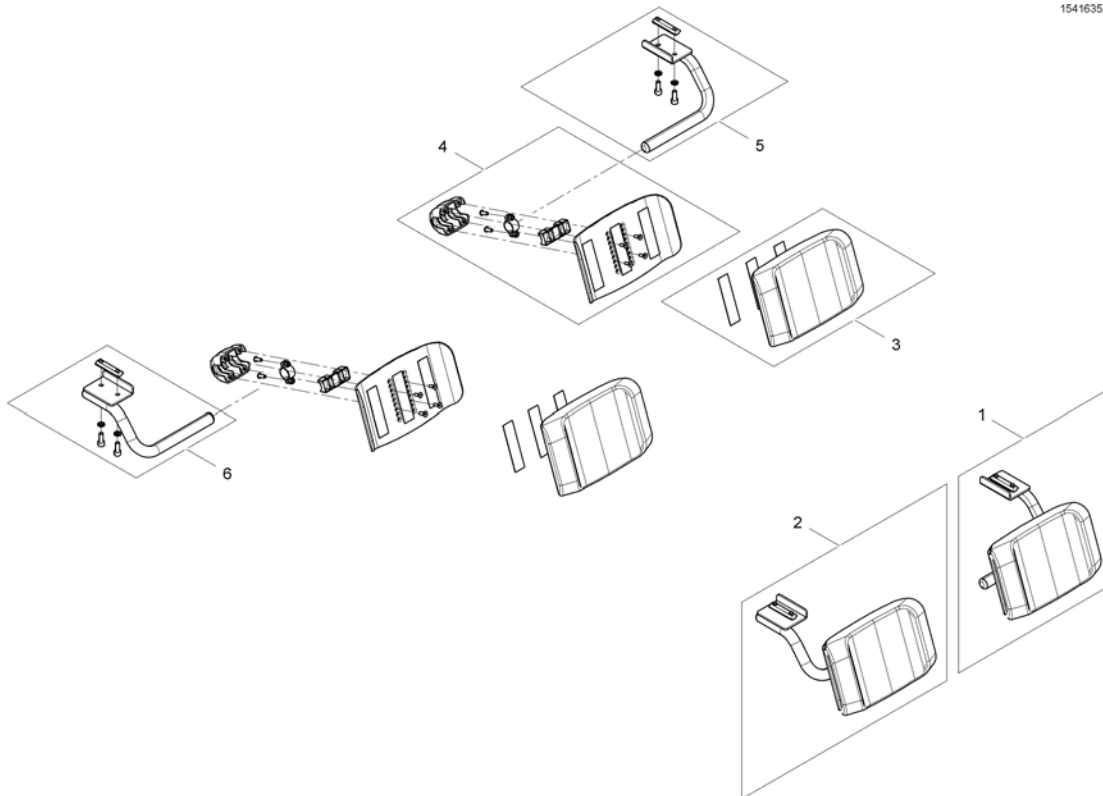


1	SP1544875	(Calf pad, SW 490mm, LH, cpl.)	1
1	SP1541635	(Calf pad, SW 550mm & 610mm, LH, cpl.)	1
2	SP1544876	(Calf pad, SW 490mm, LH, cpl.)	1
2	SP1541636	(Calf pad, SW 550mm & 610mm, RH, cpl.)	1
3	SP1541672	(Calf Pad SW 490mm)	1
3	SP1541491	(Calf Pad SW 550mm & 610mm)	1
4	SP1541669	(Calf pad bracket SW 490mm, LH/RH, cpl.)	1
4	SP1541489	(Calf pad bracket SW 550mm & 610mm, LH/RH, cpl.)	1
5	SP1541633	(Calf pad bracket, LH, cpl.)	1
6	SP1541634	(Calf pad bracket, RH, cpl.)	1

(Legrest Support, Vari series, Flex2-/Max-Seat)

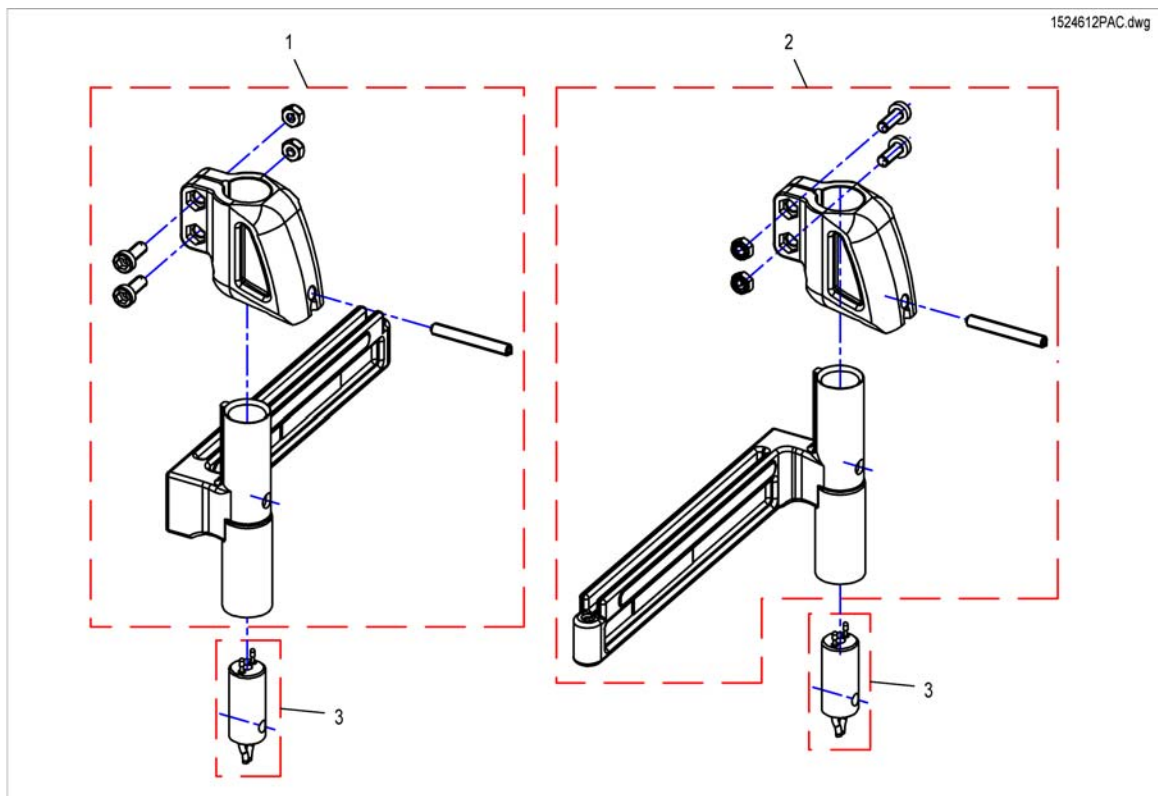


1	SP1526910	FESTE BENST MAN BORA <i>Flex2 & MAX</i>	1
1	SP1524612	HOLDER BENST EL ADE HØ BORA <i>for ACS, Flex2</i>	1
1	SP1555995	(Footrest/Legrest receptacle, powered, RH, 5cm) <i>for ACS, MAX</i>	1
1	SP1534880	(Footrest/Legrest receptacle, powered, RH, 5cm) <i>for R-Net, Flex2 & MAX without Lifter</i>	1
1	SP1537399	(Footrest/Legrest receptacle, powered, RH, 5cm) <i>for R-Net, Flex2 & MAX with Lifter</i>	1
2	SP1526909	FESTE BENST MAN BORA <i>Flex2 & MAX</i>	1
2	SP1524613	HOLDER BENST EL ADE VE BORA <i>for ACS, Flex2</i>	1
2	SP1555996	(Footrest/Legrest receptacle, powered, LH, 5cm) <i>for ACS, MAX</i>	1
2	SP1534881	(Footrest/Legrest receptacle, powered, LH, 5cm) <i>for R-Net, Flex2 & MAX without Lifter</i>	1
2	SP1537400	(Footrest/Legrest receptacle, powered, LH, 5cm) <i>for R-Net, Flex2 & MAX with Lifter</i>	1
3	SP1527775	KABEL HOLDER BENST 65MM TDX <i>for ACS, Flex2 RH</i>	1
3	SP1530795	KABEL I HOLDER F BENST STORM4 BORA TDX <i>for ACS, Max-Seat RH & LH / Flex2 LH</i>	1
3	SP1533694	(Power Supply Cable Legrest (850mm)) <i>for R-Net without Lifter, RH & LH</i>	1
3	SP1450232	KABEL EL BENST KORT STORM3 <i>for R-Net with Lifter, RH & LH</i>	1

1552004.png

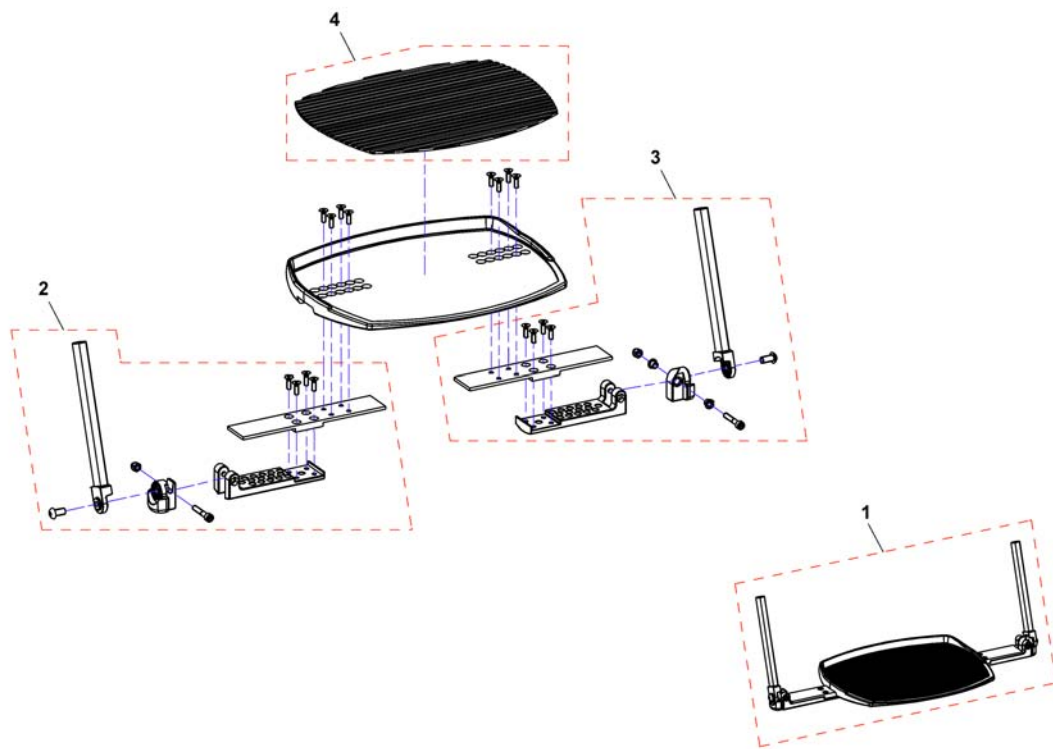


1	SP1552004	FOTBRETT SB55 STORM4 MAX	1
2	SP1552599	(Footboard bracket RH)	1
3	SP1552600	(Footboard bracket LH)	1
4	Z174159	FOTBRETT UNDERLAG HEL FOTPLA STORM/TWISTER	1

(Fixed footboard)

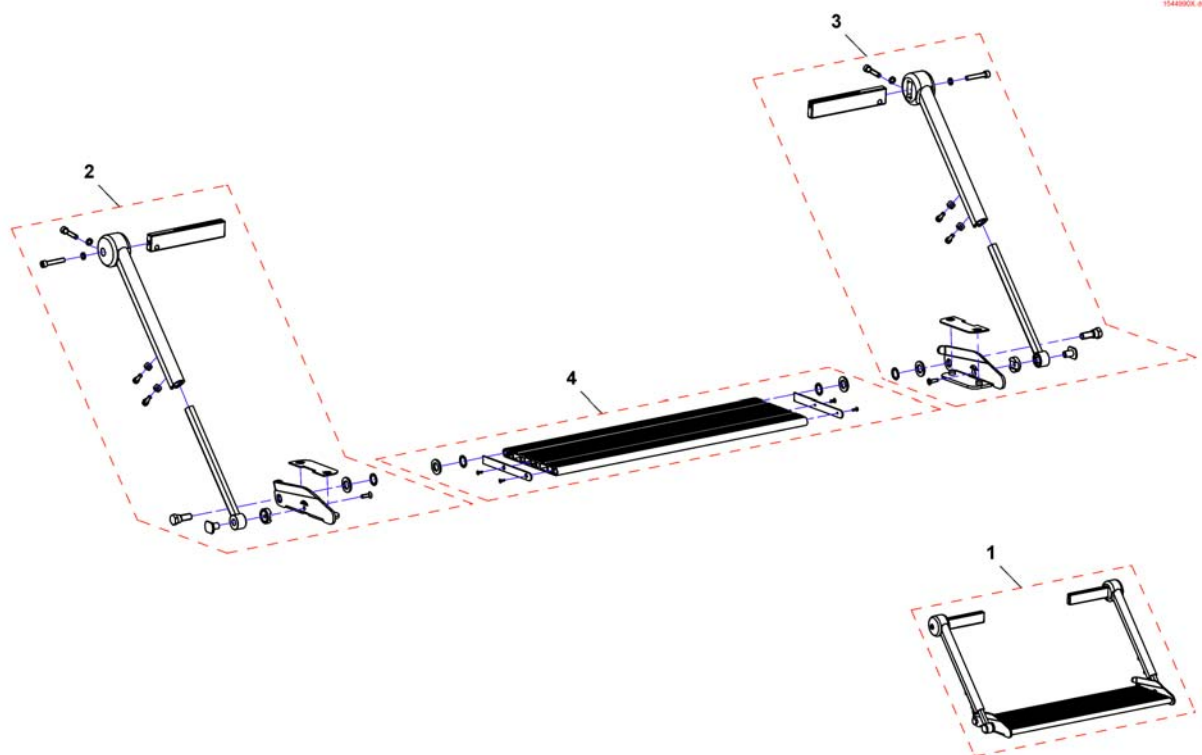


1	SP1544990	FOTBRETT 55 FAST STORM4 MAX	1
1	SP1545454	FOTBRETT 55 FAST STORM4 MAX	1
1	SP1545455	FOTBRETT FAST MAX SB49 STORM4 MAX	1
2	SP1552445	(Footboard bracket RH)	1
3	SP1552446	(Footboard bracket LH)	1
4	SP1552466	(Footboard SW49)	1
4	SP1552465	(Footboard SW55)	1
4	SP1552447	(Footboard SW61)	1

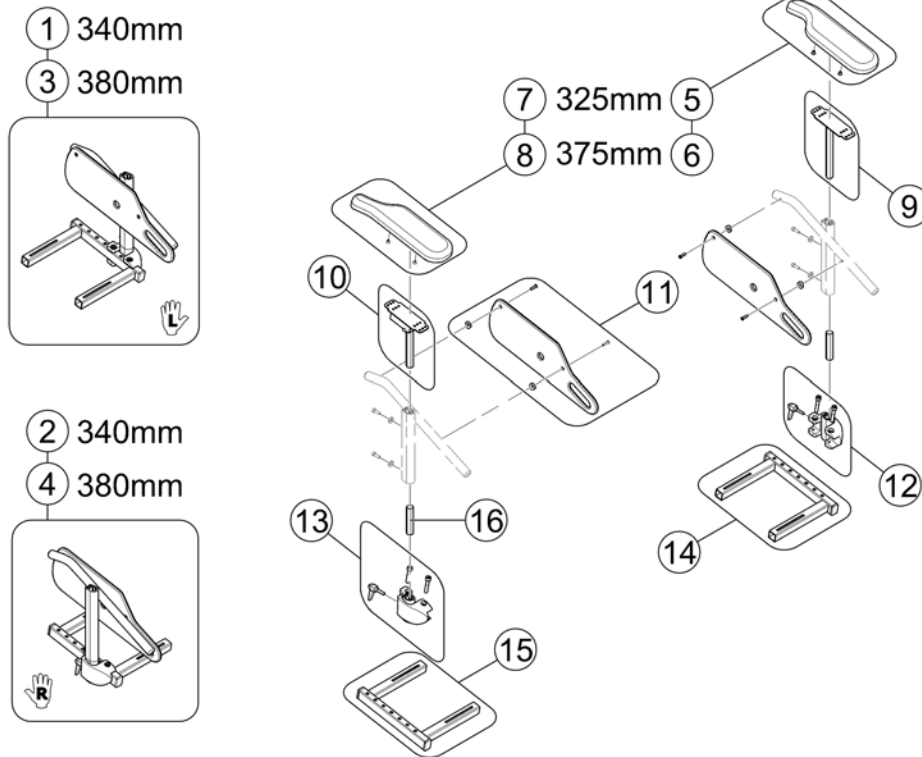

1	SP1583906	(Siderest 340mm LH, without Armrest - Max Seat)	1
2	SP1583907	(Siderest 340mm RH, without Armrest - Max Seat)	1
3	SP1583908	ARMLEN VE H 38 KPL MAX STORM4 MAX	1
4	SP1583909	ARMLEN HØ H38 KPL MAX STORM4 MAX	1
5	SP1443004	ARMLENSPOLSTER VE STORM STORM TDX BORA	1
6	SP1528846	ARMLENSPOLSTER VE LANG STORM TDX BORA	1
7	SP1449043	ARMLENSPOLSTER HØ STORM STORM TDX BORA	1
8	SP1528845	ARMLENSPOLSTER HØ LANG STORM4 TDX BORA	1
9	SP1510857	ARMLENSPOLSTER HOLDER VE STORM3/4 2009->	1
10	SP1510856	ARMLENSPOLSTER HOLDER HØ STORM3/4 2009->	1
11	SP1428078	KLESBESKYTTER STORM4 TDX	1
12	SP1541647	(Clamp, LH, cpl.)	1
13	SP1541646	(Clamp, RH, cpl.)	1
14	SP1537209	(Bracket, cpl., LH)	1
15	SP1537208	(Bracket, cpl., RH)	1
16	1422092-0002	(CONNECTION PIECE)	1


(Front wheels)

1546208.iso

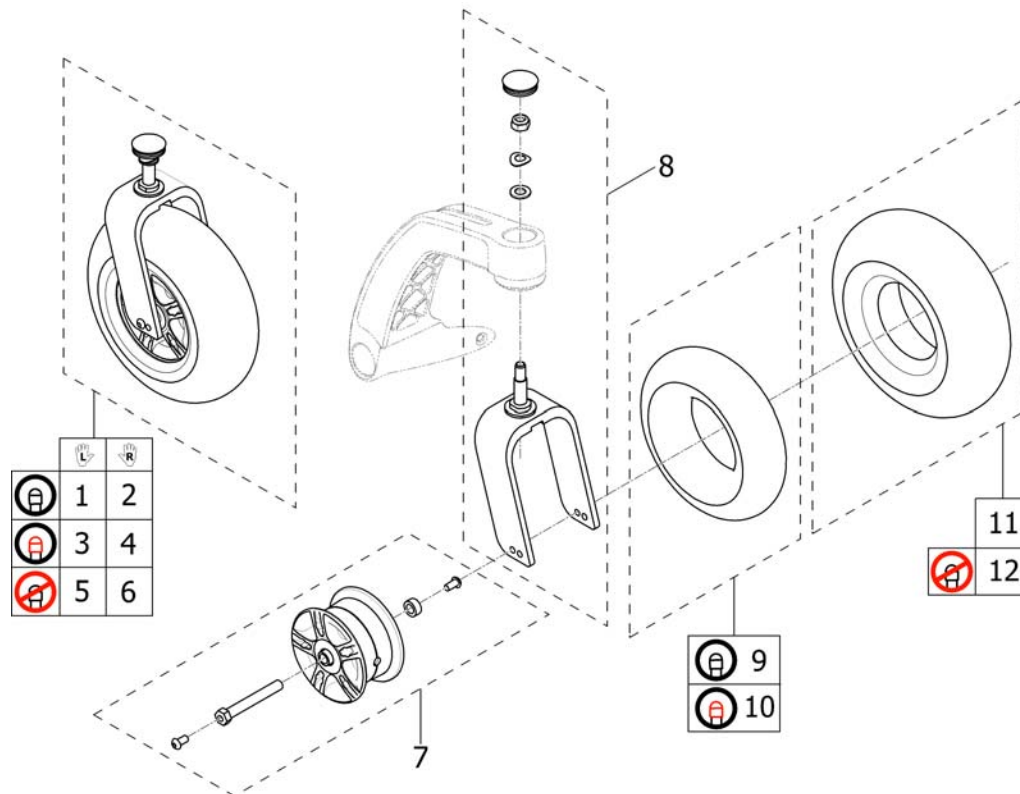

-	(New Wheel Design since 03/2013! Please order an additional Rim Kit for other side, if required.)	0
1	SP1546211 SVINGHJUL KPL LUFT VE STORM4 MAX <i>Design change 03/2013.</i>	1
2	SP1546208 SVINGHJUL LUFT KPL HØ STORM4 MAX <i>Design change 03/2013.</i>	1
3	SP1546213 (Front wheel, puncture protected, LH, complete) <i>Design change 03/2013.</i>	1
4	SP1546210 (Front wheel, puncture protected, RH, complete) <i>Design change 03/2013.</i>	1
5	SP1546212 SVINGHJUL KOMPAKT KPL STORM4 MAX <i>Design change 03/2013.</i>	1
6	SP1546209 SVINGHJUL KOMPAKT KPL STORM4 MAX <i>Design change 03/2013.</i>	1
7	SP1571812 (Frontwheel Rim Kit, complete) <i>Design change 03/2013. Replaces SP1552551</i>	1
8	SP1552552 (Fork, complete)	1
9	SP1546752 (Tube 3.00-4")	1
10	SP1552564 (Tube 3.00-4", puncture protected)	1
11	SP1546742 (Tire 3.00-4" grey)	1
12	SPM11960 DEKK FRAM PUNKTERINGSF	1


1522414.no

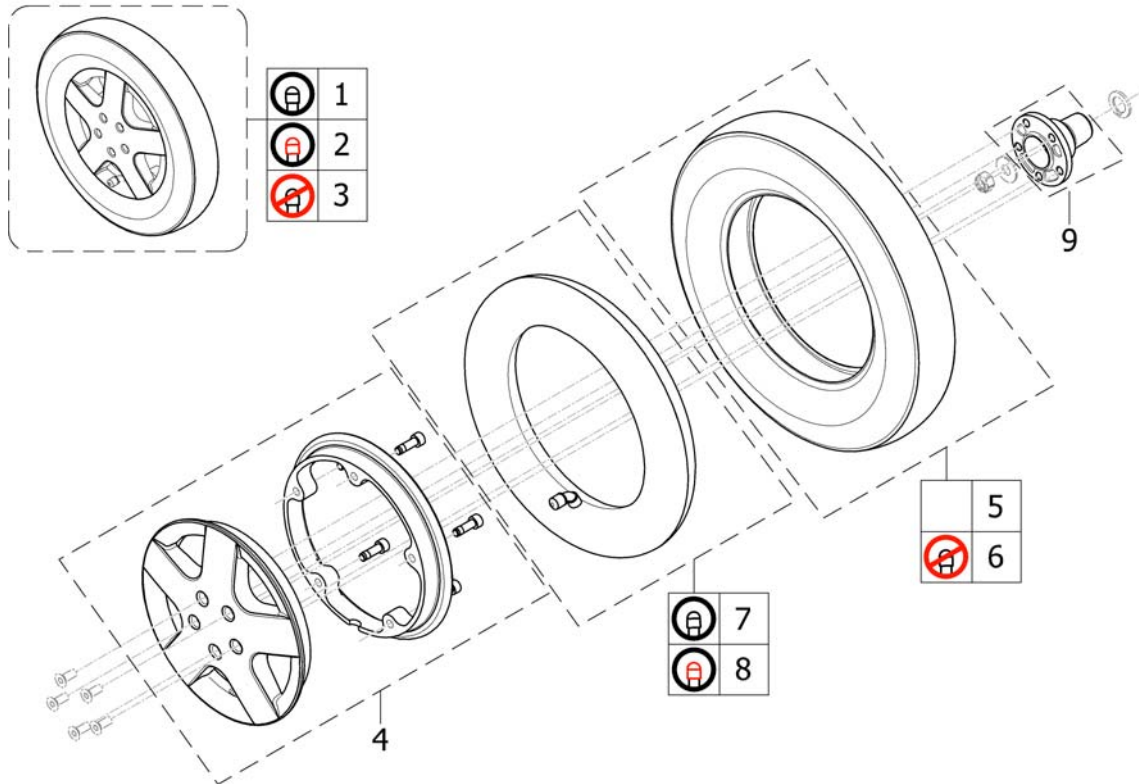

1	SP1532414	DRIVHJUL KPL M LUFT STORM4	1
2	SP1532415	DRIV HJUL PUNKT SIKKERT STORM4 KPL	1
3	SP1532416	HJUL PUNKTERINGSFRI STORM4	1
4	SP1541373	FELG KPL DRIVHJUL STORM4	1
5	SP1522940	DEKK LUFT 14X 3 (3,00-8) STORM4	1
6	SP1141342	DEKK KOMPAKT GB2 14X3 STORM4	1
7	SP1039651	(Tube 14")	1
8	SP1541374	(Tube 14" puncture protected)	1
9	SP1527821	(Wheel Hub 5-hole)	1


(Motor with swingarm)

1547873.00

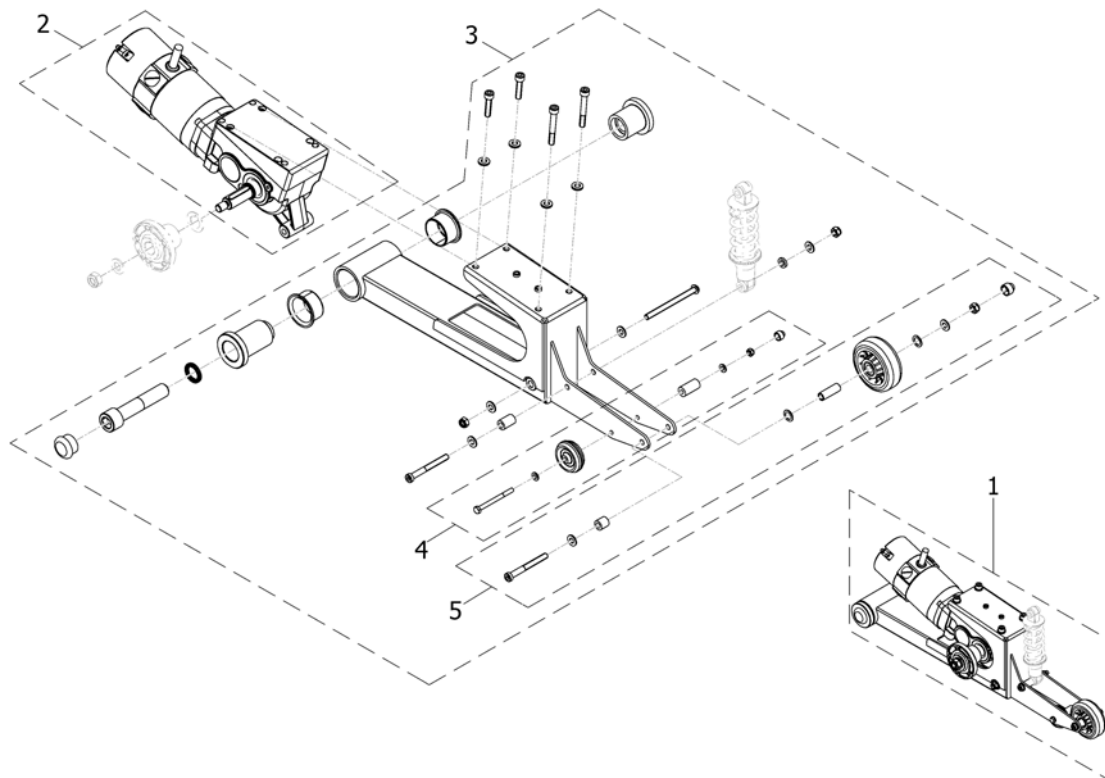

-	1510059	(Magnetbrake-kit RH)	1
-	1510064	(Magnetbrake-kit LH)	1
1	SP1547872	(Motor with swingarm, RH, cpl.)	1
1	SP1547873	(Motor with swingarm, LH, cpl.)	1
2	1440363	(Motor 6km/h (SRG0112) - Typhoon: left / Storm4 Max: right)	1
2	1440364	(Motor 6km/h (SRG0112) - Typhoon: right / Storm4 Max: left)	1
3	SP1552661	(Motorswingarm, LH, cpl.)	1
3	SP1552660	(Motorswingarm, RH, cpl.)	1
4	SP1552662	(Anti tipp wheel Ø50x20)	1
5	SP1552688	(Anti tipp wheel Ø80x30)	1




1548540X.dwg

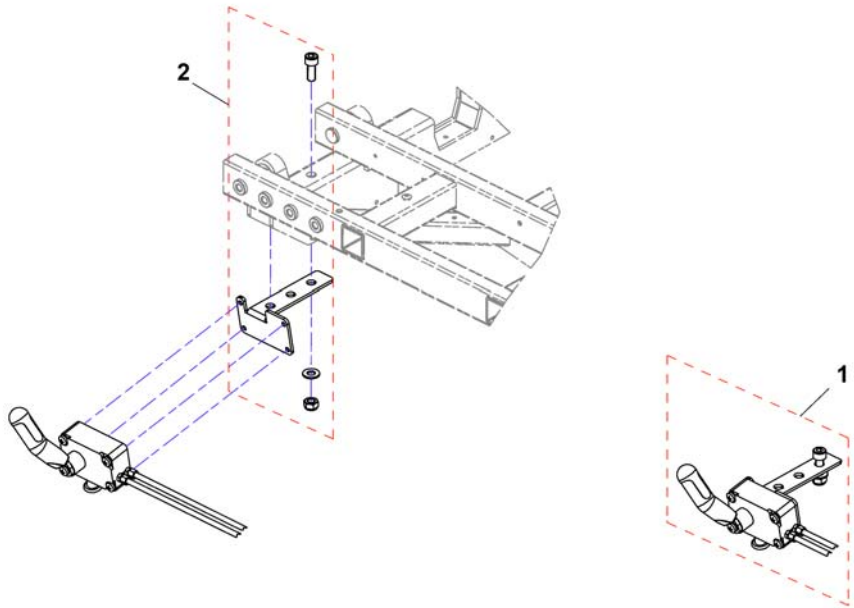

1478428	(ASSEMBLY KIT BOWDEN CABLE, CPL.) <i>not shown</i>	1
1 SP1548540	(Decoupling, cpl.)	1
2 SP1552567	(Declutching bracket, cpl.)	1

Elektronikk

1500784.no

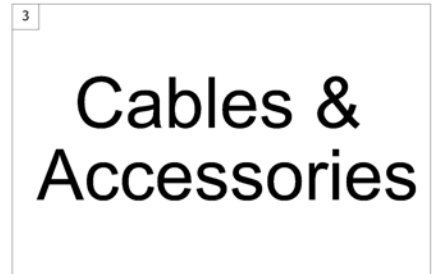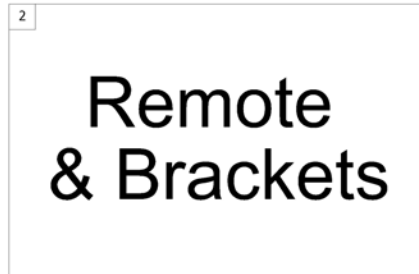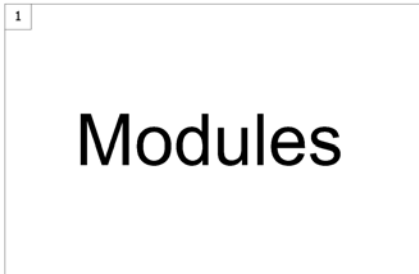


1	-	(ACS Electronic Modules)	1
2	-	(ACS Remote & Brackets)	1
3	-	(ACS Cables & Accessories)	1



-	(Power modules ACS 2)	1
-	(Power module holder / Gyro)	1
-	(ACT for ACS 2)	1
-	(G-Trac-sensor)	1
-	(Light board)	1

(Power modules ACS 2)

1524244X.dwg

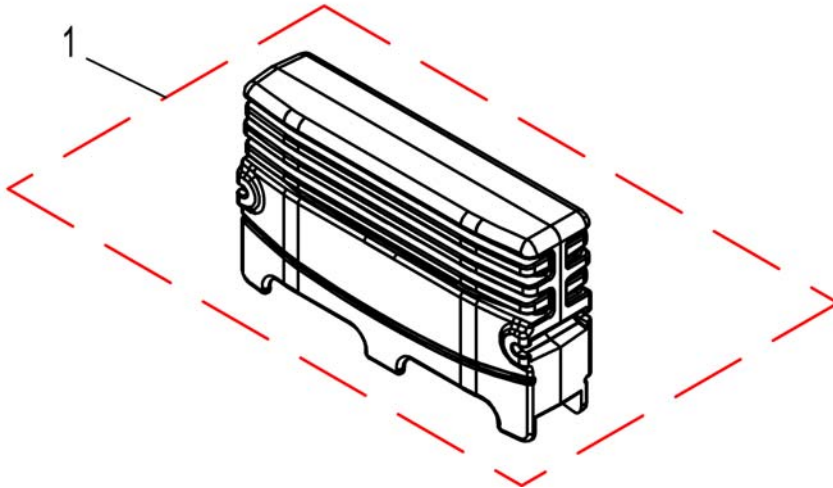

-	SP1555702	(Gasket kit for ACS2, Powermodule and ACT)	1
1	SP1524244	POWERMODUL DX2 STORM4 <i>Lock washer between module and bracket!</i>	1
1	SP1537975	POWERMODUL ACS2 FOR GYRO ACS2 <i>Lock washer between module and bracket!</i>	1

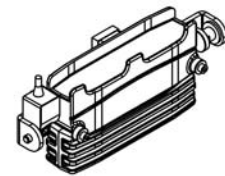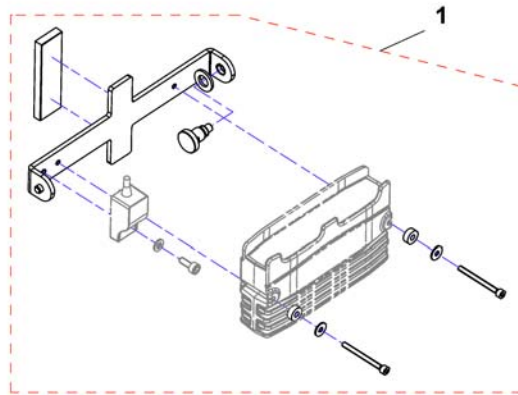


1 SP1546769

(Power module holder / Gyro)
Lock washer between module and bracket!


1

(ACT for ACS 2)

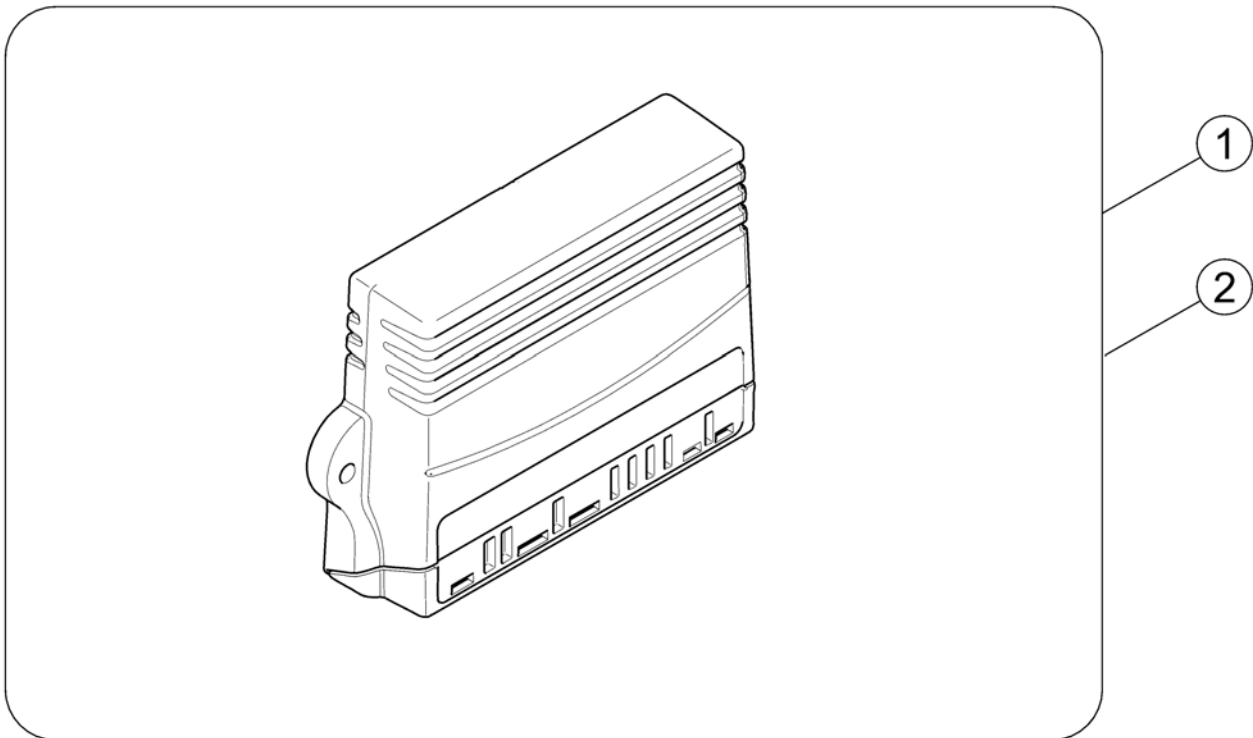


1	SP1528542	ACTUATORMODUL ACT2 DX2 STORM4 TDX BORA	1
2	SP1529382	ACTUATORMODUL ACT4 DX2 STORM4 TDX BORA	1

1537848X.dwg

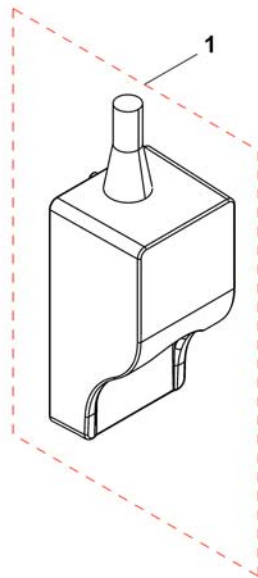


1 SP1541535 (G-Trac)

1

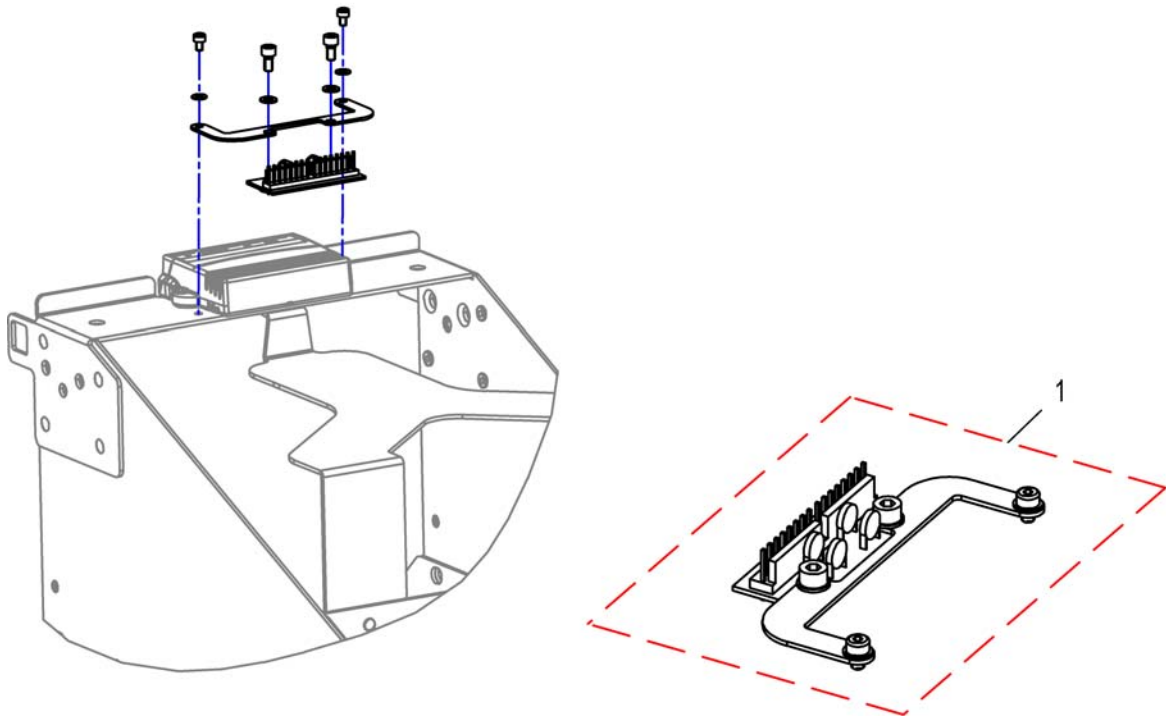


-	SP1524237	(Light cable ACS2)	1
1	SP1528890	LYSFORDELER STORM4	1

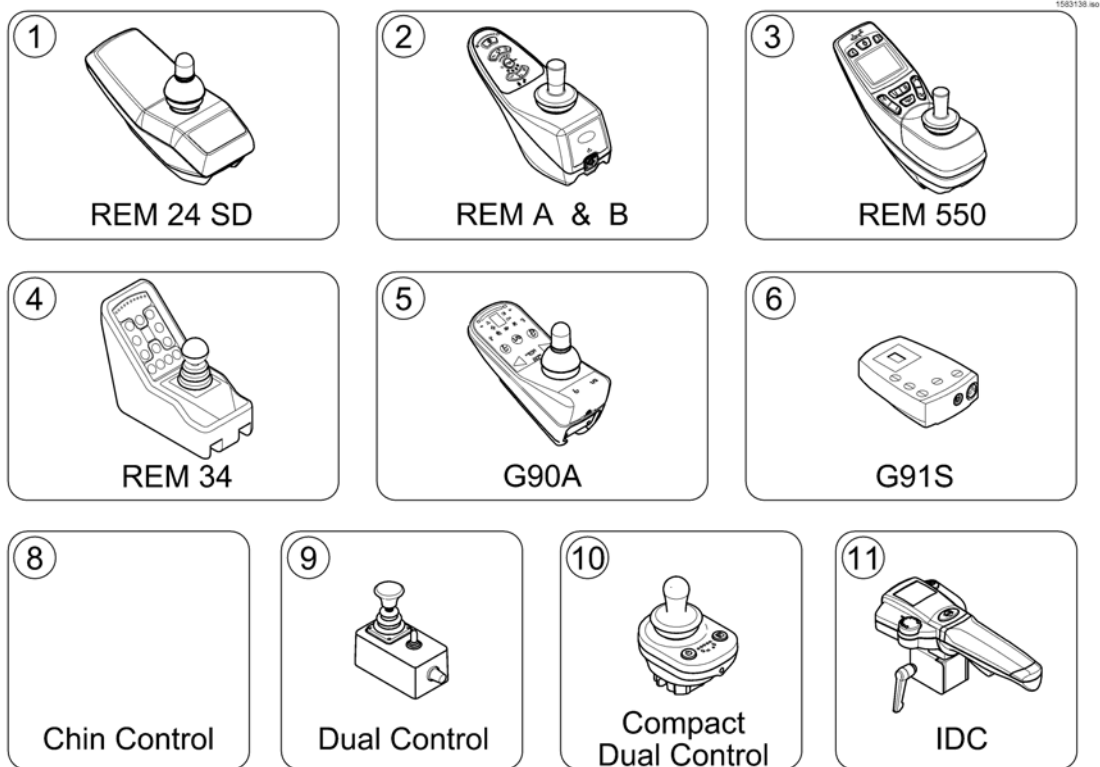


1	-	(REM 24 SD with brackets) [1601/1602]		1
2	-	(REM A & B with brackets) [1638/1639/1643/1644]		1
3	-	(REM 550 with brackets) [1617/1618]		1
4	-	(REM 34 with brackets) [1611]		1
5	-	(G90A with brackets) [1614/1616]		1
6	-	(G91S with brackets) [1615]		1
7	-	(Small Chin Control with brackets) [1609]		1
8	-	(Dual Control with brackets) [1291/1292]	→ 30.06.2015	1
9	-	(Compact Dual Control with brackets) [1720]	01.07.2015 →	1
10	-	(Intuitive Dual Control with brackets) [1645]		1

(REM 24 SD with brackets)

1562696 iso

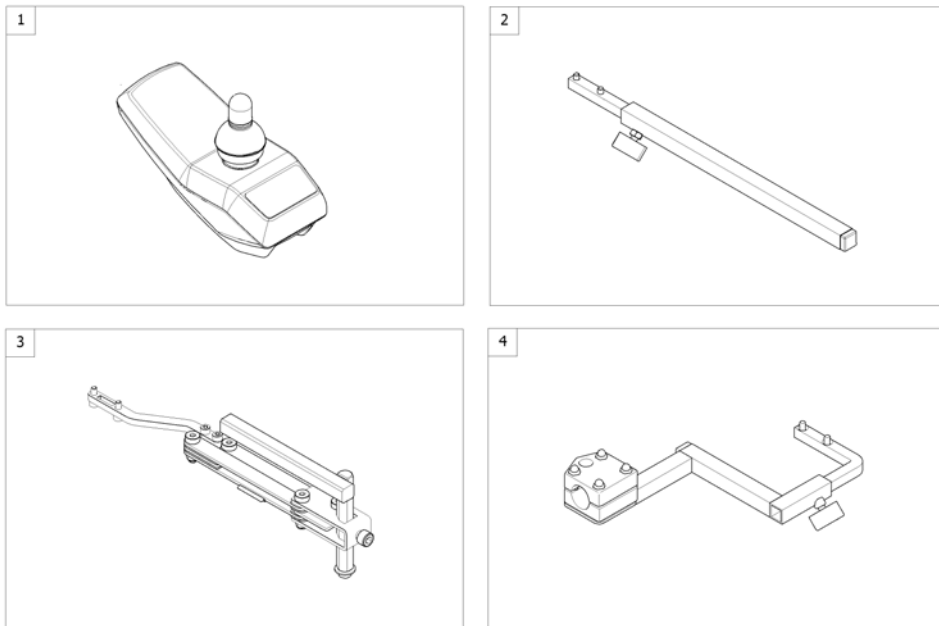


1	-	(Remote REM 24SD) <i>[1601/1602]</i>	0
2	-	(Remote Bracket Standard) <i>[1601+1207/1208]</i>	1
3	-	(Remote Bracket Swing away & height adjust.) <i>[1601+1707/1708]</i>	1
4	-	(Remote Bracket Attendant) <i>[1602+1207/1208]</i>	1

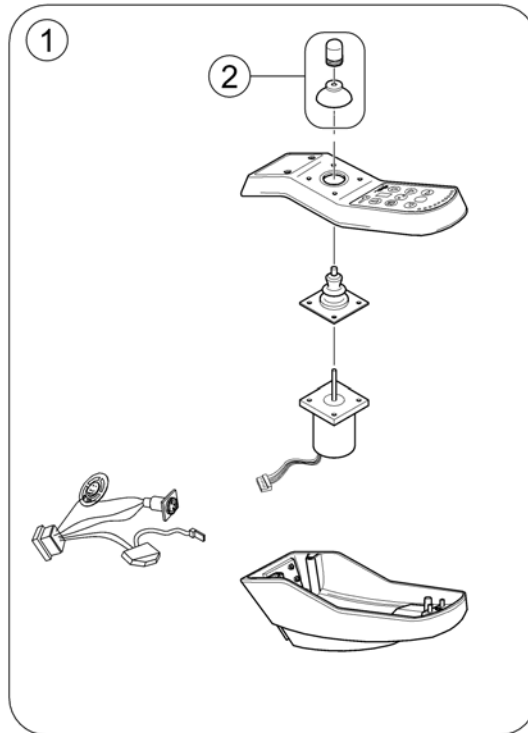

1	1436541	(Remote REM24SD)	1
2	SPZ183229	JOYSTIKK KOMPLETT	1

(Remote Bracket Standard)


1559851.iso

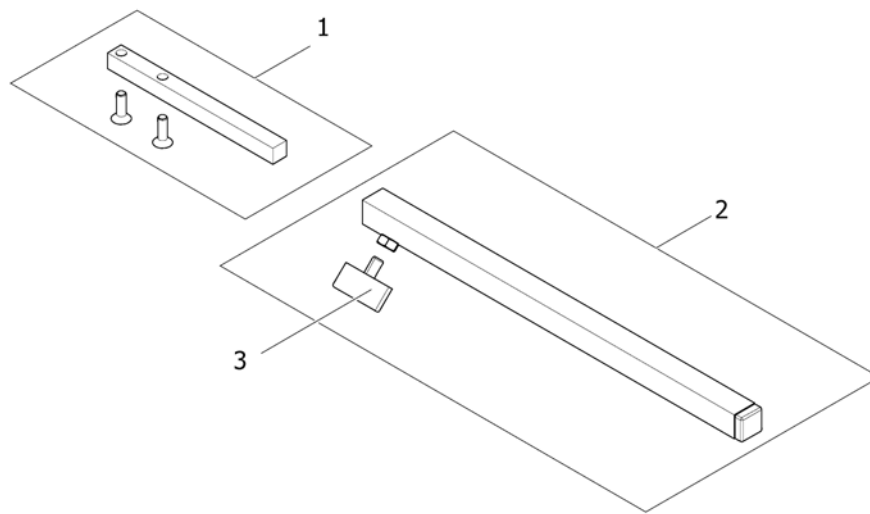

1	SP1436542	HOLDER DX-REM 24 SD RETT STORM SPORT/3 OPPFL.ARML	1
2	SPS183040	STYREBOKSHOLDER STD KPL STORM3 G40	1
3	L31001	VINGSKRUV M 6X10 GARANT G24 S	1

(Remote Bracket Swing away & height adjust.)


1559852.iso

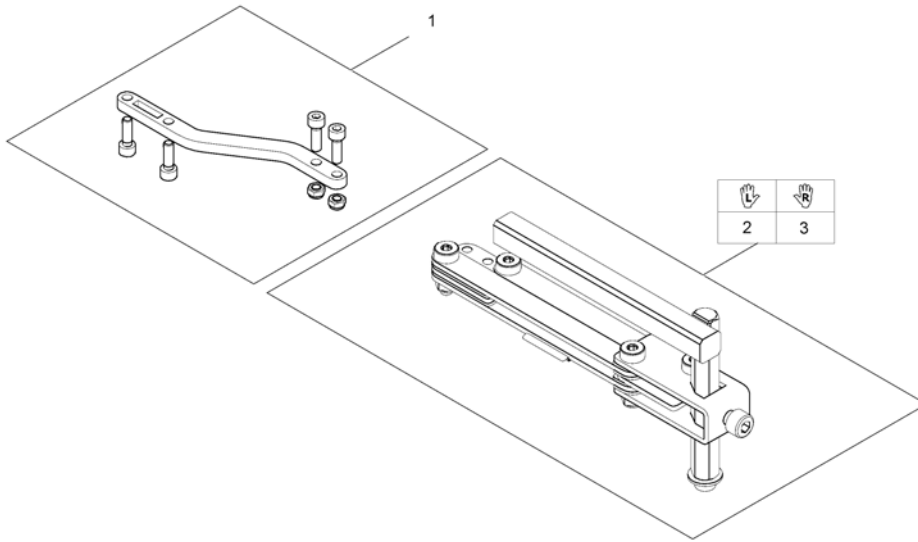


1	SP1498222	FESTE FOR STYREBOKST TIL SWING AWAY	1
2	SP1496249	SWING AWAY VE NY MOD2008 KPL	1
3	SP1496248	SWING AWAY HØ NY MOD2008 KPL	1

(Remote Bracket Attendant)

1560837.no

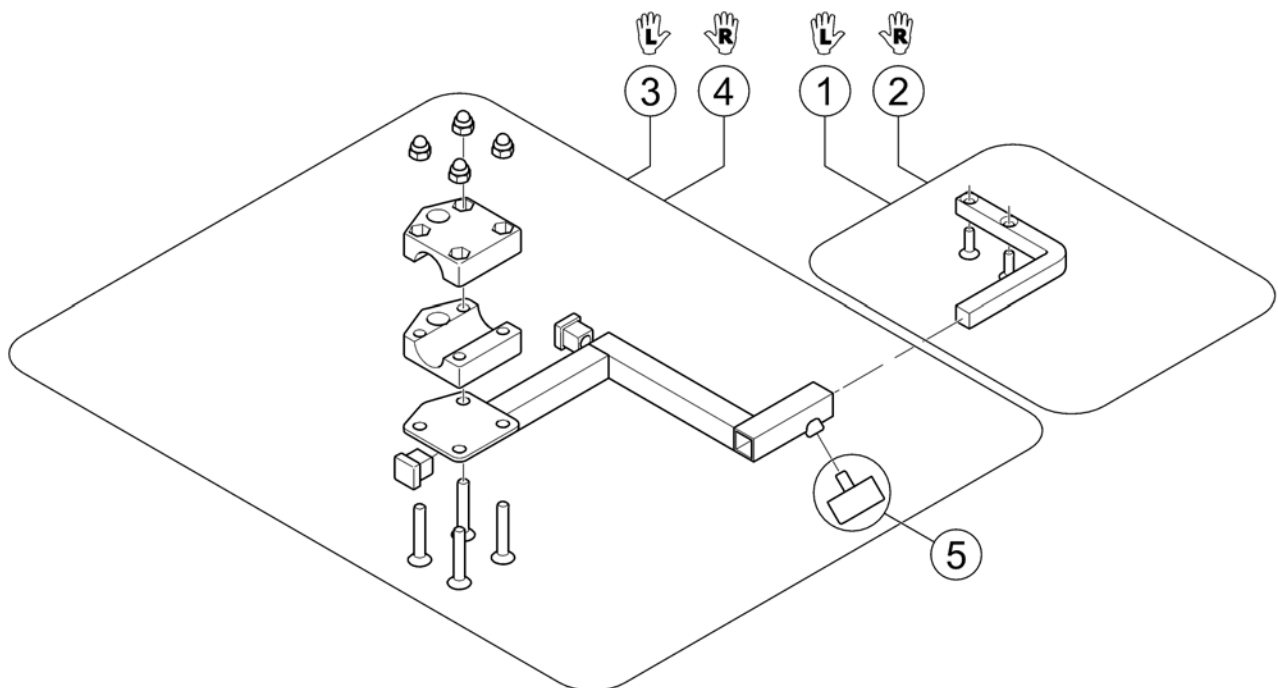


1	1422806	(Remote Support LH)	1
2	1422810	(Remote Support RH)	1
3	SP1422937	(Bracket for attendant control LH)	1
4	SP1422936	(Bracket for attendant control RH)	1
5	L31001	VINGSKRUV M 6X10 GARANT G24 S	1

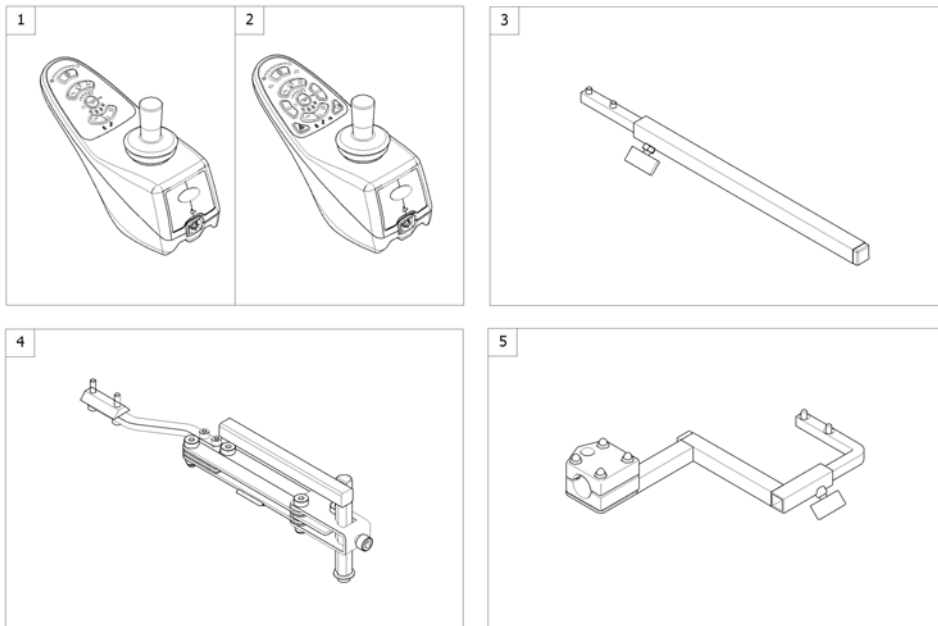


1	SP1523440	(Remote "REM A") [1638/1643]	1
2	SP1523445	(Remote "REM B") [1639/1644]	1
3	-	(Remote Bracket Standard) [1638/1639+1207/1208]	1
4	-	(Remote Bracket Swing away & height adjust.) [1638/1639+1707/1708]	1
5	-	(Remote Bracket Attendant) [1643/1644+1207/1208]	1

(Remote Bracket Swing away & height adjust.)



1559857.iso

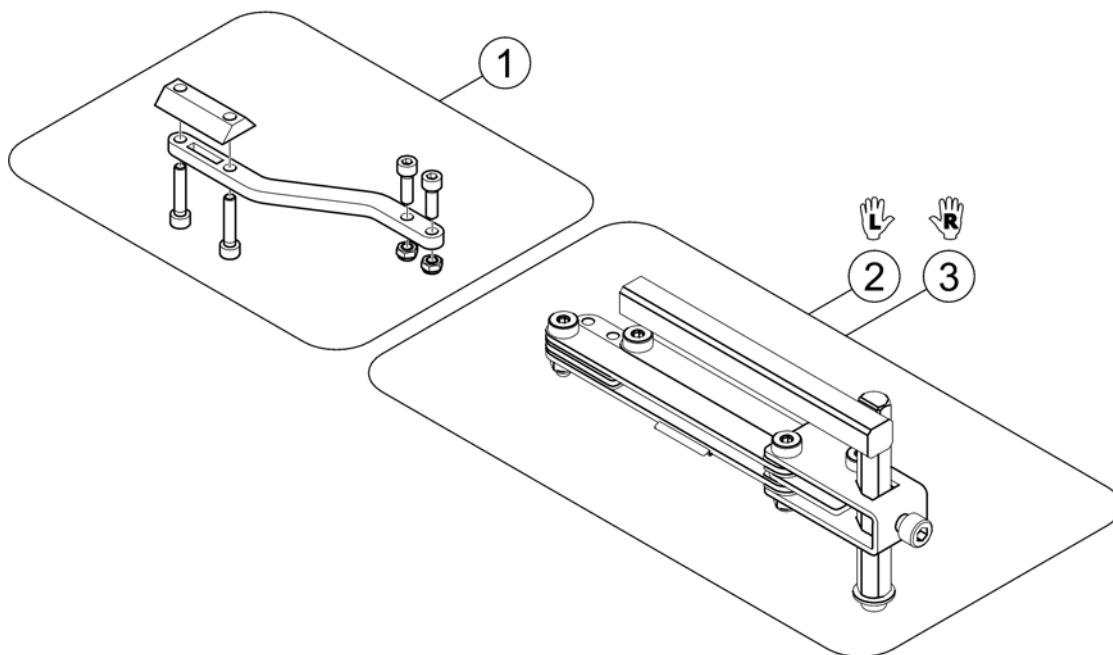


1	SP1536180	HOLDER STYREBOKS SHARK/REM A/REM B	1
2	SP1496249	SWING AWAY VE NY MOD2008 KPL	1
3	SP1496248	SWING AWAY HØ NY MOD2008 KPL	1

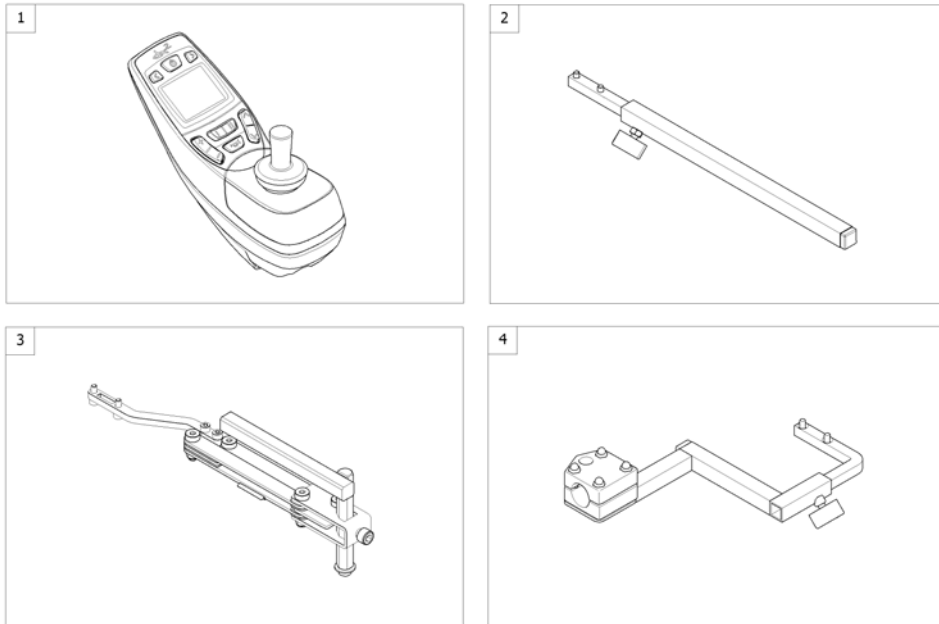

1	SP1523444	STYREBOKS REM550 STORM4 TDX BORA [1617/1618]	1
2	-	(Remote Bracket Standard) [1617+1207/1208]	1
3	-	(Remote Bracket Swing away & height adjust.) [1617+1207/1208]	1
4	-	(Remote Bracket Attendant) [1618+1207/1208]	1

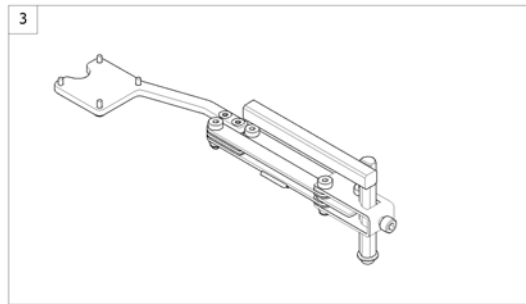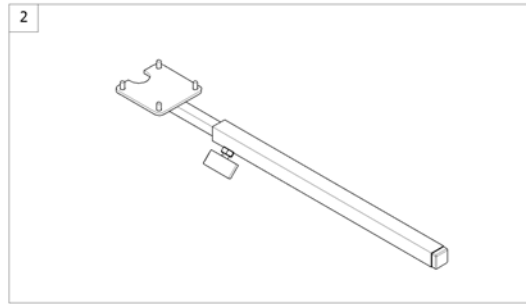


1	SP1440677	(Remote REM 34) [1611]	1
2	-	(Remote Bracket Standard) [1611+1207/1208]	1
3	-	(Remote Bracket Swing away & height adjust.) [1611+1707/1708]	1

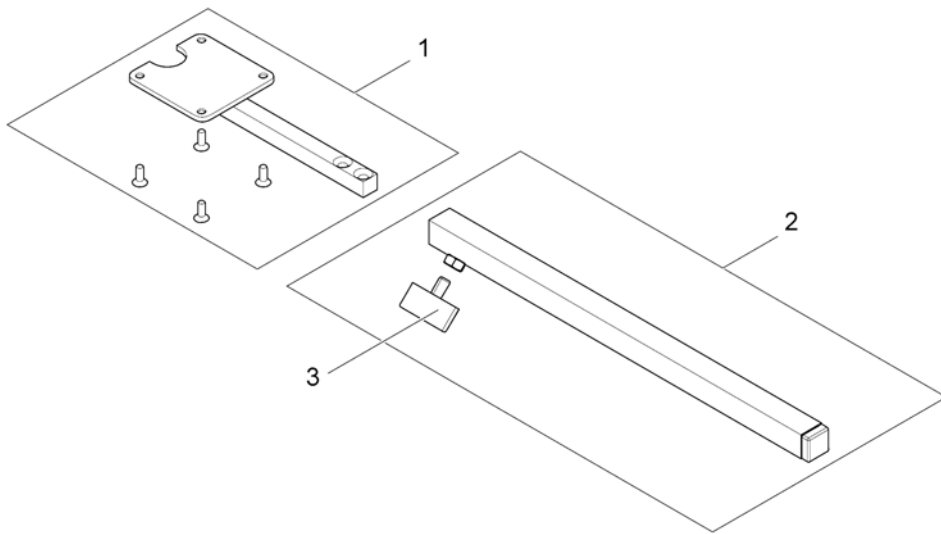


1	SP1434921	HOLDER FOR REM 34 STORM4	1
2	SPS183040	STYREBOKSHOLDER STD KPL STORM3 G40	1
3	L31001	VINGSKRUV M 6X10 GARANT G24 S	1

(Remote Bracket Swing away & height adjust.)

1560845.iso

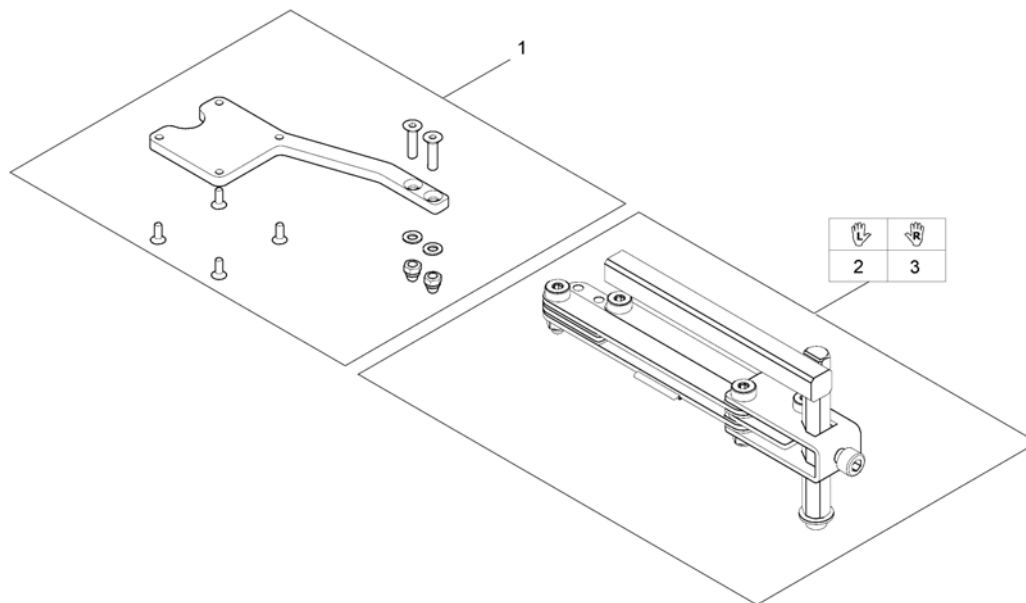


1	SP1527125	FESTE FOR REM34 STORM4	1
2	SP1496249	SWING AWAY VE NY MOD2008 KPL	1
3	SP1496248	SWING AWAY HØ NY MOD2008 KPL	1

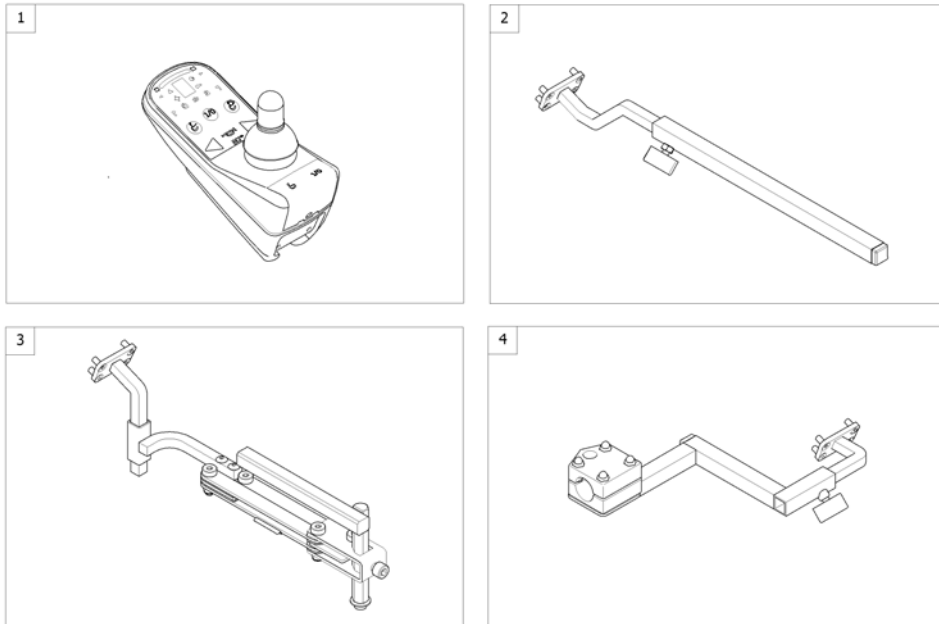

1	SP1504720	STYREBOKS DX-REM G90A STORM4 [1614/1616]	1
2	-	(Remote Bracket Standard) [1614+1207/1208]	1
3	-	(Remote Bracket Swing away & height adjust.) [1614+1707/1708]	1
4	-	(Remote Bracket Attendant) [1616+1207/1208]	1

(Remote Bracket Standard)


1562939.iso

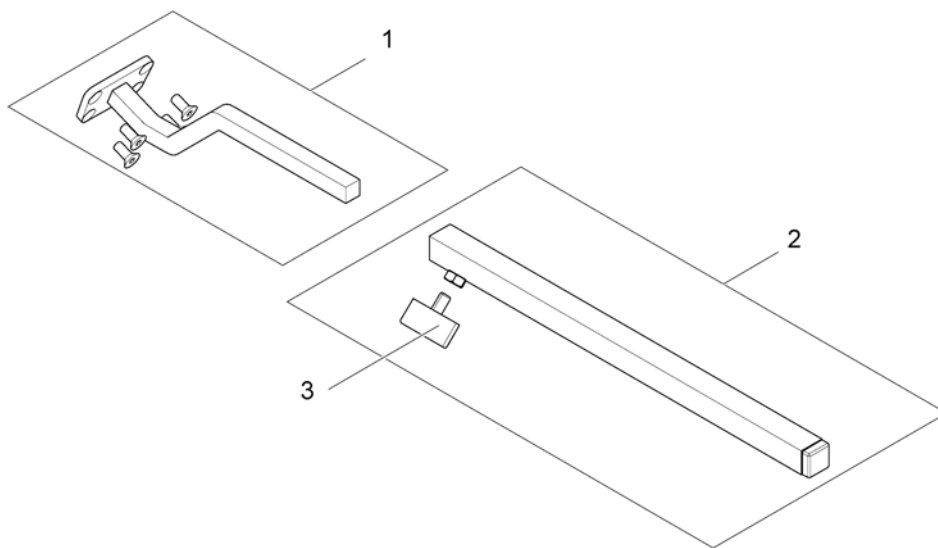

1	SP1504822	(Remote Support Standard)	1
2	SPS183040	STYREBOKSHOLDER STD KPL STORM3 G40	1
3	L31001	VINGSKRUV M 6X10 GARANT G24 S	1

(Remote Bracket Swing away & height adjust.)


1560310.iso

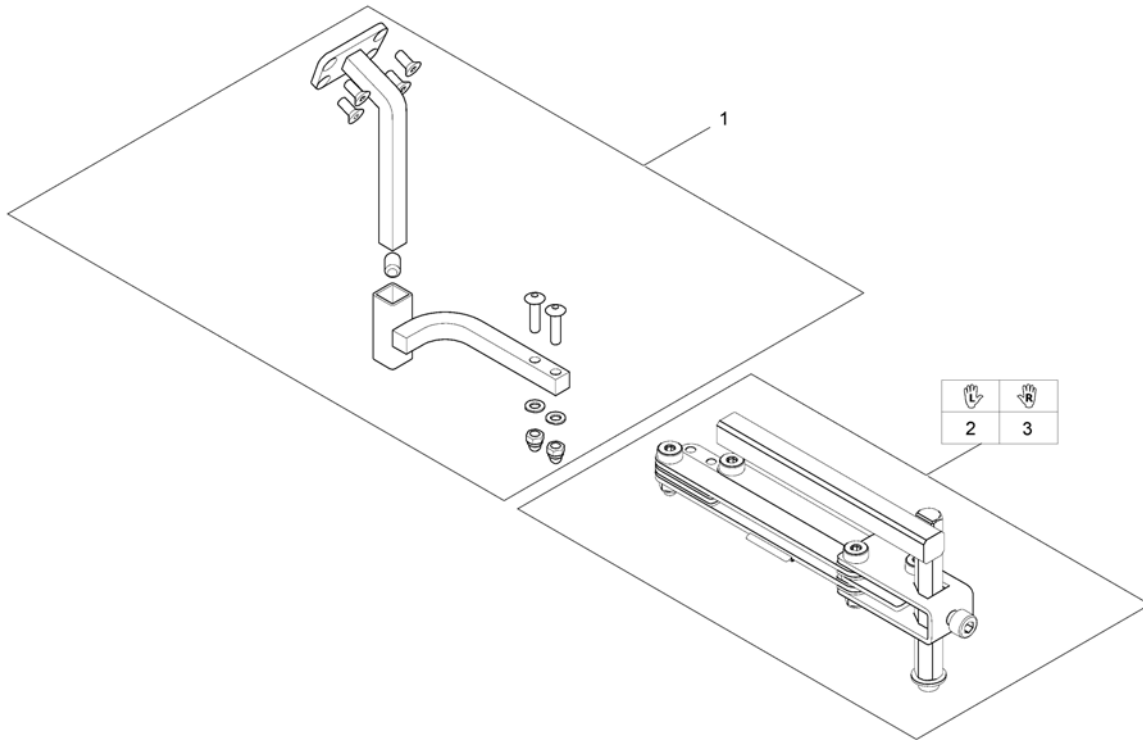



1	SP1517352	FESTE FOR G90 SWINGAWAY STORM4 TDX BORA	1
2	SP1496249	SWING AWAY VE NY MOD2008 KPL	1
3	SP1496248	SWING AWAY HØ NY MOD2008 KPL	1

(Remote Bracket Attendant)

1560853.iso

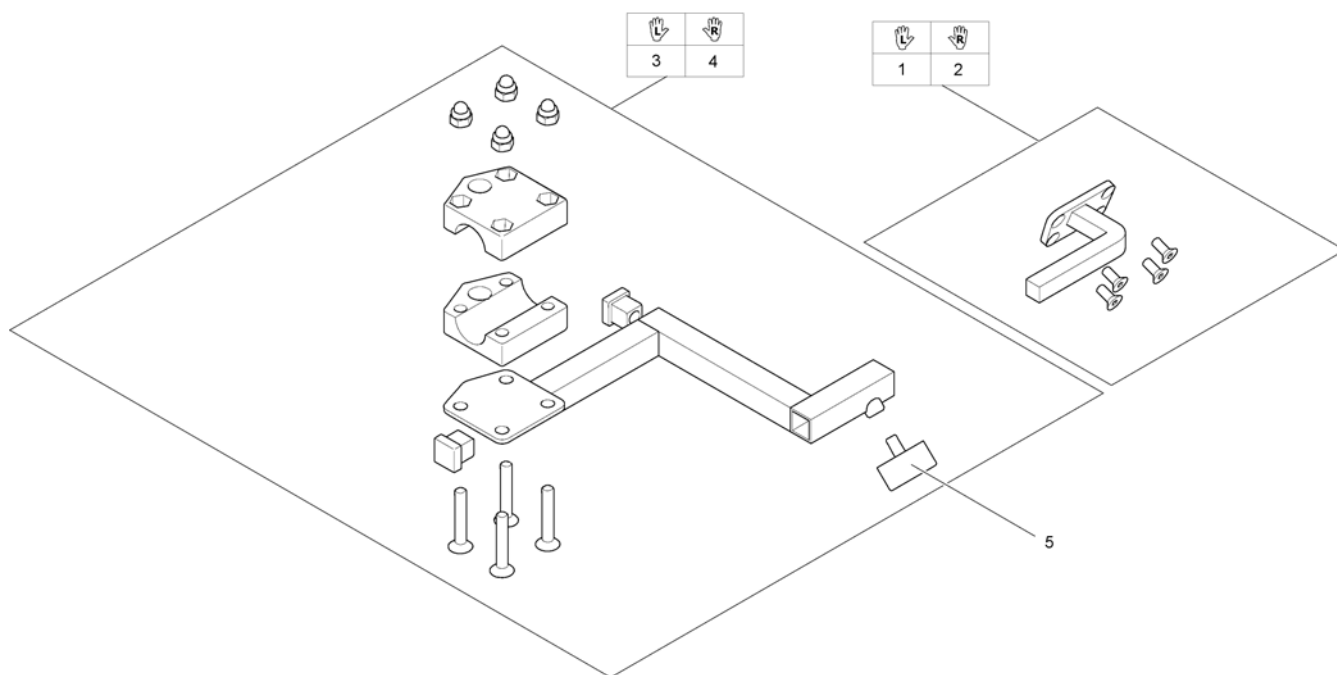


1	SP1509435	(Remote Support attendant control G90A, LH)	1
2	SP1509439	(Remote Support attendant control G90A, RH)	1
3	SP1422937	(Bracket for attendant control LH)	1
4	SP1422936	(Bracket for attendant control RH)	1
5	L31001	VINGSKRUV M 6X10 GARANT G24 S	1

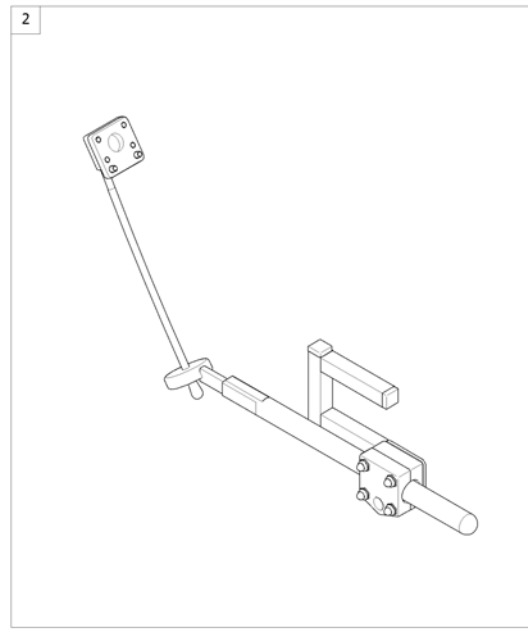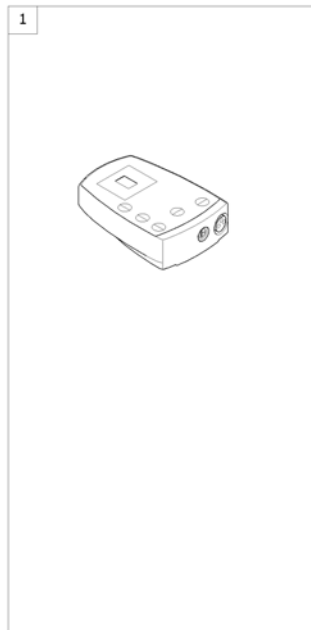

1	SP1505187	(Remote Control G91S) [1615]	1
2	-	(Remote Bracket Standard) [1615+1207/1208]	1

(Remote Bracket Standard)


1560654.iso

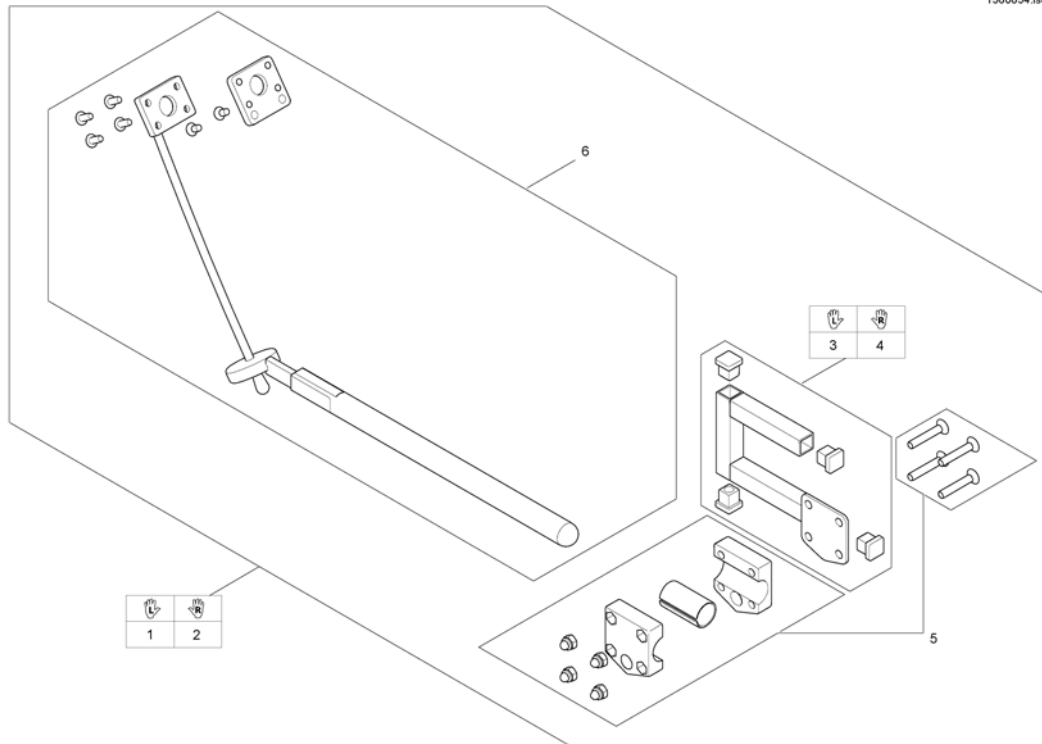

1	SP1509718	HOLDER FOR DX-REMG91 VE JUSTERBAR HØYDE/DYBDE	1
2	SP1509690	HOLDER FOR DX-REMG90 HØ JUSTERBAR HØYDE/DYBDE	1
3	SP1509716	(Adapter G91 LH)	1
4	SP1509711	(Adapter G91 RH)	1
5	SP1527503	(Gripper clamp cpl.)	1
6	SP1527504	(Swan neck remote bracket with adapter plate)	1


-	(Small Chincontrol)	1
-	(Framework for chincontrol)	1

(Small Chincontrol)

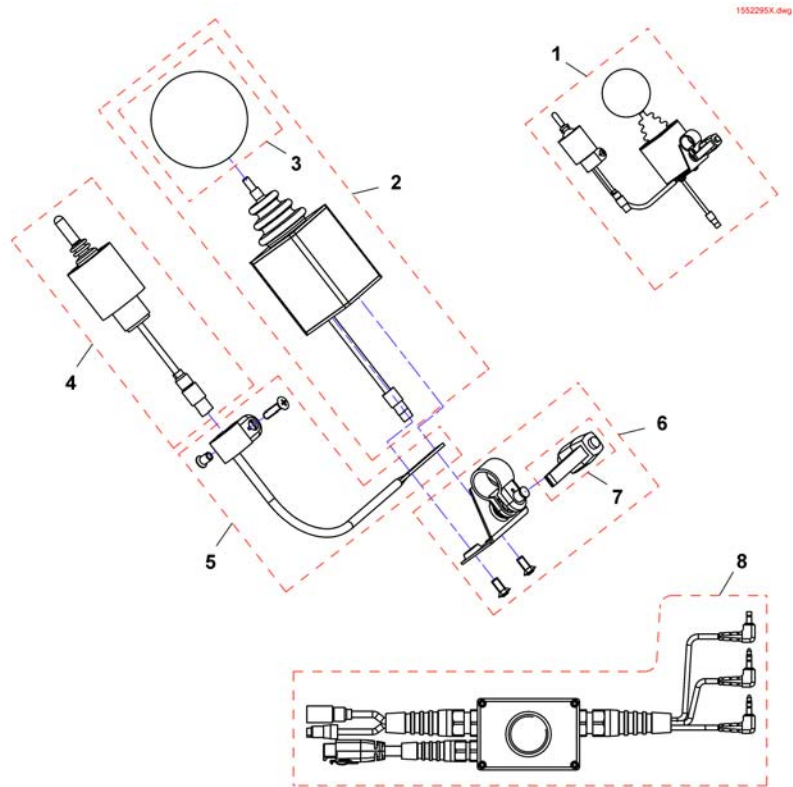

1	1448705	\$HAKESTYRING KPL MONT EKSTRA JOYSTICK	1
1	1421770	(Chin control G80/G90, cpl.)	1
1	SP1544255	(Chin control G91, cpl.)	1
2	Z183282	(Joystick, kmpl.)	1
3	Z183265	BRUK GJS26826 DYNAMICS	1
4	1442943	(Switchjoystick REM24, kmpl.)	1
4	Z183283	(Switchjoystick G80/G90, cpl.)	1
4	SP1544250	(Switchjoystick G91, kmpl.)	1
5	SP1552294	(Switchjoystick bracket)	1
6	SP1552295	(Joystick clamp)	1
7	L32901	SPAK M8 STORM4	1
8	SP1442944	(Harness REM24)	1
8	Z183275	(Harness G80/G90)	1
8	SP1541715	(Harness G91)	1



(Framework for chincontrol)

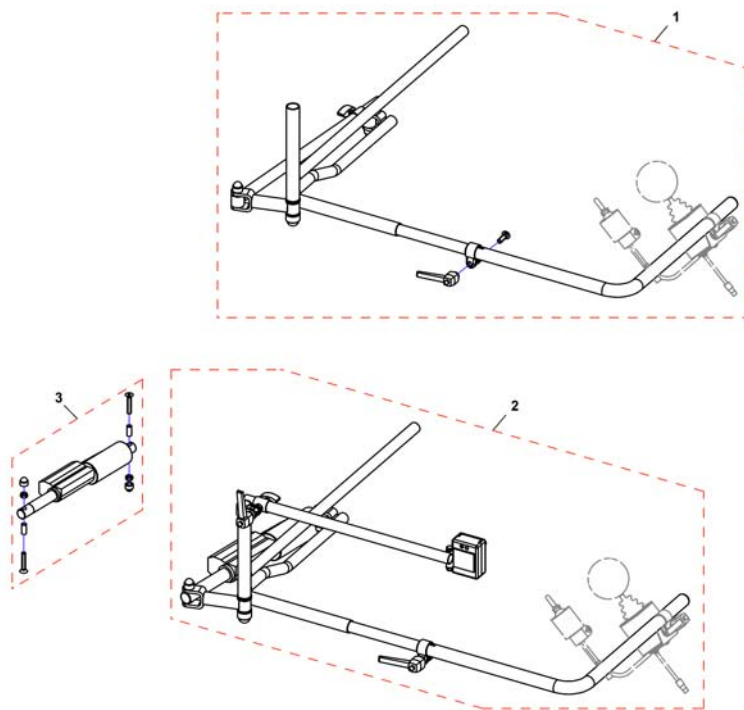


-	Z167994	(CHIN CONTROL SUPPORT . CPL.) <i>Not shown / Nicht abgebildet / Non illustré</i>	1
1	SP1422943	(Framework for chin control, manual, horizontal swing away, LH, cpl.)	1
1	SP1422942	(Framework for chin control, manual, horizontal swing away, RH, cpl.)	1
2	SP1422945	HOLDER HAKEBETJ EL HORIS SVINGAWAY VE KPL <i>incl. Actuator</i>	1
2	SP1422944	(Framework for chin control, powered, horizontal swing away, RH, cpl.) <i>incl. Actuator</i>	1
-	SP1552335	(Hardware, chin control, powered, horizontal swing away, LH) <i>Not shown / Nicht abgebildet / Non illustré</i>	1
-	SP1552336	(Hardware, chin control, powered, horizontal swing away, RH) <i>Not shown / Nicht abgebildet / Non illustré</i>	1
-	SP1422946	(Framework for chincontrol, powered, vertikal swing away) <i>Not shown / Nicht abgebildet / Non illustré</i>	1
3	SP1552337	(Actuator for horizontal swing away chin control, cpl.)	1
3	SP1552418	(Gestell für Kinnbedienung, elektrisch, vertikal abschenkbar) <i>Not shown / Nicht abgebildet / Non illustré</i>	1

(Dual Control with brackets)

1562893 iso

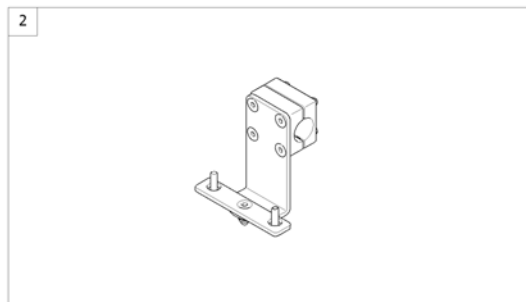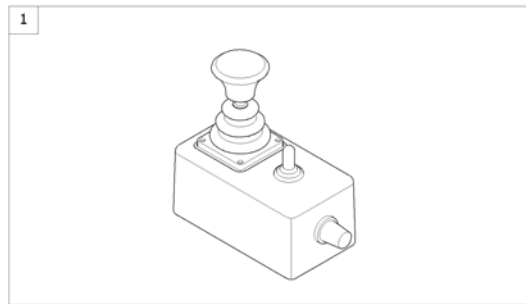

1	6001425	(ACS dual control) [1291/1292]	1
2	-	(Remote Bracket Dual Control) [1291/1292]	1

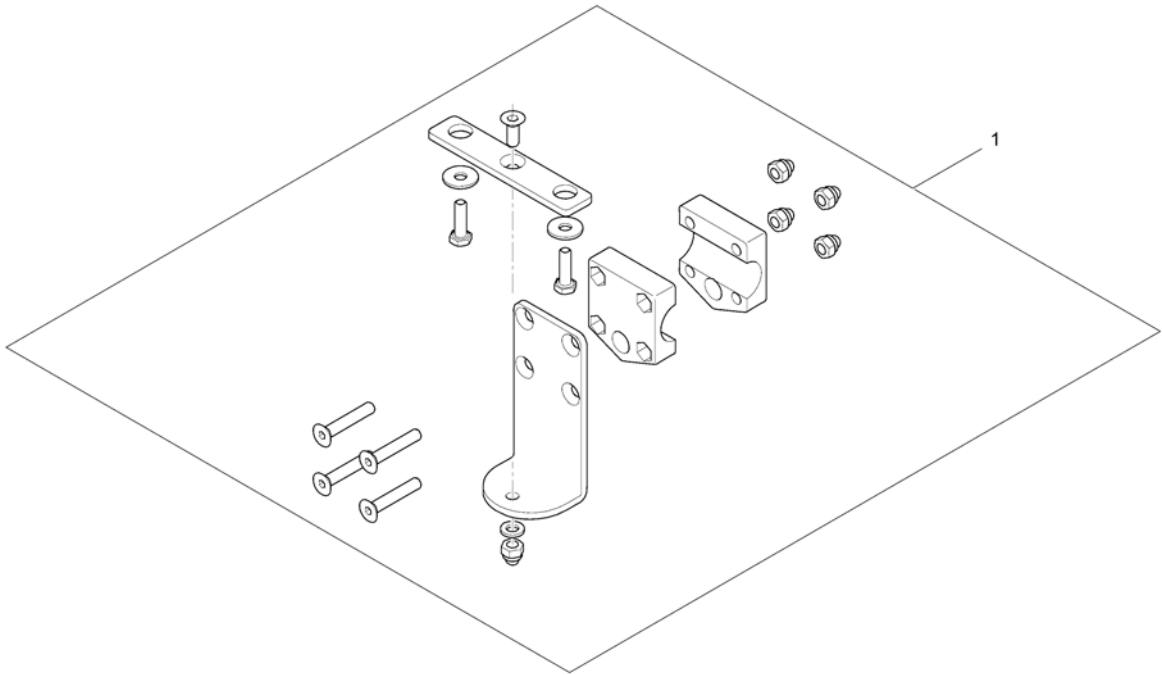


1 SP1422818

FESTEBRAKETT LEDSAGERSTY RYGGMONT ENEKL


1

(Compact Dual Control with brackets)

1588802 Rev

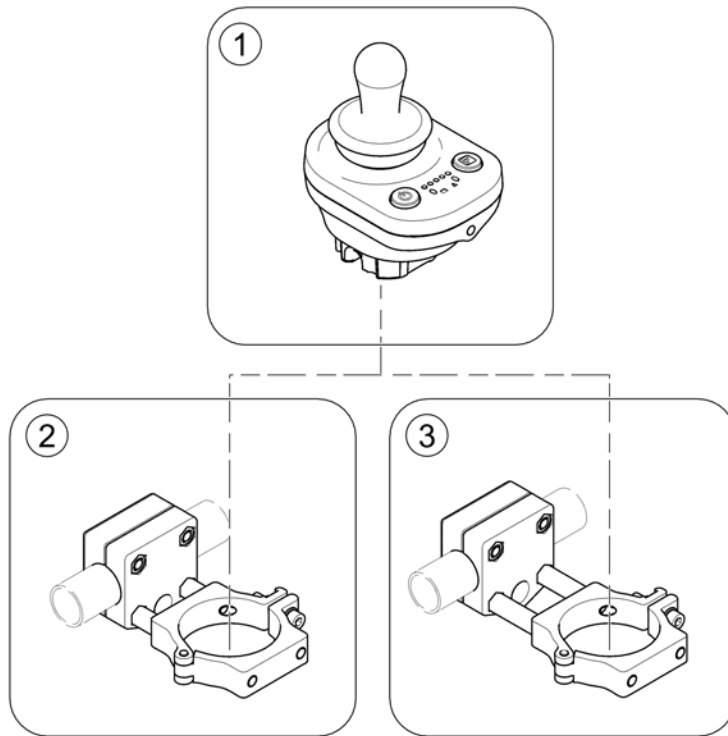

1	-	(Compact Dual Control)	1
2	-	(Bracket, Compact Dual Control - short)	1
3	-	(Bracket, Compact Dual Control - long)	1

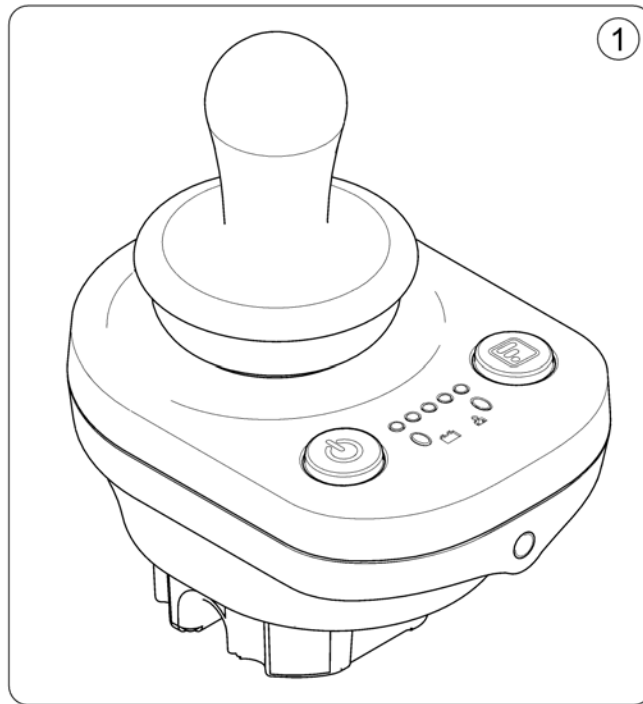


1 SP1586988

(Remote DX2-ACU)

1

(Bracket, Compact Dual Control - short)

1588857.jpg

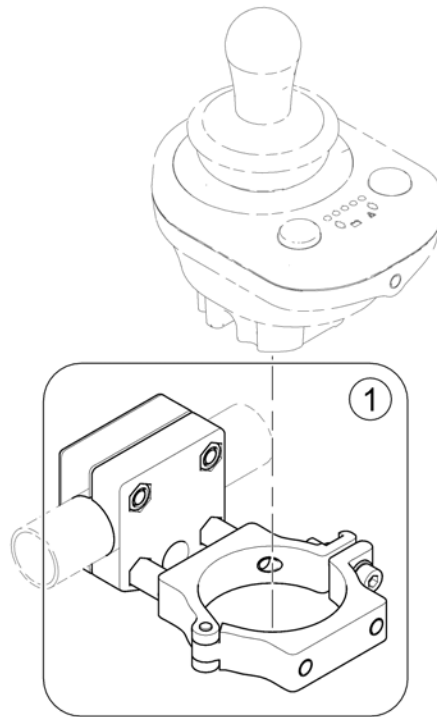


1 SP1587904

(Remote Attachment - short)

1

(Bracket, Compact Dual Control - long)

158856 iso

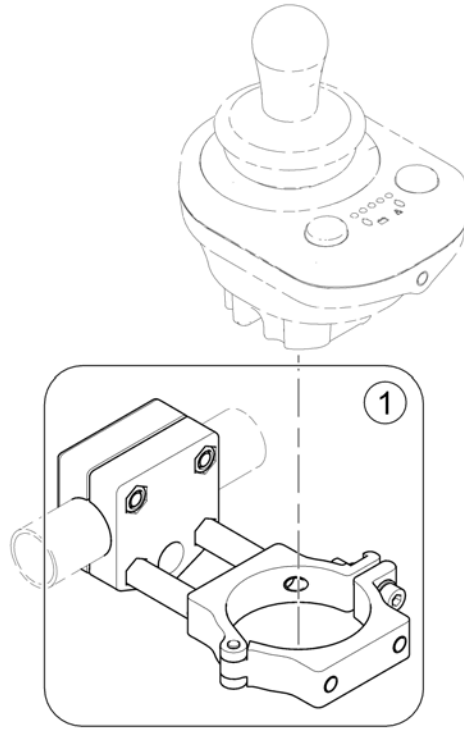


1 SP1587119

(Remote Attachment - long)

1

(Intuitive Dual Control with brackets)

1509875X.iso

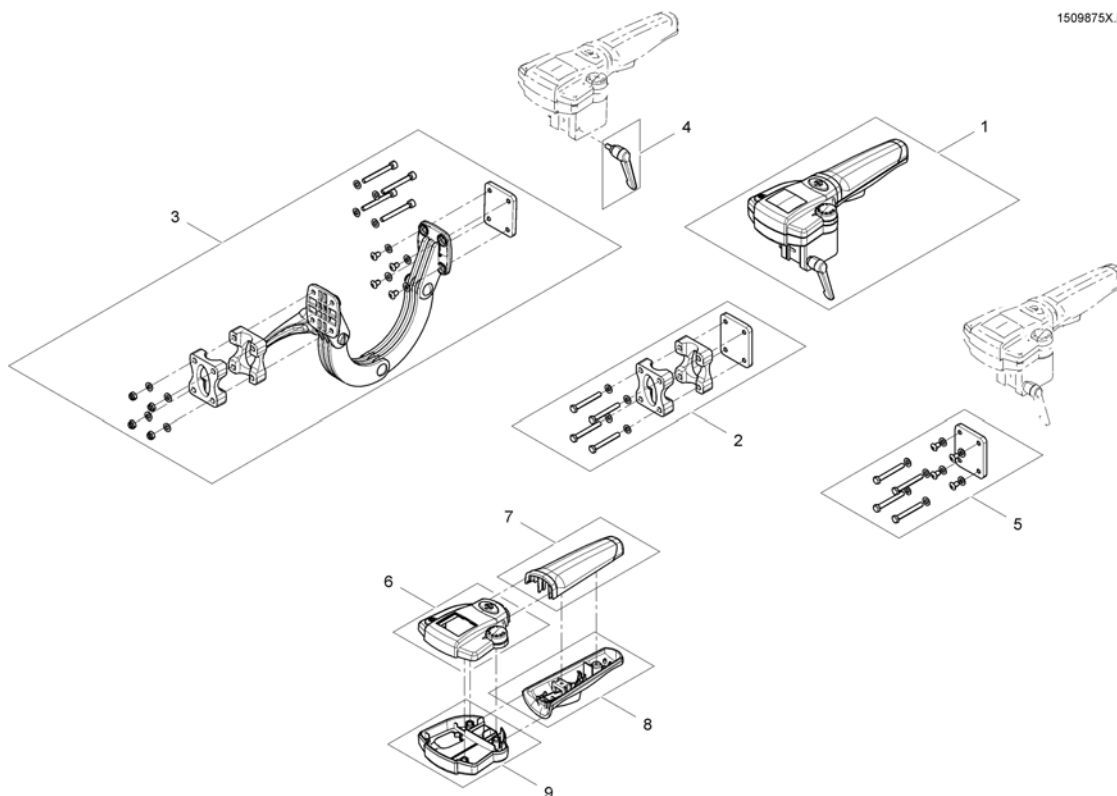


1	SP1509875	LEDSAGERSTYRING INTUITIV ALLE STOLER MED ACS <i>For chairs built before 01/02/10 please also order SP1537148</i>	1
2	SP1522310	KLEMME RYGGRØR IDC FESTE INTUITIV LEDESAGERSTYRING	1
3	SP1521488	HOLDER INTUITIV LEDESAGER RYGGER MED RUNDT RØR	1
4	SP1510135	FESTE IDC MED FINGERSKR STORM4	1
5	SP1537148	(Supportplate IDC for chairs before 01.02.10)	1
6	SP1579573	(Upper Housing)	1
7	SP1511060	(Handhold upper part)	1
8	SP1511099	(Handhold lower part)	1
9	SP1510470	DEKSEL INT LEDESAGERST ACS NEDRE DEKSEL	1

-	(Bus-Cables ACS)	1
-	(T-Adapter ACS)	1
-	(Slow-stop-sensor & Holder)	1
-	(Programming Kits)	1
-	(Magnetkey for Remotes)	1

(Bus-Cables ACS)


ACSBUSKABEL.iso

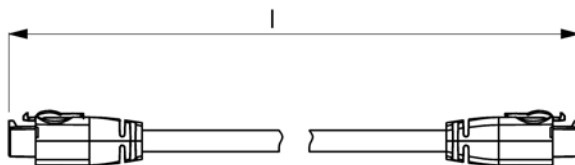

Position	length l [mm]
1	300
2	500
3	700
4	1100
5	1700
6	2000
7	2500
8	2700
9	1060

1	GSM63003	KABEL BUS 0,3M ACS,	1
2	SPGSM63005	(Bus cable ACS, 500mm) <i>Replace GSM63004</i>	1
3	GSM63007	(Bus cable ACS 700mm)	1
4	GSM63011	KABEL-BUS 1,1M ACS, ST utgått bruk 63010	1
5	1559862	KABEL BUS 1,7 M <i>Replace GSM63016</i>	1
6	GSM63020	DX BUS CABLE 2.0M STORM <i>Replace 1419386</i>	1
7	1440424	(Bus cable ACS, 2500mm)	1
8	SP1559873	(Bus cable ACS, 2700mm) <i>Replace GSM63026</i>	1
9	SP1550402	(Extension cable ACS, 1060mm)	1


1534069X.dwg

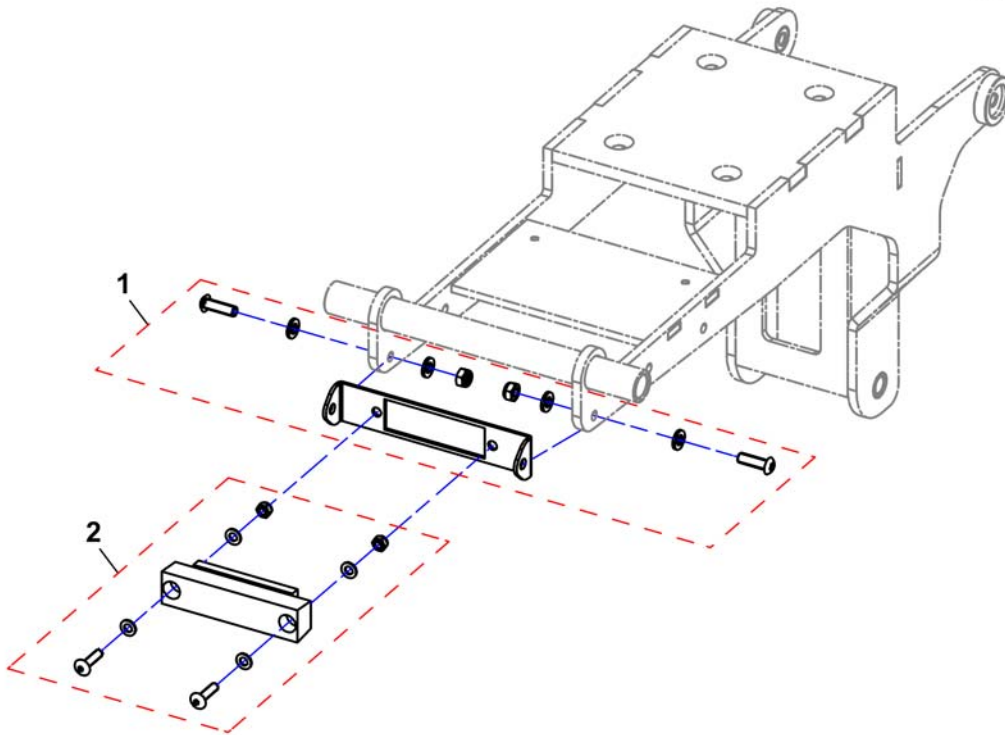


-	GSM63003	KABEL BUS 0,3M ACS,	1
1	SP1537966	HOLDER DX-SKT4 STORM4	1
2	SP1537967	FORGRENING BUSS SKT4 ACS	1

(Slow-stop-sensor & Holder)


1546440X.dwg

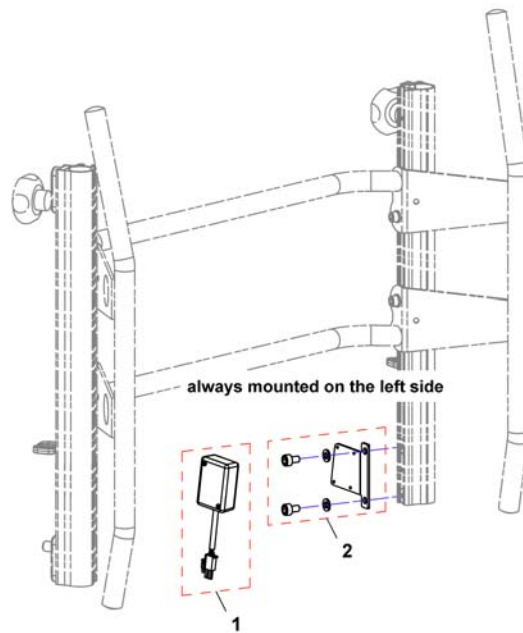

1	SP1533662	(Slow-stop-sensor, ACS2, cpl.)	1
2	SP1546433	(Slow-stop-sensor bracket)	1

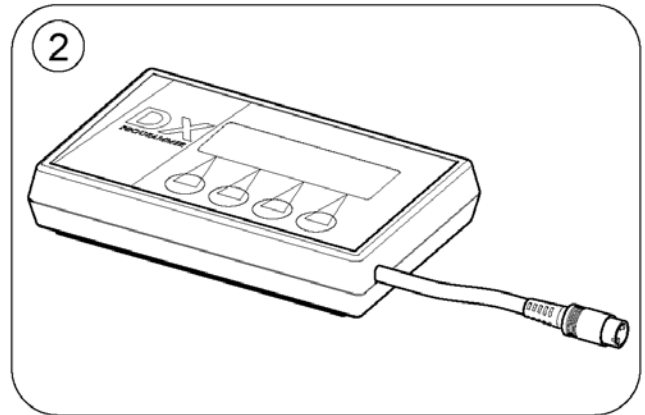

1	1449280	(USW wizard dongle kit, EDL-USW, cpl.)	1
2	6001450	(Hand Programmer, Instrucions In French, German, Spanish)	1
	SP1441551	(Programming Adapter Shark DK-ADAPT (necessary to program Shark))	1

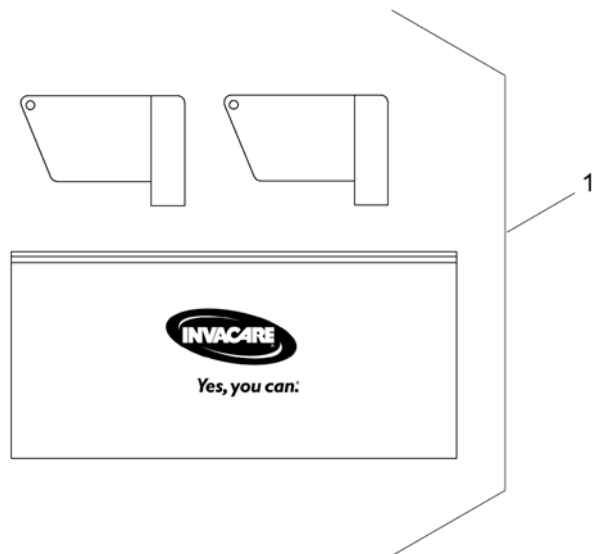

1 SP1419695 (Magnet key with bag)

1


	Capacity [Ah]															
	1,2	12	30	31	32	36	38	40	40	50	50	50,9	60	73,6	75	97,6
Bora/SpectraXTR								8		10						
Dragon										10			13			
Dragon Vertic													13			
FDX													13			
G50													13	14		16
Kite													13	14		
Mirage				4				8								
Mistral3				4									12			
P9000XDT					5											
Pronto M41					5											
Pronto M61					5											
Spectra Blitz								8					12			
Spectra Comfort								7								
Spectra Plus								7								
Spectra Seat Riser								7								
Storm3										10			13	14		
Storm3 True Track	1													14		
Storm3 Vertic														14		
Storm4														14		
Storm4 Max														14		
Stream										10			13			
TDX-SP														14		
TDX-SP Narrow												12				
Tiger										10			13			
Topan													13			
Tornado								8								
Typhoon													13			
Typhoon GB	1												13			
Typhoon GB	1												13			
Zephyr	1												13			
Auriga			3					8		10						
Comet														14	15	
Leo						6			9							
Lynx		2														
Meteor														14		
Orion											11					
length [mm]	97	152	197	196	211	196	210	197	198	198	198	238	259	260	260	329
width [mm]	49	99	132	132	132	130	175	168	165	166	165	140	169	171	168	171
height [mm]	51	102	180	183	183	179	175	175	171	176	170	235	178	210	223	238
weight [kg]	0,6	3,9	11,1	10,5	11	12	14,6	14,4	14,2	14	16	17,1	19,3	24,3	26	32,5



i	-	(Change batteries only pairwise and from the same type!)	1
1	1440386	BATTERY 1,2AH 12V FRIKPL.THYPOON	1
2	SP1506493	(Battery 12V/12Ah AGM)	1
3	P50021	(Battery, 12V/30Ah C20, GEL)	1
4	P50029	BATTERY 30AH MK12-31 TROJAN	1
5	SP1521459	(Battery, 12V/34Ah C20, AGM)	1
6	1505498	(Battery, 12V/36Ah C20, AGM)	1
7	P50028	(Battery, 12V/38Ah C20, GEL (UK & DK only))	1
8	PG50988	BATTERY 40AH MK12-40 TROJAN	1
9	SP1550692	(Battery, 12V/40Ah C20, AGM)	1
10	SP1522107	(Battery, 12V/50Ah C20, AGM)	1
11	SP1521695	(Battery, 12V/50Ah C20 AGM)	1
12	PG50989	BATTERY 50AH MK12-50 TROJAN	1
13	1432944	BATTERY 60AH MK12-60 TROJAN	1
14	P50016	BATTERY 70AH MK12-73 TROJAN	1
15	SP1522026	(Battery, 12V/75Ah C20, AGM)	1
16	1478675	BATTERY 97AH MK12-97 TROJAN	1


(Chargers)

SP51981.iso

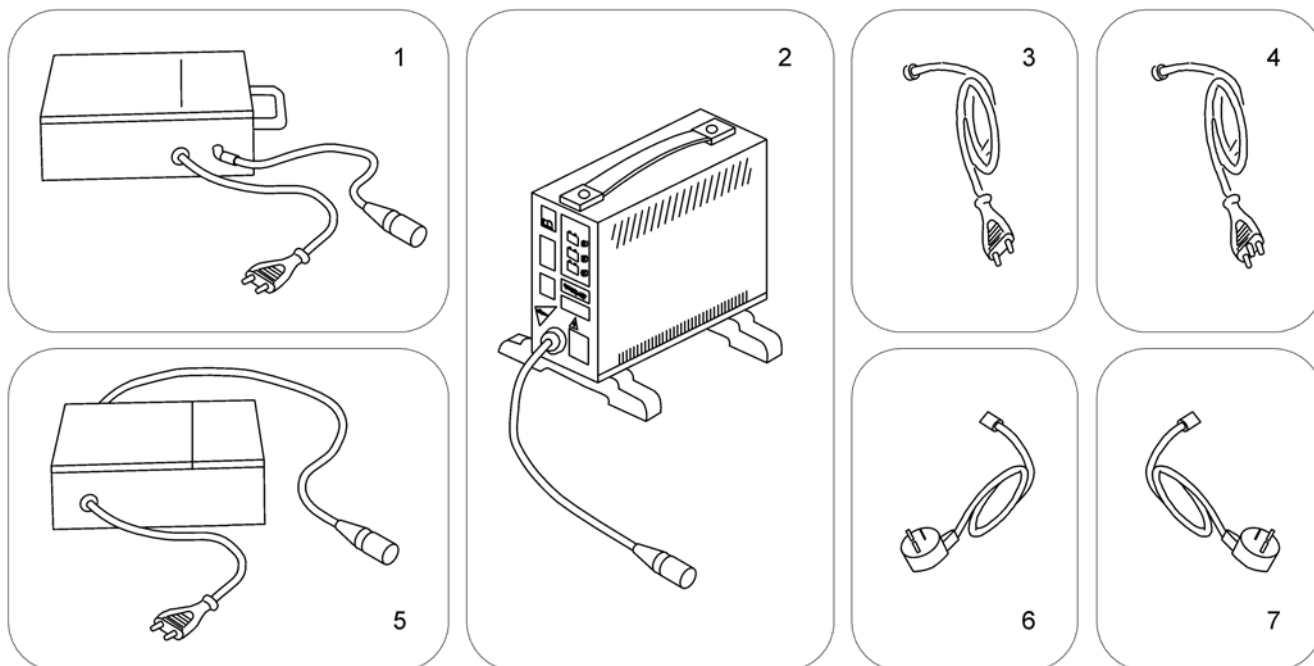


1	5813987	(Charger 8A, Gi Popular (Sweden only))	1
2	SPP51981	(Charger 8A for Batteries up to 73Ah (without power cord))	1
3	P21079	(Power Cord, Continental Shockproof Plug)	1
4	1443841	(Power Cord, Swiss)	1
5	1510201	(Charger 10A, Powercharge Classic (G50), for Batteries from 97AH (with power cord))	1
5	SP1545511	LADER LADAC/DEFA 701239 MED PLUGG	1
6	P21080	(Power Cord, UK)	1
7	SP1553849	(Power Cord, Israel)	1

(Battery support)

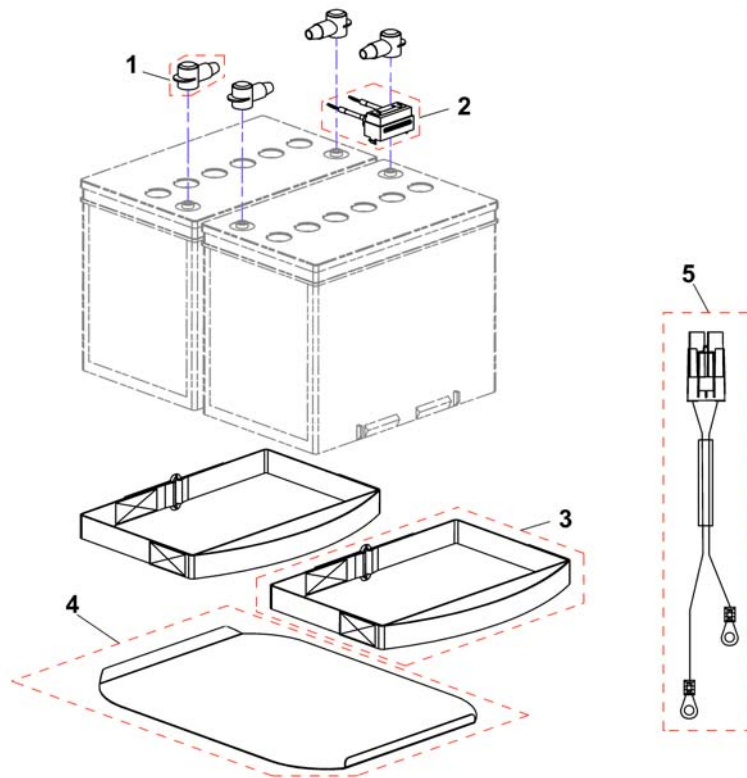


1	P52996	POLSKO DEKSEL	1
2	SP1525580	SIKRINGSHOLDER MED SIKRING KPL	1
3	SP1518073	(Battery strap)	1
4	SP1524214	STØTTEPLATE BATTERI STORM4	1
5	SP1524666	(Battery cable)	1

(LED-Frontlight)

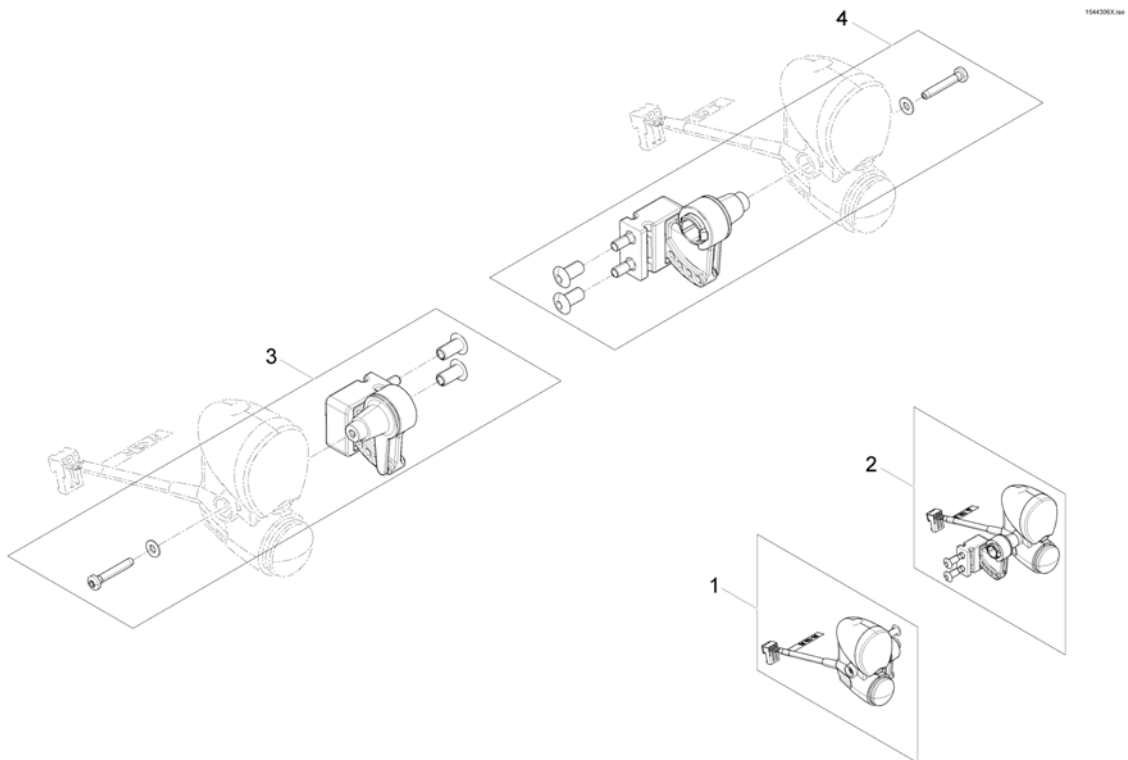


1	SP1544306	(Front light RH)	1
2	SP1544307	(Front light LH)	1
3	SP1560653	(Frontlight bracket LH)	1
4	SP1562296	(Frontlight bracket RH)	1

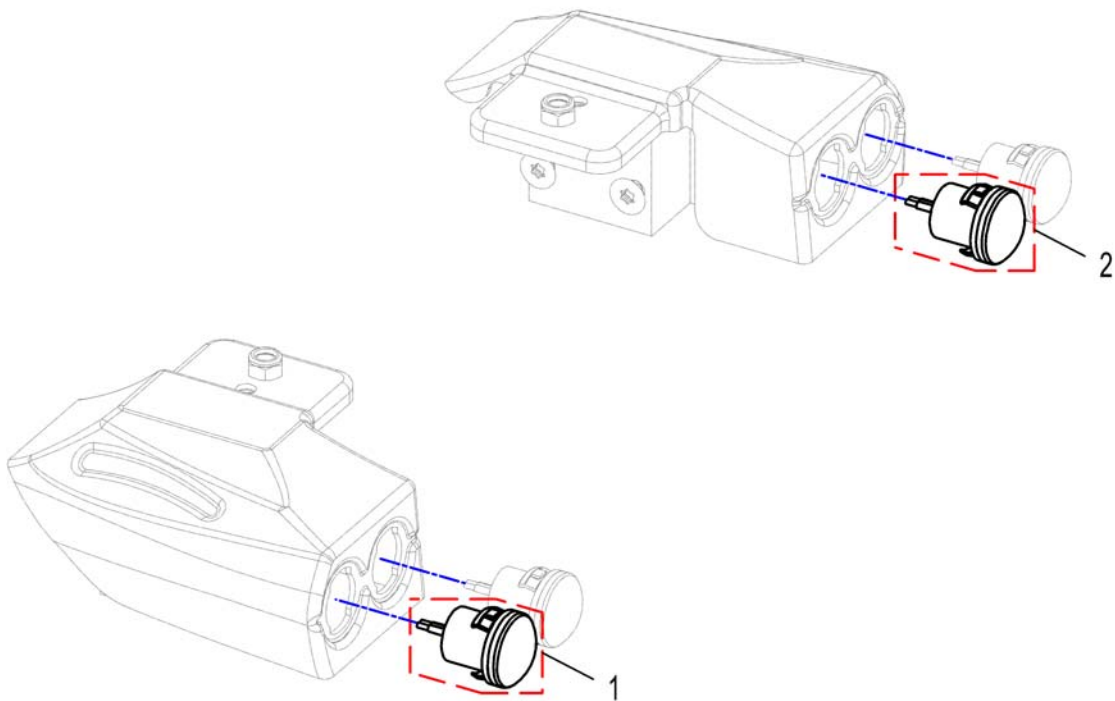


-	SP1529712	BAKLYS CABLE STORM4	1
1	SP1530287	BLINKLYS BAK STORM4	1
2	SP1530288	BAKLYS STORM4	1

(Lap Belts / H-Belt)

1529460X.dwg

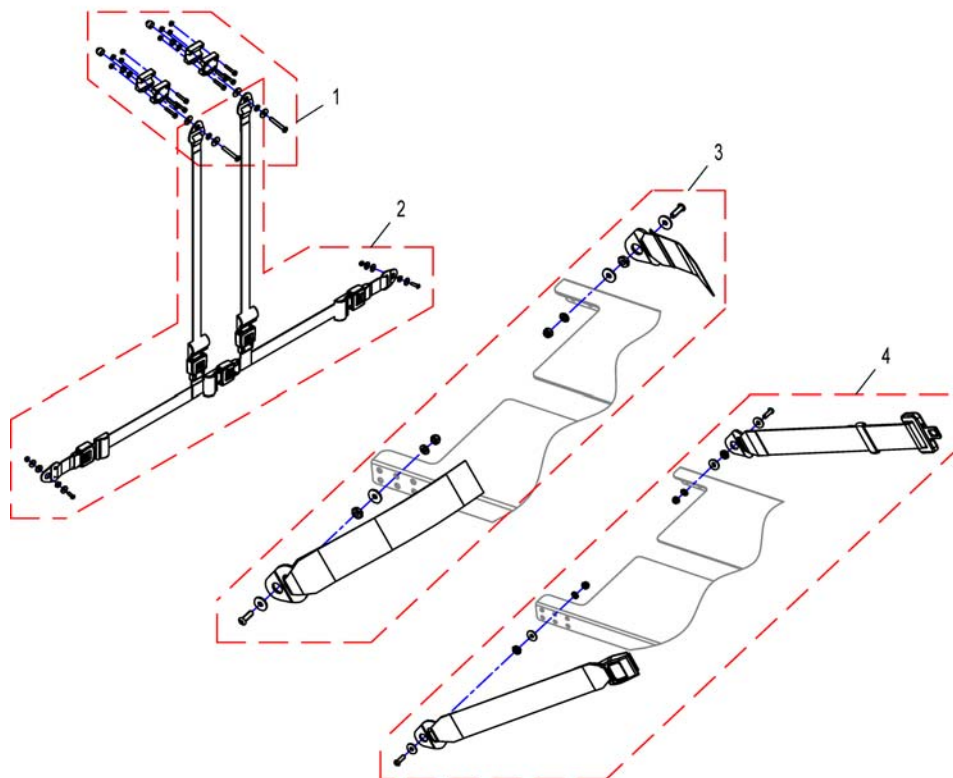


1	SP1530575	H-SELE FESTE EASYADAPT STORM4	1
2	SP1530576	H-SELE EASYADAPT STORM4	1
3	SP1529460	(Lap Belt Velcro, cpl.)	1
4	SP1529461	HOFTEBELTE M HURTIGLÅS STORM4	1

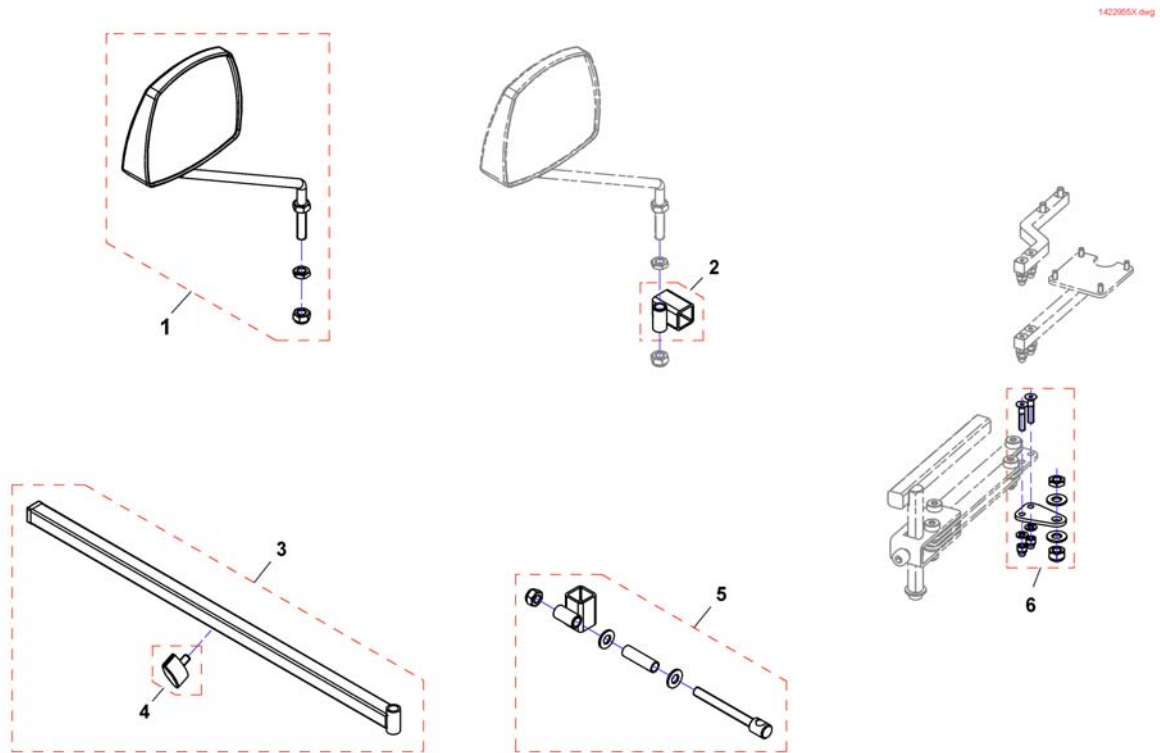


1	SP1422955	SPEIL KPL KUN SPEIL STORM 3	1
2	F276005.00	HOLDER SPEIL DRAGON STORM4 HUSK SP1527151	1
3	SP1423435	HOLDER SPEIL STORM3	1
4	L31001	VINGSKRUV M 6X10 GARANT G24 S	1
5	SP1422957	SPEIL FESTE BRACKETT STORM4 PÅ SWINGAWAY	1
6	SP1527151	FESTE FOR SPEILFESTE STORM4 TDX BORA	1

(Headrest & Bracket)

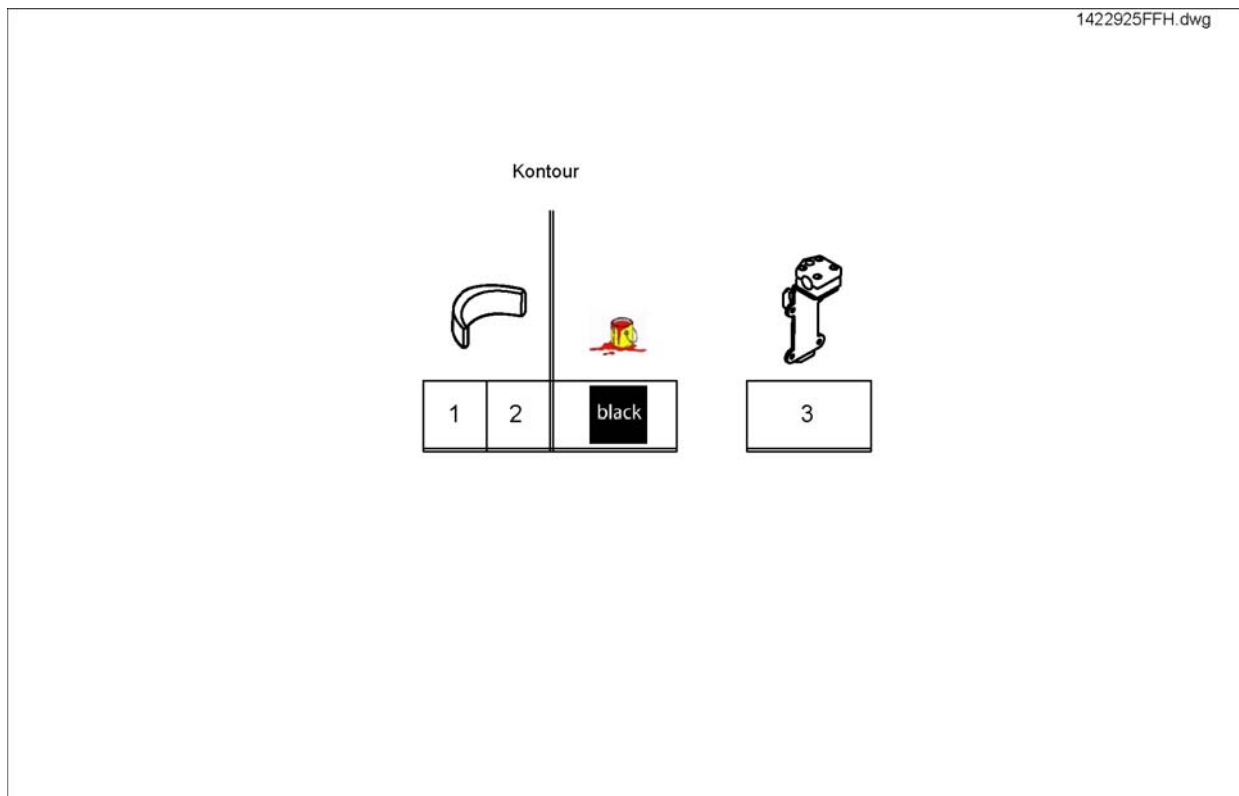


1	SP1422925	HODESTØTTE KPL CONTOUR SORT STORM3/DRAK U/FESTE	1
2	SP1422865	HODESTØTTEPUTE U FESTE STORM4	1
3	SP1422914	HOLDER HODESTØTTE RYGG STD RYGG STORM3 DRAGON	1

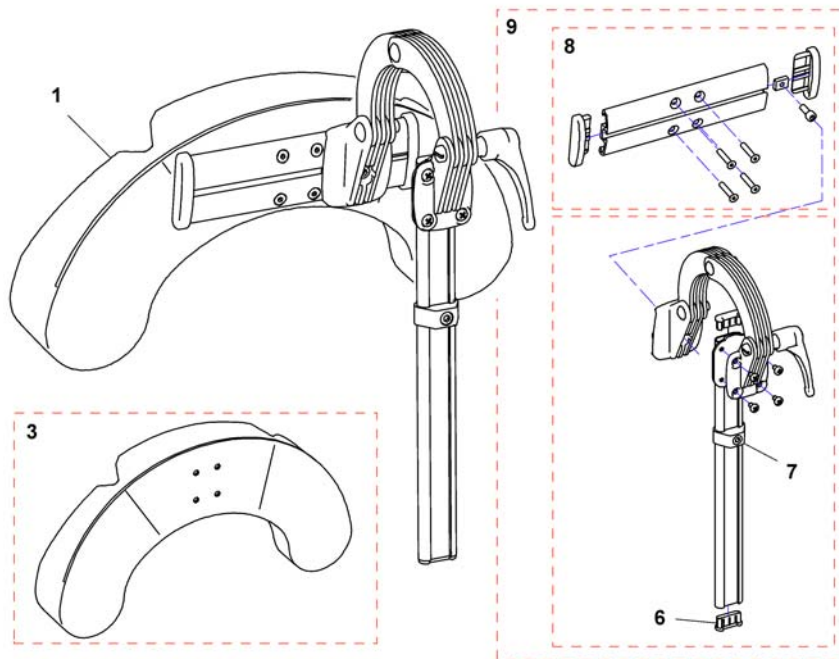


1	1446560	HODE/SKINN STØTTE KPL TR26 MED SIDEFORFLYTNING	1
1	SP1541658	(Head/cheek support TR26 cpl. sidewise adjustable EasyAdapt)	1
3	1444304	PUTE HAKESTØTTE BELLIS/A ZALEA	1
5	SP1529572	(Headrest holder for EasyAdapt)	1
6	5716497	HYLSE MESSINGSKRUE FOR LOMME REA	1
7	1448661	STOPPER FOR HREG NAKKE OG HODEST REA M SKRUE	1
8	1445838	SKINNE HODESTØTTE KPL BE LLIS AZAL FOC SPI3	1
9	1485538	FESTE HODESTØTTE STORM4	1

(Headrest, sidewise adjustable)

153091X.no

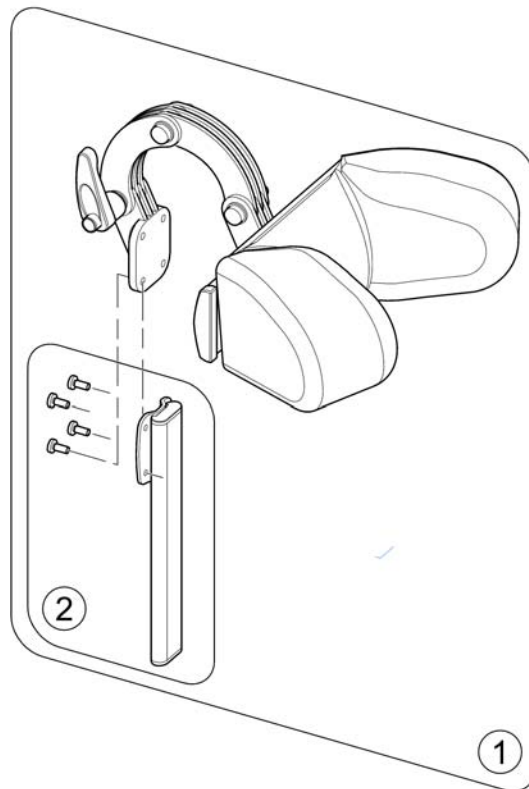


1	SP1541657	NAKKESTØTTE SIDEV JUST BRUK SP1530119	1
2	SP1530567	(Holder, cpl.)	1

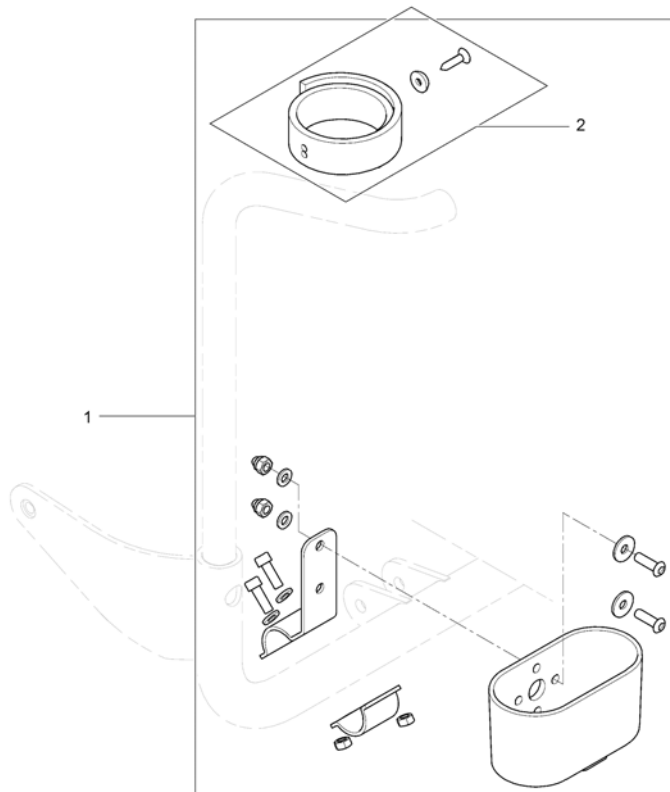


1	SP1543078	(Caneholder, cpl.)	1
2	SP1420493	(Belt for caneholder)	1

(Fender for Drive Wheel)

154805X.png

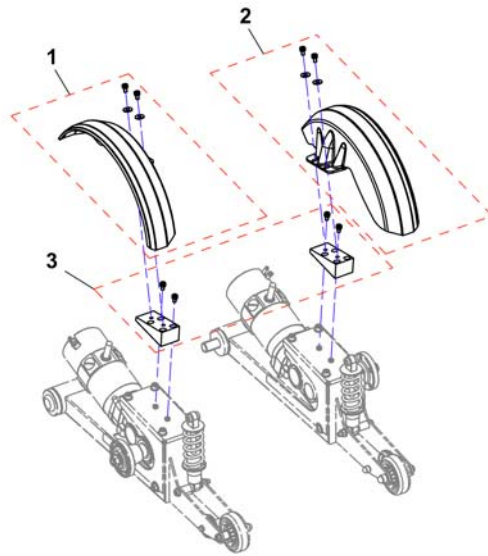


1	SP1533975	(Mudguard, cpl., RH)	1
2	SP1533976	(Mudguard, cpl., LH)	1
3	SP1548538	(Mudguard adapter)	1

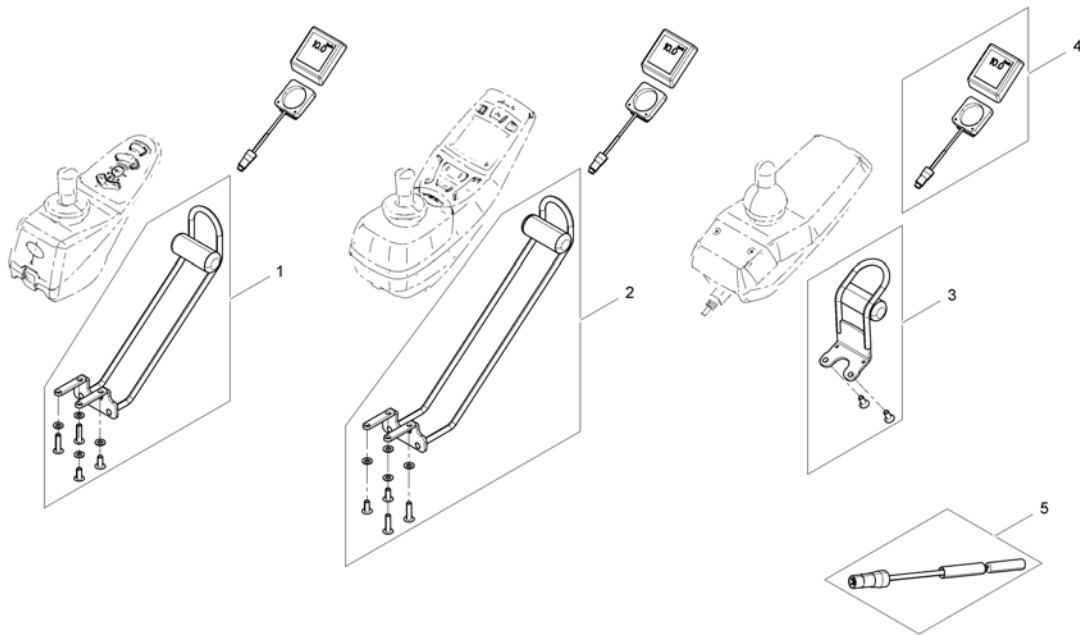


1	SP1530563	(Speedometer holder REM A & REM B)	1
2	SP1530564	(Speedometer holder REM 550)	1
3	SP1546263	(Speedometer holder REM24)	1
4	Z183259	(Speedometer)	1
5	1421733	(Reed-sensor cable l=2500mm)	1

(Mounting-kit km-Counter)

1540353X.iso

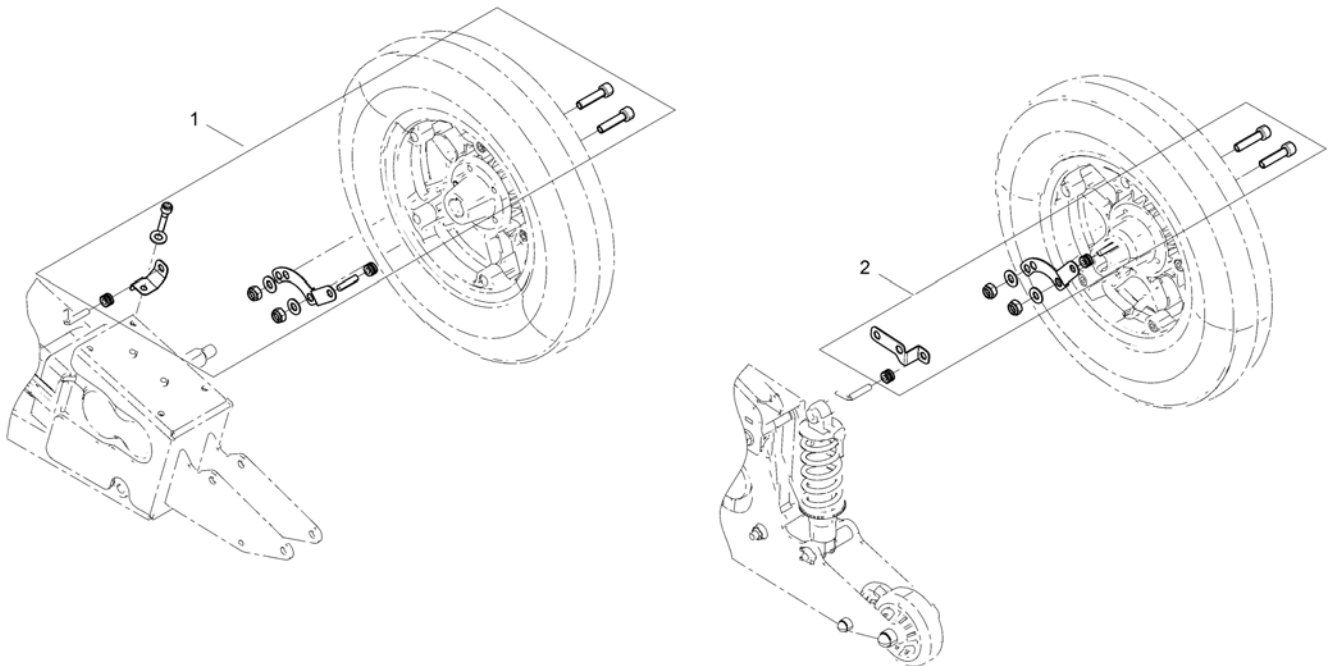


1	SP1548705	(Mounting-kit Speedometer sensor)	1
2	SP1540353	(Mounting-kit Speedometer sensor SSD-motors)	1

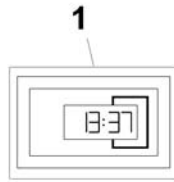

1 SP1530275 (Hour counter)

1

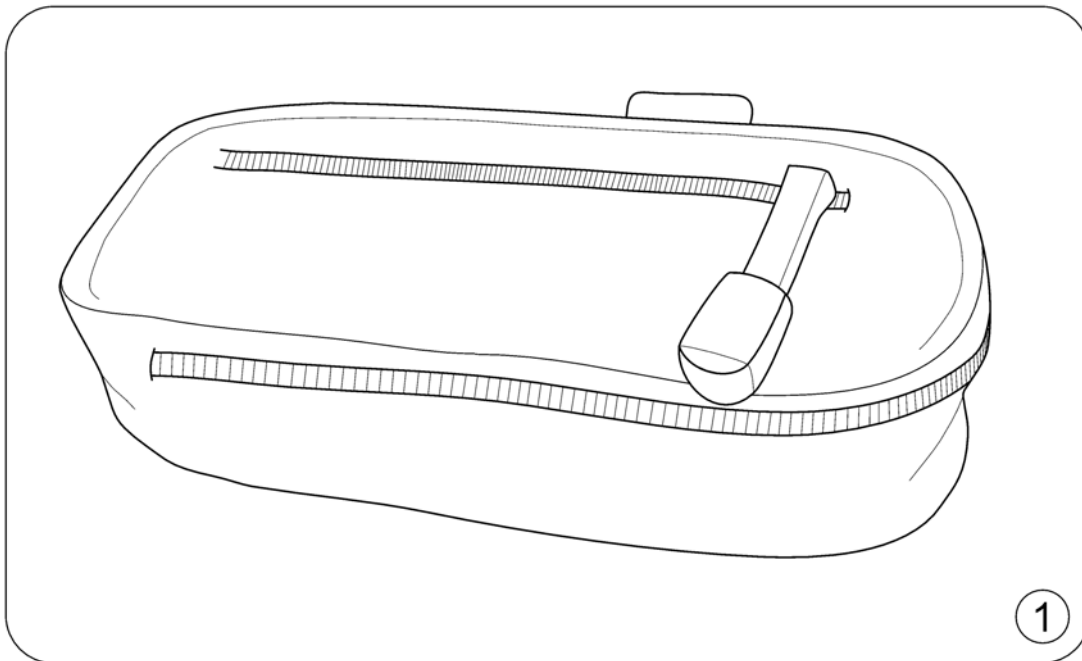


1 SP1557532

ARMLENSVESKE KLICK FIX STORM4 TDX KLICK FIX

1

(KLICKfix® bracket for Fixed Armrest)

1562515.iss

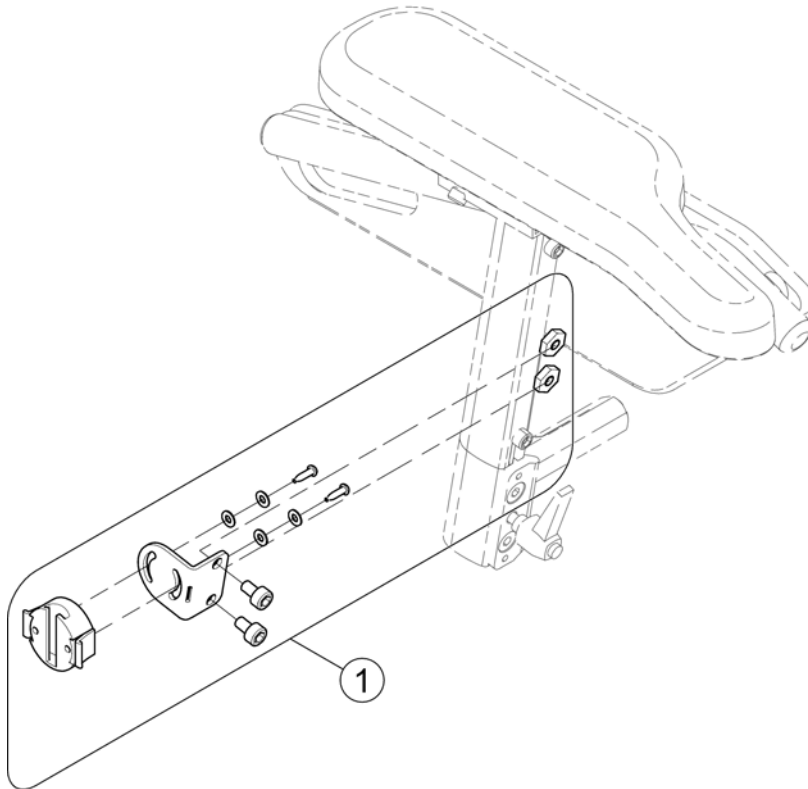


1 SP1554320

FESTE ARMLENSVESKE STORM4 TDX KLICK FIX


1

(Bag)

1567256.no

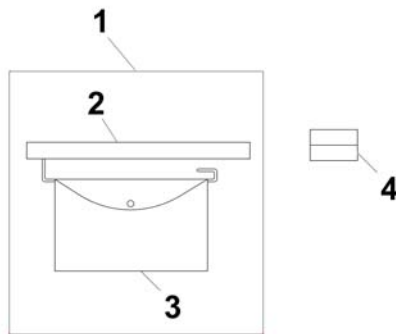

1	SP1430417	ARMLENSVESKE VE KPL STORM 3	1
2	SP1430855	HOLDER FOR ARMLENSVESKE STORM4 TDX BORA	1
3	SP5314967	VESKE ARMLEN STORM4	1
4	F370032-9005	(Holder for side part with mirror/brake RAL 9)	1

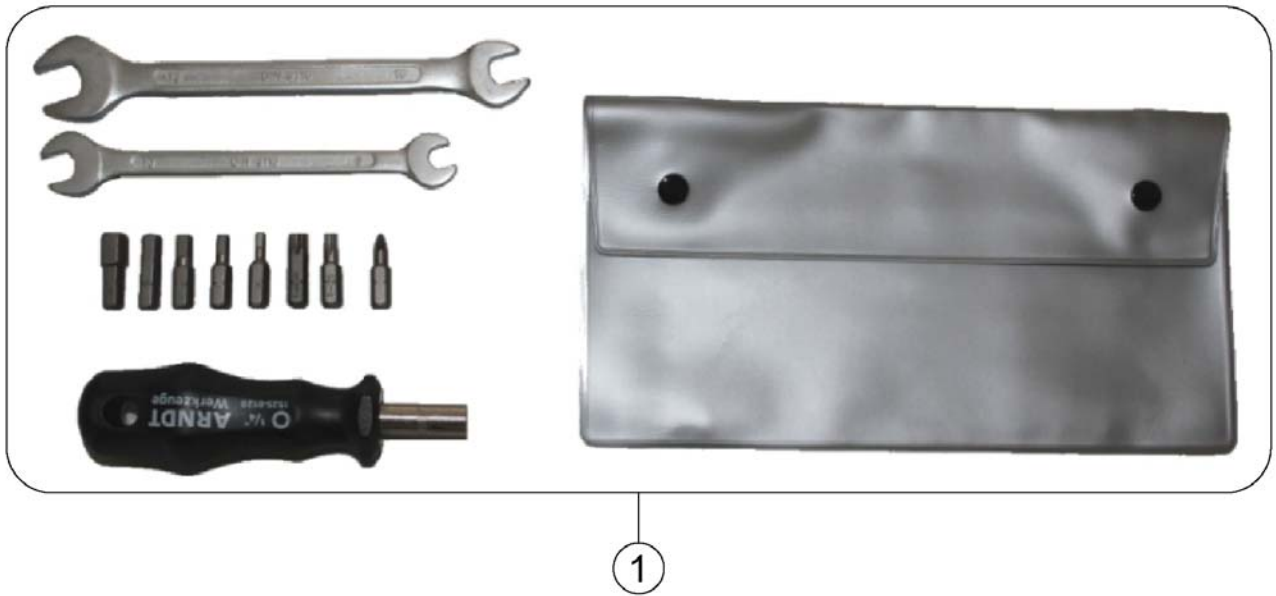

1 SP1422868

VERKTØY SETT

1



Yes, you can.®



Belgium & Luxemburg:

Invacare nv, Autobaan 22, B-8210 Loppem
Tel: (32) (0)50 83 10 10, Fax: (32) (0)50 83 10 11
belgium@invacare.com / www.invacare.be

Denmark:

Invacare A/S, Sdr. Ringvej 37, DK-2605 Brøndby
Tel: (45) (0)36 90 00 00, Fax: (45) (0)36 90 00 01
denmark@invacare.com / www.invacare.dk

Deutschland:

Invacare Aquatec GmbH, Alemannenstraße 10, D-88316 Isny
Tel: (49) (0)75 62 7 00 0, Fax: (49) (0)75 62 7 00 66
info@invacare-aquatec.com / www.invacare-aquatec.de

European Distributor Organisation:

Invacare, Kleiststraße 49, D-32457 Porta Westfalica
Tel: (49) (0)57 31 754 540, Fax: (49) (0)57 31 754 541
edo@invacare.com / www.invacare.eu.com

España:

Invacare SA, c/Areny s/n, Polígon Industrial de Celrà, E-17460 Celrà (Girona)
Tel: (34) (0)972 49 32 00, Fax: (34) (0)972 49 32 20
contactsp@invacare.com / www.invacare.es

France:

Invacare Poirier SAS, Route de St Roch, F-37230 Fondettes
Tel: (33) (0)2 47 62 64 66, Fax: (33) (0)2 47 42 12 24
contactfr@invacare.com / www.invacare.fr

Ireland:

Invacare Ireland Ltd, Unit 5 Seatown Business Campus, Seatown Road, Swords, County Dublin - Ireland
Tel: (353) 1 810 7084, Fax: (353) 1 810 7085
ireland@invacare.com / www.invacare.ie

Italia:

Invacare Mecc San s.r.l., Via dei Pini 62, I-36016 Thiene (VI)
Tel: (39) 0445 38 00 59, Fax: (39) 0445 38 00 34
italia@invacare.com / www.invacare.it

Nederland:

Invacare BV, Celsiusstraat 46, NL-6716 BZ Ede
Tel: (31) (0)318 695 757, Fax: (31) (0)318 695 758
nederland@invacare.com, csede@invacare.com / www.invacare.nl

Norge:

Invacare AS, Grensingsvingen 9, Postboks 6230, Etterstad, N-0603 Oslo
Tel: (47) (0)22 57 95 00, Fax: (47) (0)22 57 95 01
norway@invacare.com, island@invacare.com / www.invacare.no

Österreich:

Invacare Austria GmbH, Herzog Odilostrasse 101, A-5310 Mondsee
Tel: (43) 6232 5535 0, Fax: (43) 6232 5535 4
info-austria@invacare.com / www.invacare.at

Portugal:

Invacare Lda, Rua Estrada Velha, 949, P-4465-784 Leça do Balio
Tel: (351) (0)225 1059 46/47, Fax: (351) (0)225 1057 39
portugal@invacare.com / www.invacare.pt

Sverige & Suomi:

Invacare AB, Fagerstagatan 9, S-163 91 Spånga
Tel: (46) (0)8 761 70 90, Fax: (46) (0)8 761 81 08
sweden@invacare.com, finland@invacare.com / www.invacare.se

Switzerland:

Invacare AG, Benkenstrasse 260, CH-4108 Witterswil
Tel: (41) (0)61 487 70 80, Fax: (41) (0)61 487 70 81
switzerland@invacare.com / www.invacare.ch

United Kingdom:

Invacare Limited, Pencoed Technology Park, Pencoed, Bridgend CF35 5AQ
Switchboard Tel: (44) (0)1656 776200, Fax: (44) (0)1656 776201
Customer services Tel: (44) (0)1656 776222, Fax: (44) (0)1656 776220
uk@invacare.com / www.invacare.co.uk