

Spare Parts Catalogue

Version: 2015/07/08b NO



Invacare® **COMET**



Yes, you can.®

General Information

For any order of spare parts the following information is required:

- Product type*
- Product name & model*
- Serial Number*
- Part-Number*
- Part-Description*

Exploded view drawings and code numbers are not contractual and subject to modification, in relation to Invacare® continuous product improvement policy to meet end users needs.

The reproduction or use of this work, or of any substantial part of this work, in any form or by any electronic, mechanical or other means, including photocopying, scanning and recording, and in any information storage and retrieval system, including electronic transmission, is forbidden without the prior written permission of Invacare® International Sarl.



Yes, you can.®

TABLE OF CONTENTS

Informasjon

(Accessory Compatibility)	5
(User Manuals Comet)	6

Chassis

(Front frame 3-Wheeler)	7
(Front shroud 3-Wheeler)	8
(Front frame 4-Wheeler)	9
(Front shroud 4-Wheeler)	10
(Rear frame)	11
(Rear shroud)	12

Sete

(Seat & Seat Options)	13
(Seat & Seat Options)	14

Sving-og Drivhjul

(Front wheel 3-Wheeler)	15
(Front wheel 4-Wheeler)	16
(Rear wheel)	17
(Anti-tipp-wheel)	18

Drivenhet

(Propulsion 10/12.8/15 km/h)	19
(Motors, Powermodules & Speed Reduction)	20
(Motors, Powermodules & Speed Reduction)	20
(Upgrade Kits - Powermodules & Speed Reduction)	21
(Batteries)	22

Bremser

(Brakes 3 Wheeler)	23
(Brakes 4 Wheeler)	24

Styret

(Steering)	25
(Drive Electronics Steering Bar)	26
(Frontfork 3-Wheeler)	27
(Steering linkage 4-Wheeler)	28

kabel og lader

(Chargers)	29
(Cable)	30

Sikkerhetsutstyr

(Warning Triangle)	31
--------------------------	----













Yes, you can.®

TABLE OF CONTENTS

Annet Utstyr & Tilbehør

(Accessories)	32
(Accessory Compatibility)	5
(Rollator bracket)	33
(Storage box rear bracket)	34
(Veltop Rain- & Suncover)	35

ComComp.dr

					
	û				
	û	û			
	û	û	û		
	ü	û	û	û	
	ü	û	û	ü	ü


(For further information please contact your customer service.)

(User Manuals Comet)

languages.io

BG	Български	FR	Français	NL	Nederlands	SV	Svenska
CS	Čeština	HE	עברית	NO	Norsk	UK	Українська
DA	Dansk	HR	Hrvatski	PL	Polski		
DE	Deutsch	HU	Magyar	PT	Português		
EN	English	IT	Italiano	RU	Русский		
ES	Español	JA	日本語	RO	Română		
ET	Eesti	LV	Latviešu	SK	Slovenčina		
FI	Suomi	LT	Lietuvių	SL	Slovenščina		


CZ	1514593	(User Manual Comet Czech)	1
DA	1514594	(User Manual Comet Danish)	1
DE	1514596	(User Manual Comet German)	1
EN	1514597	(User Manual Comet English)	1
ES	1514598	(User Manual Comet Spanish)	1
ET	1514599	(User Manual Comet Estonian)	1
FR	1514600	(User Manual Comet French)	1
IT	1514601	(User Manual Comet Italian)	1
LT	1514602	(User Manual Comet Lithuanian)	1
NL	1514603	(User Manual Comet Dutch)	1
NO	1514604	BRUKSANVISNING COMET COMET	1
PL	1514605	(User Manual Comet Polish)	1
PT	1514606	(User Manual Comet Portuguese)	1
SK	1514607	(User Manual Comet Slovenian)	1
SV	1514608	(User Manual Comet Swedish)	1
FI	1527700	(User Manual Comet Finsk)	1
DE	SP1550400	(User Manual Country German)	1
UK	SP1579822	(User Manual Comet UK)	1

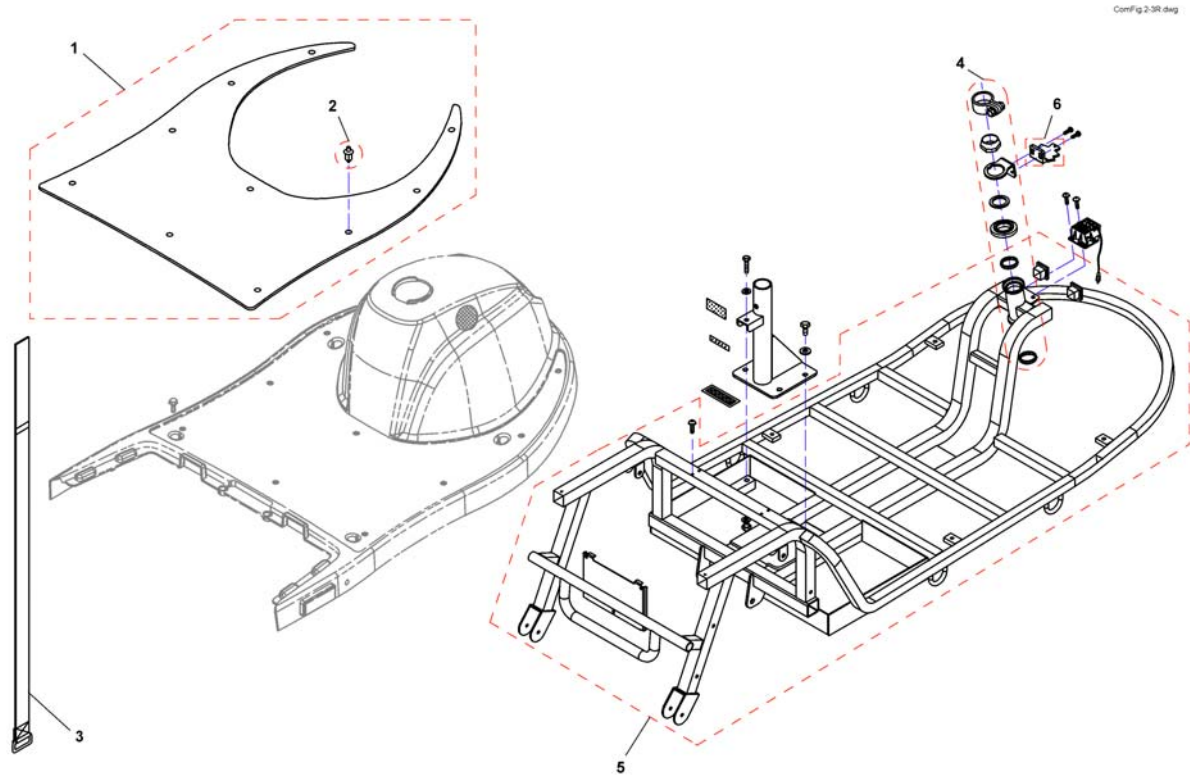

1	SP1521765	MATTE STOR COMET ALPINE 3 HJUL	1
2	SP1505476	FESTEKLIPS MATTE STOR COMET 8STK	1
3	SP1521758	BATTERISTROPP COMET	1
4	SP1505487	(Bearing Kit)	1
5	SP1521729	RAMME 3-HJUL COMET	1
6	SP1536164	FARTSREDUKSJONS BRAKETT COMET	1


(Front shroud 3-Wheeler)

ConFig 2-1-3R.dwg

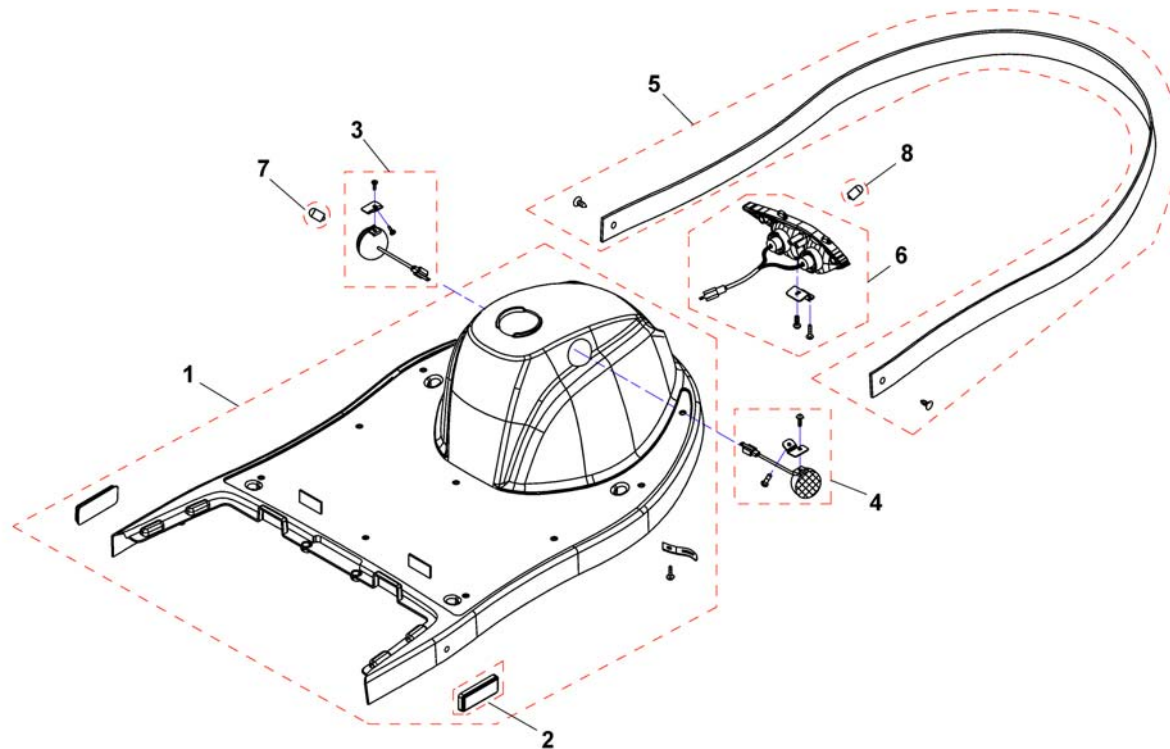


-	SP1523450	(Tuch Up Paint Moonstone Silver)		1
-	SP1523462	(Tuch Up Paint Onyx Blue)		1
1	SP1521730	(Shroud, silver)		1
1	SP1521731	(Shroud, blue)		1
1	SP1559815	DEKSEL FORAN COMET3 HJUL COMET		1
2	SP1517331	REFLEKS COMET		1
3	SP1518016	RETNIGSLYS VE COMET 4W		1
4	SP1518017	RETNIGSLYS HØ COMET 4W		1
5	SP1557367	KANTLISTE TIL DEKSEL COMET		1
6	SP1517878	LYS FORAN KPL	→ 18.08.2008	1
6	SP1527473	HOVEDLYKT KPL COMET	19.08.2008 →	1
7	SP1522030	PÆRE 24V 10W COMET BLINKLYS		1
8	SP1522029	\$LYSPÆRE FORAN COMET 21W COMET BRUK SP1527474	→ 18.08.2008	1
8	SP1527474	PÆRE 12V 15W COMET	19.08.2008 →	1

Invacare® Comet | Chassis
(Front frame 4-Wheeler)

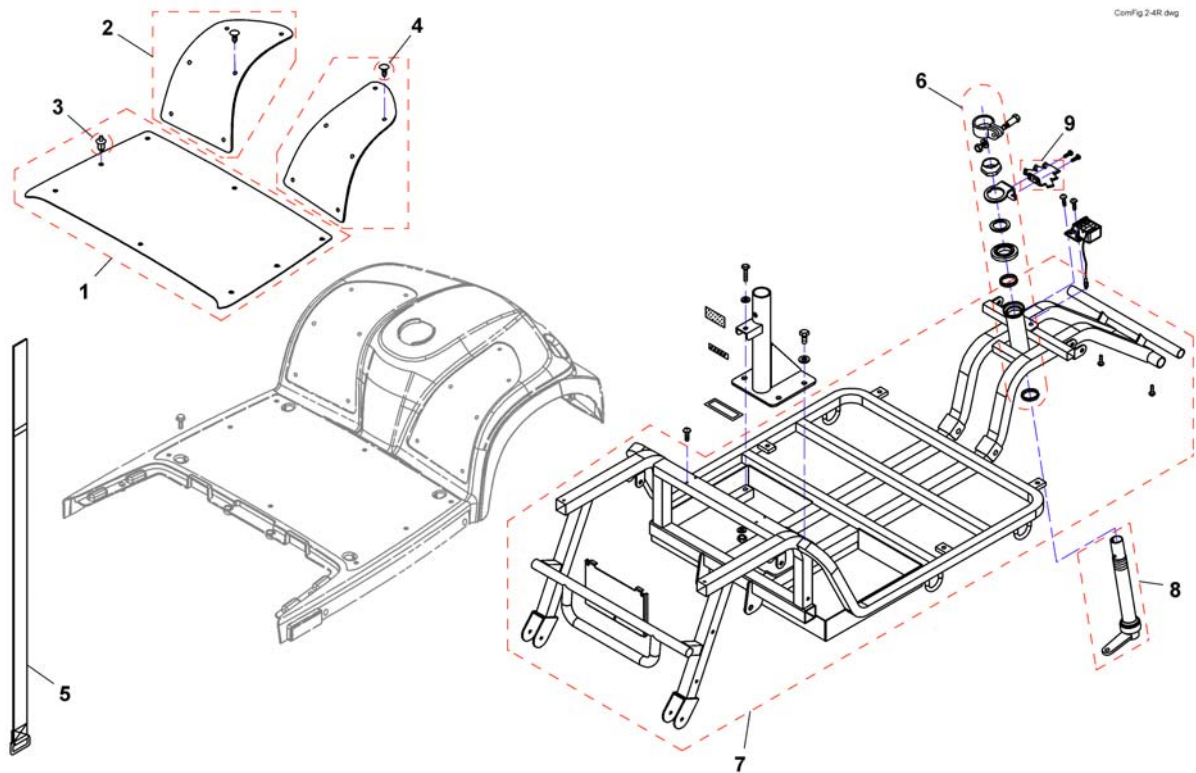


1	SP1521755	MATTE NEDRE COMET	1
2	SP1521756	MATTE FORAN HØ VE COMET	1
3	SP1505476	FESTEKLIPS MATTE STOR COMET 8STK	1
4	SP1505479	FESTEKLIPS MATTE FORAN COMET ALPINE 6 STK	1
5	SP1521758	BATTERISTROPP COMET	1
6	SP1505487	(Bearing Kit)	1
7	SP1521760	RAMME COMET	1
8	SP1561395	(Steering link)	1
9	SP1536164	FARTSREDUKSJONS BRAKETT COMET	1

(Front shroud 4-Wheeler)

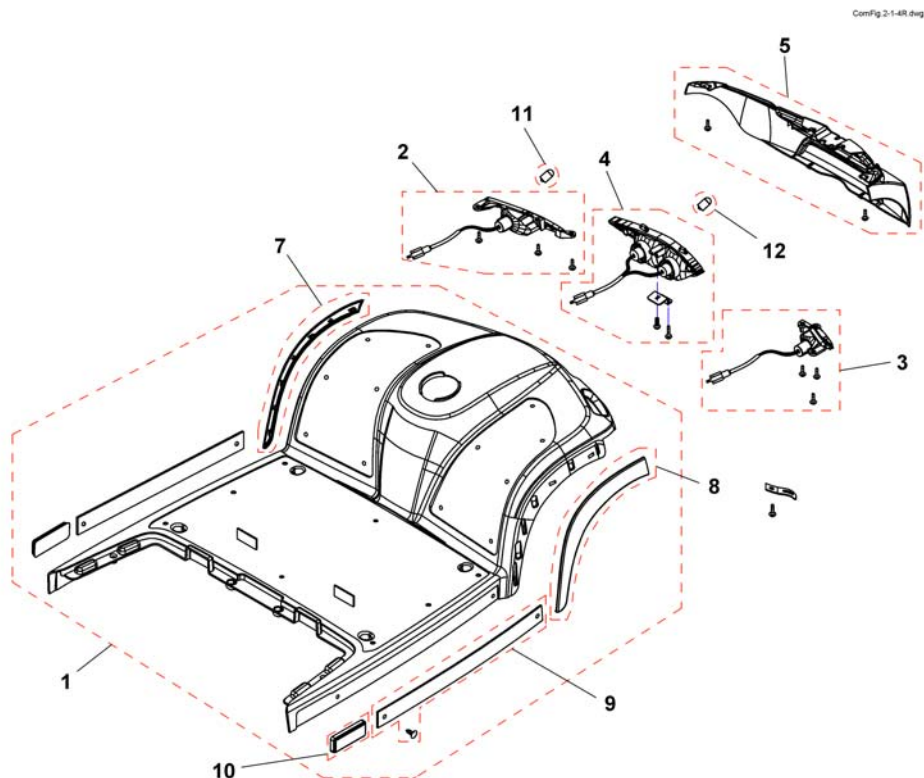

	SP1523462	(Tuch Up Paint Onyx Blue)		1
	SP1523450	(Tuch Up Paint Moonstone Silver)		1
	SP1537798	LAKKSTIFT COMET GRÅ COMET		1
1	SP1521732	(Front Shroud, silver)		1
1	SP1521733	(Front Shroud, blue)		1
1	SP1537792	DEKSEL FORAN/MIDT STOR COMET		1
1	SP1551635	(Front Shroud, red (UK only))		1
2	SP1517733	RETNINGSLYS VE COMET 4W		1
3	SP1517734	RETNINGSLYSHØ COMET 4W		1
4	SP1517878	LYS FORAN KPL	→ 18.08.2008	1
4	SP1527473	HOVEDLYKT KPL COMET	19.08.2008 →	1
5	SP1521762	DEKSEL STØTFANGER COMET		1
7	SP1521735	DEKSEL VED FESTE RATTST. COMET VE		1
8	SP1521734	DEKSEL VED FESTE RATTST. COMET HØ		1
9	SP1521754	DEKSEL NEDE COMET COMET		1
10	SP1517331	REFLEKS COMET		1
11	SP1522030	PÆRE 24V 10W COMET BLINKLYS		1
12	SP1522029	\$LYSPÆRE FORAN COMET 21W COMET BRUK SP1527474	→ 18.08.2008	1
12	SP1527474	PÆRE 12V 15W COMET	19.08.2008 →	1


(Rear frame)

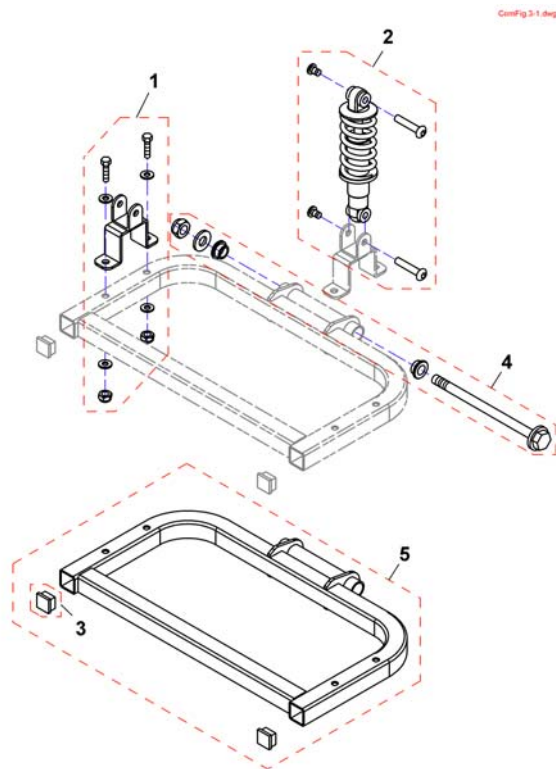


1	SP1521888	(Holder Differenzial)	1
1	SP1533846	HOLDER DIFFERENSIAL COMET	1
2	SP1521886	STØTDEMPER COMET	1
2	SP1533847	STØTDEMPER 165MM COMET	1
2	SP1533848	STØTDEMPER 350LBS COMET	1
3	SP1521597	(plastic cap)	1
4	SP1541614	BAKRAMME MONTERINGS KIT COMET	1
5	SP1541615	(Rear frame, Comet standard)	1
5	SP1541616	CHASSI BAK COMET COMET	1

(Rear shroud)

Conf. Fig. 3.10a

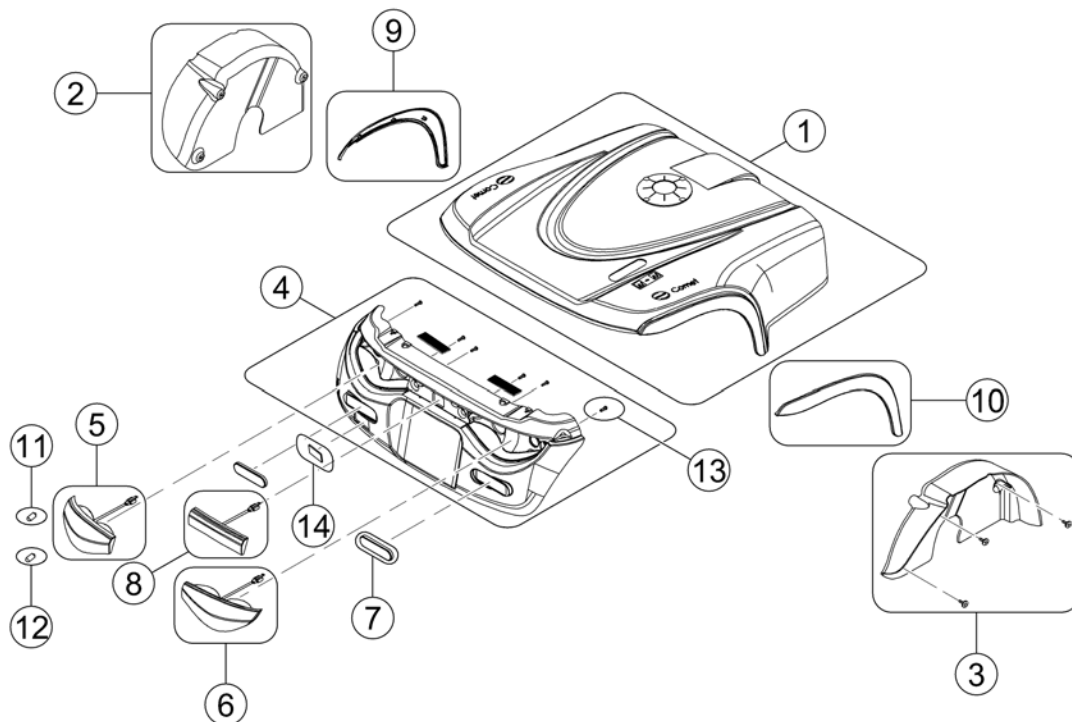


1	SP1521837	(Rear Shroud, silver)	1
1	SP1521838	(Rear Shroud, blue)	1
1	SP1537793	(Rear Shroud, slate grey, (Comet HD))	1
1	SP1537794	DEKSEL OVER BATTERIER COMET	1
1	SP1551636	(Rear Shroud, red (UK only))	1
2	SP1521862	SKJERM BAK VE COMET	1
3	SP1521839	SKJERM BAK HØ COMET	1
4	SP1521683	DEKSEL BAK V STØTFANGER COMET	1
5	SP1518034	RETNINGSLYS VE BAK COMET	1
6	SP1518033	RETNINGSLYS HØ BAK COMET	1
7	SP1518036	REFLEKS BAK COMET	2
8	SP1518035	BREMSSELYS KPL COMET	1
9	SP1521882	BAKRE DEKSEL VE COMET	1
10	SP1521883	BAKRE DEKSEL HØ COMET	1
11	SP1522030	PÆRE 24V 10W COMET BLINKLYS	2
12	SP1522034	LYSPÆRE 24V 5W COMET BAK	2
13	1505418	(Tapping Screw (4x12))	6
14	SP1521879	LIST BAKRE DEKSEL LITEN COMET	1

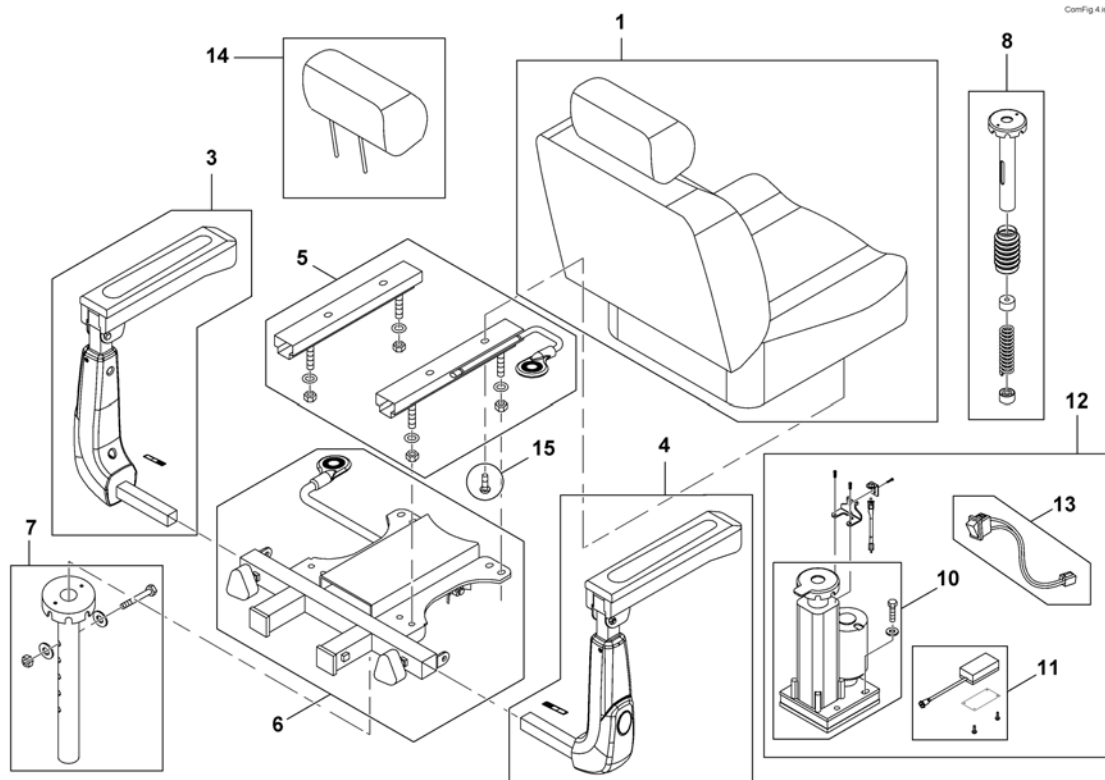


1	SP1586116	(Seat 20" Standard, double white stitching)	1
1	SP1586113	(Seat 22" Standard, double white stitching)	1
1	SP1586115	(Seat 26" HD, double white stitching)	1
3	SP1521689	ARMLÉN VE COMET	1
3	SP1533706	(Armrest LH, Comet HD)	1
4	SP1521688	ARMLÉN HØ COMET	1
4	SP1533707	(Armrest RH, Comet HD)	1
5	SP1521690	GLIDER COMET	1
6	1505533	SETEBRAKETT COMET	1
6	SP1533708	SETEBRAKETT COMET HD	1
7	SP1521691	SETEBRAKETT M ARMLENSFESTE COMET	1
8	SP1528986	(Seat Suspension Comet 508,5mm up to 100kg) <i>not available for Comet HD</i>	1
8	SP1528987	(Seat Suspension Comet 533,5mm up to 100kg) <i>not available for Comet HD</i>	1
8	SP1528988	(Seat Suspension Comet 558,5mm up to 100kg) <i>not available for Comet HD</i>	1
8	SP1522459	SETEFJÆRING SH50 COMET	1
8	SP1522460	SETEFJÆRING SH54 COMET	1
8	SP1522461	SETEFJÆRING SH56 COMET	1
10	SP1521979	(Lifter)	1
11	SP1521986	CONTROLLER SETELØFT COMET	1
12	SP1521980	SETELØFT KPL COMET <i>When retrofitting a seatlifter to a scooter => 18/08/08 remember to order the main cable-loom - not available for Comet HD</i>	1
13	SP1522037	BRYTER SETELØFT COMET	1
14	SP1529380	HODESTØTTE COMET	1
15	1505534	(Bolt, Slide cpl. (M8x15))	4

(Seat & Seat Options)

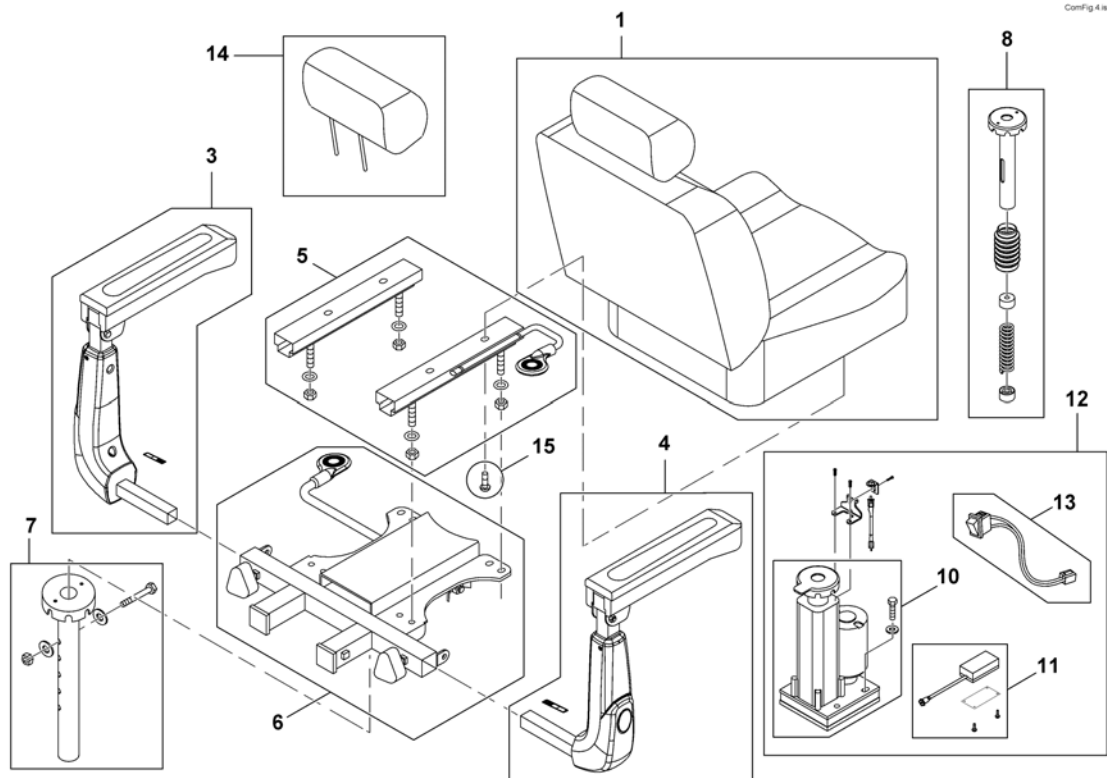

1	SP1521890	STANDARD SETE 20" COMET"	1
1	SP1521693	SETE COMET 22" COMET HUSK SP1552594	1
1	SP1584887	(Seat 20" HD)	1
1	SP1533705	SETE COMET HD SB26" COMET HUSK SP1552594	1
3	SP1521689	ARMLÉN VE COMET	1
3	SP1533706	(Armrest LH, Comet HD)	1
4	SP1521688	ARMLÉN HØ COMET	1
4	SP1533707	(Armrest RH, Comet HD)	1
5	SP1521690	GLIDER COMET	1
6	1505533	SETEBRAKETT COMET	1
6	SP1533708	SETEBRAKETT COMET HD	1
7	SP1521691	SETEBRAKETT M ARMLENSFESTE COMET	1
8	SP1528986	(Seat Suspension Comet 508,5mm up to 100kg) <i>not available for Comet HD</i>	1
8	SP1528987	(Seat Suspension Comet 533,5mm up to 100kg) <i>not available for Comet HD</i>	1
8	SP1528988	(Seat Suspension Comet 558,5mm up to 100kg) <i>not available for Comet HD</i>	1
8	SP1522459	SETEFJÆRING SH50 COMET	1
8	SP1522460	SETEFJÆRING SH54 COMET	1
8	SP1522461	SETEFJÆRING SH56 COMET	1
10	SP1521979	(Lifter)	1
11	SP1521986	CONTROLLER SETELØFT COMET	1
12	SP1521980	SETELØFT KPL COMET <i>When retrofitting a seatlifter to a scooter => 18/08/08 remember to order the main cable-loom - not available for Comet HD</i>	1
13	SP1522037	BRYTER SETELØFT COMET	1
14	SP1529380	HODESTØTTE COMET	1
15	1505534	(Bolt, Slide cpl. (M8x15))	4


15219865.dwg

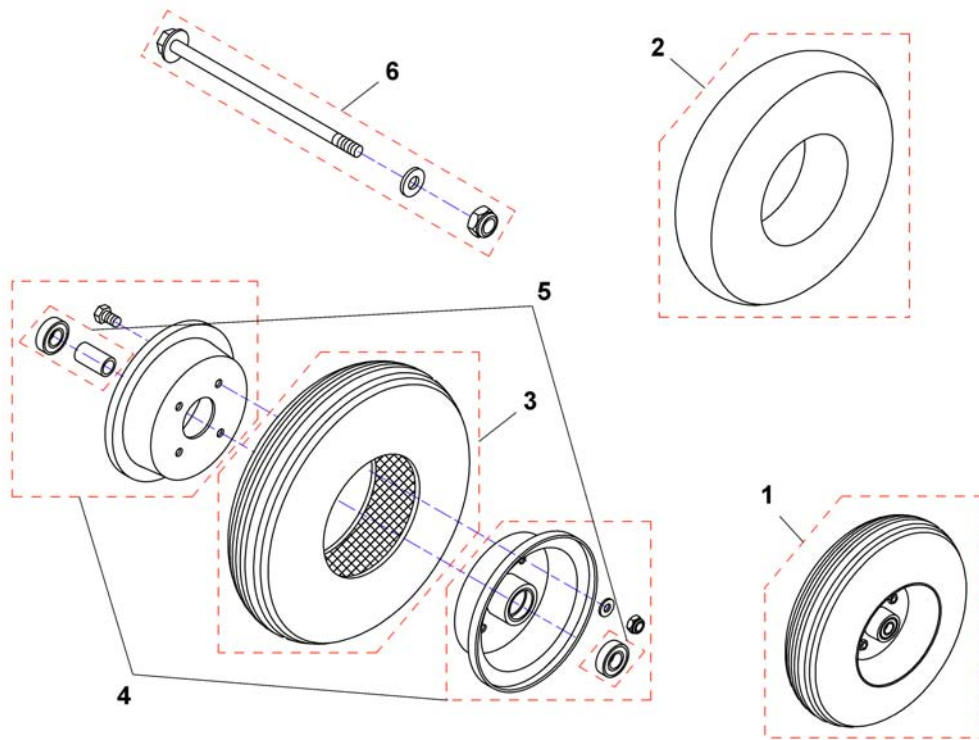

1	SP1521998	FORHJUL KPL COMET 3 HJUL	1
1	SP1586117	(Front Wheel (3W) with tube 13", cpl. - black)	1
2	SP1527714	SLANGE COMET	1
3	SP1527750	DEKK MED SLANGE STD 1STK COMET	1
3	SP1586118	(Tyre with tube 13" - black)	1
3	SP1543159	DEKK KOMPAKT COMET COMET	1
3	SP1586119	(Solid Tyre PU 13" - black)	1
4	SP1527754	FELG KOMPLETT FRONT COMET 3 WH	1
5	SP1527747	LAGER KIT HJUL COMET ALPINE 3W	1
6	SP1557458	AKSLING FORHJUL COMET COMET	1


(Front wheel 4-Wheeler)

1521999X.iso

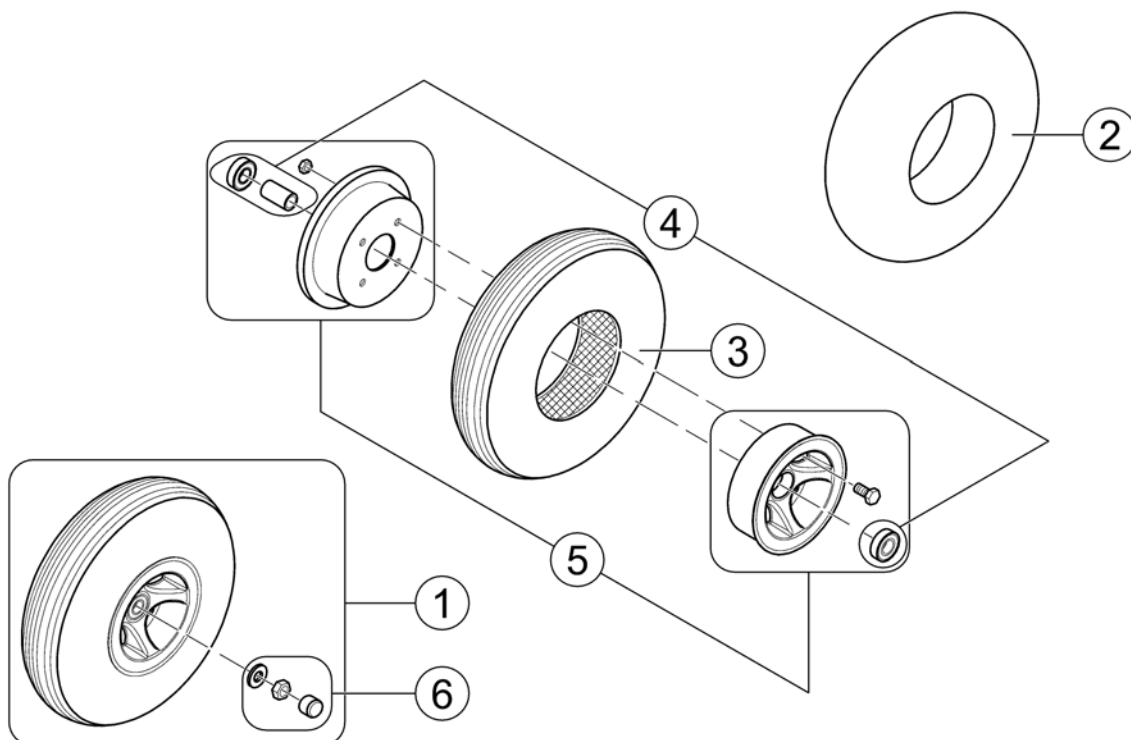

1	SP1521999	FORHJUL KPL COMET 4 HJUL	1
1	SP1586120	(Front Wheel (4W) with tube 13", cpl. - black)	1
2	SP1527714	SLANGE COMET	1
3	SP1527750	DEKK MED SLANGE STD 1STK COMET	1
3	SP1586118	(Tyre with tube 13" - black)	1
3	SP1543159	DEKK KOMPAKT COMET COMET	1
3	SP1586119	(Solid Tyre PU 13" - black)	1
4	SP1527751	(Bearing-Kit)	1
5	SP1580174	(Front Wheel (4W) Rim Kit)	1
6	SP1557460	SKRUE FOR FESTE HJUL COMET	1


1122020x.dwg

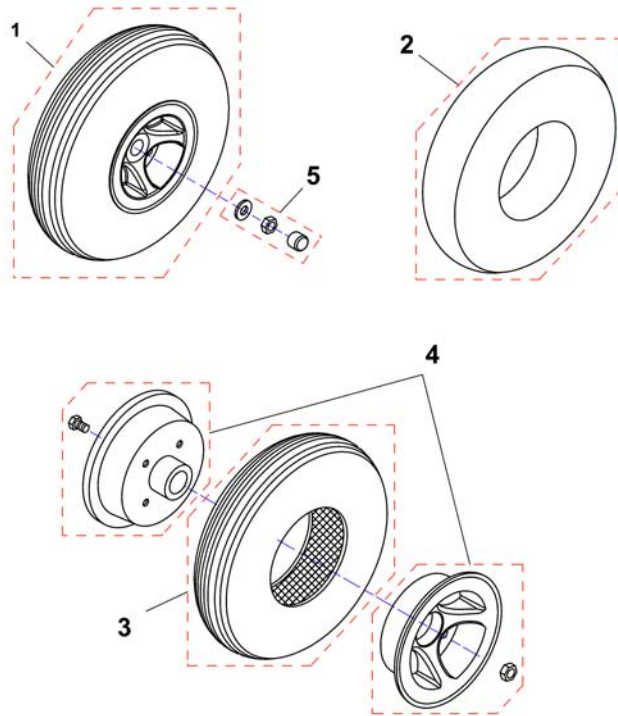

1	SP1522020	BAKHJUL KPL COMET	1
1	SP1586121	(Rear Wheel with tube 13", cpl. - black)	1
2	SP1527714	SLANGE COMET	1
3	SP1527750	DEKK MED SLANGE STD 1STK COMET	1
3	SP1586118	(Tyre with tube 13" - black)	1
3	SP1543159	DEKK KOMPACT COMET COMET	1
3	SP1586119	(Solid Tyre PU 13" - black)	1
4	SP1527749	FELG KOMPLETT BAK COMET COMET	1
5	SP1557459	MUTTER FOR FESTE BAKHJUL COMET	1

Invacare® Comet | Sving-og Drivhjul
(Anti-tipp-wheel)


1522021X.dwg

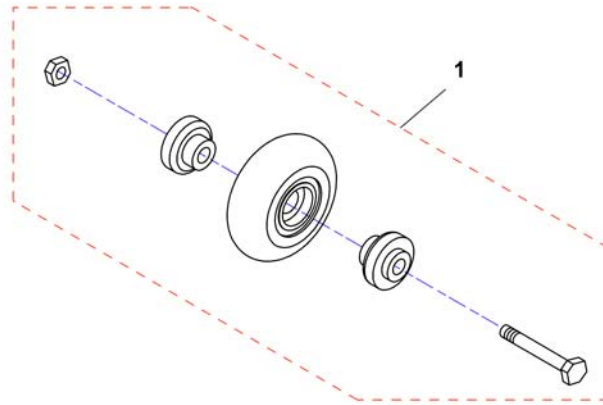

1 SP1522021

HJUL ANTI-TIPPER COMET

1

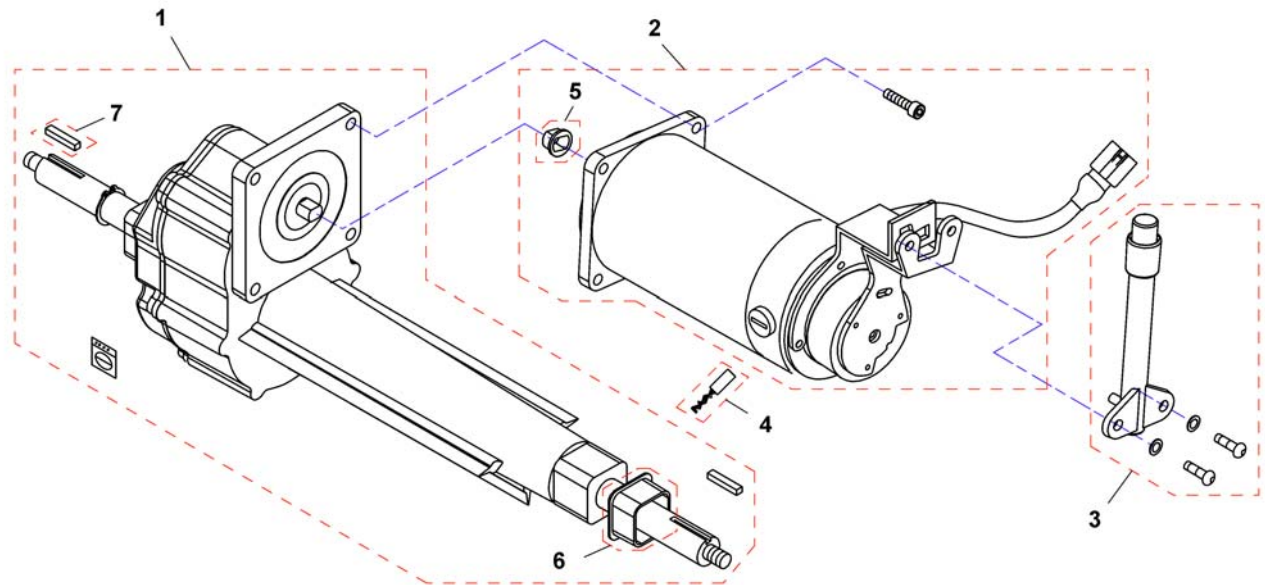

1	SP1522011	(Differential 10 km/h)	1
1	SP1522013	(Differential 12,8 km/h & 15 km/h)	1
1	SP1533710	DIFFERENSIAL COMET	1
2	-	(Motors, Powermodules & Speed Reduction)	0
3	SP1505672	FRIKOPLINGSHENDEL COMET	1
-	SP1587600	(Brush Cap Set)	1
4	SP1533853	BØRSTER COMET	1
-	SP1587601	(Brush Cap Set (HD / Alpine))	1
5	1505667	FØRING COMET	1
5	SP1533842	GLIDER COMET	1
6	SP1533852	GUMMIMATTE COMET	1
7	1505669	(Key 5x30)	1
7	SP1553438	SPLINT AKSLING TRANSAKS COMET <i>Comet HD/Alpine</i>	1
4	SP1527845	(Carbon Brush (4 Pieces))	1

(Motors, Powermodules & Speed Reduction)

1577207.no

Name	Wheels	Color	Notice	km/h	Serialnumber	Rhino	Motor	Power-module	Speed Reduction
Comet	3	blue		15	-> 11FRA0049	1	9	25	33
Comet	3	blue		15	11FRA0050 - 11GRA0005	1	9	25	30
Comet	3	blue		15	11GRA0006 ->	2	10	20	30
Comet	3	blue	⊘	15	-> 11FRA0096	1	9	25	33
Comet	3	blue	⊘	15	11FRA0097 - 11GRA0023	1	9	25	30
Comet	3	blue	⊘	15	11GRA0024 ->	2	10	20	30
Comet	3	silver		15	-> 11FRA0045	1	9	25	33
Comet	3	silver		15	11FRA0046 - 11GRA0000	1	9	25	30
Comet	3	silver		15	11GRA0001 ->	2	10	20	30
Comet	4	blue		12,8	-> 11FRB0416	1	9	25	33
Comet	4	blue		12,8	11FRB0417 - 11GRB0122	1	9	25	30
Comet	4	blue		12,8	11GRB0123 ->	2	10	20	30
Comet	4	silver		12,8	-> 11FRB0411	1	9	25	33
Comet	4	silver		12,8	11FRB0412 - 11GRB0115	1	9	25	30
Comet	4	silver		12,8	11GRB0116 ->	2	10	20	30
Comet	4	blue		15	-> 11FRB0376	1	9	25	33
Comet	4	blue		15	11FRB0377 - 11GRB0024	1	9	25	30
Comet	4	blue		15	11GRB0025 ->	2	10	20	30
Comet	4	silver		15	-> 11FRB0391	1	9	25	33
Comet	4	silver		15	11FRB0392 - 11FRB0786	1	9	25	30
Comet	4	silver		15	11FRB0787 ->	2	10	20	30
Comet HD	4	grey		10	-> 11FRB0406	1	11	17/18	32
Comet HD	4	grey		10	11FRB0407 - 11GRB0129	1	11	17/18	28
Comet HD	4	grey		10	11GRB0130 ->	2	12	21	28
Comet Alpine	3	grey	⊘	10	->11FRA0042	1	13	17/18	32
Comet Alpine	3	grey	⊘	10	11FRA0043 - 11GRA0013	1	13	17/18	28
Comet Alpine	3	grey	⊘	10	11GRA0014 ->	2	14	21	28
Comet Alpine	4	grey		10	-> 11FRB0401	1	13	17/18	32
Comet Alpine	4	grey		10	11FRB0402 - 11GRB0020	1	13	17/18	28
Comet Alpine	4	grey		10	11GRB0021 ->	2	14	21	28
Comet Alpine	4	grey	⊘	10	-> 11FRB0299	1	13	17/18	32
Comet Alpine	4	grey	⊘	10	11FRB0300 - 11GRB0025	1	13	17/18	28
Comet Alpine	4	grey	⊘	10	11GRB0026 ->	2	14	21	28

⊘ = without batteries

9	SP1522017	(Motor 12,8 km/h & 15 km/h, Rhino)	1
10	SP1553429	(Motor 12,8 km/h & 15 km/h, Rhino2)	1
11	SP1533850	\$MOTOR BRUK SP1533851	1
12	SP1553426	\$MOTOR BRUK SP1553427 COMET ALPINE RHINO2	1
13	SP1533851	MOTOR COMET ALPINE RHINO COMET TIL TRANSAKSEL	1
14	SP1553427	MOTOR COMET ALPINE RHIN2 COMET	1
17	-	(No longer available.)	1
18	-	(No longer available.)	1
20	SP1553399	(Controller, Rhino2, 120A, DS120)	1
21	SP1553400	POWERMODUL MODUL COMET ALPINE FRA OKT2011	1
25	SP1559396	(Upgrade Kit Comet 12,8 & 15km/h Rhino1 - 110A to Rhino2 - 120A)	1
-	-	(For Mounting the Speed Reduction Switch, the Adjustment Gauge SP1581766 is needed!)	0
-	SP1581766	(Adjustment Gauge, Speed Reduction Switch)	1
28	SP1556266	BRYTER FARTSREDUKSJON COMET 2 GENERASJON	1
30	SP1556282	(Speed Limit Switch Box 2. Generation, Orion 3 & 4-wheel 12 km/h - Comet 3 & 4-wheel 12,8 & 15 km/h)	1
32	SP1554026	BRYTER FARTSRED OPPGRAD COMET KIT GAMMEL=>NY TYP	1
33	SP1554027	(Speed Limit Switch Box Upgrade Kit to 2. Generation, Orion 4-Wheel 12 km/h - Comet 3 & 4-Wheel 12,8 & 15 km/h)	1

(Upgrade Kits - Powermodules & Speed Reduction)

RHINOUPGRADE.iss

Upgrade Kit Powermodule - Rhino >>> Rhino2*incl. Controller & Motor Assembly*

Orion			Comet		Comet Alpine	Comet HD
6 km/h	10 km/h	12 km/h	12,8 km/h	15 km/h	10 km/h	10 km/h
22	23	24	25		26	27

Upgrade Kit Speed Reduction - Generation 1 >>> Generation 2

	3-Wheel					4-Wheel				
	6 km/h	10 km/h	12 km/h	12,8 km/h	15 km/h	6 km/h	10 km/h	12 km/h	12,8 km/h	15 km/h
Orion	34	35	36	-	-	31	32	33	-	-
Comet	-	32	-	33	33	-	32	-	33	33
Comet HD	-	32	-	-	-	-	32	-	-	-
Comet Alpine	-	32	-	-	-	-	32	-	-	-


22	SP1553627	(Upgrade Kit Orion 6km/h Rhino - 70A to Rhino2 - 90A)	1
23	SP1559360	(Upgrade Kit Orion 10km/h Rhino - 110A to Rhino2 - 120A)	1
24	SP1559395	(Upgrade Kit Orion 12km/h Rhino - 110A to Rhino2 - 120A)	1
25	SP1559396	(Upgrade Kit Comet 12,8 & 15km/h Rhino1 - 110A to Rhino2 - 120A)	1
26	SP1559397	(Upgrade Kit Comet Alpine Rhino1 - 160A to Rhino2 - 160A)	1
27	SP1559398	(Upgrade Kit Comet HD Rhino1 - 160A to Rhino2 - 160A)	1
-		(For Mounting the Speed Reduction Switch, the Adjustment Gauge SP1581766 is needed!)	0
	SP1581766	(Adjustment Gauge, Speed Reduction Switch)	1
31	SP1554025	(Speed Limit Switch Box Upgrade Kit to 2. Generation, Orion 4-Wheel 6km/h)	1
32	SP1554026	BRYTER FARTSRED OPPGRAD COMET KIT GAMMEL=>NY TYP	1
33	SP1554027	(Speed Limit Switch Box Upgrade Kit to 2. Generation, Orion 4-Wheel 12 km/h - Comet 3 & 4-Wheel 12,8 & 15 km/h)	1
34	SP1554028	(Speed Limit Switch Box Upgrade Kit to 2. Generation, Orion 3-Wheel 6 km/h)	1
35	SP1554029	(Speed Limit Switch Box Upgrade Kit to 2. Generation, Orion 3-Wheel 10 km/h)	1
36	SP1554030	(Speed Limit Switch Box Upgrade Kit to 2. Generation, Orion 3-Wheel 12 km/h)	1

(Batteries)

	Capacity [Ah]															
	1,2	12	30	31	32	36	38	40	40	50	50	50,9	60	73,6	75	97,6
Bora/SpectraXTR								8		10						
Dragon										10			13			
Dragon Vertic													13			
FDX													13			
G50													13	14		16
Kite													13	14		
Mirage				4				8								
Mistral3				4									12			
P9000XDT					5											
Pronto M41					5											
Pronto M61					5											
Spectra Blitz								8					12			
Spectra Comfort								7								
Spectra Plus								7								
Spectra Seat Riser								7								
Storm3										10			13	14		
Storm3 True Track	1														14	
Storm3 Vertic															14	
Storm4															14	
Storm4 Max															14	
Stream										10			13			
TDX-SP														14		
TDX-SP Narrow													12			
Tiger										10			13			
Topan													13			
Tornado								8								
Typhoon													13			
Typhoon GB	1												13			
Typhoon GB	1												13			
Zephyr	1												13			
Auriga			3					8		10						
Comet														14	15	
Leo						6			9							
Lynx		2														
Meteor														14		
Orion												11				
length [mm]	97	152	197	196	211	196	210	197	198	198	198	238	259	260	260	329
width [mm]	49	99	132	132	132	130	175	168	165	166	165	140	169	171	168	171
height [mm]	51	102	180	183	183	179	175	175	171	176	170	235	178	210	223	238
weight [kg]	0,6	3,9	11,1	10,5	11	12	14,6	14,4	14,2	14	16	17,1	19,3	24,3	26	32,5




i	-	(Change batteries only pairwise and from the same type!)	1
1	1440386	BATTERY 1,2AH 12V FRIKPL.THYPOON	1
2	SP1506493	(Battery 12V/12Ah AGM)	1
3	P50021	(Battery, 12V/30Ah C20, GEL)	1
4	P50029	BATTERY 30AH MK12-31 TROJAN	1
5	SP1521459	(Battery, 12V/34Ah C20, AGM)	1
6	1505498	(Battery, 12V/36Ah C20, AGM)	1
7	P50028	(Battery, 12V/38Ah C20, GEL (UK & DK only))	1
8	PG50988	BATTERY 40AH MK12-40 TROJAN	1
9	SP1550692	(Battery, 12V/40Ah C20, AGM)	1
10	SP1522107	(Battery, 12V/50Ah C20, AGM)	1
11	SP1521695	(Battery, 12V/50Ah C20 AGM)	1
12	PG50989	BATTERY 50AH MK12-50 TROJAN	1
13	1432944	BATTERY 60AH MK12-60 TROJAN	1
14	P50016	BATTERY 70AH MK12-73 TROJAN	1
15	SP1522026	(Battery, 12V/75Ah C20, AGM)	1
16	1478675	BATTERY 97AH MK12-97 TROJAN	1

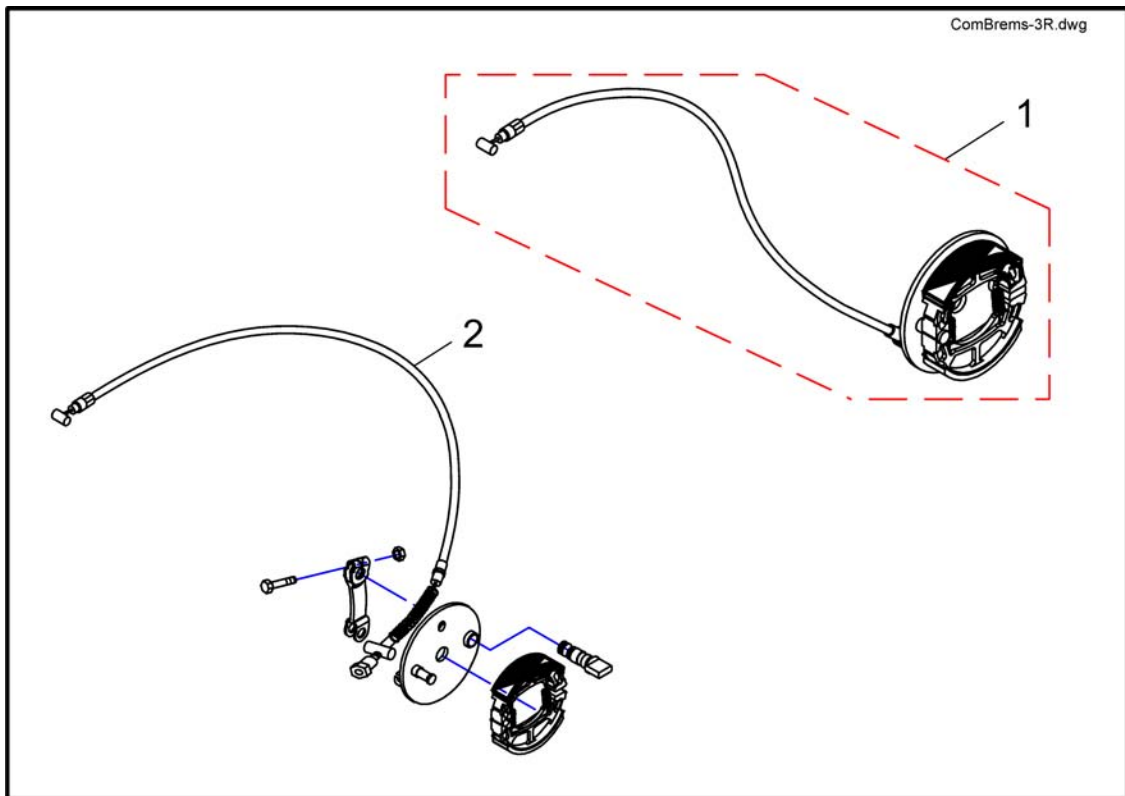


1	SP1522025	BREMS KPL M WIRE COMET ALPINE 3W	1
2	SP1518419	BREMSEWIRE 3 HJUL COMET	1

(Brakes 4 Wheeler)

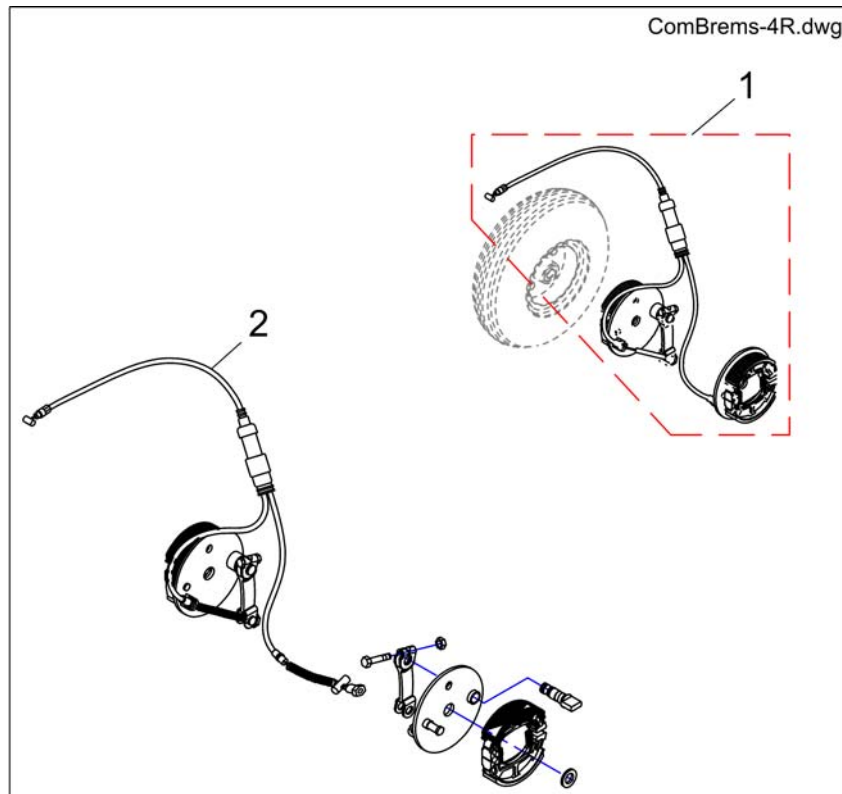


1	SP1521589	BREMS 4 HJUL COMET	1
2	SP1518422	BREMSEWIRE 4 HJUL COMET	1

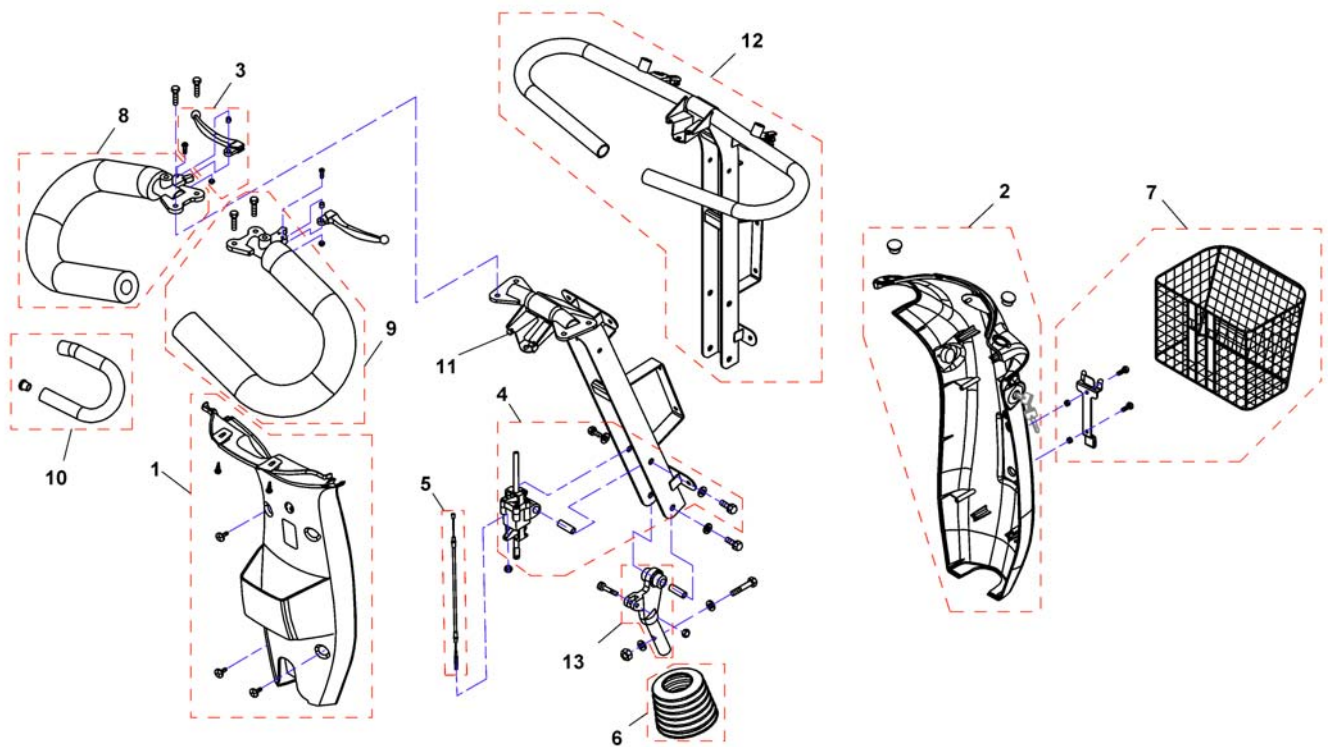


1	SP1521725	DEKSEL BAKRE TIL STYRING COMET		1
2	SP1521495	(Front Cover Steering, silver)		1
2	SP1521727	FRONTDEKSEL STYREENHET BLÅ 4 HJUL COMET		1
2	SP1537796	FRONTDEKSEL STYREENHET GRÅ/SORT (STD)4 HJ COMET		1
2	SP1551637	(Front Cover Steering, red (UK only))		1
3	SP1521578	REGULATOR HENDEL COMET		1
4	SP1505279	STYREJUSTERING COMET		1
5	SP1521583	KABEL COMET		1
6	SP1521586	GUMMI MUFFE RATTSTAMME COMET		1
7	SP1521584	KURV FORAN COMET		1
8	SP1527788	HÅNDTAK VE COMET	→ 28.02.2009	1
8	SP1527828	(Ergo Handle Bar LH)	01.03.2009 → 16.01.2012	1
9	SP1527789	HÅNDTAK HØ COMET	→ 28.02.2009	1
9	SP1527827	(Ergo Handle Bar RH)	01.03.2009 → 16.01.2012	1
10	SP1527792	MOSEGUMMI HÅNDTAK COMET		1
11	SP1541396	(Steering unit)	→ 15.01.2012	1
12	SP1555848	(Steering unit)	16.01.2012 →	1
13	1505297	JOINT STYRING NEDRE COMET		1

(Drive Electronics Steering Bar)

ComFig.1-1.dwg

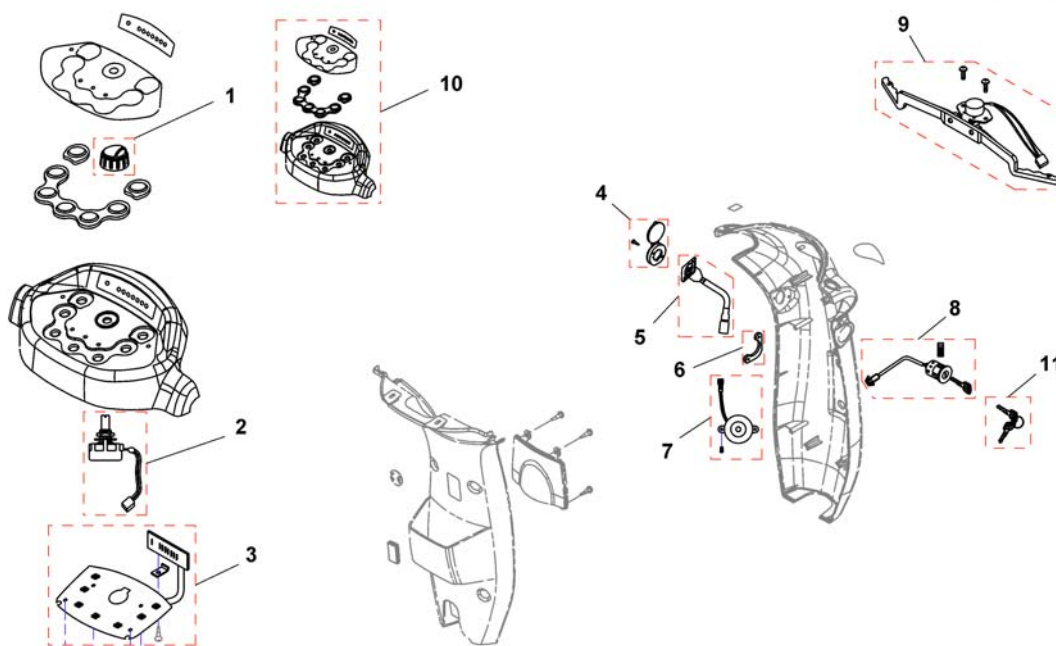


1	1505460	KNOTT HASTIGHETSKONTROLL COMET		1
2	1505462	VARIABLE MOTSTAND COMET		1
3	SP1524542	(P.C. Board)	→ 18.08.2008	1
3	SP1527395	(P.C. Board)	19.08.2008 → 24.05.2009	1
3	SP1532909	PRINTPLATE COMET <i>Serialnumber (Comet 3w xxxxxxxxx =>) (Comet 4w 09ERB0068 =>)</i>	25.05.2009 →	1
4	1523534	DEKSEL FOR LADEKONTAKT COMET		1
5	1523543	LADEKONTAKT COMET		1
6	SP1523544	(Plate)		1
7	1506153	BUZZER COMET		1
8	1505451	MAIN SWITCH COMET		1
8	SP1543154	LÅS M 2 NØKLER UNIK COMET		1
9	SP1521575	POTMETER KPL COMET ALPINE		1
10	SP1526435	(Cover, Upper Steering)	→ 24.05.2009	1
10	SP1532908	DEKSEL STYRING COMET <i>Serialnumber (Comet 3w xxxxxxxxx =>) (Comet 4w 09ERB0068 =>)</i>	25.05.2009 →	1
11	SP1530211	NØKKELE 2 STK STANDARD COMET		1

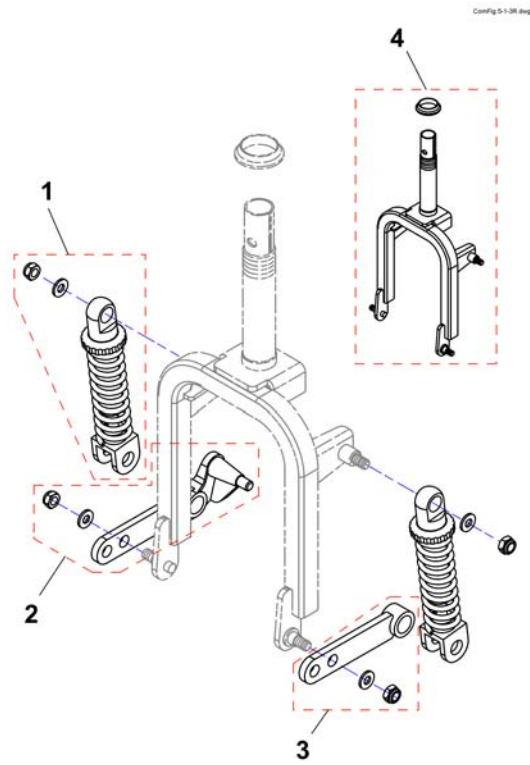


1	SP1521990	STØTDEMPER 150LBS COMET	1
2	SP1521992	(Rocker, front LH)	1
3	SP1521991	(Rocker, front RH)	1
4	SP1521989	HJUL GAFFEL COMET	1

(Steering linkage 4-Wheeler)

Conf: 5.1-4R.dwg


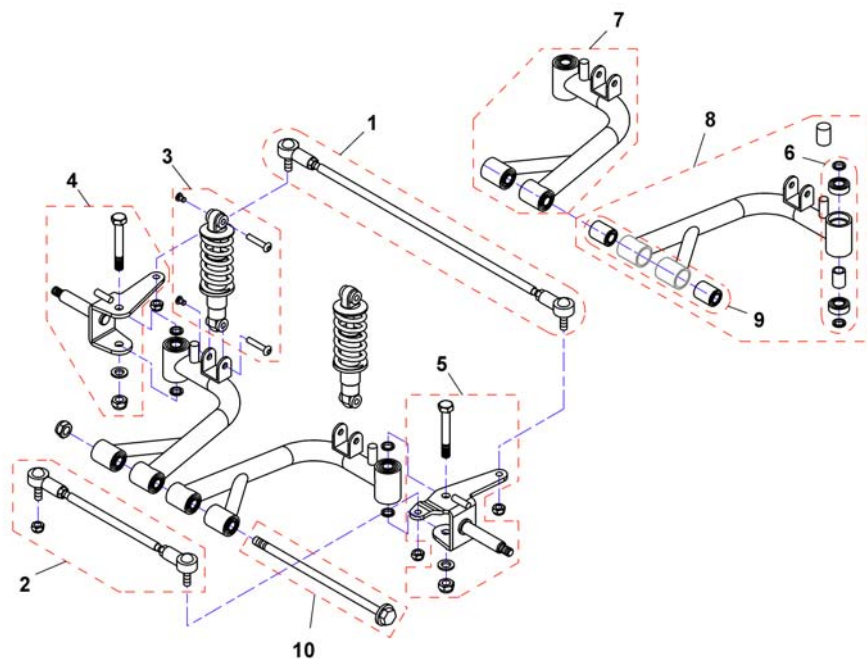


1	SP1518077	FRONT STAG KIT COMET	1
2	SP1521994	STYRESTAG FORAN KORT COMET	1
3	SP1521995	STØTDEMPER FORAN COMET	1
3	SP1533849	STØTDEMPER COMET	1
4	SP1521996	FRONT AKSEL VE COMET	1
5	SP1521997	FRONT AKSEL HØ COMET APINE	1
6	SP1522000	LAGER HJULOPPHENG 1 SETT COMET	1
7	SP1526394	HJUL STØTDEMPER VE COMET	1
8	SP1526393	HJUL STØTDEMPER HØ COMET	1
9	SP1526349	GLIDELAGER COMET	1
10	SP1545459	BOLT M10 X 215 COMET APINE	1

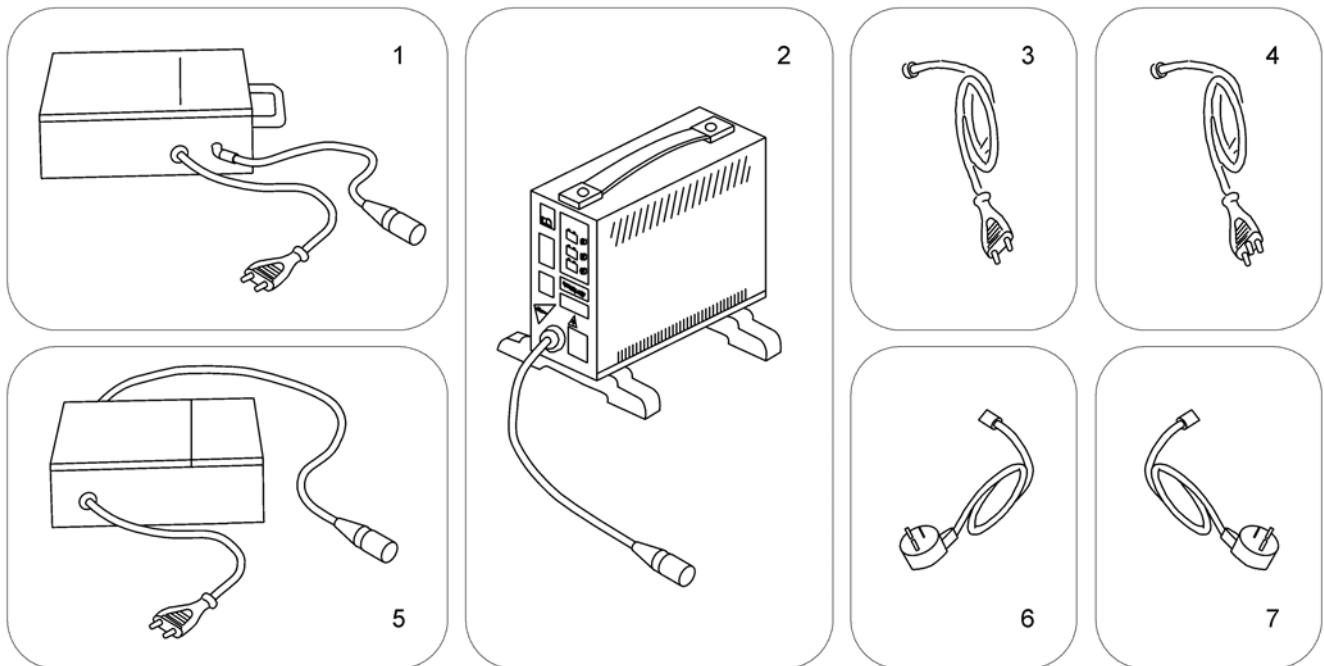


1	5813987	(Charger 8A, Gi Popular (Sweden only))	1
2	SPP51981	(Charger 8A for Batteries up to 73Ah (without power cord))	1
3	P21079	(Power Cord, Continental Shockproof Plug)	1
4	1443841	(Power Cord, Swiss)	1
5	1510201	(Charger 10A, Powercharge Classic (G50), for Batteries from 97AH (with power cord))	1
5	SP1545511	LADER LADAC/DEFA 701239 MED PLUGG	1
6	P21080	(Power Cord, UK)	1
7	SP1553849	(Power Cord, Israel)	1

(Cable)

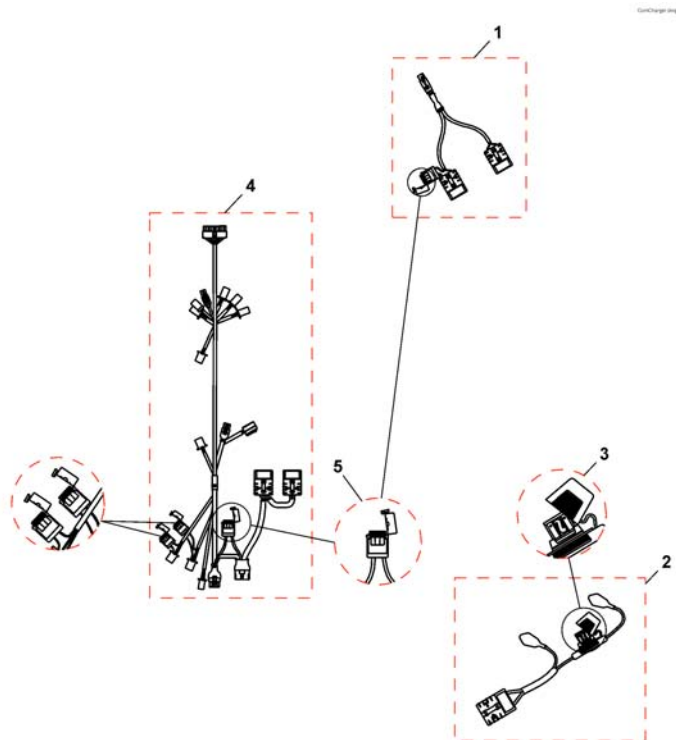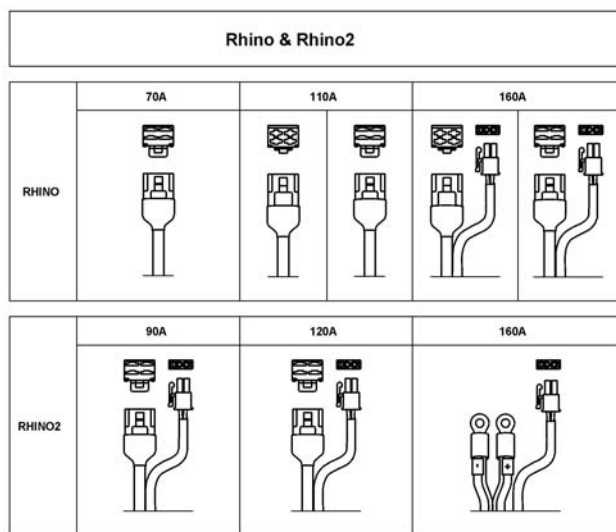


1	SP1521644	(Off Board Charging Cable Orion & Comet)		1
2	SP1521696	BATTERIKABEL COMET		1
2	SP1537917	BATTERIKABEL COMET COMET		1
3	1505500	SIKRING 70AMP COMET		1
3	SP1537918	SIKRING 100Amp. COMET ALPINE		1
4	SP1521761	KABELSTAMME COMET	→ 18.08.2008	1
4	SP1527400	LEDNINGSNETT COMET BRUK 1532911	18.08.2008 → 24.05.2009	1
		<i>Serialnumber (Comet 3w 08JRA0001) (Comet 4w 08HRB0082)</i>		
4	SP1532911	KABELSTAMME COMET	25.05.2009 →	1
		<i>Serialnumber (Comet 3w 10ARA0001 ==>) (Comet 4w 09ERB0068 ==>)</i>		
4	SP1553401	(Wire Harness, 120A, Rhino2)		1
4	SP1553402	KABELSTAMME COMET RHINO2 COMET		1
5	SP1521577	SIKRING 10A COMET		1

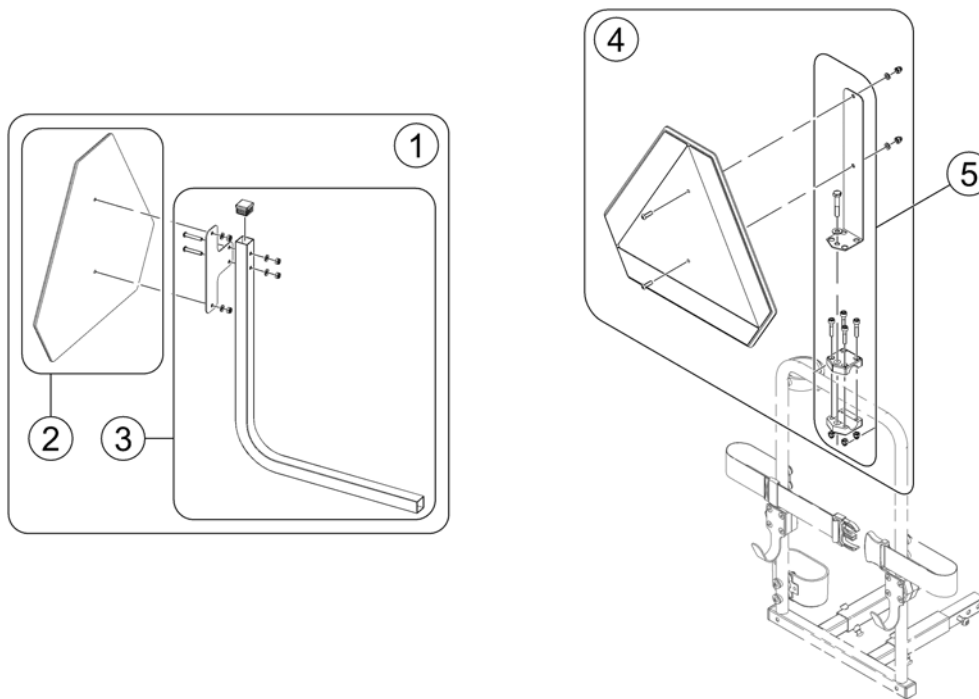


1	SP1523648	(Warning Triangle Scooter)	1
2	1430834	(Rear Marking Sign ECE-R 69)	1
3	SP1526913	(Holder Warning Triangel)	1
4	SP1430410	(Reflective Sign ECE-R 69, complete)	1
5	SP1580671	(Bracket, Reflective Sign, cpl.)	1

(Accessories)

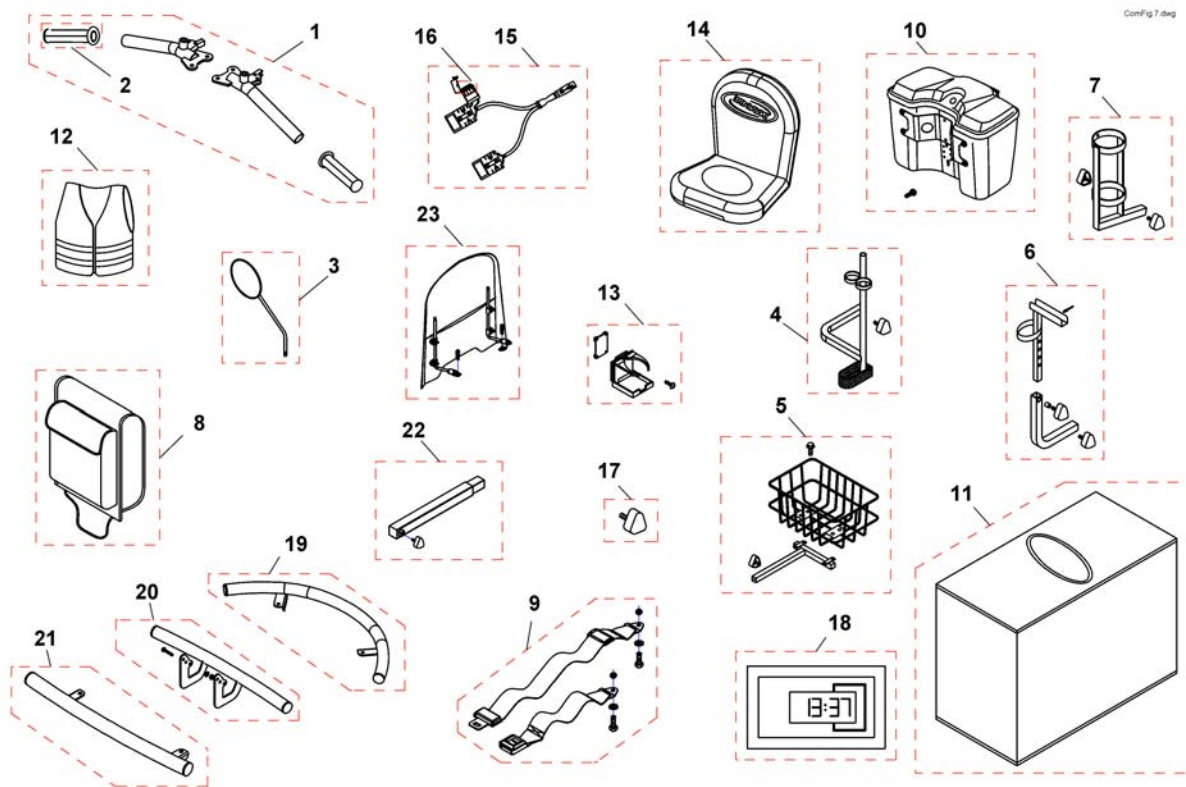


1449280	(USW wizard dongle kit, EDL-USW, cpl.)		1
1 SP1518330	(T-Shape Steering Bars)	→ 15.01.2012	1
2 1505415	(Grip, Handle Bar)	→ 15.01.2012	1
3 SP1518241	SPEIL COMET		1
4 SP1528274	KRYKKEHOLDER COMET COMET <i>If retrofitting Comet HD, also order SP1541502</i>		1
5 SP1505727	KURV BAK COMET COMET <i>If retrofitting Comet HD, also order SP1541502</i>		1
6 SP1505728	HOLDER FOR RULLATOR COMET		1
7 SP1505729	HOLDER OKSYGENFLASKE COMET		1
8 SP1505731	(Bag, Seat Rear)		1
9 SP1518274	HOFTEBELTE COMET		1
10 SP1521595	(Lockable Storage Box - silver)		1
10 SP1521661	(Lockable Storage Box - onyx blue)		1
10 SP1537710	BAGASJEBOKS FORAN M LÅS COMET		1
10 SP1551408	(Lockable Storage Box Velvet Grey)		1
10 SP1551638	(Lockable Storage Box - ruby red)		1
11 SP1518240	PRESENNING COMET COMET <i>currently not available for Comet HD</i>		1
12 SP1521592	(Safety vest)		1
13 SP1521594	HOLDER KOPP COMET		1
14 SP1521684	SETETREKK STD COMET <i>currently not available for Comet HD</i>		1
15 SP1521644	(Off Board Charging Cable Orion & Comet)		1
16 SP1521577	SIKRING 10A COMET		1
17 SP1507005	(Star Handle)		1
18 SP1571491	(Hourcounter)		1
19 SP1528297	(Front Bumper, 3-Wheeler)		1
20 SP1528298	STØTFANGER FORAN COMET 4W		1
21 SP1528299	STØTFANGER BAK COMET COMET		1
22 SP1541502	(Extension Tube Comet HD) <i>Comet HD only</i>		1
23 SP1543160	VINDSKJERM COMET		1

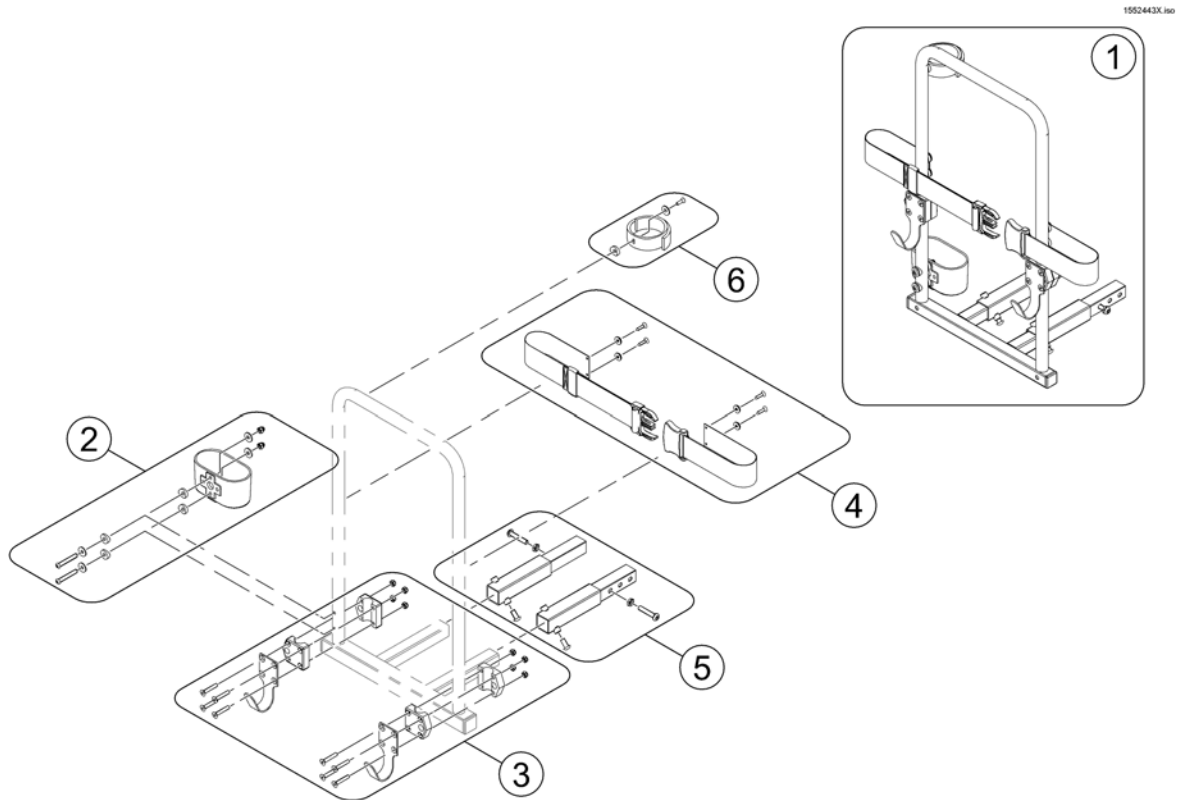



1	SP1553851	RULLATORHOLDER UNIVERSAL COMET ALLE RULLATORER	1
2	SP1560465	KRYKKEHOLDER KUN SKÅL COMET	1
3	SP1553890	(Rollator hooks)	1
4	SP1553891	(Belt, cpl.)	1
5	SP1552594	FORLENGER FESTE KRYKKEH COMET <i>only needed for Orion & Comet</i>	1
6	SP5009551	VELCRO KRYKKEHOLDER COMET	1

(Storage box rear bracket)

150257X.png

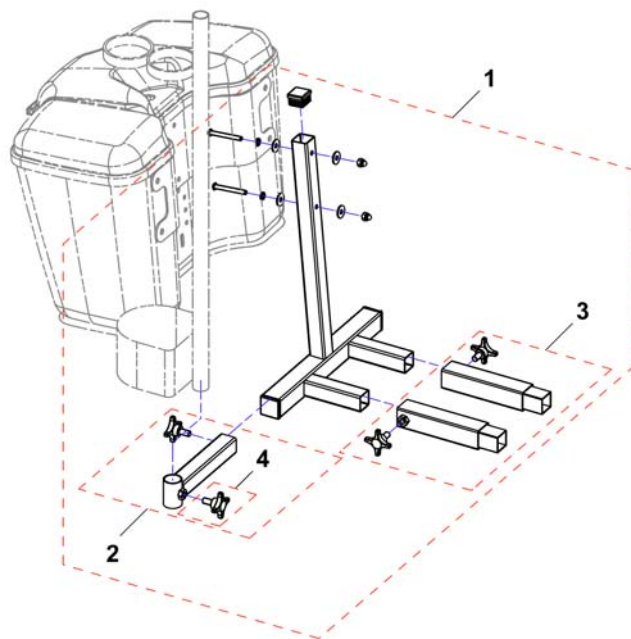

1	SP1552537	BRAKETT BAGASJEBOKS BAK COMET	1
2	SP1553854	FESTE KRYKKEHOLDER COMET	1
3	SP1552594	FORLENGER FESTE KRYKKEH COMET <i>only needed for Orion & Comet</i>	1
4	5824117	STJERNEGREP FESTE NY MANØVERBOKS MOOVER	1

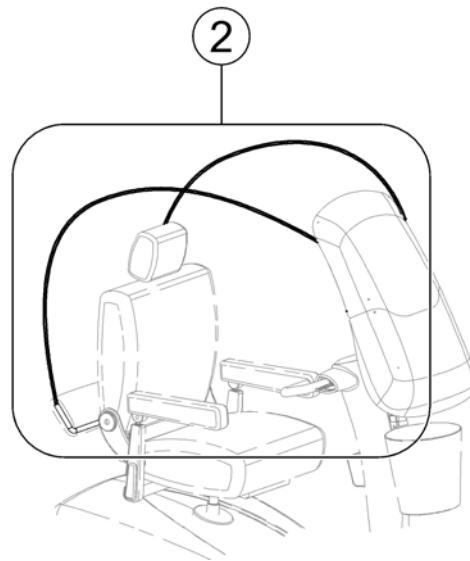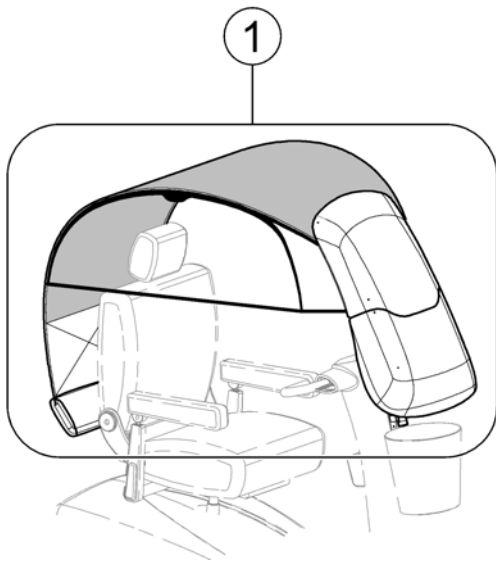

1	SP1572494	TAK COMET COMET	1
2	SP1572507	(Fiber Rods for Rain- & Suncover)	1
-	SP1587493	(Rear Holder (KlickFix)) <i>not shown</i>	1



Yes, you can.®



Belgium & Luxemburg:

Invacare nv, Autobaan 22, B-8210 Loppem
Tel: (32) (0)50 83 10 10, Fax: (32) (0)50 83 10 11
belgium@invacare.com / www.invacare.be

Denmark:

Invacare A/S, Sdr. Ringvej 37, DK-2605 Brøndby
Tel: (45) (0)36 90 00 00, Fax: (45) (0)36 90 00 01
denmark@invacare.com / www.invacare.dk

Deutschland:

Invacare Aquatec GmbH, Alemannenstraße 10, D-88316 Isny
Tel: (49) (0)75 62 7 00 0, Fax: (49) (0)75 62 7 00 66
info@invacare-aquatec.com / www.invacare-aquatec.de

European Distributor Organisation:

Invacare, Kleiststraße 49, D-32457 Porta Westfalica
Tel: (49) (0)57 31 754 540, Fax: (49) (0)57 31 754 541
edo@invacare.com / www.invacare.eu.com

España:

Invacare SA, c/Areny s/n, Polígon Industrial de Celrà, E-17460 Celrà (Girona)
Tel: (34) (0)972 49 32 00, Fax: (34) (0)972 49 32 20
contactsp@invacare.com / www.invacare.es

France:

Invacare Poirier SAS, Route de St Roch, F-37230 Fondettes
Tel: (33) (0)2 47 62 64 66, Fax: (33) (0)2 47 42 12 24
contactfr@invacare.com / www.invacare.fr

Ireland:

Invacare Ireland Ltd, Unit 5 Seatown Business Campus, Seatown Road, Swords, County Dublin - Ireland
Tel: (353) 1 810 7084, Fax: (353) 1 810 7085
ireland@invacare.com / www.invacare.ie

Italia:

Invacare Mecc San s.r.l., Via dei Pini 62, I-36016 Thiene (VI)
Tel: (39) 0445 38 00 59, Fax: (39) 0445 38 00 34
italia@invacare.com / www.invacare.it

Nederland:

Invacare BV, Celsiusstraat 46, NL-6716 BZ Ede
Tel: (31) (0)318 695 757, Fax: (31) (0)318 695 758
nederland@invacare.com, csede@invacare.com / www.invacare.nl

Norge:

Invacare AS, Grensesvingen 9, Postboks 6230, Etterstad, N-0603 Oslo
Tel: (47) (0)22 57 95 00, Fax: (47) (0)22 57 95 01
norway@invacare.com, island@invacare.com / www.invacare.no

Österreich:

Invacare Austria GmbH, Herzog Odilostrasse 101, A-5310 Mondsee
Tel: (43) 6232 5535 0, Fax: (43) 6232 5535 4
info-austria@invacare.com / www.invacare.at

Portugal:

Invacare Lda, Rua Estrada Velha, 949, P-4465-784 Leça do Balio
Tel: (351) (0)225 1059 46/47, Fax: (351) (0)225 1057 39
portugal@invacare.com / www.invacare.pt

Sverige & Suomi:

Invacare AB, Fagerstagatan 9, S-163 91 Spånga
Tel: (46) (0)8 761 70 90, Fax: (46) (0)8 761 81 08
sweden@invacare.com, finland@invacare.com / www.invacare.se

Switzerland:

Invacare AG, Benkenstrasse 260, CH-4108 Witterswil
Tel: (41) (0)61 487 70 80, Fax: (41) (0)61 487 70 81
switzerland@invacare.com / www.invacare.ch

United Kingdom:

Invacare Limited, Pencoed Technology Park, Pencoed, Bridgend CF35 5AQ
Switchboard Tel: (44) (0)1656 776200, Fax: (44) (0)1656 776201
Customer services Tel: (44) (0)1656 776222, Fax: (44) (0)1656 776220
uk@invacare.com / www.invacare.co.uk