

Spare Parts Catalogue

Version: 2015/06/25a NO



Invacare® TDX-SP



Yes, you can.®

General Information

For any order of spare parts the following information is required:

- Product type*
- Product name & model*
- Serial Number*
- Part-Number*
- Part-Description*

Exploded view drawings and code numbers are not contractual and subject to modification, in relation to Invacare® continuous product improvement policy to meet end users needs.

The reproduction or use of this work, or of any substantial part of this work, in any form or by any electronic, mechanical or other means, including photocopying, scanning and recording, and in any information storage and retrieval system, including electronic transmission, is forbidden without the prior written permission of Invacare® International Sarl.



Yes, you can.®

TABLE OF CONTENTS

Informasjon

(User Manuals TDX-SP)	9
(User Manuals REM550)	10
(User Manuals REM24SD)	11
(User Manuals Shark2)	12
(User Manuals REM A & REM B)	13
(User Manuals G90)	14
(User Manuals G91)	15
(User Manuals IDC Dual Control)	16
(User Manuals REM34 & REM41 Tray/Centre Control)	17
(User Manuals ACS Handprogrammer)	18
(User Manual Proximity Head Remote DSL-105)	19
(User Manuals Ultra Low Maxx Seat)	20

Chassis

(Walking Beam ==> 06.09.11)	21
(Walking Beam SSD-Motors)	22
(Rear Pivot Assembly)	23
(Rear Pivot Assembly (Mounting))	24
(Cylinders, Locking Gas)	25
(Fender)	26
(Shrouds and Hardware)	27
(Battery Box Hardware)	28

Sete

Sete	29
(Standard Seat)	30
(Powered Tilt 50° & manuell)	31
(Tiltmodule 28°)	32
(Lifter (incl. 28°-Tilt))	33
(Bellow)	34
(Seat frames for Balle modules)	35
(Adapter sheets)	36
(Seatplates)	37
(Seat Cushion for Standard Seat)	38
(Contour Seat)	39
(Powered Tilt 50° & manuell)	31
(Tiltmodule 28°)	32
(Lifter (incl. 28°-Tilt))	33
(Bellow)	34
(Seat frames for Balle modules)	35
(Adapter sheets)	36
(Seatplates)	37
(Seat Cushion for Contour Seat)	40
(Flex Seat)	41
(Powered Tilt 50° & manuell)	31
(Tiltmodule 28°)	32
(Lifter (incl. 28°-Tilt))	33
(Bellow)	34
(Seat frames for Balle modules)	35



Yes, you can.®

TABLE OF CONTENTS

(Adapter sheets)	36
(Seatplates)	37
(Seat Cushion for Flex Seat)	42
(Junior Seat)	43
(Powered Tilt 50° & manuell)	31
(Tiltmodule 28°)	32
(Lifter (incl. 28°-Tilt))	33
(Bellow)	34
(Seat frames for Balle modules)	35
(Adapter sheets)	36
(Seatplates)	37
(Seat Cushion for Junior Seat)	44
(Modulite Seat)	45
(man. settable Tilt)	46
(powered Tilt 0°-30°)	47
(Lifter)	48
(One Piece Seat Plate)	49
(Telescopic Seat Frame and Plate)	50
(Telescopic Seat Frame for Tilt 12° / 20°)	51
(Telescopic Seat Frame for Tilt 30° / Lifter)	52
(Armrest bracket)	53
(Legrest Support)	54
(Seat Plate)	55
(Telescopic Seat Frame and Sling Seat)	56
(Telescopic Seat Frame for Tilt 12° / 20°)	57
(Telescopic Seat Frame for Tilt 30° / Lifter)	58
(Armrest bracket)	53
(Legrest Support)	59
(Sling Seat)	60
(Telescopic Seat Frame and Universal Adapter)	61
(Telescopic Seat Frame for Tilt 12° / 20°)	51
(Telescopic Seat Frame for Tilt 30° / Lifter)	52
(Armrest bracket)	53
(Legrest Support)	54
(Adapter Kit for Universal Seat)	62
(Seat Cushion)	63

Rygg

Rygg	66
(Backrest for Standard Seat)	67
(Backrest Standard Seat, settable)	68
(Backrest Recliner)	69
(Backrest for Contour Seat)	70
(Backrest Flex Seat)	71
(Backrest Adjustment Flex/Contour manual)	72
(Backrest adjustment Flex/Contour electric)	73
(Backrest Cushion for Contour Seat)	75
(Backrest for Flex Seat)	76
(Backrest Flex Seat)	71
(Backrest Adjustment Flex/Contour manual)	72
(Backrest adjustment Flex/Contour electric)	73
(Backrest Cushion for Flex Seat)	77



Yes, you can.®

TABLE OF CONTENTS

(Backrest for Junior Seat)	78
(Backrest / Junior Seat)	79
(Backrest Cushion for Junior Seat)	80
(Backrest for Modulite Seat)	81
(Support for Backrest Frame)	82
(Backrest Frame)	83
(Sling Back, tension adjustable)	84
(Backrest support)	85
(Backposts)	86
(Connection tube)	87
(Back Recliner / Stabilization bracket)	88
(Backrest Cushions)	89
(Flex 3 / Laguna)	90
(Matrx Posture Back)	91
(Matrx Elite T/R Back)	92
(Matrx Elite Deep Back)	93
(Matrx Elite Full Back)	94

Benstøtter og holdere

(Legrest fixation)	95
(Hanger Vari A (04/2009 =>))	96
(Hanger Vari F)	97
(Hangers ADM LH/RH manual)	98
(Hanger Vari ADE/ADP electric (07/2005 =>))	99
(Central Legrest, manual - Standard- / Easy Adapt-Seat)	100
(Central Legrest, manual - Modulite Seat)	101
(Central Legrest LNX electr. - Standard- / Easy Adapt-Seat)	102
(Central Legrest LNX electr. - Modulite Seat)	103
(Centre-mounted Legrests)	104
(Bracket Centr. Legrest LNX - Narrow Base)	105

Fotplate / Innerrør

(Footrests Vari, Flip-Up (01.08.10 =>))	106
(Footrests Vari, Flip-Up & angle adjustable (01.08.10 =>))	107
(Footplate)	108
(Footplate with support tube fix RH/LH)	109
(Footplate with support tube angle adjust. RH/LH)	110
(Support Tube with Footplate for central Legrest)	111
(Footrest Lower Parts)	112

Armlen

Armlen	113
(Armrest for Standard Seat)	114
(Armrest Standard)	115
(Armrests cpl.)	116
(T-Armrest, detachable, with Armpad, for Standard Seat 2008-08-01 ==>)	117
(Armrest for Contour Seat)	118
(Armrest Standard)	115
(Armrests cpl.)	116
(Mounting bracket Flip-up armrests)	119
(Following armrests)	120



Yes, you can.®

TABLE OF CONTENTS

(Sliding Armrest)	121
(Skirtguard for following armrest)	122
(Armrest for Flex Seat)	123
(Armrest Standard)	115
(Armrests cpl.)	116
(Mounting bracket Flip-up armrests)	119
(Following armrests)	120
(Sliding Armrest)	121
(Skirtguard for following armrest)	122
(Armrest for Junior Seat)	124
(Armrests cpl.)	116
(Armrests for Modulite Seat)	125
(T-Armrest)	126
(Padded Armrest)	127
(Inrigged Armrest)	128
(Flip-Up Armrest)	129
(Following Armrest)	130
(Hip Support)	131

Sving-og Drivhjul

(Castors Front/Rear)	132
----------------------------	-----

Senter (Driv) hjul

(Drive Wheels)	133
----------------------	-----

Drivenhet

(Motor & Gearbox ==> 06.09.11)	134
(Propulsion SSD - Details)	135

Elektronikk

Elektronikk	138
(ACS Electronic)	139
(ACS Electronic Modules)	140
(Powermodule ACS2)	141
(ACT2/4 for ACS2)	142
(Bracket ACT narrow base)	143
(Lightboard ACS2)	144
(G-Trac-sensor)	145
(Holder for G-Trac)	146
(ACS Remote & Brackets)	147
(REM 24 SD with brackets)	148
(Remote REM 24SD)	149
(Remote Bracket Standard)	150
(Remote Bracket Swing away & height adjust.)	151
(Remote Bracket Attendant)	152
(REM A & B with brackets)	153
(Remote Bracket Standard)	150
(Remote Bracket Swing away & height adjust.)	154
(Remote Bracket Attendant)	152
(REM 550 with brackets)	155



Yes, you can.®

TABLE OF CONTENTS

(Remote Bracket Standard)	150
(Remote Bracket Swing away & height adjust.)	151
(Remote Bracket Attendant)	152
(REM 34 with brackets)	156
(Remote Bracket Standard)	157
(Remote Bracket Swing away & height adjust.)	158
(G90A with brackets)	159
(Remote Bracket Standard)	160
(Remote Bracket Swing away & height adjust.)	161
(Remote Bracket Attendant)	162
(G91S with brackets)	163
(Remote Bracket Standard)	164
(Centre control REM 41)	165
(Small Chin Control with brackets)	166
(Small Chincontrol)	167
(Framework for chincontrol)	168
(Remote Dual Control with Brackets)	169
(Remote Bracket Dual Control for Standard-/Contour-/Flex-/Junior-Seat)	170
(Remote Bracket Dual Control for Modulite Seat)	171
(Intuitive Dual Control with brackets)	172
(ACS Cables & Accessories)	173
(Bus-Cables ACS)	174
(T-Adapter)	175
(T-Adapter - Modulite Seat)	176
(Programming Kits)	177
(Magnetkey for Remotes)	178
(Shark Electronic)	179
(Shark Electronic Modules)	180
(Shark Powermodule)	181
(Lightboard ACS2)	144
(Shark Remote & Brackets)	182
(Remote Shark with Brackets)	183
(Remote Bracket Standard)	150
(Remote Bracket Swing away & height adjust.)	154
(Remote Dual Control with Brackets)	184
(Remote Bracket Dual Control)	185
(Shark Cables & Accessories)	186
(Bus-Cables Shark)	187
(Cables Shark)	188
(Programming Kits)	177

kabel og lader

(Batteries)	189
(Chargers)	190
(Battery support)	191
(Battery Tray Assembly and Wire Harnesses)	192

Sikkerhedsutstyr

(Light System)	193
(Rear Light Protector Kit)	194
(Lightsystem LED Front)	195



Yes, you can.®

TABLE OF CONTENTS

(Reflective Sign)	196
(Lap Belt)	197
(Lap Belts (Modulite Seat))	198
(H-Belts (Modulite Seat))	199
(Rearview Mirror)	200

Seteustyr

(Backrest Extension Flex)	201
(Abduction Wedge)	202
(Lateral Support - Flex-2 Seat)	203
(Lateral Support, multi adjustable - Easy Adapt)	204
(Lateral Support - Modulite Seat)	205
(Lateral Support - Modulite Seat)	206
(Lateral Support - Junior Seat)	207
(Head- & Neckrests)	208
(Push Handle)	209

Tilbehør og andre deler

(Caneholder)	210
(Caneholder for Standard Seat)	211
(Caneholder for Contour/Flex Seat)	212
(Cane holder (Modulite: backrest frame))	213
(Cane holder (Modulite: back posts man recliner))	214
(Cane holder (Modulite: back posts pow recliner))	215
(Trays)	216
(Hour Counter)	217
(KLICKfix® bag)	218
(KLICKfix® bracket for Fixed Armrest)	219
(KLICKfix® bracket for T-Armrest)	220
(KLICKfix® bracket for Inrigged/Flip-Up/Following Armrest (Modulite))	221
(Bag)	222
(Bumper/Sport Accessories)	223
(T-Nuts for Accessories Modulite Seat)	224

BG	Български	FR	Français	NL	Nederlands	SV	Svenska
CS	Čeština	HE	עברית	NO	Norsk	UK	Українська
DA	Dansk	HR	Hrvatski	PL	Polski		
DE	Deutsch	HU	Magyar	PT	Português		
EN	English	IT	Italiano	RU	Русский		
ES	Español	JA	日本語	RO	Română		
ET	Eesti	LV	Latviešu	SK	Slovenčina		
FI	Suomi	LT	Lietuvių	SL	Slovenščina		

CS	1531137	(User Manual TDX-SP Czech)	1
DA	1537969	(User Manual TDX-SP Dansk)	1
DE	1521463	(User Manual TDX-SP German)	1
EN	1521464	(User Manual TDX-SP English)	1
ES	1522212	(User Manual TDX-SP Spanish)	1
FR	1522172	(User Manual TDX-SP French)	1
IT	1522187	(User Manual TDX-SP Italian)	1
LT	1531140	(User Manual TDX-SP Lithuanian)	1
NO	1537970	BRUKSANVISNING TDX MANUAL NORSK	1
PL	1531138	(User Manual TDX-SP Polish)	1
RU	SP1549025	(User Manual TDX-SP Russian)	1
SL	1531139	(User Manual TDX-SP Slovenian)	1
SV	1522687	(User Manual TDX-SP Swedish)	1

(User Manuals REM550)

languages.io

BG	Български	FR	Français	NL	Nederlands	SV	Svenska
CS	Čeština	HE	עברית	NO	Norsk	UK	Українська
DA	Dansk	HR	Hrvatski	PL	Polski		
DE	Deutsch	HU	Magyar	PT	Português		
EN	English	IT	Italiano	RU	Русский		
ES	Español	JA	日本語	RO	Română		
ET	Eesti	LV	Latviešu	SK	Slovenčina		
FI	Suomi	LT	Lietuvių	SL	Slovenščina		

DA	1528088	(User Manual REM550 Danish)	1
DE	1528089	(User Manual REM550 German)	1
EN	1528090	BRUKSANVISNING DX-REM550 MANUAL	1
ES	1528091	(User Manual REM550 Spanish)	1
FI	1528092	(User Manual REM550 Finsk)	1
FR	1528093	(User Manual REM550 French)	1
IT	1528094	(User Manual REM550 Italiano)	1
NL	1528095	(User Manual REM550 Dutch)	1
NO	1528096	BRUKSANVISNING DX REM550 MANUAL NORSK	1
PT	1528097	(User Manual REM550 Portuguese)	1
SV	1528098	(User Manual REM550 Swedish)	1
HR	SP1533279	(User Manual REM550 Croatian)	1
CS	SP1533280	(User Manual REM551 Czech)	1
ET	SP1533281	(User Manual REM552 Estonian)	1
HU	SP1533282	(User Manual REM553 Hungarian)	1
LT	SP1533283	(User Manual REM554 Lithuanian)	1
PL	SP1533284	(User Manual REM555 Polish)	1
SL	SP1533285	(User Manual REM556 Slovenian)	1

languages.io

BG	Български	FR	Français	NL	Nederlands	SV	Svenska
CS	Čeština	HE	עברית	NO	Norsk	UK	Українська
DA	Dansk	HR	Hrvatski	PL	Polski		
DE	Deutsch	HU	Magyar	PT	Português		
EN	English	IT	Italiano	RU	Русский		
ES	Español	JA	日本語	RO	Română		
ET	Eesti	LV	Latviešu	SK	Slovenčina		
FI	Suomi	LT	Lietuvių	SL	Slovenščina		

DE	1436142	(User Manual REM24SD German)	1
FR	1436143	(User Manual REM24SD French)	1
EN	1436144	(User Manual REM24SD English)	1
SV	1436145	(User Manual REM24SD Swedish)	1
DA	1436146	(User Manual REM24SD Danish)	1
NO	1436147	(User Manual REM24SD Norwegian)	1
NL	1436148	(User Manual REM24SD Dutch)	1
ES	1436149	(User Manual REM24SD Spanish)	1
PT	1436150	(User Manual REM24SD Portuguese)	1
IT	1436385	(User Manual REM24SD Italian)	1
FI	1436386	(User Manual REM24SD Finnish)	1
HR	SP1533251	(User Manual REM24SD Croatian)	1
CS	SP1533252	(User Manual REM24SD Czech)	1
ET	SP1533253	(User Manual REM24SD Estonian)	1
HU	SP1533254	(User Manual REM24SD Hungarian)	1
LT	SP1533255	(User Manual REM24SD Lithuanian)	1
PL	SP1533256	(User Manual REM24SD Polish)	1
SL	SP1533257	(User Manual REM24SD Slovenian)	1
LV	SP1543152	(User Manual REM24SD Latvian)	1

(User Manuals Shark2)

languages.io

BG	Български	FR	Français	NL	Nederlands	SV	Svenska
CS	Čeština	HE	עברית	NO	Norsk	UK	Українська
DA	Dansk	HR	Hrvatski	PL	Polski		
DE	Deutsch	HU	Magyar	PT	Português		
EN	English	IT	Italiano	RU	Русский		
ES	Español	JA	日本語	RO	Română		
ET	Eesti	LV	Latviešu	SK	Slovenčina		
FI	Suomi	LT	Lietuvių	SL	Slovenščina		

DE	1451346	(User Manual Shark II German)	1
FR	1451347	(User Manual Shark II French)	1
SV	1451348	(User Manual Shark II Swedish)	1
DA	1451349	(User Manual Shark II Danish)	1
NO	1451350	(User Manual Shark II Norwegian)	1
NL	1451351	(User Manual Shark II Dutch)	1
ES	1451352	(User Manual Shark II Spanish)	1
PT	1451353	(User Manual Shark II Portuguese)	1
IT	1451354	(User Manual Shark II Italian)	1
FI	1451355	(User Manual Shark II Finnish)	1
EN	1451356	(User Manual Shark II English)	1
SL	1451584	(User Manual Shark II Slovenian)	1
CS	SP1533297	(User Manual Shark2 CS)	1
ET	SP1533298	(User Manual Shark2 EE)	1
HU	SP1533299	(User Manual Shark2 HU)	1
LT	SP1533300	(User Manual Shark2 LT)	1
PL	SP1533301	(User Manual Shark2 PL)	1
BG	SP1546061	(User Manual Shark II Bulgarian)	1
RU	SP1570186	(User Manual Shark II Russian)	1
HR	SP1574028	(User Manual Shark II Croatian)	1
LV	SP1576075	(User Manual Shark II Latvian)	1

(User Manuals REM A & REM B)

languages.io

BG	Български	FR	Français	NL	Nederlands	SV	Svenska
CS	Čeština	HE	עברית	NO	Norsk	UK	Українська
DA	Dansk	HR	Hrvatski	PL	Polski		
DE	Deutsch	HU	Magyar	PT	Português		
EN	English	IT	Italiano	RU	Русский		
ES	Español	JA	日本語	RO	Română		
ET	Eesti	LV	Latviešu	SK	Slovenčina		
FI	Suomi	LT	Lietuvių	SL	Slovenščina		

DA	1529171	(User Manual REM A & B Dansk)	1
DE	1529172	(User Manual REM A & B German)	1
EN	1529173	(User Manual REM A & B English)	1
ES	1529174	(User Manual REM A & B Spanish)	1
FI	1529228	(User Manual REM A & B Finsk)	1
FR	1529229	(User Manual REM A & B French)	1
IT	1529230	(User Manual REM A & B Italian)	1
NL	1529231	(User Manual REM A & B Dutch)	1
NO	1529232	(User Manual REM A & B Norsk)	1
PT	1529233	(User Manual REM A & B Portuguese)	1
SV	1529234	(User Manual REM A & B Swedish)	1
HR	SP1533258	(User Manual REM A & B Croatian)	1
CS	SP1533259	(User Manual REM A & B Czech)	1
ET	SP1533265	(User Manual REM A & B Estonian)	1
HU	SP1533266	(User Manual REM A & B Hungarian)	1
LT	SP1533267	(User Manual REM A & B Lithuanian)	1
PL	SP1533268	(User Manual REM A & B Polish)	1
SL	SP1533269	(User Manual REM A & B Slovenian)	1
RO	SP1533270	(User Manual REM A & B Romanian)	1

BG	Български	FR	Français	NL	Nederlands	SV	Svenska
CS	Čeština	HE	עברית	NO	Norsk	UK	Українська
DA	Dansk	HR	Hrvatski	PL	Polski		
DE	Deutsch	HU	Magyar	PT	Português		
EN	English	IT	Italiano	RU	Русский		
ES	Español	JA	日本語	RO	Română		
ET	Eesti	LV	Latviešu	SK	Slovenčina		
FI	Suomi	LT	Lietuvių	SL	Slovenščina		

languages.io

BG	Български	FR	Français	NL	Nederlands	SV	Svenska
CS	Čeština	HE	עברית	NO	Norsk	UK	Українська
DA	Dansk	HR	Hrvatski	PL	Polski		
DE	Deutsch	HU	Magyar	PT	Português		
EN	English	IT	Italiano	RU	Русский		
ES	Español	JA	日本語	RO	Română		
ET	Eesti	LV	Latviešu	SK	Slovenčina		
FI	Suomi	LT	Lietuvių	SL	Slovenščina		

DE	1536441	(User Manual G91 German)	1
DA	1537929	(User Manual G91 Danish)	1
NL	1537930	(User Manual G91 Dutch)	1
EN	1537931	(User Manual G91 English)	1
FI	1537932	(User Manual G91 Finnish)	1
FR	1537933	(User Manual G91 French)	1
IT	1537934	(User Manual G91 Italian)	1
NO	1537935	(User Manual G91 Norwegian)	1
PT	1537936	(User Manual G91 Portuguese)	1
ES	1537937	(User Manual G91 Spanish)	1
SV	1537938	(User Manual G91 Swedish)	1

(User Manuals IDC Dual Control)

languages.io

BG	Български	FR	Français	NL	Nederlands	SV	Svenska
CS	Čeština	HE	עברית	NO	Norsk	UK	Українська
DA	Dansk	HR	Hrvatski	PL	Polski		
DE	Deutsch	HU	Magyar	PT	Português		
EN	English	IT	Italiano	RU	Русский		
ES	Español	JA	日本語	RO	Română		
ET	Eesti	LV	Latviešu	SK	Slovenčina		
FI	Suomi	LT	Lietuvių	SL	Slovenščina		

DE	1521480	(User Manual Dual-Control German)	1
EN	1521481	(User Manual Dual-Control English)	1
FR	1521482	(User Manual Dual-Control French)	1
NL	1521483	(User Manual Dual-Control Dutch)	1
NO	1521484	BRUKSANVISNING INTUITIV LEDSAGERSTYRING	1
SV	1521485	(User Manual Dual-Control Swedish)	1
DA	1521486	(User Manual Dual-Control Danish)	1
IT	1522252	(User Manual Dual-Control Italian)	1
PT	1522254	(User Manual Dual-Control Portuguese)	1
ES	1522255	(User Manual Dual-Control Spanish)	1

(User Manuals REM34 & REM41 Tray/Centre Control)

languages.io

BG	Български	FR	Français	NL	Nederlands	SV	Svenska
CS	Čeština	HE	עברית	NO	Norsk	UK	Українська
DA	Dansk	HR	Hrvatski	PL	Polski		
DE	Deutsch	HU	Magyar	PT	Português		
EN	English	IT	Italiano	RU	Русский		
ES	Español	JA	日本語	RO	Română		
ET	Eesti	LV	Latviešu	SK	Slovenčina		
FI	Suomi	LT	Lietuvių	SL	Slovenščina		

DA	SP1538022	(User Manual REM41 Tray Control Danish)	1
EN	SP1538023	(User Manual REM41 Tray Control English)	1
ES	SP1538024	(User Manual REM41 Tray Control Spanish)	1
FR	SP1538025	(User Manual REM41 Tray Control French)	1
IT	SP1538026	(User Manual REM41 Tray Control Italian)	1
NL	SP1538027	(User Manual REM41 Tray Control Dutch)	1
NO	SP1538028	(User Manual REM41 Tray Control Norwegian)	1
PT	SP1538029	(User Manual REM41 Tray Control Portuguese)	1
SV	SP1538030	(User Manual REM41 Tray Control Swedish)	1
CS	SP1538031	(User Manual REM41 Tray Control Czech)	1
SL	SP1538032	(User Manual REM41 Tray Control Slovenian)	1
DE	SP1541402	(User Manual REM41 Tray Control German)	1

(User Manuals ACS Handprogrammer)

languages.io

BG	Български	FR	Français	NL	Nederlands	SV	Svenska
CS	Čeština	HE	עברית	NO	Norsk	UK	Українська
DA	Dansk	HR	Hrvatski	PL	Polski		
DE	Deutsch	HU	Magyar	PT	Português		
EN	English	IT	Italiano	RU	Русский		
ES	Español	JA	日本語	RO	Română		
ET	Eesti	LV	Latviešu	SK	Slovenčina		
FI	Suomi	LT	Lietuvių	SL	Slovenščina		

DE	1419316	(User Manual Handheld Programmer German)	1
EN	1419317	(User Manual Handheld Programmer English)	1

(User Manual Proximity Head Remote DSL-105)

languages.io

BG	Български	FR	Français	NL	Nederlands	SV	Svenska
CS	Čeština	HE	עברית	NO	Norsk	UK	Українська
DA	Dansk	HR	Hrvatski	PL	Polski		
DE	Deutsch	HU	Magyar	PT	Português		
EN	English	IT	Italiano	RU	Русский		
ES	Español	JA	日本語	RO	Română		
ET	Eesti	LV	Latviešu	SK	Slovenčina		
FI	Suomi	LT	Lietuvių	SL	Slovenščina		

EN	SP1555731	(User Manual Proximity Head Remote DSL-105 English)	1
DE	SP1555732	(User Manual Proximity Head Remote DSL-105 German)	1
FR	SP1555733	(User Manual Proximity Head Remote DSL-105 French)	1
DA	SP1555839	(User Manual Proximity Head Remote DSL-105 Danish)	1
ES	SP1555840	(User Manual Proximity Head Remote DSL-105 Spanish)	1
FI	SP1555841	(User Manual Proximity Head Remote DSL-105 Finnish)	1
IT	SP1555842	(User Manual Proximity Head Remote DSL-105 Italian)	1
NL	SP1555843	(User Manual Proximity Head Remote DSL-105 Dutch)	1
NO	SP1555844	(User Manual Proximity Head Remote DSL-105 Norwegian)	1
PT	SP1555845	(User Manual Proximity Head Remote DSL-105 Portuguese)	1
SV	SP1555846	(User Manual Proximity Head Remote DSL-105 Swedish)	1


(User Manuals Ultra Low Maxx Seat)

languages.io

BG	Български	FR	Français	NL	Nederlands	SV	Svenska
CS	Čeština	HE	עברית	NO	Norsk	UK	Українська
DA	Dansk	HR	Hrvatski	PL	Polski		
DE	Deutsch	HU	Magyar	PT	Português		
EN	English	IT	Italiano	RU	Русский		
ES	Español	JA	日本語	RO	Română		
ET	Eesti	LV	Latviešu	SK	Slovenčina		
FI	Suomi	LT	Lietuvių	SL	Slovenščina		


DA	SP1585726	(User Manual Ultra Low Maxx Seat - Danish)	1
DE	SP1585727	(User Manual Ultra Low Maxx Seat - German)	1
EN	SP1585725	(User Manual Ultra Low Maxx Seat - English)	1
FI	SP1585728	(User Manual Ultra Low Maxx Seat - Finish)	1
FR	SP1585729	(User Manual Ultra Low Maxx Seat - French)	1
IT	SP1588266	(User Manual Ultra Low Maxx Seat - Italian)	1
NL	SP1585730	(User Manual Ultra Low Maxx Seat - Dutch)	1
NO	SP1585731	(User Manual Ultra Low Maxx Seat - Norwegian)	1
PT	SP1585732	(User Manual Ultra Low Maxx Seat - Portuguese)	1
SV	SP1585733	(User Manual Ultra Low Maxx Seat - Swedish)	1

114858.dwg

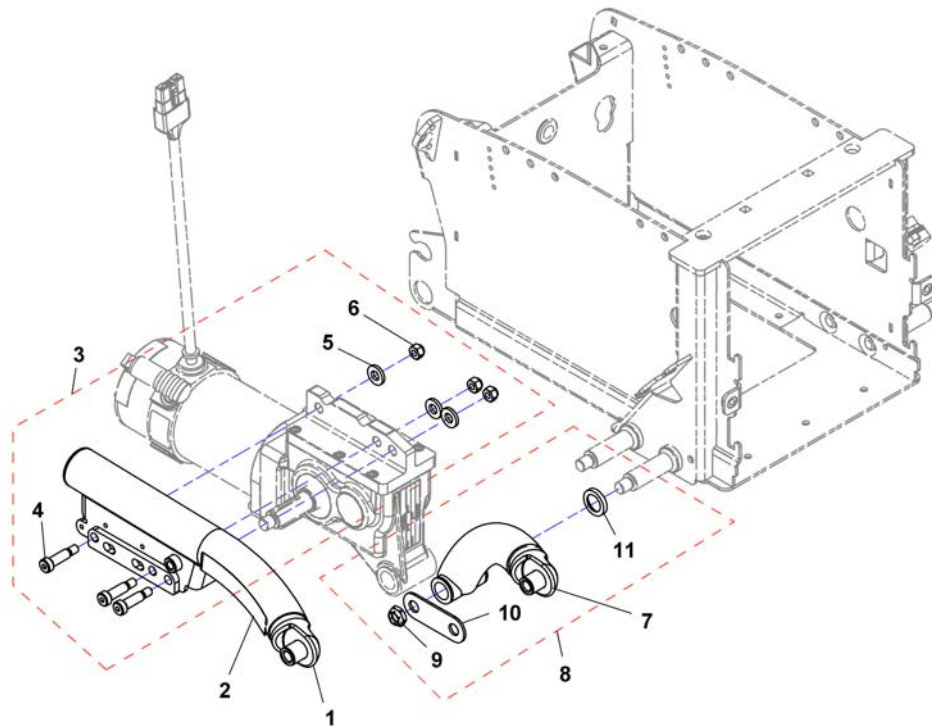


1	SP1523256	(Walking Beam Assembly (electric red) with TDX SP Decal, LH)	1
1	SP1523255	(Walking Beam Assembly (electric red) with TDX SP Decal, RH)	1
1	SP1523258	(Walking Beam Assembly (wet black) with TDX SP Decal, LH)	1
1	SP1523257	(Walking Beam Assembly (wet black) with TDX SP Decal, RH)	1
2	SP1523167	(Decal, TDX SP (Yellow))	1
2	SP1523012	(Decal, TDX SP (Black))	1
3	SP1523178	(Kit, Walking Beam Mounting Hardware)	1
7	SP1523199	(Link, Lower Pivot (wet black)-Left)	1
7	SP1523198	(Link, Lower Pivot (wet black)-Right)	1
7	SP1563163	(Link, Lower Pivot (electric red)-Left)	1
7	SP1563169	(Link, Lower Pivot (electric red)-Right)	1
8	SP1523168	(Kit, Lower Pivot Link / 4-Pole Motor Mounting Hardware)	1
4	SP1522942	(Screw, Socket Head Shoulder (3/8 x1"))	1
5	SP1522966	(Washer (7/16 x13/16 x1/8"))	1
10	-	(No longer available.)	0

(Walking Beam SSD-Motors)

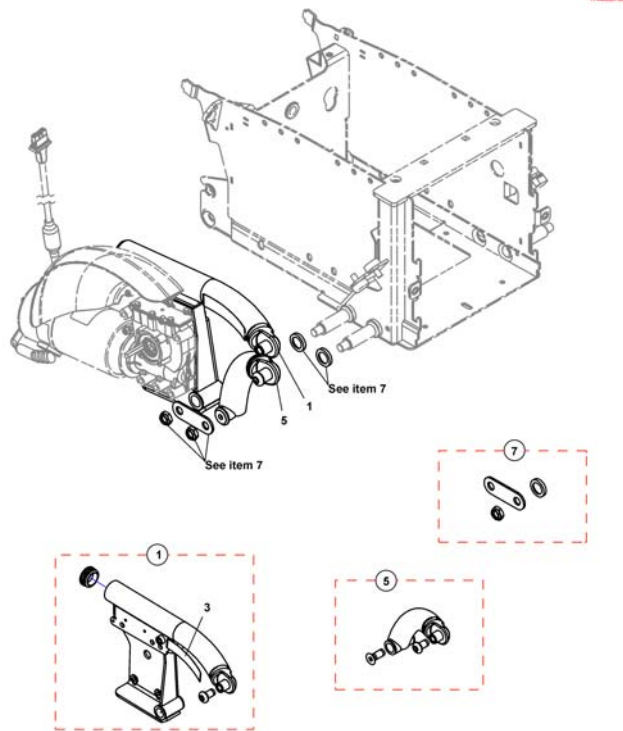


1	SP1551713	(Walking Beam Assembly (wet black) with TDX-SP Decal-Right)	1
1	SP1557069	(Walking Beam Assembly (electric red) with TDX-SP Decal-Right)	1
1	SP1551714	(Walking Beam Assembly (wet black) with TDX-SP Decal-Left)	1
1	SP1557070	(Walking Beam Assembly (electric red) with TDX-SP Decal-Left)	1
3	SP1523167	(Decal, TDX SP (Yellow))	1
5	SP1523198	(Link, Lower Pivot (wet black)-Right)	1
5	SP1563169	(Link, Lower Pivot (electric red)-Right)	1
5	SP1523199	(Link, Lower Pivot (wet black)-Left)	1
5	SP1563163	(Link, Lower Pivot (electric red)-Left)	1
7	SP1551715	(Kit, Lower Pivot Link/4-Pole Motor Mounting Hardware)	1

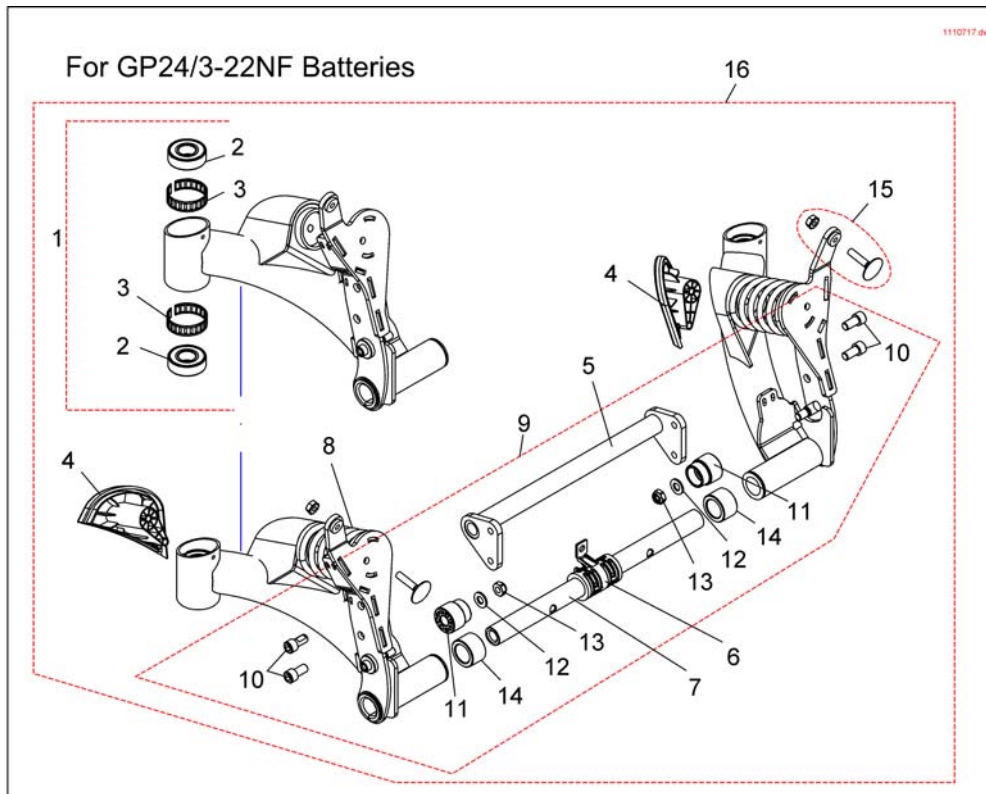

1	SP1523260	SVINGARM BAK KPL VE TDX	1
1	SP1523259	SVINGARM BAK HØ KPL TDX	1
1	SP1564785	(Kit, Swing Arm Assembly (wet black) with Bearings, Rear-Left)	1
1	SP1564781	(Kit, Swing Arm Assembly (wet black) with Bearings, Rear-Right)	1
2	SP40117X006	(Bearing (5/8"))	1
3	SP1110717	(Ring, Tolerance)	1
4	SP1523089	(Cap, Rear-Left)	1
4	SP1523088	(Cap, Rear-Right)	1
5	SP1523083	(Cross-Member, Pivot (TDX-SP Narrow Base))	1
5	SP1142287	()	1
6	SP1523166	FESTE DEKSEL TDX TDX	1
7	SP1523194	(Bar, Pivot)	1
7	SP1523197	(Bar, Pivot)	1
7	SP1155018	()	1
8	SP1523002	FJÆR HEAVYDUTY TDX <i>replaced by SP1579643 (silver)</i>	1
8	SP1579643	(Spring - silver) <i>replaces SP1523002 (yellow)</i>	1
9	SP1523177	FESTESETT ROTASJONSAKSEL TDX <i>included parts 10-14</i>	1
13	SP1522937	(Locknut (5/16-18))	1
15	SP1523144	BOLT FESTE FJÆRINGENHET TDX	1
16	SP1523261	(Rear Pivot Assembly (electric red) (ForGP24/3-22NF Battery Box) (wide base))	1
16	SP1564779	(Rear Pivot Assembly (wet black) (ForGP24/3-22NF Battery Box) (wide base))	1
16	SP1551708	(Rear Pivot Assembly (electric red) (For22NF Battery Box) (Narrow base))	1
16	SP1564780	(Rear Pivot Assembly (wet black) (For 22NF Battery Box) (Narrow base))	1

(Rear Pivot Assembly (Mounting))


1149285 Rev

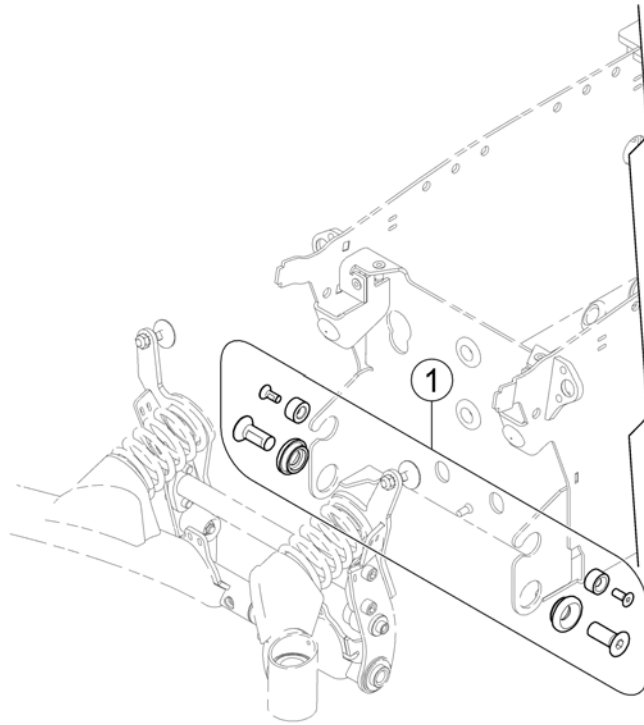

1 SP1523193



FESTE HODESTØTTE BAKRE TDX

1

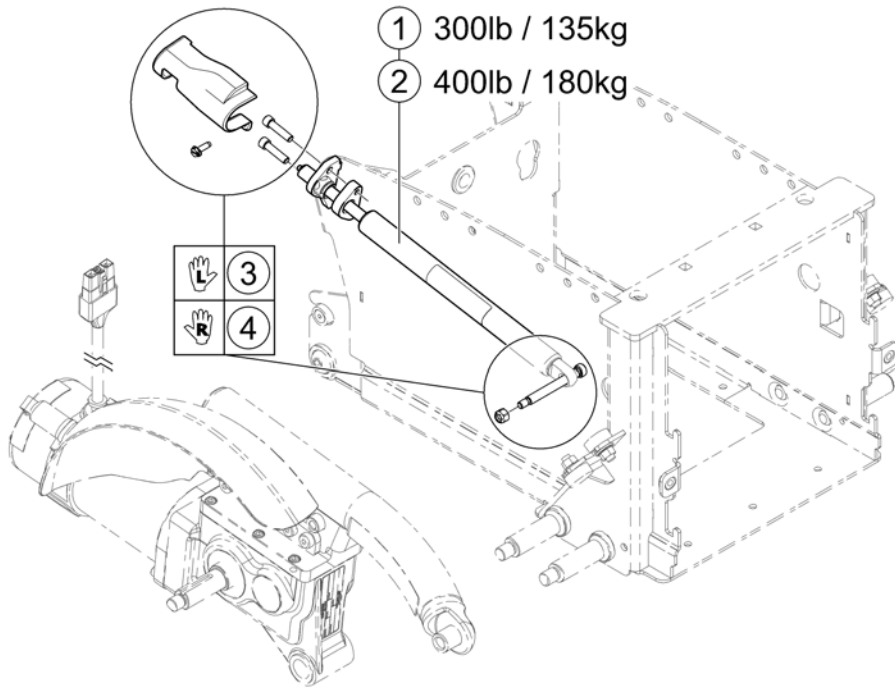


1	SP1523162	(Cylinder, Locking Gas (300 Lb. User Weight Limit))	1
2	SP1523158	(Cylinder, Locking Gas (400 Lb. User Weight Limit))	1
3	SP1523149	(Kit, Locking Gas Cylinder Mounting Hardware -Left)	1
4	SP1523148	(Kit, Locking Gas Cylinder Mounting Hardware -Right)	1

(Fender)

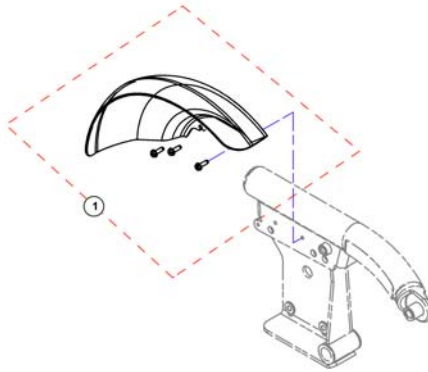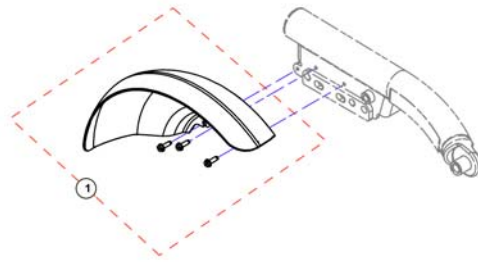


1	SP1523170	SKJERM KPL HØ TDX TDX	1
1	SP1523171	SKJERM KPL VE TDX TDX	1

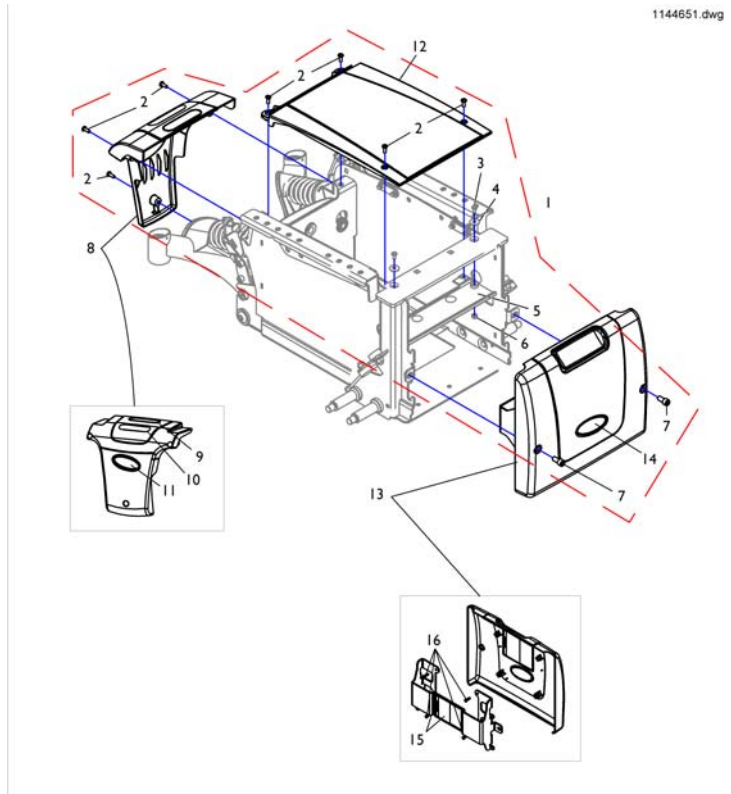

1	SP1523187	DEKSEL BATTERIER TDX	1
1	SP1144651	(Kit, Shroud Hardware for 22NF Batteries (narrow base))	1
6	SP1522936	(Locknut (10-32))	1
7	SP1536212	(knurled screw 5/16")	1
8	SP1523183	DEKSEL BAK KPL TDX TDX	1
8	SP1523155	DEKSEL BRUK SP1523183 TDX	1
12	SP1523182	DEKSEL SENTER BATTERI TDX	1
12	SP1523153	(Shroud, Center (TRRO/TRBKTS) (narrow base))	1
13	SP1523189	FRONTDEKSEL TDX TDX	1
13	SP1144642	(Front Shroud and Bracket Assembly, for 2-22NF Batteries (narrow base))	1
15	SP1523190	(Bracket w/Decal, Front Shroud (ForGP24 Batteries))	1
16	SP1523003	(Screw, Phillips Head (8-16 x5/8"))	1


(Battery Box Hardware)

1140071.00

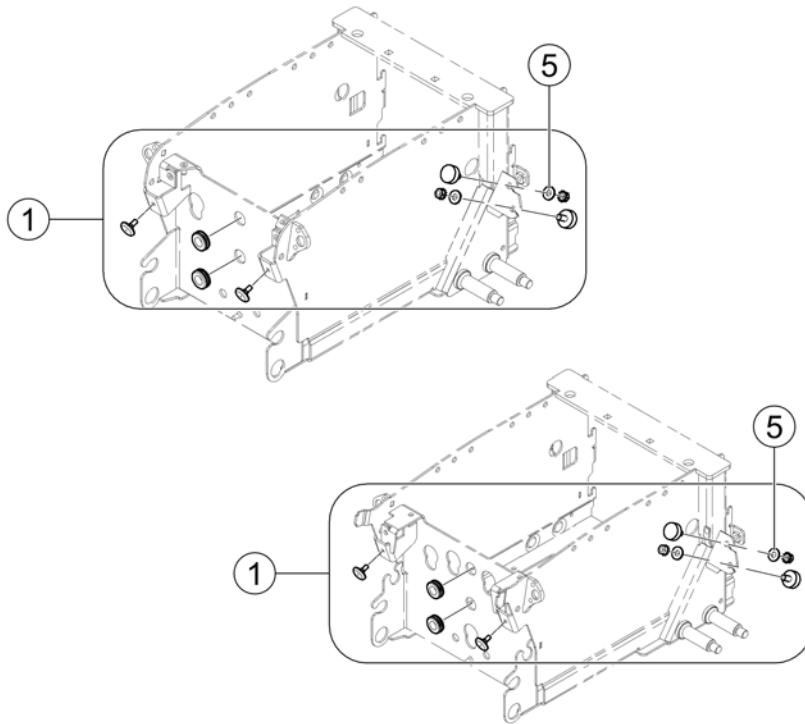

1	SP1523179	BATTERIBOKS TDX EXTRA GUMMIPAD OG FORINGER	1
5	SP1522968	(Washer (5/16 x3/4 x1/16"))	1

SPCTDX_SP20.00

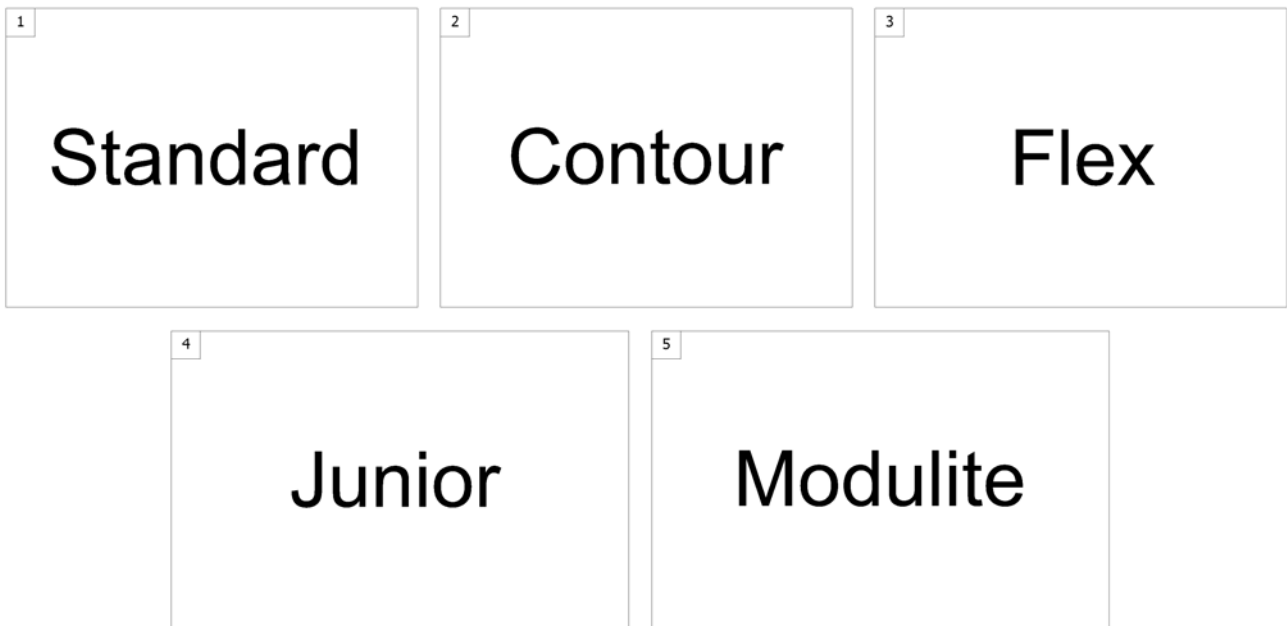


1	-	(Standard Seat)	1
2	-	(Contour Seat)	1
3	-	(Flex Seat)	1
4	-	(Junior Seat)	1
5	-	(Modulite Seat)	1

-	(Powered Tilt 50° & manuell)	1
-	(Tiltmodule 28°)	1
-	(Lifter (incl. 28°-Tilt))	1
-	(Bellow)	1
-	(Seat frames for Balle modules)	1
-	(Adapter sheets)	1
-	(Seatplates)	1
-	(Seat Cushion for Standard Seat)	1

(Powered Tilt 50° & manuell)


1513344X.iso

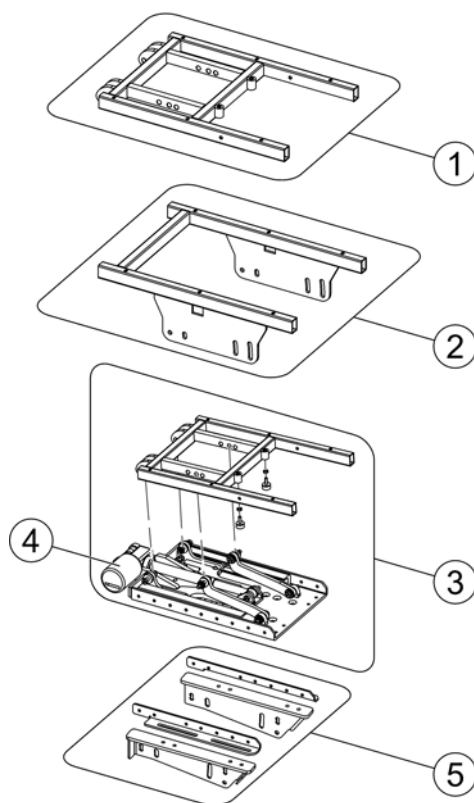


-	SP1535829	(Mercury Switch for tilt up to 50°)	1
1	SP1513344	SETERAMME TILT 50° TDX	1
2	SP1513657	SETERAMME MAN TILT TDX	1
2	SP1548900	(Seatframe for Manual Tilt (254mm))	1
3	SP1528314	TILT MED SETERAMME TDX	1
4	SP1143801	(Tilt-Actuator)	1
5	SP1580051	(Angle Bracket for CG-Tilt (pair))	1

Incl. screws / avec vis / inkl. Schrauben

Invacare® TDX-SP | Sete
(Tiltmodule 28°)

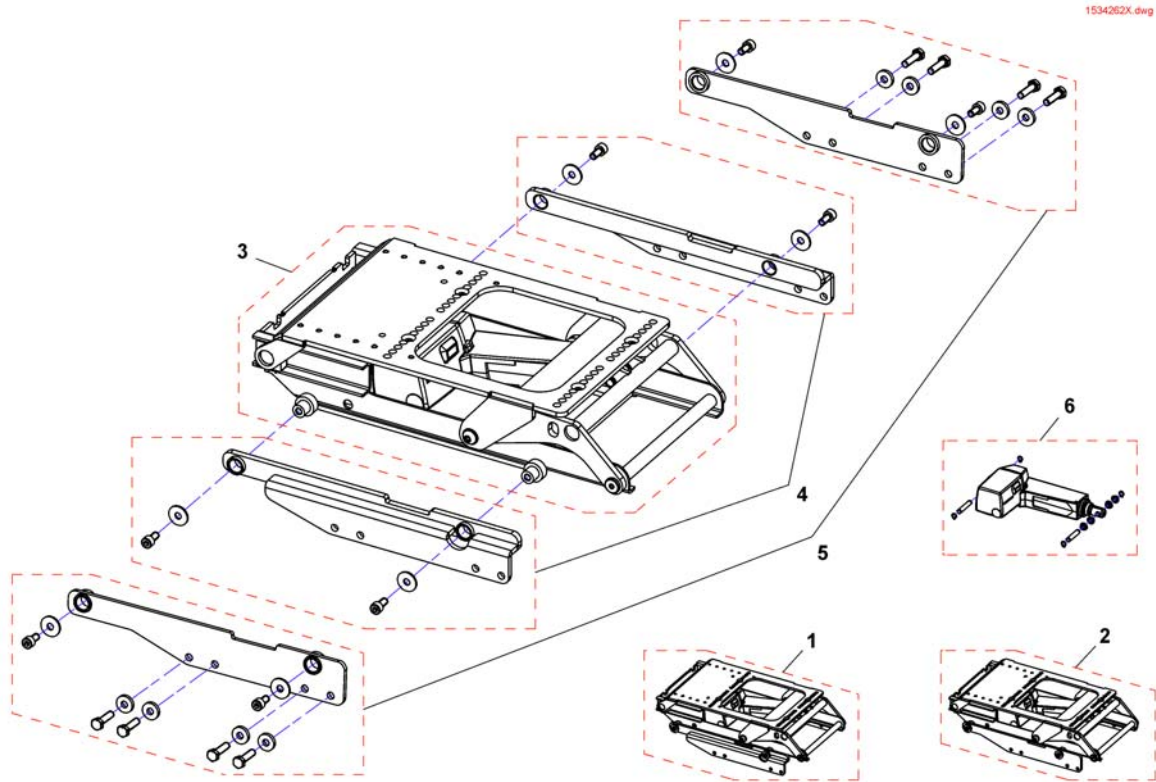


1	SP1534262	SETETILT 0-30° KPL TDX	1
2	SP1548546	(Seattilt 0-28°, cpl., Narrowbase)	1
3	SP1533063	TILTENHET 0-30° TDX	1
4	SP1536176	STYRESKINNER TILTMOTOR TDX	1
5	SP1551035	(Guiding rails, cpl., Narrowbase)	1
6	SP1544888	(Actuator)	1

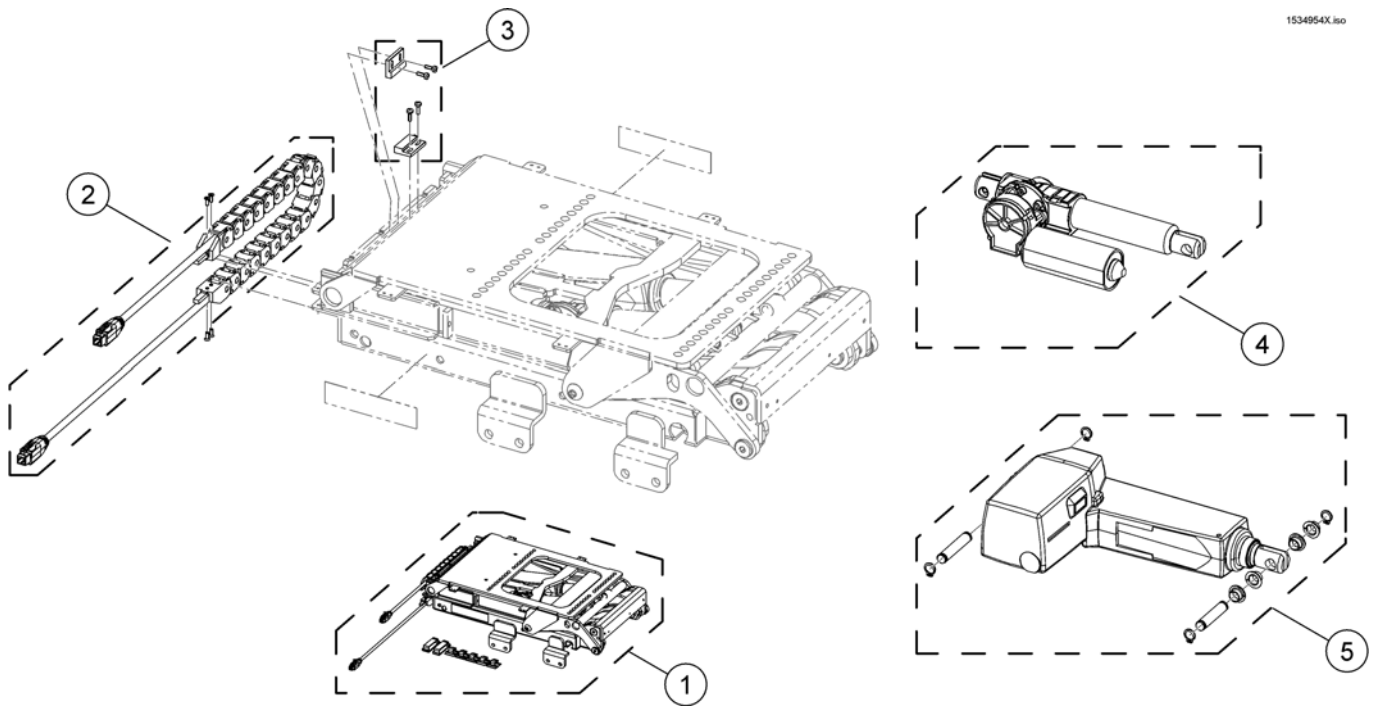

-	SP1533562	(Actuator cable ACS2 (~1570mm))	1
-	SP1571139	(Tilt actuator cable ACS2 (~920mm))	1
1	SP1534954	SETELØFT M TILT IN SPACE TDX	1
2	SP1535848	KABELGUIDE MED KABLE TDX	1
3	SP1536184	SENSOR FARTSREDUKSJON TDX	1
4	SP1533557	MOTOR TILT BORA	1
5	SP1533616	SETELØFT MOTOR TDX	1

1545673X.dwg

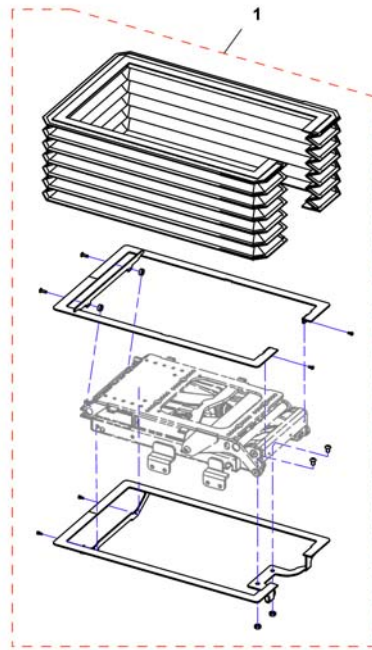


SP1545673

BELG FOR LØFT TDX

1

(Seat frames for Balle modules)

1552400X.dwg

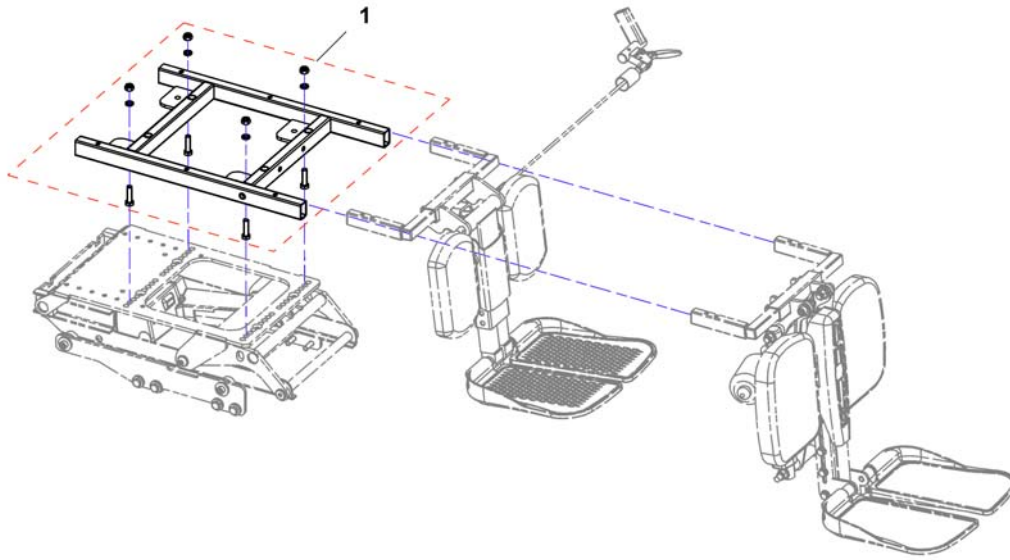




1 SP1552400

(Seatframes for Balle modules)

1



Invacare® TDX-SP | Sete
(Adapter sheets)

1534951X.dwg

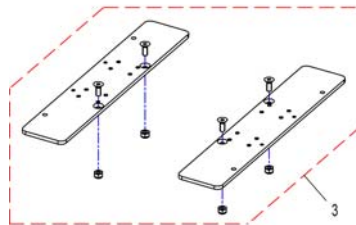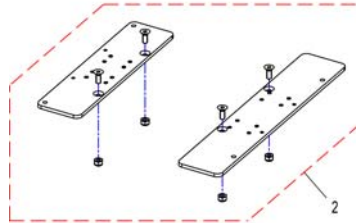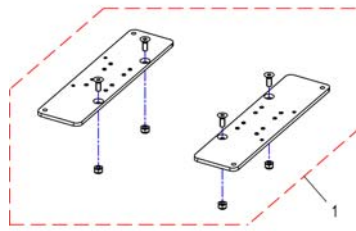

1	SP1534951	ADAPTERPLATER TILT SB39 TDX	1
2	SP1534952	ADAPTERPLATER TILT SB44 TDX	1
3	SP1534953	ADAPTERPLATER TILT SB49 TDX	1

1420459X.dwg

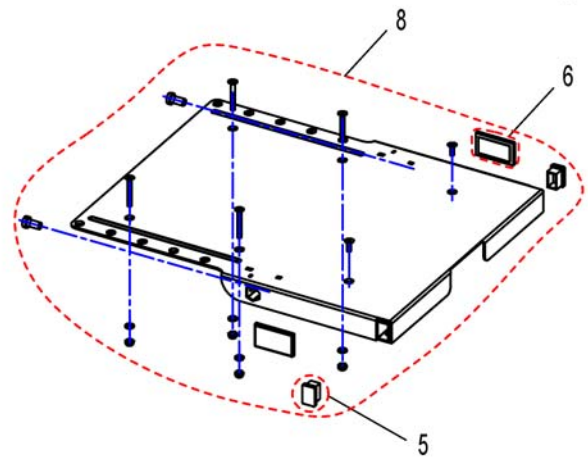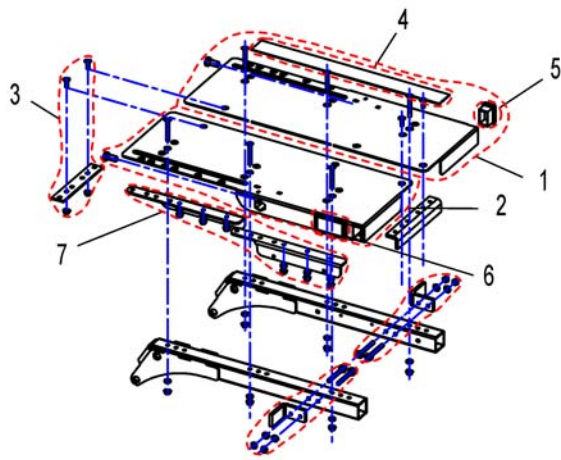

Kontour



Flex




Standard



1	SP1420458	SETEPLATE VE TDX	1
2	SP1420457	SETEPLATE ERS TDX HØ TDX	1
3	SP1420496	(Joining Plate)	1
4	SPU21087	(Velcro)	1
5	SPL01060	(Darning)	1
6	SPP31298	(Reflector, yellow)	1
7	1510637	(Stiffing Kit)	1
8	SP1440623	MONTERINGSKIT SETEPLATE TDX	1

(Seat Cushion for Standard Seat)

1571799.no

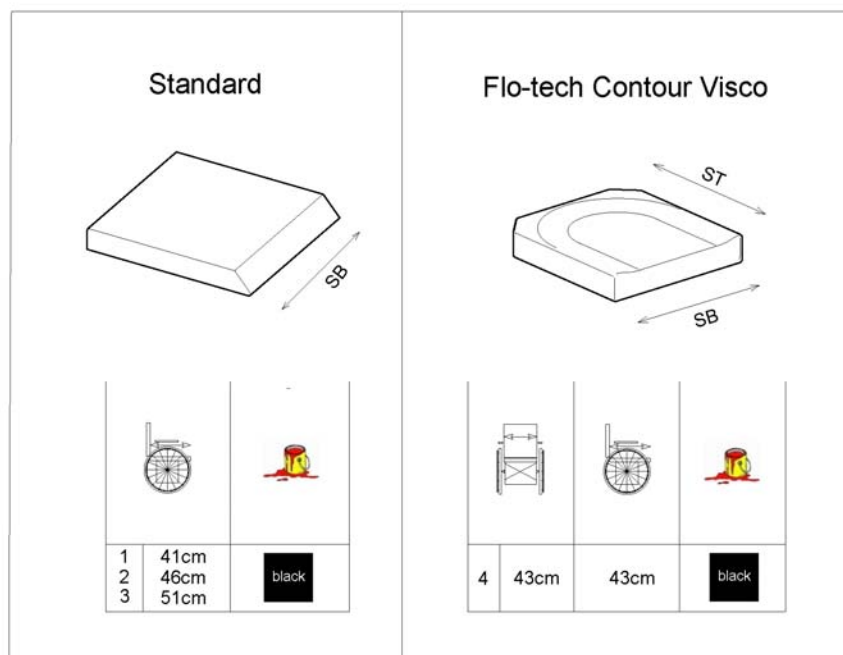

1	1420460	SETEPUTE SD41 STD DRAGON VERTIC <i>[1336+0326]</i>	1
2	1420461	SETEPUTE SD46 STD DRAGON VERTIC <i>[1336+0327]</i>	1
3	1420462	SETEPUTE SD51 STD DRAGON VERTIC <i>[1336+0330]</i>	1
4	1501113	(Seat Cushion Flo-tech Contour Visco SW 43cm x SD 43cm) <i>[1901]</i>	1

-	(Powered Tilt 50° & manuell)	1
-	(Tiltmodule 28°)	1
-	(Lifter (incl. 28°-Tilt))	1
-	(Bellow)	1
-	(Seat frames for Balle modules)	1
-	(Adapter sheets)	1
-	(Seatplates)	1
-	(Seat Cushion for Contour Seat)	1


(Seat Cushion for Contour Seat)

1573660.no

Contour



Flo-tech Contour Visco

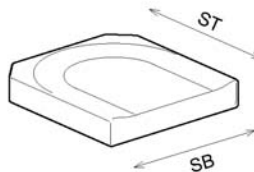

1	41cm	black
2	46cm	

3	43cm	43cm	black
4	45,5cm	53cm	
5	48cm	40,5cm	
6	48cm	43cm	


1	1422072	SETEPUTE 43 CONTUR SORT <i>[1337]</i>	1
2	1422074	SETEPUTE 48 SVART CONTUR <i>[1335]</i>	1
3	SP1501113	SETEPUTE 43*43 TDX FLOTEC CONTUR VISCO <i>[1901]</i>	1
4	SP1500993	SETEPUTE 45*53 TDX <i>[1902]</i>	1
5	SP1501116	SETEPUTE 48*40 TDX <i>[1903]</i>	1
6	SP1501117	SETEPUTE 48*43 TDX <i>[1904]</i>	1

-	(Powered Tilt 50° & manuell)	1
-	(Tiltmodule 28°)	1
-	(Lifter (incl. 28°-Tilt))	1
-	(Bellow)	1
-	(Seat frames for Balle modules)	1
-	(Adapter sheets)	1
-	(Seatplates)	1
-	(Seat Cushion for Flex Seat)	1


(Seat Cushion for Flex Seat)

1573688.rss


Flex



VIC AIR



Flo-tech Contour Visco

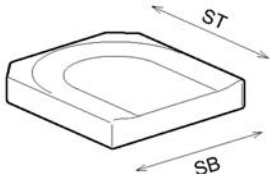


Item No.	Width	Color	Notes
1	39cm	Bris blue	
2	43cm	Bris blue	
3	48cm	Bris blue	==> 31.03.13
4	39cm	black	
5	43cm	black	
6	48cm	black	==> 31.03.13
7	39cm	Dartex grey	
8	43cm	Dartex grey	
9	48cm	Dartex grey	
10	39cm	black Plush	
11	43cm	black Plush	
12	48cm	black Plush	01.04.13 ==>


1	1422529	SETEPUTE 39 FLEX BLÅ	→ 31.03.2013	1
2	1422530	SETEPUTE 43 FLEX BLÅ	→ 31.03.2013	1
3	1422532	SETEPUTE 48 FLEX BLÅ	→ 31.03.2013	1
4	1436561	SETEPUTE 39 FLEX SORT	→ 31.03.2013	1
5	-	(No longer available - please choose Black Plush) <i>replaced by SP1436562-TR35</i>	→ 31.03.2013	0
6	-	(No longer available - please choose Black Plush) <i>replaced by SP1436563-TR35</i>	→ 31.03.2013	0
7	5252582-TR23	SETEPUTE 36 TROMB DARTEX		1
8	5252583-TR23	SETEPUTE 41 TROMB DARTEX		1
9	5252584-TR23	SETEPUTE 46 TROMB DARTEX		1
10	SP1436561-TR35	(Seat cushion SW39cm, black Plush)	01.04.2013 →	1
11	SP1436562-TR35	SETEPUTE SB43 SORT FLEX STORM4 TDX BORA	01.04.2013 →	1
12	SP1436563-TR35	SETEPUTE FLEX SB48 TDX BORA	01.04.2013 →	1
13	1422531	(Cushion Flex SW43cm, VIC AIR)		1
14	SP1501113	SETEPUTE 43*43 TDX FLOTEC CONTUR VISCO		1
15	SP1500993	SETEPUTE 45*53 TDX		1
16	SP1501116	SETEPUTE 48*40 TDX		1
17	SP1501117	SETEPUTE 48*43 TDX		1

-	(Powered Tilt 50° & manuell)	1
-	(Tiltmodule 28°)	1
-	(Lifter (incl. 28°-Tilt))	1
-	(Bellow)	1
-	(Seat frames for Balle modules)	1
-	(Adapter sheets)	1
-	(Seatplates)	1
-	(Seat Cushion for Junior Seat)	1


(Seat Cushion for Junior Seat)

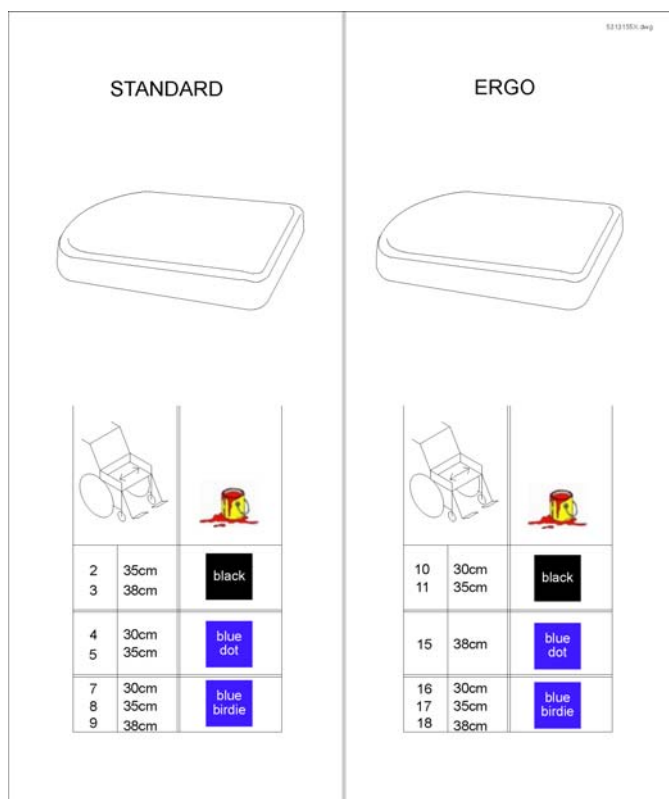

2	5313181	SETEPUTE SB35CM TDX	1
3	5313207	SETEPUTE SB38CM TDX	1
4	5313152	SETEPUTE 30 MOO85 KID BLUE BIRDIE	1
5	5313178	SETEPUTE 35 STD MOO85 KID BLUE BIRDIE	1
7	5313154	SETEPUTE 30 MOO85 KID BLÅ	1
8	5313180	SETEPUTE 35 STD MOO85 KID BLÅ	1
9	5313206	SETEPUTE 38 MOO85 KID BLÅ	1
10	5313159	(Seat Cushion Ergo SW30cm, Vinyl Black)	1
11	5313185	(Seat Cushion Ergo SW35cm, Vinyl Black)	1
15	5313208	SETEPUTE 38 ERGONOMISK MOO85 KID BLUE BIRDIE	1
16	5313158	SETEPUTE 30 ERGONOMISK MOO85 KID BLÅ	1
17	5313184	SETEPUTE 35 ERGONOMISK MOO85 KID BLÅ	1
18	5313210	SETEPUTE 38 ERGONOMISK MOO85 KID BLÅ	1

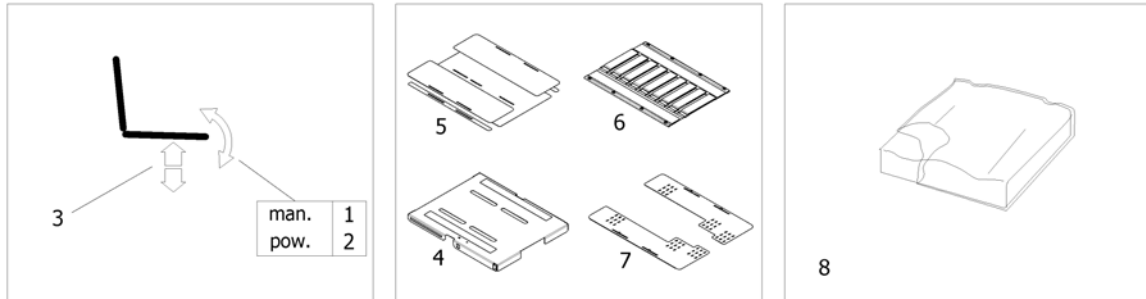


1	-	(man. settable Tilt) [1091]	1
2	-	(powered Tilt 0°-30°) [1093]	1
3	-	(Lifter) [1094]	1
4	-	(One Piece Seat Plate) [2110]	1
5	-	(Telescopic Seat Frame and Plate) [2111]	1
6	-	(Telescopic Seat Frame and Sling Seat) [2112]	1
7	-	(Telescopic Seat Frame and Universal Adapter) [2109]	1
8	-	(Seat Cushion)	1

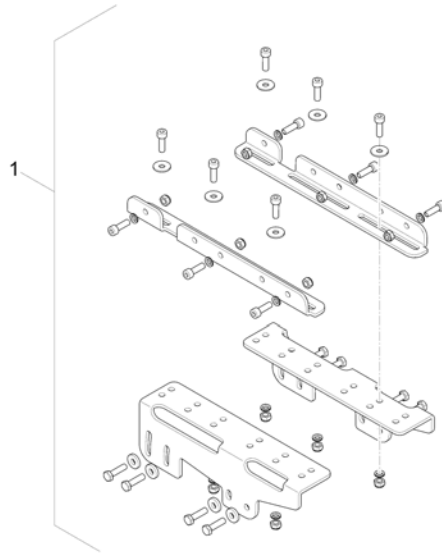

1 SP1563229

(Mounting Rails)


1

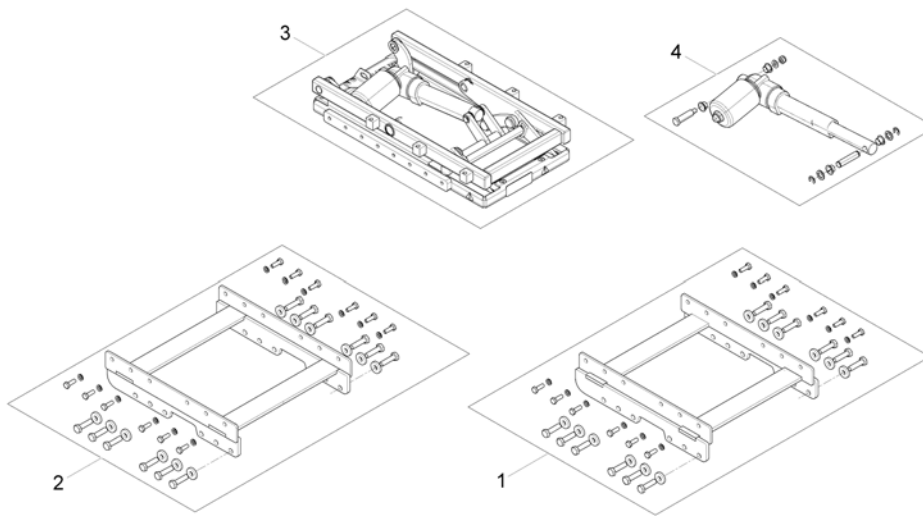

1	SP1568426	(Lifter- /Tilt Bracket for wide Base)	1
2	SP1568424	(Lifter- /Tilt Bracket for narrow Base)	1
3	SP1558714	(Tilt unit)	1
4	SP1571809	(Actuator)	1
-	SP1570291	(Adapter Cable)	1


(Lifter)

1577303 iso

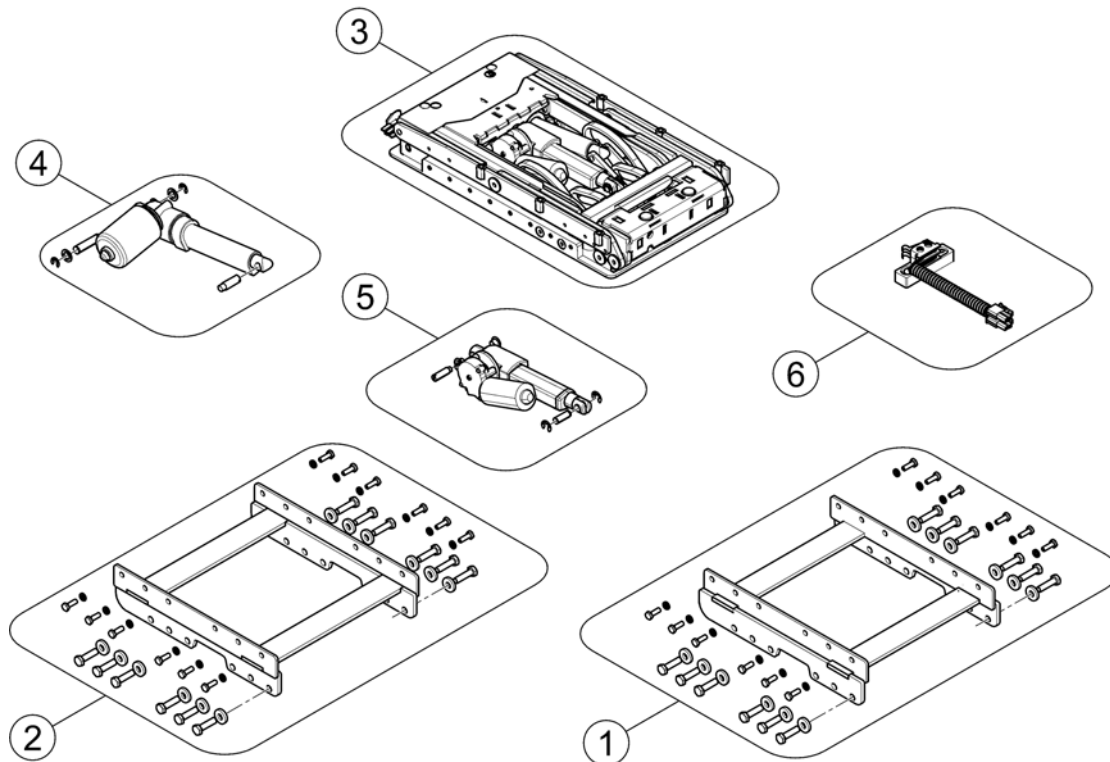

-	SP1570291	(Adapter Cable) <i>(2x for wide Base / 1x for Narrow Base)</i>	1
-	SP1576062	(Extension Cable) <i>(only for narrow Base)</i>	1
-	SP1570294	(Extension Cable)	1
-	SP1533561	(Light Cable w. DCI)	1
1	SP1568426	(Lifter- /Tilt Bracket for wide Base)	1
2	SP1568424	(Lifter- /Tilt Bracket for narrow Base)	1
3	SP1558715	SETELØFT MED TILT KPL TDX SP2 LOW-RIDER	1
4	SP1571823	(Lifter actuator)	1
5	SP1571822	(Tilt actuator)	1
6	SP1577762	(Slow-Stop Switchcable with Mounting)	1

1571064.000

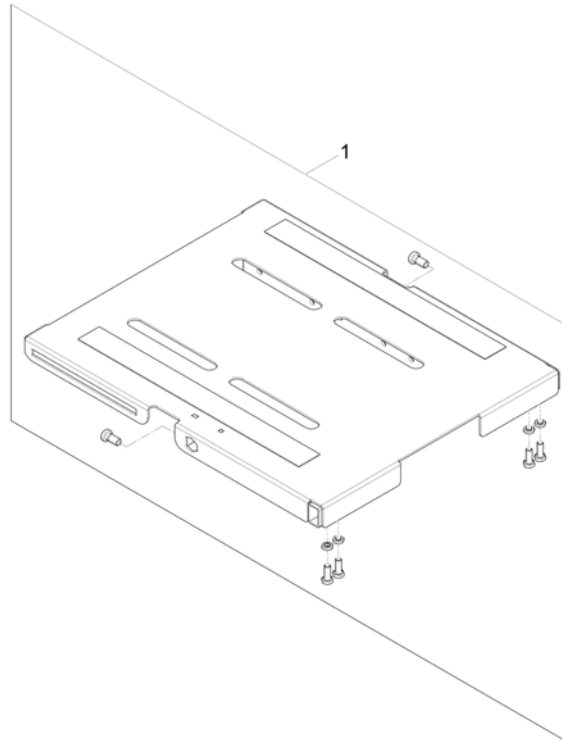


1 SP1571297

(One Piece Seat Plate)

1

(Telescopic Seat Frame and Plate)


1572312 iso

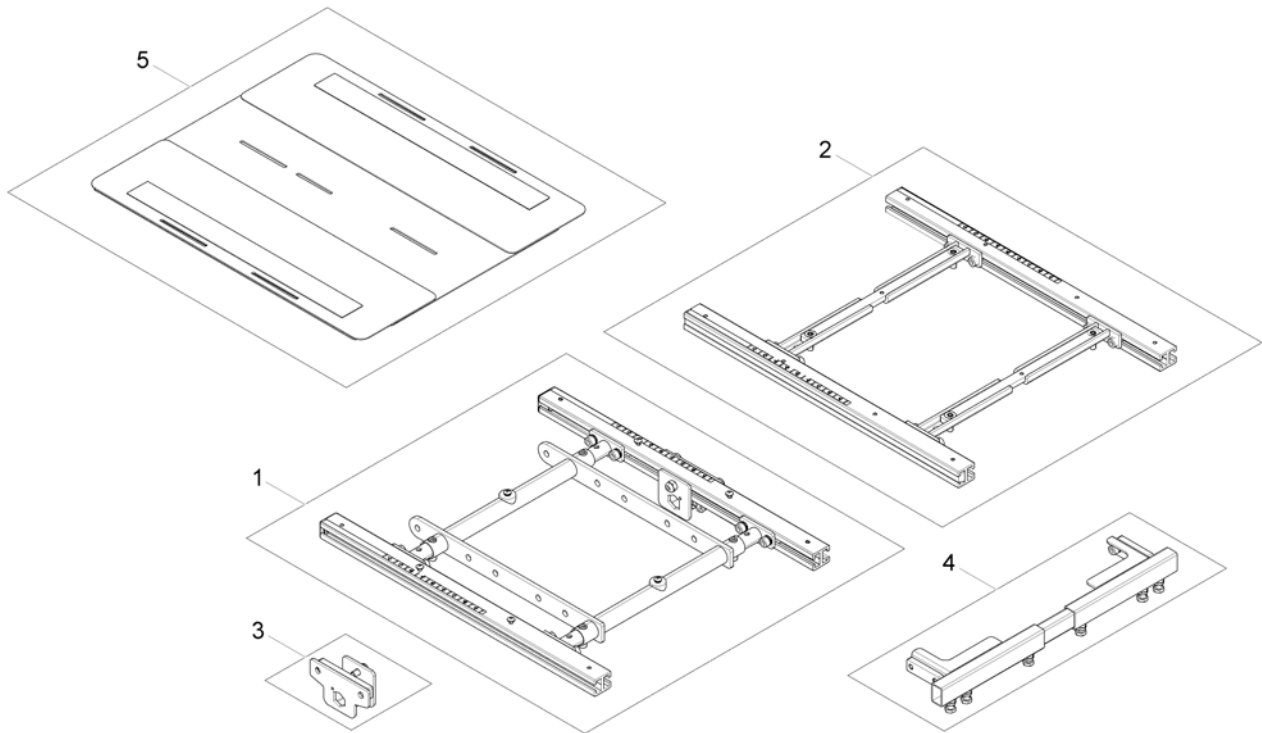

1	-	(Telescopic Seat Frame for Tilt 12° / 20°)	1
2	-	(Telescopic Seat Frame for Tilt 30° / Lifter)	1
3	-	(Armrest bracket)	1
4	-	(Legrest Support)	1
5	-	(Seat Plate)	1

(Telescopic Seat Frame for Tilt 12° / 20°)


1572009 iso

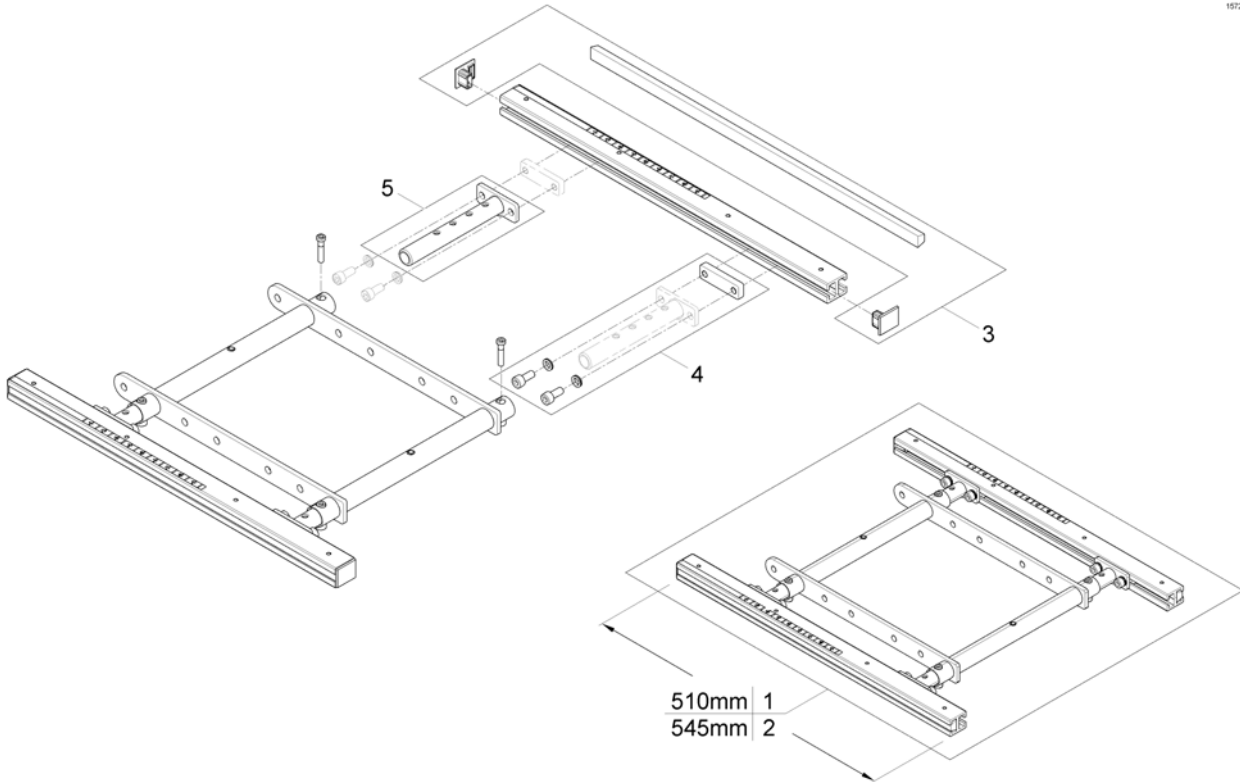

-	SP1588443	(Seat Rails 510mm (2pcs kit))	1
-	SP1588444	(Seat Rails 545mm (2pcs kit))	1
1	SP1572977	(Seatframe 510mm, cpl., pre set up to SW 43cm)	1
2	SP1582216	(Seatframe 545mm, cpl., pre set up to SW 43cm)	1
3	SP1571810	(Sealing strap with plugs)	1
4	SP1571811	(Groove stone with mounting parts)	1
5	SP1554646	(Adjustment Tube)	1


(Telescopic Seat Frame for Tilt 30° / Lifter)

1572004 Rev

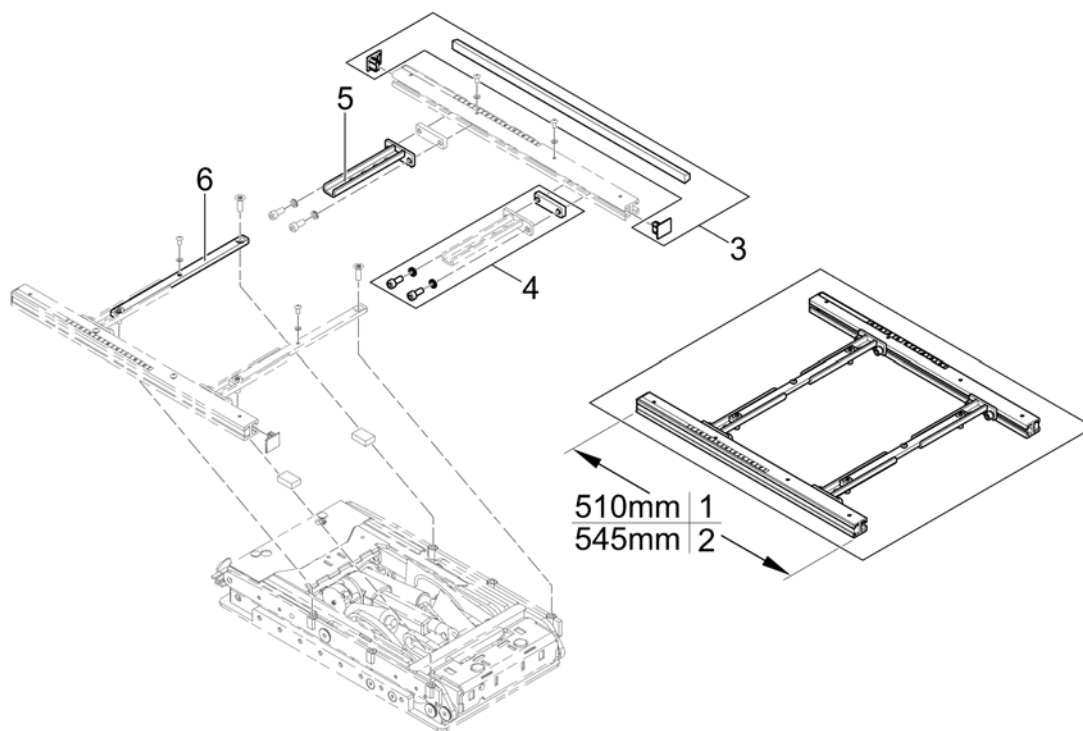

-	SP1588443	(Seat Rails 510mm (2pcs kit))	1
-	SP1588444	(Seat Rails 545mm (2pcs kit))	1
1	SP1572992	(Seatframe 510mm, pre set up to SW 430mm, cpl.)	1
2	SP1582219	(Seatframe 545mm, pre set up to SW 430mm, cpl.)	1
3	SP1571810	(Sealing strap with plugs)	1
4	SP1571811	(Groove stone with mounting parts)	1
5	SP1567007	(Seat Frame Adapter)	1
6	SP1567008	(Middle Rail)	1

1571834 Rev

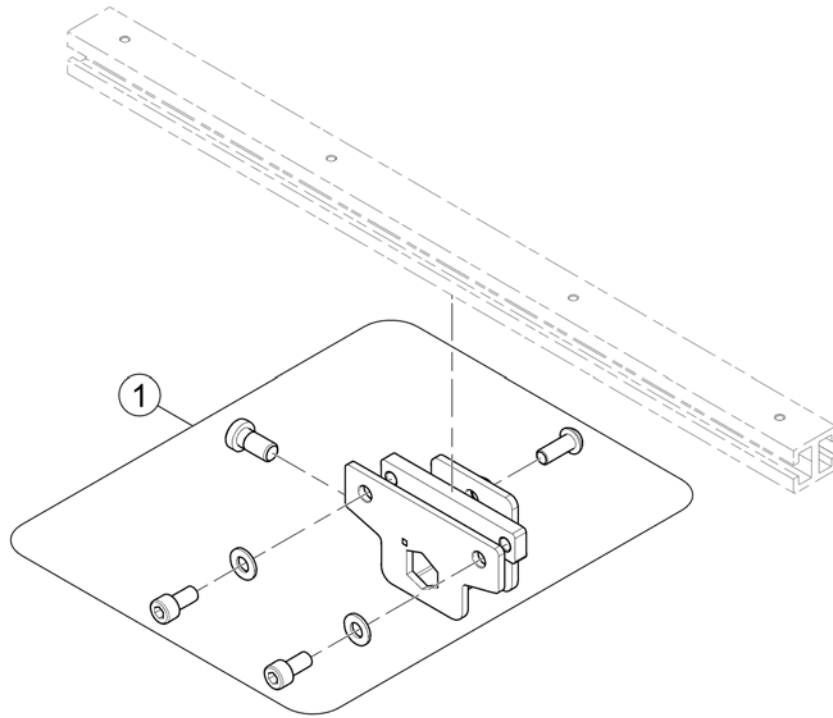


1 SP1570189

(Armrest bracket)


1

(Legrest Support)

1572021.000

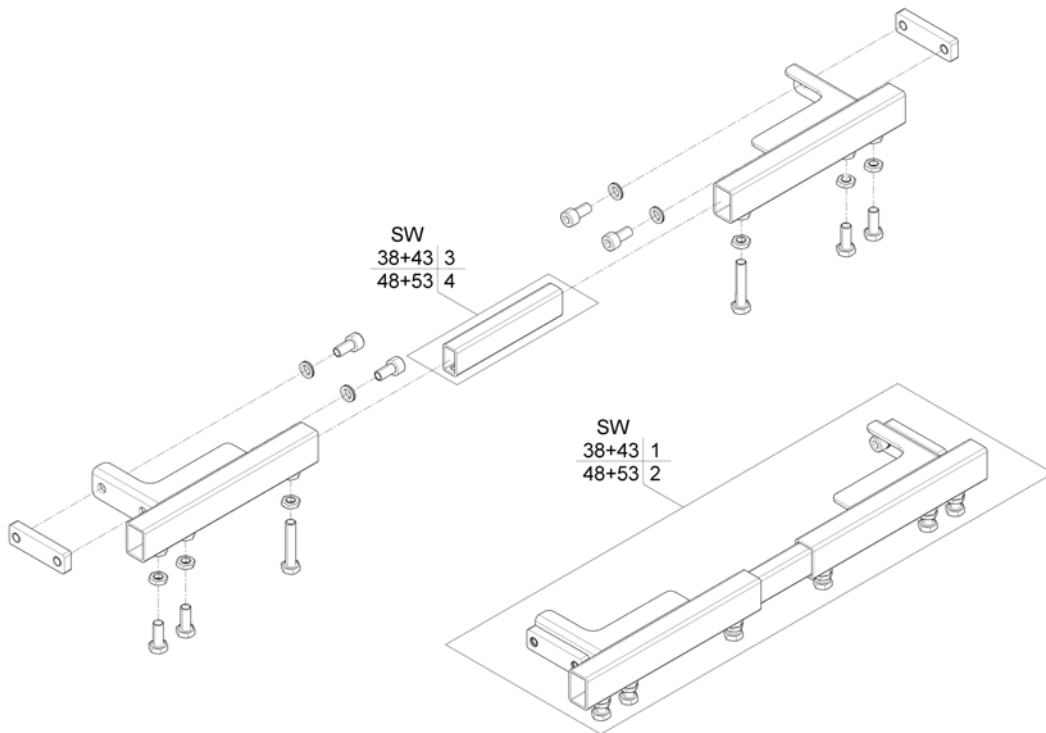

1	SP1558935	(Legrest Support SW38+43, cpl.)	1
2	SP1559294	BENSTØTTE FESTE SB48+53 MODULITE	1
3	SP1558134	(Connection Tube SW380+430mm)	1
4	SP1558135	BRAKETT SETEPLATE BENST MODULITE SB48-53	1

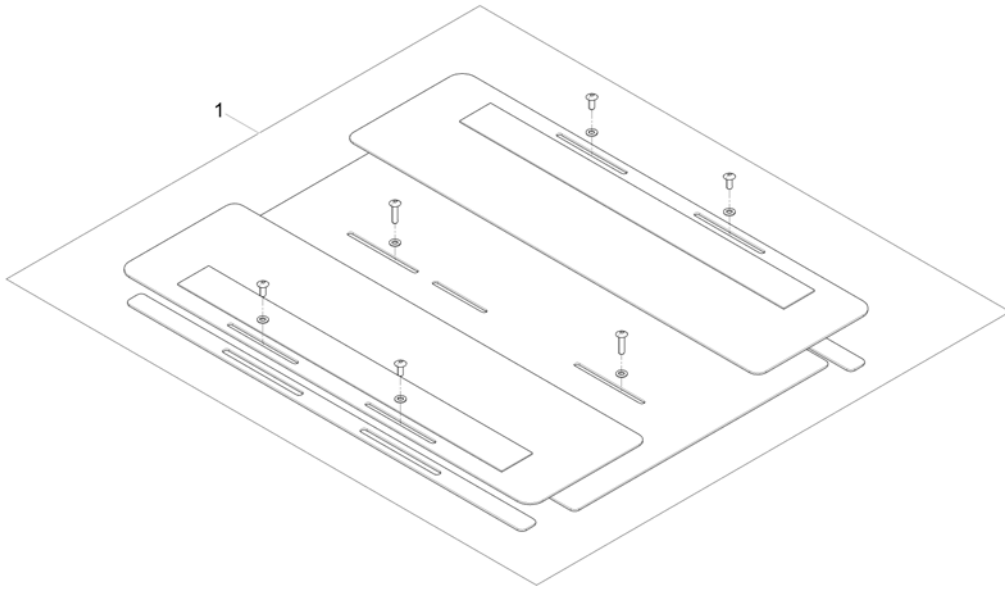


1 SP1572199

(Seat Plate 510mm, cpl.)

1

(Telescopic Seat Frame and Sling Seat)


1572311.00

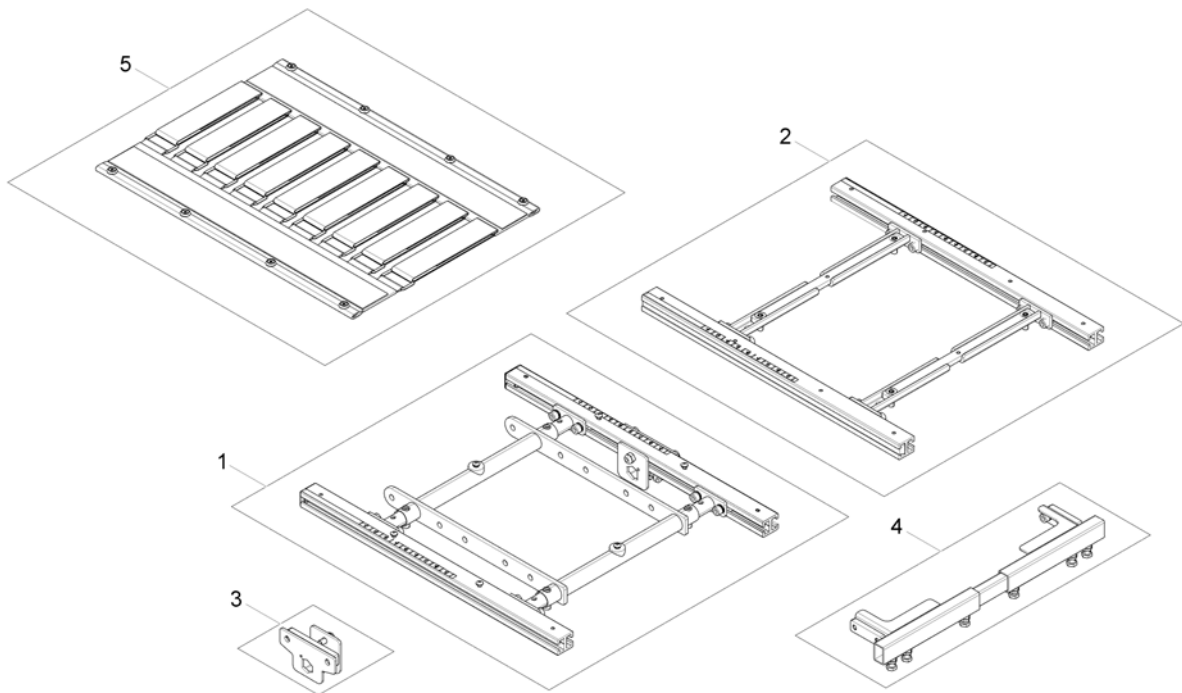

1	-	(Telescopic Seat Frame for Tilt 12° / 20°)	1
2	-	(Telescopic Seat Frame for Tilt 30° / Lifter)	1
3	-	(Armrest bracket)	1
4	-	(Legrest Support)	1
5	-	(Sling Seat)	1

(Telescopic Seat Frame for Tilt 12° / 20°)


1571900-00

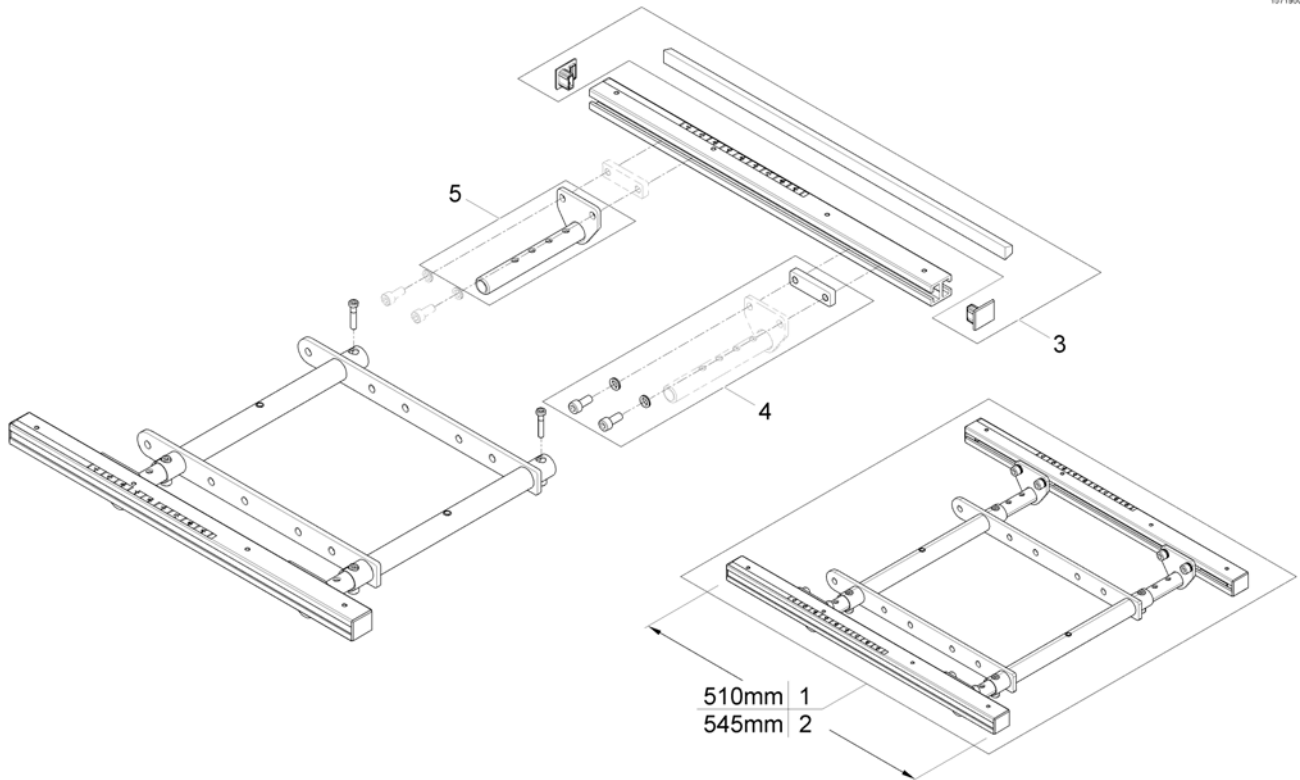


-	SP1588443	(Seat Rails 510mm (2pcs kit))	1
-	SP1588444	(Seat Rails 545mm (2pcs kit))	1
1	SP1572978	(Seatframe 510mm, cpl., pre set up to SW 43cm)	1
2	SP1582217	(Seatframe 545mm, cpl., pre set up to SW 43cm)	1
3	SP1571810	(Sealing strap with plugs)	1
4	SP1571811	(Groove stone with mounting parts)	1
5	SP1555400	(Adjustment Tube)	1

(Telescopic Seat Frame for Tilt 30° / Lifter)

1571832 no

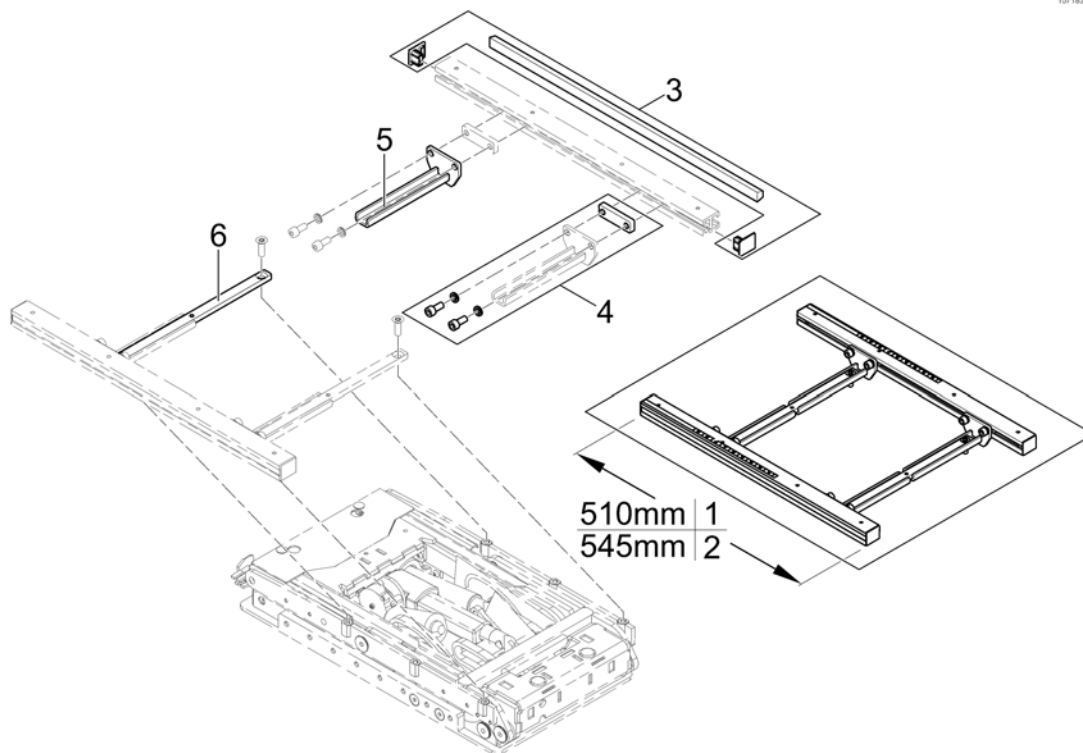


-	SP1588443	(Seat Rails 510mm (2pcs kit))	1
-	SP1588444	(Seat Rails 545mm (2pcs kit))	1
1	SP1572993	(Seatframe 510mm, pre set up to SW 430mm, cpl.)	1
2	SP1582230	(Seatframe 545mm, pre set up to SW 430mm, cpl.)	1
3	SP1571810	(Sealing strap with plugs)	1
4	SP1571811	(Groove stone with mounting parts)	1
5	SP1567720	(Seatframe Adapter)	1
6	SP1567008	(Middle Rail)	1

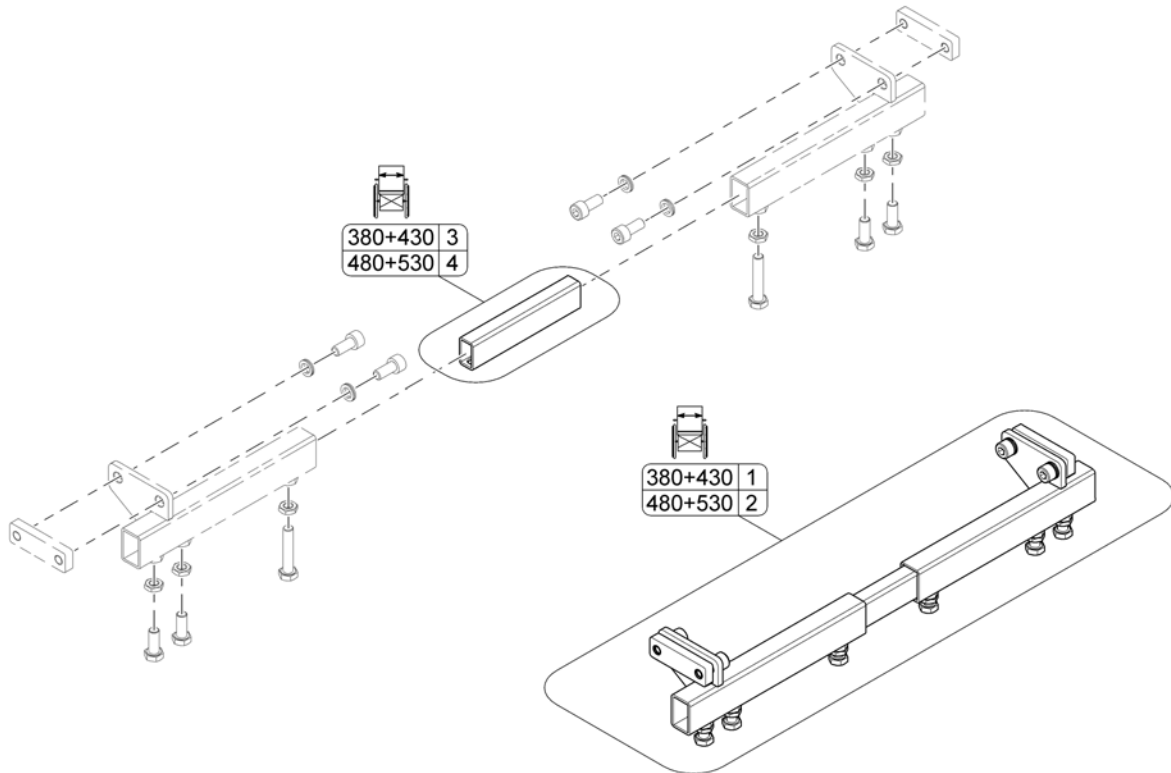

1	SP1559305	(Legrest Support SW380+430mm, cpl.)	1
2	SP1559307	(Legrest Support SW480+530mm, cpl.)	1
3	SP1558134	(Connection Tube SW380+430mm)	1
4	SP1558135	BRACKET SETEPLATE BENST MODULITE SB48-53	1

Invacare® TDX-SP | Sete
(Sling Seat)


1511762no

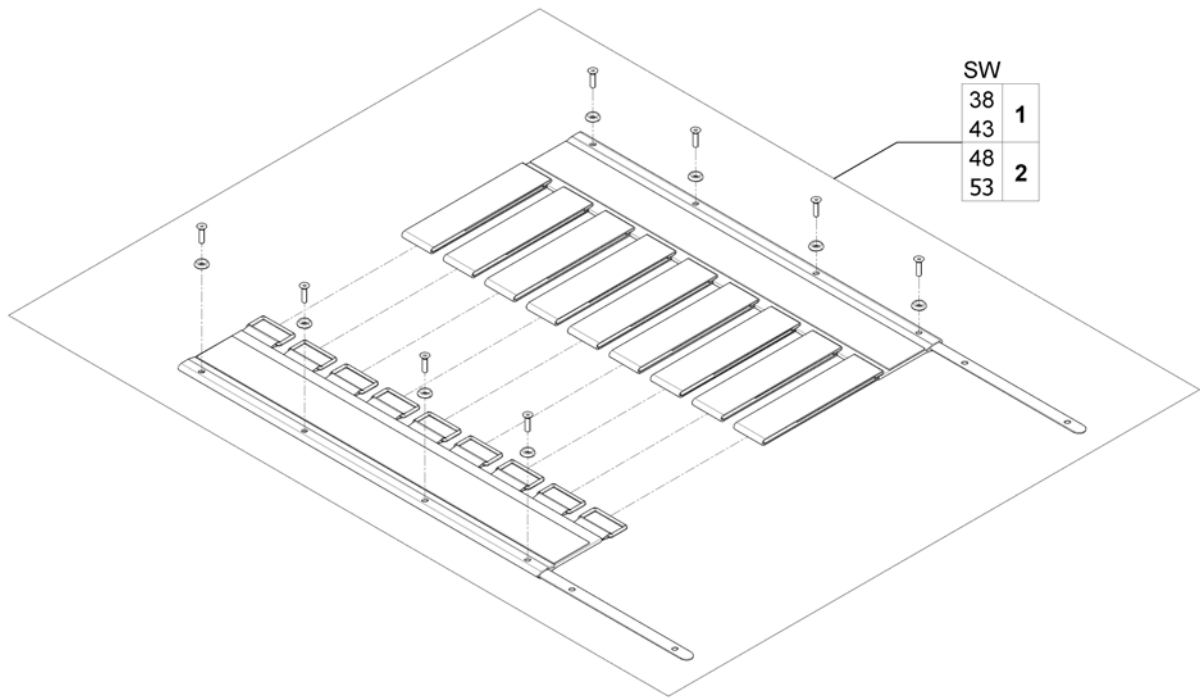

1	SP1557877	VELCRO SETE SB38-44 STORM4 TDX SP2	1
2	SP1555911	VELCRO SETEBUNN SB46 53 STORM4 TDX SP2	1

(Telescopic Seat Frame and Universal Adapter)


1581864 iso

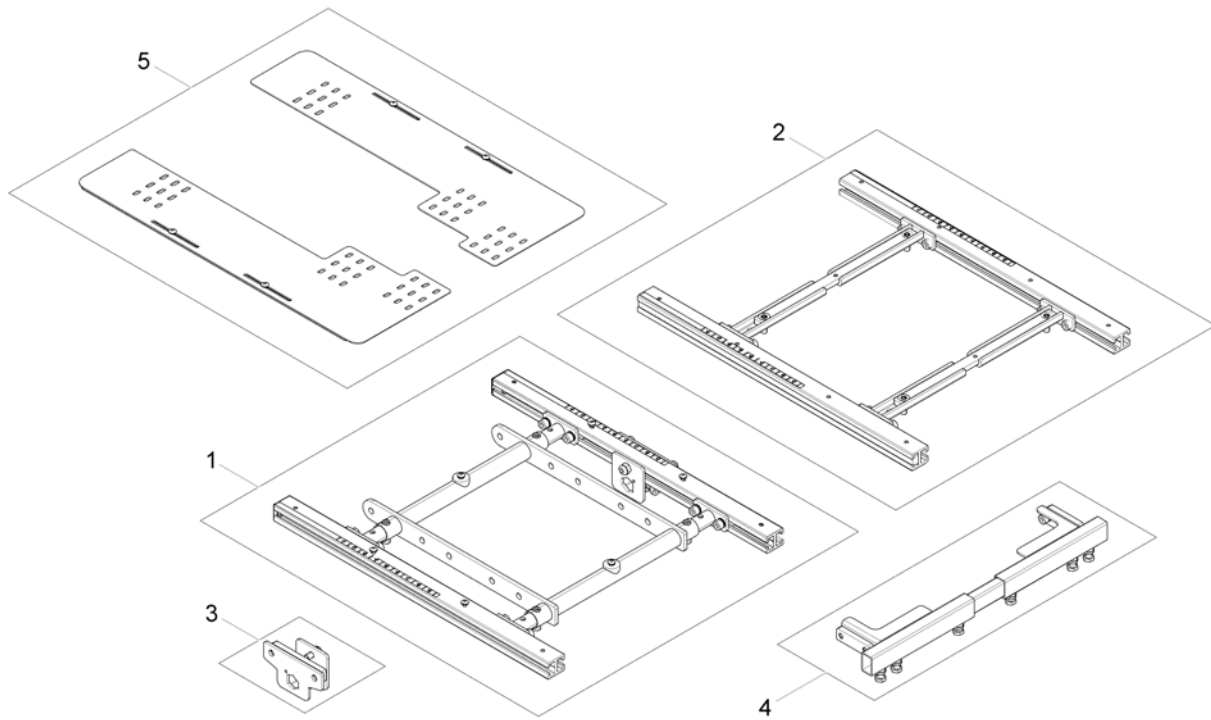

1	-	(Telescopic Seat Frame for Tilt 12° / 20°)	1
2	-	(Telescopic Seat Frame for Tilt 30° / Lifter)	1
3	-	(Armrest bracket)	1
4	-	(Legrest Support)	1
5	-	(Adapter Kit for Universal Seat)	1

(Adapter Kit for Universal Seat)


1581063.iso

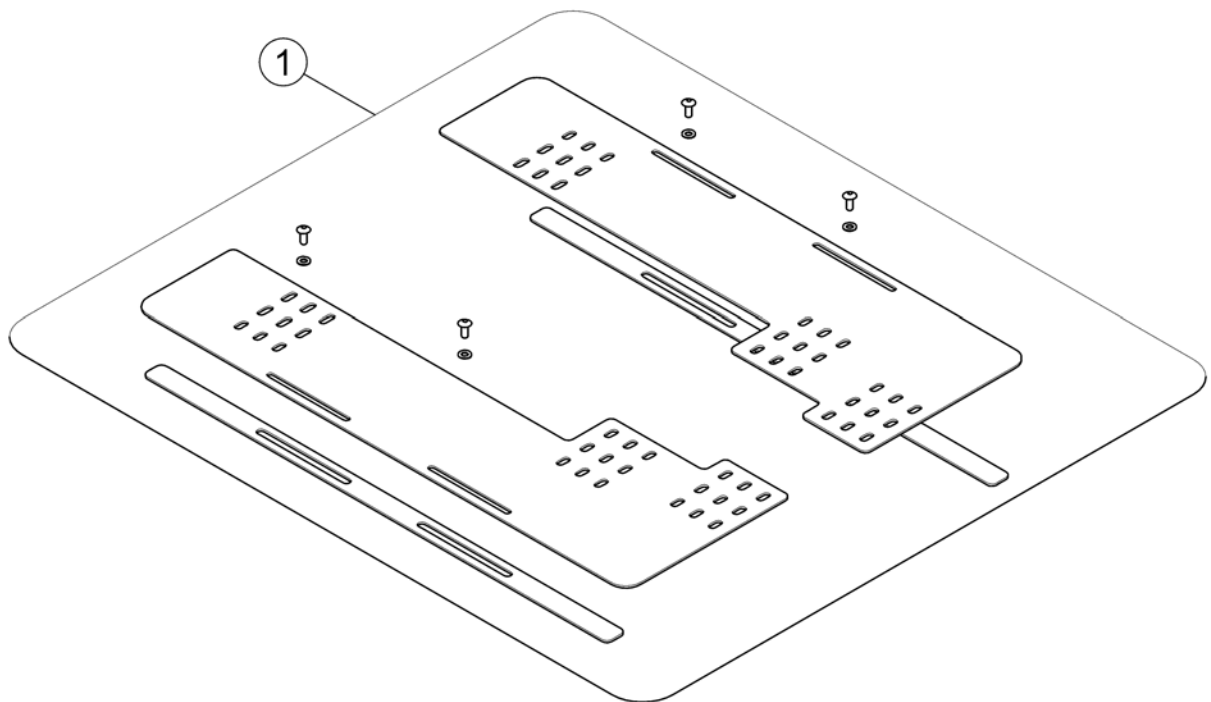

1 SP1580750

(Adapter Kit for Universal Seat 510mm)

1

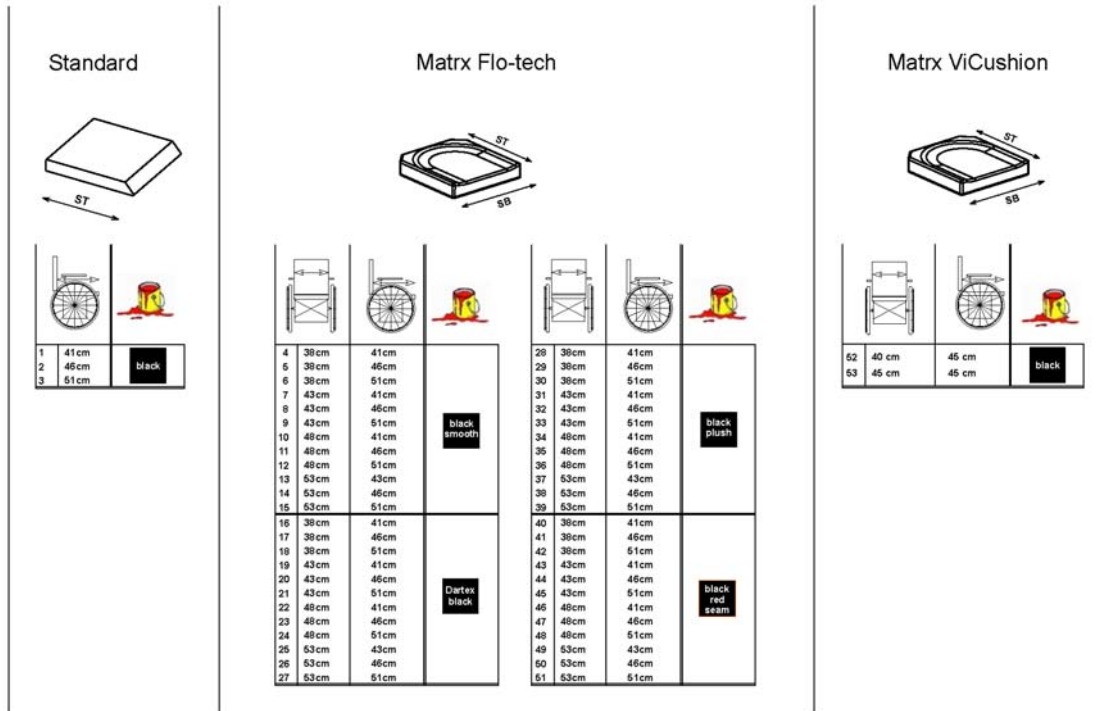


1	1420460	SETEPUTE SD41 STD DRAGON VERTIC [2141]	1
2	1420461	SETEPUTE SD46 STD DRAGON VERTIC [2141]	1
3	1420462	SETEPUTE SD51 STD DRAGON VERTIC [2141]	1
4	SPFLB15/16	SETEPUTE SB38 SD41 SORT STORM4 [2142+2151]	1
5	SPFLB15/18	SETEPUTE SB38 SD46 SORT STORM4 [2142+2151]	1
6	SPFLB15/20	SETEPUTE SB38 SD51 SORT STORM4 [2142+2151]	1
7	SPFLB17/16	SETEPUTE SB43 SD41 SORT STORM4 [2142+2151]	1
8	SPFLB17/18	SETEPUTE SB43 SD46 STD STORM4 [2142+2151]	1
9	SPFLB17/20	SETEPUTE SB43 SD51 SORT STORM4 [2142+2151]	1
10	SPFLB19/16	SETEPUTE SB48 SD41 SORT STORM4 [2142+2151]	1
11	SPFLB19/18	SETEPUTE SB48 SD46 SORT STORM4 [2142+2151]	1
12	SPFLB19/20	SETEPUTE SB48 SD51 SORT STORM4 [2142+2151]	1
13	SPFCB21/17	SETEPUTE SB53 SD43 SORT STORM4 [2142+2151]	1
14	SPFCB21/18	SETEPUTE SB53 SD46 SORT STORM4 [2142+2151]	1
15	SPFCB21/20	SETEPUTE SB53 SD51 SORT STORM4 [2142+2151]	1
16	SP1529546-TR26	(seat cushion Flo-tech Lite SW 38 x SD 41 Dartex black) [2142+2152]	1
17	SP1529547-TR26	(seat cushion Flo-tech Lite SW 38 x SD 46 Dartex black) [2142+2152]	1
18	SP1529548-TR26	(seat cushion Flo-tech Lite SW 38 x SD 51 Dartex black) [2142+2152]	1

(Seat Cushion)

19	SP1529549-TR26	(seat cushion Flo-tech Lite SW 43 x SD 41 Dartex black) [2142+2152]	1
20	SP1529550-TR26	SETEPUTE SB43SD46 DARTEX STORM4 TDX [2142+2152]	1
21	SP1529551-TR26	(seat cushion Flo-tech Lite SW 43 x SD 51 Dartex black) [2142+2152]	1
22	SP1529552-TR26	(seat cushion Flo-tech Lite SW 48 x SD 41 Dartex black) [2142+2152]	1
23	SP1529553-TR26	(seat cushion Flo-tech Lite SW 48 x SD 46 Dartex black) [2142+2152]	1
24	SP1529554-TR26	(seat cushion Flo-tech Lite SW 48 x SD 51 Dartex black) [2142+2152]	1
25	SP1529555-TR26	(seat cushion Flo-tech Contour SW 53 x SD 43 Dartex black) [2142+2152]	1
26	SP1529556-TR26	(seat cushion Flo-tech Contour SW 53 x SD 46 Dartex black) [2142+2152]	1
27	SP1529570-TR26	(seat cushion Flo-tech Contour SW 53 x SD 51 Dartex black) [2142+2152]	1
28	SP1529546-TR35	(seat cushion Flo-tech Lite SW 38 x SD 41 black chenille fabric) [2142+2154]	1
29	SP1529547-TR35	(seat cushion Flo-tech Lite SW 38 x SD 46 black chenille fabric) [2142+2154]	1
30	SP1529548-TR35	(seat cushion Flo-tech Lite SW 38 x SD 51 black chenille fabric) [2142+2154]	1
31	SP1529549-TR35	(seat cushion Flo-tech Lite SW 43 x SD 41 black chenille fabric) [2142+2154]	1
32	SP1529550-TR35	(seat cushion Flo-tech Lite SW 43 x SD 46 black chenille fabric) [2142+2154]	1
33	SP1529551-TR35	(seat cushion Flo-tech Lite SW 43 x SD 51 black chenille fabric) [2142+2154]	1
34	SP1529552-TR35	(Seat cushion Flo-tech Lite SW 48 x SD 41 black chenille fabric) [2142+2154]	1
35	SP1529553-TR35	(seat cushion Flo-tech Lite SW 48 x SD 46 black chenille fabric) [2142+2154]	1
36	SP1529554-TR35	(seat cushion Flo-tech Lite SW 48 x SD 51 black chenille fabric) [2142+2154]	1
37	SP1529555-TR35	(seat cushion Flo-tech Contour SW 53 X SD 43 black chenille fabric) [2142+2154]	1
38	SP1529556-TR35	(seat cushion Flo-tech Contour SW 53 x SD 46 black chenille fabric) [2142+2154]	1
39	SP1529570-TR35	(seat cushion Flo-tech Contour SW 53 x SD 51 black chenille fabric) [2142+2154]	1
40	SP1529546	SETEPUTE EASY-A SB38SD41 SORT M ORANG SØM [2142+2153]	1
41	SP1529547	SETEPUTE EASY-A SB38SD46 SORT M ORANG SØM [2142+2153]	1
42	SP1529548	SETEPUTE EASY-A SB38SD51 SORT M ORANG SØM [2142+2153]	1
43	SP1529549	SETEPUTE EASY-A SB43SD41 SORT M ORANG SØM [2142+2153]	1
44	SP1529550	SETEPUTE EASY-A SB43SD46 SORT M ORANG SØM [2142+2153]	1
45	SP1529551	SETEPUTE EASY-A SB43SD51 SORT M ORANG SØM [2142+2153]	1
46	SP1529552	SETEPUTE EASY-A SB48SD41 SORT M ORANG SØM [2142+2153]	1
47	SP1529553	SETEPUTE EASY-A SB48SD46 SORT M ORANG SØM [2142+2153]	1
48	SP1529554	SETEPUTE EASY-A SB48SD51 SORT M ORANG SØM [2142+2153]	1
49	SP1529555	SETEPUTE EASY-A SB53SD43 SORT M ORANG SØM [2142+2153]	1
50	SP1529556	SETEPUTE EASY-A SB53SD46 SORT M ORANG SØM [2142+2153]	1

51	SP1529570	SETEPUTE EASY-A SB53 SD5 SORT M ORANG SØM <i>[2142+2153]</i>	1
52	SP1531760	SETEPUTE 40x45 MATRIX MATRIX <i>[2144]</i>	1
53	SP1531761	SeatCushion 45x45cm für PB-&Full Back Cushions <i>[2144]</i>	1
54	SP1501113	SETEPUTE 43*43 TDX FLOTEC CONTUR VISCO <i>[2143]</i>	1

Rygg

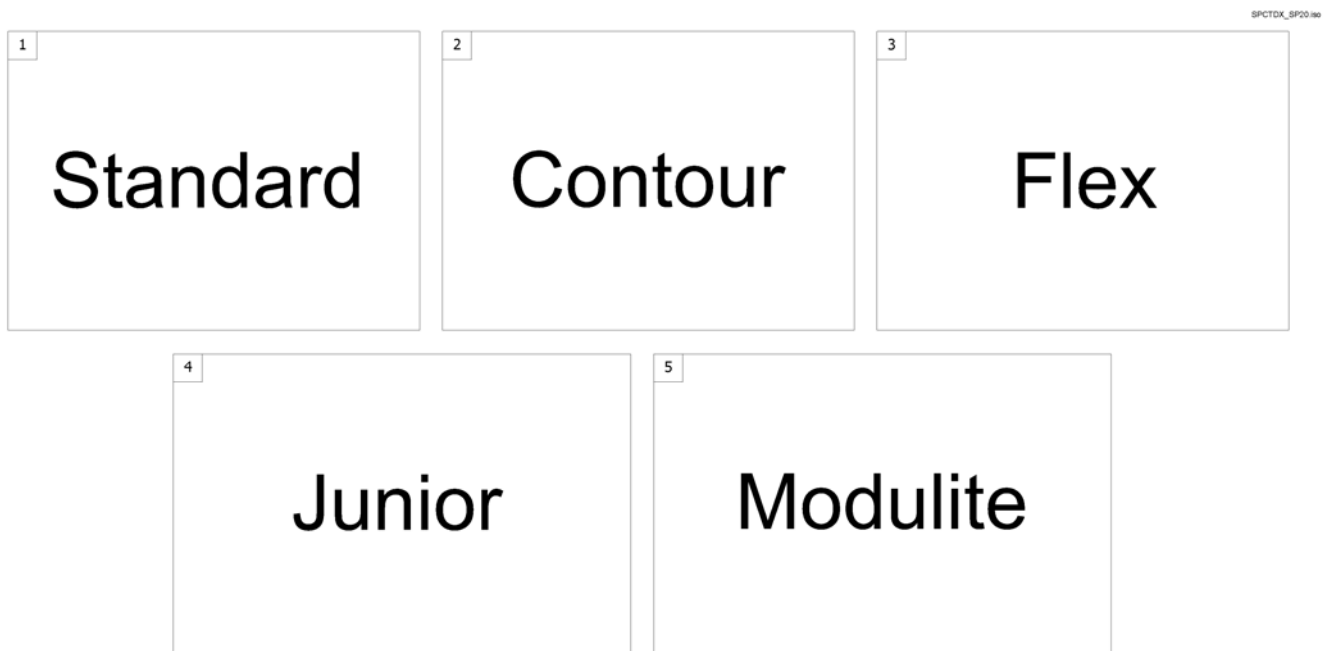


1	-	(Backrest for Standard Seat)	1
2	-	(Backrest for Contour Seat)	1
3	-	(Backrest for Flex Seat)	1
4	-	(Backrest for Junior Seat)	1
5	-	(Backrest for Modulite Seat)	1


-	(Backrest Standard Seat, settable) [0958]	1
-	(Backrest Recliner) [0953/0954/0947]	1

(Backrest Standard Seat, settable)

1422061X.no

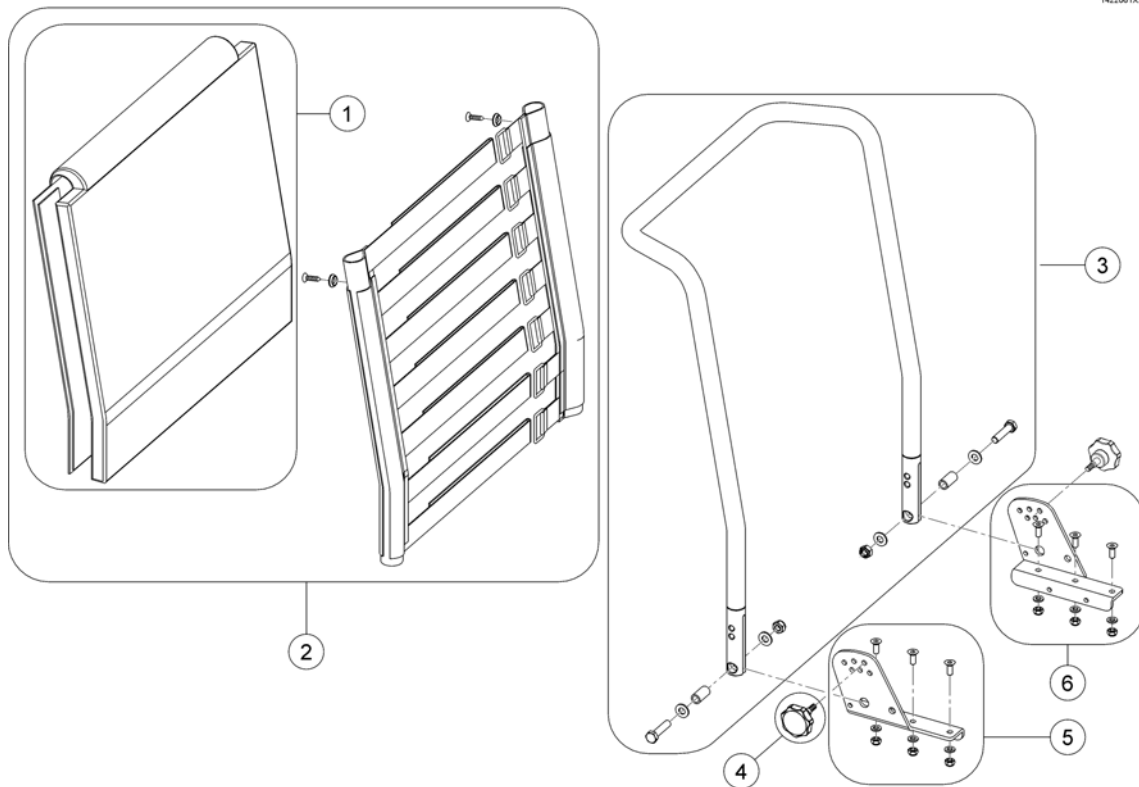


1	1420445	RYGGPUTE BORA	1
1	1420452	RYGGPUTE 54 CM BORA	1
2	1422062	VELCRO RYGG STANDARD SETE STORM3	1
2	1422063	VELCRO RYGG STANDARD SETE RH54 STORM3	1
3	SP1419996	RYGGGRAMME HØ 48CM STORM4	1
3	SP1420465	RYGGGRAMME HØ 54CM STORM4	1
4	SP1420444	FINGERSKRUE M 8 RYGGREG.DRAGON	1
5	SP1541567	(Backrest bracket RH)	1
6	SP1541568	(Backrest bracket LH)	1

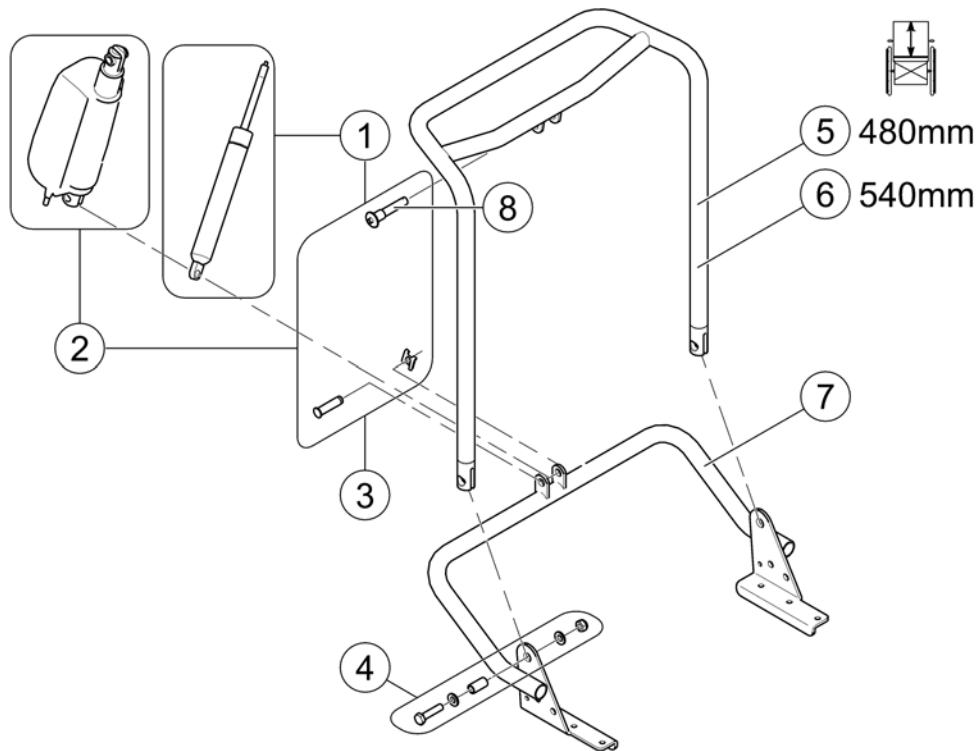

-	SP1562940	(Adaptercable Shark to reverse actuator)	1
1	SPN368007	GASSFJÆR RYGGREGULERING TDX	1
2	SP1446520	AKTUATOR EL RYGG DRAGON	1
3	SP1583897	(Fixation Kit)	1
4	SP1579749	(Fixation Kit, Standard Backrest)	2
5	1442864-9005	(Backrest frame for 480 mm height (push handle), adjustable)	1
6	1442866-9005	(Backrest frame for 540 mm height (push handle), adjustable)	1
7	1442845-9005	RYGGBØYE NEDRE BORA	1
8	F368016.99	LÅSESPLINT TIL RYGG AKTUATOR	1


(Backrest for Contour Seat)

-	(Backrest Flex Seat)	1
-	(Backrest Adjustment Flex/Contour manual)	1
-	(Backrest adjustment Flex/Contour electric)	1
-	(Backrest Cushion for Contour Seat)	1

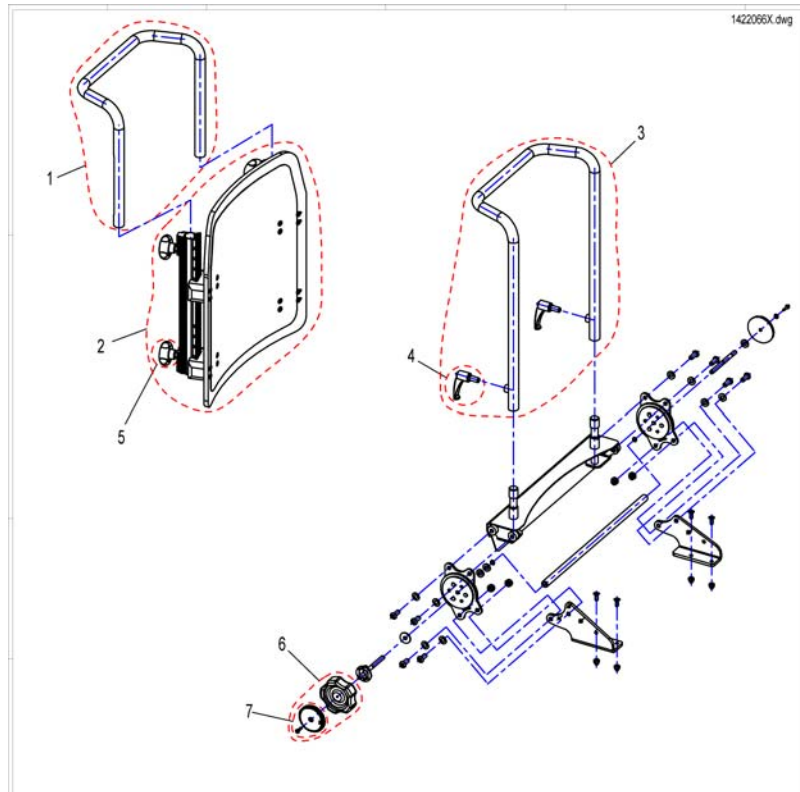

1	SP1425840	RYGGRAMME SB39CM TDX	1
1	SP1425839	KJØREHÅNDTAK SB43-48CM TDX	1
2	SP1422126	RYGG FLEX U PUTE SB39 STORM3	1
2	SP1422125	RYGG FLEX U PUTE SB48 STORM 3	1
2	SP1422124	RYGG FLEX U PUTE SB43 STORM 3	1
3	SP1422070	RYGGRØR KPL SB43/48 SORT CONTUR	1
4	L31996	LÅSEKRUE M/HENDEL	1
5	SP5253088	(Pressure Screw)	1
6	SP1420483	PLASTRATT STORM4	1
7	SP1421696	REFLEX STORM4	1


(Backrest Adjustment Flex/Contour manual)

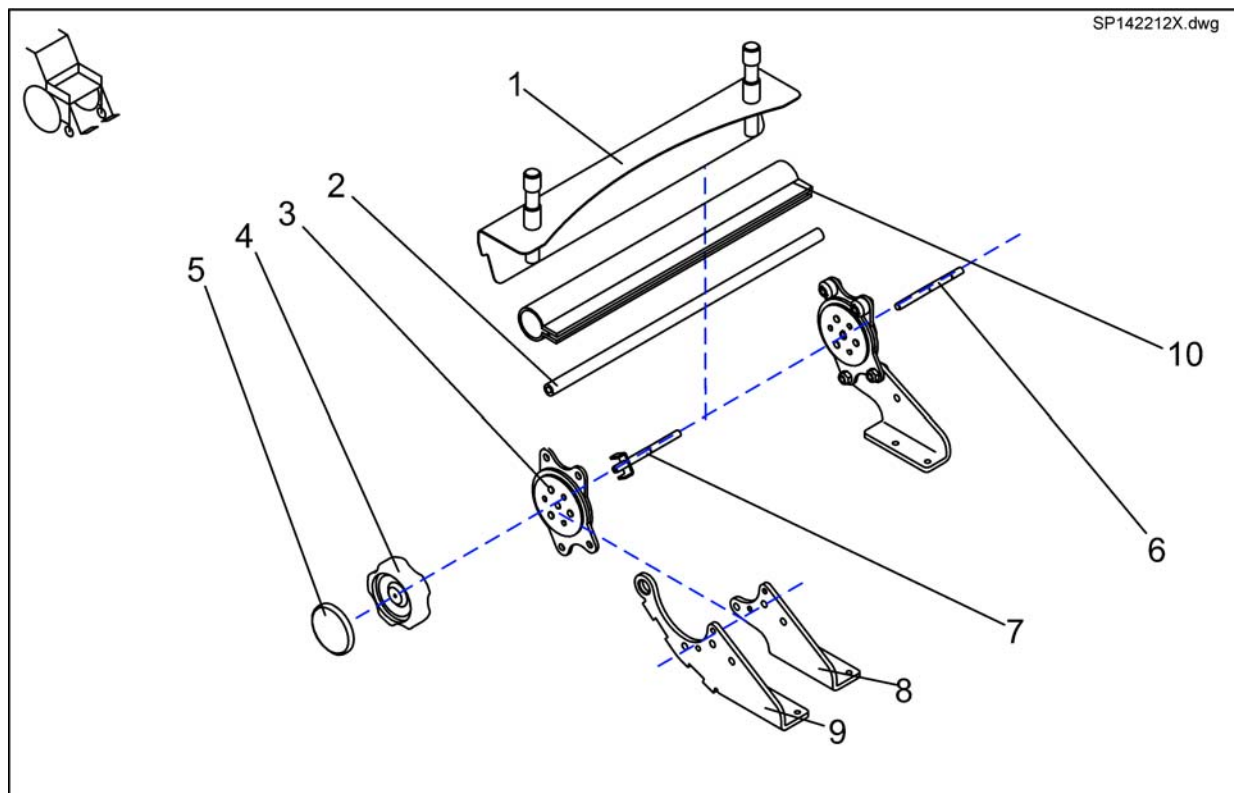

-	1442942	(MANUAL BACK ADJUSTER, SEAT WIDTH 43CM, FOR SLIDING ARMRESTS, CPL.)	1
-	SP1422067	RYGGJUSTERING MAN SB 48 STORM3	1
-	SP1422122	RYGGJUSTERING MAN SB 39 FLEX STORM3	1
-	SP1425854	(MANUAL BACK ADJUSTER, SEAT WIDTH 48CM, FOR SLIDING ARMRESTS, CPL.)	1
-	SP1423210	MONT SETT FOR RYGGJUST STO/TYP/TO	1
-	SP1422066	RYGGJUSTERING MAN SB 43 STORM3	1
-	SP1425857	(MANUAL BACK ADJUSTER, SEAT WIDTH 39CM, FOR SLIDING ARMRESTS, CPL.)	1
1	1422112-9005	RYGGFESTE SB 48 FLEX CONTUR	1
1	1419992-9005	RYGGFESTE SB 43 FLEX CONTUR	1
1	1422119-9005	RYGGFESTE SB 39 FLEX CONTUR	1
2	1420491	FORBINDELSERØR SB 48 FLEX CONTUR	1
2	1420489	FORBINDELSERØR SB 39 FLEX CONTUR	1
2	1420490	FORBINDELSERØR SB 43 FLEX CONTUR	1
3	SP1450234	BESLAG SETE RYGG MAN PAR STORM4	1
4	1420483	(HAND WHEEL)	1
5	1421696	REFLEKS GUL	1
6	1420488-0002	BESLAG SPINDEL TIL VREG RYGG	1
7	1421713-0002	(HAND WHEEL SHAFT)	1
8	1420476-9005	(BRACE BRACKET MANUAL RH)	1
8	1420477-9005	FORSTERKING EL RYGG VE TDX	1
9	1429564-9005	HOLDER RYGGJUSTERING VE OPPFELLB ARMLEN	1
9	1429563-9005	HOLDER RYGGJUSTERING HØ OPPFELLB ARMLEN	1
10	1442579	STREKKAPSLING SB 48 FLEX CONTUR	1
10	1442578	STREKKAPSLING SB 43 FLEX CONTUR	1
10	1442577	STREKKAPSLING SB 39 FLEX CONTUR	1

(Backrest adjustment Flex/Contour electric)

1440456X.dwg

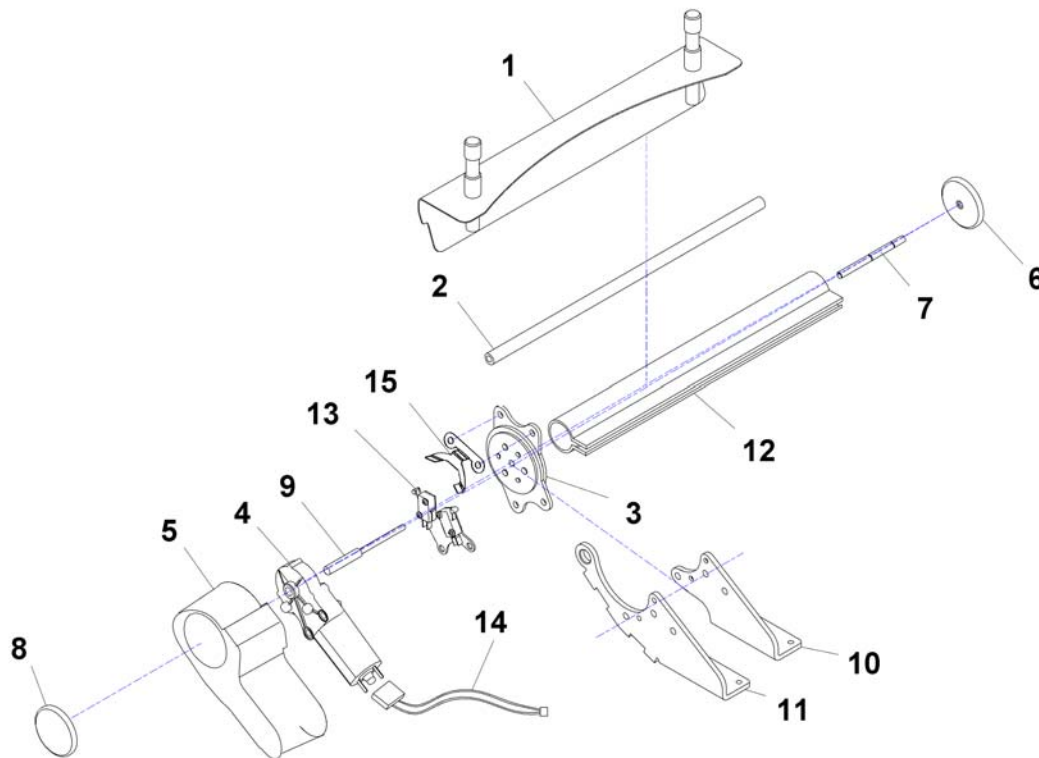


	SP1440407	RYGGJUSTERING EL SB39 FLEX TORNADO	1
	SP1440405	RYGGJUSTERING EL SB43 KONTUR TYPHOON	1
	SP1440406	RYGGJUSTERING EL SB48 CONTOUR	1
	SP1440410	RYGGREG EL F OPPF ARML TDX	1
	SP1440408	RYGGJUSTERING EL SB43 OPPFELLB ARMLEN	1
	SP1440409	RYGGJUSTERING EL SB48 OPPFELLB ARMLEN	1
	SP1423210	MONT SETT FOR RYGGJUST STO/TYP/TO	1
	SP1529386	()	1
1	1422119-9005	RYGGFESTE SB 39 FLEX CONTUR	1
1	1419992-9005	RYGGFESTE SB 43 FLEX CONTUR	1
1	1422112-9005	RYGGFESTE SB 48 FLEX CONTUR	1
2	1420489	FORBINDELSERØR SB 39 FLEX CONTUR	1
2	1420490	FORBINDELSERØR SB 43 FLEX CONTUR	1
2	1420491	FORBINDELSERØR SB 48 FLEX CONTUR	1
3	SP1423211	BESLAG SETE RYGG PAR STORM4	1
4	1420475	GIRMOTOR RYGG STORM 3	1
5	1420998	MOTORDEKSEL EL RYGG STORM3 TDX	1
5	1428057	DEKSEL RYGGMOTOR TDX	1
6	1421696	REFLEKS GUL	1
7	1420488-0002	BESLAG SPINDEL TIL VREG RYGG	1
8	1420482	REFLEKS GUL	1
9	1420487-0002	(MOTOR SHAFT)	1
10	1420477-9005	FORSTERKING EL RYGG VE TDX	1
10	1421708-9005	FORSTERKING EL RYGG HØ TDX	1
11	1429563-9005	HOLDER RYGGJUSTERING HØ OPPFELLB ARMLEN	1
11	1429564-9005	HOLDER RYGGJUSTERING VE OPPFELLB ARMLEN	1
12	1442577	STREKKAPSLING SB 39 FLEX CONTUR	1
12	1442578	STREKKAPSLING SB 43 FLEX CONTUR	1
12	1442579	STREKKAPSLING SB 48 FLEX CONTUR	1
13	SP1440711	MIKROBRYTER KPL FERDIG MONTERT EL RYGG	1
14	SP1440726	(ACTUATORCABLE RECLINER ACS2 / SHARK)	1




(Backrest adjustment Flex/Contour electric)

(Backrest Cushion for Contour Seat)

1573703.iso

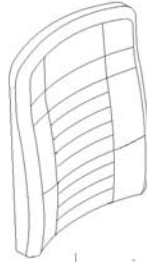Contour



		
1	43cm	
2	48cm	

1	SP1422076	RYGGPUTE 43 CONTUR M/KLEMMER SORT STORM 3	1
2	SP1422078	RYGGPUTE 48 CONTUR SORT	1
-	SP1423216	KLEMMER KPL SETT 4 STK CONTUR RYGG.STORM3/TORNA	1

(Backrest for Flex Seat)

-	(Backrest Flex Seat)	1
-	(Backrest Adjustment Flex/Contour manual)	1
-	(Backrest adjustment Flex/Contour electric)	1
-	(Backrest Cushion for Flex Seat)	1

(Backrest Cushion for Flex Seat)

1572719.no




Flex



		
1	39cm	
2	43cm	
3	48cm	==> 31.03.13
4	39cm	
5	43cm	
6	48cm	==> 31.03.13
7	39cm	
8	43cm	
9	48cm	
10	39cm	
11	43cm	
12	48cm	01.04.13 ==>

VIC AIR




		
13	43cm	

1	1422534	RYGGPUTE 39 FLEX BLÅ STORM3	→ 31.03.2013	1
2	1422535	RYGGPUTE 43 FLEX BLÅ STORM3	→ 31.03.2013	1
3	1422536	RYGGPUTE 48 FLEX BLÅ STORM3	→ 31.03.2013	1
4	1436564	RYGGPUTE 39 FLEX SORT	→ 31.03.2013	1
5	-	(No longer available - please choose Black Plush)	→ 31.03.2013	0
6	-	(No longer available - please choose Black Plush)	→ 31.03.2013	0
7	5252506-TR23	\$RYGGPUTE 39 REA COMFORT DARTEX		1
8	5252507-TR23	RYGGPUTE 44 REA COMFORT		1
9	5252508-TR23	\$RYGGPUTE 49 REA COMFORT		1
10	SP1436564-TR35	(Back Cushion SW39cm, black Plush)	01.04.2013 →	1
11	SP1436565-TR35	RYGGPUTE FLEX SB43 SORT TDX	01.04.2013 →	1
12	SP1436566-TR35	RYGGPUTE 48 FLEX SORT PLYSJ	01.04.2013 →	1
13	1422537	(Back Cushion SW43cm, VIC AIR)		1


(Backrest for Junior Seat)

-	(Backrest / Junior Seat)	1
-	(Backrest Cushion for Junior Seat)	1

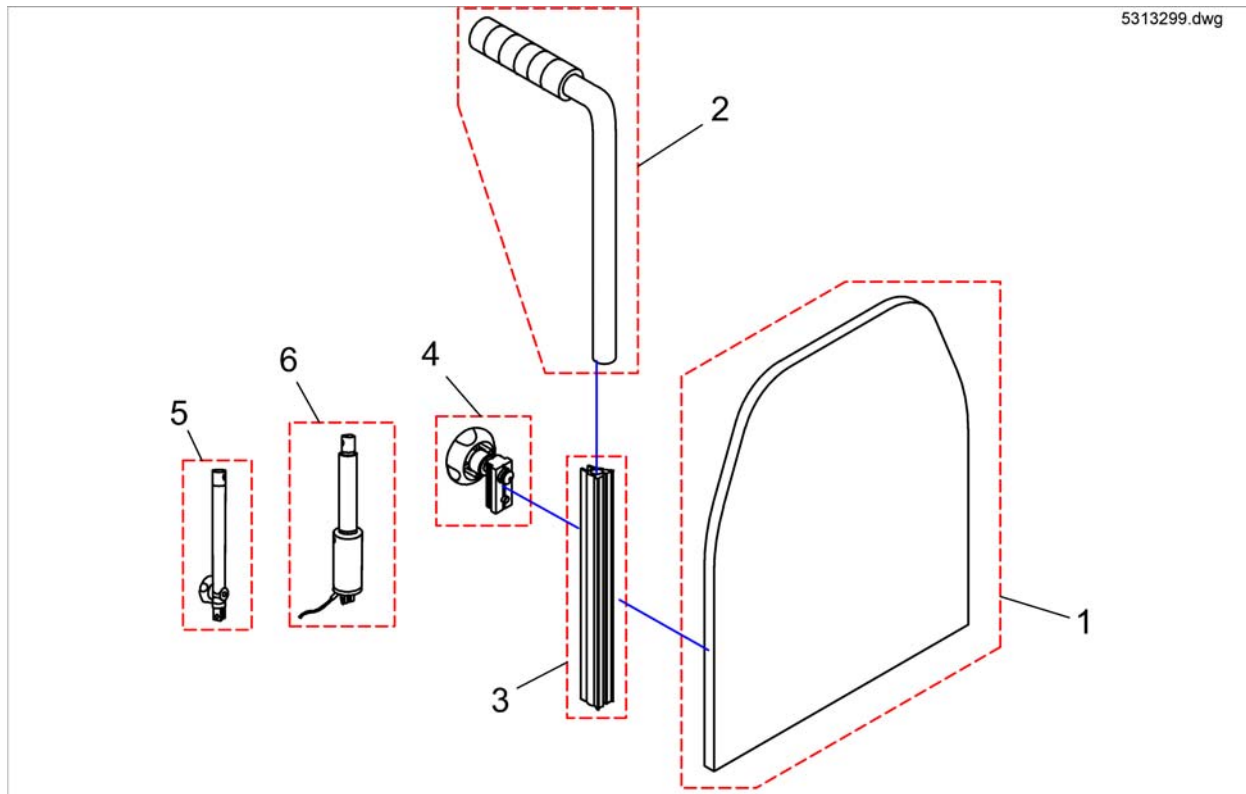

1	5313162	RYGGPLATE JUNIOR SB35	1
1	5313136	RYGGPLATE JUNIOR SB30	1
1	5313188	(BACKREST PLATE SB 38)	1
2	SP1524672	(Push Handle (single one))	1
3	5252204	RYGGRØR STK U FESTERATT REA COMFORT	1
4	5253088	LÅSKLOSS KPL F RYGGRØR REA COMFORT	1
5	5313220	RYGGREGULERING MAN TDX	1
5	SP1445030	RYGGREGULERINGKIT KPL TDX	1
6	SP1445029	RYGGJUSTERING EL JUNIOR	1
6	5316008	MOTOR TDX	1


(Backrest Cushion for Junior Seat)

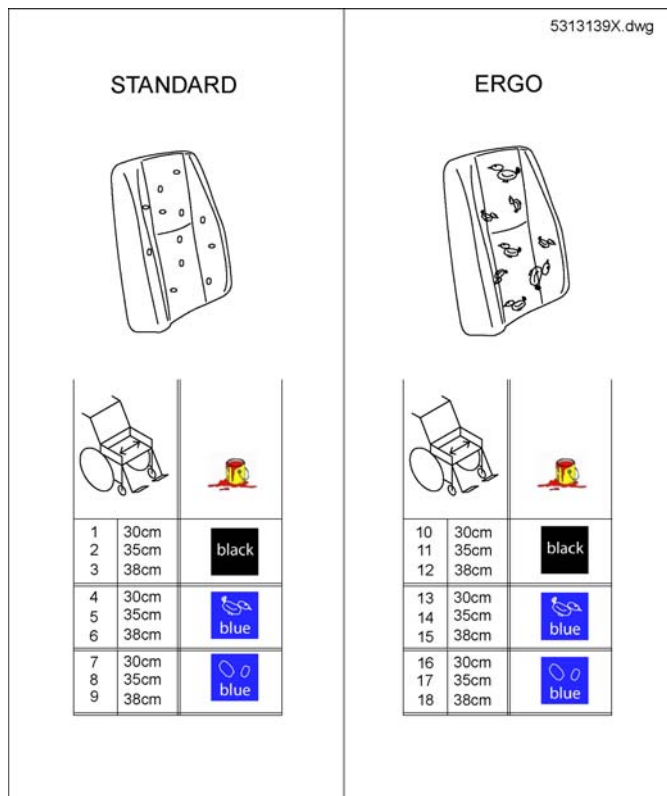

1	5313142	(Back Cushion STD SW30cm, Vinyl Black)	1
2	5313168	(Back Cushion STD SW35cm, Vinyl Black)	1
3	5313194	(Back Cushion STD SW38cm, Vinyl Black)	1
4	5313139	RYGGPUTE 30 STD MOO85 KID BLUE BIRDIE	1
5	5313165	RYGGPUTE 35 STD MOO85 KID BLUE BIRDIE	1
6	5313191	RYGGPUTE 38 STD MOO85 KID BLUE BIRDIE	1
7	5313141	RYGGPUTE 30 MOO85 KID BLÅ	1
8	5313167	RYGGPUTE 35 MOO85 KID BLÅ	1
9	5313193	RYGGPUTE 38 MOO85 KID BLÅ	1
10	5313146	(Back Cushion Ergo SW30cm, Vinyl Black)	1
11	5313172	(Back Cushion Ergo SW35cm, Vinyl Black)	1
12	5313198	RYGGPUTE 38 ERGONOMISK MOO85 KID SORT SKAI	1
14	5313169	RYGGPUTE 35 ERGONOMISK MOO85 KID BLUE BIRDIE	1
15	5313195	RYGGPUTE 38 ERGONOMISK MOO85 KID BLUE BIRDIE	1
16	5313145	RYGGPUTE 30 ERGONOMISK MOO85 KID BLÅ	1
17	5313171	RYGGPUTE 35 ERGONOMISK MOO85 KID BLÅ	1
18	5313197	RYGGPUTE 38 ERGONOMISK MOO85 KID BLÅ	1

(Backrest for Modulite Seat)


1586095.000

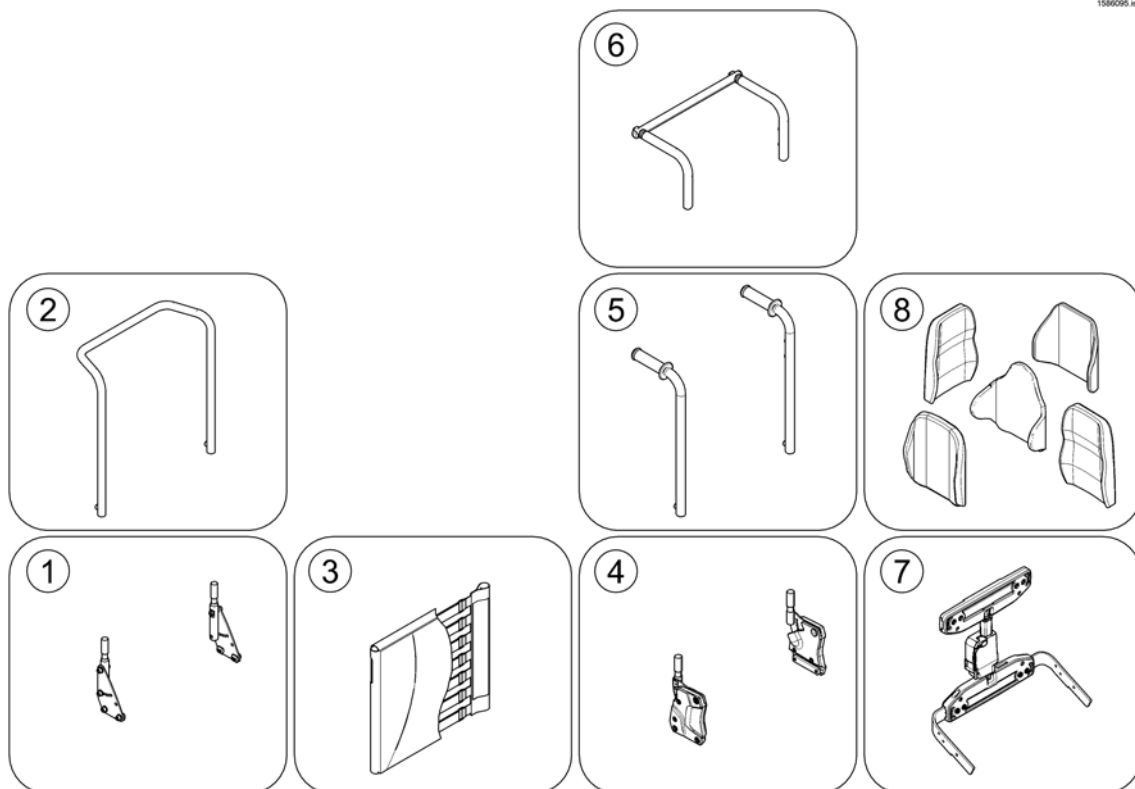

1	-	(Support for Backrest Frame) [0975+2161]	0
2	-	(Backrest Frame) [2161]	0
3	-	(Sling Back, tension adjustable) [2171]	0
4	-	(Backrest support)	0
5	-	(Backposts) [2162/2163]	0
6	-	(Connection tube) [1307]	0
7	-	(Back Recliner / Stabilization bracket) [0977]	0
8	-	(Backrest Cushions) [2172/2173/2174/2175/2176/2177/2178]	1


(Support for Backrest Frame)

1571553 no

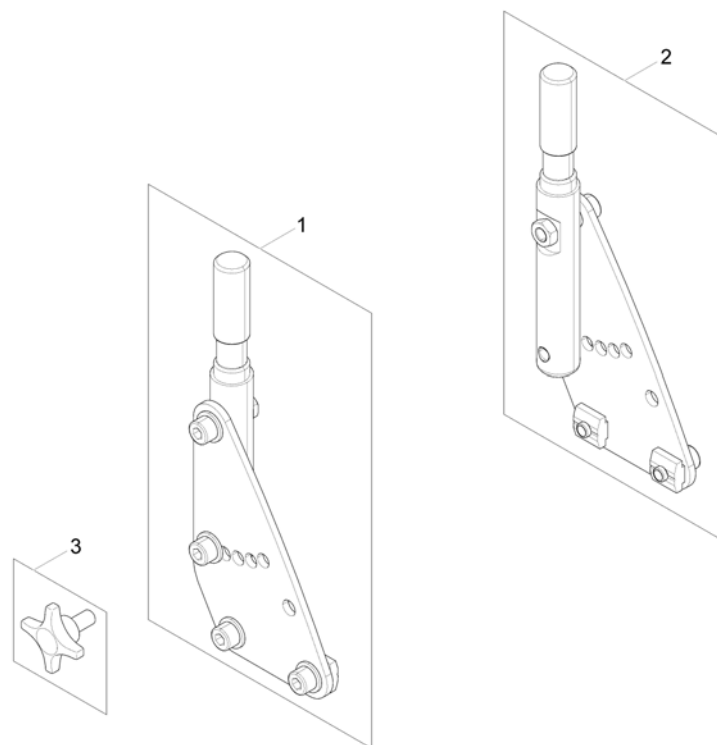

1	SP1571542	(Backrest Support RH)	1
2	SP1571553	(Backrest Support LH)	1
3	5824117	STJERNEGREP FESTE NY MANØVERBOKS MOOVER <i>(only swedish homologation)</i>	2


1571831.no

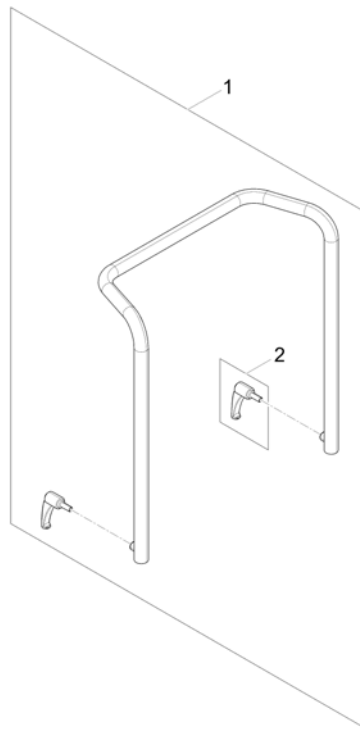


1	SP1567810	(Backrest Frame)	1
2	0021-70002	(Indexable lever 63mm M8x16)	1

(Sling Back, tension adjustable)

1571829.no

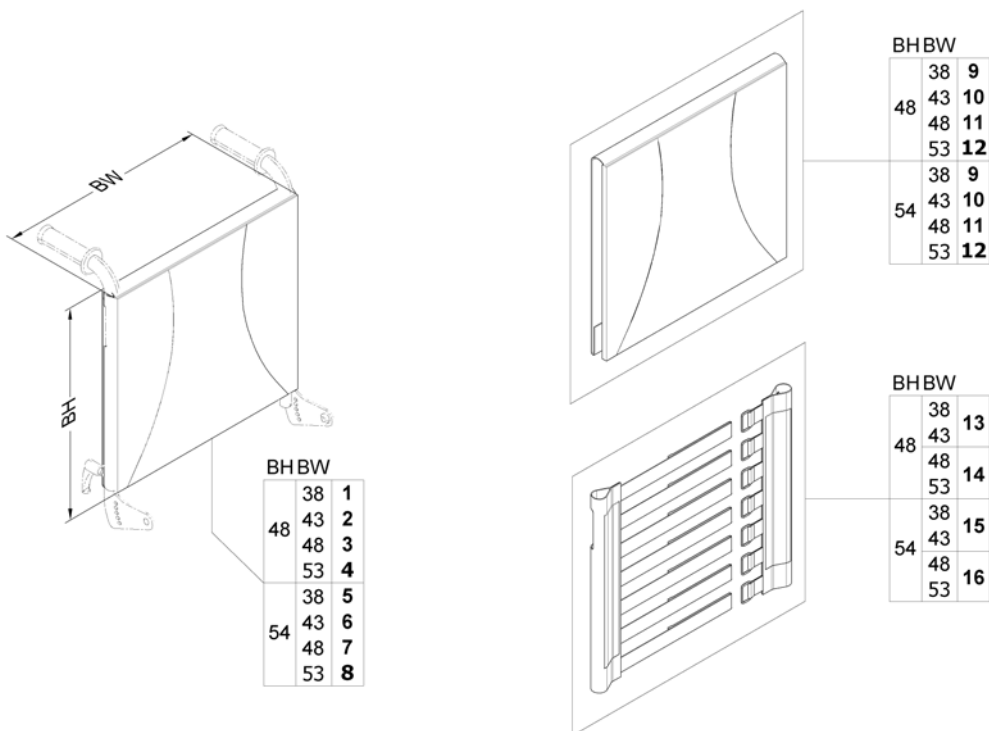


1	SP1567529	(Sling Back, tension adjustable & Upholstery, BW 380mm / BH 480mm, cpl.)	1
2	SP1567465	(Sling Back, tension adjustable & upholstery, BW 430mm / BH 480mm, cpl.)	1
3	SP1567530	RYGG VELCRO SB48 RH48 MODULITE	1
4	SP1567531	(Sling Back, tension adjustable & upholstery, BW 530mm / BH 480mm, cpl.)	1
5	SP1567627	(Sling Back, tension adjustable & upholstery, BW 380mm / BH 540mm, cpl.)	1
6	SP1567746	RYGG VELCRO SB43 RH43 MODULITE	1
7	SP1567747	RYGG VELCRO SB48 RH54 MODULITE	1
8	SP1567748	(Sling Back, tension adjustable & upholstery, BW 530mm / BH 540mm, cpl.)	1
9	SP1563563	(Backrest upholstery)	1
10	SP1563653	(Backrest upholstery)	1
11	SP1563654	(Backrest upholstery)	1
12	SP1563655	(Backrest upholstery)	1
13	SP1571492	(Sling back)	1
14	SP1571493	(Sling back)	1
15	SP1571494	(Sling back)	1
16	SP1571510	(Sling back)	1

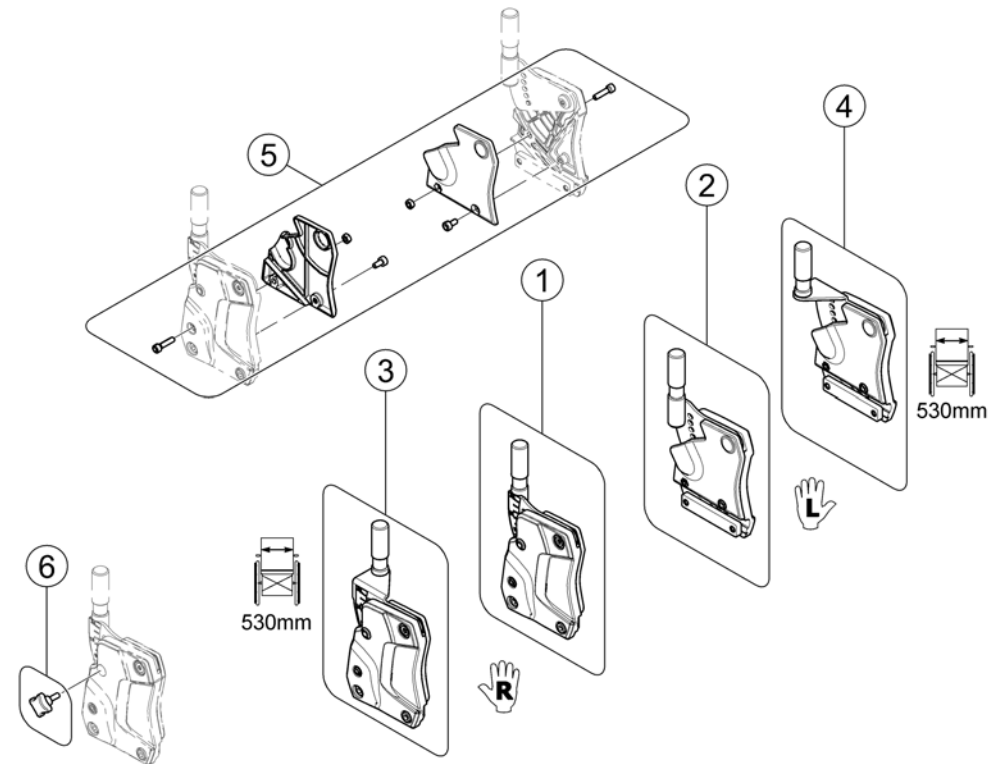


1	SP1582709	(Backrest Support - RH) Compatible with Flip-Up & Following Armrest [2193/2192/2194] - replaces SP1571560	1
2	SP1582708	(Backrest Support - LH) Compatible with Flip-Up & Following Armrest [2193/2192/2194] - replaces SP1571561	1
3	SP1582952	(Backrest Support SW 530mm - RH) Compatible with Flip-Up & Following Armrest [2193/2192/2194] - replaces SP1571568	1
4	SP1582951	(Backrest Support SW 530mm - LH) Compatible with Flip-Up & Following Armrest [2193/2192/2194] - replaces SP1571569	1
5	SP1571676	(Cover)	1
6	SP1579877	(Hand Screw M6) (Swedish Homologation only!)	2

(Backposts)

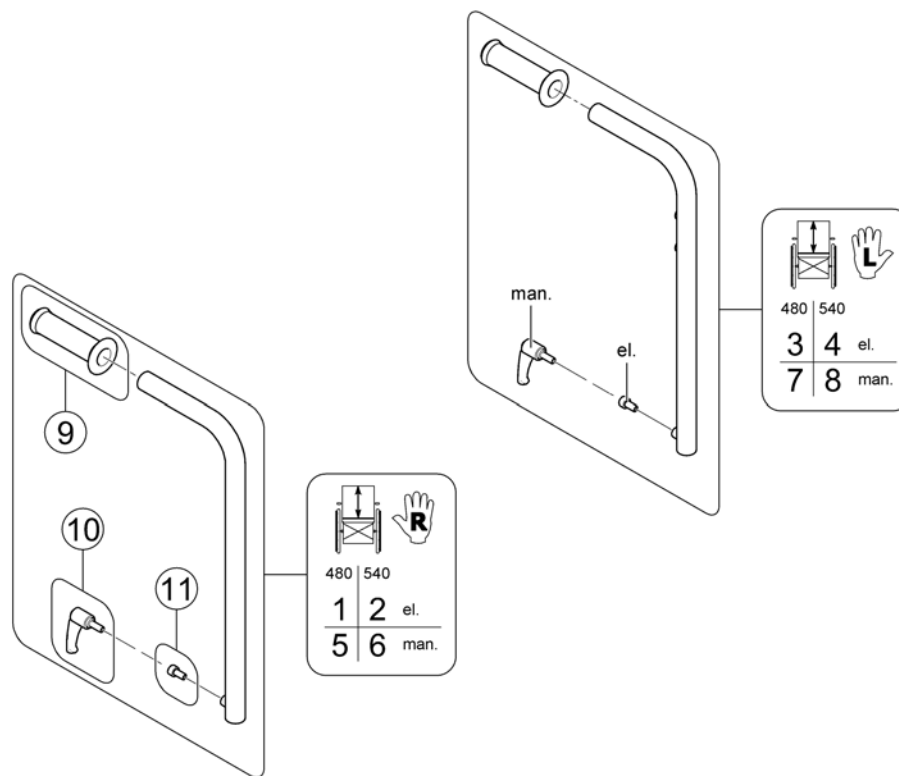


1	SP1587177	(Backpost el. BH 480mm - RH) <i>Compatible with Flip-Up & Following Armrest [2193/2192/2194] - replaces SP1563102</i>	1
2	SP1587184	(Backpost el. BH 540mm - RH) <i>Compatible with Flip-Up & Following Armrest [2193/2192/2194] - replaces SP1563103</i>	1
3	SP1587176	RYGGRØR HØ 48 VE MODULIT STORM4 <i>Compatible with Flip-Up & Following Armrest [2193/2192/2194] - replaces SP1560576</i>	1
4	SP1587183	(Backpost el. BH 540mm - LH) <i>Compatible with Flip-Up & Following Armrest [2193/2192/2194] - replaces SP1559578</i>	1
5	SP1587163	(Backpost man. BH 480mm - RH) <i>Compatible with Flip-Up & Following Armrest [2193/2192/2194] - replaces SP1572489</i>	1
6	SP1587182	(Backpost man. BH 540mm - RH) <i>Compatible with Flip-Up & Following Armrest [2193/2192/2194] - replaces SP1572490</i>	1
7	SP1587162	(Backpost man. BH 480mm - LH) <i>Compatible with Flip-Up & Following Armrest [2193/2192/2194] - replaces SP1572491</i>	1
8	SP1587178	(Backpost man. BH 540mm - LH) <i>Compatible with Flip-Up & Following Armrest [2193/2192/2194] - replaces SP1572492</i>	1
9	SP1563651	(Handles (pair))	1
10	0021-70002	(Indexable lever 63mm M8x16)	1
11	1567999	(Bolt M8x16 8.8 black DIN912)	1

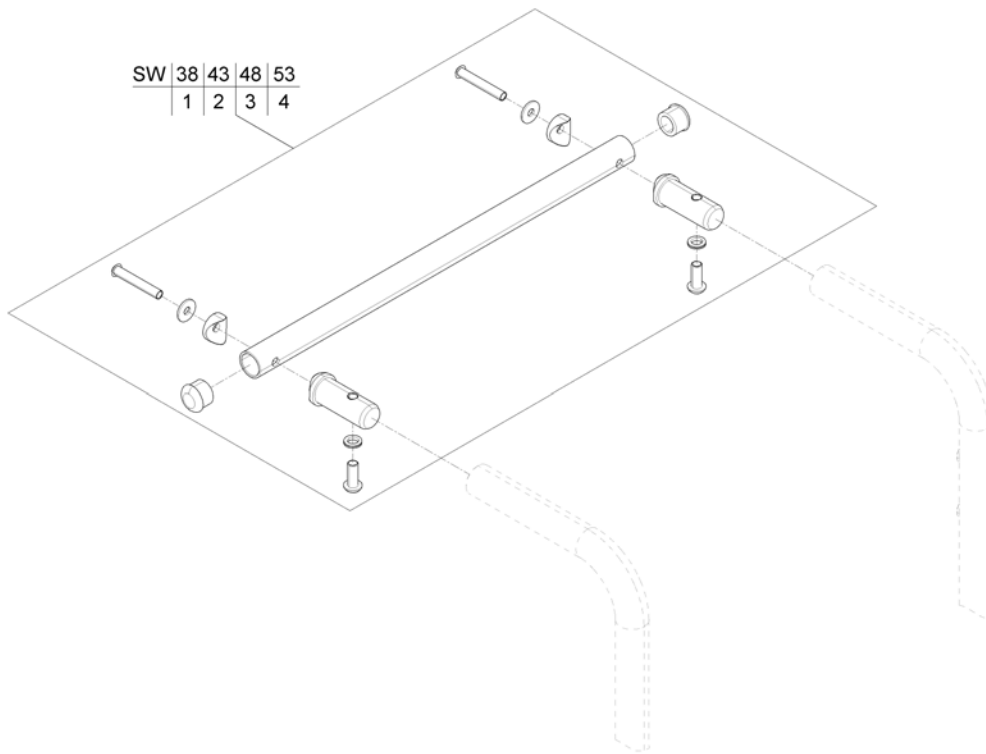


1	SP1570370	(Connection tube)	1
2	SP1570371	KJØREBØYLE SB43 MODULITE STORM4	1
3	SP1570372	BRAKETT RYGG SB48 LS1 MODULITE	1
4	SP1570373	(Connection tube)	1

(Back Recliner / Stabilization bracket)

1571097.kno

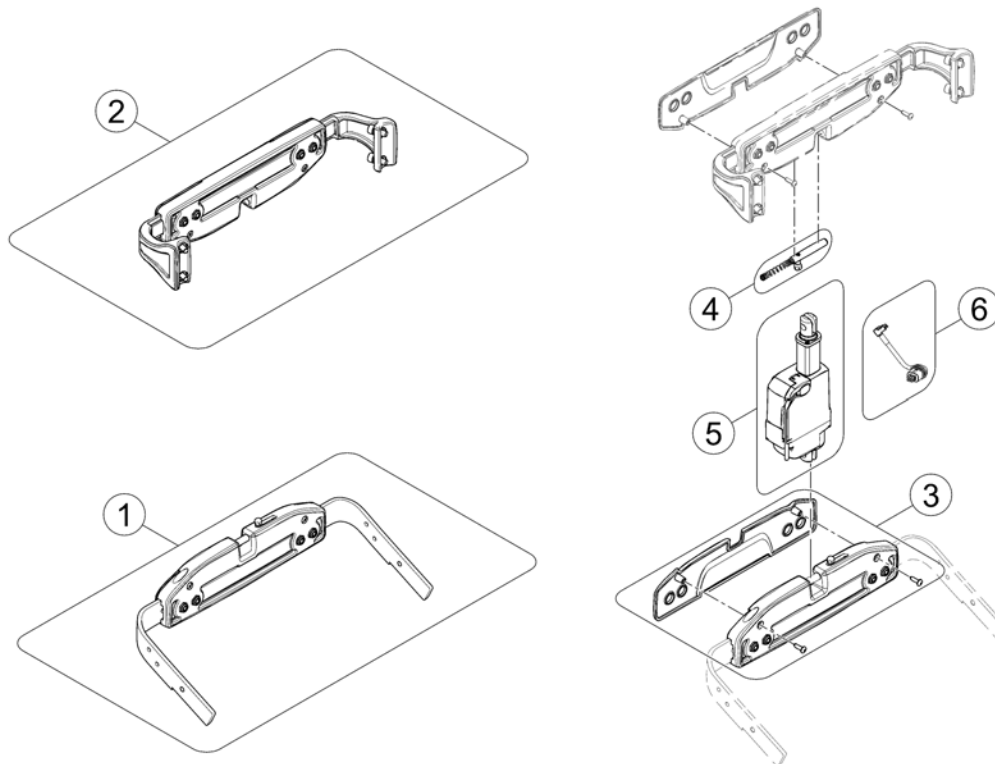


1	SP1567169	(Actuator bracket (pre set to SW 43cm))	1
2	SP1567168	BRACKET STABILISATOR RYG STORM4 MODULITE	1
3	SP1571702	(Cover)	1
4	SP1571704	(Locking bolt)	1
5	SP1557820	(Actuator)	1
6	SP1572166	(Actuator Cable LA23 ACS2 (~950mm))	1

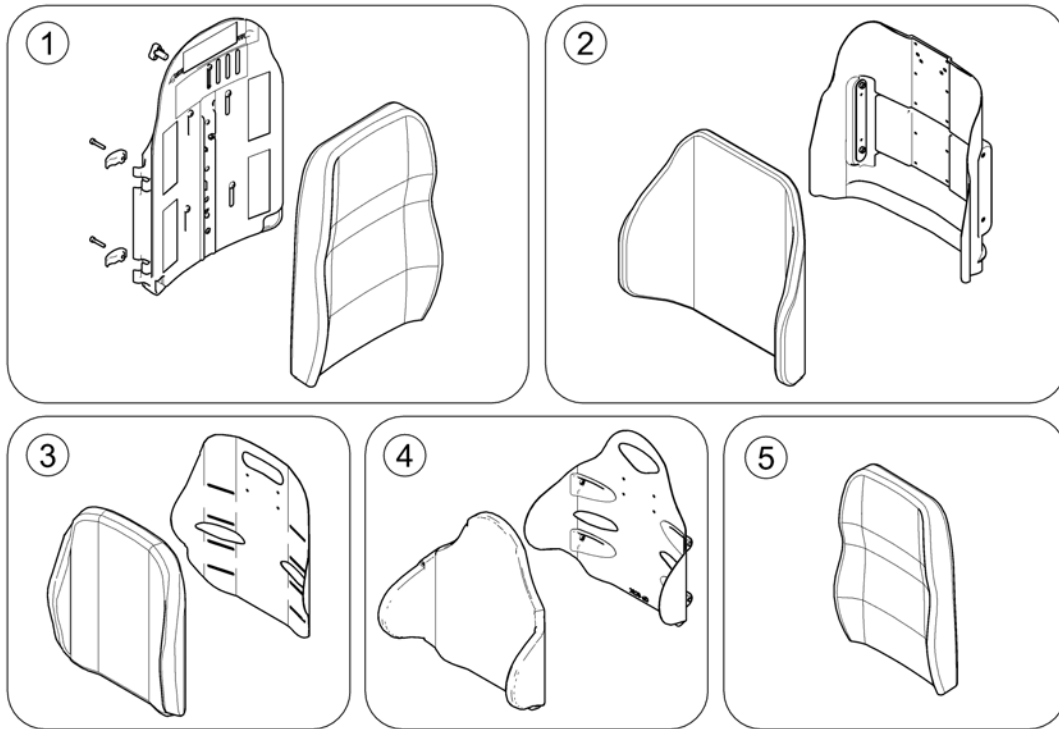

1	-	(Flex 3 / Laguna)		1
2	-	(Matrx Posture Back)	01.07.2015 →	1
3	-	(Matrx Elite T/R Back)	01.07.2015 →	1
4	-	(Matrx Elite Deep Back)	01.07.2015 →	1
5	-	(Matrx Elite Full Back)	01.07.2015 →	1

(Flex 3 / Laguna)

1573106.no

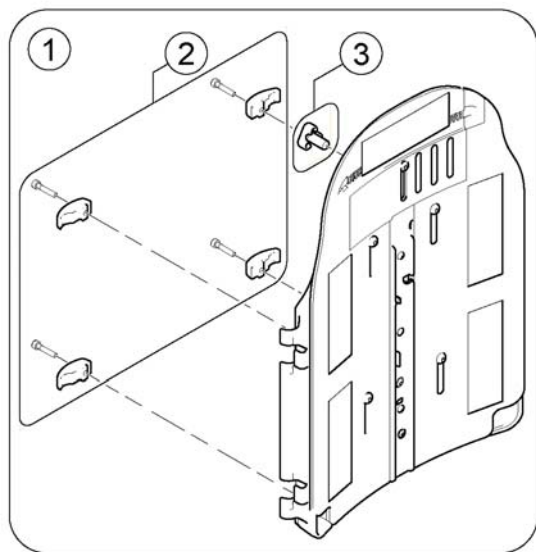


Flex 3

Laguna

4	380mm	black
5	430mm	
6	480mm	
7	530mm	
8	380mm	Dartex black
9	430mm	
10	480mm	
11	530mm	Plush black
12	380mm	
13	430mm	
14	480mm	
15	530mm	black orange seam
16	380mm	
17	430mm	
18	480mm	
19	530mm	

1	SP1568489	FLEX3 RYGG TIL MODULITE STORM4 TDX [2172]	1
2	SP1572321	(Clamps)	1
3	1497411	(HANDWHEEL M12 LONG NECK/HEADREST)	1
4	SP1529585	RYGGPUTE EASYADAPT SB38 STORM4 M SORT [2173+2151]	1
5	SP1529586	RYGGPUTE EASYADAPT SB43 STORM4 STD SORT [2173+2151]	1
6	SP1529587	RYGGPUTE EASYADAPT SB48 STORM4 SORT [2173+2151]	1
7	SP1529596	RYGGPUTE EASYADAPT SB53 STORM4 M SORT [2173+2151]	1
8	SP1529538-TR26	(Backrest Cushion Laguna / EasyAdapt, Dartex black - SW 380mm) [2173+2152]	1
9	SP1529539-TR26	(Backrest Cushion Laguna / EasyAdapt, Dartex black - SW 430mm) [2173+2152]	1
10	SP1529540-TR26	(Backrest Cushion Laguna / EasyAdapt, Dartex black - SW 480mm) [2173+2152]	1
11	SP1529545-TR26	(Backrest Cushion Laguna / EasyAdapt, Dartex black - SW 530mm) [2173+2152]	1
12	SP1529538-TR35	(Backrest Cushion Laguna, Plush black - SW 380mm) [2173+2154]	1
13	SP1529539-TR35	(Backrest Cushion Laguna, Plush black - SW 430mm) [2173+2154]	1
14	SP1529540-TR35	(Backrest Cushion Laguna, Plush black - SW 480mm) [2173+2154]	1
15	SP1529545-TR35	(Backrest Cushion Laguna, Plush black - SW 530mm) [2173+2154]	1
16	SP1529538	RYGGPUTE LAGUNA 37CM SORT M ORANG SØM [2173+2153]	1
17	SP1529539	RYGGPUTE LAGUNA 43CM SORT M ORANG SØM [2173+2153]	1
18	SP1529540	RYGGPUTE LAGUNA 48CM SORT M ORANG SØM [2173+2153]	1
19	SP1529545	RYGGPUTE LAGUNA 50CM SORT M ORANG SØM [2173+2153]	1

Posture Back

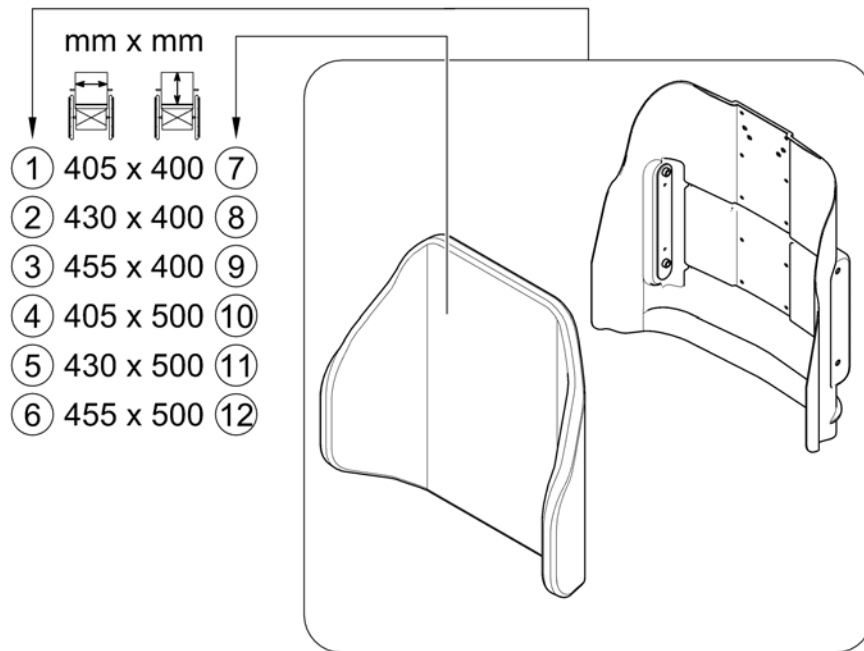


1	SP1531762	PB Back Cushion 40x40cm <i>[2174]</i>	1
2	SP1531792	PB Back Cushion 43x40cm <i>[2174]</i>	1
3	SP1531822	PB Back Cushion 45x40cm <i>[2174]</i>	1
4	SP1531763	PB Back Cushion 40x50cm <i>[2175]</i>	1
5	SP1531793	RYGG POSTURE BACK SB43 RH50 <i>[2175]</i>	1
6	SP1531823	POSTURE BACK 45x50cm MATRIX <i>[2175]</i>	1
7	SPTRS0139-1616	(Backrest Cushion - Posture Back 405x400mm (16x16"))	1
8	SPTRS0139-1716	(Backrest Cushion - Posture Back 430x400mm (17x16"))	1
9	SPTRS0139-1816	(Backrest Cushion - Posture Back 455x400mm (18x16"))	1
10	SPTRS0139-1620	(Backrest Cushion - Posture Back 405x500mm (16x20"))	1
11	SPTRS0139-1720	(Backrest Cushion - Posture Back 430x500mm (17x20"))	1
12	SPTRS0139-1820	(Backrest Cushion - Posture Back 455x500mm (18x20"))	1

(Matrx Elite T/R Back)


1588886 No

Elite T/R Back

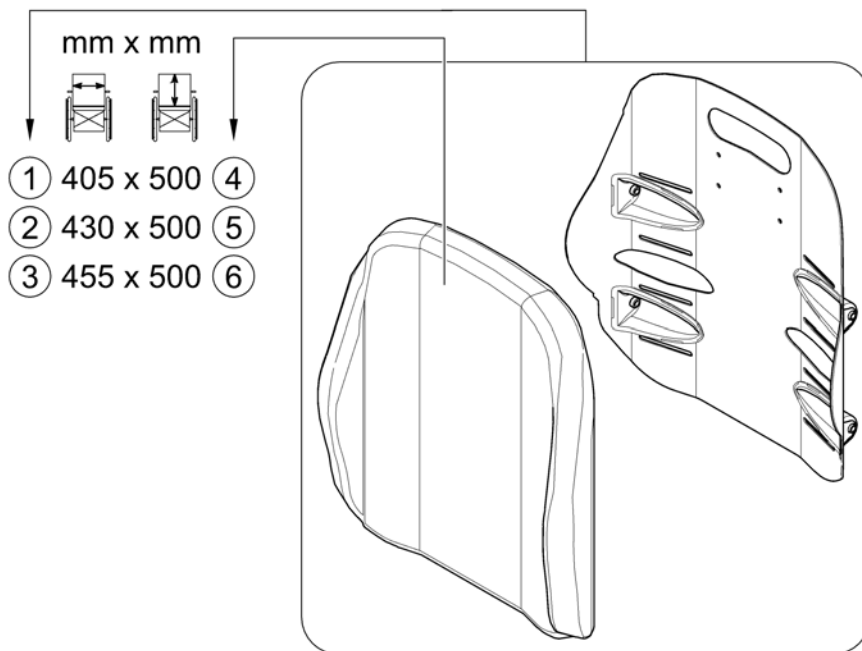

1	SP1588270	(Backrest, Matrx Elite T/R Back 16x20", cpl.) <i>[2177+2121]</i>	1
2	SP1588271	(Backrest, Matrx Elite T/R Back 17x20", cpl.) <i>[2177+2122]</i>	1
3	SP1588272	(Backrest, Matrx Elite T/R Back 18x20", cpl.) <i>[2177+2123]</i>	1
4	SPTRS0375-1620	(Backrest Cushion - Elite T/R Back 405x500mm (16x20"))	1
5	SPTRS0375-1720	(Backrest Cushion - Elite T/R Back 430x500mm (17x20"))	1
6	SPTRS0375-1820	(Backrest Cushion - Elite T/R Back 455x500mm (18x20"))	1

Elite Deep Back

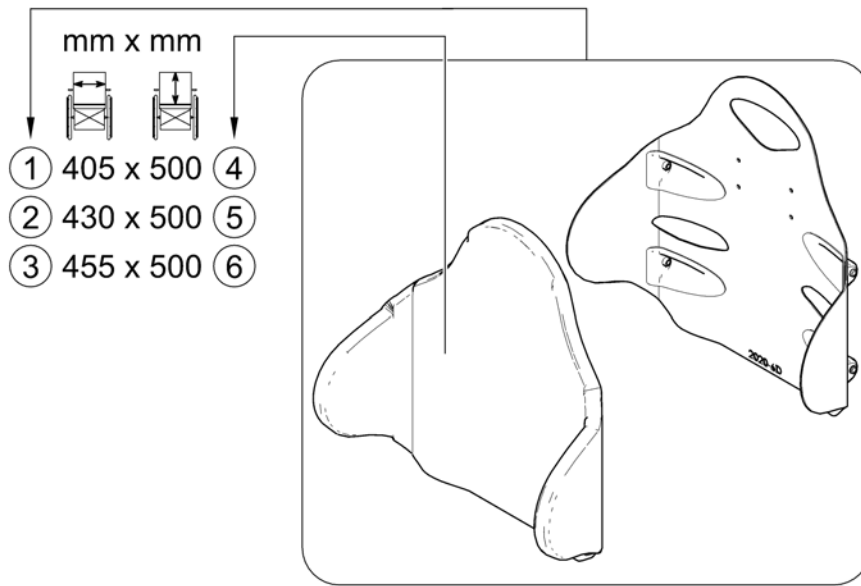


1	SP1588273	(Backrest, Matrx Elite Deep Back 16x20", cpl.) [2178+2121]	1
2	SP1588274	(Backrest, Matrx Elite Deep Back 17x20", cpl.) [2178+2122]	1
3	SP1588275	(Backrest, Matrx Elite Deep Back 18x20", cpl.) [2178+2123]	1
4	SPTRS0344-1620	(Backrest Cushion - Elite Deep Back 405x500mm (16x20"))	1
5	SPTRS0344-1720	(Backrest Cushion - Elite Deep Back 430x500mm (17x20"))	1
6	SPTRS0344-1820	(Backrest Cushion - Elite Deep Back 455x500mm (18x20"))	1


(Matrx Elite Full Back)

1588714.no

Elite Full Back

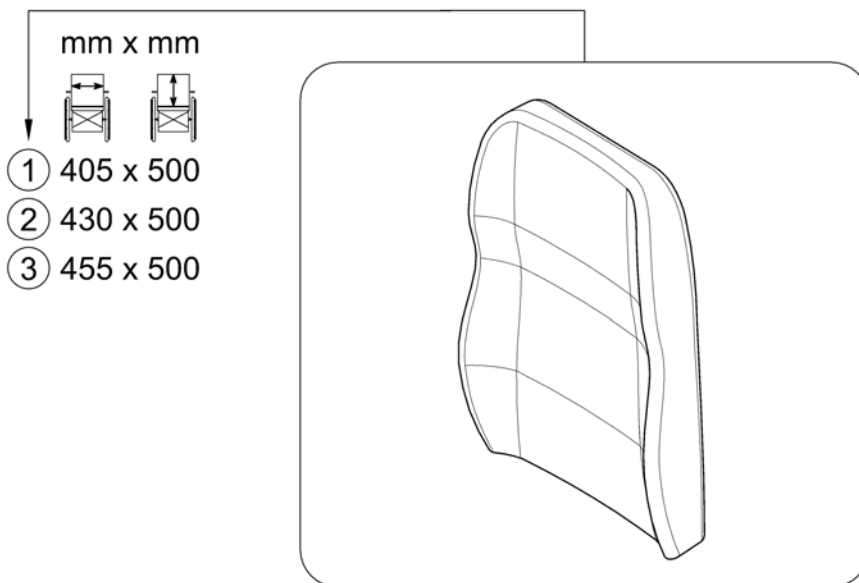


1	SP1531824	Back Cushion 40x50cm <i>[2176]</i>	1
2	SP1531831	FULL BACK 43x50cm MATRIX <i>[2176]</i>	1
3	SP1531832	Back Cushion 45x50cm <i>[2176]</i>	1

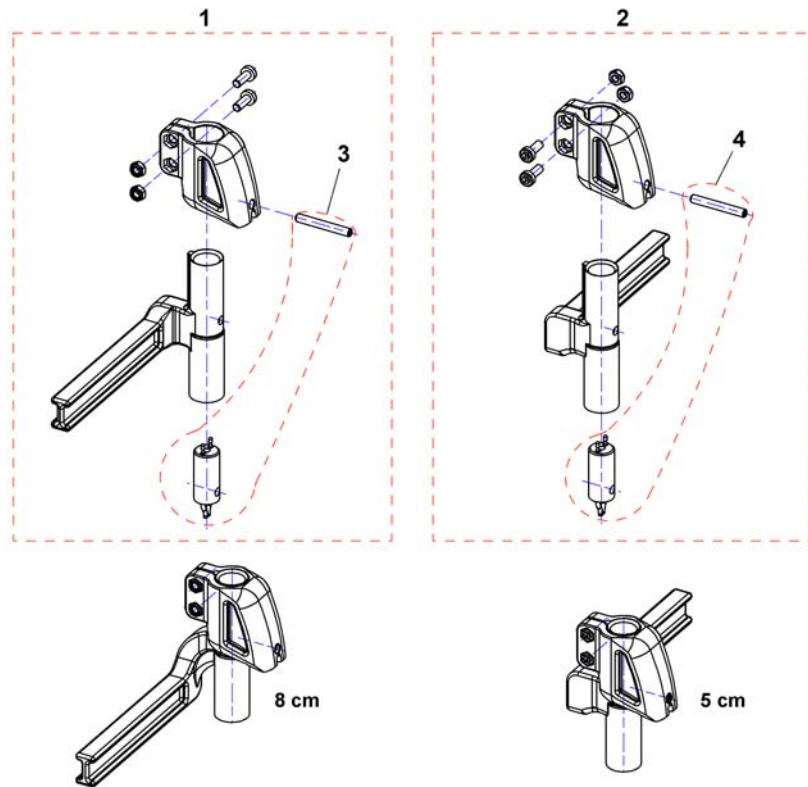


1	SP1449733	BENSTØTTEHOLDER MAN VE 5CM STORM	1
1	SP1449737	BENSTØTTEHOLDER MAN VE 8CM STORM TDX BORA	1
1	SP1449735	BENSTØTTEHOLDER EL VE 5CM STORM3 TO TY 010705	1
1	SP1449739	BENSTØTTEHOLDER EL VE 8CM ST03 TO TY NY 010705	1
1	SP1529951	BENSTHOLD EL 5CM VE TDX <i>For TDX-SP with lifter</i>	1
1	SP1535937	BENSTHOLD EL 8CM VE TDX <i>For TDX-SP with lifter</i>	1
2	SP1449732	BENSTØTTEHOLDER MAN HØ 5CM STORM	1
2	SP1449736	BENSTØTTEHOLDER MAN HØ 8CM STORM TDX BORA	1
2	SP1449314	BENSTØTTEHOLDER EL HØ 5CM STORM3 NY 010705	1
2	SP1449315	BENSTØTTEHOLDER EL HØ 8CM STORM3 NY 010705	1
2	SP1529949	BENSTHOLDER 5CM HØ TDX <i>For TDX-SP with lifter</i>	1
2	SP1535856	BENSTHOLD EL 8CM HØ TDX <i>For TDX-SP with lifter</i>	1
3	SP1450228	BENST FESTEDEL TDX	1
3	SP1527775	KABEL HOLDER BENST 65MM TDX <i>For TDX-SP with lifter</i>	1
4	SP1450232	KABEL EL BENST KORT STORM3	1
4	SP1527775	KABEL HOLDER BENST 65MM TDX <i>For TDX-SP with lifter</i>	1
-	SP1527899	(Extension Cable for powered Legrest) <i>For TDX-SP without lifter</i>	1

(Hanger Vari A (04/2009 =>))

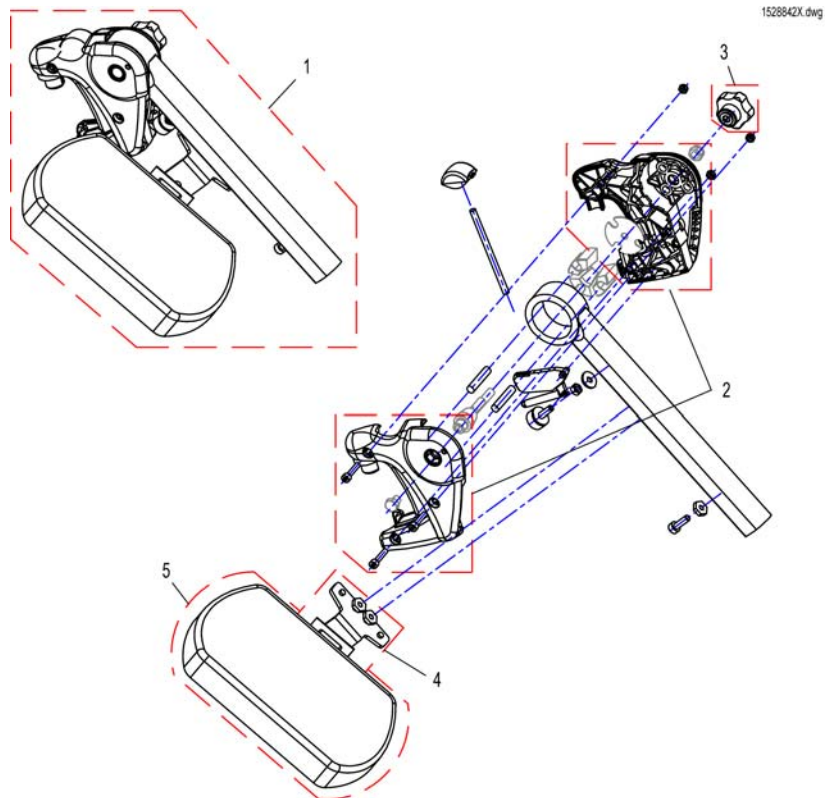


-	SP1527820	(Rework-Kit Vari A Legrest Version 1 => Version 2) <i>Vari A Version 1 => Version 2 Upgrade Kit</i>	1
1	SP1527498	BENSTØTTE OVERDEL VE MAN VREG LKOMP FRA 07/2005	1
1	SP1527492	BENST OVERDEL MAN HØ STORM4	1
2	SP1434955	(Cover Legrest cpl.)	1
3	L34011	(Handknob M8 Tristarfemale)	1
4	SP1450447	LEGGPLATE KPL VE NY MAI 05	1
4	SP1450446	LEGGPLATE KPL HØ NY MAI 05	1
5	6301333	LEGGPLATE EL OG MAN VREG BENST STORM/TWISTER	1

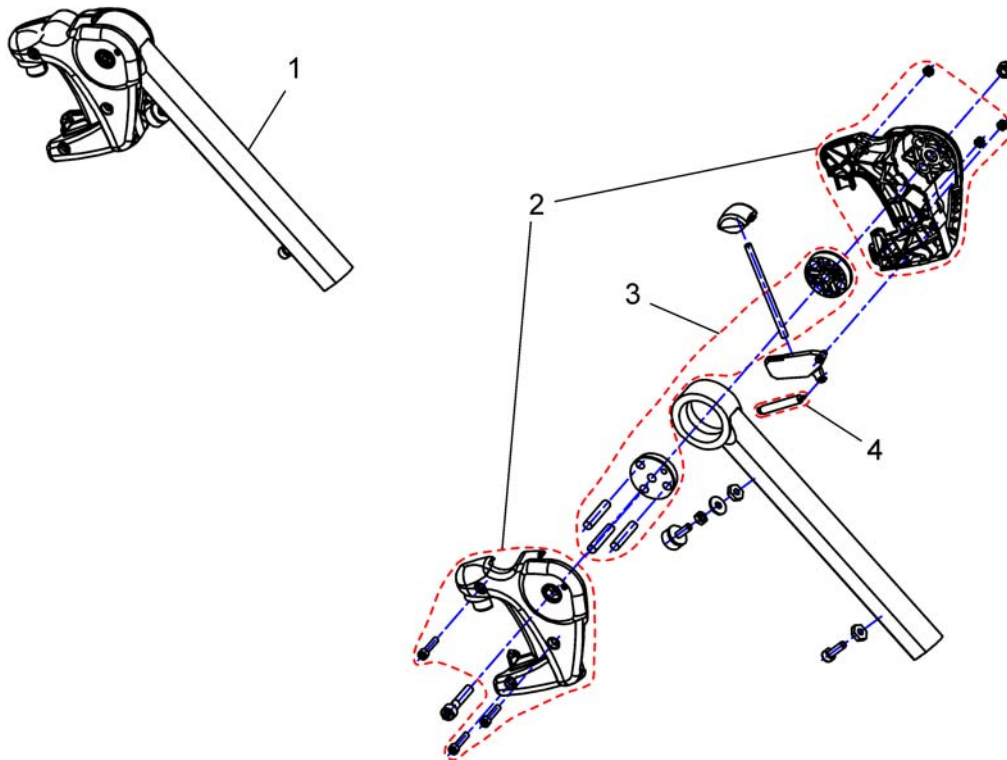

1	SP1449347	BENSTØTTE OVERDEL VARI F STORM TDX BORA VE	1
1	SP1449346	BENSTØTTE OVERDEL VARI F STORM TDX BORA HØ	1
2	SP1434955	(Cover Legrest cpl.)	1
3	1434956	(Mechanic)	1
4	SPZ173178	FJÆR STORM4 BORA	1

(Hangers ADM LH/RH manual)


1449409X.dwg

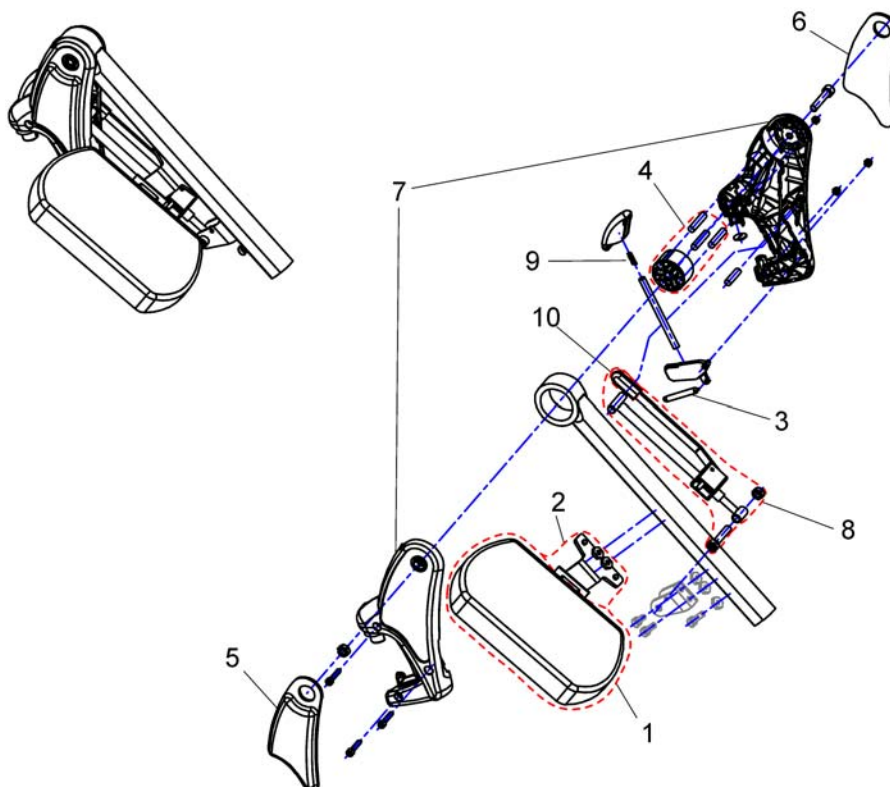

1	6301333	LEGGPLATE EL OG MAN VREG BENST STORM/TWISTER	1
2	SP1450447	LEGGPLATE KPL VE NY MAI 05	1
2	SP1450446	LEGGPLATE KPL HØ NY MAI 05	1
3	SPZ173178	FJÆR STORM4 BORA	1
4	SPZ174134	(Floating Cylinder With Pin)	1
5	SP1450410	KNEPOLSTER BENST EL VE 07.2005->	1
6	SP1450448	KNEPOLSTER BENSTG EL HØ 07.2005->	1
7	SP1449293	BENST HOLDER OVERDEL PAR EL VREG 07.2005->	1
8	SPZ174130	BENST ERGO REGULLERINGSMEK KPL	1
9	SPN12196	FJÆR STORM4 BORA	1
10	SP1006490	(Cap for tripping lever)	1
11	SP1449410	BENSTØTTE OVERDEL ADM HØ STORM4 MAN VREG	1
11	SP1449409	BENSTØTTE OVERDEL ADM VE STORM4 MAN VREG	1

(Hanger Vari ADE/ADP electric (07/2005 =>))


1449407X.dwg

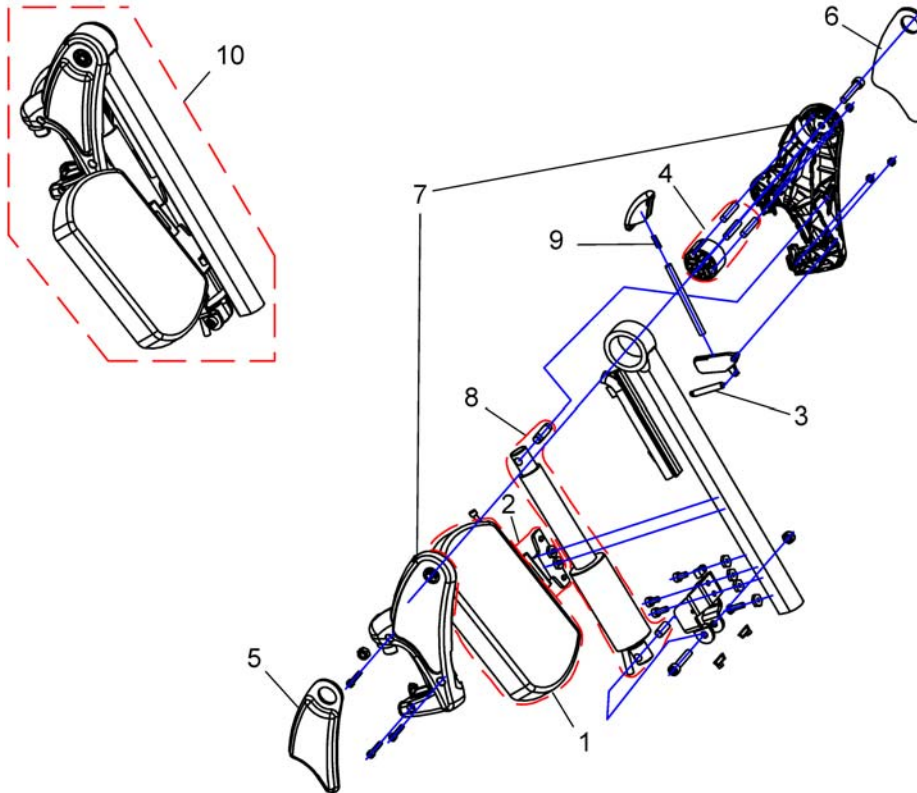


1	6301333	LEGGPLATE EL OG MAN VREG BENST STORM/TWISTER	1
2	SP1450447	LEGGPLATE KPL VE NY MAI 05	1
2	SP1450446	LEGGPLATE KPL HØ NY MAI 05	1
3	SPZ173178	FJÆR STORM4 BORA	1
4	SPZ174134	(Floating Cylinder With Pin)	1
5	SP1450448	KNEPOLSTER BENSTG EL HØ 07.2005->	1
6	SP1450410	KNEPOLSTER BENST EL VE 07.2005->	1
7	SP1449293	BENST HOLDER OVERDEL PAR EL VREG 07.2005->	1
8	SPZ151006	MOTOR EL BENST STORM4	1
9	SPN12196	FJÆR STORM4 BORA	1
10	SP1449408	BENST OVERDEL ADP HØ STORM4	1
10	SP1449407	BENST OVERDEL ADP VE STORM4	1

(Central Legrest, manual - Standard- / Easy Adapt-Seat)

1537926X.iso

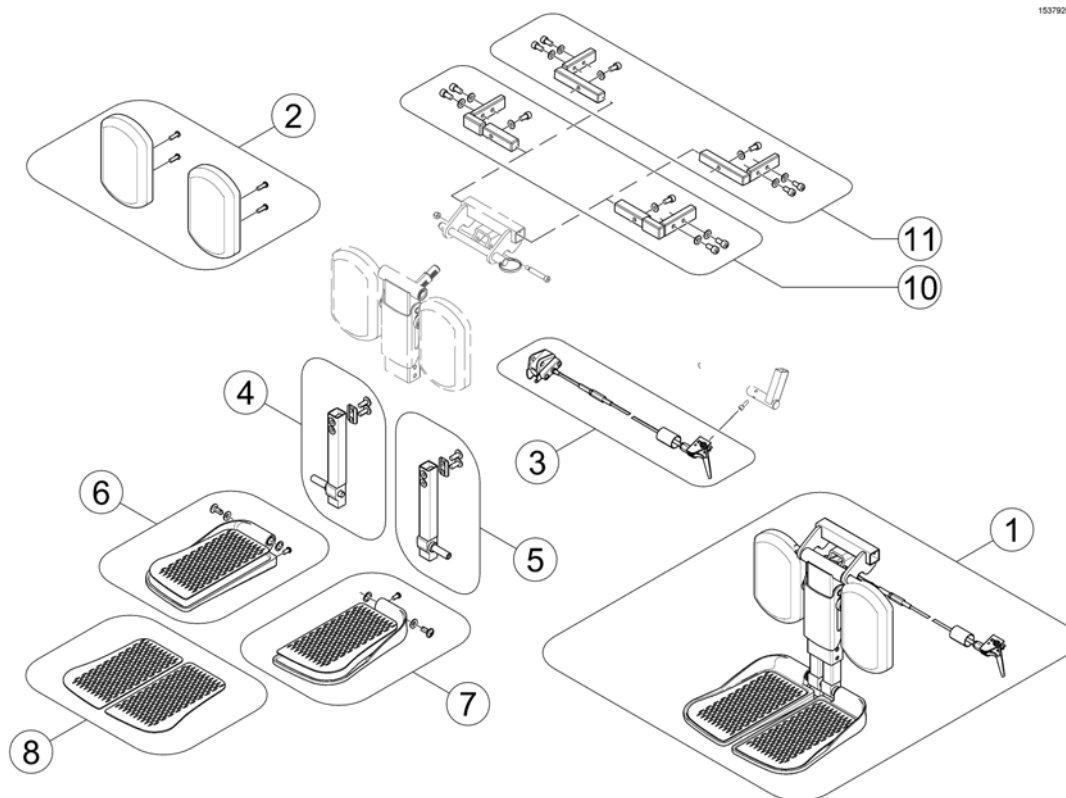


1	SP1537926	(Central Legrest manual, cpl.)	1
2	SP1163373	LEGGPUTER TDX (PAR)	1
3	SP1121079	(Bowden cable central legrest, manual)	1
4	SP1161986	(Kit, extension tube, cpl. RH)	1
5	SP1161987	(Kit, extension tube, cpl. LH)	1
6	SP1161990	(Footplate, cpl. RH)	1
7	SP1161991	(Footplate, cpl. LH)	1
8	SP1163375	(Footrest pad (Pair))	1
10	SP1541397	(Support center mounted legrest 430-530 mm)	1
11	SP1541398	(Support center mounted legrest 380-410 mm)	1

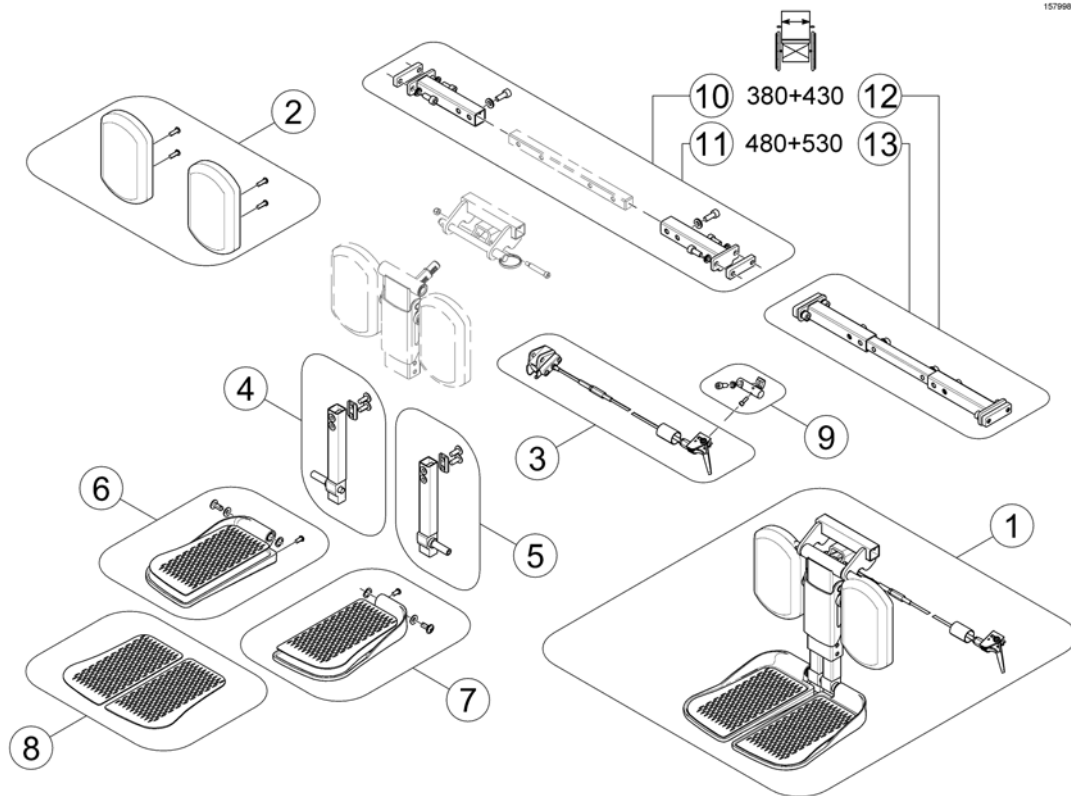(Central Legrest, manual - Modulte Seat)

1	SP1537926	(Central Legrest manual, cpl.)	1
2	SP1163373	LEGGPUTER TDX (PAR)	1
3	SP1121079	(Bowden cable central legrest, manual)	1
4	SP1161986	(Kit, extension tube, cpl. RH)	1
5	SP1161987	(Kit, extension tube, cpl. LH)	1
6	SP1161990	(Footplate, cpl. RH)	1
7	SP1161991	(Footplate, cpl. LH)	1
8	SP1163375	(Footrest pad (Pair))	1
9	SP1579988	(Support, Lever, cpl.)	1
10	SP1579989	(Support, Centr. Legrest man. LNX - SW 380+430mm)	1
11	SP1579990	(Support, Centr. Legrest man. LNX - SW 480+530mm)	1
12	SP1579991	(Support, Centr. Legrest man. LNX - SW 380+430mm, cpl.)	1
13	SP1579992	(Support, Centr. Legrest man. LNX - SW 480+530mm, cpl.)	1

(Central Legrest LNX electr. - Standard- / Easy Adapt-Seat)


1543031X.iss

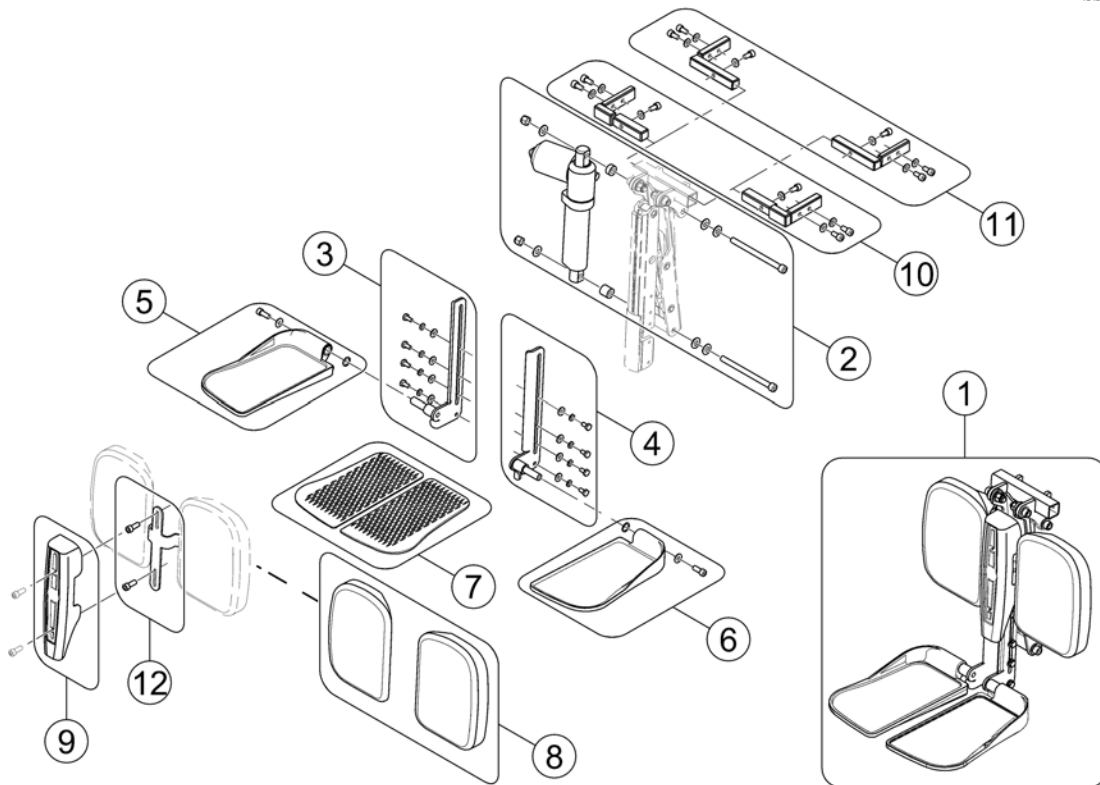

-	SP1541677	(Adaptercable 300mm) <i>For chairs with lifter</i>	1
-	SP1543012	(Adaptercable 1200mm) <i>For chairs without lifter</i>	1
1	SP1537489	(Centre Mounted Legrest LNX (Standard / Easy Adapt) complete)	1
2	SP1543031	(Actuator, complete)	1
3	SP1543035	(Footplate mount, RH)	1
4	SP1543034	(Footplate mount, LH)	1
5	SP1543037	(Footplate, RH)	1
6	SP1543036	(Footplate, LH)	1
7	SP1163375	(Footrest pad (Pair))	1
8	SP1543033	(Calfplates (Pair))	1
9	SP1543032	(Cover)	1
10	SP1541397	(Support center mounted legrest 430-530 mm)	1
11	SP1541398	(Support center mounted legrest 380-410 mm)	1
12	SP1579985	(Calf Plate Bracket)	1


(Central Legrest LNX electr. - Modulite Seat)

1575133 iso

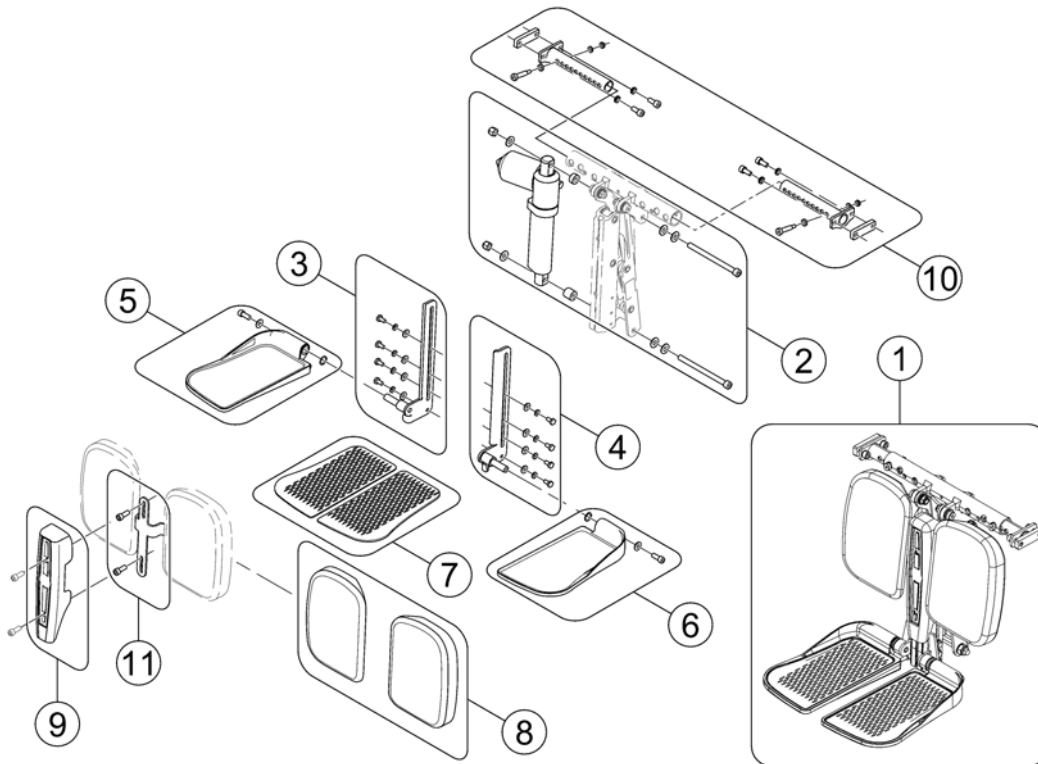

-	SP1541677	(Adaptercable 300mm) <i>For chairs with lifter</i>	1
-	SP1543012	(Adaptercable 1200mm) <i>For chairs without lifter</i>	1
1	SP1571959	BENSTØTTE SENTER LNX EL TDX SP2	1
2	SP1543031	(Actuator, complete)	1
3	SP1543035	(Footplate mount, RH)	1
4	SP1543034	(Footplate mount, LH)	1
5	SP1543037	(Footplate, RH)	1
6	SP1543036	(Footplate, LH)	1
7	SP1163375	(Footrest pad (Pair))	1
8	SP1543033	(Calfplates (Pair))	1
9	SP1543032	(Cover)	1
10	SP1575144	(Support, Centr. Legrest el. LNX - Modulite)	1
11	SP1579985	(Calf Plate Bracket)	1


(Centre-mounted Legrests)

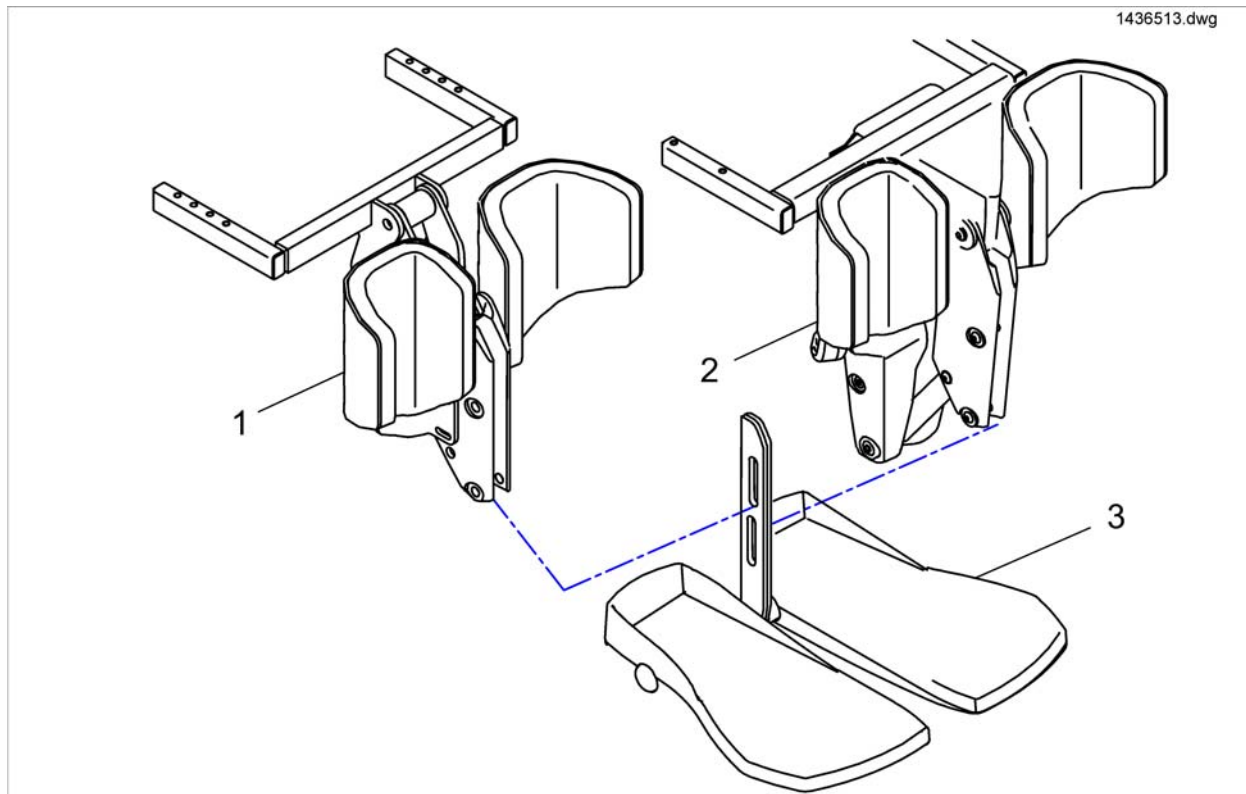

1	SP1436512	BENSTØTTE VREG SENTER MONT MAN	1
2	SP1440413	BENST SENTER V100 VREG EL TORNADO TYPHOON	1
3	SP1440292	FOTPLATE OPPFEL INNERØR 44-50 HØ/VE STORM4 T2	1
3	SP1440291	FOTPLATE OPPFELLB MEDIUM INNERRØR SENTER BENST	1
3	SP1440290	FOTPLATE OPPFELL INNERØR 32-38 HØ/VE STORM4 T2	1

(Bracket Centr. Legrest LNX - Narrow Base)


1541573X.iso

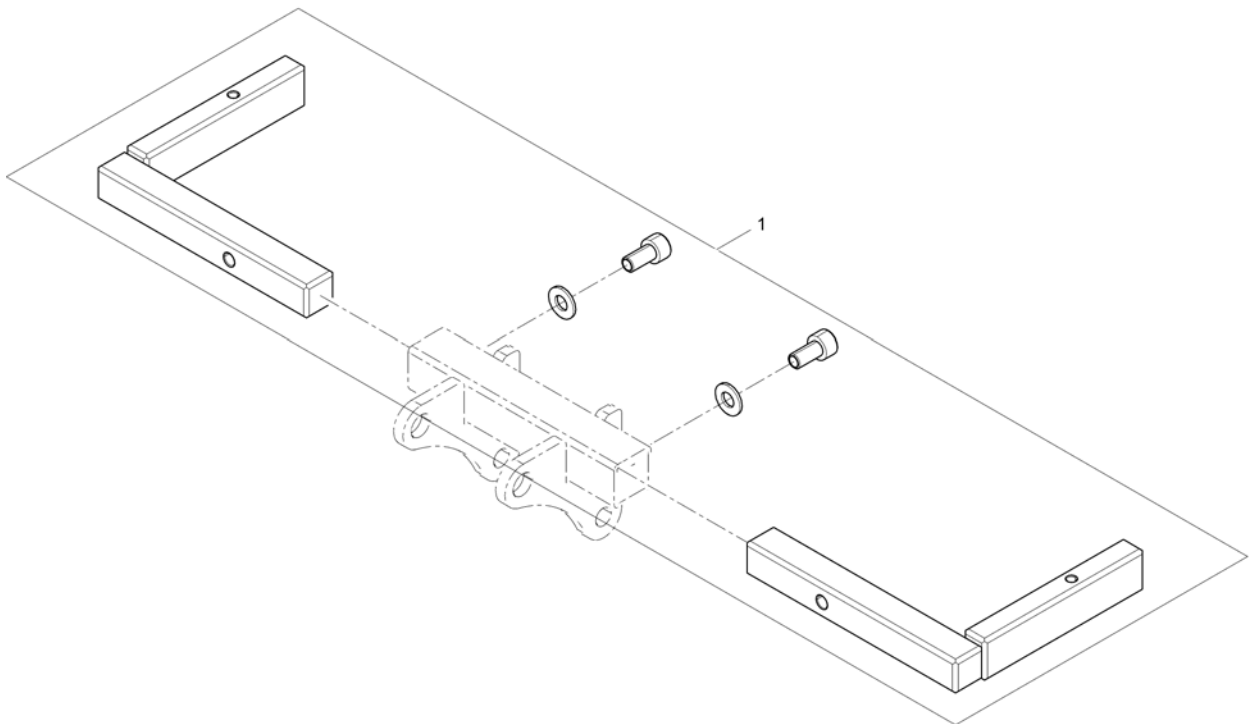

1	SP1543042	(Bracket central legrest LNX & manuell)	1
1	SP1554509	(Bracket central legrest LNX & manuell (Narrow base))	1


(Footrests Vari, Flip-Up (01.08.10 =>))

1529644X.jpg

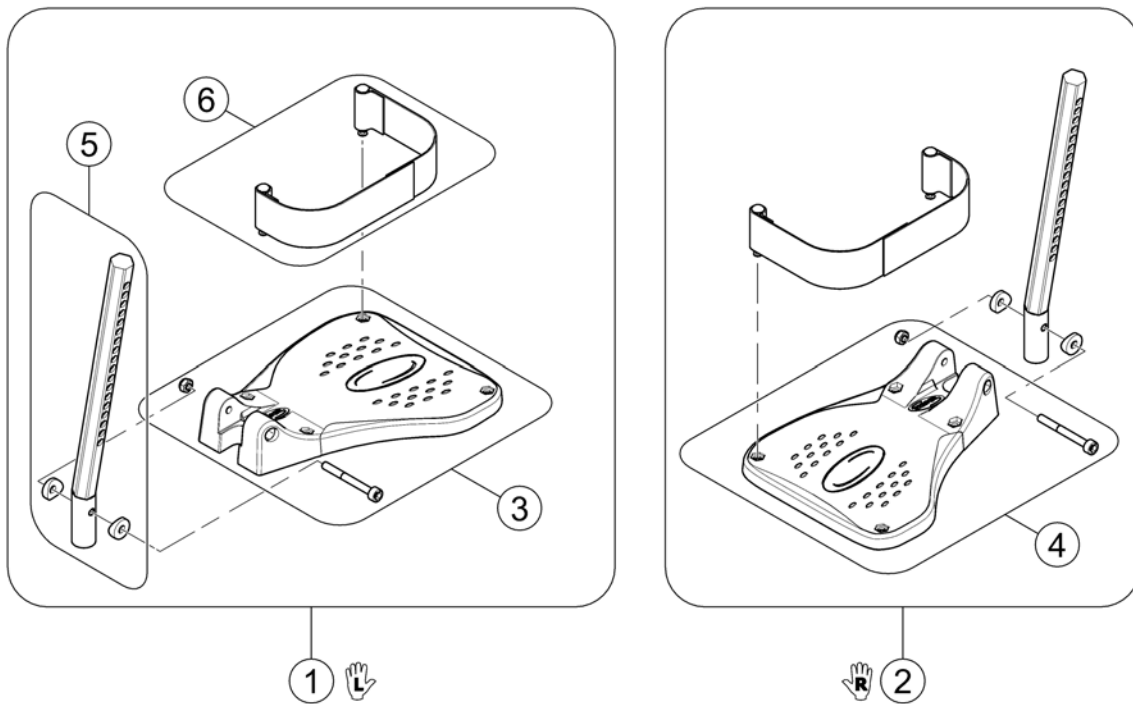


1	SP1541575	FORPLATE OPPFELLBAR KPL VENSTRE	1
2	SP1541574	FOTPLATE OPPFELLBAR KPL HØYRE	1
3	SP1529645	FOTPLATE KLP SB43-45 VE STORM4	1
4	SP1529644	FOTPLATE KLP SB43-45 HØ STORM4	1
5	SP1543045	(Plug-in tube LH/RH)	1
6	SP1442467	LEGGBÅND HELKAPPE PAR MIRAGE, STORM4, TDX	1

(Footrests Vari, Flip-Up & angle adjustable (01.08.10 =>))

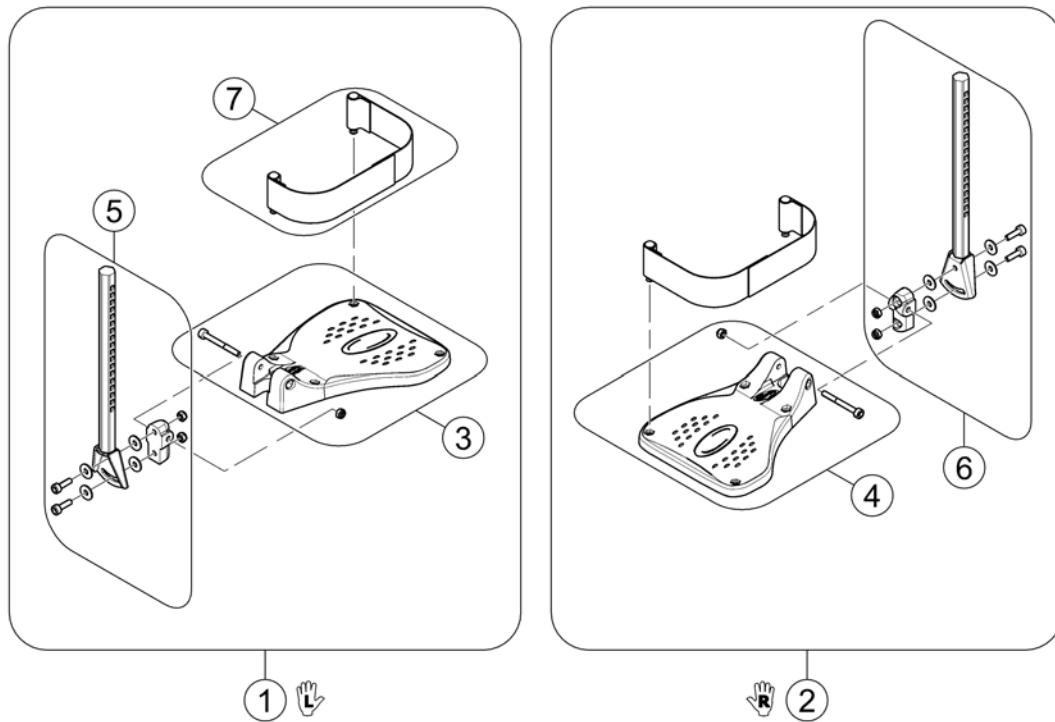


1	SP1541580	FOTPLATE M/INNERRØR KPL VE VREG	1
2	SP1541579	FOTPLATE M/INNERRØR KPL	1
3	SP1529645	FOTPLATE KLP SB43-45 VE STORM4	1
4	SP1529644	FOTPLATE KLP SB43-45 HØ STORM4	1
5	SP1423204	(FOOT PLATE TUBE, ANGLE ADJUSTMENT LH CPL.)	1
6	SP1423203	(FOOT PLATE TUBE, ANGLE ADJUSTMENT RH CPL.)	1
7	SP1442467	LEGGBÅND HELKAPPE PAR MIRAGE, STORM4, TDX	1

(Footplate)

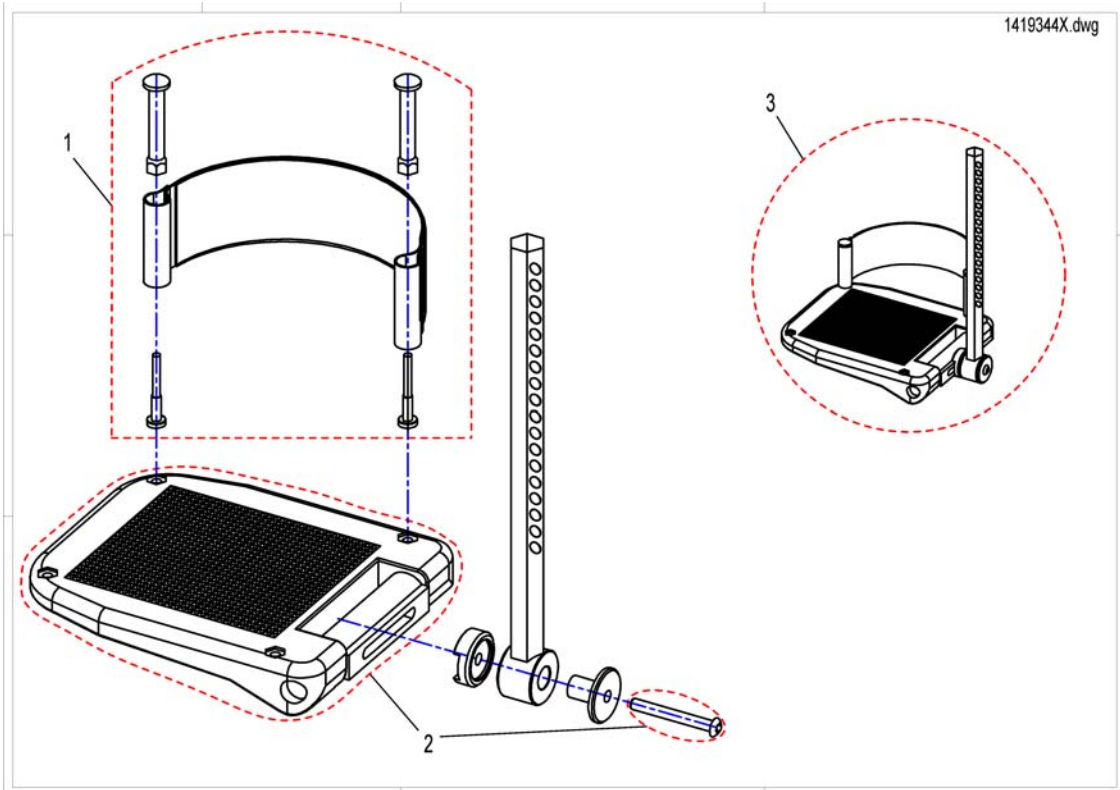


1	SP5321579	HÆLBÅND TDX	1
2	SP5106376	(Footplate)	1
3	SP1419344	FOTPLATE VE VREG OG DREG KPL MED INNERRØR	1
3	SP1419345	FOTPLATE HØ VREG OG DREG KPL MED INNERRØR	1

(Footplate with support tube fix RH/LH)

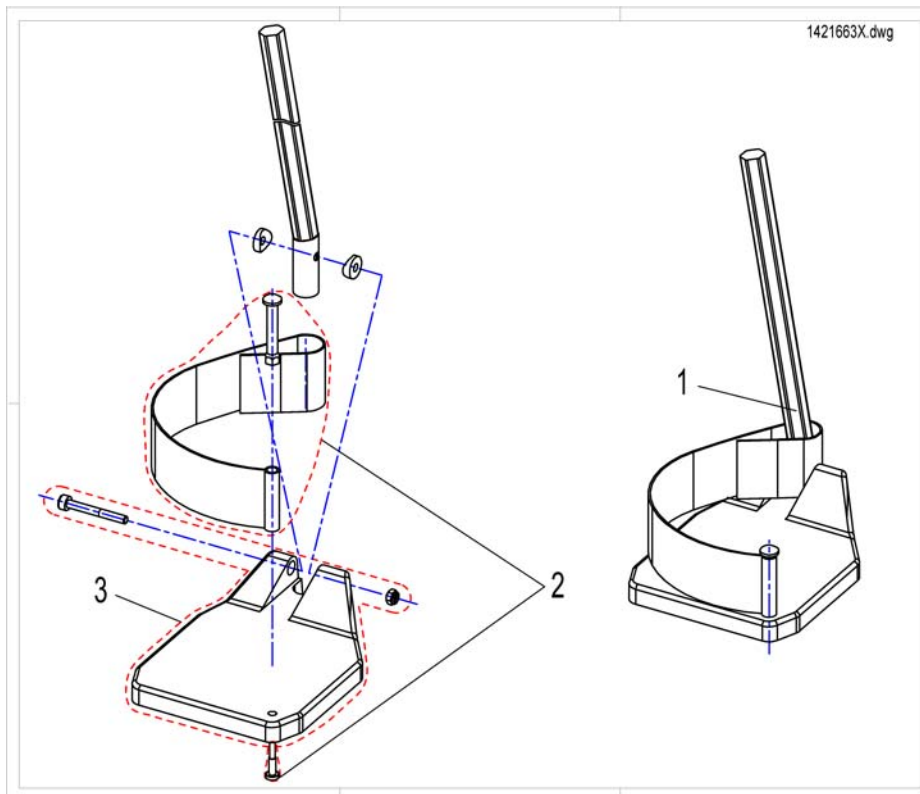

1	SP1421662	BENSTØTTEUNDERDEL HØ STORM3	1
1	SP1421663	BENSTØTTEUNDERDEL VE STORM3	1
2	SPB273011	HÆLBÅND BORA TDX	1
3	SPZ173217	(Footplate LH)	1
3	SPZ173218	(Footplate RH)	1


(Footplate with support tube angle adjust. RH/LH)

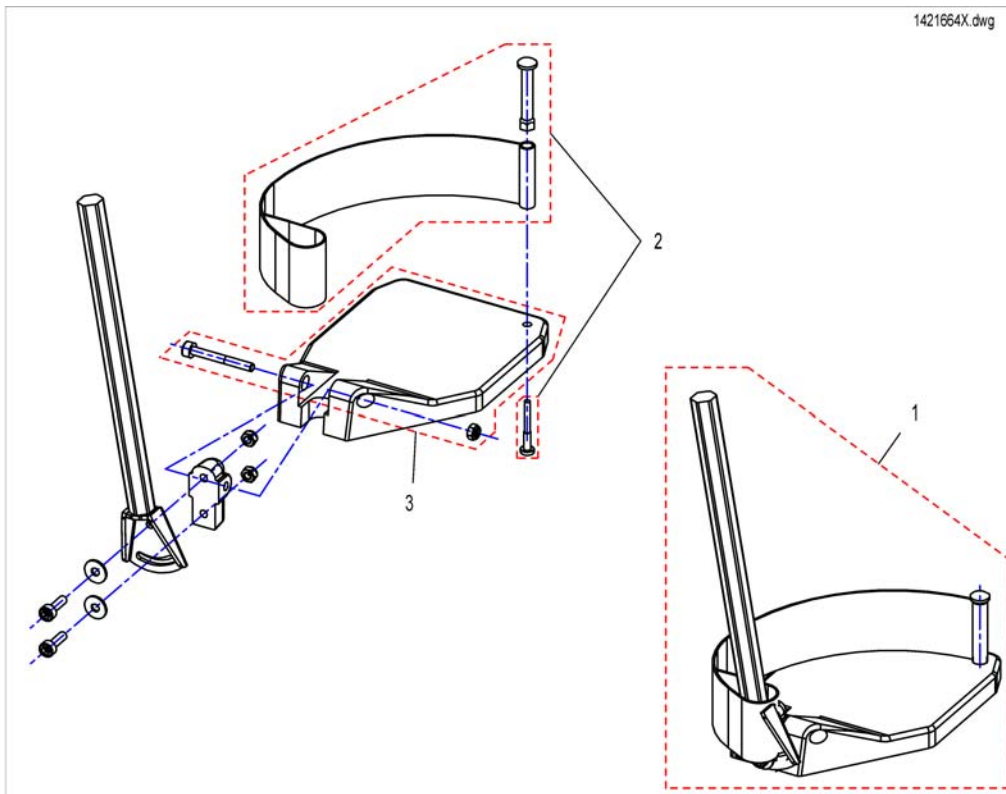

1	SP1421664	BENSTØTTEUNDERDEL HØ VINKELREG. STORM	1
1	SP1421665	BENSTØTTEUNDERDEL VE VINKELREG. STORM	1
2	SPB273011	HÆLBÅND BORA TDX	1
3	SPZ173217	(Footplate LH)	1
3	SPZ173218	(Footplate RH)	1


(Support Tube with Footplate for central Legrest)

1440290X.dwg

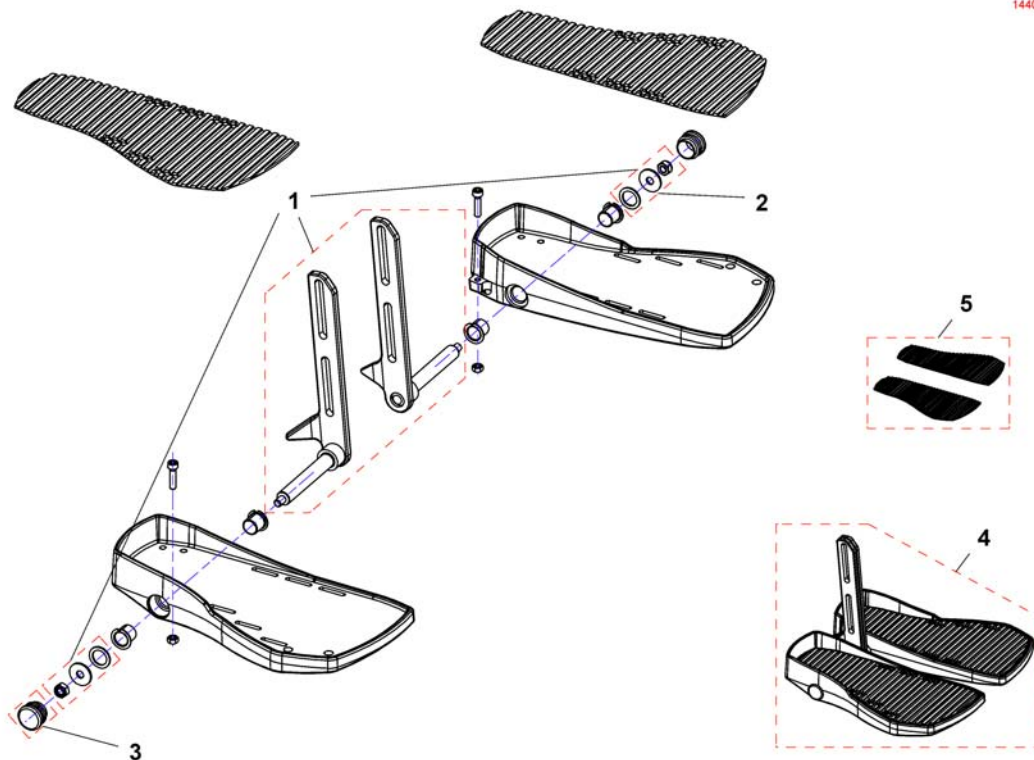


1	SP1440442	INNERRØR SENTERBENSTØTTE STORM 4 EXTRA LITEN	1
1	SP1440443	INNERRØR SENTERBENSTØTTE STORM 4 EXTRA MEDIUM	1
1	SP1440444	(Central Lower Legrest, large)	1
2	SPF00002	(Fixation Kit)	1
3	SP1440517	(Darning)	1
4	SP1440290	FOTPLATE OPPFELL INNERØR 32-38 HØ/VE STORM4 T2	1
4	SP1440291	FOTPLATE OPPFELLB MEDIUM INNERRØR SENTER BENST	1
4	SP1440292	FOTPLATE OPPFEL INNERØR 44-50 HØ/VE STORM4 T2	1
5	SP1546720	(Footboardrubber)	1

(Footrest Lower Parts)

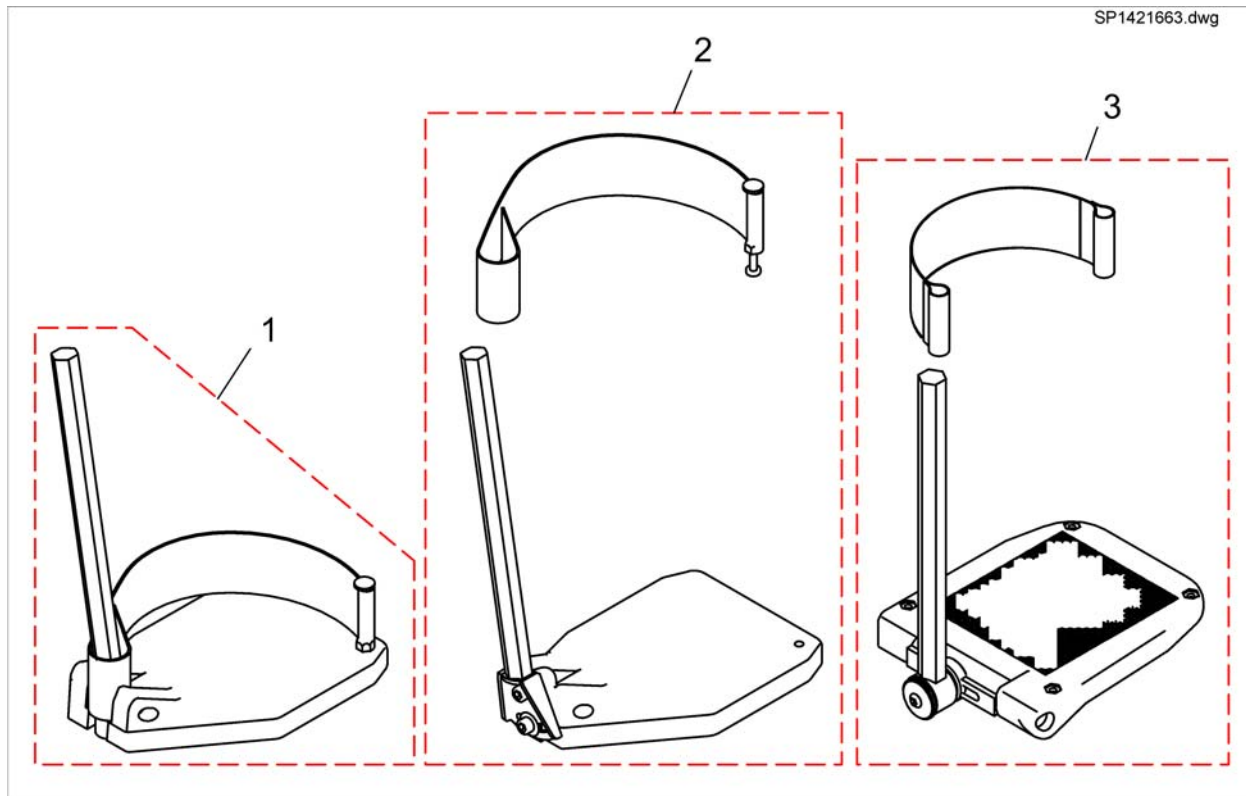

1	SP1421663	BENSTØTTEUNDERDEL VE STORM3	1
1	SP1421662	BENSTØTTEUNDERDEL HØ STORM3	1
2	SP1421664	BENSTØTTEUNDERDEL HØ VINKELREG. STORM	1
2	SP1421665	BENSTØTTEUNDERDEL VE VINKELREG. STORM	1
3	SP1419344	FOTPLATE VE VREG OG DREG KPL MED INNERRØR	1
3	SP1419345	FOTPLATE HØ VREG OG DREG KPL MED INNERRØR	1
4	SP1446549	()	1
5	SP1446548	()	1


SPCTDX_SP20.00

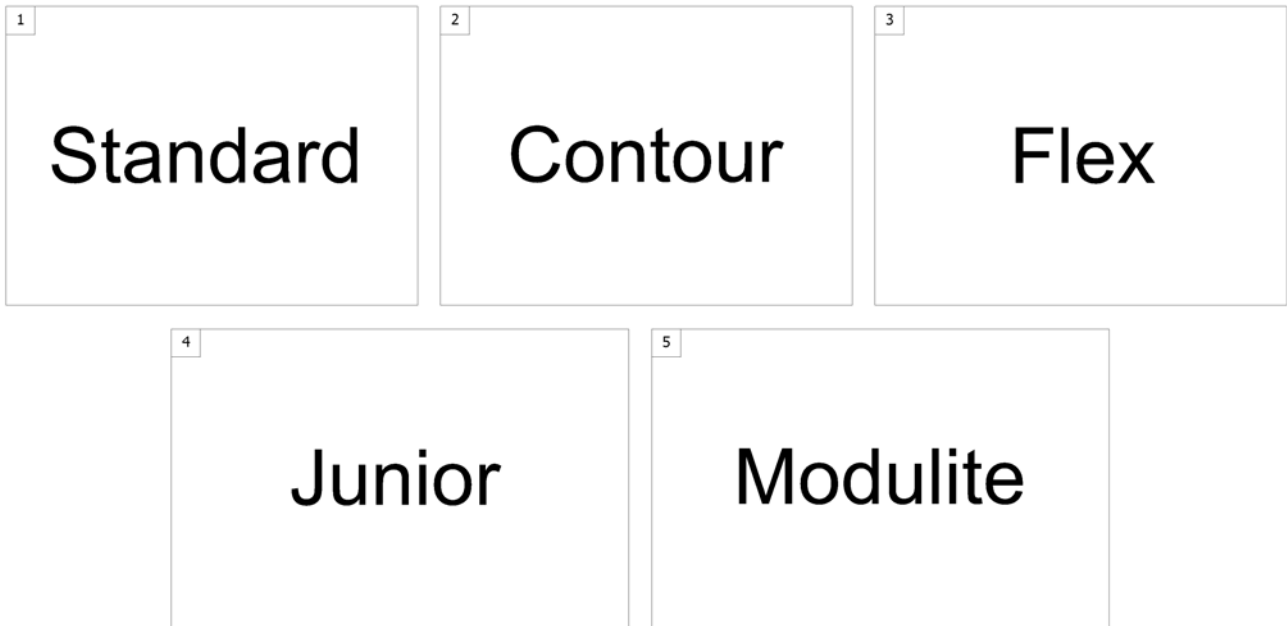

1	-	(Armrest for Standard Seat)	1
2	-	(Armrest for Contour Seat)	1
3	-	(Armrest for Flex Seat)	1
4	-	(Armrest for Junior Seat)	1
5	-	(Armrests for Modulite Seat)	1

(Armrest for Standard Seat)

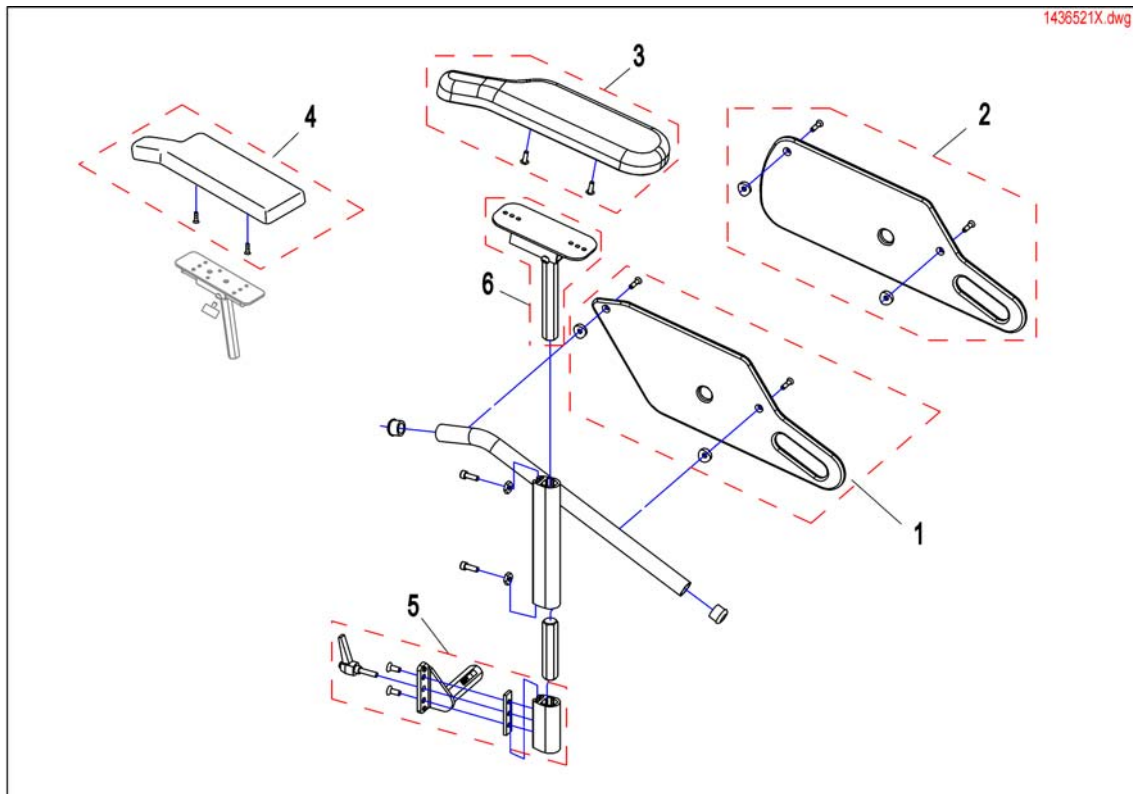

-	(Armrest Standard) <i>[0635/0636]</i>	1
-	(Armrests cpl.) <i>[0635/0636]</i>	1
-	(T-Armrest, detachable, with Armpad, for Standard Seat 2008-08-01 ==>) <i>[0606]</i>	1



1	SP1428078	KLESBESKYTTER STORM4 TDX		1
2	SP1420471	KLESBESKYTTER TDX		1
3	SP1449043	ARMLENSPOLSTER HØ STORM STORM TDX BORA	01.01.2009 →	1
3	SP1443004	ARMLENSPOLSTER VE STORM STORM TDX BORA	01.01.2009 →	1
3	SP1528845	ARMLENSPOLSTER HØ LANG STORM4 TDX BORA		1
3	SP1528846	ARMLENSPOLSTER VE LANG STORM TDX BORA		1
4	SP1423221	ARMLENSPOLS.HØ STOR G50/ STORM3/4/THYPOON2/2009->	→ 31.12.2008	1
4	SP1423224	ARMLENSPOLS.VE STOR G50/ STORM3/4/THYPOON2/2009->	→ 31.12.2008	1
5	SP1442550	FESTE ARMLEN HØ TDX		1
5	SP1442549	FESTE ARMLEN VE TDX		1
6	SP1510857	ARMLENSPOLSTER HOLDER VE STORM3/4 2009->		1
6	SP1510856	ARMLENSPOLSTER HOLDER HØ STORM3/4 2009->		1


(Armrests cpl.)

1421694X.dwg

Seat height 45cm/47cm Seatraiser - seatheight 47cm - 77cm											
Standard, Contour, Flex				01.01.09 ==>							
		Armrest height 340mm		Armrest height 380mm							
		RH	LH	RH	LH						
> 39cm	> 41cm	1	2	3	4						
39cm	> 41cm	5	6	7	8						
> 39cm	<= 41cm	9	10	11	12						
39cm	<= 41cm	13	14	15	16						
<table border="1" style="margin: auto;"> <tr> <td colspan="2" style="text-align: center;">child seat</td> </tr> <tr> <td style="text-align: center;">RH</td> <td style="text-align: center;">LH</td> </tr> <tr> <td style="text-align: center;">17</td> <td style="text-align: center;">18</td> </tr> </table>						child seat		RH	LH	17	18
child seat											
RH	LH										
17	18										

1	SP1421692	ARMLEN HØ KPL HREG AVTGB 340MM STORM TDX	1
2	SP1421694	ARMLEN VE KPL HREG AVTGB 340MM STORM TDX	1
3	SP1421693	ARMLEN HØ KPL DHREG 380MM STORM3	1
4	SP1421695	ARMLEN VE KPL DHREG 380MM STORM3	1
5	SP1422542	ARMLEN KORT HØ KPL SB39 FLEX STORM3	1
6	SP1422543	ARMLEN KORT VE KPL SB39 FLEX STORM3	1
7	SP1422548	ARMLEN DHREG HØ KORT 38 STORM3	1
8	SP1422549	ARMLEN DHREG VE KORT 38 STORM3	1
9	SP1428096	ARMLEN 340MM SB41 HØ TDX	1
10	SP1428097	ARMLEN 340MM SB41 VE TDX	1
11	SP1428098	ARMLEN 380MM SB39 HØ TDX	1
12	SP1428099	ARMLEN 380MM SB41 VE TDX	1
13	SP1428100	ARMLEN HØ KPL DHREG SB39 340MM STORM3	1
14	SP1428101	ARMLEN VE KPL DHREG SB39 340MM STORM3	1
15	SP1428102	ARMLEN 380MM SB39 HØ TDX	1
16	SP1428103	ARMLEN VE KPL DHREG SB39 380MM STORM3	1
17	5313557	ARMLEN DHREG OPPFELL HØ JUNIOR	1
18	5313558	ARMLEN DHREG OPPFELL VE JUNIOR	1

(T-Armrest, detachable, with Armpad, for Standard Seat 2008-08-01 ==>)

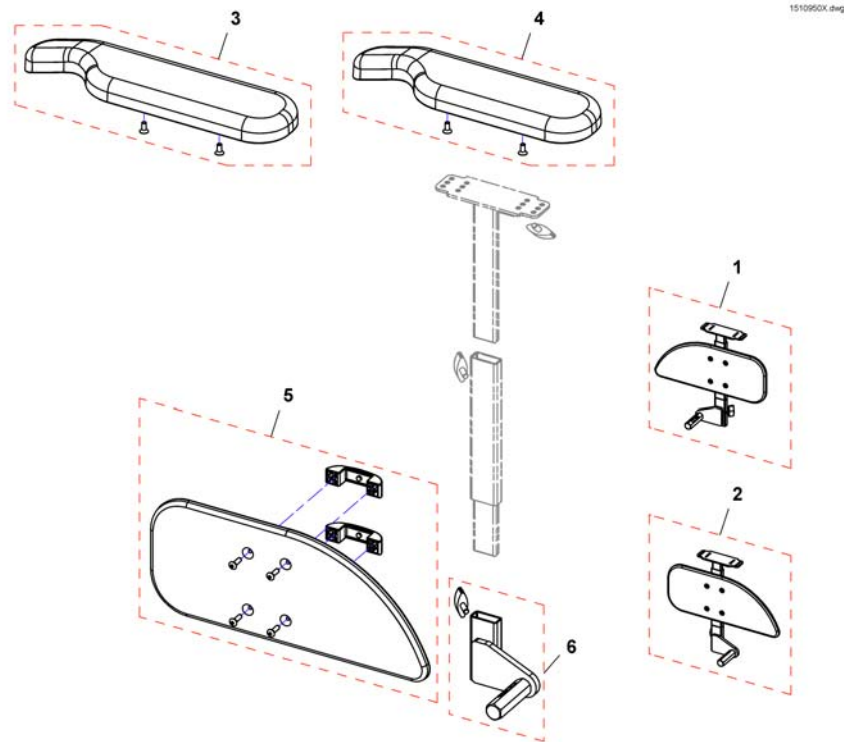


1	SP1510935	(Armrest, without armpad, RH)	1
2	SP1510950	(Armrest, without armpad, LH)	1
3	SP1528846	ARMLENSPOLSTER VE LANG STORM TDX BORA	1
3	SP1528845	ARMLENSPOLSTER HØ LANG STORM4 TDX BORA	1
4	SP1443004	ARMLENSPOLSTER VE STORM STORM TDX BORA	1
4	SP1449043	ARMLENSPOLSTER HØ STORM STORM TDX BORA	1
5	SP1541505	(Skirt guard, RH)	1
5	SP1541506	(Skirt guard, LH)	1
6	SP1541510	(Bracket for T-arm, RH)	1
6	SP1541511	(Bracket for T-arm, LH)	1

(Armrest for Contour Seat)

-	(Armrest Standard)	1
-	(Armrests cpl.)	1
-	(Mounting bracket Flip-up armrests)	1
-	(Following armrests)	1
-	(Sliding Armrest)	1
-	(Skirtguard for following armrest)	1


(Mounting bracket Flip-up armrests)

1548568.dwg

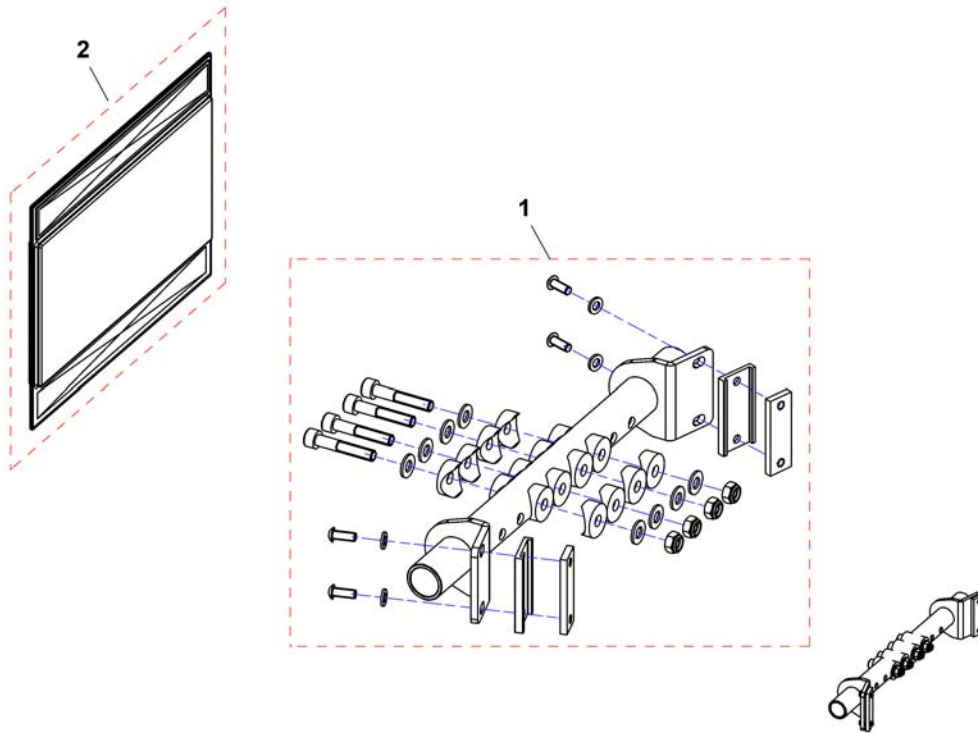


1	SP1548568	FESTER OPPFEL ARML SB39 BORA	1
1	SP1548569	FESTER OPPFEL ARML SB44 BORA	1
1	SP1548631	FESTER OPPFEL ARML SB49 BORA	1
2	SP1550410	(Protection cover)	1

(Following armrests)

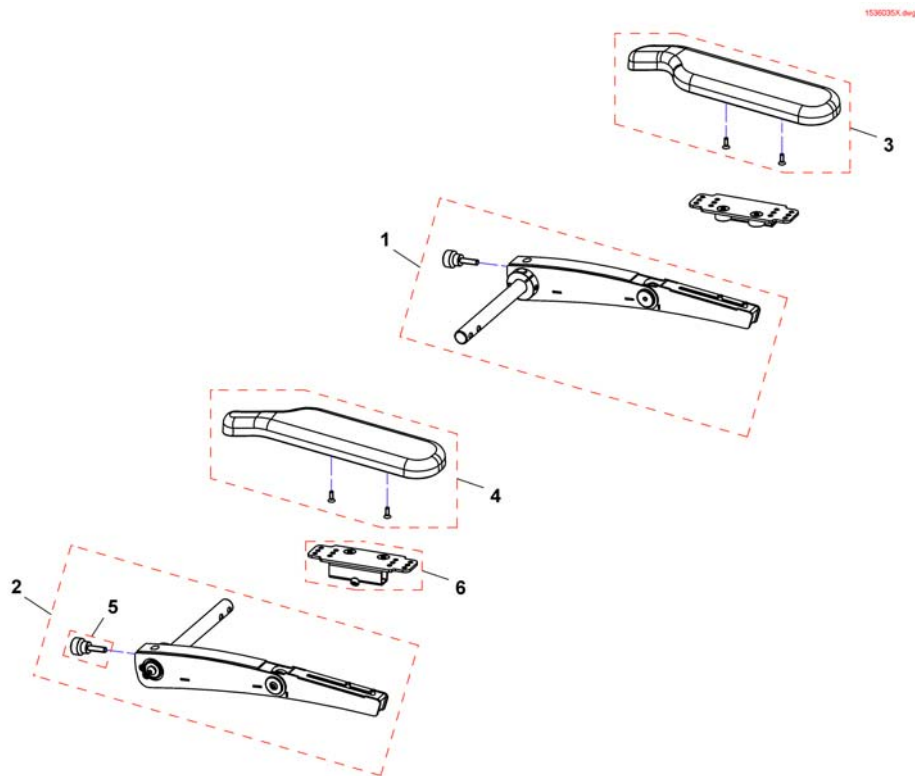


1	SP1536036	ARMLÉN OPPFELL VE STORM4	1
1	SP1536100	ARMLÉN LANG VE OPPFELLB STORM4	1
2	SP1536035	ARMLÉN OPPFELL HØ STORM4	1
2	SP1536099	ARMLÉN LANG HØ OPPFELLB STORM4	1
3	SP1443004	ARMLÉNSPOLSTER VE STORM STORM TDX BORA	1
3	SP1528846	ARMLÉNSPOLSTER VE LANG STORM TDX BORA	1
4	SP1449043	ARMLÉNSPOLSTER HØ STORM STORM TDX BORA	1
4	SP1528845	ARMLÉNSPOLSTER HØ LANG STORM4 TDX BORA	1
5	SP1535504	(Knurled screw M8x30)	1
6	SP1535244	FESTE ARMLÉN RYGGMONT STORM4	1

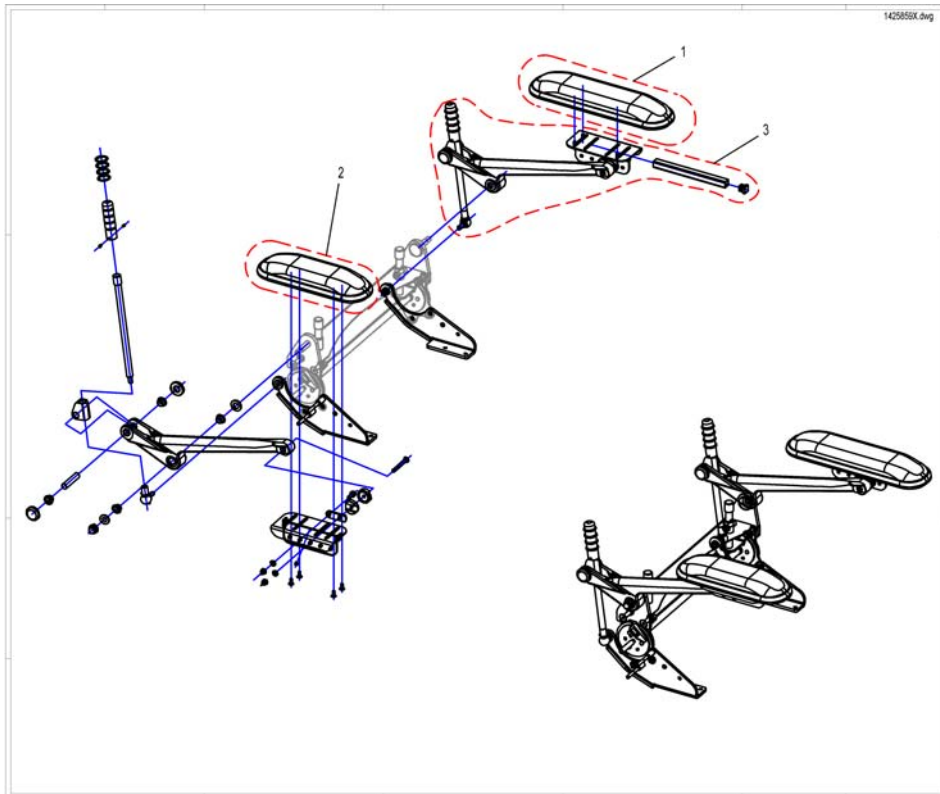

1	SP1513484	ARMLEN LANG TDX	1
2	SP1513485	ARMLEN KORT TDX	1
3	SP1425844	ARMLEN OPPFELLBAR KPL PA TDX	1

(Skirtguard for following armrest)

1425860X.dwg

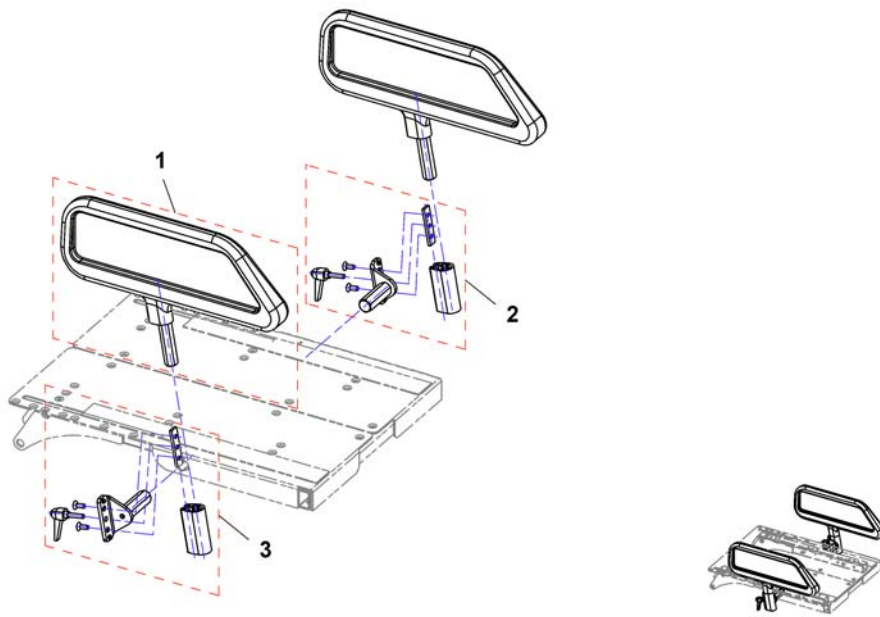

1	SP1426303	KLESBESKYTTER OPPF ARML STORM4	1
2	SP1442549	FESTE ARMLEN VE TDX	1
3	SP1442550	FESTE ARMLEN HØ TDX	1

-	(Armrest Standard)	1
-	(Armrests cpl.)	1
-	(Mounting bracket Flip-up armrests)	1
-	(Following armrests)	1
-	(Sliding Armrest)	1
-	(Skirtguard for following armrest)	1

(Armrest for Junior Seat)


-

(Armrests cpl.)

1

(Armrests for Modulite Seat)


1571738.000

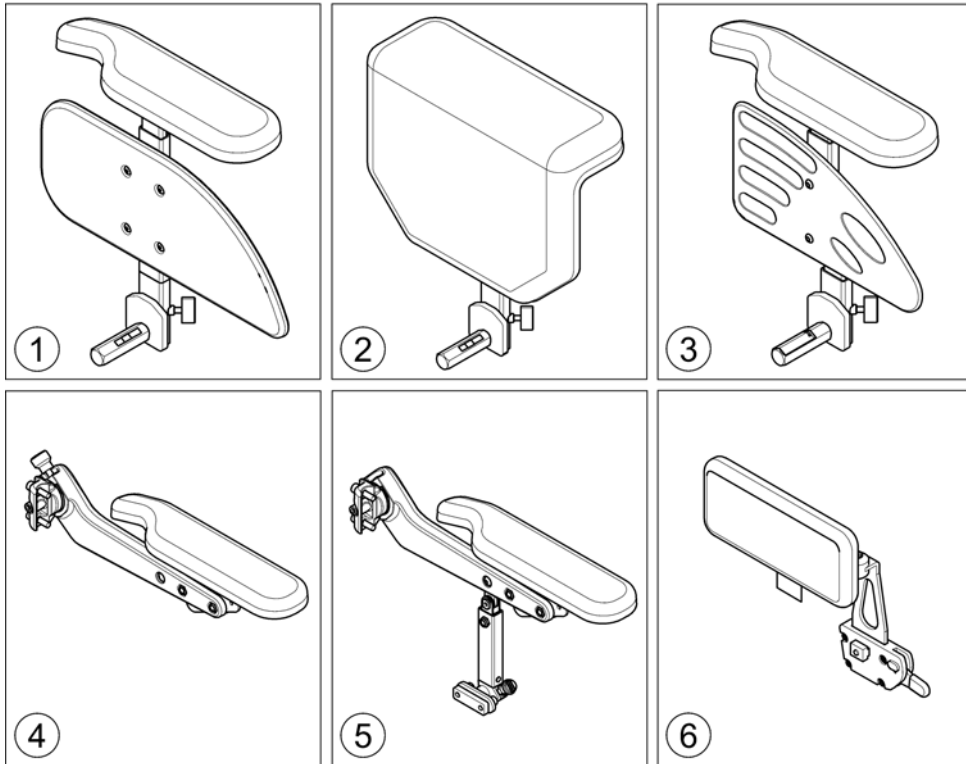


1	-	(T-Armrest) [2191+2195/2196/2197]	1
2	-	(Padded Armrest) [2198]	1
3	-	(Inrigged Armrest) [2183+2195/2196/2197/2199]	0
4	-	(Flip-Up Armrest) [2193+2195/2196/2197/2199]	0
5	-	(Following Armrest) [2192/2194+2195/2196/2197/2199]	0
6	-	(Hip Support) [2184/2185/2186/2187/2188/2189]	0

(T-Armrest)

1572570.iso

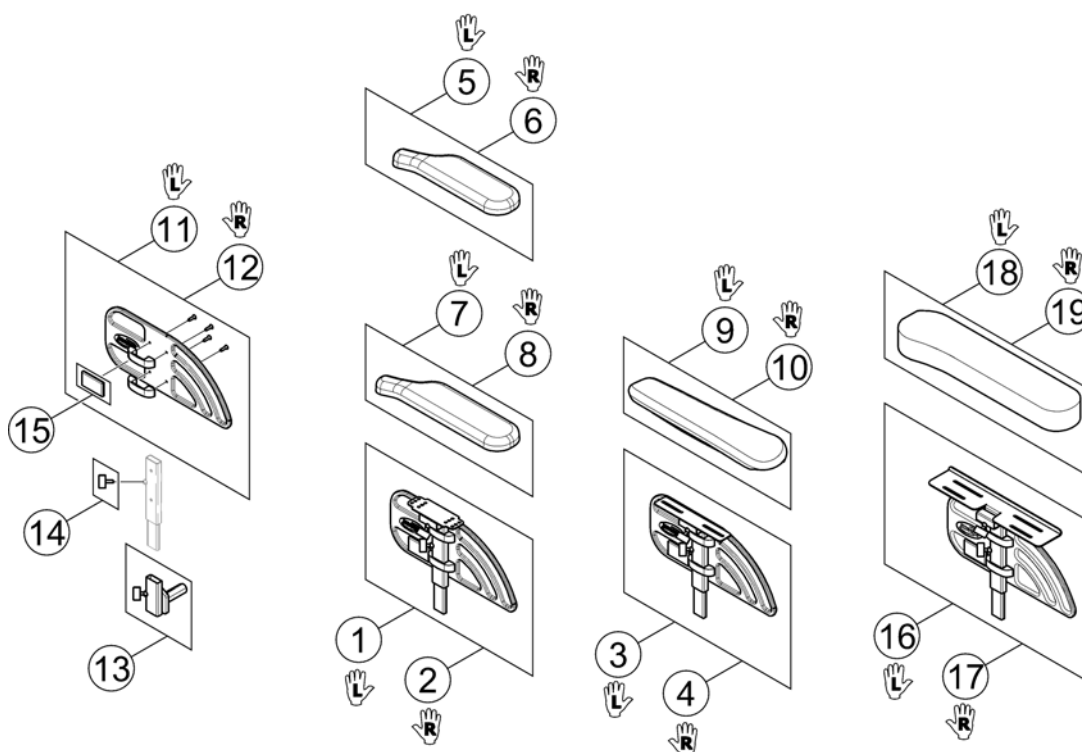


1	SP1570318	SIDESTØTTE VE STORM4 MODULITE	1
2	SP1570368	SIDESTØTTE HØ STORM4 MODULITE	1
3	SP1578114	(Siderest LH (for Ergonomic Armpad))	1
4	SP1578115	(Siderest RH (for Ergonomic Armpad))	1
5	SP1443004	ARMLENSPOLSTER VE STORM STORM TDX BORA	1
6	SP1449043	ARMLENSPOLSTER HØ STORM STORM TDX BORA	1
7	SP1528846	ARMLENSPOLSTER VE LANG STORM TDX BORA	1
8	SP1528845	ARMLENSPOLSTER HØ LANG STORM4 TDX BORA	1
9	SP1571589	ARMLENSPOLSTER ERGONO VE STORM4 TDX	1
10	SP1571588	ARMLENSPOLSTER ERGONO HØ STORM4 TDX	1
11	SP1572940	(Skirt Guard LH)	1
12	SP1572948	(Skirt Guard RH)	1
13	SP1567749	(Siderest Bracket - LH/RH)	1
14	SPL10024	(Wing Screw M6x16)	1
15	SPP31298	(Reflector, yellow)	1
16	SP1585205	(Siderest LH (for Soft Armpad))	1
17	SP1585206	(Siderest RH (for Soft Armpad))	1
18	SP1585068	(Armpad, Soft, LH)	1
19	SP1585069	(Armpad, Soft, RH)	1

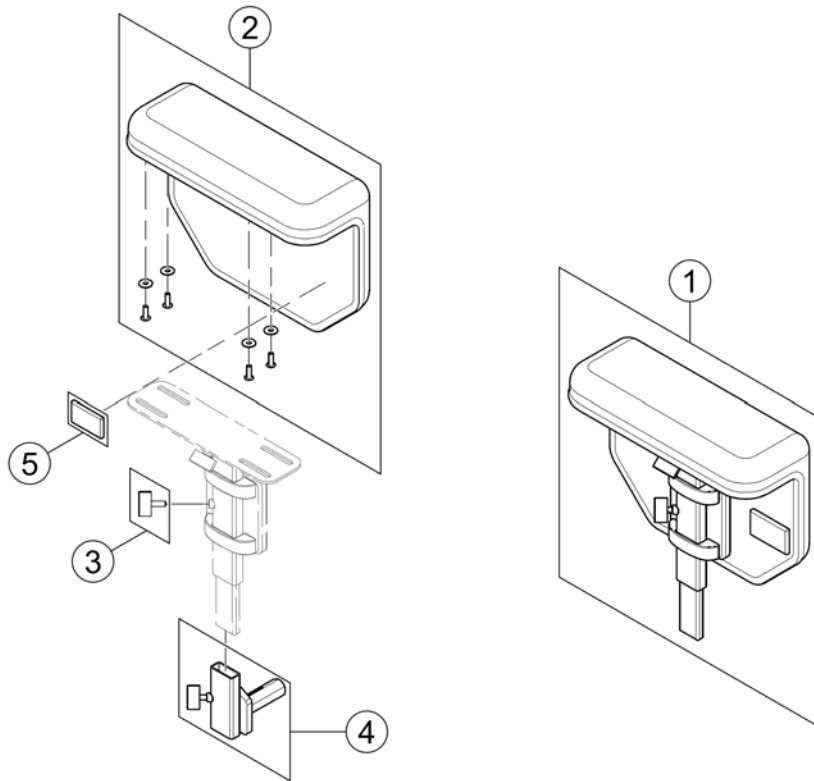


1	SP1571987	(Padded Armrest RH/LH, cpl.)	1
2	SP1571962	ARMLENSPOLSTER PADDED MODULITE	1
3	SPL10024	(Wing Screw M6x16)	1
4	SP1567749	(Siderest Bracket - LH/RH)	1
5	SPP31298	(Reflector, yellow)	1

(Inrigged Armrest)

1585180 iso

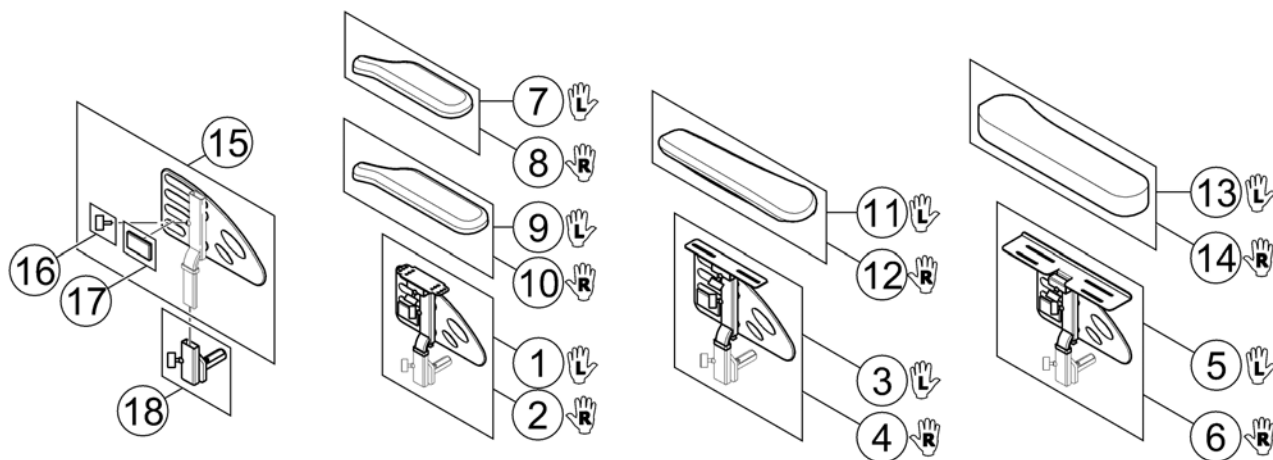


1	SP1585448	ARMLEN VE KORT BORA	1
2	SP1585449	ARMLEN HØ KORT BORA	1
3	SP1585450	(Armrest LH (for Ergonomic Armpad))	1
4	SP1585451	(Armrest RH (for Ergonomic Armpad))	1
5	SP1585452	(Armrest LH (for Soft Armpad))	1
6	SP1585453	(Armrest RH (for Soft Armpad))	1
7	SP1443004	ARMLENSPOLSTER VE STORM STORM TDX BORA	1
8	SP1449043	ARMLENSPOLSTER HØ STORM STORM TDX BORA	1
9	SP1528846	ARMLENSPOLSTER VE LANG STORM TDX BORA	1
10	SP1528845	ARMLENSPOLSTER HØ LANG STORM4 TDX BORA	1
11	SP1571589	ARMLENSPOLSTER ERGONO VE STORM4 TDX	1
12	SP1571588	ARMLENSPOLSTER ERGONO HØ STORM4 TDX	1
13	SP1585068	(Armpad, Soft, LH)	1
14	SP1585069	(Armpad, Soft, RH)	1
15	SP1578119	(Skirt Guard LH/RH)	1
16	SPL10024	(Wing Screw M6x16)	1
17	SPP31298	(Reflector, yellow)	1
18	SP1567749	(Siderest Bracket - LH/RH)	1

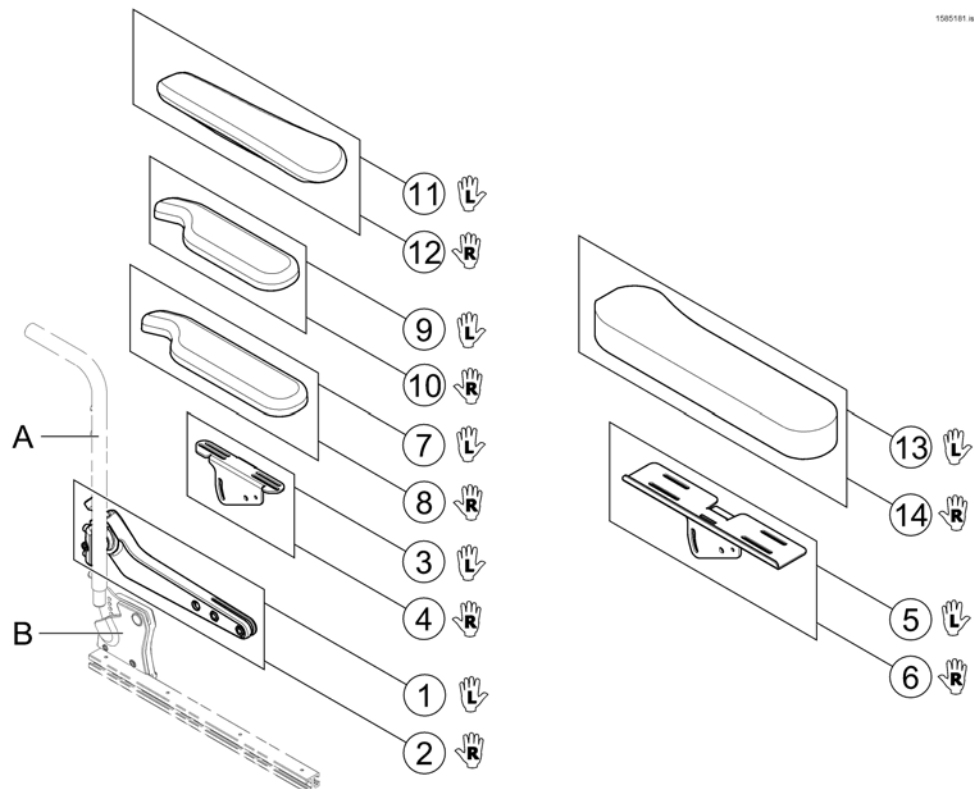


-	-	(Retrofitting the armrests on chairs built before 15-03-27 also requires new Pos. A & B and Sling Back with respective cut-outs!) !!!		0
1	SP1582668	ARMLLEN FLIP UP VE KPL MUDULITE	19.01.2015 →	1
2	SP1582644	ARMLLEN FLIP UP HØ KPL MODULITE	19.01.2015 →	1
3	SP1585412	(Armpad Support (Standard, Extended, Ergonomic), LH)		1
4	SP1585413	(Armpad Support (Standard, Extended, Ergonomic), RH)		1
5	SP1585414	(Armpad Support (Soft), LH)		1
6	SP1585415	(Armpad Support (Soft), RH)		1
7	SP1528846	ARMLENSPOLSTER VE LANG STORM TDX BORA		1
8	SP1528845	ARMLENSPOLSTER HØ LANG STORM4 TDX BORA		1
9	SP1443004	ARMLENSPOLSTER VE STORM STORM TDX BORA		1
10	SP1449043	ARMLENSPOLSTER HØ STORM STORM TDX BORA		1
11	SP1571589	ARMLENSPOLSTER ERGONO VE STORM4 TDX		1
12	SP1571588	ARMLENSPOLSTER ERGONO HØ STORM4 TDX		1
13	SP1585068	(Armpad, Soft, LH)		1
14	SP1585069	(Armpad, Soft, RH)		1

(Following Armrest)

1585182 NO

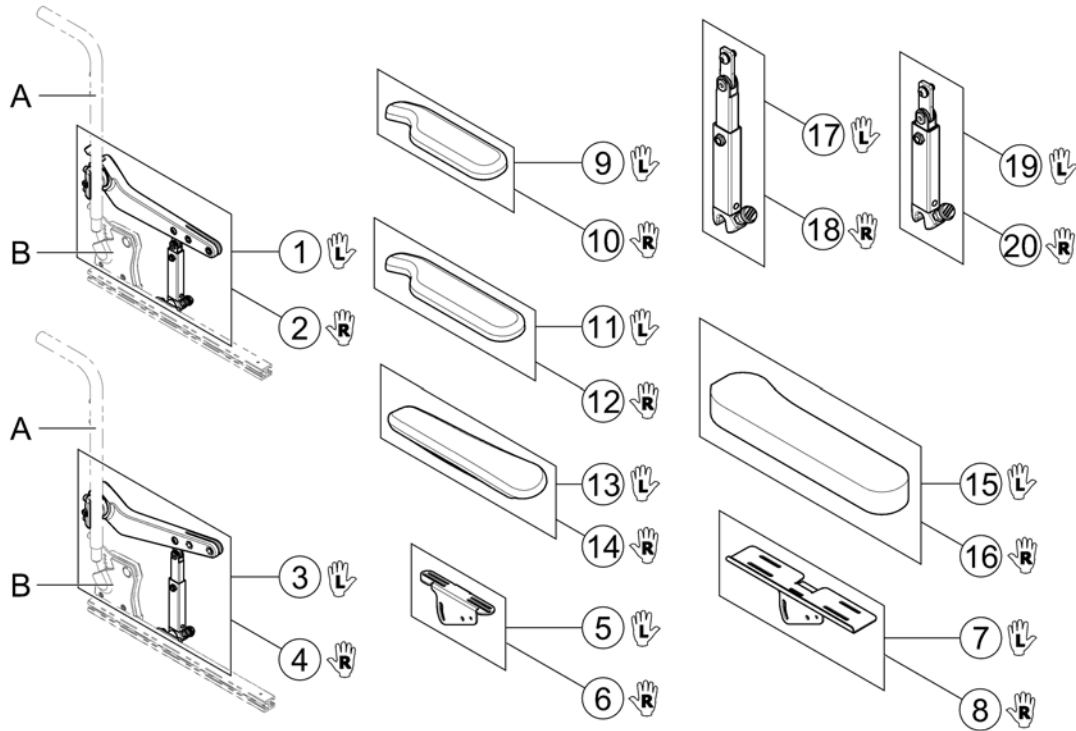


-	-	(Retrofitting the armrests on chairs built before 15-03-27 also requires new Pos. A & B and Sling Back with respective cut-outs!) !!!		0
1	SP1585263	ARMLLEN OPPFELLBAR VE MODULITE KORT	19.01.2015 →	1
2	SP1585264	ARMLLEN OPPFELLBAR HØ MODULITE KORT	19.01.2015 →	1
3	SP1585267	ARMLLEN OPPFELL VE LANG MODULITE FOLLOWING	19.01.2015 →	1
4	SP1585268	ARMLLEN OPPFELLB HØ LANG MODULITE FOLLOWING	19.01.2015 →	1
5	SP1585412	(Armpad Support (Standard, Extended, Ergonomic), LH)		1
6	SP1585413	(Armpad Support (Standard, Extended, Ergonomic), RH)		1
7	SP1585414	(Armpad Support (Soft), LH)		1
8	SP1585415	(Armpad Support (Soft), RH)		1
9	SP1443004	ARMLLENSPOLSTER VE STORM STORM TDX BORA		1
10	SP1449043	ARMLLENSPOLSTER HØ STORM STORM TDX BORA		1
11	SP1528846	ARMLLENSPOLSTER VE LANG STORM TDX BORA		1
12	SP1528845	ARMLLENSPOLSTER HØ LANG STORM4 TDX BORA		1
13	SP1571589	ARMLLENSPOLSTER ERGONO VE STORM4 TDX		1
14	SP1571588	ARMLLENSPOLSTER ERGONO HØ STORM4 TDX		1
15	SP1585068	(Armpad, Soft, LH)		1
16	SP1585069	(Armpad, Soft, RH)		1
17	SP1583156	(Amrest Support, long, LH)		1
18	SP1583155	(Amrest Support, long, RH)		1
19	SP1583154	(Amrest Support, short, LH)		1
20	SP1582639	(Amrest Support, short, RH)		1

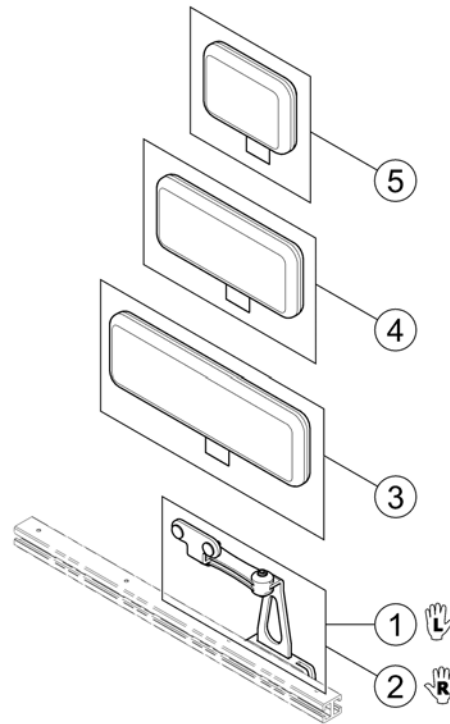

1	SP1584411	(Hip Support, Quick Release bracket, LH)	1
2	SP1584410	FESTE SIDESTØTTE MODULIT STORM4 TDX LOW-RIDER	1
3	SP1584414	(Hip Support Pad, large (330x100 mm))	1
4	SP1584413	PUTE SIDESTØTTE MEDIUM STORM4	1
5	SP1584412	(Hip Support Pad, small (140x100 mm))	1


(Castors Front/Rear)

1142342.no

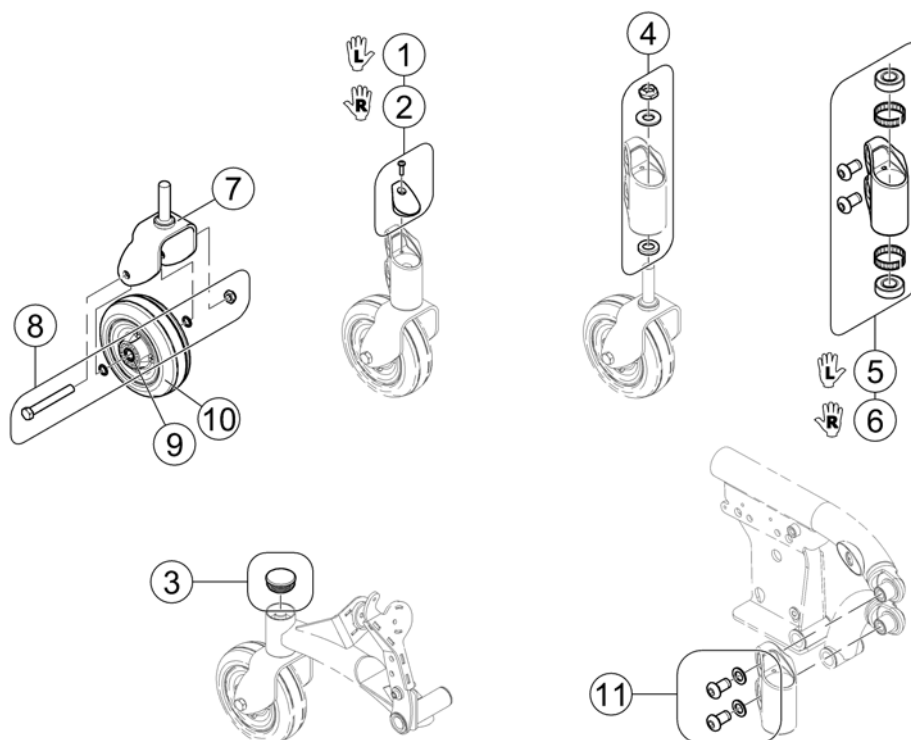

-	SP1537420	HYLSE GAFFEL TDX	1
1	SP1523141	TOPPHETTE FORAN VE TDX	1
2	SP1523140	TOPPHETTE FORAN HØ TDX	1
3	SP1523093	TOPPHETTE BAK HØ/VE TDX	1
4	SP1522999	GAFFELHUS TDX	1
5	SP1523139	GAFFELHUS M FESTE VE TDX	1
6	SP1523094	GAFFELHUS M FESTE HØ TDX	1
7	SP1523016	GAFFEL TDX	1
8	SP1537431	AKSLINGSBOLT SVINGHJUL TDX	1
9	SP1522935	DISTANSELAGER TDX	1
10	SP1175881	(6x2" Wheel with bearings, silver hub)	1
10	SP1511369	SVINGHJUL KPL TDX TDX TORNADO TYPHOON	1
11	SP1523176	SKRUER GAFFELHUS TDX	1


(Drive Wheels)

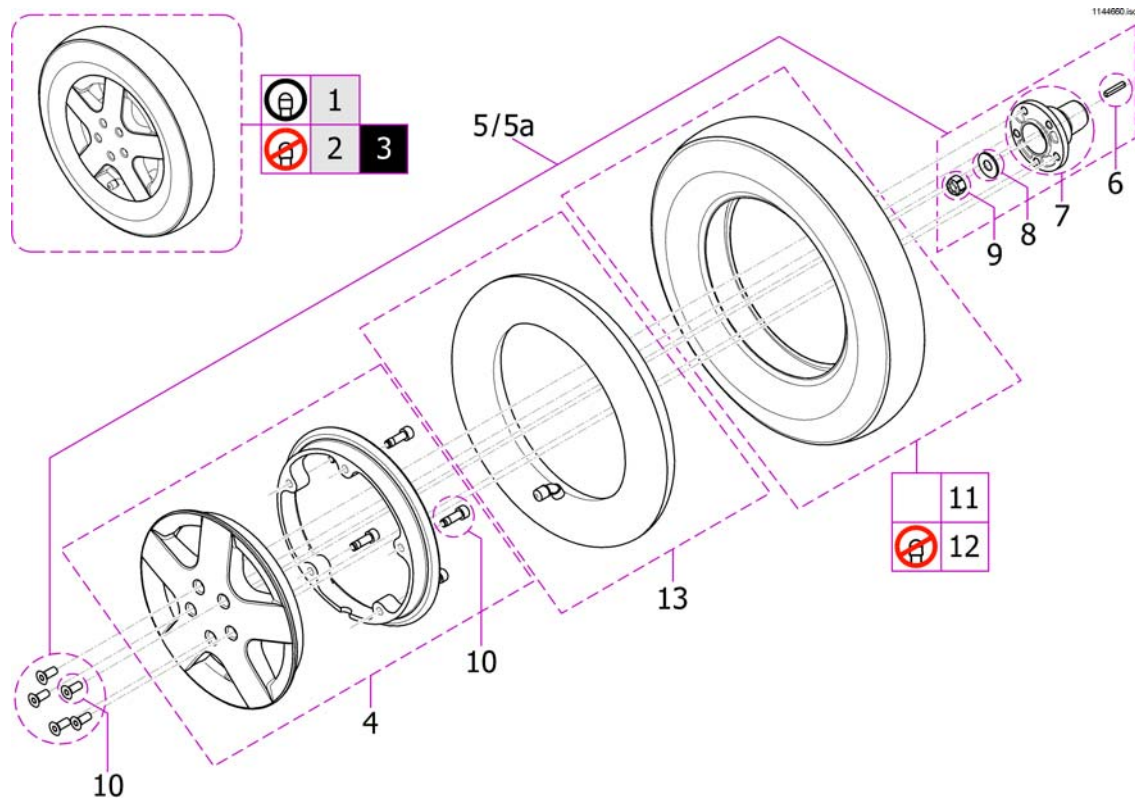

1	SP1532414	DRIVHJUL KPL M LUFT STORM4	1
2	SP1532416	HJUL PUNKTERINGSFRI STORM4	1
3	SP1523161	DRIVHJUL KOMPAKT KPL STORM4 TDX	1
4	SP1523013	HJUL MED FESTE TDX	1
5	SP1523157	MONTERINGSETT DRIVHJUL TDX	1
5a	SP1168890	(Drive Wheel 14 x 3" Mounting Kit (2-Pole Motor!))	1
6	SP1522934	(Key (3/16 x 1-1/4"))	1
7	SP1523092	HJULNAV TDX	1
8	SP1522939	(Washer (1/2 x 1-9/32 x 1/8"))	1
9	1025737	(Axle Locknut 1/2"-20)	1
10	SP1522975	Bolt 5/16-18 x 7/8" TDX	1
11	SP1522940	DEKK LUFT 14X 3 (3,00-8) STORM4	1
12	SP1141342	DEKK KOMPACT GB2 14X3 STORM4	1
13	SP1039651	(Tube 14")	1


(Motor & Gearbox ==> 06.09.11)

1144057 no

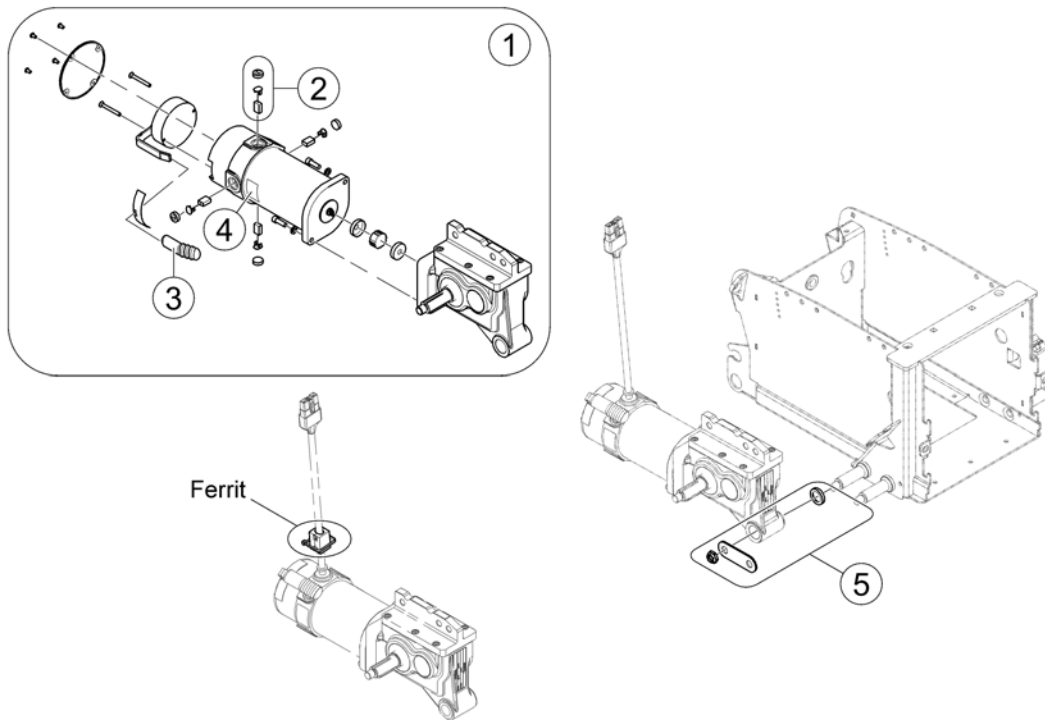

1	SP1523164	DRIVERK VE TDX KPL TDX	1
1	SP1523163	DRIVERK TDX HØ KPL TDX	1
1	SP1144657	MOTOR KPL VE TDX	1
1	SP1144656	MOTOR KPL HØ TDX	1
2	SP1522967	BØRSTER 4 STK KIT TDX STORM 4	4
3	SP1522976	(Cap, Declutching Lever)	1
4	1104797	(Disengaging information label)	1
5	SP1523168	(Kit, Lower Pivot Link / 4-Pole Motor Mounting Hardware)	1

1163096X.iso

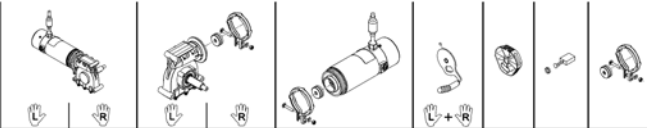

Bora/ SpectraXTR	6km/h	c	ACS/ Shark	1	2	33	34	37	---	51	45
			VR2	3	4	33	34	43	---	51	45
			R-Net	5	6	33	34	41	---	51	45
	10km/h	d	ACS/ Shark	7	8	31	32	36	---	51	45
			VR2	9	10	31	32	40	---	51	45
			R-Net	11	12	31	32	42	---	51	45
BoraHD/ Spectra XTR2HD	6km/h		ACS/ Shark	54	55	33	34	39	---	51	46
	10km/h		ACS/ Shark	56	57	31	32	38	---	51	46
	6km/h		R-Net	58	59	33	34	62	---	51	46
	10km/h		R-Net	60	61	31	32	63	---	51	46
FDX	6km/h	e		13	14	33	34	37	53	---	45
	6km/h	f	ACS/ Shark	54	55	33	34	39	53	---	46
	10km/h			15	16	31	32	38	53	---	46
Kite	6km/h		ACS/ Shark	17	18	33	34	37	---	51	45
	10km/h		ACS/ Shark	19	20	31	32	36	---	51	45
Kite HD	6km/h		ACS/ Shark	21	22	33	34	39	---	51	46
	10km/h		ACS/ Shark	23	24	31	32	38	---	51	46
Storm4/ Storm4 X-plore	6km/h		ACS/ Shark	47	48	34	33	39	---	51	46
	10km/h		ACS/ Shark	27	28	32	31	38	---	51	46
TDX-SP	6km/h	a		25	26	34	33	39	53	---	46
	8km/h	a	ACS/ Shark	29	30	34	33	38	53	---	46
	10km/h	b		49	50	32	31	38	53	---	46
TDX-SP Narrow	6km/h		ACS/ Shark	25	26	34	33	39	53	---	46
	10km/h		ACS/ Shark	49	50	32	31	38	53	---	46

a) used from 28.09.11 / b) used from 07.09.11 / c) used from 18.04.11 / d) used from 01.08.11 / e) used till 09.09.13 / f) used from 10.09.13

-	N23004	KIL KRYSS DRIVAXEL (MO	1
1	SP1524195	DRIVENHET 6KMT VE BORA	1
2	SP1524194	DRIVENHET 6KMT HØ BORA	1
3	SP1541408	(Propulsion cpl. LH, Gearbox i = 23:1, Motor, 2-pole, 250W, slow, 600mm cable, VR2)	1
4	SP1541410	(Propulsion cpl. RH, Gearbox i = 23:1, Motor, 2-pole, 250W, slow, 600mm cable, VR2)	1
5	SP1541409	(Propulsion cpl. LH, Gearbox i = 23:1, Motor, 2-pole, 250W, slow, 600mm cable, R-Net)	1
6	SP1541411	(Propulsion cpl. RH, Gearbox i = 23:1, Motor, 2-pole, 250W, slow, 600mm cable, R-Net)	1
7	SP1524197	(Propulsion cpl. LH, Gearbox i = 18:1, Motor, 2-pole, 250W, fast, 600mm cable)	1
8	SP1524196	(Propulsion cpl. RH, Gearbox i = 18:1, Motor, 2-pole, 250W, fast, 600mm cable)	1
9	SP1541404	(Propulsion cpl. LH, Gearbox i = 18:1, Motor, 2-pole, 250W, fast, 600mm cable, VR2)	1
10	SP1541406	(Propulsion cpl. RH, Gearbox i = 18:1, Motor, 2-pole, 250W, fast, 600mm cable, VR2)	1
11	SP1541405	(Propulsion cpl. LH, Gearbox i = 18:1, Motor, 2-pole, 250W, fast, 600mm cable, R-Net)	1
12	SP1541407	(Propulsion cpl. RH, Gearbox i = 18:1, Motor, 2-pole, 250W, fast, 600mm cable, R-Net)	1
13	SP1168983	(Propulsion cpl. LH, Gearbox i = 23:1, Motor, 2-pole, 250W, slow, 600mm cable)	1
14	SP1168984	(Propulsion cpl. RH, Gearbox i = 23:1, Motor, 2-pole, 250W, slow, 600mm cable)	1
15	SP1168989	(Propulsion cpl. LH, Gearbox i = 18:1, Motor, 4-pole, 340W, fast, 760mm cable)	1
16	SP1168988	(Propulsion cpl. RH, Gearbox i = 18:1, Motor, 4-pole, 340W, fast, 760mm cable)	1
17	SP1568147	(Propulsion cpl. LH, Gearbox i = 23:1, Motor, 2-pole, 250W, slow, 600mm cable)	1
18	SP1568148	(Propulsion cpl. RH, Gearbox i = 23:1, Motor, 2-pole, 250W, slow, 600mm cable)	1

(Propulsion SSD - Details)

19	SP1568149	(Propulsion cpl. LH, Gearbox i = 18:1, Motor, 2-pole, 250W, fast, 600mm cable)	1
20	SP1568150	(Propulsion cpl. RH, Gearbox i = 18:1, Motor, 2-pole, 250W, fast, 600mm cable)	1
21	SP1568151	(Propulsion cpl. LH, Gearbox i = 23:1, Motor, 4-pole, 340W, slow, 760mm cable)	1
22	SP1568152	(Propulsion cpl. RH, Gearbox i = 23:1, Motor, 4-pole, 340W, slow, 760mm cable)	1
23	SP1568153	(Propulsion cpl. LH, Gearbox i = 18:1, Motor, 4-pole, 340W, fast, 760mm cable)	1
24	SP1568208	(Propulsion cpl. RH, Gearbox i = 18:1, Motor, 4-pole, 340W, fast, 760mm cable)	1
25	SP1536022	(Propulsion cpl. LH, Gearbox i = 23:1, Motor, 4-pole, 340W, slow, 760mm cable)	1
26	SP1536021	(Propulsion cpl. RH, Gearbox i = 23:1, Motor, 4-pole, 340W, slow, 760mm cable)	1
27	SP1541366	DRIVMOTOR KPL VE STORM4 SSD <i>S/N 061122610151 =></i>	1
28	SP1541365	DRIVMOTOR KPL HØ STORM4 SSD <i>S/N 061122610151 =></i>	1
29	SP1170307	DRIVMOTOR KPL VE TDX TDX SSD	1
30	SP1170306	DRIVMOTOR KPL HØ TDX TDX SSD	1
31	SP1163413	(Gearbox, LH, i = 18:1)	1
32	SP1163412	(Gearbox, RH, i = 18:1)	1
33	SP1163415	(Gearbox, LH, i = 23:1)	1
34	SP1163414	(Gearbox, RH, i = 23:1)	1
35	SP1542953	(Clamp band with clutch)	1
36	SP1163099	(Motor, 2-pole, 250W, fast, 600mm cable)	1
37	SP1164046	(Motor, 2-pole, 250W, slow, 600mm cable)	1
38	SP1165863	(Motor, 4-pole, 340W, fast, 760mm cable)	1
39	SP1165865	(Motor, 4-pole, 340W, slow, 760mm cable)	1
40	SP1544840	(Motor, 2-pole, 250W, fast, 600mm cable, VR2)	1
41	SP1544796	(Motor, 2-pole, 250W, slow, 600mm cable, R-Net)	1
42	SP1544841	(Motor, 2-pole, 250W, fast, 600mm cable, R-Net)	1
43	SP1544795	(Motor, 2-pole, 250W, slow, 600mm cable, VR2)	1
45	SP1540358	(Kit, brushes with caps, 2-pole (Set of 2))	1
46	SP1522967	BØRSTER 4 STK KIT TDX STORM 4	1
47	SP1583821	(Propulsion cpl. LH, Gearbox i = 23:1, Motor, 4-pole, 340W, slow, 760mm cable) <i>S/N 061122812868 =></i>	1
48	SP1583822	(Propulsion cpl. RH, Gearbox i = 23:1, Motor, 4-pole, 340W, slow, 760mm cable) <i>S/N 061122812868 =></i>	1
49	SP1534060	(Propulsion cpl. LH, Gearbox i = 18:1, Motor, 4-pole, 340W, fast, 760mm cable)	1
50	SP1534059	(Propulsion cpl. RH, Gearbox i = 18:1, Motor, 4-pole, 340W, fast, 760mm cable)	1
51	SP1579595	FRIKOPLINGS HÅNDTAK STORM4 OG ALLE SSD MOTOR <i>Single part. Consider to buy another one for the other side if required. - Replaces SP1168668, SP1168669 & SP1169005</i>	1
53	SP1169004	(Declutching lever, complete set for both sides)	1
54	SP1552782	(Propulsion cpl. LH, Gearbox i = 23:1, Motor, 4-pole, 340W, slow, 760mm cable)	1
55	SP1552783	(Propulsion cpl. RH, Gearbox i = 23:1, Motor, 4-pole, 340W, slow, 760mm cable)	1
56	SP1552872	(Propulsion cpl. LH, Gearbox i = 18:1, Motor, 4-pole, 340W, fast, 760mm cable)	1
57	SP1552873	(Propulsion cpl. RH, Gearbox i = 18:1, Motor, 4-pole, 340W, fast, 760mm cable)	1
58	SP1552832	(Propulsion cpl. LH, Gearbox i = 23:1, Motor, 4-pole, 340W, slow, 760mm cable, R-Net)	1

59	SP1552833	(Propulsion cpl. RH, Gearbox i = 23:1, Motor, 4-pole, 340W, slow, 760mm cable, R-Net)	1
60	SP1552836	(Propulsion cpl. LH, Gearbox i = 18:1, Motor, 4-pole, 340W, fast, 760mm cable, R-Net)	1
61	SP1552837	(Propulsion cpl. RH, Gearbox i = 18:1, Motor, 4-pole, 340W, fast, 760mm cable, R-Net)	1
62	SP1562498	(Motor, 4-Pol, 340W, slow, 760mm cabel, R-Net)	1
63	SP1562499	(Motor, 4-Pol, 340W, fast, 760mm cabel, R-Net)	1



1	-	(ACS Electronic)	1
2	-	(Shark Electronic)	1

1500784.no

1
Modules


2
**Remote
& Brackets**

3
**Cables &
Accessories**


1	-	(ACS Electronic Modules)	1
2	-	(ACS Remote & Brackets)	1
3	-	(ACS Cables & Accessories)	1

(ACS Electronic Modules)

-	(Powermodule ACS2)	1
-	(ACT2/4 for ACS2)	1
-	(Bracket ACT narrow base)	1
-	(Lightboard ACS2)	1
-	(G-Trac-sensor)	1
-	(Holder for G-Trac)	1

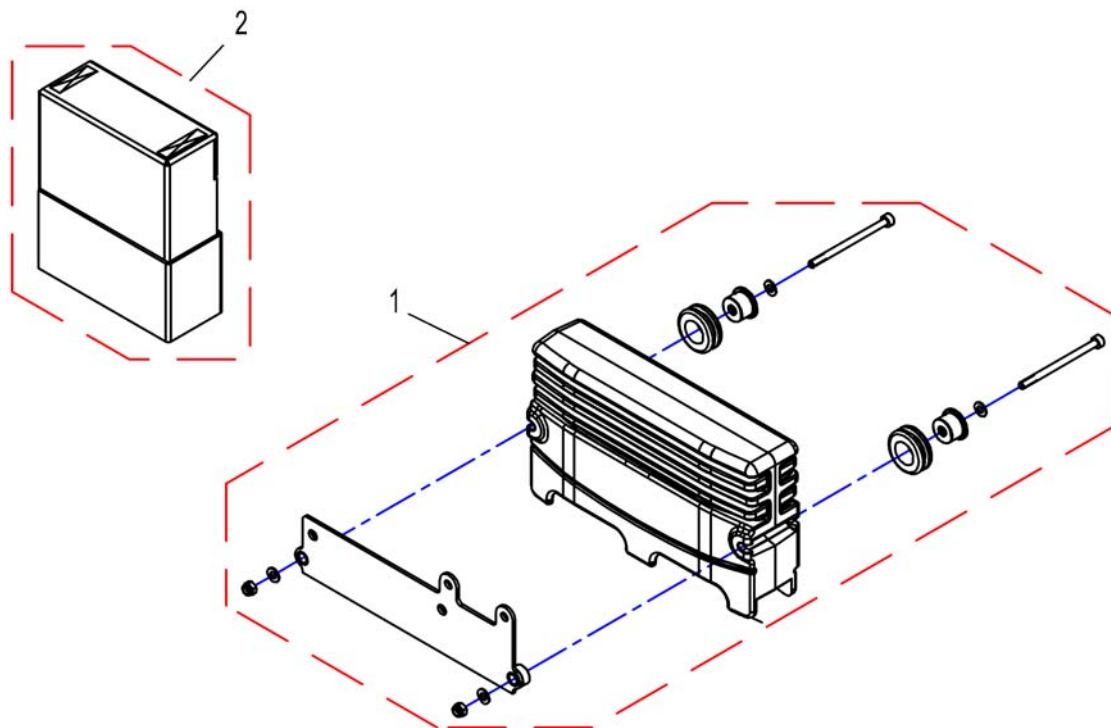


1	SP1529381	POWERMODUL TDX STORM4	1
1	SP1541534	POWERMODUL DX2 TDX M GYRO	1
2	SP1532518	HJULSKJERM TDX	1

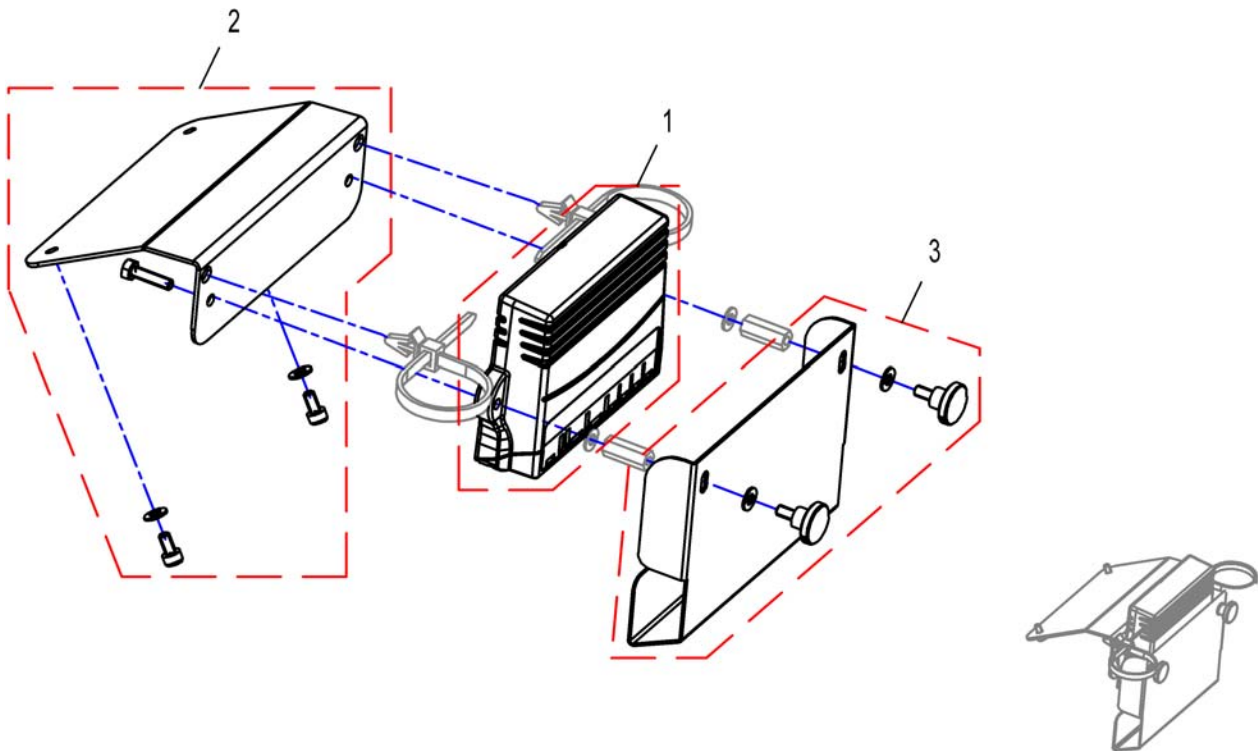


1	SP1529382	ACTUATORMODUL ACT4 DX2 STORM4 TDX BORA	1
1	SP1534955	(ACT2 for ACS2, cpl.)	1
2	SP1536213	HJULSKJERM TDX	1
3	SP1536273	HOLDER ELEKTRONIKK TDX	1
-	SP1536274	FESTE-KIT ERS	1

151183X.dwg

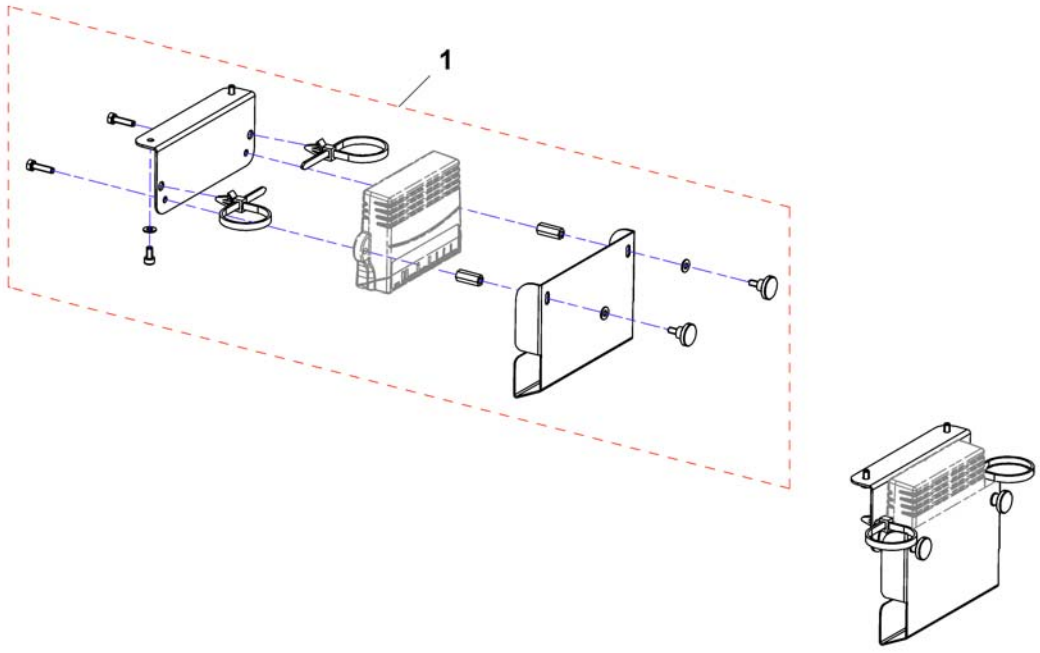

1 SP1554510 (Bracket ACT narrow base)

1

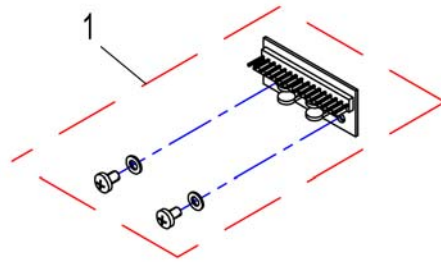


1 SP1529383

LYSFORDELER TDX


1

1537848X.dwg

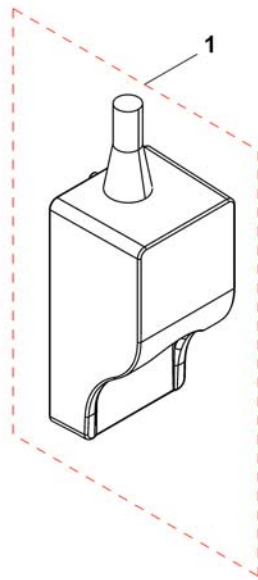

1 SP1541535 (G-Trac)


1

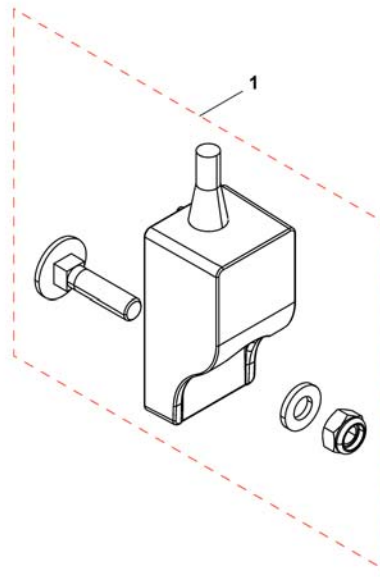

1 SP1541752

(Holder for G-Trac)


1

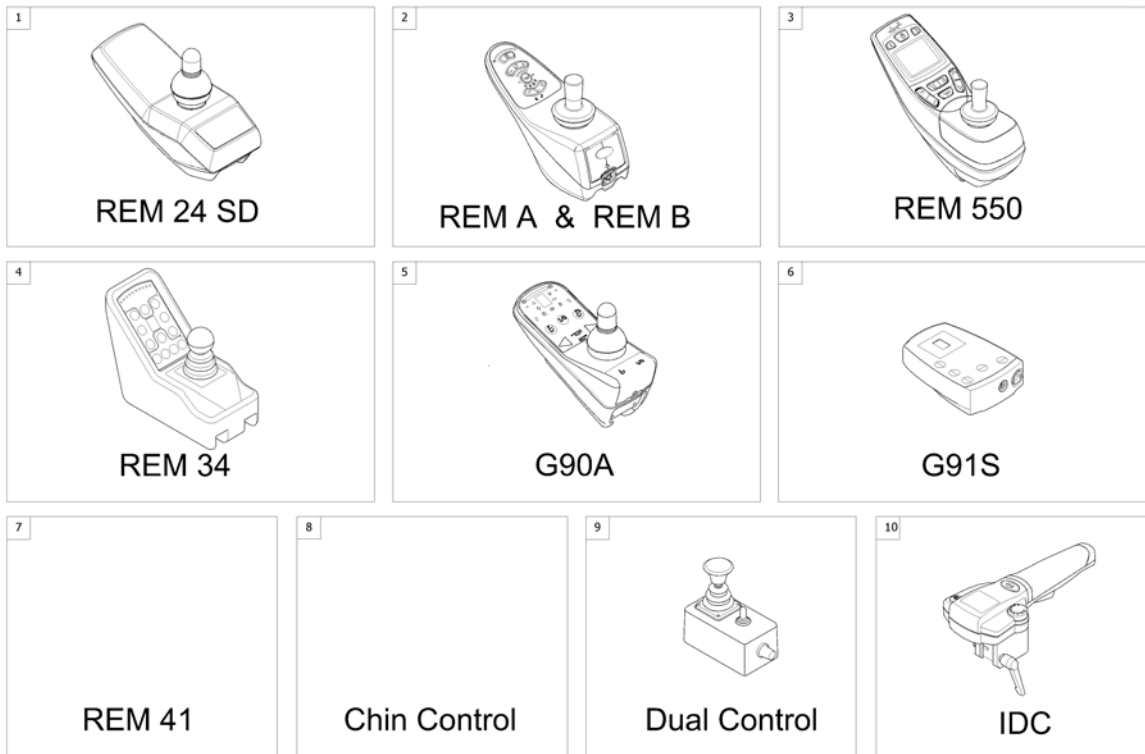


1	-	(REM 24 SD with brackets) [1601/1602]	1
2	-	(REM A & B with brackets) [1638/1639/1643/1644]	1
3	-	(REM 550 with brackets) [1617/1618]	1
4	-	(REM 34 with brackets) [1611]	1
5	-	(G90A with brackets) [1614/1616]	1
6	-	(G91S with brackets) [1615]	1
7	-	(Centre control REM 41) [1607/1608]	1
8	-	(Small Chin Control with brackets) [1609]	1
9	-	(Remote Dual Control with Brackets) [1291/1292]	1
10	-	(Intuitive Dual Control with brackets) [1645]	1

(REM 24 SD with brackets)

1562696 iso

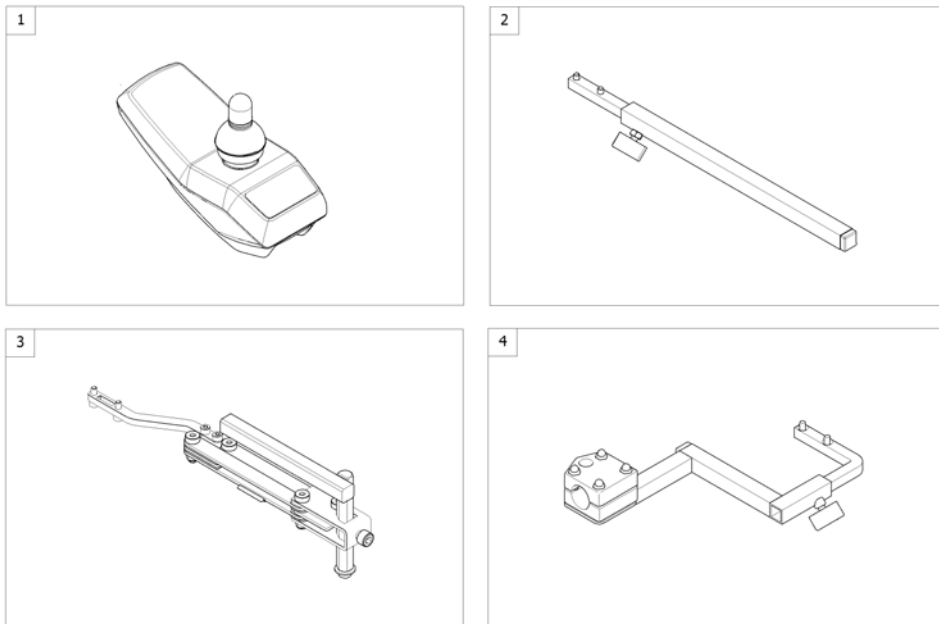


1	-	(Remote REM 24SD) <i>[1601/1602]</i>	0
2	-	(Remote Bracket Standard) <i>[1601+1207/1208]</i>	1
3	-	(Remote Bracket Swing away & height adjust.) <i>[1601+1707/1708]</i>	1
4	-	(Remote Bracket Attendant) <i>[1602+1207/1208]</i>	1

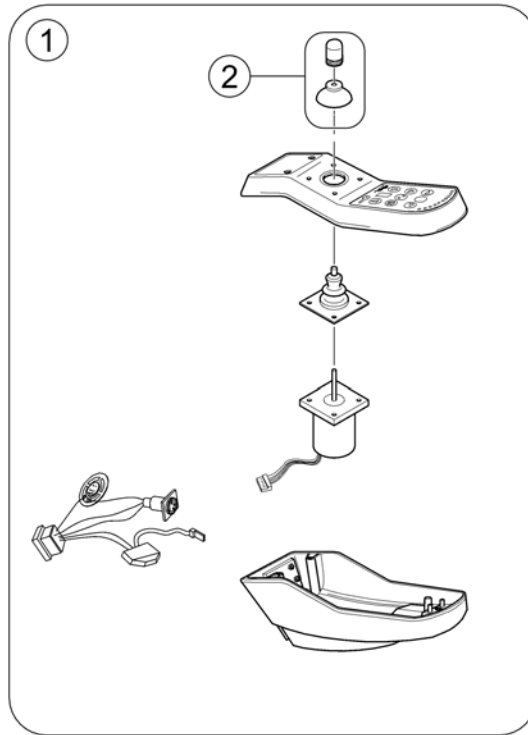

1	1436541	(Remote REM24SD)	1
2	SPZ183229	JOYSTIKK KOMPLETT	1

(Remote Bracket Standard)


1559851.iso

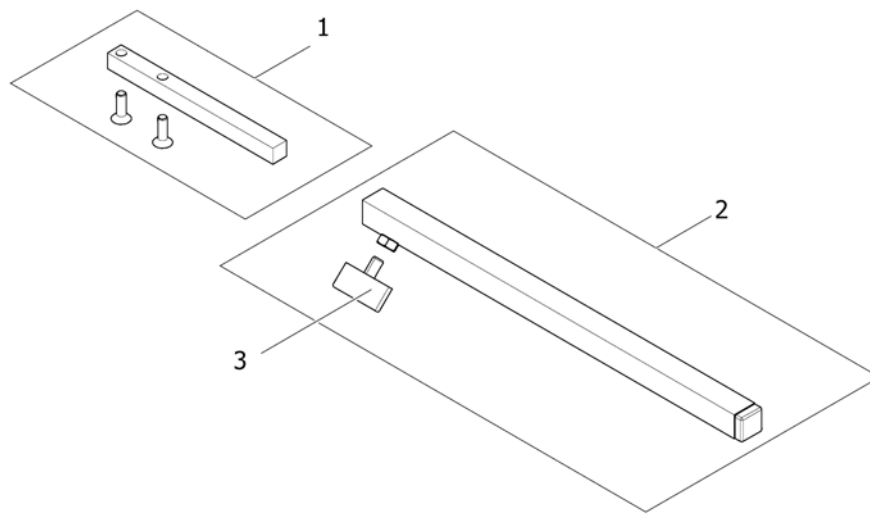

1	SP1436542	HOLDER DX-REM 24 SD RETT STORM SPORT/3 OPPFL.ARML	1
2	SPS183040	STYREBOKSHOLDER STD KPL STORM3 G40	1
3	L31001	VINGSKRUV M 6X10 GARANT G24 S	1

(Remote Bracket Swing away & height adjust.)


1559852.iso

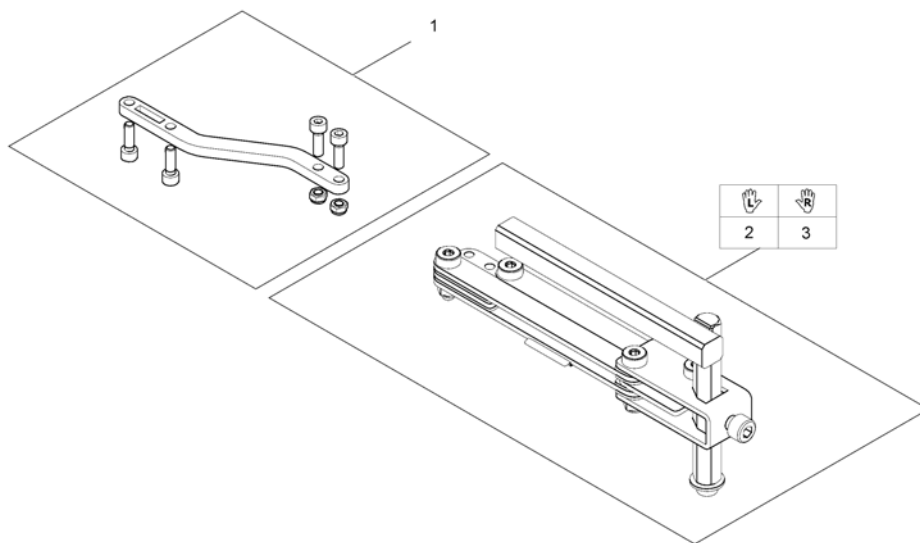


1	SP1498222	FESTE FOR STYREBOKST TIL SWING AWAY	1
2	SP1496249	SWING AWAY VE NY MOD2008 KPL	1
3	SP1496248	SWING AWAY HØ NY MOD2008 KPL	1

(Remote Bracket Attendant)

1560837.no

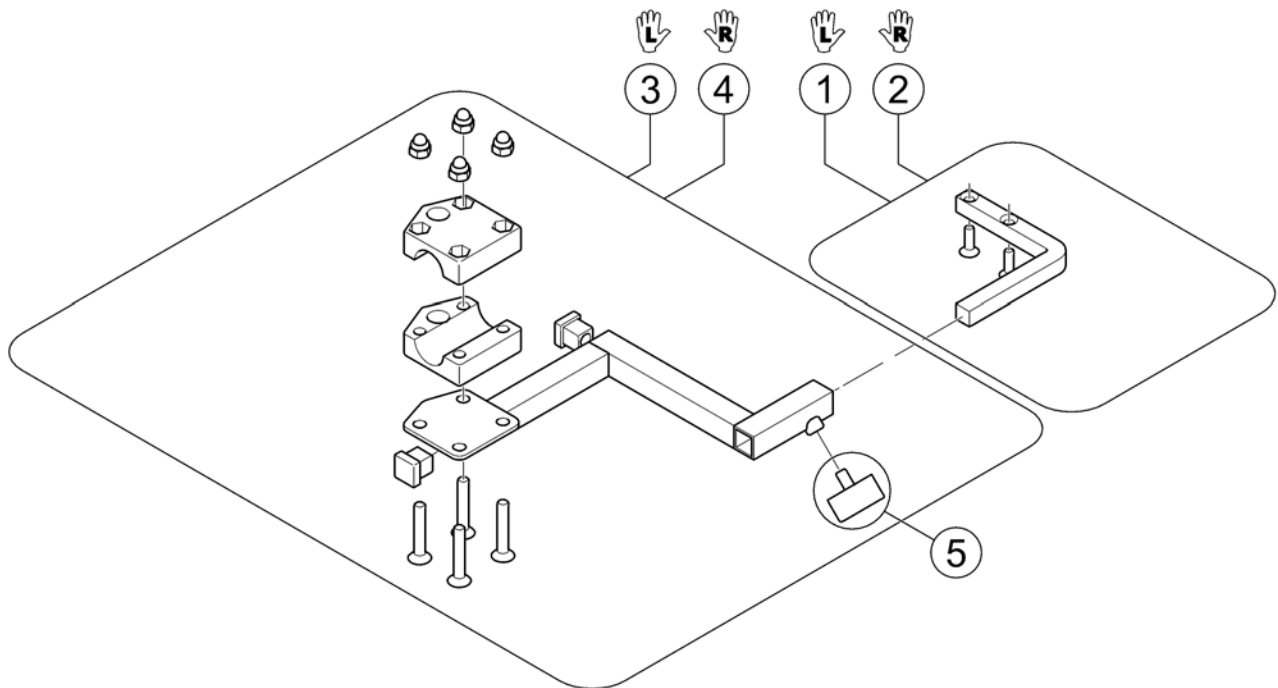


1	1422806	(Remote Support LH)	1
2	1422810	(Remote Support RH)	1
3	SP1422937	(Bracket for attendant control LH)	1
4	SP1422936	(Bracket for attendant control RH)	1
5	L31001	VINGSKRUV M 6X10 GARANT G24 S	1

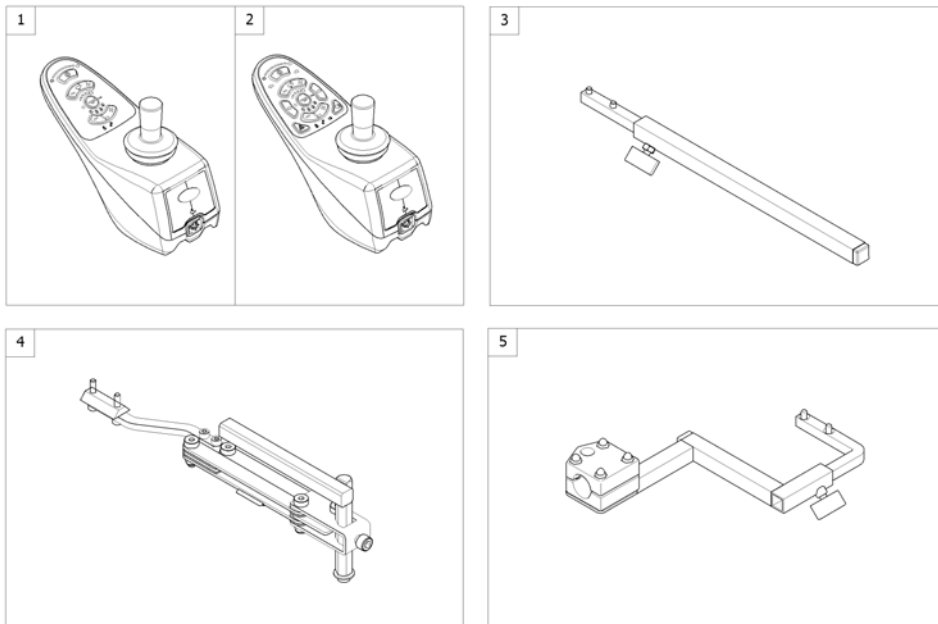


1	SP1523440	(Remote "REM A") [1638/1643]	1
2	SP1523445	(Remote "REM B") [1639/1644]	1
3	-	(Remote Bracket Standard) [1638/1639+1207/1208]	1
4	-	(Remote Bracket Swing away & height adjust.) [1638/1639+1707/1708]	1
5	-	(Remote Bracket Attendant) [1643/1644+1207/1208]	1

(Remote Bracket Swing away & height adjust.)



1559857.iso

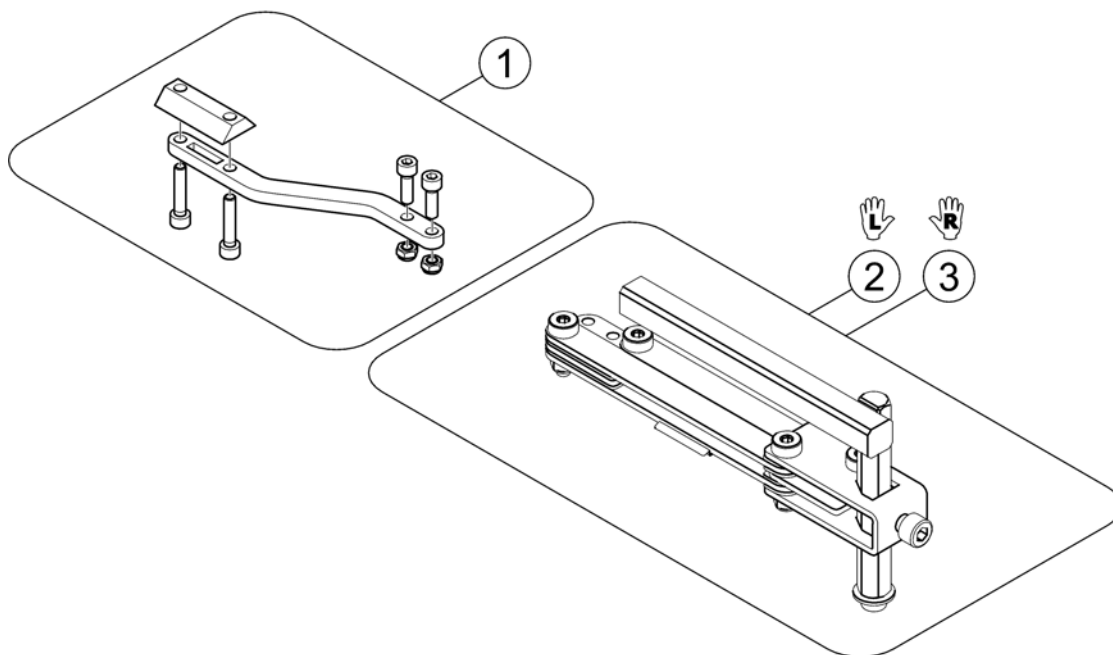


1	SP1536180	HOLDER STYREBOKS SHARK/REM A/REM B	1
2	SP1496249	SWING AWAY VE NY MOD2008 KPL	1
3	SP1496248	SWING AWAY HØ NY MOD2008 KPL	1

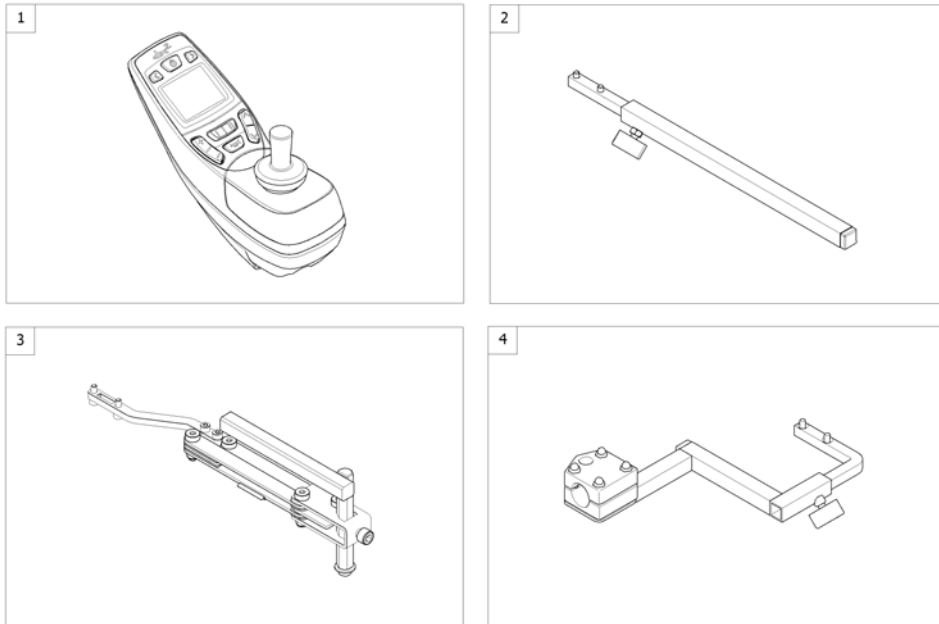

1	SP1523444	STYREBOKS REM550 STORM4 TDX BORA [1617/1618]	1
2	-	(Remote Bracket Standard) [1617+1207/1208]	1
3	-	(Remote Bracket Swing away & height adjust.) [1617+1207/1208]	1
4	-	(Remote Bracket Attendant) [1618+1207/1208]	1

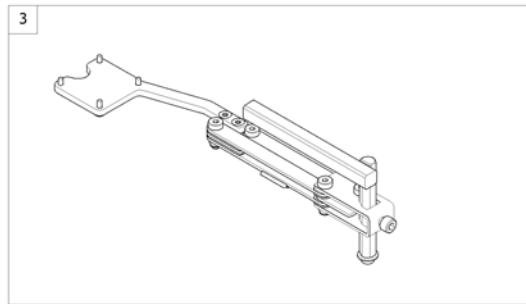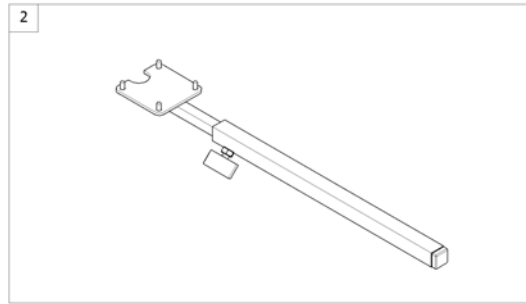


1	SP1440677	(Remote REM 34) [1611]	1
2	-	(Remote Bracket Standard) [1611+1207/1208]	1
3	-	(Remote Bracket Swing away & height adjust.) [1611+1707/1708]	1

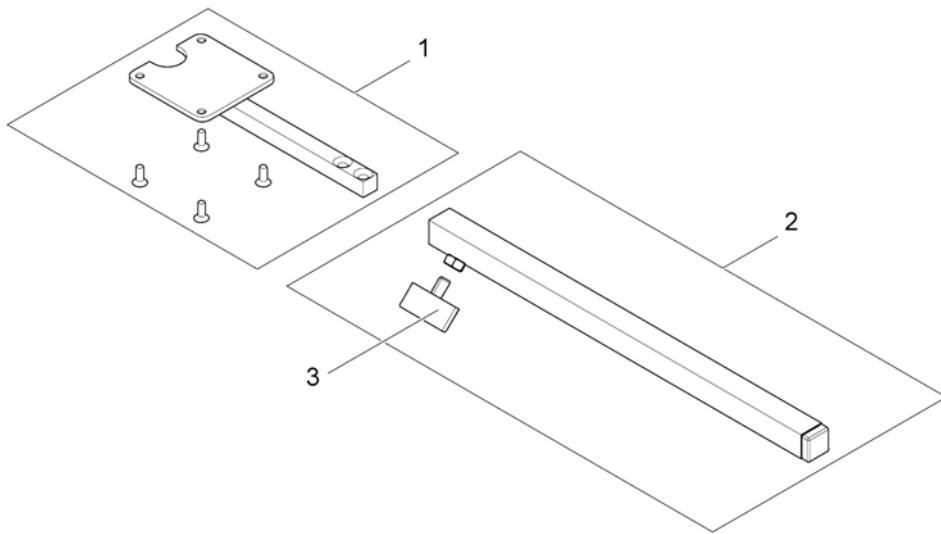


1	SP1434921	HOLDER FOR REM 34 STORM4	1
2	SPS183040	STYREBOKSHOLDER STD KPL STORM3 G40	1
3	L31001	VINGSKRUV M 6X10 GARANT G24 S	1

(Remote Bracket Swing away & height adjust.)

1560845.iso

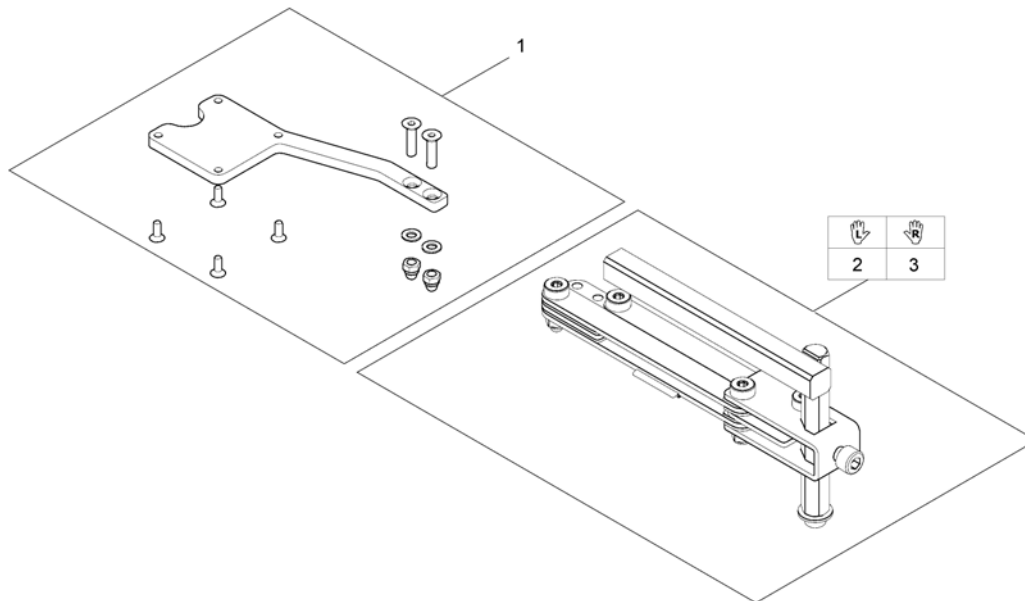


1	SP1527125	FESTE FOR REM34 STORM4	1
2	SP1496249	SWING AWAY VE NY MOD2008 KPL	1
3	SP1496248	SWING AWAY HØ NY MOD2008 KPL	1

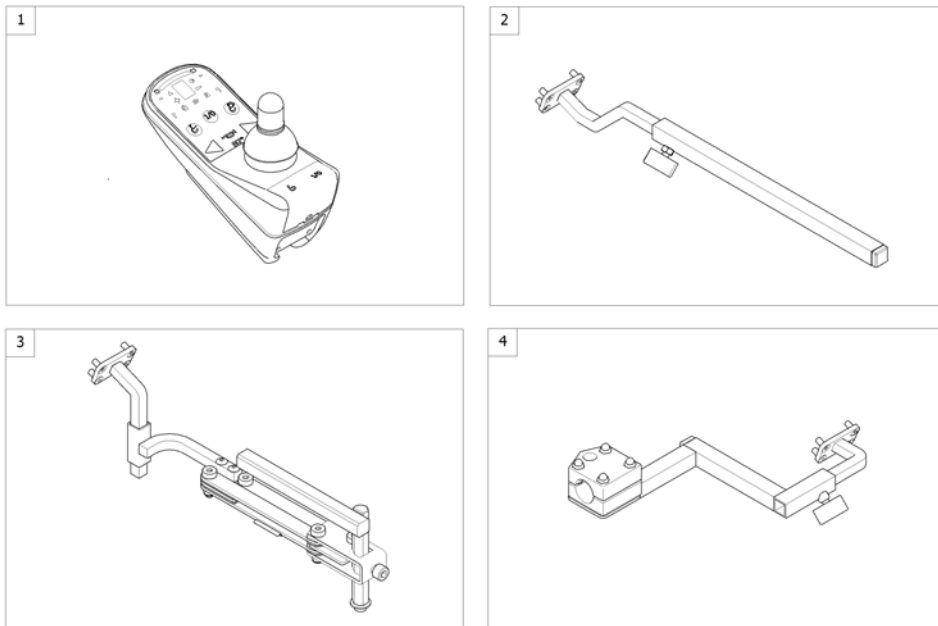

1	SP1504720	STYREBOKS DX-REM G90A STORM4 [1614/1616]	1
2	-	(Remote Bracket Standard) [1614+1207/1208]	1
3	-	(Remote Bracket Swing away & height adjust.) [1614+1707/1708]	1
4	-	(Remote Bracket Attendant) [1616+1207/1208]	1

(Remote Bracket Standard)


1562939.iso

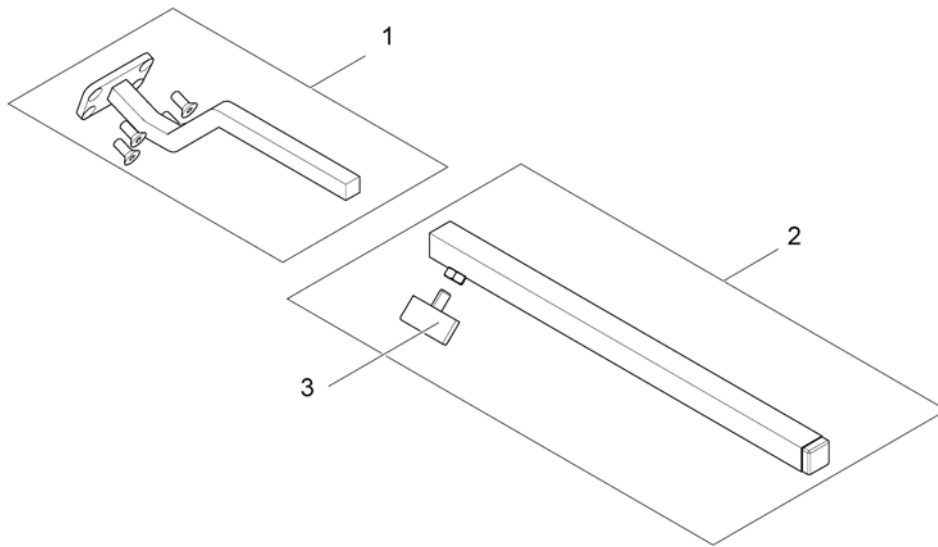

1	SP1504822	(Remote Support Standard)	1
2	SPS183040	STYREBOKSHOLDER STD KPL STORM3 G40	1
3	L31001	VINGSKRUV M 6X10 GARANT G24 S	1

(Remote Bracket Swing away & height adjust.)


1560310.iso

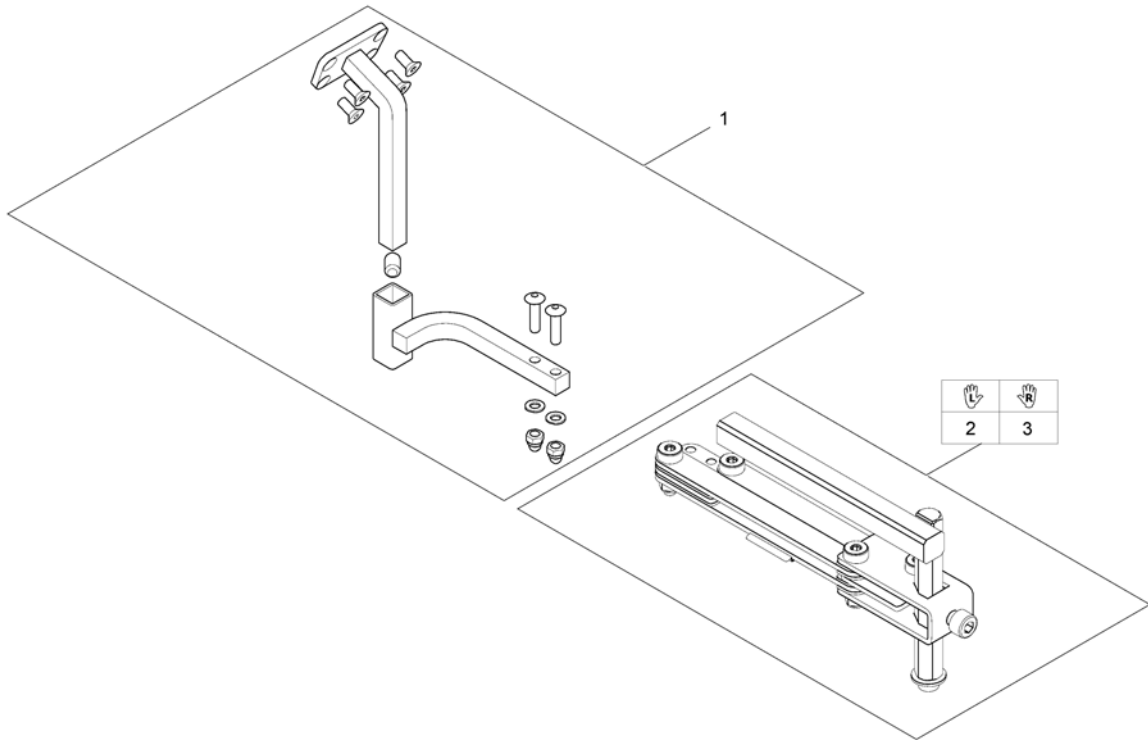



1	SP1517352	FESTE FOR G90 SWINGAWAY STORM4 TDX BORA	1
2	SP1496249	SWING AWAY VE NY MOD2008 KPL	1
3	SP1496248	SWING AWAY HØ NY MOD2008 KPL	1

(Remote Bracket Attendant)

1560853.iso

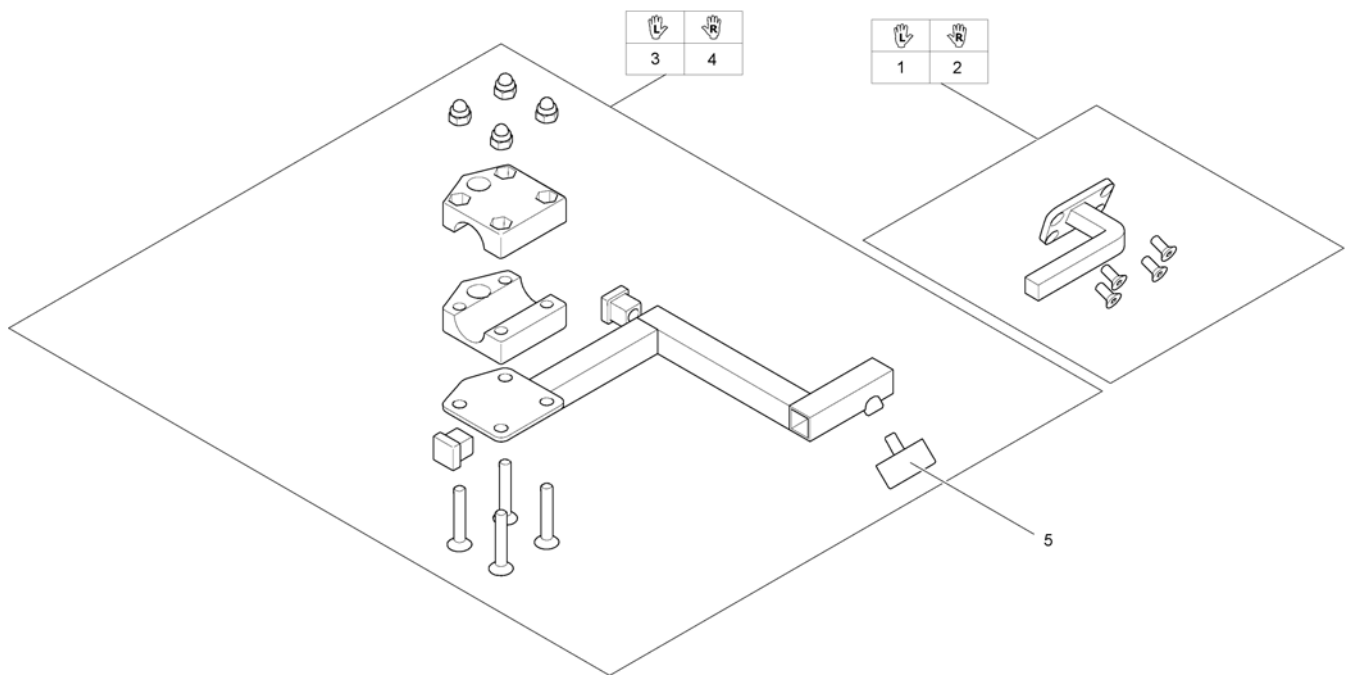


1	SP1509435	(Remote Support attendant control G90A, LH)	1
2	SP1509439	(Remote Support attendant control G90A, RH)	1
3	SP1422937	(Bracket for attendant control LH)	1
4	SP1422936	(Bracket for attendant control RH)	1
5	L31001	VINGSKRUV M 6X10 GARANT G24 S	1

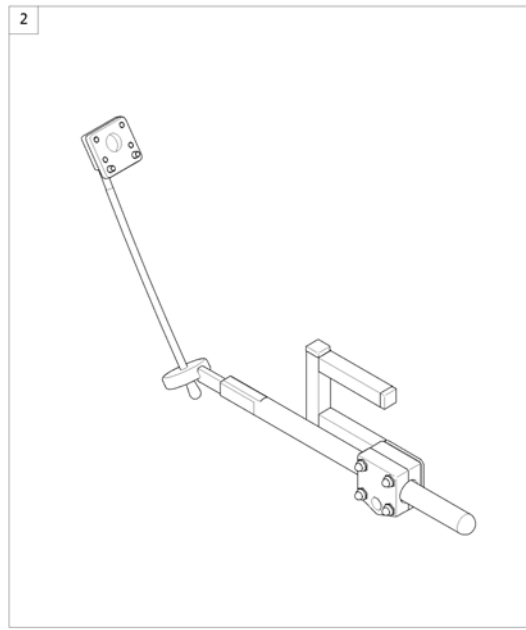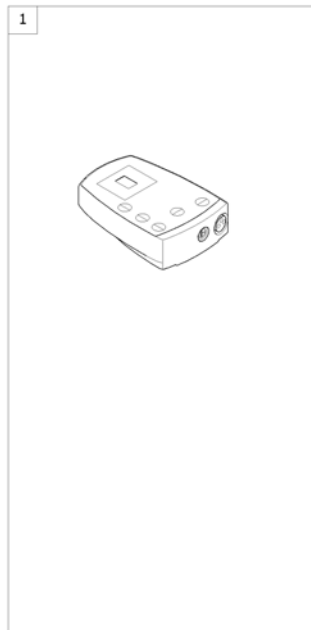

1	SP1505187	(Remote Control G91S) [1615]	1
2	-	(Remote Bracket Standard) [1615+1207/1208]	1


(Remote Bracket Standard)

1560654.iso

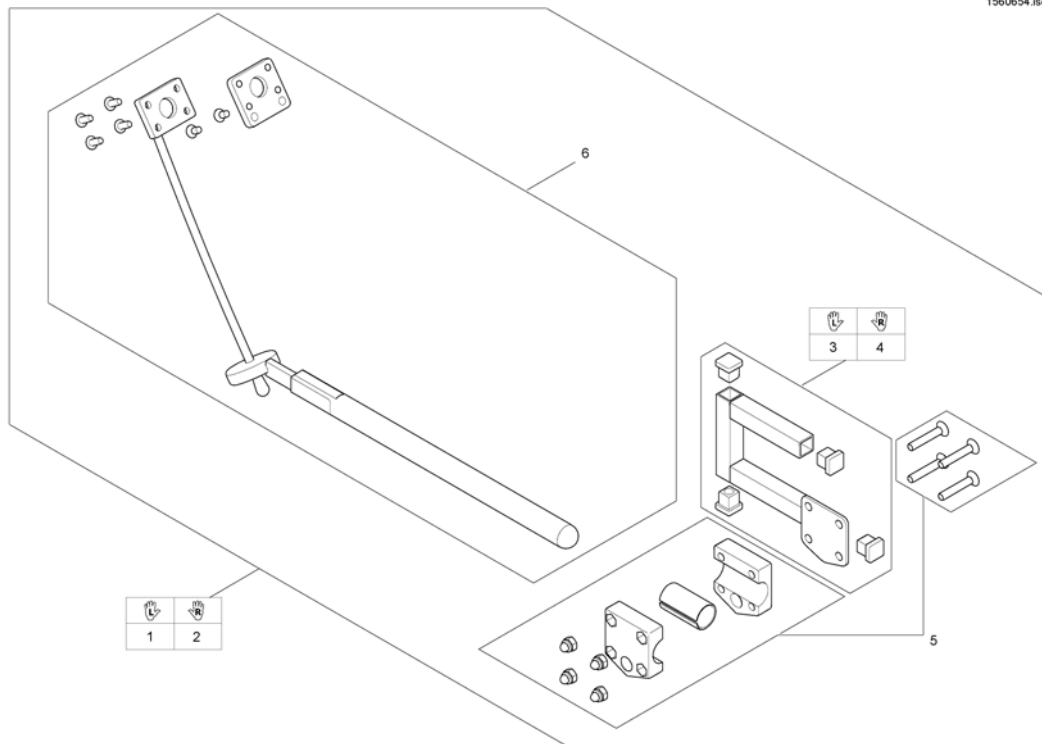

1	SP1509718	HOLDER FOR DX-REMG91 VE JUSTERBAR HØYDE/DYBDE	1
2	SP1509690	HOLDER FOR DX-REMG90 HØ JUSTERBAR HØYDE/DYBDE	1
3	SP1509716	(Adapter G91 LH)	1
4	SP1509711	(Adapter G91 RH)	1
5	SP1527503	(Gripper clamp cpl.)	1
6	SP1527504	(Swan neck remote bracket with adapter plate)	1


F383057.dwg

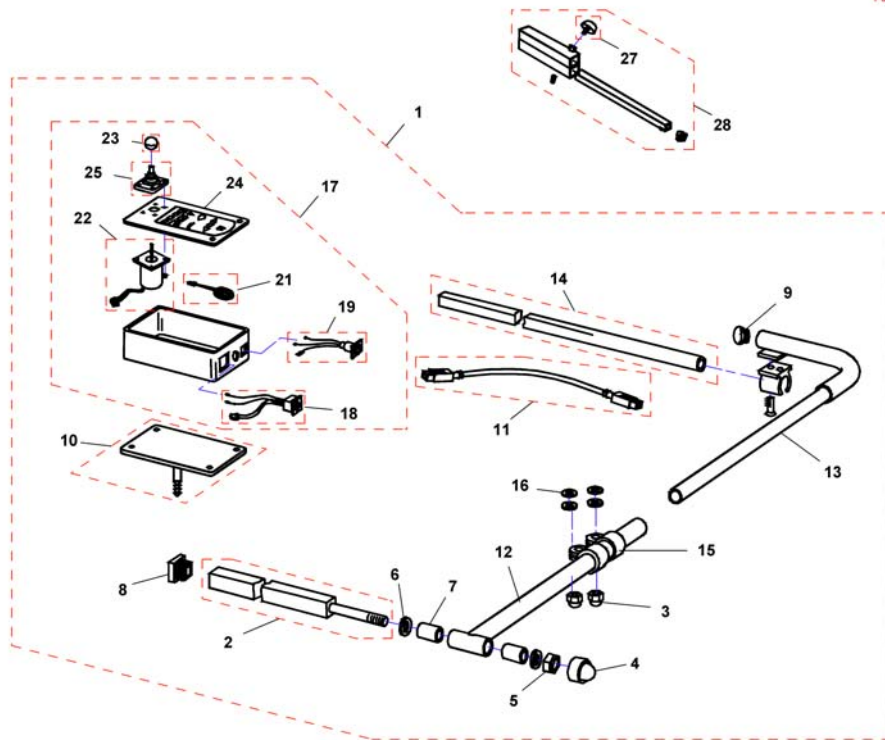

-	Q99081	(User Manual Table- & Middle Control REM41-S German)	1
1	SP1530588	DX-REM41 MED SENTERFESTE TDX	1
1	SP1530589	(CENTRAL STEERING CPL.; SWING-AWAY LH, REM41-D)	1
2	Z177024.00	FESTE F MITTSTYRNING,	1
3	F60208	(Cap nut DIN1587-M08 zc)	1
4	L01037	(Sks Nut Cap M12)	1
5	F20106	MUTTER SEKSKANT SELVLÅSENDE G40	1
6	G00104	SKIVE 13X24X2,5 G40	1
7	Z17032A	(Bush Rd. 19 X 3.5 X 25)	1
8	L01055	ENDESTYKKE FESTEBR.	1
9	L01008	(Tube End Plug SI 22)	1
10	Z183216.P40	FESTE F MANBOX MITTST,	1
11	1559862	KABEL BUS 1,7 M	1
12	Z183089.00	BREDD JUST ARM MITTST,	1
13	Z183086.00	ÄNDRÖR HÖ, STORM	1
13	Z183092.00	ÄNDRÖR VÄ, STORM	1
14	H281005A.00	(MIRROR SUPPORT RAIL; CHROME PLATED)	1
15	5006720	KLEMME 22MM TDX	1
16	G00106	SKIVE M8 DRIVHJUL MIRAGE	1
17	SP1527930	BORDSTYRING DX-REM41-D DOLPHIN MASTER	1
18	P41108	KONTAKT BUS KABEL DX-REM41	1
19	P41106	(CHARGER SOCKET)	1
21	P30493	(HORN)	1
22	GSM55120	(ACS-remote WITH 180MM CABLE)	1
23	L36999	(KNOB FOR remote 32 MM, HOLE 5.8 MM)	1
24	-	(No longer available.)	0
25	Z183001A	JOYSTICK GUMMI ACS	1
26	Z183289	SENTERSTYRT STØTTE RØR KPL	1
27	L31001	VINGSKRUV M 6X10 GARANT G24 S	1
28	Z183289	SENTERSTYRT STØTTE RØR KPL	1


(Small Chin Control with brackets)

-	(Small Chincontrol)	1
-	(Framework for chincontrol)	1

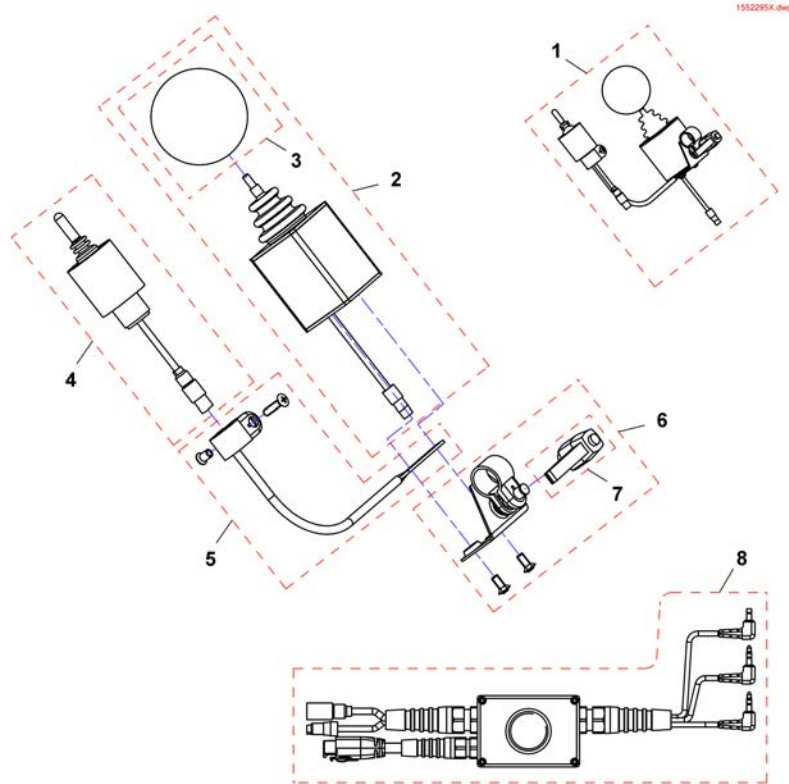

1	1448705	\$HAKESTYRING KPL MONT EKSTRA JOYSTICK	1
1	1421770	(Chin control G80/G90, cpl.)	1
1	SP1544255	(Chin control G91, cpl.)	1
2	Z183282	(Joystick, kmpl.)	1
3	Z183265	BRUK GJS26826 DYNAMICS	1
4	1442943	(Switchjoystick REM24, kmpl.)	1
4	Z183283	(Switchjoystick G80/G90, cpl.)	1
4	SP1544250	(Switchjoystick G91, kmpl.)	1
5	SP1552294	(Switchjoystick bracket)	1
6	SP1552295	(Joystick clamp)	1
7	L32901	SPAK M8 STORM4	1
8	SP1442944	(Harness REM24)	1
8	Z183275	(Harness G80/G90)	1
8	SP1541715	(Harness G91)	1




(Framework for chincontrol)

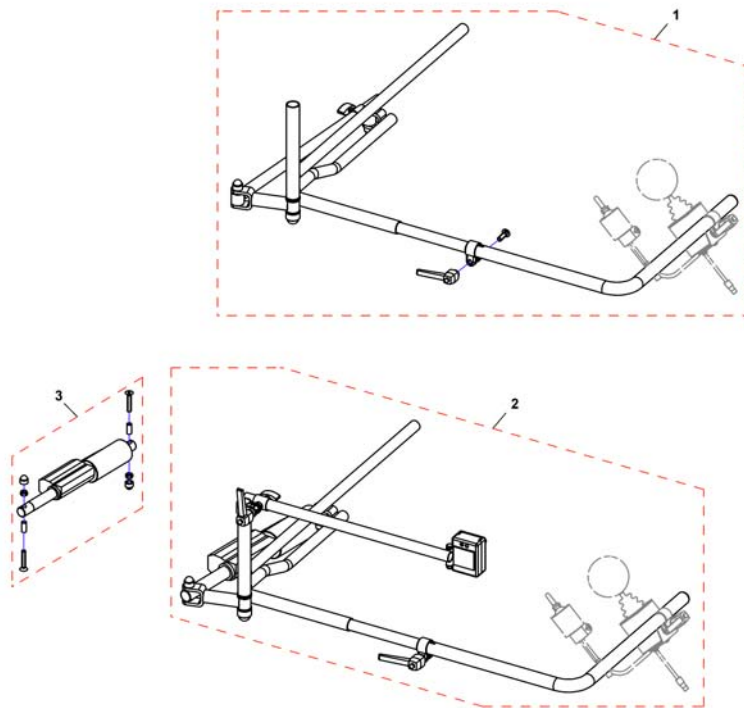

-	Z167994	(CHIN CONTROL SUPPORT . CPL.) <i>Not shown / Nicht abgebildet / Non illustré</i>	1
1	SP1422943	(Framework for chin control, manual, horizontal swing away, LH, cpl.)	1
1	SP1422942	(Framework for chin control, manual, horizontal swing away, RH, cpl.)	1
2	SP1422945	HOLDER HAKEBETJ EL HORIS SVINGAWAY VE KPL <i>incl. Actuator</i>	1
2	SP1422944	(Framework for chin control, powered, horizontal swing away, RH, cpl.) <i>incl. Actuator</i>	1
-	SP1552335	(Hardware, chin control, powered, horizontal swing away, LH) <i>Not shown / Nicht abgebildet / Non illustré</i>	1
-	SP1552336	(Hardware, chin control, powered, horizontal swing away, RH) <i>Not shown / Nicht abgebildet / Non illustré</i>	1
-	SP1422946	(Framework for chincontrol, powered, vertikal swing away) <i>Not shown / Nicht abgebildet / Non illustré</i>	1
3	SP1552337	(Actuator for horizontal swing away chin control, cpl.)	1
3	SP1552418	(Gestell für Kinnbedienung, elektrisch, vertikal abschenkbar) <i>Not shown / Nicht abgebildet / Non illustré</i>	1

(Remote Dual Control with Brackets)


1574292 iso

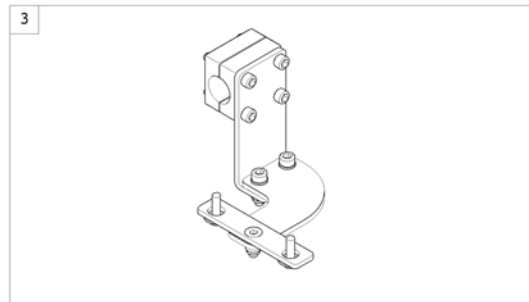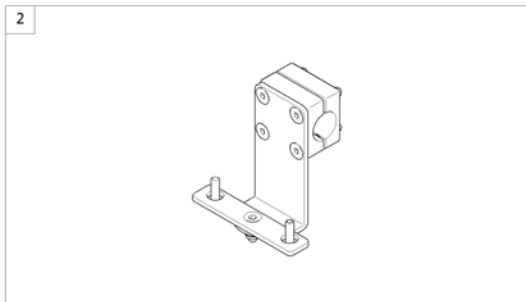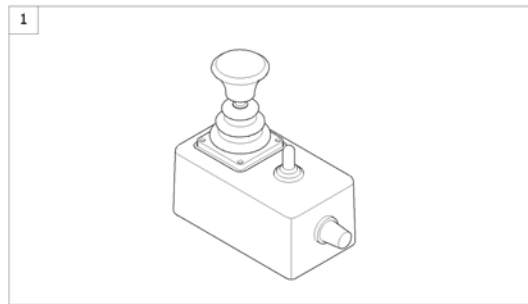

1	6001425	(ACS dual control) [1291/1292]	1
2	-	(Remote Bracket Dual Control for Standard-/Contour-/Flex-/Junior-Seat)	1
3	-	(Remote Bracket Dual Control for Modulite Seat)	1

(Remote Bracket Dual Control for Standard-/Contour-/Flex-/Junior-Seat)

1560885.iso

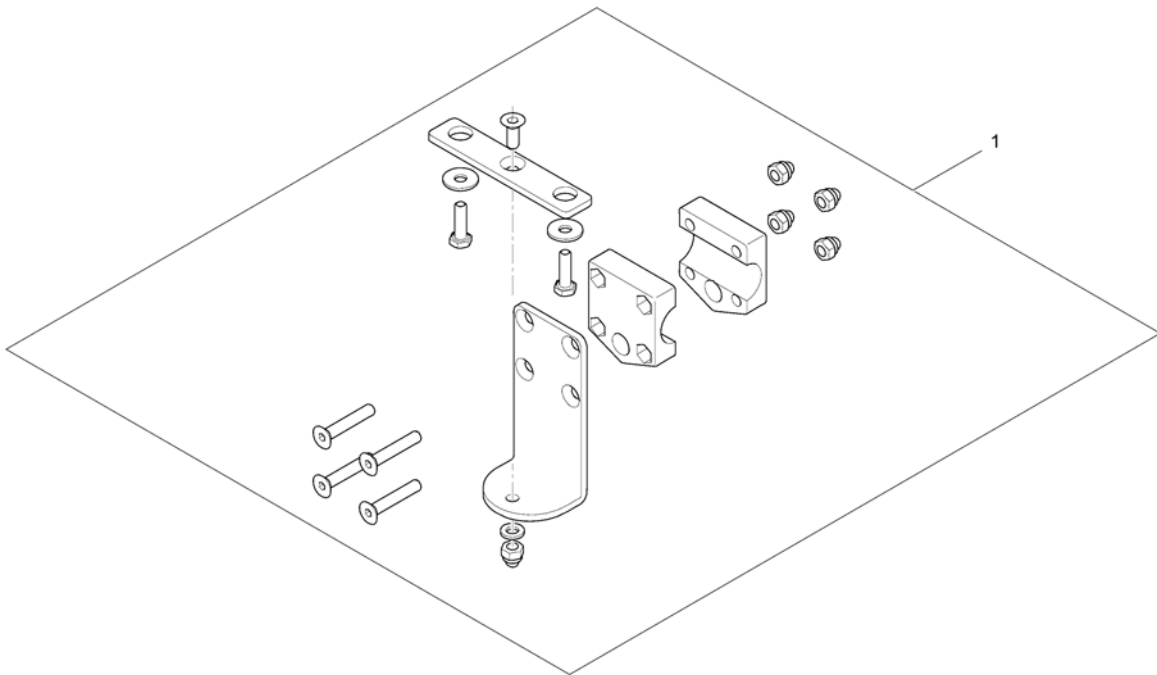


1 SP1422818

FESTEBRAKETT LEDSAGERSTY RYGGMONT ENEKL

1

(Remote Bracket Dual Control for Modulite Seat)


1574291 iso

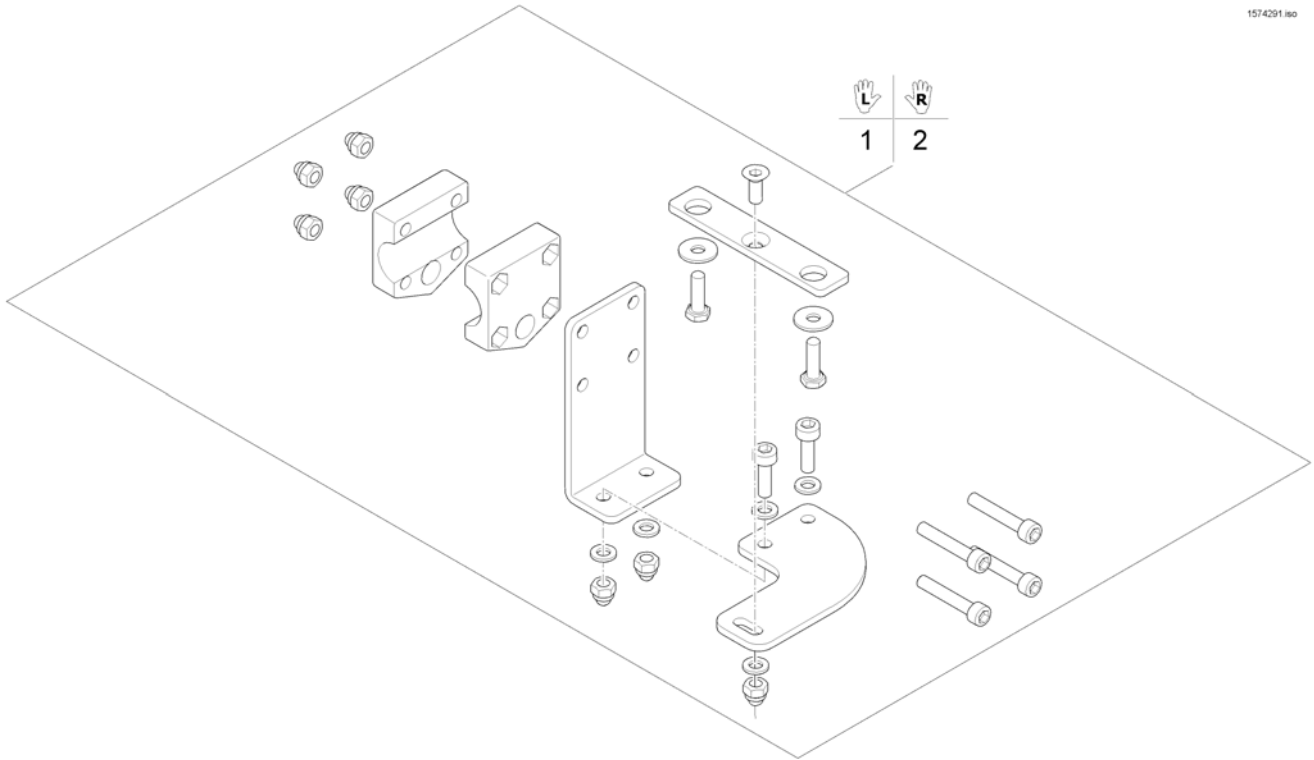

1	SP1571799	(Remote Bracket Dual Control LH)	1
2	SP1571800	(Remote Bracket Dual Control RH)	1

(Intuitive Dual Control with brackets)

1509875X.iso

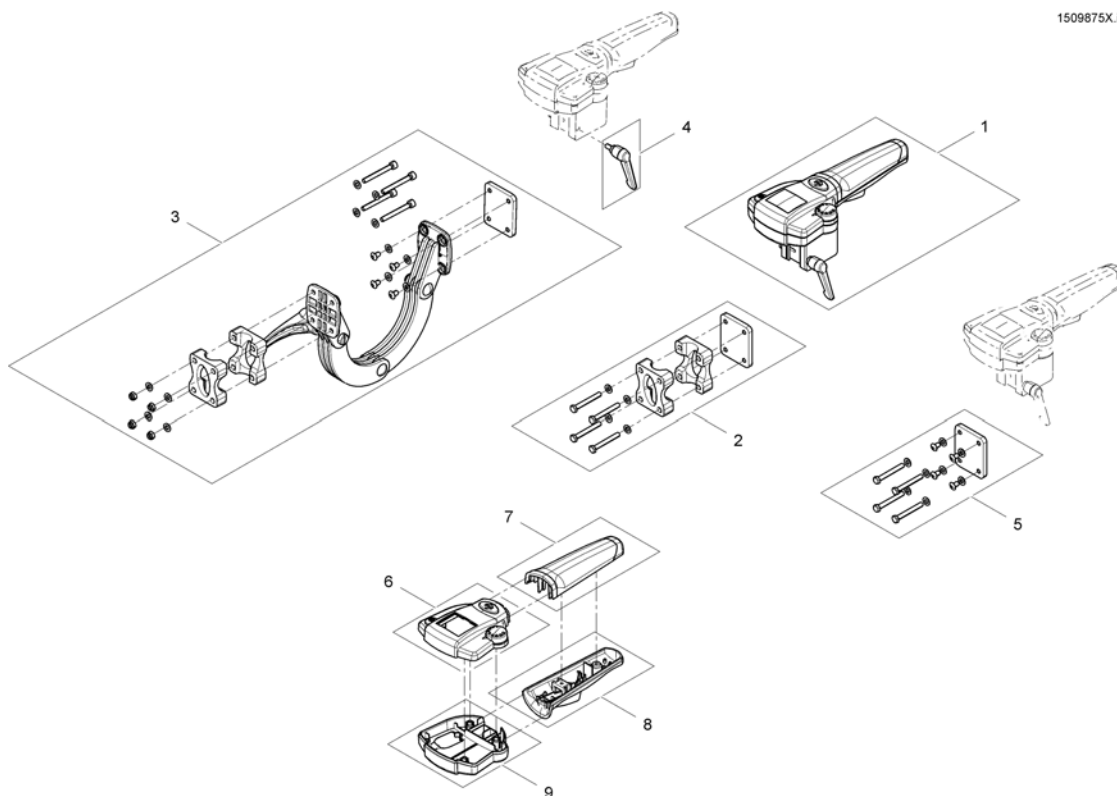


1	SP1509875	LEDSAGERSTYRING INTUITIV ALLE STOLER MED ACS <i>For chairs built before 01/02/10 please also order SP1537148</i>	1
2	SP1522310	KLEMME RYGGRØR IDC FESTE INTUITIV LEDESAGERSTYRING	1
3	SP1521488	HOLDER INTUITIV LEDESAGER RYGGER MED RUNDT RØR	1
4	SP1510135	FESTE IDC MED FINGERSKR STORM4	1
5	SP1537148	(Supportplate IDC for chairs before 01.02.10)	1
6	SP1579573	(Upper Housing)	1
7	SP1511060	(Handhold upper part)	1
8	SP1511099	(Handhold lower part)	1
9	SP1510470	DEKSEL INT LEDESAGERST ACS NEDRE DEKSEL	1

-	(Bus-Cables ACS)	1
-	(T-Adapter)	0
-	(T-Adapter - Modulte Seat)	1
-	(Programming Kits)	1
-	(Magnetkey for Remotes)	1


(Bus-Cables ACS)

ACSBUSKABEL.iso

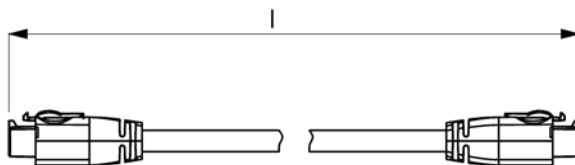

Position	length l [mm]
1	300
2	500
3	700
4	1100
5	1700
6	2000
7	2500
8	2700
9	1060

1	GSM63003	KABEL BUS 0,3M ACS,	1
2	SPGSM63005	(Bus cable ACS, 500mm) <i>Replace GSM63004</i>	1
3	GSM63007	(Bus cable ACS 700mm)	1
4	GSM63011	KABEL-BUS 1,1M ACS, ST utgått bruk 63010	1
5	1559862	KABEL BUS 1,7 M <i>Replace GSM63016</i>	1
6	GSM63020	DX BUS CABLE 2.0M STORM <i>Replace 1419386</i>	1
7	1440424	(Bus cable ACS, 2500mm)	1
8	SP1559873	(Bus cable ACS, 2700mm) <i>Replace GSM63026</i>	1
9	SP1550402	(Extension cable ACS, 1060mm)	1

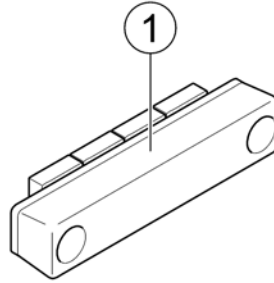


1 P10072

(ACS-Bus T-Adapter (Splitter))


1

(T-Adapter - Modulite Seat)


1583360 iso

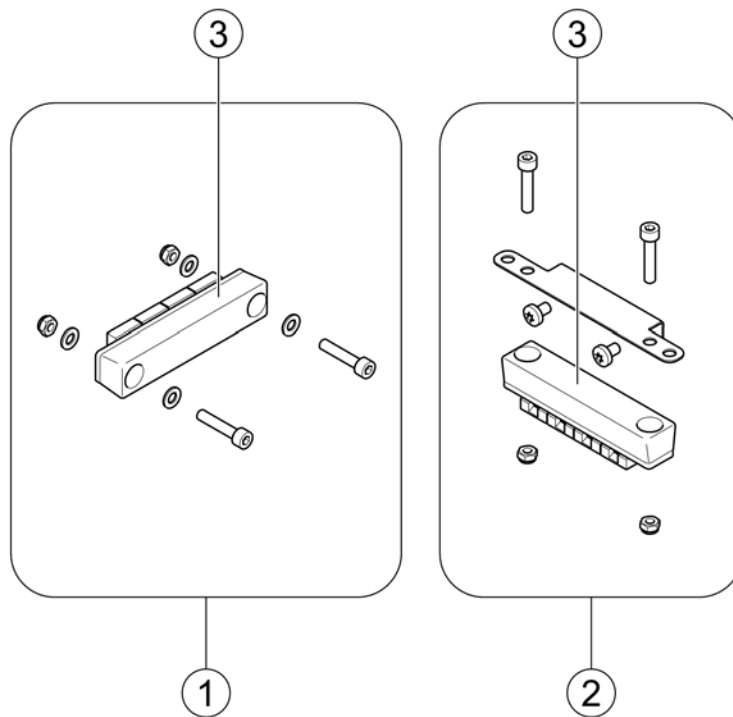

1	SP1571826	(T-Adapter - Modulite Seat)	1
2	SP1569958	(T-Adapter - Modulite Seat with MBL-Modul)	1
3	P10072	(ACS-Bus T-Adapter (Splitter))	1

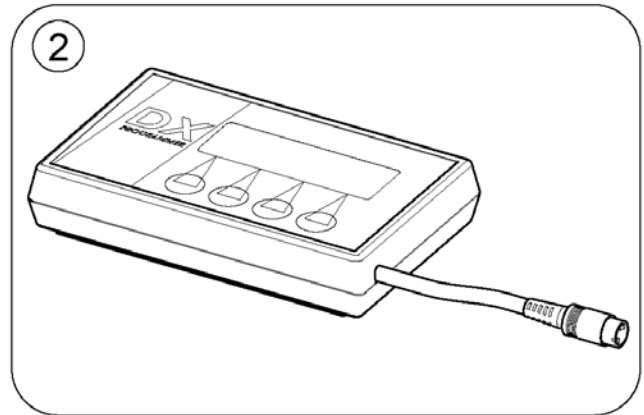

1	1449280	(USW wizard dongle kit, EDL-USW, cpl.)	1
2	6001450	(Hand Programmer, Instrucions In French, German, Spanish)	1
	SP1441551	(Programming Adapter Shark DK-ADAPT (necessary to program Shark))	1

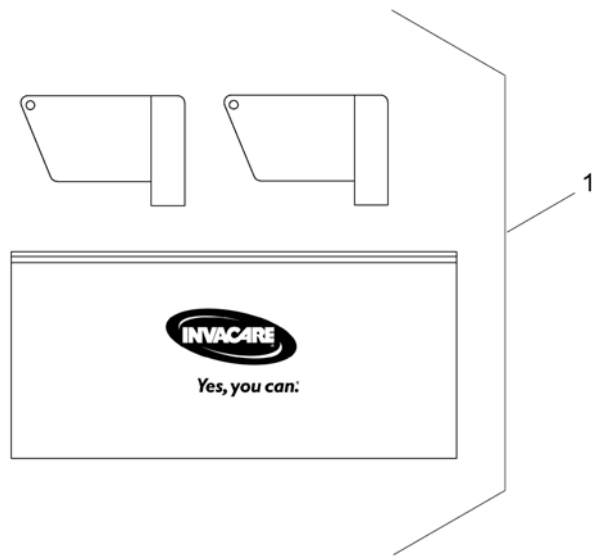

1 SP1419695 (Magnet key with bag)

1

1500784.no

1
Modules


2
**Remote
& Brackets**

3
**Cables &
Accessories**

1	-	(Shark Electronic Modules)	1
2	-	(Shark Remote & Brackets)	1
3	-	(Shark Cables & Accessories)	1

(Shark Electronic Modules)


-	(Shark Powermodule)	1
-	(Lightboard ACS2)	1

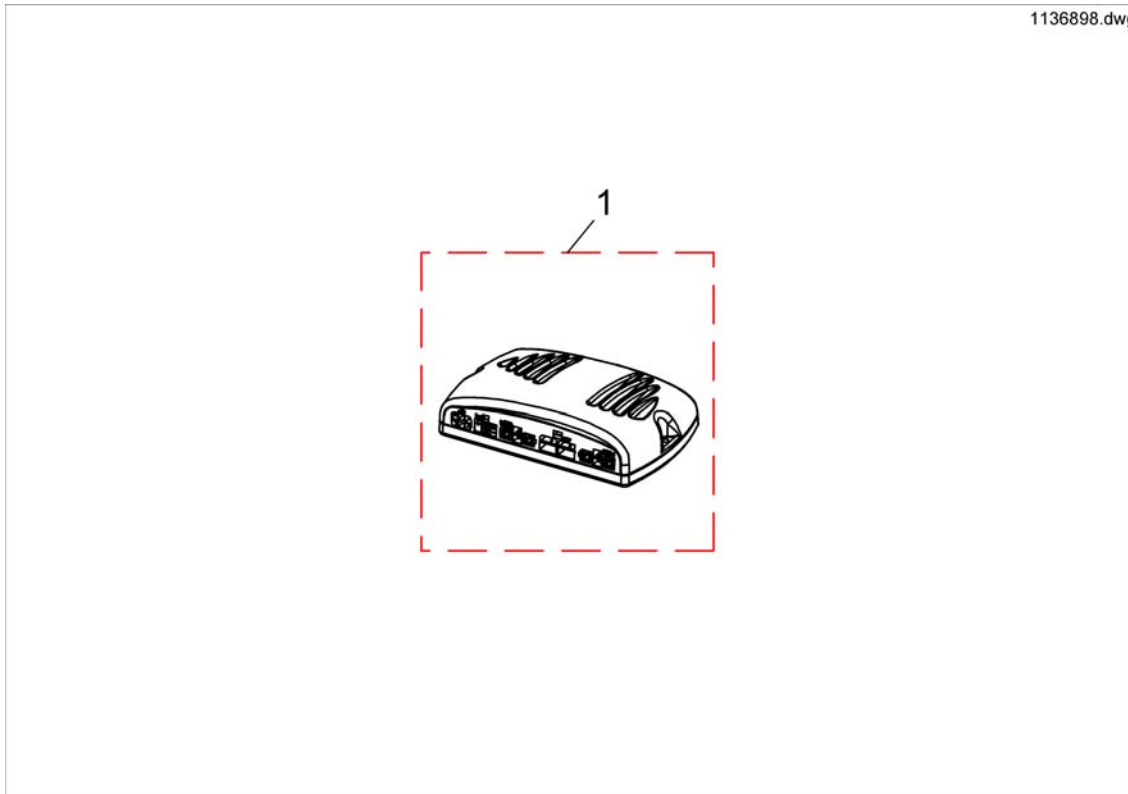

-	1450313	BUSKABEL 1500 MM SHARK	1
1	1136898	(Powermodule, Shark, 90A, 2 actuators, DK-PMC28)	1

(Shark Remote & Brackets)


1560925 no

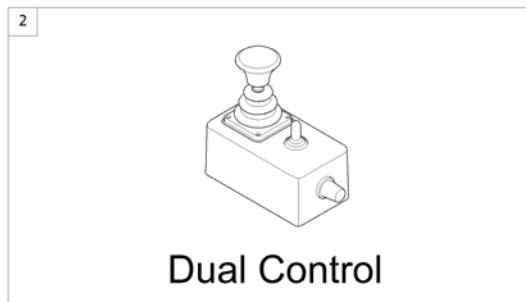

1	-	(Remote Shark with Brackets) <i>[1801]</i>	1
2	-	(Remote Dual Control with Brackets) <i>[1891/1892]</i>	1

(Remote Shark with Brackets)



1563326 iso

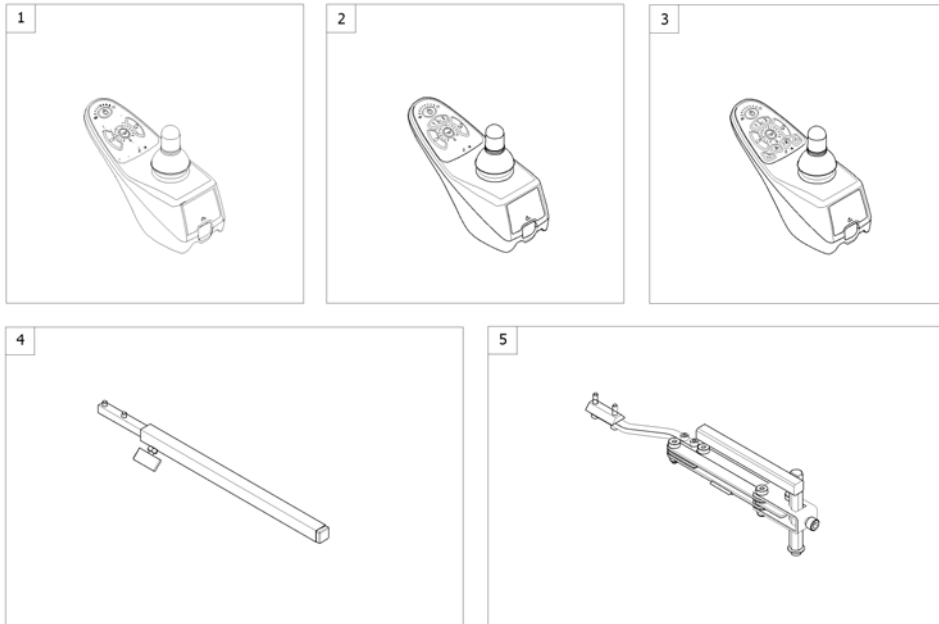


1	1492567	STYREBOKS SHARK DKREMD01 DRAGON <i>[1801]</i>	1
2	1449177	STYREBOKS SHARK II DX-REMD11 DRAGON <i>[1801]</i>	1
3	1484904	STYREBOKS SHARK II DX-REMD11 LYS OG EL DRAG <i>[1801]</i>	1
4	-	(Remote Bracket Standard) <i>[1801+1207/1208]</i>	1
5	-	(Remote Bracket Swing away & height adjust.) <i>[1801+1707/1708]</i>	1

(Remote Dual Control with Brackets)

1563329.iss

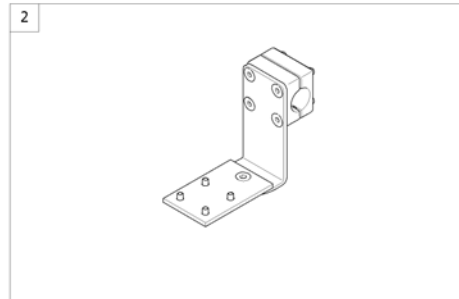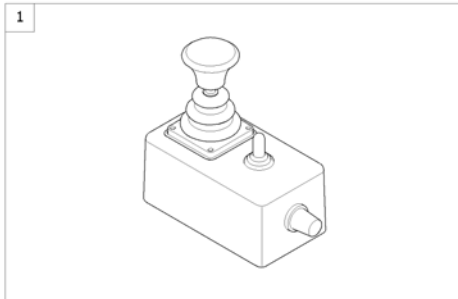

1	1444717	(Shark Dual Control, switchable) <i>[1891/1892]</i>	1
2	-	(Remote Bracket Dual Control) <i>[1891/1892]</i>	1

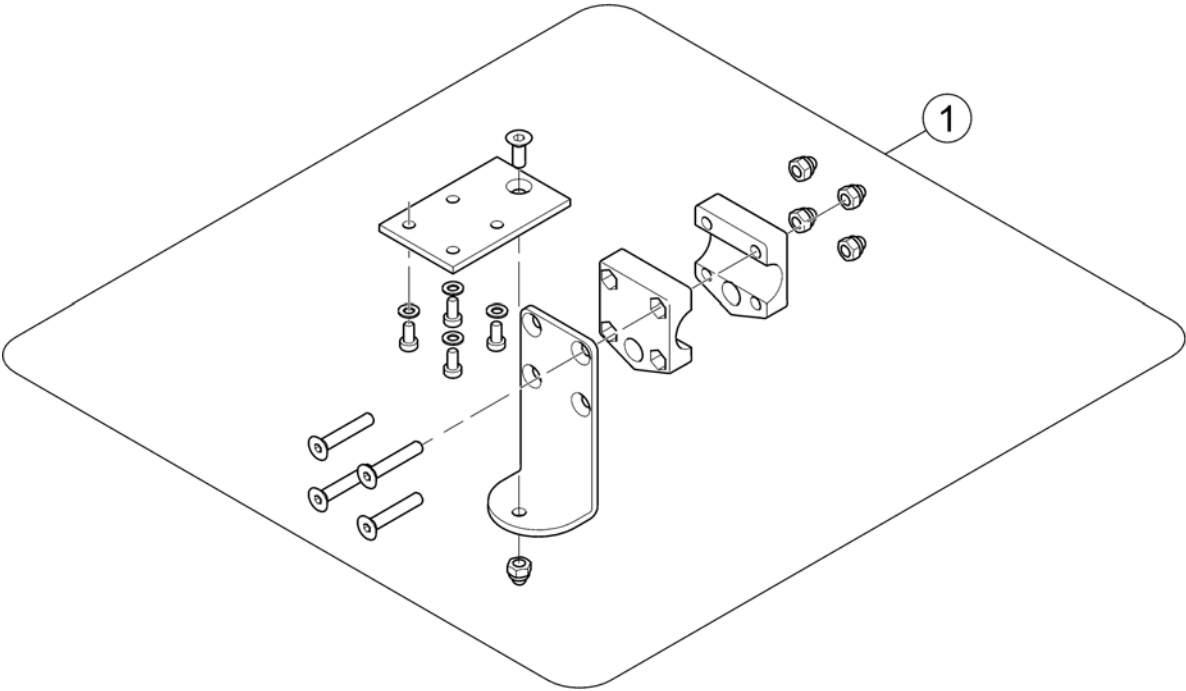

1 1450307


(Remote Bracket Attendent Control Shark)

1


(Shark Cables & Accessories)

-	(Bus-Cables Shark)	0
-	(Cables Shark)	0
-	(Programming Kits)	0

SHARKBUSKABEL.00

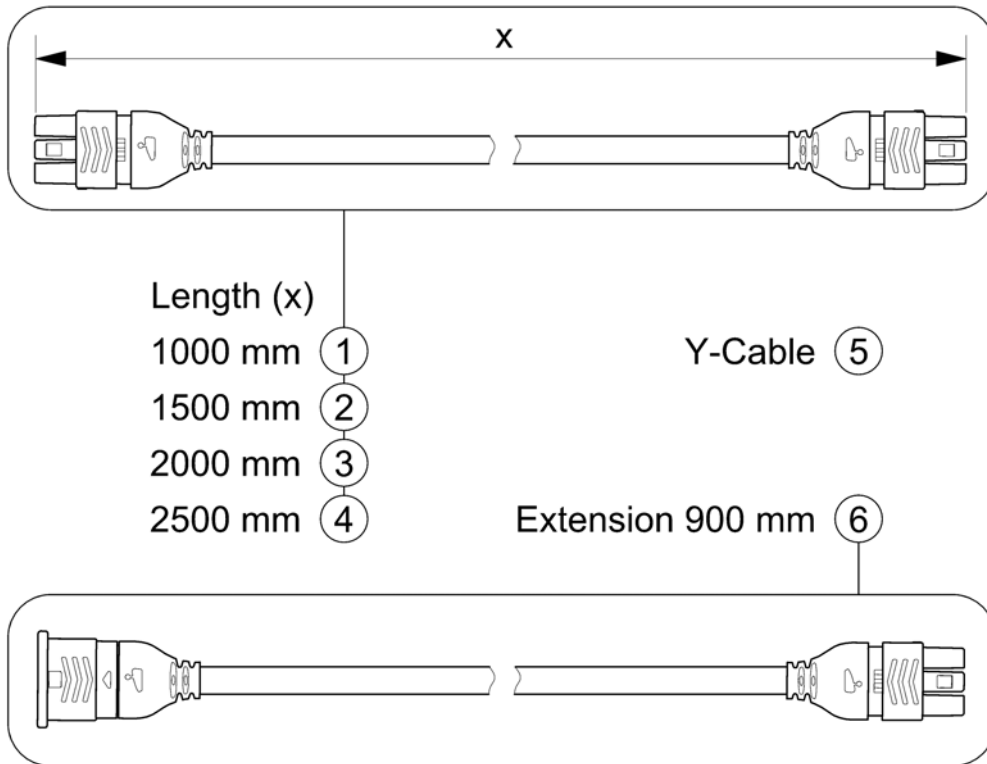

1	1448922	(Buscable Shark 1000mm)	1
2	1450313	BUSKABEL 1500 MM SHARK	1
3	1484911	(Buscable Shark 2000mm)	1
4	SP1510073	(Buscable Shark 2500mm)	1
5	1450314	BUSKABEL Y SHARK	1
6	1116404	(Extension Cable Shark 900mm)	1

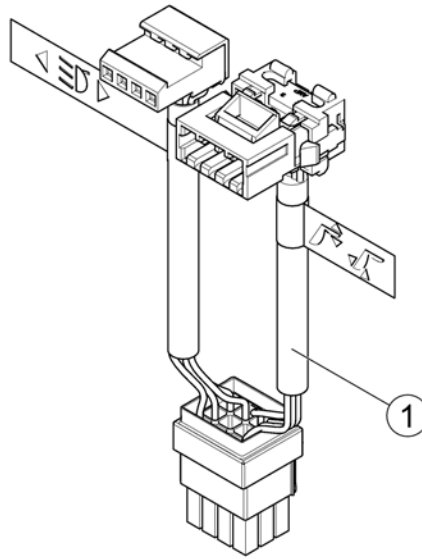

1 1488951

KABELSETT KPL FOR LYS OG AKTUATOR DRAGON

1


	Capacity [Ah]															
	1,2	12	30	31	32	36	38	40	40	50	50	50,9	60	73,6	75	97,6
Bora/SpectraXTR								8		10						
Dragon										10			13			
Dragon Vertic													13			
FDX													13			
G50													13	14		16
Kite													13	14		
Mirage				4				8								
Mistral3				4									12			
P9000XDT					5											
Pronto M41					5											
Pronto M61					5											
Spectra Blitz								8					12			
Spectra Comfort								7								
Spectra Plus								7								
Spectra Seat Riser								7								
Storm3										10			13	14		
Storm3 True Track	1													14		
Storm3 Vertic														14		
Storm4														14		
Storm4 Max														14		
Stream										10			13			
TDX-SP														14		
TDX-SP Narrow													12			
Tiger										10			13			
Topan													13			
Tornado								8								
Typhoon													13			
Typhoon GB	1												13			
Typhoon GB	1												13			
Zephyr	1												13			
Auriga			3					8		10						
Comet														14	15	
Leo						6			9							
Lynx		2														
Meteor														14		
Orion												11				
length [mm]	97	152	197	196	211	196	210	197	198	198	198	238	259	260	260	329
width [mm]	49	99	132	132	132	130	175	168	165	166	165	140	169	171	168	171
height [mm]	51	102	180	183	183	179	175	175	171	176	170	235	178	210	223	238
weight [kg]	0,6	3,9	11,1	10,5	11	12	14,6	14,4	14,2	14	16	17,1	19,3	24,3	26	32,5





i	-	(Change batteries only pairwise and from the same type!)	1
1	1440386	BATTERY 1,2AH 12V FRIKPL.THYPOON	1
2	SP1506493	(Battery 12V/12Ah AGM)	1
3	P50021	(Battery, 12V/30Ah C20, GEL)	1
4	P50029	BATTERY 30AH MK12-31 TROJAN	1
5	SP1521459	(Battery, 12V/34Ah C20, AGM)	1
6	1505498	(Battery, 12V/36Ah C20, AGM)	1
7	P50028	(Battery, 12V/38Ah C20, GEL (UK & DK only))	1
8	PG50988	BATTERY 40AH MK12-40 TROJAN	1
9	SP1550692	(Battery, 12V/40Ah C20, AGM)	1
10	SP1522107	(Battery, 12V/50Ah C20, AGM)	1
11	SP1521695	(Battery, 12V/50Ah C20 AGM)	1
12	PG50989	BATTERY 50AH MK12-50 TROJAN	1
13	1432944	BATTERY 60AH MK12-60 TROJAN	1
14	P50016	BATTERY 70AH MK12-73 TROJAN	1
15	SP1522026	(Battery, 12V/75Ah C20, AGM)	1
16	1478675	BATTERY 97AH MK12-97 TROJAN	1

(Chargers)

SP51981.iso

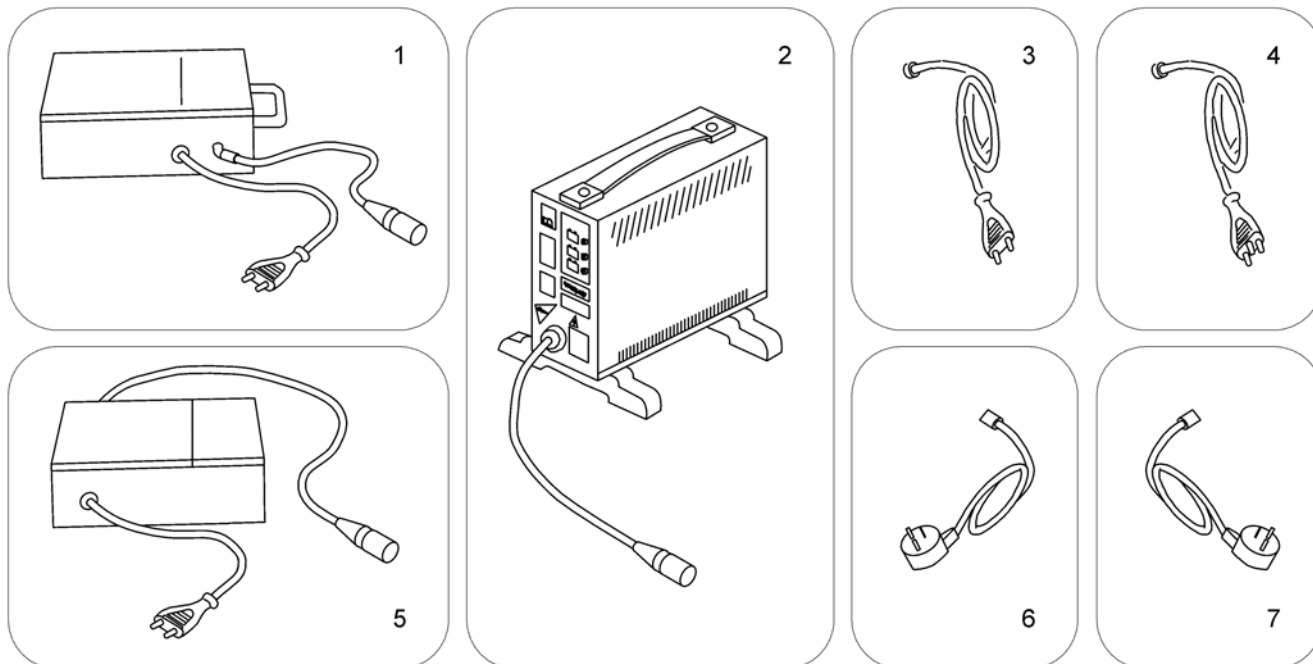


1	5813987	(Charger 8A, Gi Popular (Sweden only))	1
2	SPP51981	(Charger 8A for Batteries up to 73Ah (without power cord))	1
3	P21079	(Power Cord, Continental Shockproof Plug)	1
4	1443841	(Power Cord, Swiss)	1
5	1510201	(Charger 10A, Powercharge Classic (G50), for Batteries from 97AH (with power cord))	1
5	SP1545511	LADER LADAC/DEFA 701239 MED PLUGG	1
6	P21080	(Power Cord, UK)	1
7	SP1553849	(Power Cord, Israel)	1

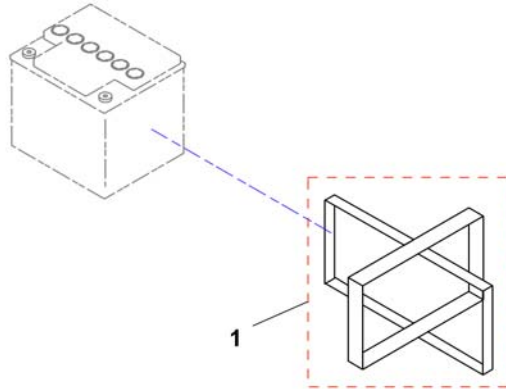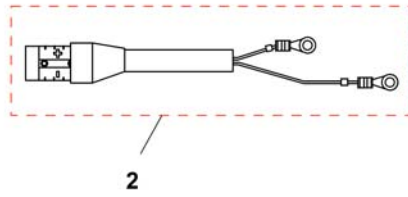


1150940	(Batteryharness for Tilt with Recline and Lighting)	1
GSM63003	KABEL BUS 0,3M ACS,	1
1432561	SIKRINGSHOLDER STORM3 KOMP.M 80A SIKRING	1
1 F384020	MONT SATS BATTERI BOKS,	1
2 SP1527780	BATTERIKABEL TDX	1

(Battery Tray Assembly and Wire Harnesses)

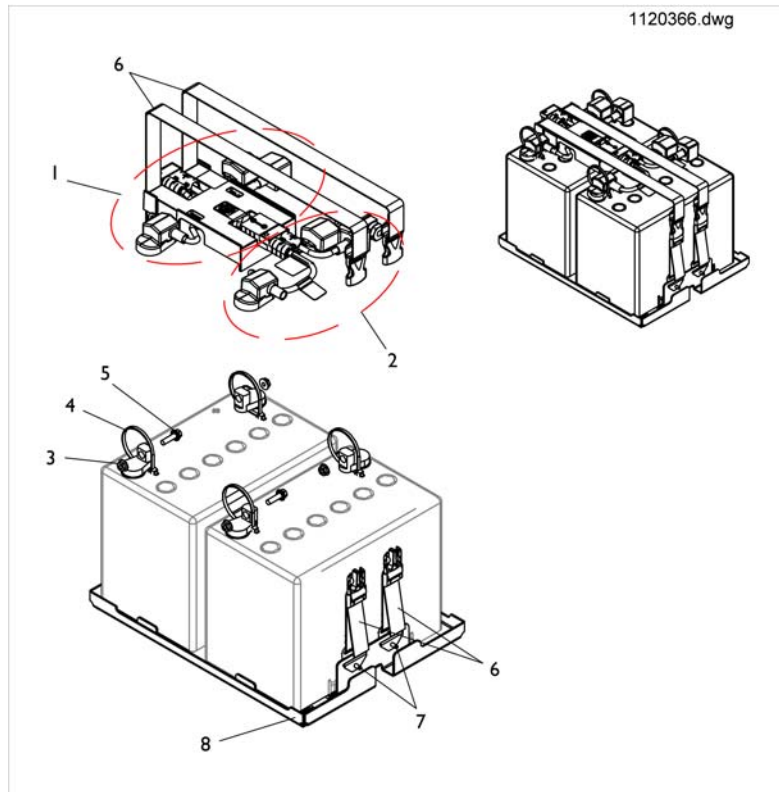

1	SP1523007	(Battery Wire Harness with Terminal Hardware, GP24 -Rear)	1
2	SP1523006	(Battery Wire Harness with Terminal Hardware, GP24 -Front)	1
4	SP1522970	(Package, Cable Ties (11-1/2"Long))	10
5	SP1522972	(Screw, Hex Flange Whiz Lock (1/4-20x7/8"))	1
6	SP1523191	FESTESTROPP TDX	1
7	SP1523266	(Rivet (3/16 x7/16"))	1
8	SP1523188	(Battery Tray Assembly, TDX SP GP24)	1
	SP1523150	(Strip, Self Adhesive Wear (2-1/2"Long x1/2"Wide))	1


1521460X.dwg

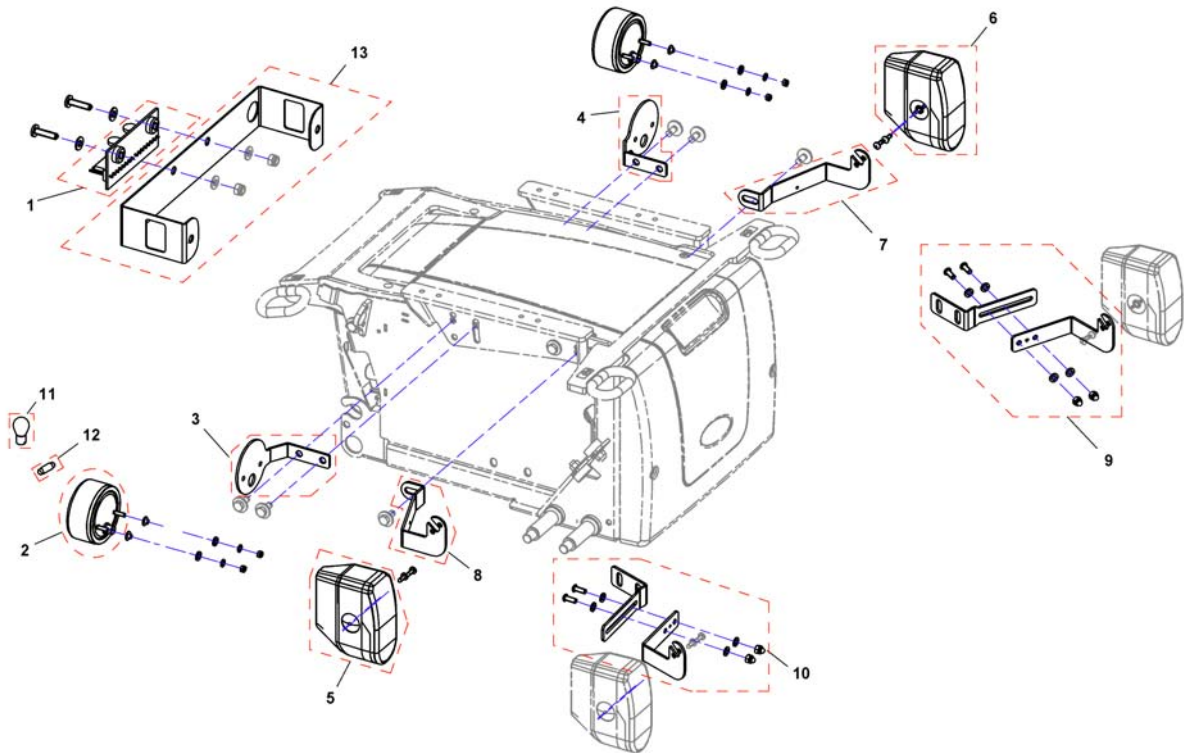

!!!	-	(Rear Light Protector Kit)	0
-	1488951	KABELSETT KPL FOR LYS OG AKTUATOR DRAGON	1
1	1415004	LYSFORDELER PRINT G40	1
2	1442736	BAKLYS KPL STORM3	1
3	SP1513692	HOLDER LAMPE BAK HØ TDX	1
4	SP1513691	HOLDER LAMPE BAK VE TDX	1
5	1420412	FRONTLYS HØ KPL STORM3 TORNADO G50	1
6	1420413	FRONTLYS VE KPL STORM3 TORNADO TDX	1
7	SP1513751	HOLDER LAMPE FRONT VE TDX	1
8	SP1513752	HOLDER LAMPE FRONT HØ TDX	1
9	SP1551039	FESTE LYKT FORAN VE TDX <i>only TDX-SP narrow base</i>	1
10	SP1551040	FESTE LYKT FORAN HØ TDX <i>only TDX-SP narrow base</i>	1
11	SP1419271	(Light Bulb 24V/21W, yellow (Turn Signal front/rear))	1
12	SPP31191	(Incandescent Lamp 24V/5W white)	1
13	SP1513830	FESTE LYSMODUL TDX	1

Invacare® TDX-SP | Sikkerhetsutstyr
(Rear Light Protector Kit)


1585420.000

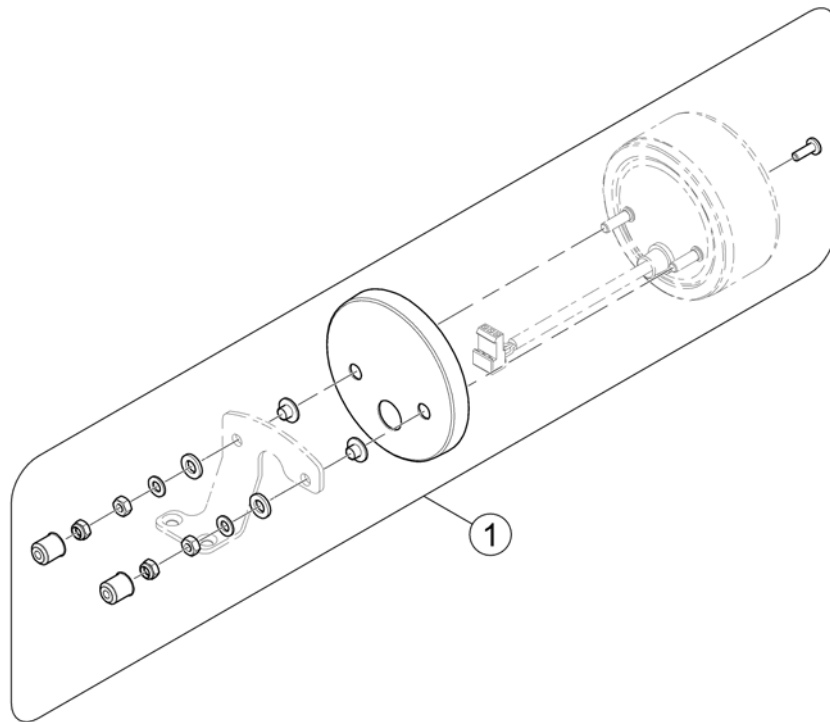

1 SP1581827

(Rear Light Protector Exchange Kit)


1

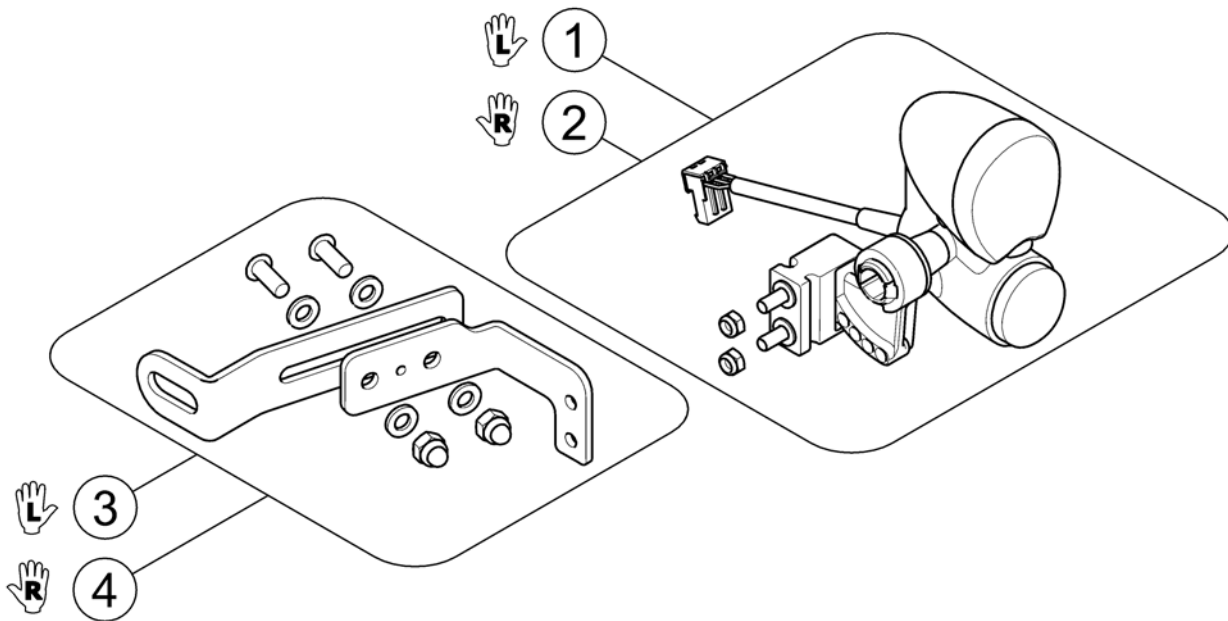


1	SP1536030	(Front light LED, LH)	1
2	SP1576193	(Front Light LED RH)	1
3	SP1583093	(Light Bracket LED, front LH)	1
4	SP1583092	(Light Bracket LED, front RH)	1

(Reflective Sign)

SP1430410.no

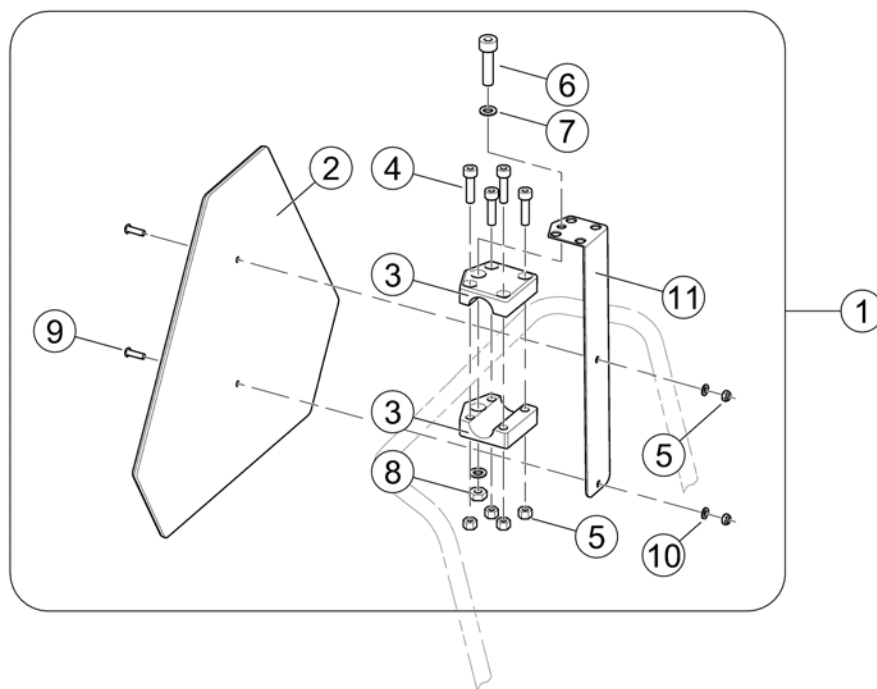


1	SP1430410	(Reflective Sign ECE-R 69, complete)	1
2	1430834	(Rear Marking Sign ECE-R 69)	1
3	Z12008A.P40	(Gripping clip with hexagon hole, RAL 9005 black)	2
4	D50004	(Cylinder Bolt M6x30-8.8 bk - DIN 912)	4
5	F20109	MUTTER 6 KT M6	6
6	D00102	SEKSKANT SKRUE M8X45	1
7	G00107	STOPPSKIVE 8,4X17X1,6	2
8	F00002	SEKSKANTMUTTER M8	1
9	D90187	(Fillister Head Screw M6x20-10.9 bk - DIN 7380)	2
10	G00115	SKIVE 6,4X12,5X1,6 MISTRAL	2
11	1430418-9005	(Mounting Bracket for Reflective Sign - RAL 9005 black)	1
12	SP1580671	(Bracket, Reflective Sign, cpl.)	1

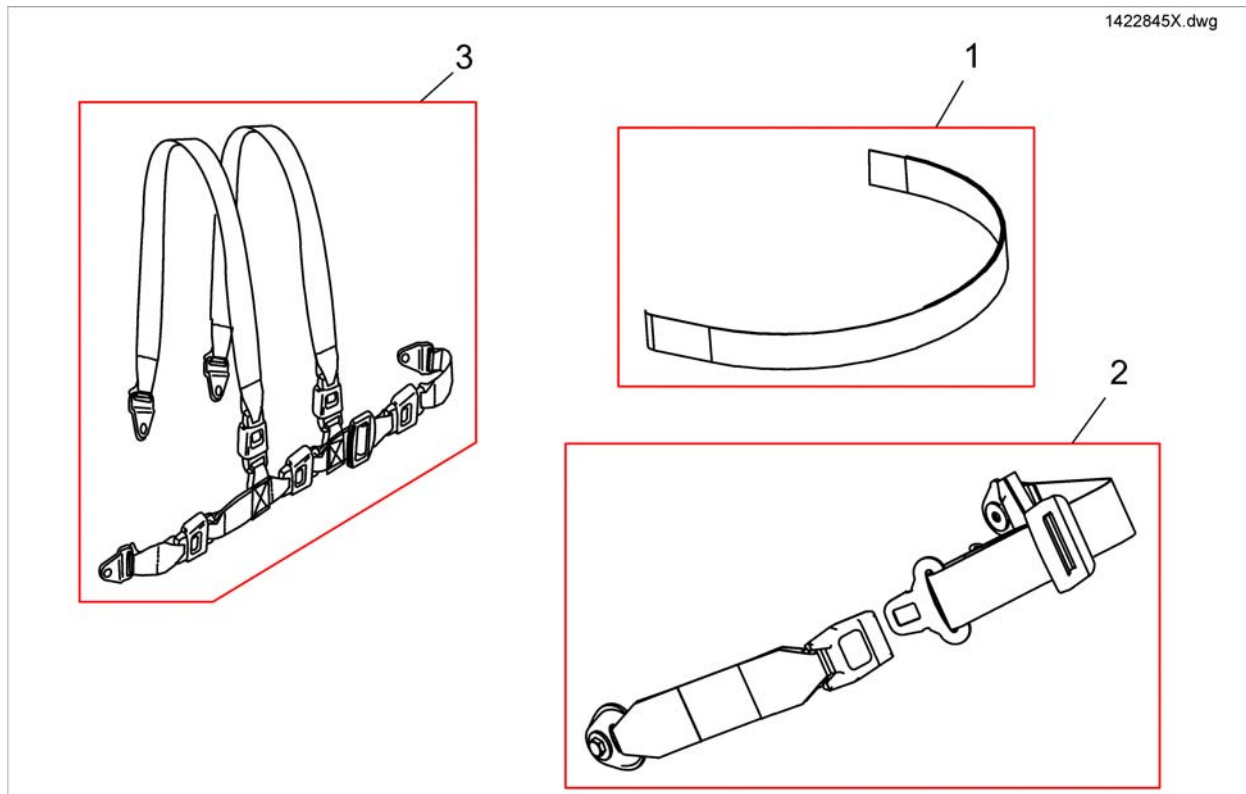


1	1422870	HOFTEBELTE MED VELCRO TDX/STORM4/BORA	1
1	1422869	HOFTEBELTE BORA EXTRA BORA INNE	1
2	1422845	HOFTEBELTE MED SNAP LÅS STORM 3/ DRAGON	1
2	1422846	HOFTEBELTE MED HURTIGLÅS	1
3	SP1422848	H-SELE CONTUR STORM3	1
3	SP1422847	(HOLDING STRAP FOR FLEX SEAT H-FORM CPL.)	1

(Lap Belts (Modulite Seat))

1571836 iso

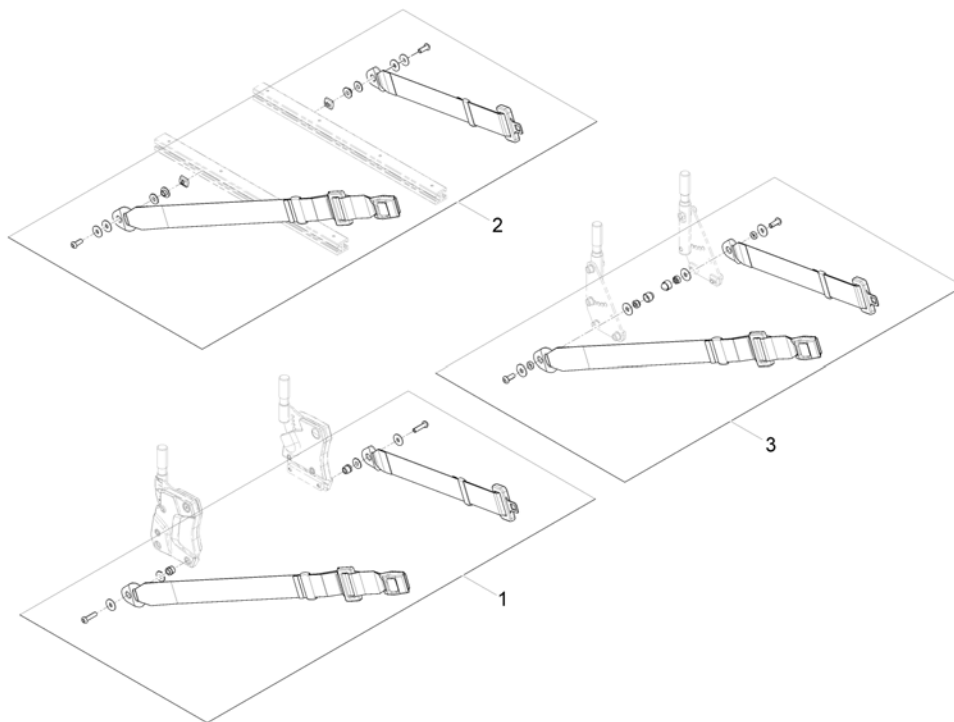


1	SP1570381	(Lap Belt (fix mounting on Backrest Support), cpl.)	1
2	SP1571183	HOFTEBELTE C TIL SKINNE MODULITE	1
3	1422845	HOFTEBELTE MED SNAP LÅS STORM 3/ DRAGON	1

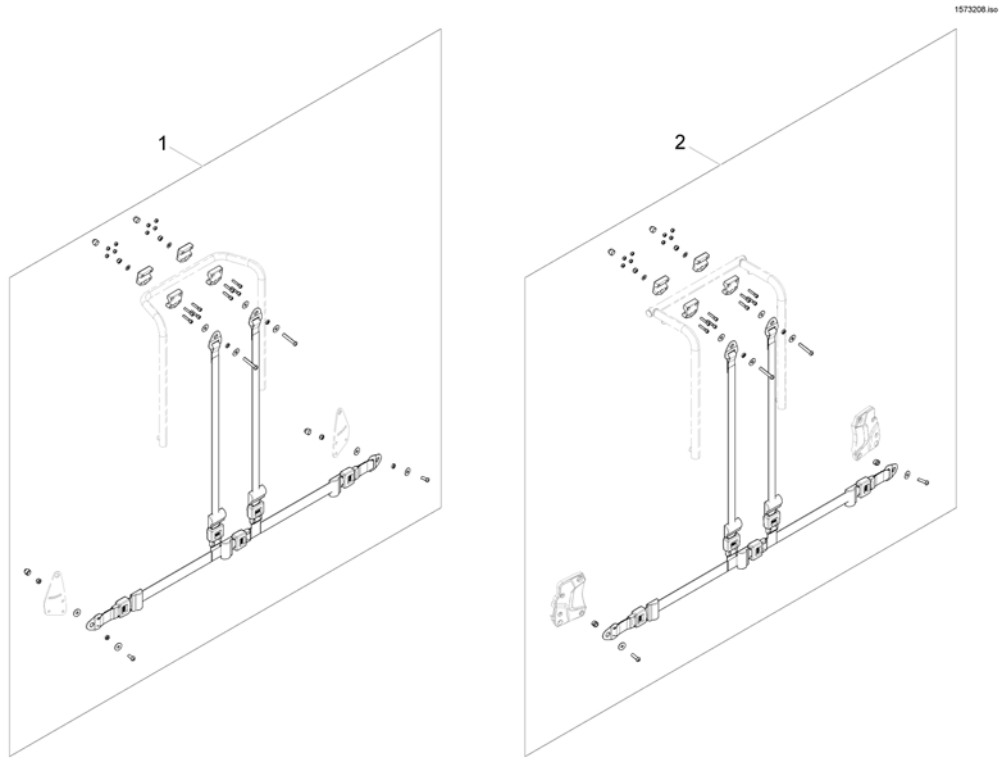


1	SP1570385	(H-Belt for Backrest Frame)	1
2	SP1570380	(H-Belt for Back Posts)	1

(Rearview Mirror)

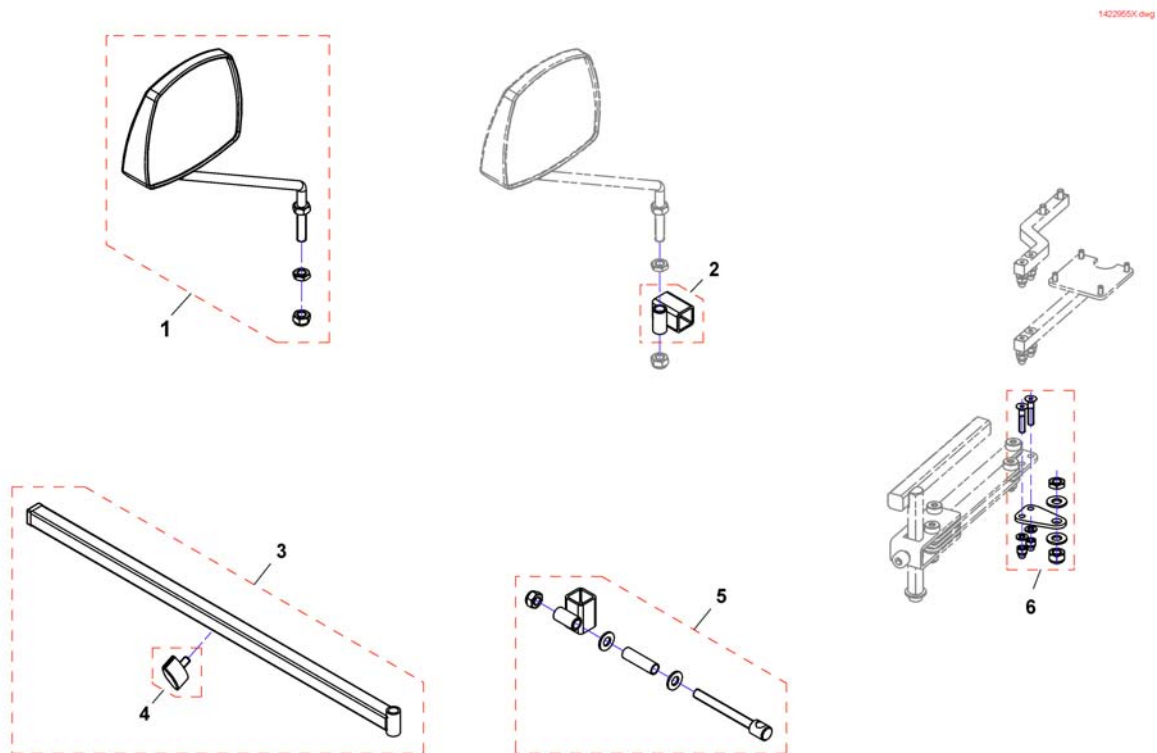

1	SP1422955	SPEIL KPL KUN SPEIL STORM 3	1
2	F276005.00	HOLDER SPEIL DRAGON STORM4 HUSK SP1527151	1
3	SP1423435	HOLDER SPEIL STORM3	1
4	L31001	VINGSKRUV M 6X10 GARANT G24 S	1
5	SP1422957	SPEIL FESTE BRAKETT STORM4 PÅ SWINGAWAY	1
6	SP1527151	FESTE FOR SPEILFESTE STORM4 TDX BORA	1


1573746.jpg

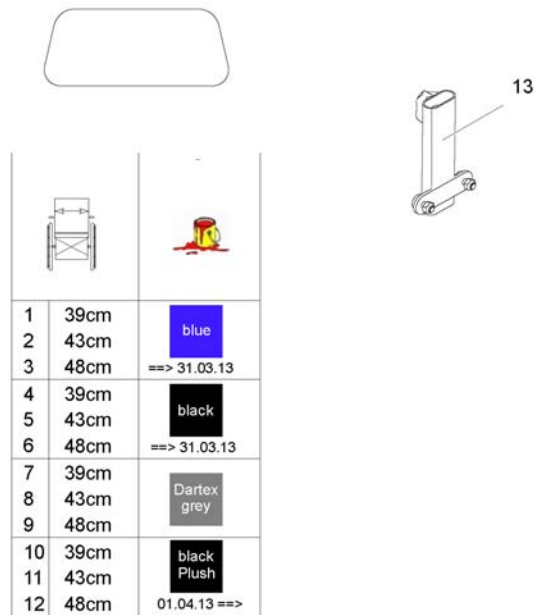


1	39cm	blue
2	43cm	
3	48cm	==> 31.03.13
4	39cm	black
5	43cm	
6	48cm	==> 31.03.13
7	39cm	Dartex grey
8	43cm	
9	48cm	
10	39cm	black Plush
11	43cm	
12	48cm	01.04.13 ==>

1	SP1422918	RYGGFORLENGER FLEX BLÅ SB 39	→ 31.03.2013	1
2	SP1422919	RYGGFORLENGER FLEX BLÅ SB 44	→ 31.03.2013	1
3	SP1422920	RYGGFORLENGER FLEX BLÅ SB 48	→ 31.03.2013	1
4	1436567	(Backrest Extension for Flex Seat, SW39cm, black)	→ 31.03.2013	1
5	1436568	RYGGFORLENGER SB44 TDX FLEX SORT	→ 31.03.2013	1
6	1440283	(Backrest Extension for Flex Seat, SW48cm, black)	→ 31.03.2013	1
7	5253035-TR23	RYGGFORLENGER 39 KPL PLUSH DARTEX		1
8	5253036-TR23	RYGGFORLENGER 44CM GRØNN DARTEX (MBUSKERUD)		1
9	5253037-TR23	RYGGFORLENGER 49CM GRØNN DARTEX (MBUSKERUD)		1
10	SP1436567-TR35	(Backrest Extension for Flex Seat, SW39cm, black Plush)	01.04.2013 →	1
11	SP1436568-TR35	(Backrest Extension for Flex Seat, SW43cm, black Plush)	01.04.2013 →	1
12	SP1440283-TR35	(Backrest Extension for Flex Seat, SW48cm, black Plush)	01.04.2013 →	1
13	SP1422921	LÅS RYGGFORLENGER TDX		1

(Abduction Wedge)

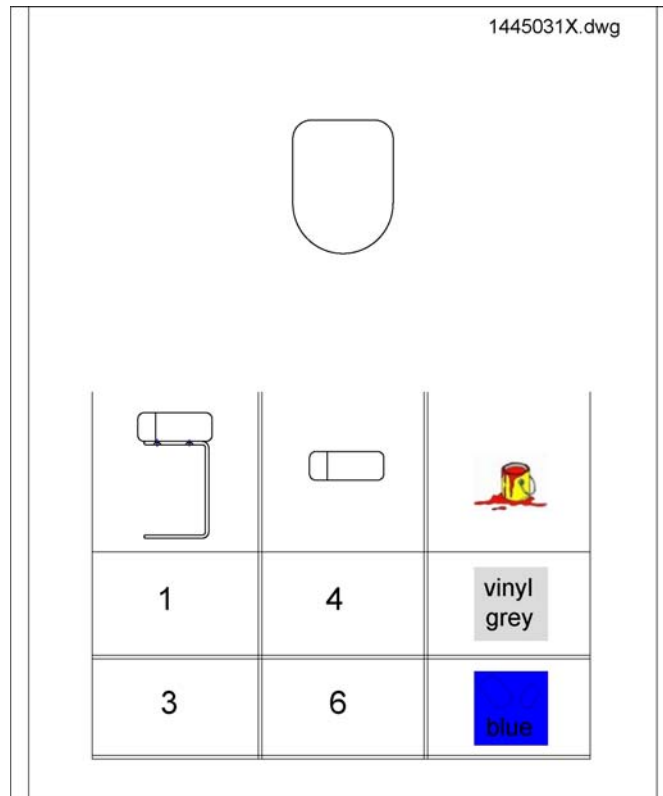


1	SP1445033	(Abduction Wedge Junior, grey Vinyl, with Bracket)	1
3	SP1445032	(Abduction Wedge Junior, Blue Dot, with Bracket)	1
4	SP5313257	(Abduction Wedge Junior, grey Vinyl, wo Bracket)	1
6	SP5313256	(Abduction Wedge Junior, Blue Dot, wo Bracket)	1

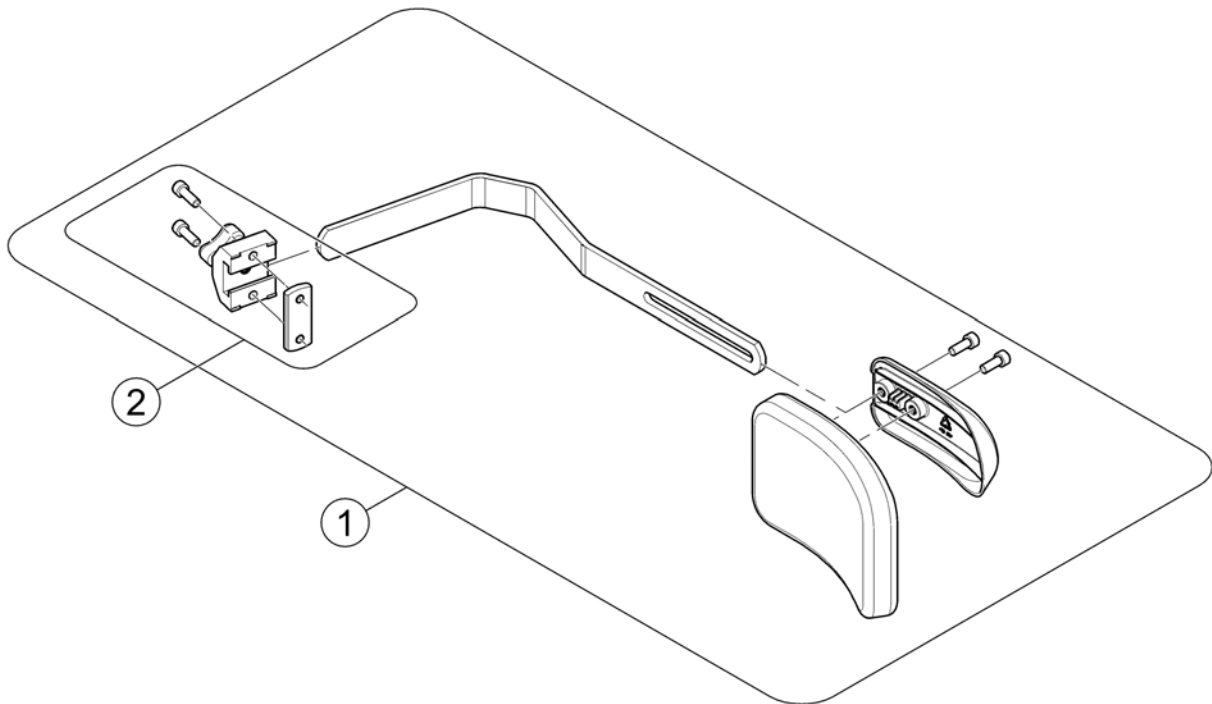


1	SP1561841	SIDESTØTTE FAST HØ/VE TDX	1
2	5253170	SIDESTØTTE FESTE KPL REA	1

(Lateral Support, multi adjustable - Easy Adapt)

1502647.no

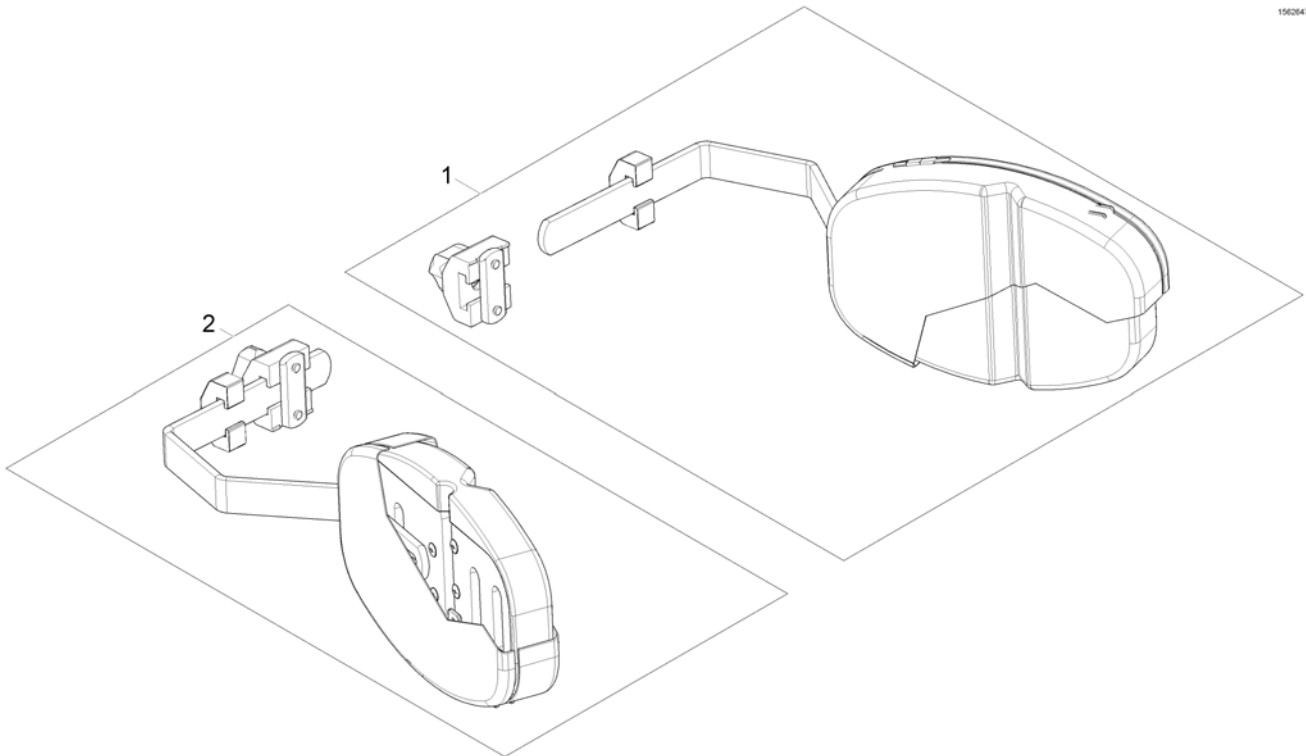


1	1448692	SIDESTØTTE VE TR26 MULTI REGULERBAR	1
2	1448693	SIDESTØTTE HØ TR26 MULTI REGULERBAR	1

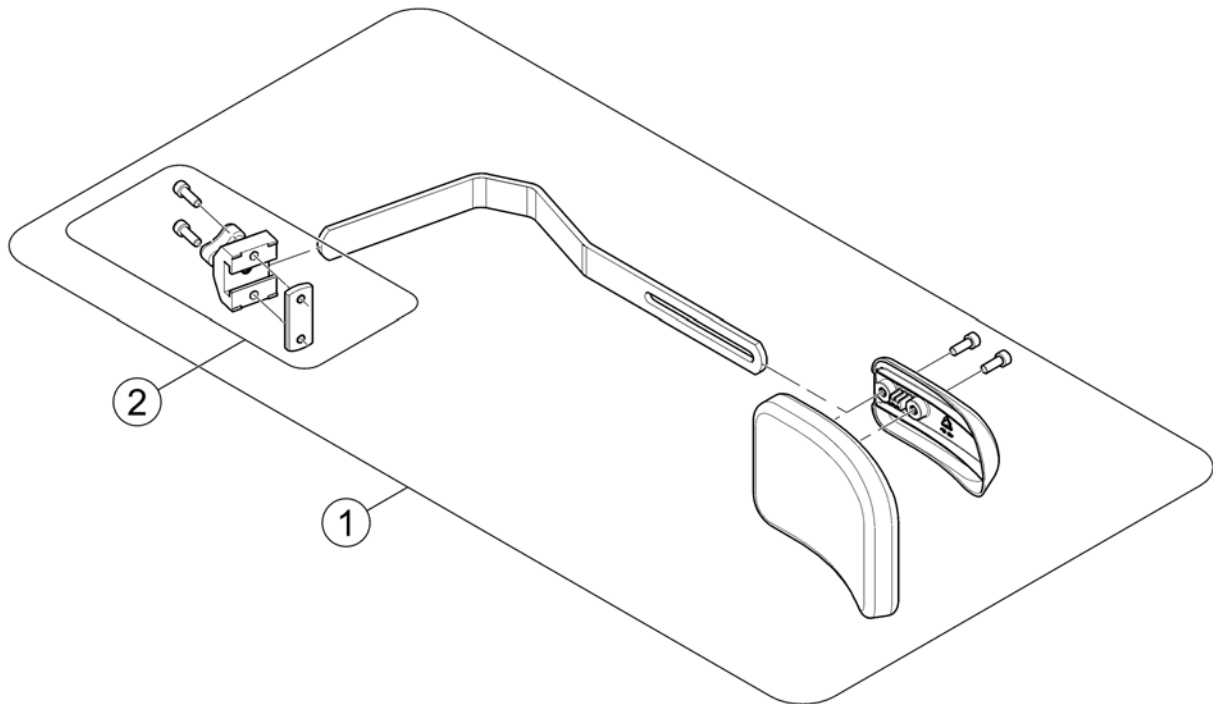

1	SP1580394	SIDESTØTTE MODULITE	1
2	1543574	SIDESTØTTEFESTE KPL LAGUNA 2, CLEMATIS	1


(Lateral Support - Modulite Seat)

158880 iso

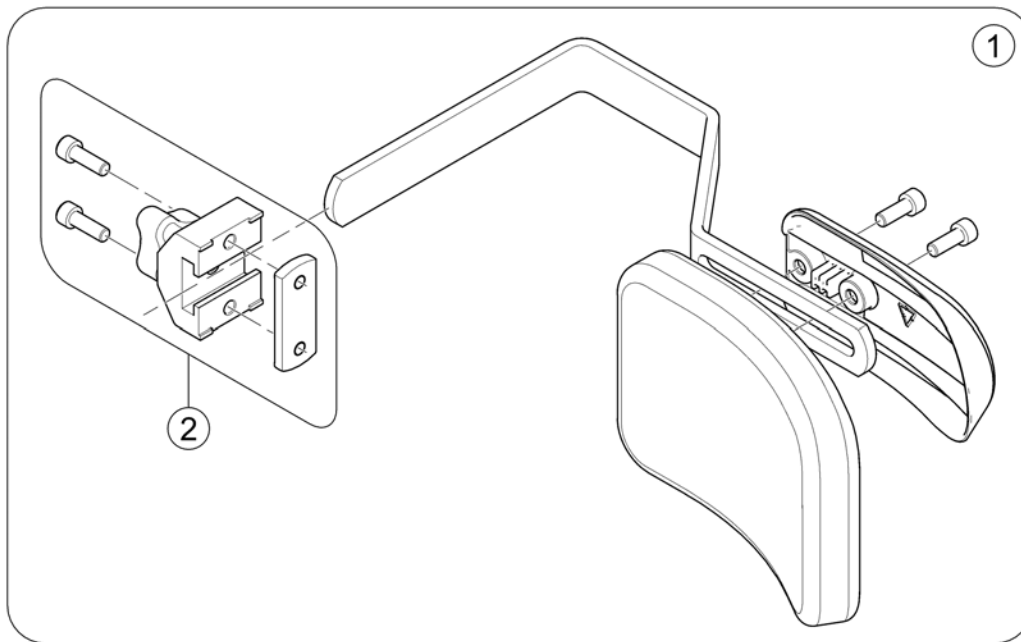


-	(Design change on 01. July 2015 - we suggest to change older Supports on both sides.)	0
1	SP1588314 (Lateral Support, cpl.)	1
2	5253170 SIDESTØTTE FESTE KPL REA	1

5313271X.dwg

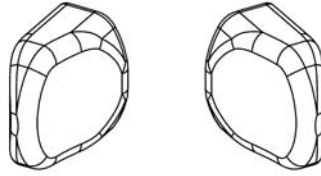


RH	LH	
1	2	blue dot
3	4	vinyl grey

1	SP5313271	(Trunk Support RH for Junior, blue Dot)	1
2	SP5313275	(Trunk Support LH for Junior, blue Dot)	1
3	5313272	(TRUNK SUPPORT VINYL BLACK RH CPL.)	1
4	5313276	(TRUNK SUPPORT VINYL BLACK LH CPL.)	1

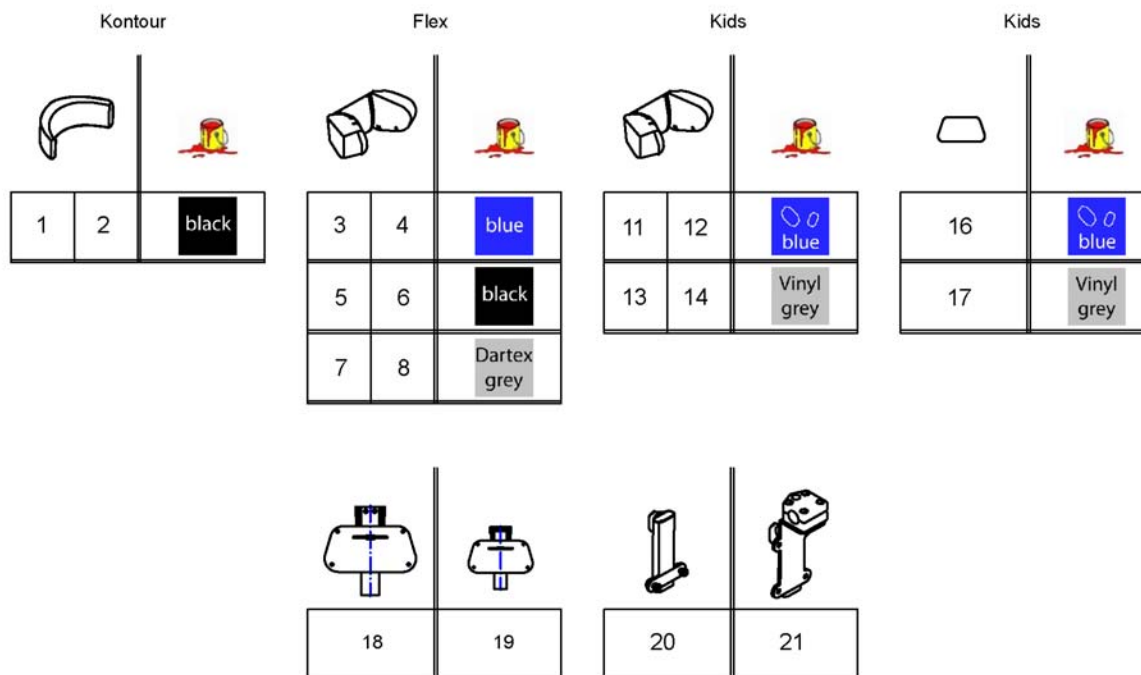


1	SP1422925	HODESTØTTE KPL KONTOUR SORT STORM3/DRAG U/FESTE	1
2	SP1422865	HODESTØTTEPUTE U FESTE STORM4	1
3	-	(No longer available.)	0
4	-	(No longer available.)	0
5	SP1440285-TR35	HODESTØTTE MULTI TDX SVART PLYSJ U/BRAKETT	1
6	SP1585039	(Headrest Cushion Flex, Plush black (chenille TR35), w/o bracket)	1
7	SP1428092	(Headrest Flex, Dartex grey, complete)	1
8	SP5252229	(Headrest cushion Flex Dartex grey, without Bracket)	1
11	SP5313307	(Headrest cushion Junior, blue Dot, complete)	1
12	SP5313261	(Headrest cushion Junior, blue Dot, without bracket)	1
13	SP5313308	(Headrest Cushion Junior, Vinyl grey, complete)	1
14	SP5313262	(Headrest Cushion Junior, Vinyl grey, without bracket)	1
16	SP5313311	(Neckrest Cushion Junior, Blue Dot)	1
17	SP5313312	(Neckrest Cushion Junior, grey Vinyl)	1
18	SP1419636	FESTE NAKKESTØTTE TDX	1
19	SP1419637	FESTE FOR HODESTØTTE AJ OPTIMIST LITEN	1
20	SP1422547	HOLDER HODESTØTTE RYGG KONTOUR/FLEX STORM3/DRAG	1
21	SP1422914	HOLDER HODESTØTTE RYGG STD RYGG STORM3 DRAGON	1

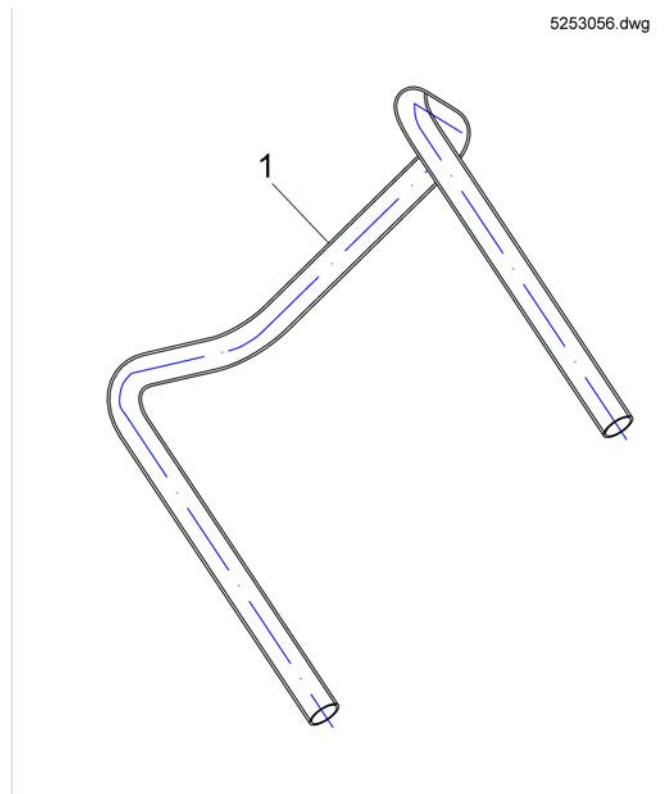

1	SP1425839	KJØREHÅNDTAK SB43-48CM TDX	1
1	SP1425840	RYGGRAMME SB39CM TDX	1
2	SP1524672	(Push Handle (single one))	1

(Caneholder)


-	(Caneholder for Standard Seat)	1
-	(Caneholder for Contour/Flex Seat)	1
-	(Cane holder (Modulite: backrest frame))	1
-	(Cane holder (Modulite: back posts man recliner))	1
-	(Cane holder (Modulite: back posts pow recliner))	1

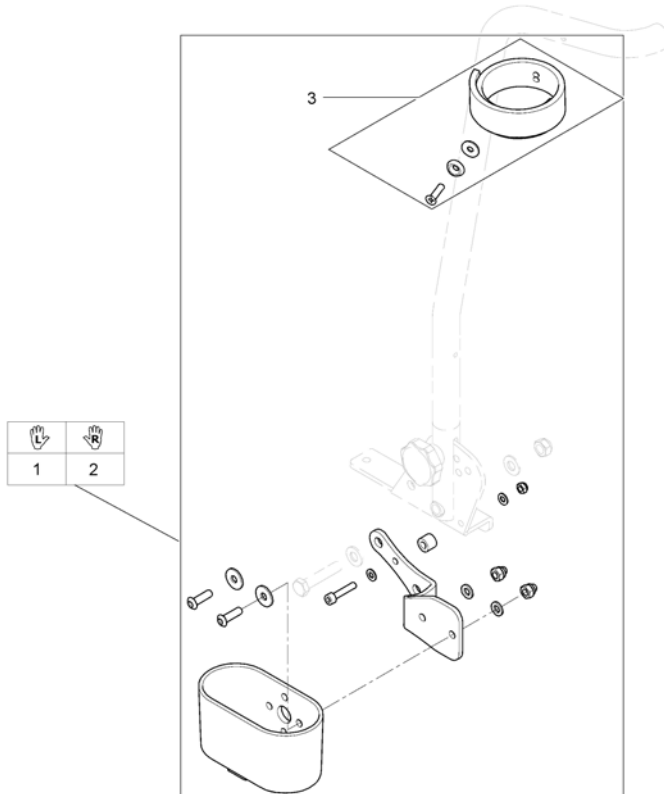

1	1442701	(CANE HOLDER FOR STANDARD-SEAT, CPL. Tornado/Typhoon)	1
2	SP1534877	KRYKKEHOLDER KPL TDX TDX	1
3	SP5009551	VELCRO KRYKKEHOLDER COMET	1


(Caneholder for Contour/Flex Seat)

1567254.iso

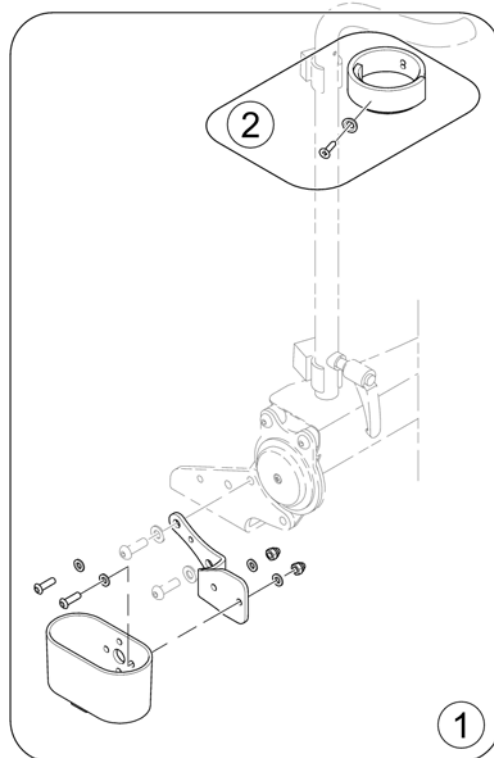


1	1442702	KRYKKEHOLDER KONTUR FLEX KPL TORNADO	1
2	SP5009551	VELCRO KRYKKEHOLDER COMET	1

(Cane holder (Modulite: backrest frame))

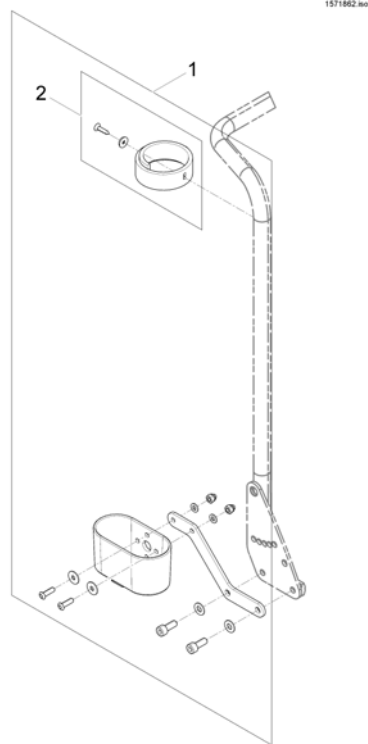

1	SP1570396	(Cane holder, standard backrest)	1
2	SP1420493	(Belt for caneholder)	1


(Cane holder (Modulite: back posts man recliner))

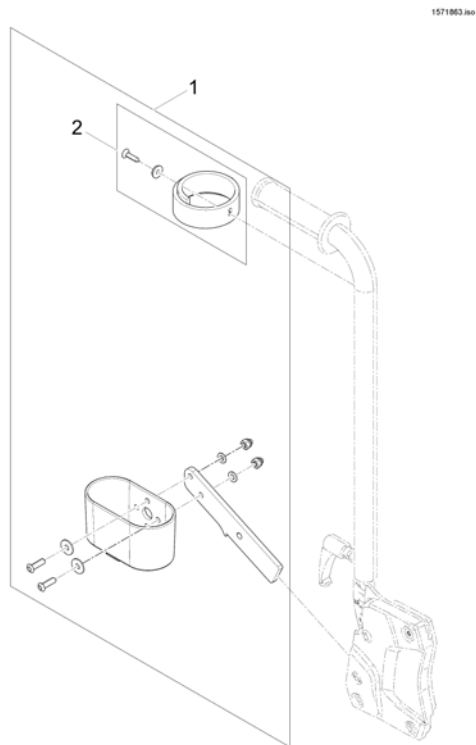

1	SP1570443	(Cane holder, manual backrest)	1
2	SP1420493	(Belt for caneholder)	1

(Cane holder (Modulite: back posts pow recliner))


1571837.no

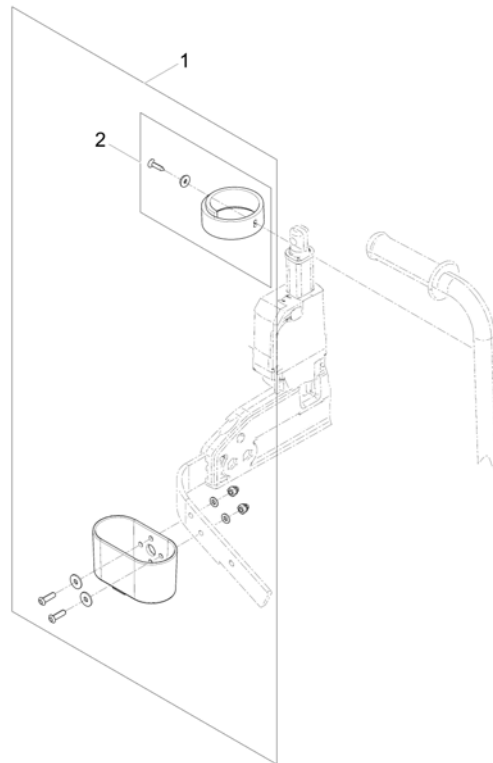

1	SP1570382	KRYKKEHOLDER STORM4 STORM4 MODULITE	1
2	SP1420493	(Belt for caneholder)	1


(Trays)

1535050X.dwg

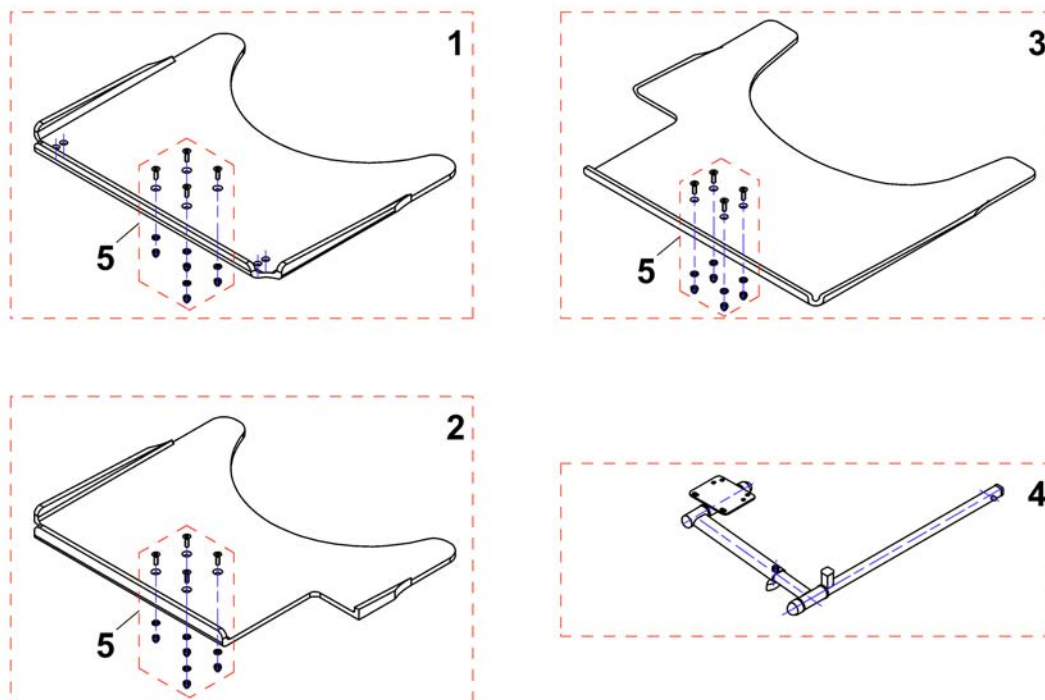

1	SP1535050	BORD 540MM SB33-43 UTEN UTSKJÆRING STORM4	1
1	SP1535010	BORD 620MM SB38-48 UTEN UTSKJÆRING STORM4	1
1	SP1535011	BORD 660MM SB44-53 UTEN UTSKJÆRING STORM4	1
2	SP1535009	BORD 540MM SB33-43 STORM4 UTSKJÆRING VE	1
2	SP1534974	BORD 620 MM SB38-48 UTSK VE STORM 4 UTEN FESTE	1
2	SP1534975	BORD 660MM SB38-48 UT VE STORM4 UTSKJÆRING VE	1
3	SP1535051	BORD 540MM SB33-43 STORM4 UTSKJÆRING HØ	1
3	SP1535012	BORD 620 MM SB38-48 UTSK HØ STORM 4 UTEN FESTE	1
3	SP1535049	BORD 660MM SB 44-53 UTSK HØ SIDE UTEN FESTE	1
4	SPZ177011	HOLDER FOR BORD SVINGBAR KPL STORM3/STORM4	1
5	Z177025	FESTE KIT BOR STORM4	1

1567259.iso

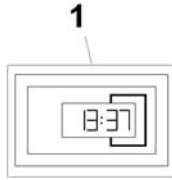


1 SP1530275 (Hour counter)

1

(KLICKfix® bag)


1562500 iso

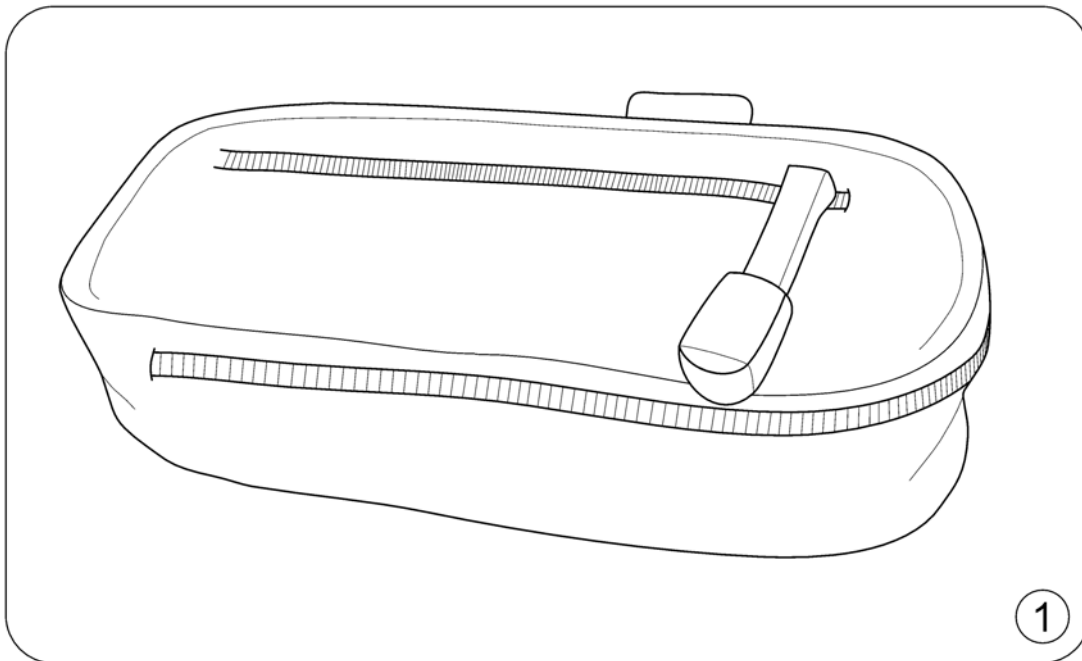

1 SP1557532

ARMLENSVESKE KLICK FIX STORM4 TDX KLICK FIX

1

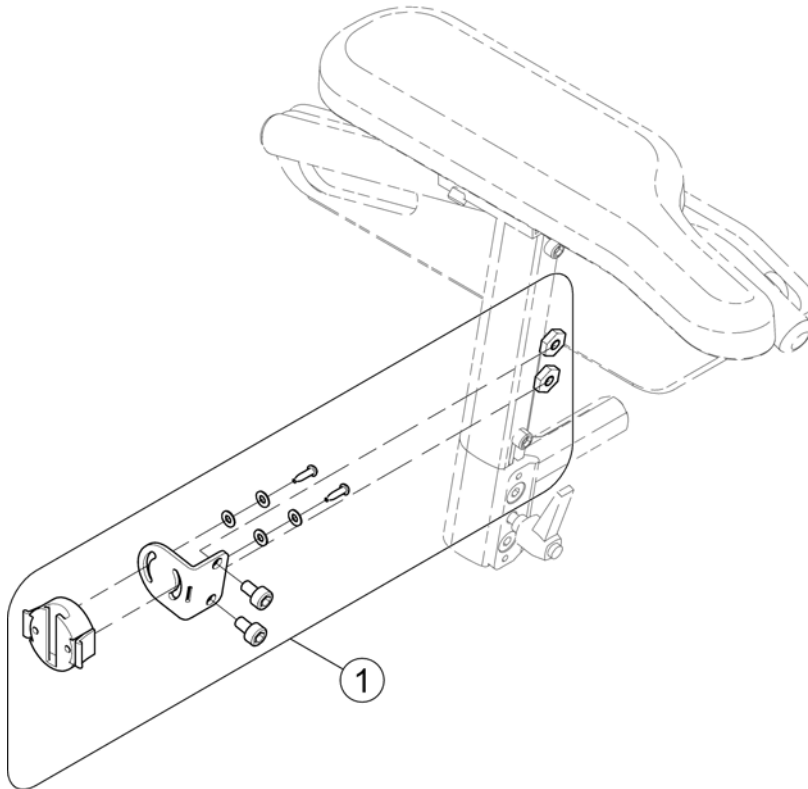


1 SP1554320

FESTE ARMLENSVESKE STORM4 TDX KLICK FIX

1

(KLICKfix® bracket for T-Armrest)

1571871 iso

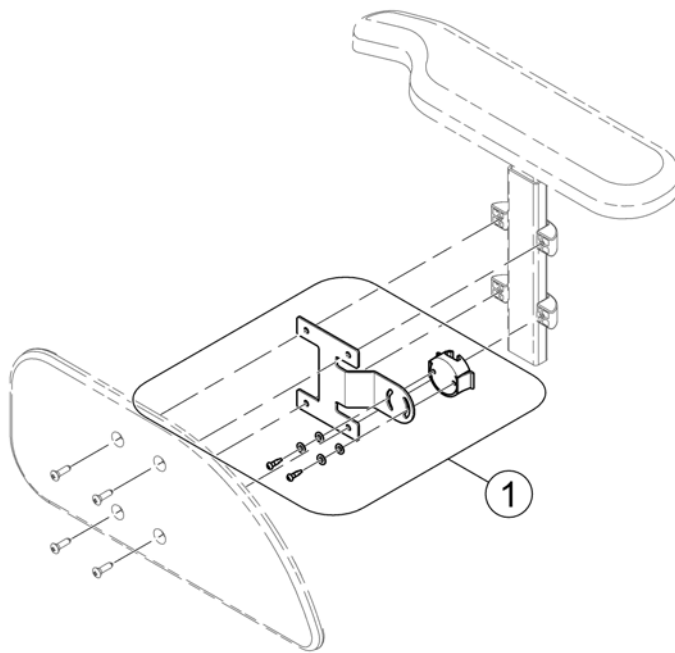


1 SP1571678

(KLICKfix® bracket)

1

(KLICKfix® bracket for Inrigged/Flip-Up/Following Armrest (Modulite))

1580167 no

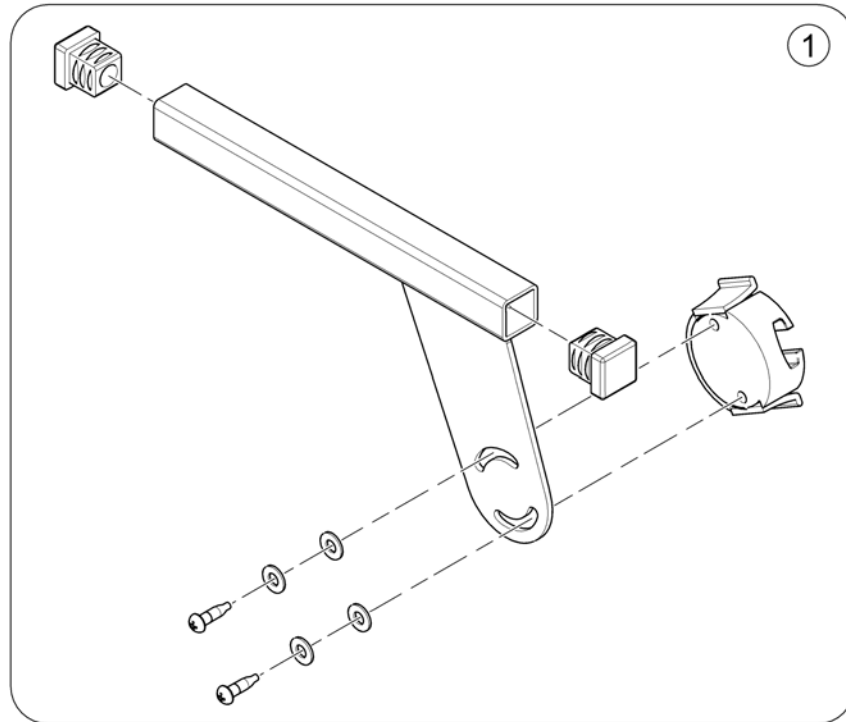


1 SP1584497

(KlickFix Bracket)


1

(Bag)

1567256.no

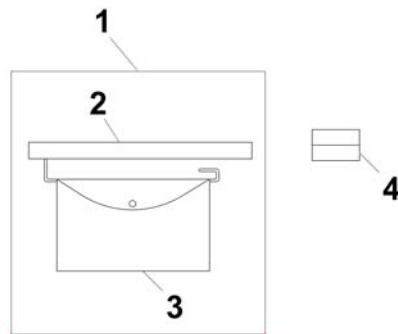


1	SP1430417	ARMLENSVESKE VE KPL STORM 3	1
2	SP1430855	HOLDER FOR ARMLENSVESKE STORM4 TDX BORA	1
3	SP5314967	VESKE ARMLEN STORM4	1
4	F370032-9005	(Holder for side part with mirror/brake RAL 9)	1

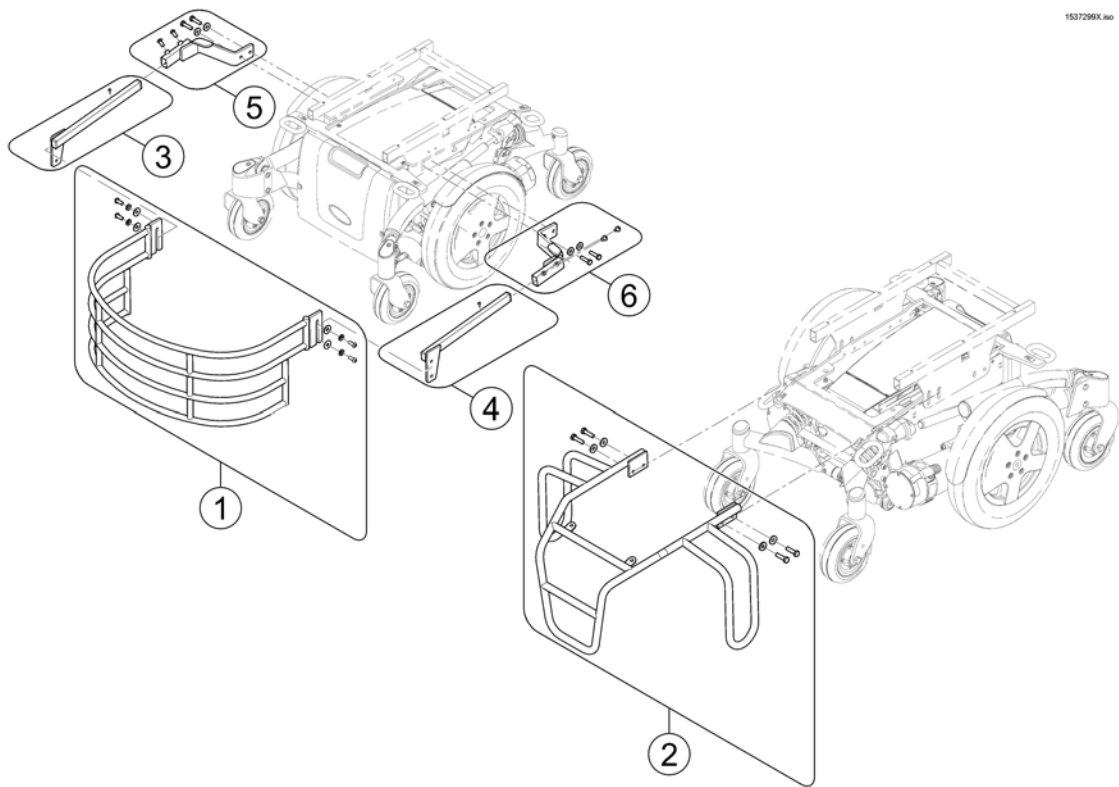

1	SP1504203	STØTFANGER KPL GUMMIERT TDX	1
2	SP1537300	FANGER BAK TDX	1
3	SP1537894	HOLDER STØTFANGER HØ TDX	1
4	SP1537895	HOLDER STØTFANGER VE TDX	1
5	SP1537896	(Frameholder, RH)	1
6	SP1537897	(Frameholder, LH)	1

(T-Nuts for Accessories Modulite Seat)

1574067 no

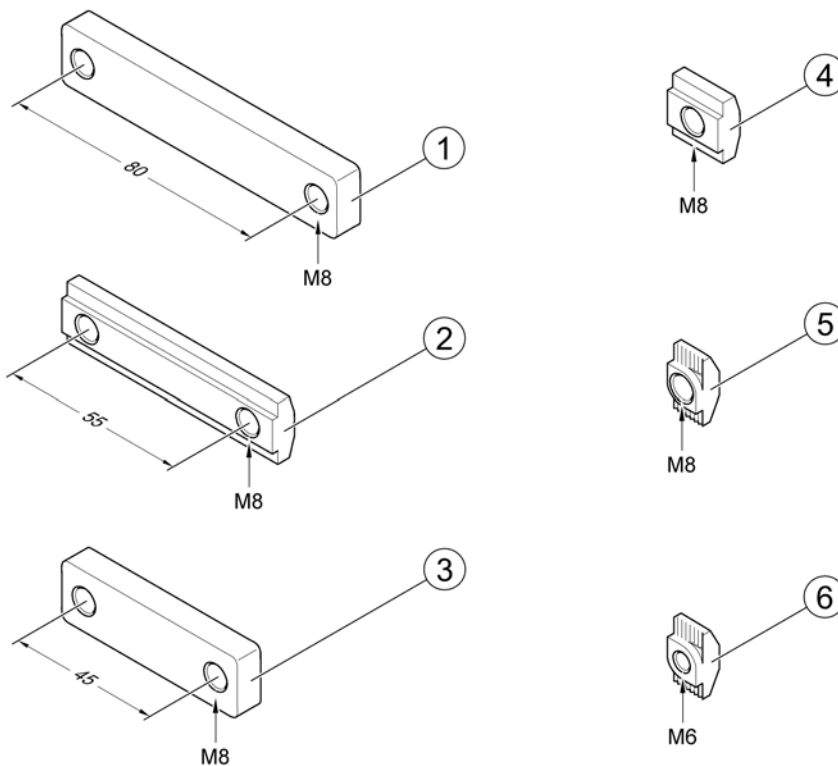

1	SP1556401	(T-Nut, hole distance 80mm, 2x M8)	1
2	SP1572444	(T-Nut, hole distance 55mm, 2x M8)	1
3	SP1555168	(T-Nut, hole distance 45mm, 2x M8)	1
4	SP1557341	(T-Nut, 1x M8)	1
5	SP1573158	(T-Nut, 1x M8)	1
6	SP1573721	(T-Nut, 1x M6)	1



Yes, you can.®



Belgium & Luxemburg:

Invacare nv, Autobaan 22, B-8210 Loppem
Tel: (32) (0)50 83 10 10, Fax: (32) (0)50 83 10 11
belgium@invacare.com / www.invacare.be

Denmark:

Invacare A/S, Sdr. Ringvej 37, DK-2605 Brøndby
Tel: (45) (0)36 90 00 00, Fax: (45) (0)36 90 00 01
denmark@invacare.com / www.invacare.dk

Deutschland:

Invacare Aquatec GmbH, Alemannenstraße 10, D-88316 Isny
Tel: (49) (0)75 62 7 00 0, Fax: (49) (0)75 62 7 00 66
info@invacare-aquatec.com / www.invacare-aquatec.de

European Distributor Organisation:

Invacare, Kleiststraße 49, D-32457 Porta Westfalica
Tel: (49) (0)57 31 754 540, Fax: (49) (0)57 31 754 541
edo@invacare.com / www.invacare.eu.com

España:

Invacare SA, c/Areny s/n, Polígon Industrial de Celrà, E-17460 Celrà (Girona)
Tel: (34) (0)972 49 32 00, Fax: (34) (0)972 49 32 20
contactsp@invacare.com / www.invacare.es

France:

Invacare Poirier SAS, Route de St Roch, F-37230 Fondettes
Tel: (33) (0)2 47 62 64 66, Fax: (33) (0)2 47 42 12 24
contactfr@invacare.com / www.invacare.fr

Ireland:

Invacare Ireland Ltd, Unit 5 Seatown Business Campus, Seatown Road, Swords, County Dublin - Ireland
Tel: (353) 1 810 7084, Fax: (353) 1 810 7085
ireland@invacare.com / www.invacare.ie

Italia:

Invacare Mecc San s.r.l., Via dei Pini 62, I-36016 Thiene (VI)
Tel: (39) 0445 38 00 59, Fax: (39) 0445 38 00 34
italia@invacare.com / www.invacare.it

Nederland:

Invacare BV, Celsiusstraat 46, NL-6716 BZ Ede
Tel: (31) (0)318 695 757, Fax: (31) (0)318 695 758
nederland@invacare.com, csede@invacare.com / www.invacare.nl

Norge:

Invacare AS, Grensesvingen 9, Postboks 6230, Etterstad, N-0603 Oslo
Tel: (47) (0)22 57 95 00, Fax: (47) (0)22 57 95 01
norway@invacare.com, island@invacare.com / www.invacare.no

Österreich:

Invacare Austria GmbH, Herzog Odilostrasse 101, A-5310 Mondsee
Tel: (43) 6232 5535 0, Fax: (43) 6232 5535 4
info-austria@invacare.com / www.invacare.at

Portugal:

Invacare Lda, Rua Estrada Velha, 949, P-4465-784 Leça do Balio
Tel: (351) (0)225 1059 46/47, Fax: (351) (0)225 1057 39
portugal@invacare.com / www.invacare.pt

Sverige & Suomi:

Invacare AB, Fagerstagatan 9, S-163 91 Spånga
Tel: (46) (0)8 761 70 90, Fax: (46) (0)8 761 81 08
sweden@invacare.com, finland@invacare.com / www.invacare.se

Switzerland:

Invacare AG, Benkenstrasse 260, CH-4108 Witterswil
Tel: (41) (0)61 487 70 80, Fax: (41) (0)61 487 70 81
switzerland@invacare.com / www.invacare.ch

United Kingdom:

Invacare Limited, Pencoed Technology Park, Pencoed, Bridgend CF35 5AQ
Switchboard Tel: (44) (0)1656 776200, Fax: (44) (0)1656 776201
Customer services Tel: (44) (0)1656 776222, Fax: (44) (0)1656 776220
uk@invacare.com / www.invacare.co.uk