Aquatec® Dot

en	Shower Stool User Manual	3
de	Duschhocker Gebrauchsanweisung	5
fr	Tabouret de douche Manuel d'utilisation	7
it	Sgabello da doccia Manuale d'uso	10
es	Taburete de ducha Manual del usuario	13
pt	Banco de duche Manual de utilização	15
nl	Douchekruk Gebruiksaanwijzing	18
sv	Duschstol Bruksanvisning	20
no	Dusjkrakk Bruksanvisning	23
fi	Suihkujakkara Käyttöohje	25
sk	Sprchovací stolček Návod na obsluhu	28
lt	Dužo kėdė Naudojimosi instrukcija	30

This manual MUST be given to the user of the product.
BEFORE using this product, this manual MUST be read and saved for future reference.

AQUATEC®

Yes, you can.®

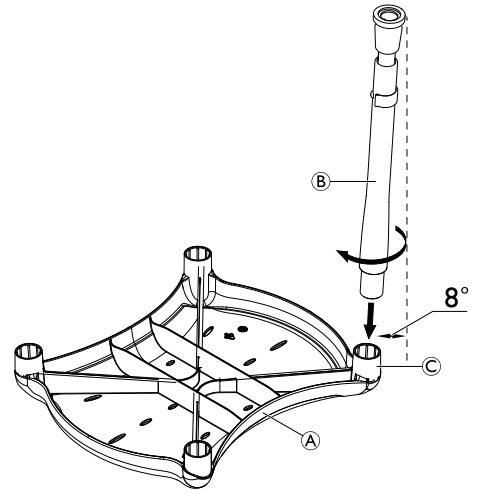
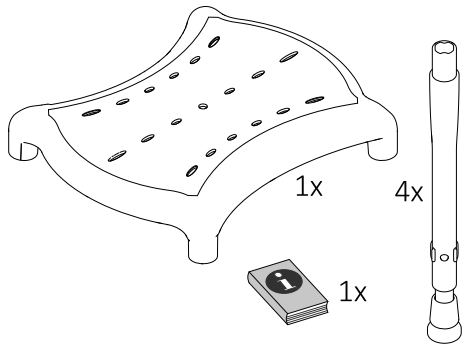
Fig. 1

Fig. 2

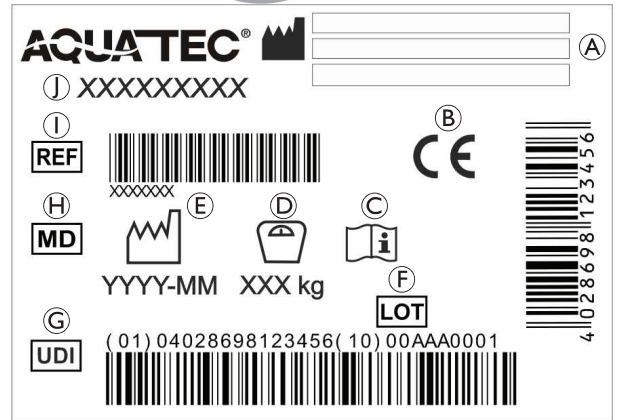
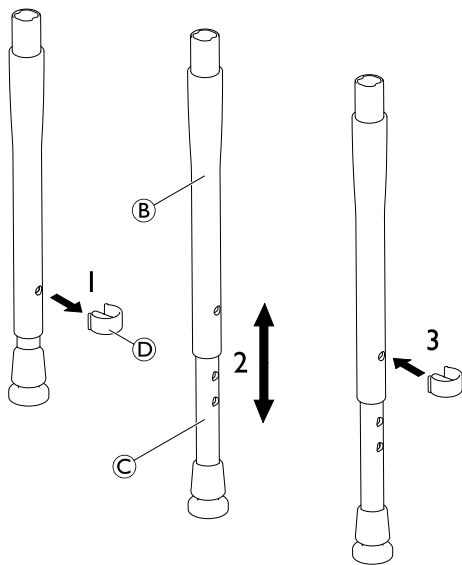
Fig. 3

Fig. 4

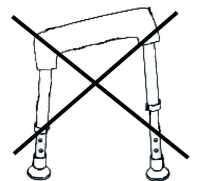
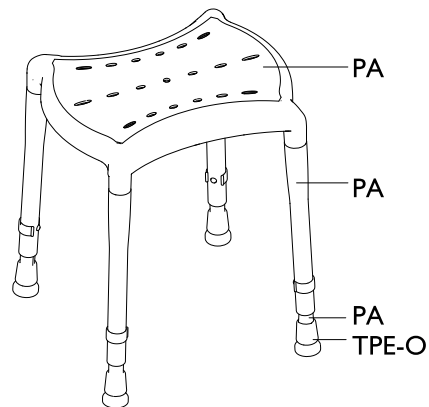
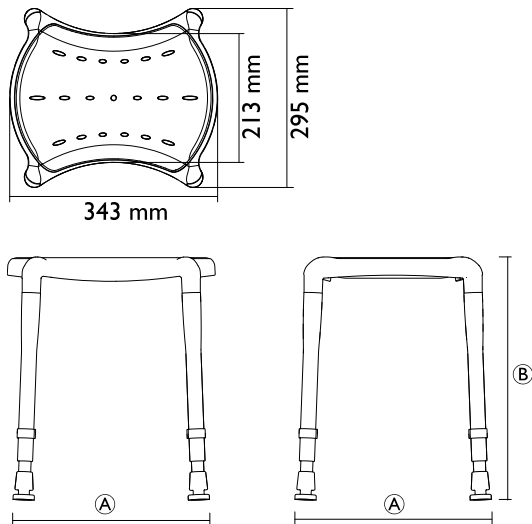
Fig. 5

Fig. 6

Fig. 7

1 General

1.1 Introduction

This user manual contains important information about the handling of the product. To ensure safety when using the product, read the user manual carefully and follow the safety instructions.

Invacare reserves the right to alter product specifications without further notice.

Before reading this document, make sure you have the latest version. You find the latest version as a PDF on the Invacare website.

If you find that the font size in the printed document is difficult to read, you can download the PDF version from the website. The PDF can then be scaled on screen to a font size that is more comfortable for you.

In case of a serious incident with the product, you should inform the manufacturer and the competent authority in your country.

1.2 Warranty Information

We provide a manufacturer's warranty for the product in accordance with our General Terms and Conditions of Business in the respective countries.

Warranty claims can only be made through the provider from whom the product was obtained.

1.3 Compliance

Quality is fundamental to the company's operation, working within the disciplines of ISO 13485.

This product features the CE mark, in compliance with the Medical Device Regulation 2017/745 Class I.

We are continuously working towards ensuring that the company's impact on the environment, locally and globally, is reduced to a minimum.

We only use REACH compliant materials and components.

Product-specific Standards

The product has been tested and conforms to EN 12182 (Assistive products for persons with disability) and related standards. For further information about local standards and regulations, contact your local Invacare representative.

1.4 Service Life

The expected service life of this product is three years when used daily and in accordance with the safety instructions, maintenance intervals and correct use, stated in this manual. The effective service life can vary according to frequency and intensity of use.

1.5 Limitation of Liability

Invacare accepts no liability for damage arising from:

- Non-compliance with the user manual
- Incorrect use
- Natural wear and tear
- Incorrect assembly or set-up by the purchaser or a third party
- Technical modifications
- Unauthorised modifications and/or use of unsuitable spare parts

2 Safety

2.1 Safety Information

WARNING!

Risk of Serious Injury or Damage

Improper use of this product may cause injury or damage.

- If you are unable to understand the warnings, cautions or instructions, contact a health care professional or provider before attempting to use this equipment.
- Do not use this product or any available optional equipment without first completely reading and understanding these instructions and any additional instructional material such as user manual, service manual or instruction sheet supplied with this product or optional equipment.

WARNING!

Risk of falling!

- Do not use if defective.
- Do not make any unauthorized alterations or modifications to the product.
- Do not move or displace the product under load or during use.
- Do not stand on the product.
- Do not use or store the product outdoors.
- Do not tilt.
- Do not sit on the product if not all four legs have full contact to the level ground.
- Do not use for transfer from the side or partial load.

WARNING!

- Before first use, clean the product thoroughly.
- Before every use, check the product for damage and ensure that it is securely assembled.

CAUTION!

Risk of Injury

Non-original or wrong parts may affect the function and safety of the product.

- Only use original parts for the product in use.
- Due to regional differences, refer to your local Invacare catalogue or website for available options or contact your Invacare distributor. See addresses at the end of this document.

2.2 Labels and Symbols on the Product (Fig. 4)

Ⓐ	Manufacturer	Ⓕ	Lot number
Ⓑ	European Conformity	Ⓖ	Unique Device Identifier
Ⓒ	Read user manual	Ⓗ	Medical Device
Ⓓ	Maximum user weight	Ⓛ	Reference number
Ⓔ	Production date	Ⓜ	Product designation

Warning label (red)

- | | |
|---|---|
| Ⓚ | Do not tilt the shower chair during usage |
|---|---|

The labels are attached to the underside of the seat plate.

3 Product Overview

3.1 Intended Use

The Aquatec Dot is a shower stool composed of four anti-slip, height adjustable feet. This product is intended to be used as a shower aid.

Intended User

Users are any adults and adolescents with restricted or limited mobility or limited muscle strength. If used without an assistant, the users must be able to keep their own balance.

Indication

- Restricted or limited mobility (e.g.: legs and/or hips) or
- having balance issues to stand up or walk due to transitional or permanent disabilities.

There are no contraindications known when using the product as intended. For the maximum user weight, see 7 *Technical Data, page 5*.

3.2 Scope of Delivery

The scope of delivery is shown in figure 1.

4 Setup

CAUTION!

Risk of Finger Trapping

- Take care during assembly.

Assembling the Product (Fig. 2)

1. Place the seat plate **A** on an even surface.

NOTICE!

Risk of Damage to the Product

The leg supports **C** are slightly angled outwards (8°) and could be damaged if pressing in the legs **B** vertically from above.

- When attaching the legs to the supports, make sure to hold them slightly angled and simultaneously carry out a rotary movement.

2. Attach the legs **B** into the seat plate by pushing them into support **C**.

WARNING!

Risk of Injury

- Make sure that all four legs are completely inserted in the supports.

Adjusting the Seat Height (Fig. 3)

1. Remove the safety clip **D** from the legs.
2. Adjust the desired height by moving the adjustment tube **C** into or out of the legs **B**.
3. Re-attach the safety clips through the desired holes of the legs and the adjustment tube.

WARNING!

Risk of Falling

- Make sure that all legs are adjusted to the same height (Fig. 7).
- Make sure that the safety clips are fully engaged in the holes of all legs.

5 Maintenance

5.1 General Maintenance Information

This product is maintenance-free, provided that the cleaning and disinfection instructions are adhered to.

5.2 Cleaning and Disinfection

General Safety Information

CAUTION!

Risk of Contamination

- Take precautions for yourself and use appropriate protective equipment.

NOTICE!

Wrong fluids or methods can harm or damage the product.

- All cleaning agents and disinfectants used must be effective, compatible with one another and must protect the materials they are used to clean.
- Never use corrosive fluids (alkalines, acid etc.) or abrasive cleaning agents. We recommend an ordinary household cleaning agent such as dishwashing liquid, if not specified otherwise in the cleaning instructions.
- Never use a solvent (cellulose thinner, acetone etc.) that changes the structure of the plastic or dissolves the attached labels.
- Always make sure that the product is completely dried before taking it into use again.

For cleaning and disinfection in clinical or long-term care environments, follow your in-house procedures.

Cleaning Intervals

NOTICE!

Regular cleaning and disinfection enhances smooth operation, increases the service life and prevents contamination.

Clean and disinfect the product:

- regularly while in use,
- before and after any service procedure,
- when it has been in contact with any body fluids,
- before using it for a new user.

Cleaning

NOTICE!

Maximum cleaning temperature:

- 60 °C by hand

1. Remove detachable parts.
2. Clean the product by wiping down all generally accessible surfaces.
3. Dry the parts with a cloth.

Disinfection Instructions

Method: Follow the application notes for the used disinfectant and wipe-disinfect all accessible surfaces.

Disinfectant: Ordinary household disinfectant.

Drying: Allow the product to air-dry.

6 After Use

6.1 Reconditioning

This product is suitable for reuse. To recondition the product for a new user, carry out the following actions:

- Inspection
- Cleaning and disinfection

For detailed information, see 5 *Maintenance, page 4*.

Make sure that the user manual is handed over with the product.

If any damage or malfunction is detected, do not reuse the product.

6.2 Disposal

Be environmentally responsible and recycle this product through your recycling facility at its end of life.

Disassemble the product and its components, so the different materials can be separated and recycled individually.

The disposal and recycling of used products and packaging must comply with the laws and regulations for waste handling in each country. Contact your local waste management company for information.

7 Technical Data

7.1 Dimensions, Weights and Materials (Fig. 5 – 6)

Ⓐ	365 mm, in highest position
Ⓑ	400 – 550 mm, in steps of 25 mm
Product weight	1.65 kg
Max. user weight	135 kg
Materials	See Fig. 6

All product components are corrosion free and do not contain natural rubber latex.

7.2 Environmental Parameters

Operating Conditions	
Ambient temperature	10 - 40 °C
Relative humidity	15% - 93%, non-condensing
Storage and Shipping Conditions	
Ambient temperature	0 - 40 °C
Relative humidity	30% - 75%, non-condensing

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Kojų atramos © yra truputį pasuktos į išorę (8°) – jei kojos Ⓑ bus spaudžiamos vertikaliai iš viršaus, jos gali būti pažeistos.
Invacare Poirier SAS
Invacare Austria GmbH
Invacare AB
Invacare GmbH