

Spare Parts Catalogue

Version: 2013/02/27b NO





Yes, you can.®

General Information

For any order of spare parts the following information is required:

- Product type*
- Product name & model*
- Serial Number*
- Part-Number*
- Part-Description*

Exploded view drawings and code numbers are not contractual and subject to modification, in relation to Invacare® continuous product improvement policy to meet end users needs.

The reproduction or use of this work, or of any substantial part of this work, in any form or by any electronic, mechanical or other means, including photocopying, scanning and recording, and in any information storage and retrieval system, including electronic transmission, is forbidden without the prior written permission of Invacare® International Sarl.



Yes, you can.®

TABLE OF CONTENTS

(Invacare® SoftAIR Super Complete Mattress & Pump)

(Invacare® SoftAIR Super Complete Mattress & Pump)	5
(Invacare® SoftAIR Super Complete Mattress & Pump)	6

(Invacare® SoftAIR Super Mattress Spares)

(Invacare® SoftAIR Super Mattress Spares)	7
(Invacare® SoftAIR Super Mattress Spares)	8

(Invacare® SoftAIR Pump Spares)

(Invacare® SoftAIR Pump Spares)	9
(Invacare® SoftAIR Pump Spares)	10

(Invacare® SoftAIR Super) | (Invacare® SoftAIR Super Complete Mattress & Pump)
(Invacare® SoftAIR Super Complete Mattress & Pump)



1 -

(Invacare® SoftAIR Super Complete Mattress & Pump)

1

(Invacare® SoftAIR Super Complete Mattress & Pump)



1	1532815	(Complete SoftAIR Super Mattress & Control Unit System. Width 830mm)	1
2	1532816	(Complete SoftAIR Super Mattress & Control Unit System. Width 880mm.)	1
3	1532858	(Control Unit SoftAIR with power cord and user manual.)	1
4	1532859	(Mattress SoftAIR Super. Width 830mm in tote bag (without control unit).)	1
5	1532860	(Mattress SoftAIR Super. Width 880mm in tote bag (without control unit).)	1

(Invacare® SoftAIR Super) | (Invacare® SoftAIR Super Mattress Spares)
(Invacare® SoftAIR Super Mattress Spares)



1 -

(Invacare® SoftAIR Super Mattress Spares)

1

(Invacare® SoftAIR Super Mattress Spares)




1	1532863	(Cushion SoftAIR Superfoot/body width 830mm)	1
2	1532864	(Cushion SoftAIR Super Head. Width 830mm.)	1
3	1532865	(Cushion SoftAIR Super foot/body. Width 880mm.)	1
4	1532866	(Cushion SoftAIR Super Head. Width 880mm.)	1
5	1532820	(Coversheet SoftAIR Super. Width 830mm)	1
6	1532821	(Coversheet SoftAIR Super. Width 880mm.)	1
7	1533365	(Tote Bag Invacare SoftAIR)	1

(Invacare® SoftAIR Pump Spares)



(Invacare® SoftAIR Pump Spares)

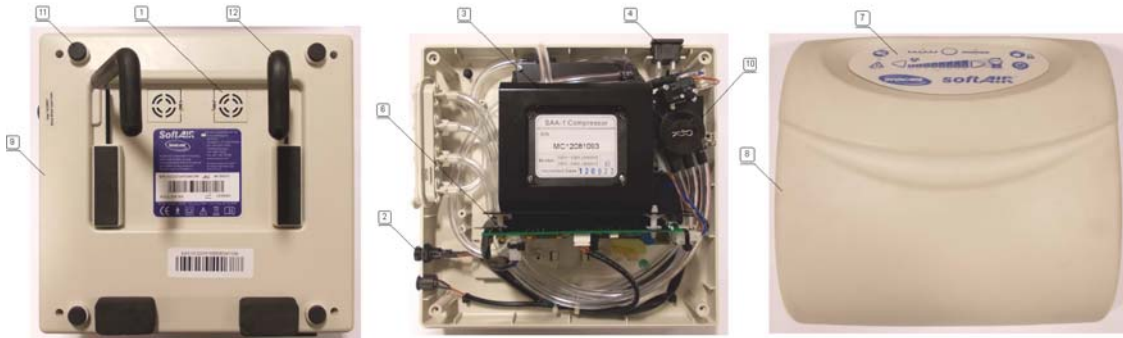

1	1532953	(Air Filter - Control Unit)	1
2	1532990	(Fuse 1A 250V, Control Unit.)	1
3	1539025	(Control unit SoftAIR with power cord and user manual.)	1
3	1533044	(Compressor, Control Unit.)	1
4	1547080	(UK Power Cord, Control Unit.)	1
5	1533195	(User Manual SoftAIR.)	1
6	1557147	(Print Circuit Board (PCB) Control Unit SoftAIR Excellence)	1
7	1533197	(Membrane, Control Unit.)	1
8	1533360	(Front Shell, Top Housing For Invacare SoftAIR.)	1
9	1533361	(Rear Shell, Bottom Housing For Invacare SoftAIR.)	1
10	1533362	(Timing Motor (Rotary Valve Motor, Synchronous Motor 220VAM-1 0.1rpm).)	1
11	1533363	(Kit Of Four (4) Rubber Feet To Rear Shell.)	1
12	1533364	(Hook Repair Kit With Cushions (35mm).)	1
12	1554898	(Hook Repair Kit With Cushions (80mm).)	1



Yes, you can.®



Belgium & Luxemburg:

Invacare nv, Autobaan 22, B-8210 Loppem
Tel: (32) (0)50 83 10 10, Fax: (32) (0)50 83 10 11
belgium@invacare.com / www.invacare.be

Denmark:

Invacare A/S, Sdr. Ringvej 37, DK-2605 Brøndby
Tel: (45) (0)36 90 00 00, Fax: (45) (0)36 90 00 01
denmark@invacare.com / www.invacare.dk

Deutschland:

Invacare Aquatec GmbH, Alemannenstraße 10, D-88316 Isny
Tel: (49) (0)75 62 7 00 0, Fax: (49) (0)75 62 7 00 66
info@invacare-aquatec.com / www.invacare-aquatec.de

European Distributor Organisation:

Invacare, Kleiststraße 49, D-32457 Porta Westfalica
Tel: (49) (0)57 31 754 540, Fax: (49) (0)57 31 754 541
edo@invacare.com / www.invacare.eu.com

España:

Invacare SA, c/Areny s/n, Polígon Industrial de Celrà, E-17460 Celrà (Girona)
Tel: (34) (0)972 49 32 00, Fax: (34) (0)972 49 32 20
contactsp@invacare.com / www.invacare.es

France:

Invacare Poirier SAS, Route de St Roch, F-37230 Fondettes
Tel: (33) (0)2 47 62 64 66, Fax: (33) (0)2 47 42 12 24
contactfr@invacare.com / www.invacare.fr

Ireland:

Invacare Ireland Ltd, Unit 5 Seatown Business Campus, Seatown Road, Swords, County Dublin - Ireland
Tel: (353) 1 810 7084, Fax: (353) 1 810 7085
ireland@invacare.com / www.invacare.ie

Italia:

Invacare Mecc San s.r.l., Via dei Pini 62, I-36016 Thiene (VI)
Tel: (39) 0445 38 00 59, Fax: (39) 0445 38 00 34
italia@invacare.com / www.invacare.it

Nederland:

Invacare BV, Celsiusstraat 46, NL-6716 BZ Ede
Tel: (31) (0)318 695 757, Fax: (31) (0)318 695 758
nederland@invacare.com, csede@invacare.com / www.invacare.nl

Norge:

Invacare AS, Grensesvingen 9, Postboks 6230, Etterstad, N-0603 Oslo
Tel: (47) (0)22 57 95 00, Fax: (47) (0)22 57 95 01
norway@invacare.com, island@invacare.com / www.invacare.no

Österreich:

Invacare Austria GmbH, Herzog Odilostrasse 101, A-5310 Mondsee
Tel: (43) 6232 5535 0, Fax: (43) 6232 5535 4
info-austria@invacare.com / www.invacare.at

Portugal:

Invacare Lda, Rua Estrada Velha, 949, P-4465-784 Leça do Balio
Tel: (351) (0)225 1059 46/47, Fax: (351) (0)225 1057 39
portugal@invacare.com / www.invacare.pt

Sverige & Suomi:

Invacare AB, Fagerstagatan 9, S-163 91 Spånåsa
Tel: (46) (0)8 761 70 90, Fax: (46) (0)8 761 81 08
sweden@invacare.com, finland@invacare.com / www.invacare.se

Switzerland:

Invacare AG, Benkenstrasse 260, CH-4108 Witterswil
Tel: (41) (0)61 487 70 80, Fax: (41) (0)61 487 70 81
switzerland@invacare.com / www.invacare.ch

United Kingdom:

Invacare Limited, Pencoed Technology Park, Pencoed, Bridgend CF35 5AQ
Switchboard Tel: (44) (0)1656 776200, Fax: (44) (0)1656 776201
Customer services Tel: (44) (0)1656 776222, Fax: (44) (0)1656 776220
uk@invacare.com / www.invacare.co.uk