

Spare Parts Catalogue

Version: 2023/02/27a NO



Invacare®
Comet^{PRO} / Comet^{ALPINE+} / Comet^{ULTRA}



Yes, you can.®

General Information

For any order of spare parts the following information is required:

- Product type*
- Product name & model*
- Serial Number*
- Part-Number*
- Part-Description*

Exploded view drawings and code numbers are not contractual and subject to modification, in relation to Invacare® continuous product improvement policy to meet end users needs.

The reproduction or use of this work, or of any substantial part of this work, in any form or by any electronic, mechanical or other means, including photocopying, scanning and recording, and in any information storage and retrieval system, including electronic transmission, is forbidden without the prior written permission of Invacare® International Sarl.



Yes, you can.[®]

TABLE OF CONTENTS

(Chassis, Shrouds & Lights)

(Front Frame)	5
(Front Shroud)	6
(Rear Frame / Motor Swing)	7
(Rear Shrouds)	8
(Colour Panels (6 pieces))	9
(Light System)	10

(Seating)

(Seats & Headrests)	11
(Seat Post)	12
(Seat Bracket)	13
(Slide Assembly)	14
(Armrests)	15

(Wheels)

(Wheels - 5" rim)	16
(Wheels - 8" rim)	17
(Anti-tipper wheel)	18

(Propulsion)

(Motors & Differentials)	19
--------------------------------	----

(Brakes)

(Brake assembly)	20
------------------------	----

(Steering)

(Steering)	21
(Suspension)	22
(Steering linkage)	23

(Batteries, Cables & Charger)

(Battery & Cables)	24
(Chargers)	25

(Electronic)

(Electronic components)	26
(Powerbox & Limit trigger speed limit)	27

(Spare parts for Accessories)

(Safety)	28
(Warning Triangle)	29
(Seating)	30
(Walking support)	31
(Rollator bracket)	32
(Storage)	33
(Storage box rear bracket)	34




Yes, you can.[®]

TABLE OF CONTENTS

(Bumpers)	35
(Other Options)	36

Invacare® Comet PRO / ALPINE+ / ULTRA|(Chassis, Shrouds & Lights)
(Front Frame)


1637039 iso

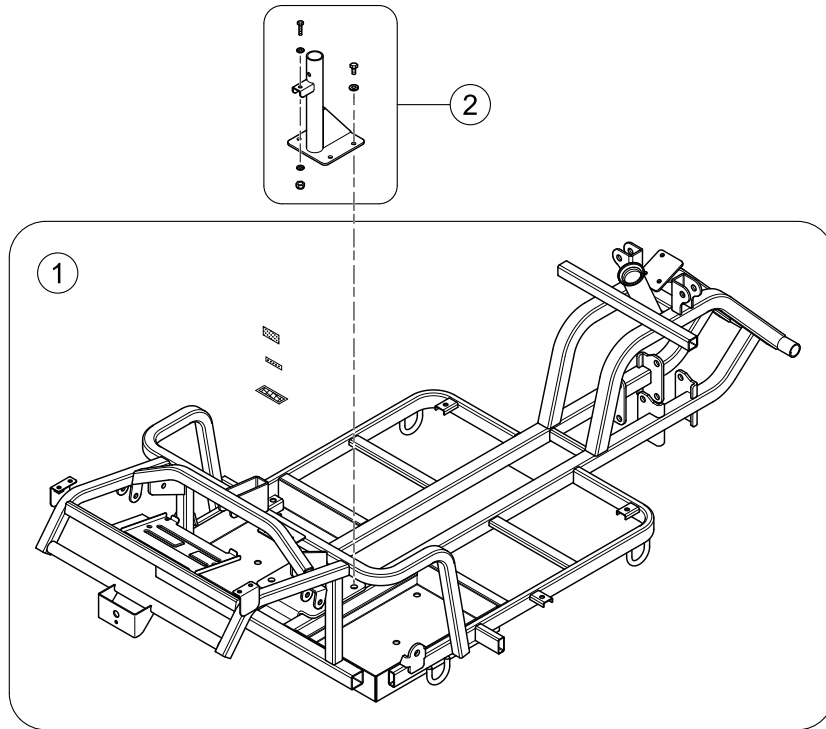

1	SP1636815	(Front frame)	1
2	SP1636817	SETEBRAKETT NEDRECOMET ALPINE+	1


(Front Shroud)

1637108 Iso

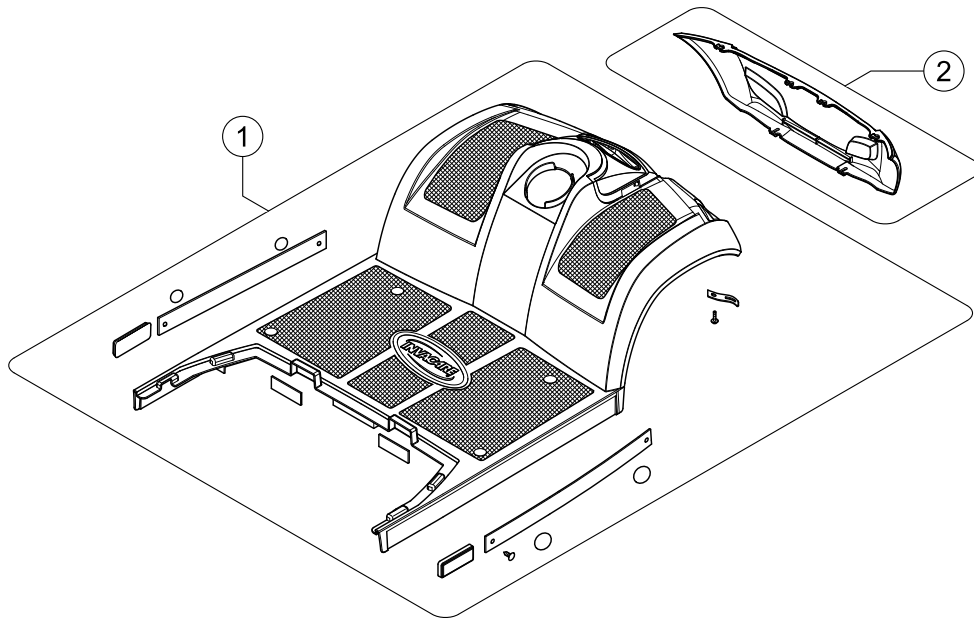

1	SP1636818	FRONTDEKSEL COMET ALPINE+	1
2	SP1608232	DEKSEL STØTFANGER COMET ALPINE+	1

Invacare® Comet PRO / ALPINE+ / ULTRA|(Chassis, Shrouds & Lights)
(Rear Frame / Motor Swing)

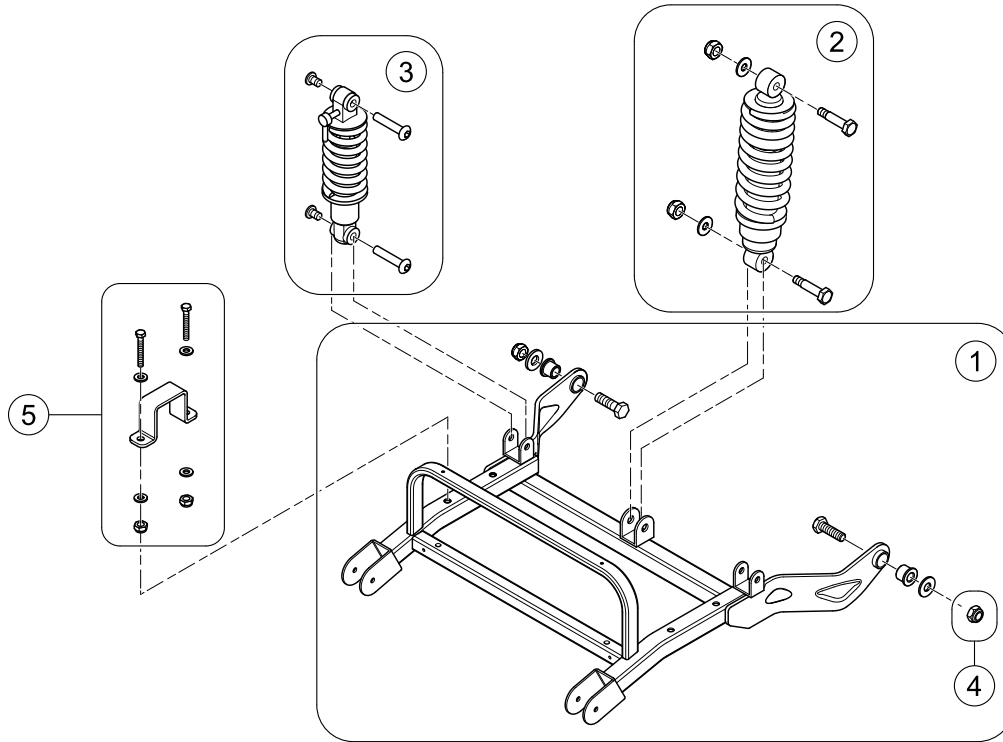


1637105.150

1	SP1636825	SVINGARM BAKRECOMET ALPINE+ <i>Design change - fully retrofittable</i>	1
2	SP1645145	STØTDEMPER SENTERCOMET ALPINE+ <i>Comet PRO / ALPINE+</i>	1
3	SP1636838	STØTDEMPER SIDELENGSCOMET ALPINE+ <i>Comet ULTRA</i>	2
4	SP1505589	NYLONMUTTERCOMET ALPINE+	2
5	SP1636837	HOLDER DIFFERENSIALCOMET ALPINE+	2


(Rear Shrouds)

1637107.iss

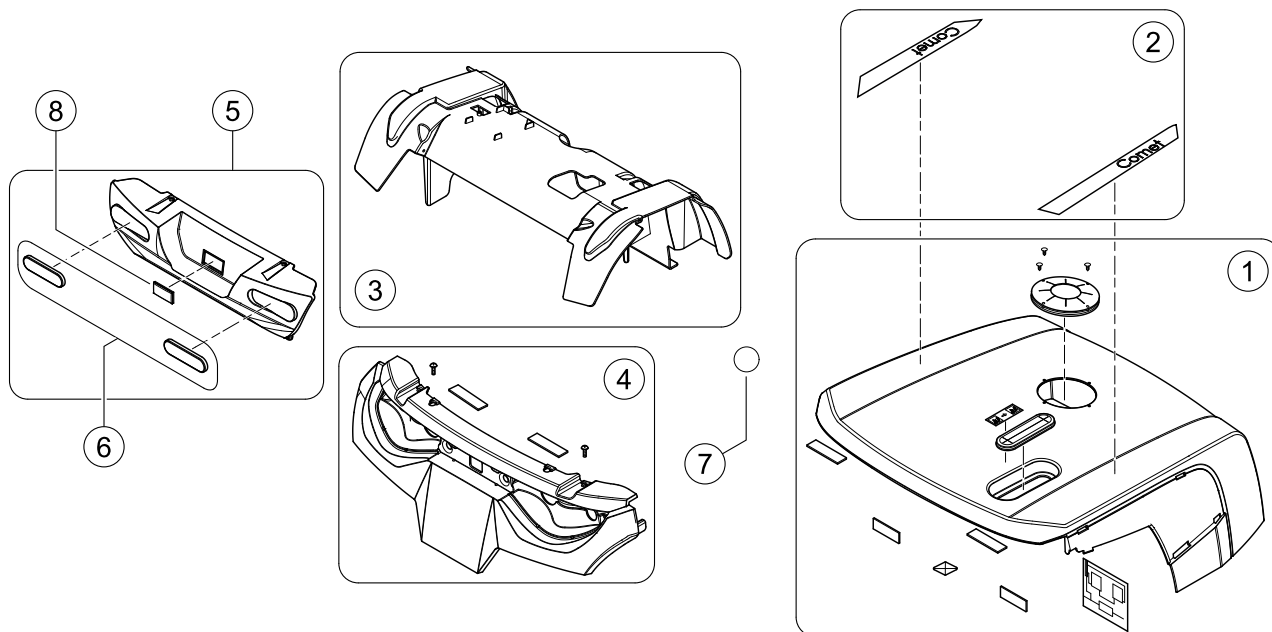

1	SP1608258	DEKSEL OVER BATTERI COMET ALPINE+	1
2	SP1608259	(Label Set - Pro)	1
2	SP1637306	ETIKETTSETT COMETCOMET ALPINE+	1
2	SP1637462	ETIKETTSETT COMETCOMET ULTRA	1
3	SP1608261	SKJERM BAK COMET ALPINE+	1
4	SP1636839	DEKSEL BAK V STØTFANGER COMET ALPINE+	1
5	SP1608260	DEKSEL BAK V STØTFANGER NEDRE DEL COMET ALPINE+	1
6	SP1518036	REFLEKS BAK COMET	1
7	SP1608262	ETIKETT STØTDEMPERCOMET ULTRA	2
8	SP1521879	LIST BAKRE DEKSEL LITEN COMET	1

(Colour Panels (6 pieces))

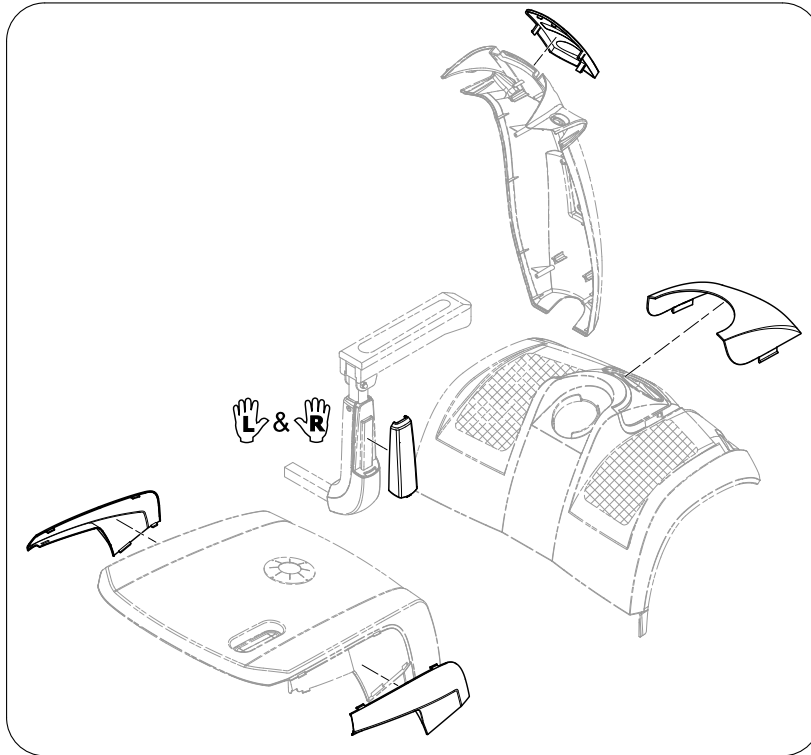


1637123.iso



1	SP1604096	DEKSEL KIT BLÅCOMET ALPINE+ <i>Replaces SP1608823</i>	1
2	SP1608821	DEKSEL SETT HVIT COMET ULTRA	1
3	SP1608822	DEKSEL SETT GRØNN COMET ULTRA ORION	1
4	-	(Color panels - Sapphire Blue no longer available - please choose Alium Blue.) <i>Replaced by SP1604096</i>	1
5	SP1608831	DEKSEL SETT SØLV ORION PRO	1
6	-	(Color panels - Jasper Red no longer available - please choose Garnet Red.) <i>Replaced by SP1654684</i>	1
7	SP1654684	Color Panels Garnet redCOMET ALPINE+	1
8	SP1637415	DEKSEL SETT CHAMPAGNE COMET ULTRA ORION	1
9	SP1637416	DEKSEL SETT KOBBER FARGE COMET ULTRA ORION	1
10	SP1637417	DEKSEL SETT RED CARMINE COMET ULTRA ORION	1
11	SP1637418	DEKSEL SETT ELECTR BLUE COMET ALPINE+	1
12	SP1637419	DEKSEL SETT CROME FARGE COMET ULTRA ORION	1
13	SP1637420	DEKSEL SETT RØD POLYGON COMET UTRA ORION	1
14	SP1637421	DEKSEL SETT BLÅ POLYGON COMET ULTRA ORION	1
15	SP1637422	DEKSEL SETT HVIT POLYGON COMET UTRA ORION	1

(Light System)

1640472.no

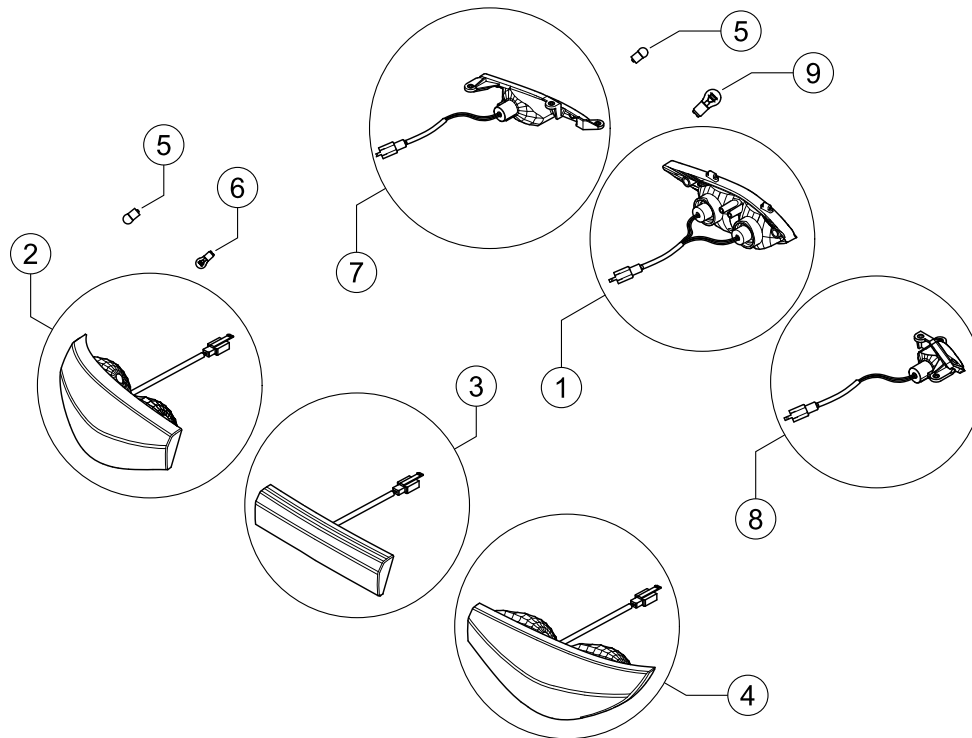



1	SP1527473	HOVEDLYKT KPL COMET	1
2	SP1518034	RETNINGSLYS VE BAK COMET	1
3	SP1518035	BREMSELYS KPL COMET	1
4	SP1518033	RETNINGSLYS HØ BAK COMET	1
5	SP1522030	LYSPÆRE 24V 12W COMET BLINKLYS	4
6	SP1522034	LYSPÆRE 24V 5W COMET BAK	1
7	SP1517733	RETNINGSLYS VE COMET 4W	1
8	SP1517734	RETNINGSLYSHØ COMET 4W	1
9	SP1527474	LYSPÆRE 12V 15W COMET	2

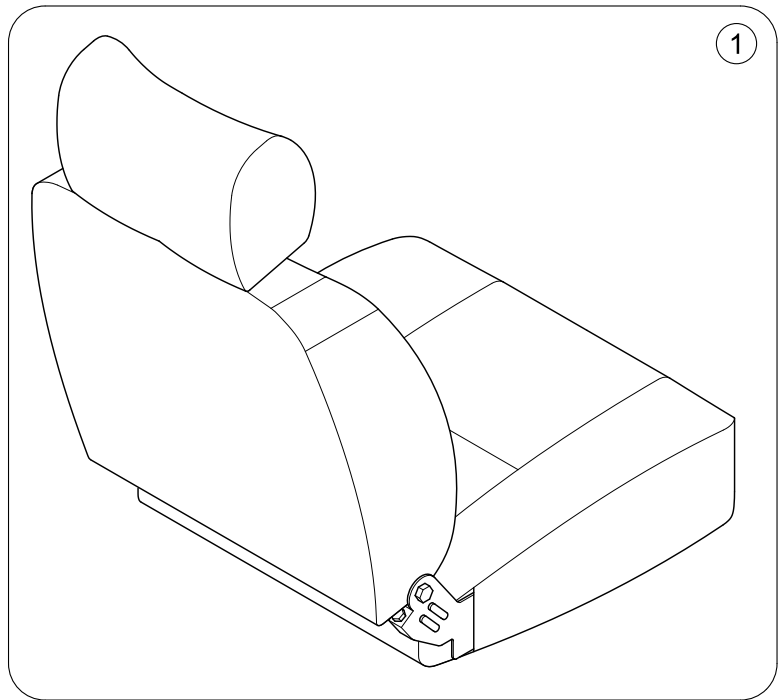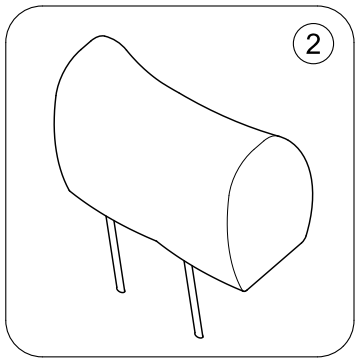

1	SP1608540	SETE COMFORT SB50 ORION PRO 4W COMET	1
1	SP1586116	SETE COMET STD HVIT SØM COMET ALPINE+ <i>Deluxe Seat (DE = Komfort Sitz)</i>	1
1	SP1608556	SETE 20PREMIUM	1
1	SP1652116	Sete 20" Deluxe LowComet Alpine+/Orion Pro <i>Not ULTRA!</i>	1
1	SP1584887	SETE 20 SORTCOMET ALPINE+ <i>NORWAY only!</i>	1
1	SP1586115	SETE COMET ULTRA SB66 COMET ULTRA	1
2	SP1529380	HODESTØTTE COMET	1
2	SP1642145	Kopfstütze Premium Sitz	1
2	SP1602195	(Headrest HD Seat)	1


Invacare® Comet PRO / ALPINE+ / ULTRA|(Seating)
(Seat Post)

1637084.no

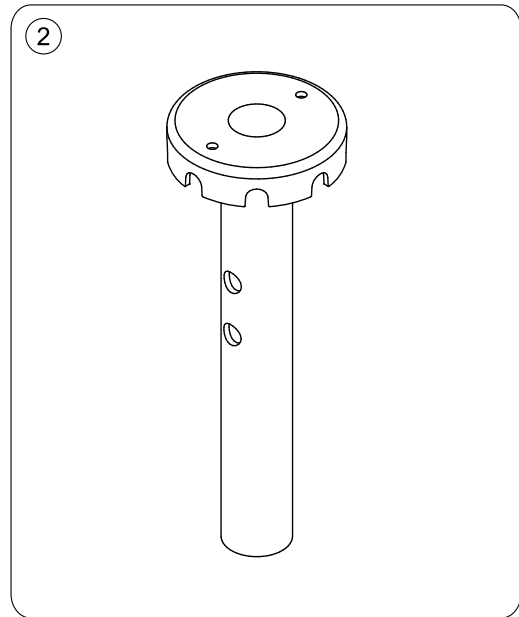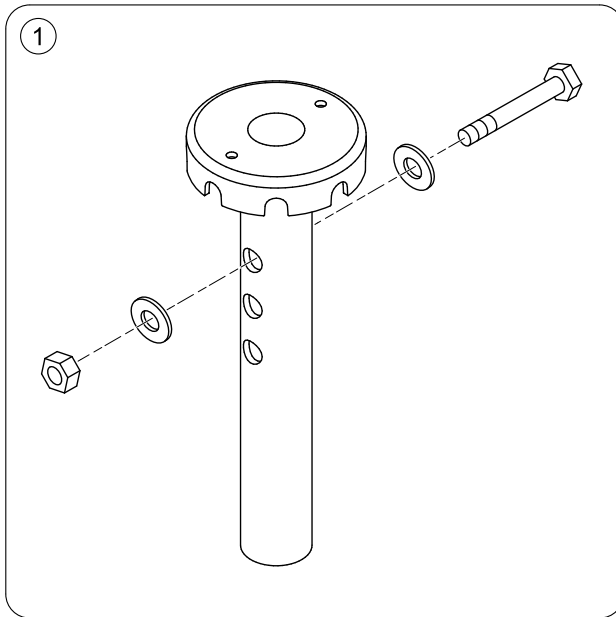

1	SP1521691	SETEBRAKETT M ARMLENSFESTE COMET	1
1	SP1640520	SetepinneComet Ultra	1

(Seat Bracket)

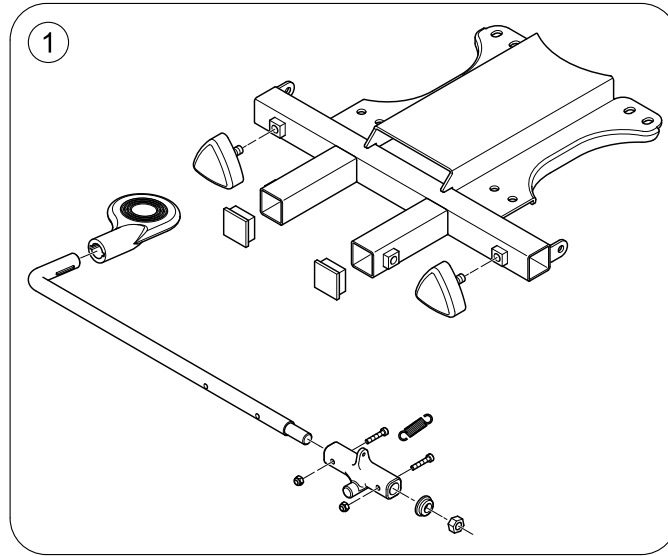


1637085.iso




1	SP1636855	BRAKETT UNDER SETE COMET ALPINE+	1
1	SP1636856	BRAKETT UNDER SETEBARE COMET ALPINE+/ULTRA <i>Comet ALPINE+ / ULTRA only!</i>	1

(Slide Assembly)

1637096 iso

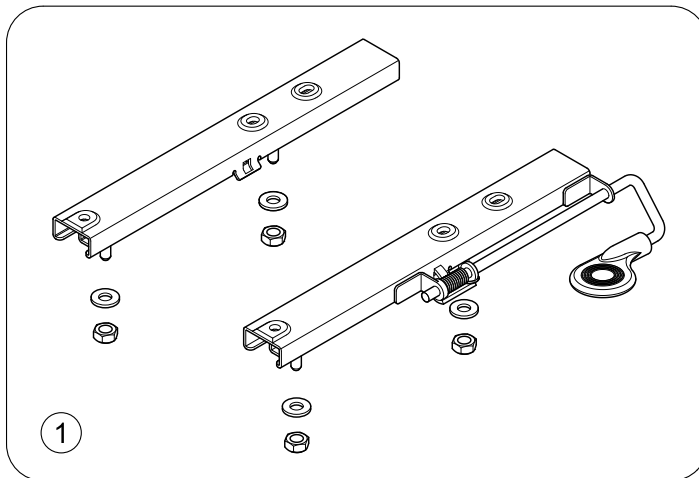


1	SP1521690	GLIDER COMET	1
1	SP1645013	(Slide Assembly - OrionPRO 3W (user weight > 127kg))	1

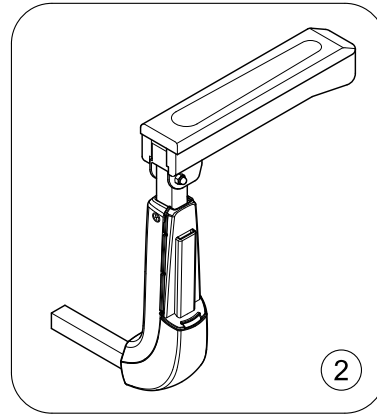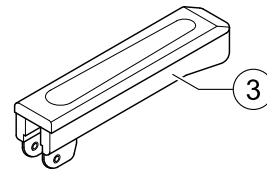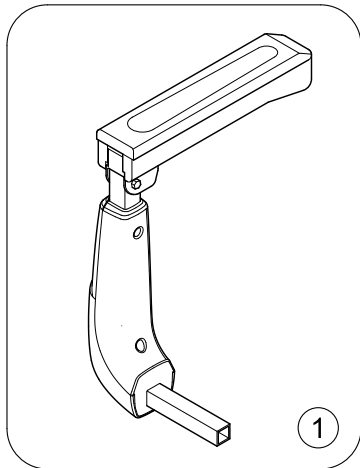

1637396.iso

1	1676255	(Armrest HD, Cpl.- LH, design 2022) <i>New design 2022</i>	1
2	1676256	(Armrest HD, Cpl.-RH, design 2022) <i>New design 2022</i>	1
1	SP1636858	ARMLÉN VENSTRE COMET ALPINE+	1
2	SP1636857	ARMLÉN HØYRECOMET ALPINE+	1
3	1509683	Armlen øvre delCOMET ULTRA	2

(Wheels - 5" rim)

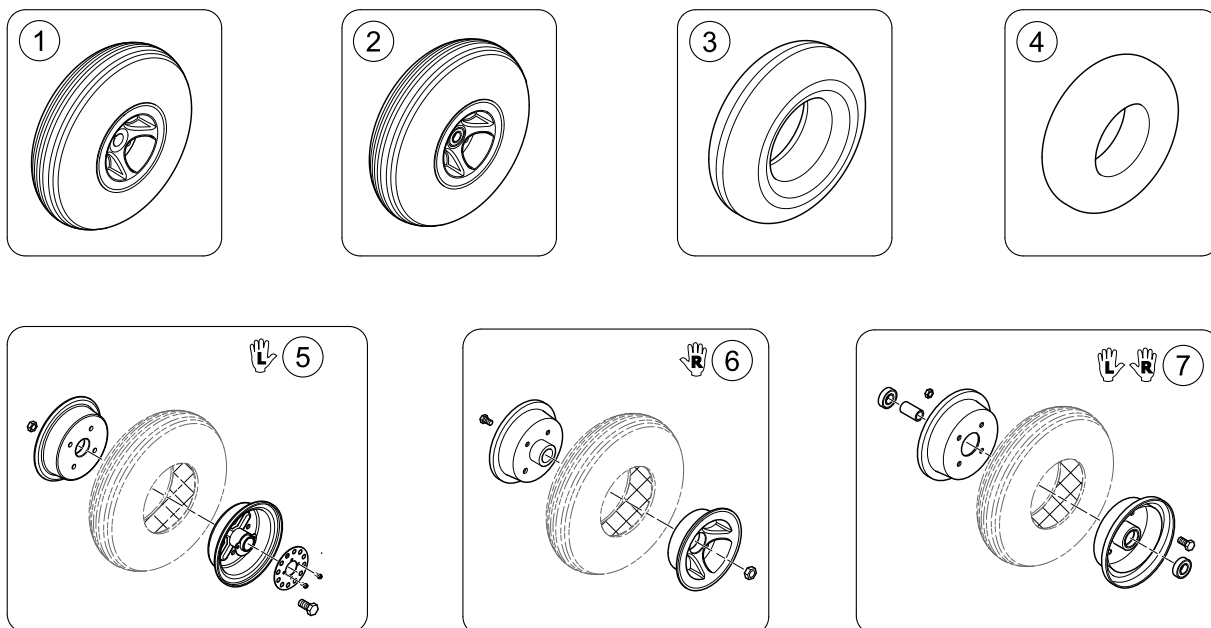


1639752.iso

1	SP1637429	BAKHJUL HØ KPL LUFT COMET ALPINE+ <i>Design change! Replacement: SP1654383</i>	→ 09.10.2019	1
1	1654353	(Rear Wheel 13", pneumatic, 5" rim, RH (new design))	10.10.2019 →	1
1	SP1654383	HJUL LUFT BAK HØ/VE NYKPL COMET ALPINE+	10.10.2019 →	1
1	SP1608775	HJUL KOMPAKT BAK VE KPL COMET ALPINE+ <i>Design change! Replacement: SP1654385</i>	→ 09.10.2019	1
1	SP1654385	HJUL KOMP BAK VE NY KPLCOMET ALPINE+ <i>New design! Replaces SP1608775</i>	10.10.2019 →	1
1	SP1654386	HJUL KOMP BAK NY HØ KPLCOMET ALPINE+ <i>New design! Replaces SP1637431</i>	10.10.2019 →	1
2	SP1608594	FORHJUL COMET ALPINE+KPL COMET ALPINE+		1
2	SP1654351	FORHJUL COMET KPL LUFTNY <i>New Design! Replaces: SP1608594</i>		1
2	SP1654357	FORHJUL KOMPAKT KPL NYCOMET ALPINE + <i>New design! Replaces SP1608696</i>	10.10.2019 →	1
3	SP1640471	DEKK SORT 4.00-5 STD COMET ALPINE+		1
3	SP1586119	DEKK KOMPACT COMET+ COMET ALPINE+		1
4	SP1527714	SLANGE COMET		1
5	SP1638864	FELG BAK VE COMET ALPINE+ <i>LH - Design change! Replacement: SP1654350</i>	→ 09.10.2019	1
5	SP1654350	FELG BAK COMET NYCOMET <i>LH - New design! Replacing: SP1638864</i>	10.10.2019 →	1
6	SP1654167	FELG BAK HØCOMET ALPINE+ <i>RH - New design! Replacing: SP1638838</i>	10.10.2019 →	1
7	SP1638837	FELG FORAN COMET ALPINE+ <i>Design change! Replacement: SP1654355</i>	→ 09.10.2019	1
7	SP1654355	FELG FORAN HØ/VE COMETCOMET NY 5" <i>New design!</i>	10.10.2019 →	1

(Wheels - 8" rim)


1639753 iso

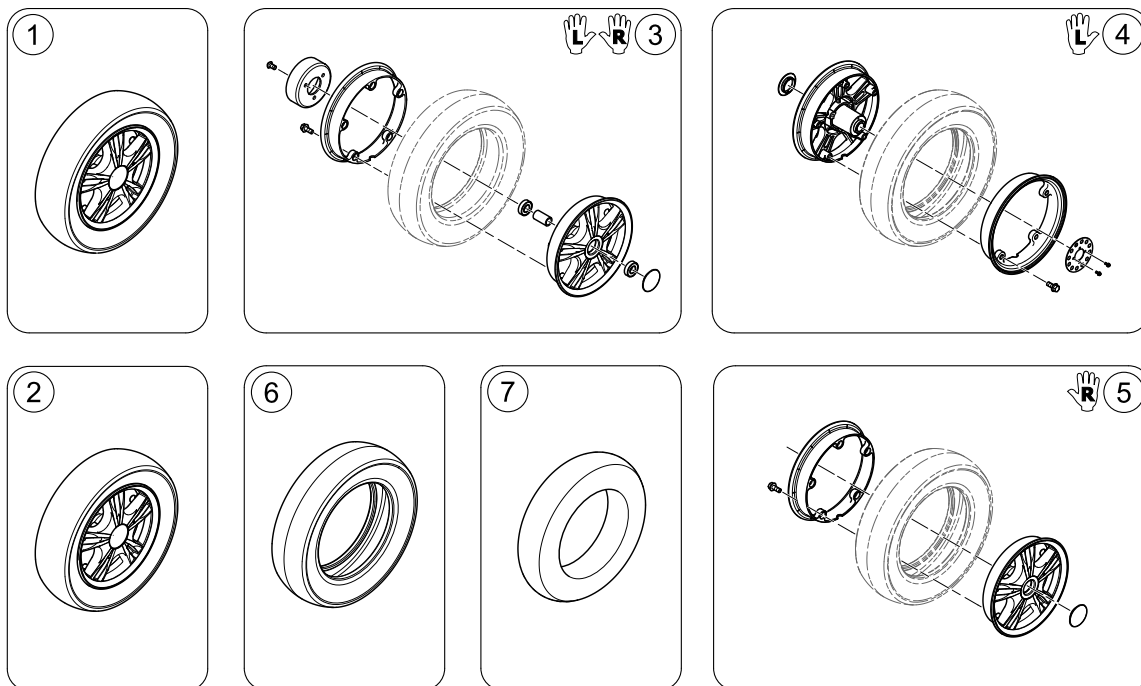

1	SP1608797	FORHJUL LAVPROFIL KPL COMET ALPINE+	1
2	SP1608819	BAKHJUL LAVPROFIL VE KPL COMET ALPINE+	1
3	SP1637433	BAKHJUL LAVPROFIL HØ KPL COMET ALPINE+	1
4	SP1638855	(8" Rim Kit, Front)	1
5	SP1638842	(8" Rim Kit, Rear LH)	1
6	SP1638856	(8" Rim Kit, Rear RH)	1
7	SP1638843	DEKK LAVPROFILCOMET ALPINE+	1
	SP1638979	(Tube 13" for 8" rim)	1

Invacare® Comet PRO / ALPINE+ / ULTRA(Wheels)
(Anti-tipper wheel)


1637088 iso

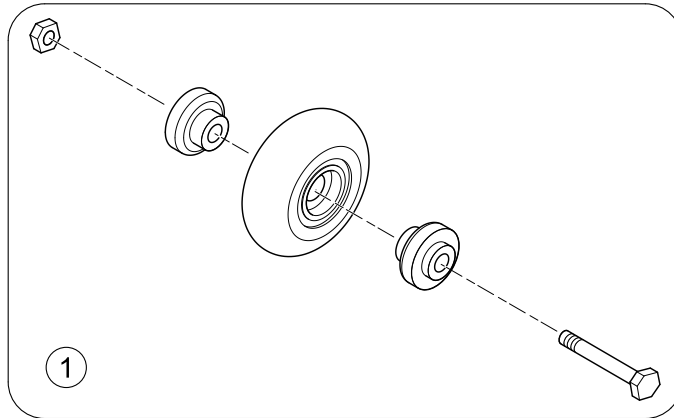

1 SP1522021

HJUL ANTI-TIPPER COMET

1

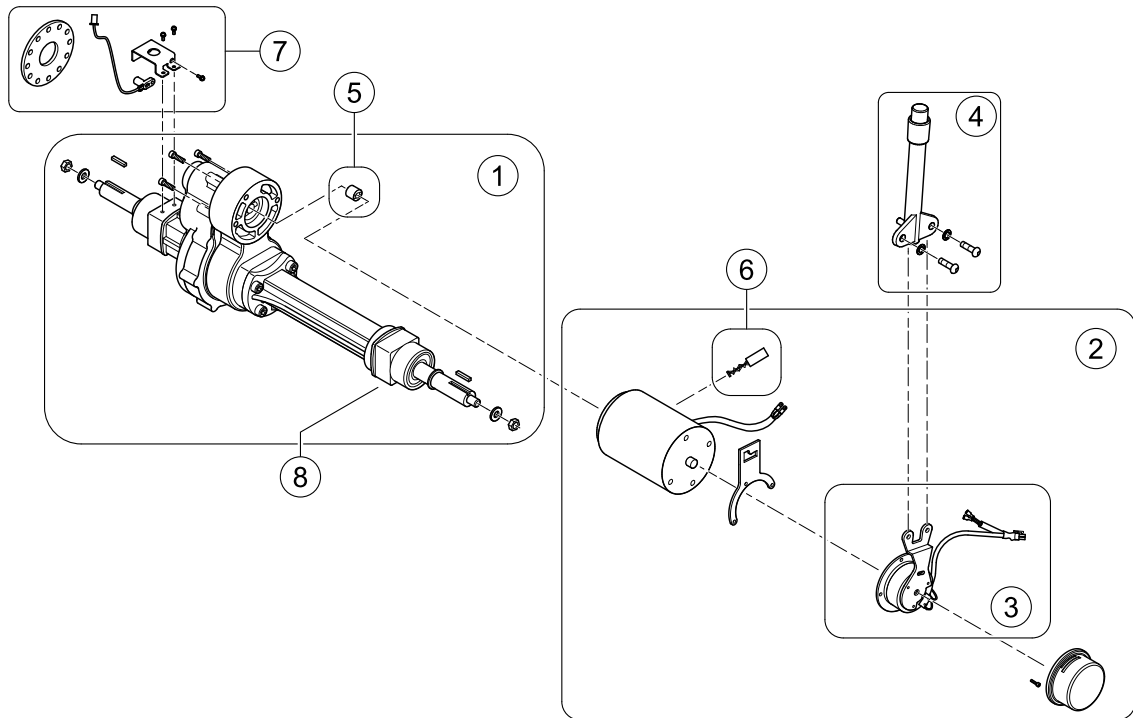


1637109.iso


1	SP1636893	(Differential - Pro)	1
1	SP1636894	DIFFERENSIAL COMET+ COMET + COMET ULTRA	1
2	SP1608264	(Motor 10 km/h - Pro)	1
2	SP1608265	(Motor 12.8 km/h - Pro)	1
2	SP1608266	(Motor 15 km/h - Pro)	1
2	SP1636875	MOTOR COMET ALPINE+ULTRA COMET ALPINE+	1
3	SP1636879	Clutch og BremsComet, Alpine, Ultra 10 km/h (Pro)	1
3	SP1636891	BREMS M CLUTCHORION PRO 12.8 & 15 km/h (Pro)	1
3	SP1636892	MAGNETBREMSCOMET ALPINE+ 10 km/h (Alpine+ / Ultra)	1
4	SP1505672	FRIKOPLINGSHENDEL COMET	1
5	SP1571584	STYREPINNE MOTORORION PRO	1
6	SP1636876	(Carbon brush) 10 km/h (Pro)	1
6	SP1636877	BØRSTORION PRO 12.8 & 15 km/h (Pro)	1
6	SP1636878	(Carbon brush) 10 km/h (Alpine+ / Ultra)	1
7	SP1643083	HASTIGHETSSENSOR KITCOMET ALPINE+ replaces item SP1636874	1
8	SP1571585	(Rubber Pad)	1

(Brake assembly)

1638922 iso

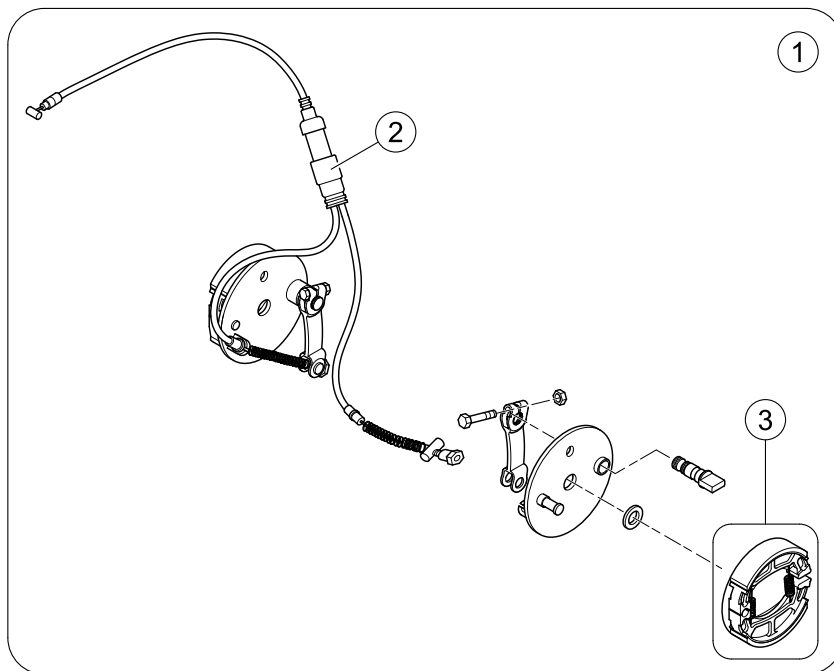


1	SP1636873	BREMS COMETCOMET ALPINE+	1
2	SP1608231	BREMSEWIRECOMET ALPINE+	1
3	SP1518418	BREMSESKOCOMET ALPINE+	1

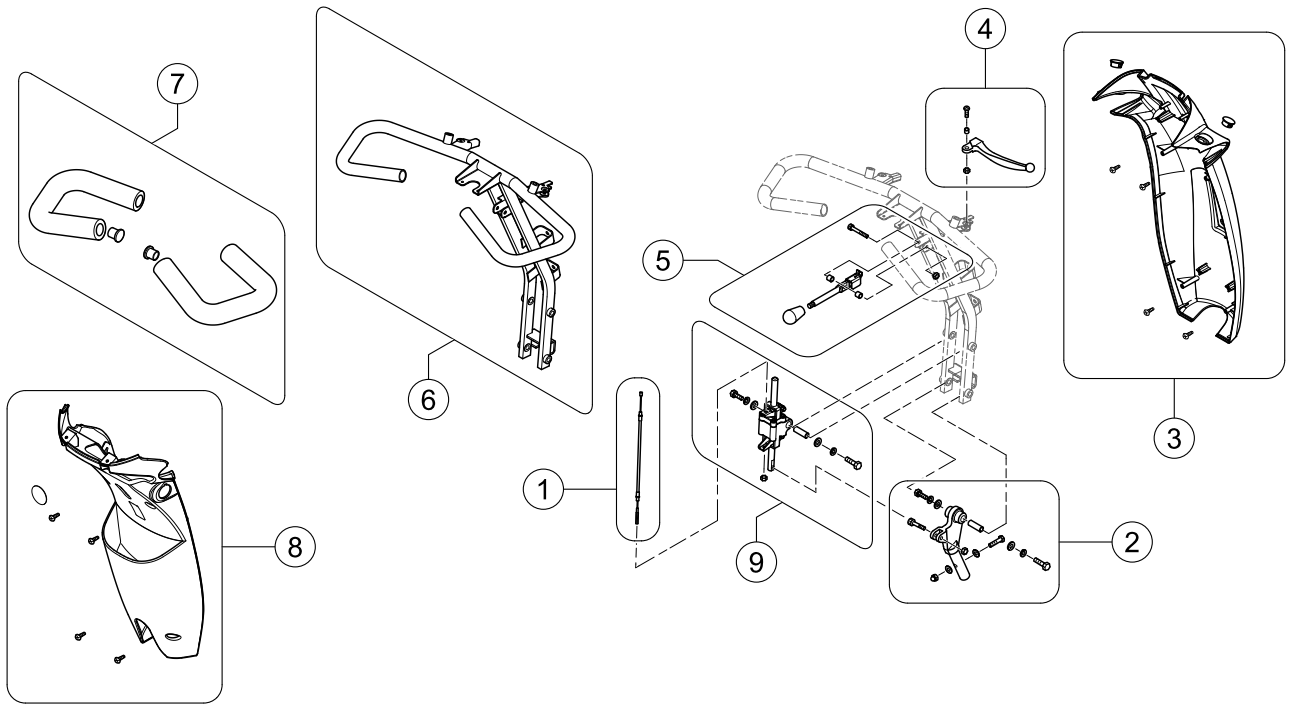


	SP1637540	(Reflective Sticker)	1
1	SP1608201	KABEL STYRINGCOMET ALPINE+	1
2	SP1636812	STYRELEDDCOMET ALPINE+	1
3	SP1608205	DEKSEL STYRE COMET ALPINE+	1
4	SP1636804	HENDEL BREMS HØCOMET ALPINE+	1
5	SP1636813	HENDEL REG STYRECOMET ALPINE+	1
6	SP1608184	STYRESTAMMECOMET ALPINE+	1
7	SP1636767	GREPCOMET ALPINE+	1
8	SP1636814	DEKSEL RATTSTAMME BAKRE COMET ALPINE+	1
9	SP1608202	STYREJUSTERINGCOMET ALPINE+	1

(Suspension)

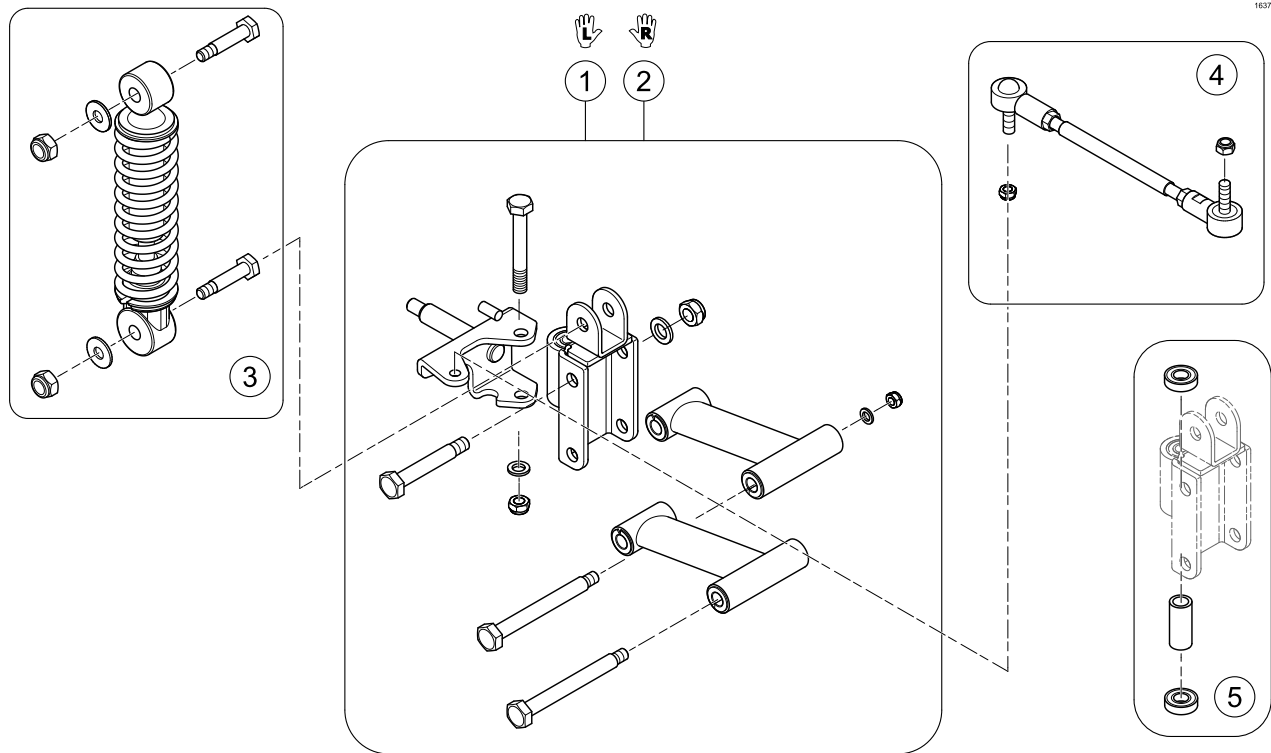


1	SP1636860	STØTDEMPER HJUL VECOMET ALPINE+	1
2	SP1636862	STØTDEMPER ARM HØCOMET ALPINE+	1
3	SP1636864	STØTDEMPER KPLCOMET ALPINE+	1
4	SP1636872	FRONT STAGCOMET ALPINE+	1
5	SP1636859	LAGERKIT COMET ALPINE+	1

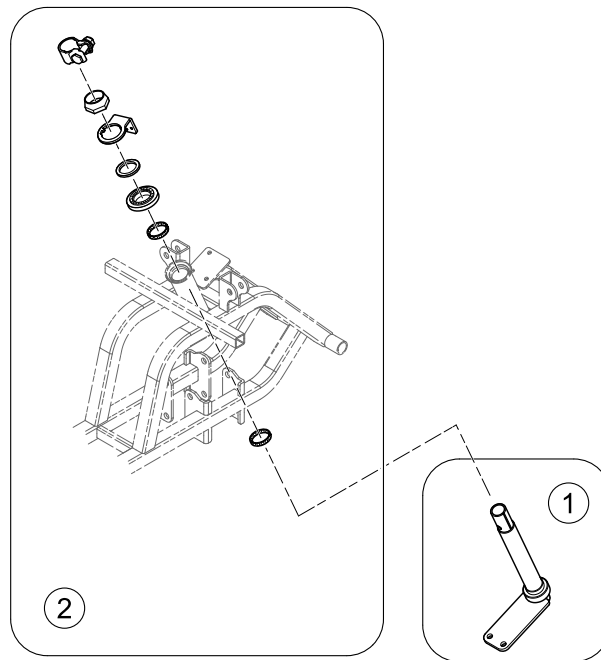

1	SP1608211	MONTERINGSKIT STYRINGCOMET ALPINE+	1
2	SP1636816	(Bearing Kit)	1


(Battery & Cables)

1637121 iso

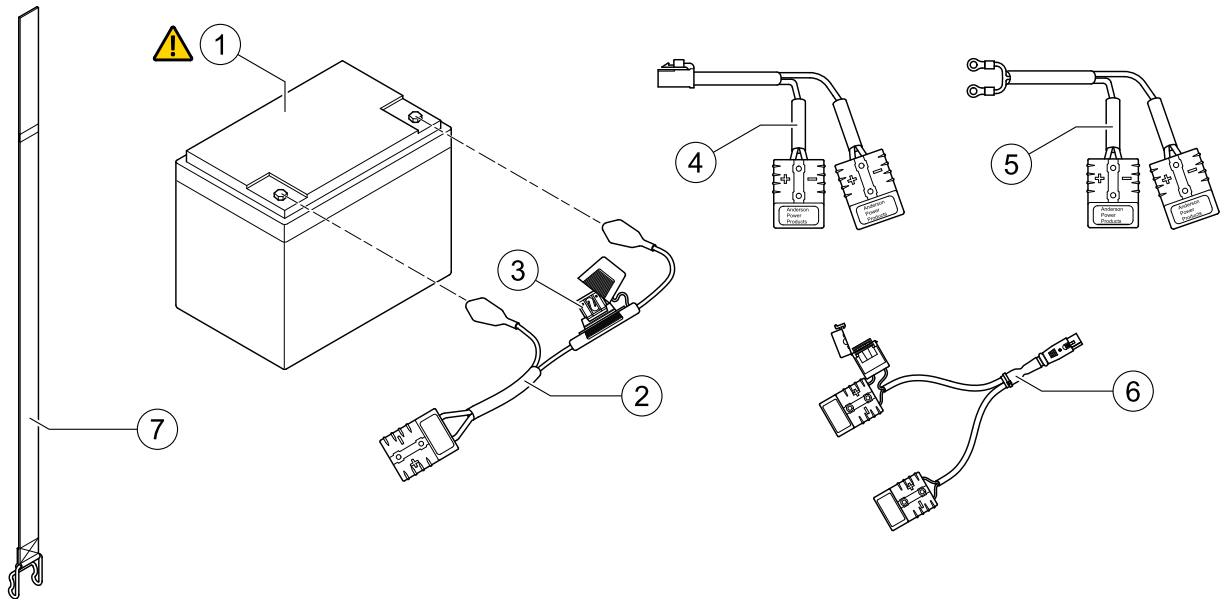

!	-	(Important advice! Replace batteries in pairs. Never mix: manufactures, date codes or technologies (Gel with AGM batteries))	0
1	SP1522026	BATTERI AGM 73.5COMET ALPINE+	2
2	SP1636819	(Fuse battery cable (70A)) <i>PRO</i>	1
2	SP1636822	SIKRING BATTERIKABELCOMET ALPINE+ <i>ALPINE+ & ULTRA</i>	1
3	SP1636823	(Fuse 70A)	1
3	SP1636824	SIKRING 100ACOMET ALPINE+	1
4	SP1608210	BATTERIKABELCOMET ALPINE+	1
5	SP1637918	BATTERIKABELCOMET ALPINE+/ULTRA	1
6	SP1641608	(Off board charging cable V0)	1
7	SP1521758	BATTERISTROPP COMET	1

(Chargers)

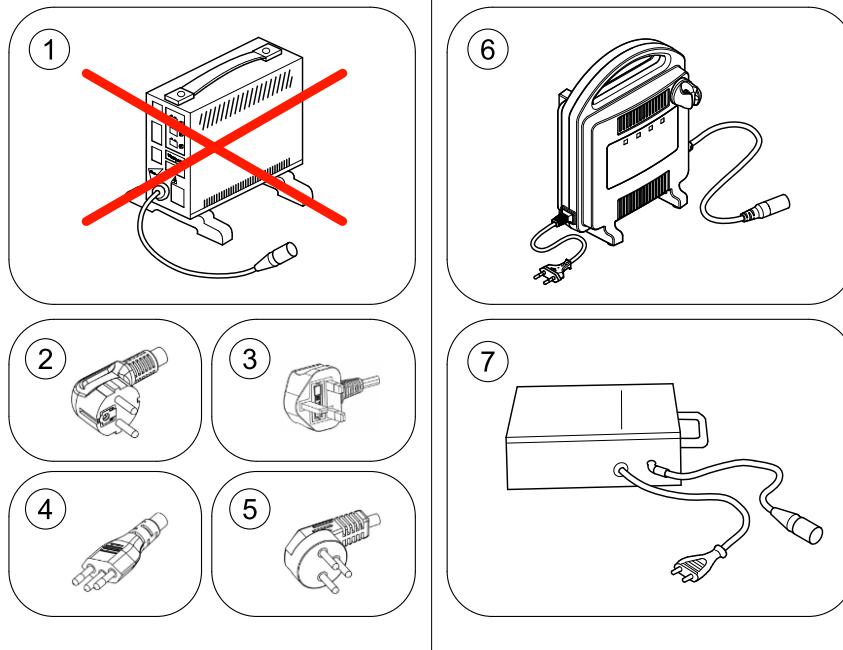


1640673.iso

1	-	(No longer available.)	1
2	P21079	Power Cord - EU PlugScooter	1
3	P21080	(Power Cord - UK Plug (IEC 60320-C13 / BS 1363 Type G))	1
4	1443841	(Power Cord - Swiss Plug (IEC 60320-C13 / SEV 1011 Type J))	1
5	SP1553849	(Power Cord - Israel Plug (IEC 60320-C13 / SI 32 Type H))	1
6	SP1606541	LADER OPTICHARGE COMET ALPINE+ [1343]	1
6	SP1606543	(Charger OptiCharge 8A - UK Plug) [1343]	1
6	SP1606544	LADER OPTICHARGE 10A OPTICHARGE [1313]	1
6	SP1606545	(Charger OptiCharge 10A - UK-Plug) [1313]	1
7	5813987	LADER 8A COMET ALPINE+ [1388]	1

(Electronic components)


1637110.iso

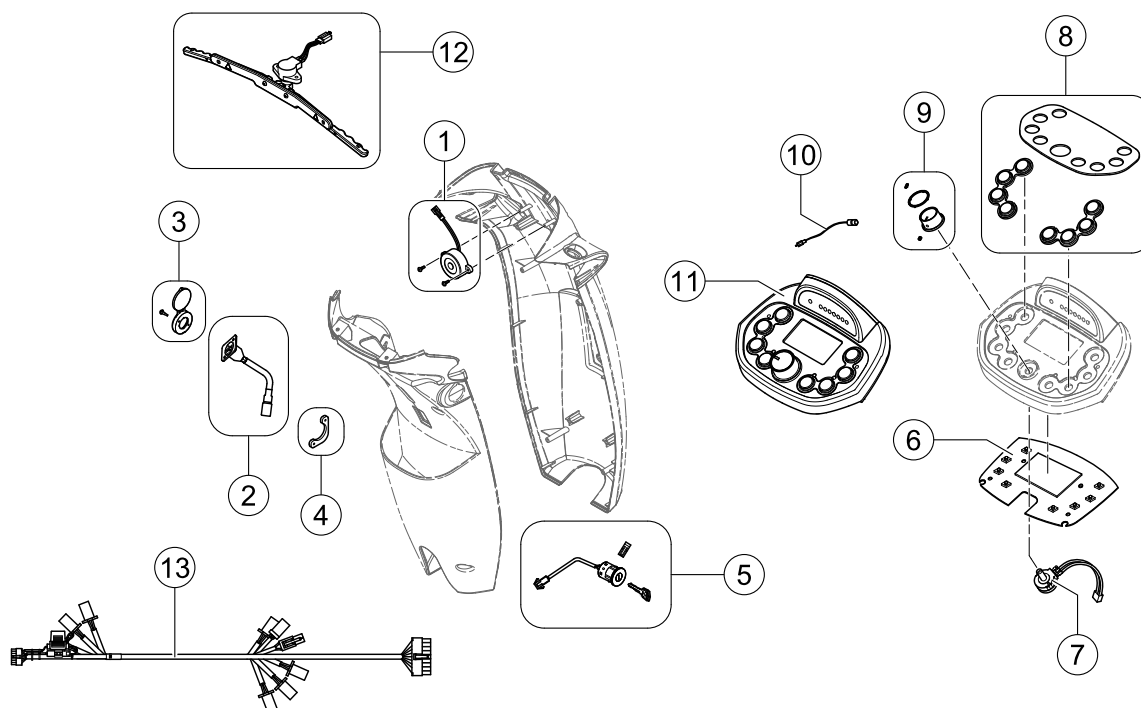

1	SP1506153	BUZZER COMET ALPINE+	1
2	SP1523543	LADEPLUGGCOMET ALPINE+	1
3	SP1523534	DEKSEL LADEKONTAKT COMET COMET ALPINE +	1
4	SP1523544	PLATE STYRE COMET	1
5	SP1505451	LÅSESYLINDERCOMET ALPINE+	1
6	SP1608208	PRINTKORT STYRE COMET ALPINE +	1
7	SP1505462	POTENSIOMETER GASSREG COMET ALPINE+	1
8	SP1637215	(Display cover & rubber keys (LCD Display))	1
9	SP1608207	BRYTER FARTSINSTILLINGCOMET ALPINE+	1
10	SP1608204	TEMPERATURFØLERCOMET ALPINE+	1
11	SP1608185	PANEL KONPLETT M LCD COMET ALPINE+	1
12	SP1608200	GASSHENDEL KPL COMET	1
13	SP1608229	LEDNINGSNETTCOMET ALPINE+	1
-	SP1521577	SIKRING 10A COMET	1

Invacare® Comet PRO / ALPINE+ / ULTRA(Electronic)
(Powerbox & Limit trigger speed limit)


1637122 Iso

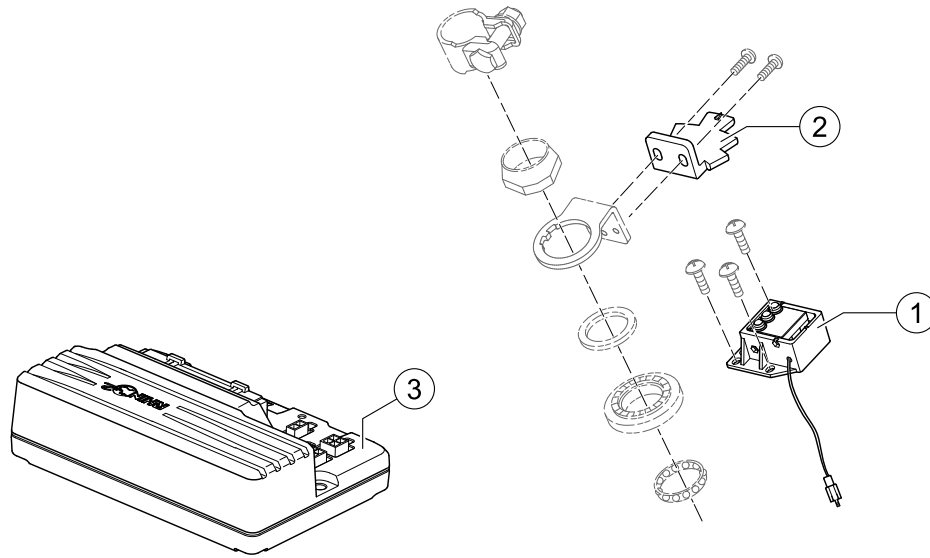


-	-	(For Mounting the Speed Reduction Switch, the Adjustment Gauge SP1581766 is needed!)	1
-	SP1581766	Fartsbegrenser CometOrion	1
1	SP1556266	BRYTER FARTSREDUKSJON COMET 2 GENERASJON <i>Comet Pro/Alpine+/Ultra 10 km/h</i>	1
1	SP1556281	FARTSBEGRENSER BRYTERCOMET <i>Comet Pro 12.8 & 15 km/h</i>	1
2	SP1608230	BRAKETT FARTSREDUKSJONCOMET ALPINE+	1
3	SP1553399	STYRING MOTORORION PRO <i>Comet Pro. Please include the model and max. speed of the scooter in the comment section of your order to receive the correct programming of the controller.</i>	1
3	SP1553400	POWERMODUL MODUL COMET ALPINE FRA OKT2011 <i>Comet Alpine+/Ultra. Please include the model and max. speed of the scooter in the comment section of your order to receive the correct programming of the controller.</i>	1
3	1553399	(Controller, Rhino2, 120A, DS120) <i>unprogrammed Module</i>	1
3	1553400	POWERMODUL MODUL COMET BRUK SP1553400 <i>unprogrammed Module</i>	1

(Safety)

1640376.no

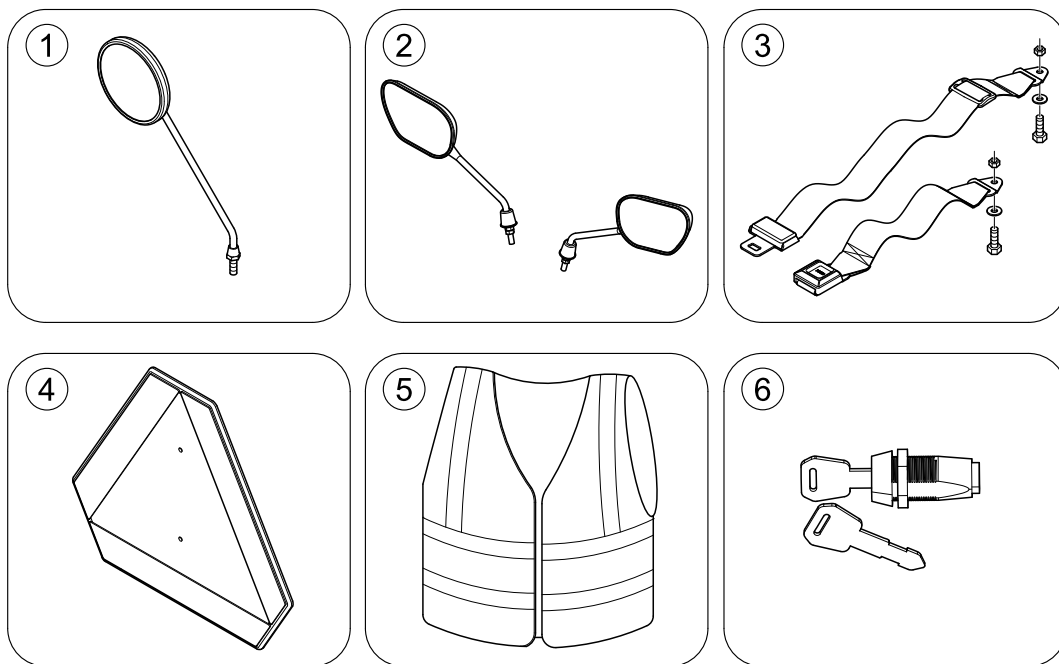


1	SP1518241	SPEIL COMET	1
2	SP1637911	SPEIL HIGH ENDCOMET ALPINE+	1
3	SP1518274	HOFTEBELTE COMET	1
4	-	(Warning Triangle)	1
5	SP1521592	SIKKERHETSVESTCOMET ALPINE+	1
6	SP1543154	LÅS M 2 NØKLER UNIK COMET	1

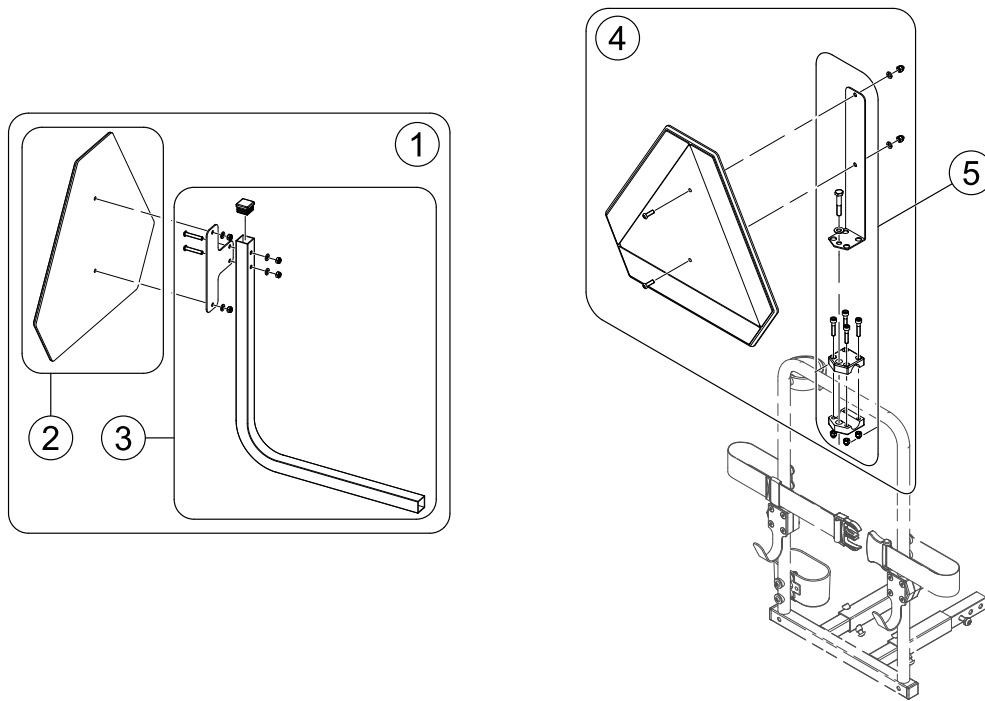


1	SP1523648	(Warning Triangle Scooter)	1
2	1430834	(Rear Marking Sign ECE-R 69)	1
3	SP1526913	(Holder Warning Triangel)	1
4	SP1430410	(Reflective Sign ECE-R 69, complete)	1
5	SP1580671	(Bracket, Reflective Sign, cpl.)	1

(Seating)

1640517.iso

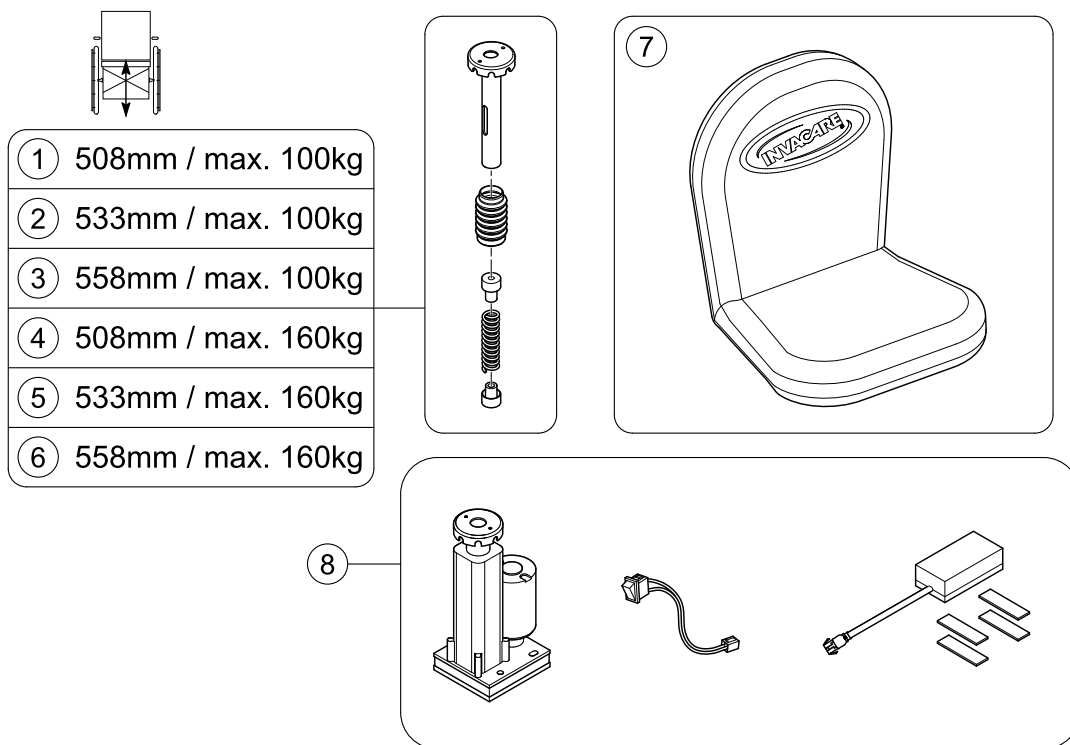


1	SP1640428	FJÆRING SETE SH50 COMET ALPINE+	1
2	SP1640429	(Seat Suspension 226mm 285lbs)	1
3	SP1640430	(Seat Suspension 251mm 201lbs)	1
4	SP1640447	SETEFJÆRING SH50COMET ALPINE NG	1
5	SP1640448	(Seat Suspension 276mm 201lbs)	1
6	SP1640449	Fjæring sete 558mmmax 160 kg Comet Alpine+	1
7	SP1521684	SETETREKK STD COMET <i>Not HD Seat!</i>	1
8	SP1641662	SETELØFTER ELEKTRISK KPLCOMET ULTRA/ALPIN+ <i>NOT Comet Ultra!</i>	1

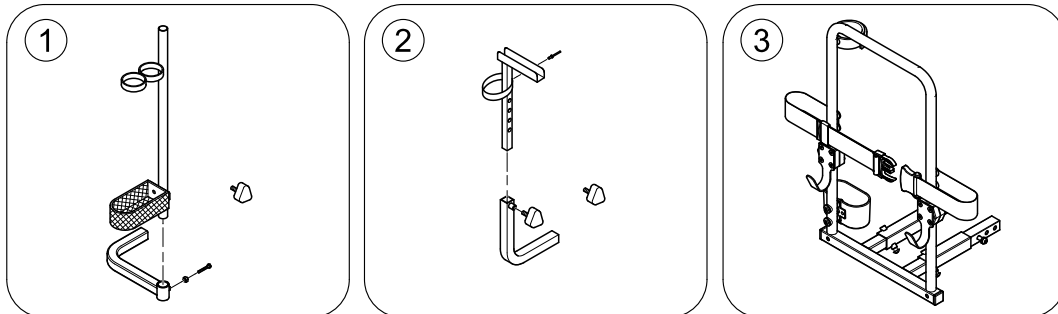


1	SP1528274	KRYKKEHOLDER COMET COMET	1
2	SP1505728	HOLDER FOR GÅRAMME COMET	1
3	-	(Rollator bracket)	0

(Rollator bracket)

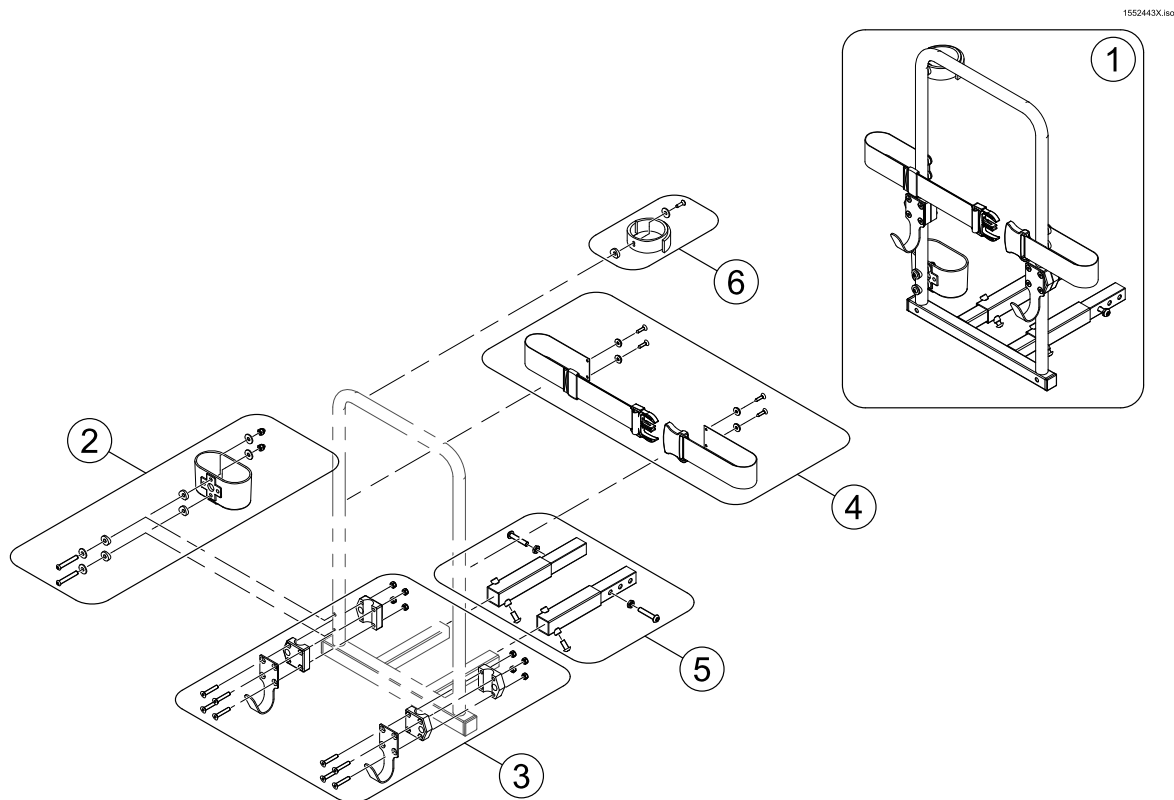

1	SP1553851	RULLATORHOLDER COMET UNI HVIS HD HUSK SP1552594	1
2	SP1560465	KRYKKEHOLDER KUN SKÅL COMET	1
3	SP1553890	KROK HOLDER RULLATORCOMET ALPINE+	1
4	SP1553891	(Belt, cpl.)	1
5	SP1552594	FORLENGER FESTE KRYKKEH COMET <i>only needed for Orion & Comet</i>	1
6	SP1640107	REM KRYKKEHOLDERCOMET ALPINE+	1

(Storage)

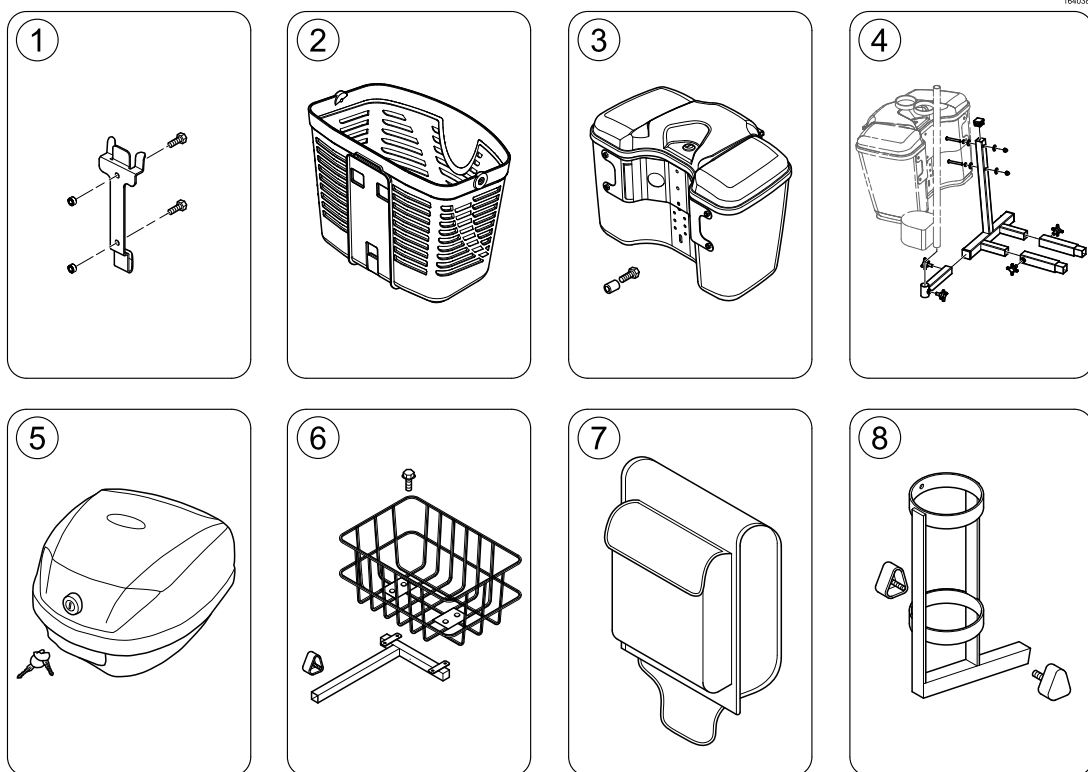


1640381.no

1	SP1505924	BRAKETT BAGASJEBOKSCOMET ALPINE+	1
2	SP1577430	KURV FORAN COMET+ ULTRA COMETA ALPINE+	1
2	SP1637915	KURV FORAN STOR COMET ALPINE+	1
		<i>Not shown / Non illustré / Nicht abgebildet</i>	
3	SP1637910	BAGASJEBOKS FORAN COMET ALPINE+	1
4	-	(Storage box rear bracket)	1
5	SP1637913	BAGASJEBOKS LÅSBAR COMET ALPINE+	1
6	SP1505727	KURV BAK COMET COMET	1
7	SP1505731	VESKE FOR RYGGCOMET ALPINE+	1
8	SP1572971	HOLDER OKSYGENFLASKE COMET ALPINE+	1

(Storage box rear bracket)



1552537X.iss

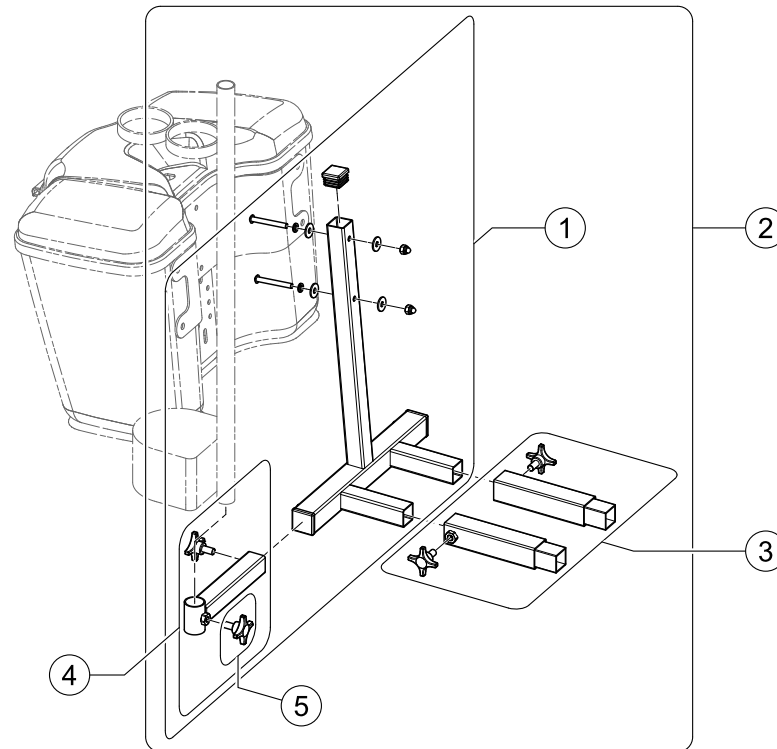

1	SP1552536	(Rear bracket for Storage box (Leo))	1
2	SP1552537	BRAKETT BAGASJEBOKS BAK COMET	1
3	SP1552594	FORLENGER FESTE KRYKKEH COMET	1
4	SP1553854	FESTE KRYKKEHOLDER COMET	1
5	5824117	STJERNEGREP FESTE NY MANØVERBOKS MOOVER	1


(Bumpers)

1640000 iso

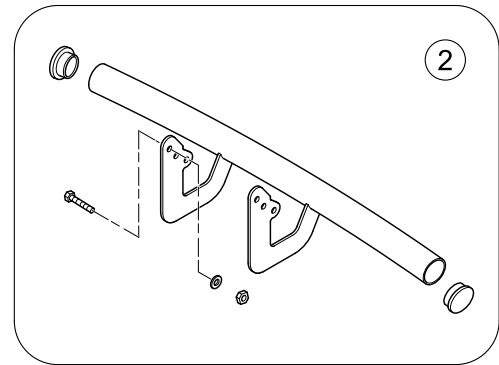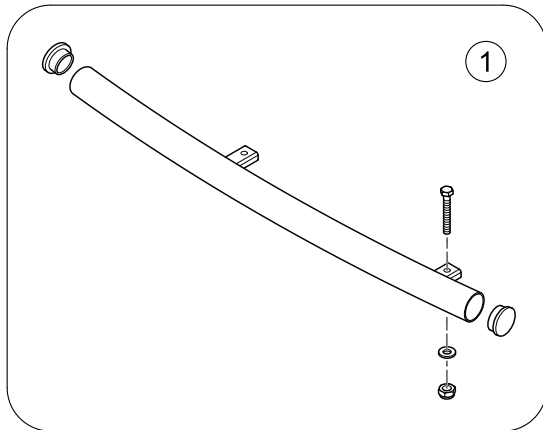

1	SP1638967	STØTFANGER BAK COMET ALPINE+	1
2	SP1528298	STØTFANGER FORAN COMET 4W	1

(Other Options)

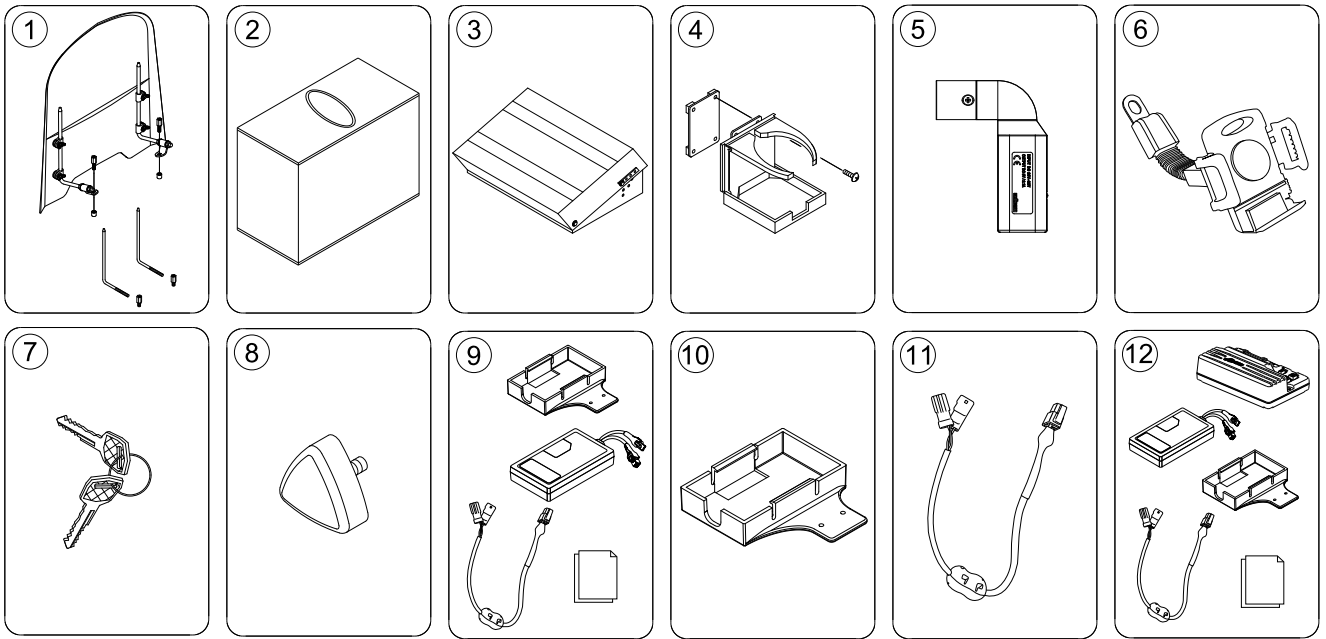

ISPS16/0382

1	SP1543160	VINDSKJERM COMET	1
2	SP1518240	PRESENNING COMET COMET	1
3	SP1637914	FOTSKAMMEL JUSTERBAR COMET ALPINE +	1
4	SP1521594	HOLDER KOPP COMET	1
5	SP1637917	USB LADEUTTAK FOR COMET COMET ALPINE+	1
6	SP1637912	HOLDER TELEFON COMET ALPINE+	1
7	SP1530211	NØKKEL 2 STK STANDARD COMET	1
8	SP1507005	TREKANTSKRUKNOTTCOMET ALPINE+	1
9	1663831	(Zeta Connectivity Kit) <i>Works with scooters that have a DS90 / DS120 / DS160 controller with firmware v1.24. This is indicated by a Connectivity Ready sticker on the inside of the tiller.</i>	1
10	1663838	(Mounting Bracket Zeta)	1
11	1663839	(Cable Loom Zeta) <i>Replacement for item in 1663831</i>	1
12	1663932	(Zeta DS120 retro fit kit) <i>For scooter with DS120 controller with firmware v1.23 and earlier (no Connectivity Ready sticker). Contains 1553399 and 1663831. Controller must be programmed for specific model after replacement.</i>	1
12	1663933	(Zeta DS160 retro fit kit) <i>For scooter with DS160 controller with firmware v1.23 and earlier (no Connectivity Ready sticker). Contains 1553400 and 1663831. Controller must be programmed for specific model after replacement.</i>	1



Yes, you can.®



Belgium & Luxemburg:

Invacare nv, Autobaan 22, B-8210 Loppem
Tel: (32) (0)50 83 10 10, Fax: (32) (0)50 83 10 11
marketingbelgium@invacare.com / www.invacare.be

Denmark:

Invacare A/S, Sdr. Ringvej 37, DK-2605 Brøndby
Tel: (45) (0)36 90 00 00, Fax: (45) (0)36 90 00 01
denmark@invacare.com / www.invacare.dk

Deutschland:

Invacare GmbH, Am Achener Hof 8, D-88316 Isny
Tel: (49) (0)7562 700 0
kontakt@invacare.com / www.invacare.de

EU Export:

Invacare Poirier SAS, Route de St Roch, F-37230 Fondettes
Tel: (33) (0)2 47 62 64 66
eu-export@invacare.com / www.invacare.eu.com

España:

Invacare SA, Avenida del Oeste Nº50 , 1ª 1ª, Valencia , CP: 46001
Tel: (34) (0)972 49 32 14
contactsp@invacare.com / www.invacare.es

France:

Invacare Poirier SAS, Route de St Roch, F-37230 Fondettes
Tel: (33) (0)2 47 62 64 66
contactfr@invacare.com / www.invacare.fr

Ireland:

Invacare Ireland Ltd, Unit 5 Seatown Business Campus, Seatown Road, Swords, County Dublin, EirCode: K67 K271 - Ireland
Tel: (353) 1 810 7084, Fax: (353) 1 810 7085
ordersireland@invacare.com / www.invacare.ie

Italia:

Invacare Mecc San s.r.l., Via dei Pini 62, I-36016 Thiene (VI)
Tel: (39) 0445 38 00 59
servizioclienti@invacare.com / www.invacare.it

Nederland:

Invacare B.V., Galvanistraat 14-3, 6716 AE Ede
Tel: (31) (0)318 695 757
nederland@invacare.com / www.invacare.nl

Norge:

Invacare AS, Brynsveien 15, 0667 Oslo, Postboks 6230, Etterstad, N-0603 Oslo
Tel: (47) (0)22 57 95 00, Fax: (47) (0)22 57 95 01
norway@invacare.com / www.invacare.no

Österreich:

Invacare Austria GmbH, Herzog Odilostrasse 101, A-5310 Mondsee
Tel: (43) 6232 5535 0, Fax: (43) 6232 5535 4
info-austria@invacare.com / www.invacare.at

Portugal:

Invacare Lda, Rua Estrada Velha, 949, P-4465-784 Leça do Balio
Tel: (+351) 225 193 360
marketing.pt@invacare.com / www.invacare.pt

Sverige & Suomi:

Invacare AB, Fagerstagatan 9, S-163 91 Spånga
Tel: (46) (0)8 761 70 90, Fax: (46) (0)8 761 81 08
sweden@invacare.com / www.invacare.se

Switzerland:

Invacare AG, Benkenstrasse 260, CH-4108 Witterswil
Tel: (41) (0)61 487 70 80, Fax: (41) (0)61 487 70 81
switzerland@invacare.com / www.invacare.ch

United Kingdom:

Invacare Limited, Unit 4, Pencoed Technology Park, Bridgend CF35 5AQ
Switchboard Tel: (44) (0)1656 776200
Customer services Tel: (44) (0)1656 776222
csmobilityuk@invacare.com
cslifestyleuk@invacare.com
uk@invacare.com / www.invacare.co.uk