# Rea<sup>®</sup> Azalea Base

en	<b>Tie down labels</b> Instruction sheet
fr	<b>Étiquettes de fixation</b> Fiche d'instruction
de	<b>Anhängeschilder</b> Merkblatt
it	<b>Etichette per l'ancoraggio</b> Foglio di istruzioni
es	<b>Fijar etiquetas</b> Hoja de instrucciones
pt	<b>Etiquetas de refreio</b> Folheto de instruções
sv	<b>Dekaler för transportfästen</b> Instruktionsblad
no	<b>Festemerking</b> Anvisningsark
nl	<b>Etiketten vastmaken</b> Instructie blad
da	<b>Fastsurringsmærkater</b> Instruktionsblad

Fig 1.1

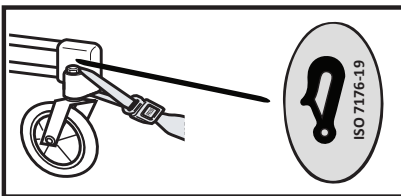
Fig 1.2

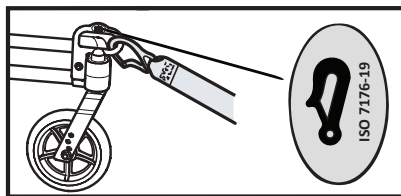
Fig 2


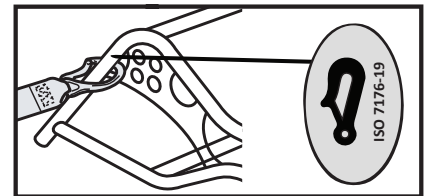
Fig 3

**rea**<sup>®</sup>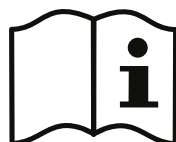

Yes, you can.<sup>®</sup>

**en**

The Rea®Azalea Base belongs to the Azalea family and serves as a wheelchair base designed to accommodate various seating systems, including custom anatomical seats produced by different manufacturers.

The Rea Azalea product family has undergone comprehensive testing as a complete product, with an Invacare seating system. However, it is essential to note that when the Rea Azalea is ordered without a specific seating system, it should not be regarded as a complete product. Full evaluation of safety only becomes possible when the base and the seating system are combined. The company responsible for installing the seating system will perform the final risk assessment. It is advisable to confirm the existence of a compatibility agreement between the seating system manufacturer and Invacare. Invacare cannot make predictions about the outcomes of accidents involving other configurations.

If the assessment considers the product as crash-safe and suitable for transportation in a vehicle, you can affix tie-down labels as depicted in Figures 1.1 or 1.2 (depending on your wheelchair's configuration) to both the front left and right sides, as well as Figure 2 for both the back left and right sides. These labels provide guidance on the placement of attachment straps when securing the chair for vehicle transport. In cases where the product is not considered crash-safe and unsuitable for vehicle transportation, it is advisable to attach the warning label (Figure 3) to the chassis.

For further vital information related to vehicle transportation, we strongly recommend a thorough reading of the user manual for the Azalea. If the user manual is not readily available, it can be obtained from your Invacare representative or downloaded from the Invacare website.

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