

# Spare Parts Catalogue

---

Version: 2014/01/17a NO





**Yes, you can.®**

## **General Information**

*For any order of spare parts the following information is required:*

- Product type*
- Product name & model*
- Serial Number*
- Part-Number*
- Part-Description*

*Exploded view drawings and code numbers are not contractual and subject to modification, in relation to Invacare® continuous product improvement policy to meet end users needs.*

*The reproduction or use of this work, or of any substantial part of this work, in any form or by any electronic, mechanical or other means, including photocopying, scanning and recording, and in any information storage and retrieval system, including electronic transmission, is forbidden without the prior written permission of Invacare® International Sarl.*



Yes, you can.<sup>®</sup>

## TABLE OF CONTENTS

---

### Informasjon

(User Manuals Bora/Spectra XTR) .....	11
(User Manuals REM550) .....	12
(User Manuals REM24SD) .....	13
(User Manuals REM A & REM B) .....	14
(User Manual Shark2) .....	15
(User Manuals IDC Dual Control) .....	16
(User Manuals REM41 Tray/Centre Control) .....	17
(User Manual Proximity Head Remote DSL-105) .....	18
(User Manuals ACS Handprogrammer) .....	19

### Chassis

(Frame) .....	20
(Battery Carrier) .....	21
(Suspension) .....	22
(Cover Lifter) .....	23
(Anti-Tipper) .....	24

### Sete

Sete .....	25
(Standard Seat) .....	26
(Tilt 20°) .....	27
(Tilt 28°) .....	28
(Lifter) .....	29
(Bellows) .....	30
(Seat Frame with Plate) .....	31
(Seat Frame Support) .....	32
(Seat Cushion) .....	33
(Flex2 Seat) .....	34
(Tilt 20°) .....	27
(Tilt 28°) .....	28
(Lifter) .....	29
(Bellows) .....	30
(Seat Frame with Plates) .....	35
(Seat Frame Support) .....	32
(Seat cushion) .....	36
(Firm Seat) .....	37
(Tilt 20°) .....	27
(Tilt 28°) .....	28
(Lifter) .....	29
(Bellows) .....	38
(Seat Frame) .....	39
(RUIK-Seat Frame) .....	40
(Seat Frame Support) .....	32
(Seat Cushion) .....	41
(Seat Board) .....	42
(Max Seat) .....	43
(Tilt 20°) .....	27
(Lifter) .....	29
(Bellows) .....	30



Yes, you can.®

## TABLE OF CONTENTS

---

(Seat Frame with Plates) .....	44
(Seat Frame Support) .....	32
(Seat Cushion Max) .....	45
(Modulite Seat) .....	46
(Tilt 20°) .....	27
(Tilt 30°) .....	47
(Lifter) .....	48
(One Piece Seat Plate) .....	49
(Telescopic Seat Frame and Plate) .....	50
(Telescopic Seat Frame for Tilt 12° / 20°) .....	51
(Telescopic Seat Frame for Tilt 30° / Lifter) .....	52
(Armrest bracket) .....	53
(Legrest Support) .....	54
(Seat Plate) .....	55
(Telescopic Seat Frame and Sling Seat) .....	56
(Telescopic Seat Frame for Tilt 12° / 20°) .....	57
(Telescopic Seat Frame for Tilt 30° / Lifter) .....	58
(Armrest bracket) .....	53
(Legrest Support) .....	59
(Sling Seat) .....	60
(Seat frame support) .....	61
(Seat Cushion) .....	62
(Leckey Seat (only UK)) .....	65
(Tilt 20°) .....	27
(Seat Frame Support) .....	32

## Rygg

Rygg .....	66
(Backrest for Standard Seat) .....	67
(Backrest Standard Seat, quick release) .....	68
(Backrest Standard Seat, settable) .....	69
(Backrest Standard Seat, Powered Recliner) .....	70
(Backrest for Flex2 Seat) .....	71
(Backrest Recliner) .....	72
(Backrest Flex2) .....	73
(Backrest frame Flex2, tension adjustable) .....	74
(Push handles Flex2) .....	75
(Push handles Flex2, attendant control) .....	76
(Backrest cushion) .....	77
(Tension adjustable) .....	78
(Backrest for Firm Seat) .....	79
(Backrest Recliner, manual, Padded Back) .....	80
(Backrest Recliner, powered, Padded Back) .....	81
(Backposts for Padded Back, manual) .....	82
(Backposts for Padded Back, powered) .....	83
(Flangeless Backpost) .....	84
(Backposts for tension adjustable) .....	85
(Padded Back) .....	86
(Tension adjustable) .....	87
(Backrest for Max Seat) .....	88
(Backrest Recliner) .....	89
(Backrest Max-Seat) .....	90





Yes, you can.®

## TABLE OF CONTENTS

---

(Backrest for Modulite Seat) .....	91
(Support for Backrest Frame) .....	92
(Backrest Frame) .....	93
(Tension adjustable) .....	94
(Backrest support) .....	95
(Backposts) .....	96
(Connection tube) .....	97
(Back Recliner / Stabilization bracket) .....	98
(Backrest Cushion) .....	99
(Flex3 adjustable Backrest Plate) .....	101
(Push handle for Leckey Seat (UK only)) .....	102

### Benstøtter og holdere

(Legrest Support, 80° & man. elevating, Standard-seat) .....	103
(Legrest Support, 80° & man. elevating, Flex2-/Max-Seat) .....	104
(Legrest Support, 80° & man. elevating, Firm-Seat) .....	105
(Legrest Support, 80° & man. elevating, Modulite Seat) .....	106
(Legrest Support, Vari series, Standard-Seat) .....	107
(Legrest Support, Vari series, Flex2-/Max-Seat) .....	108
(Legrest Support, Vari series, Firm-Seat) .....	109
(Legrest Support, Vari series, Modulite-Seat) .....	110
(Legrest Support, 70°/90°, Firm-Seat) .....	111
(Legrest Support, angle adj., Firm-Seat) .....	112
(Hanger Standard 80°) .....	113
(Hanger manually height adjustable) .....	114
(Vari F Hanger RH/LH) .....	115
(Vari A Hanger RH/LH 07.2005 ==>) .....	116
(ADE Hanger RH/LH Electr. 07.2005 ==>) .....	117
(Hanger 70 Deg, Standard) .....	118
(Hanger 70°, Firm-Seat) .....	119
(Hanger 90°, Firm-Seat) .....	120
(Hanger Angle, Adjustment) .....	121

### Fotplate / Innerrør

(Legrest 80°) .....	122
(Footplate, depth-, angleadjustable, 80°) .....	123
(Footrest, fixed, 70°) .....	124
(Footrest, angleadjustable, 70°) .....	125
(Footrest, depth-, angleadjustable, 70°) .....	126
(Footplate with support tube angle adjust. RH/LH) .....	127
(Footrests 80° & man. elevating) .....	128
(Footrests Vari series) .....	129
(Flip-up footrests Vari series 01.08.10 ==>) .....	130
(Angle adjustable footrests Vari series 01.08.10 ==>) .....	131

### Sidegring

Sidegring .....	132
(Armrest for Standard Seat) .....	133
(T-Armrest, detachable, with Armpad, for Standard Seat 2008-08-01 ==>) .....	134
(Armrest for Flex2 Seat) .....	135
(Armrest long) .....	136



Yes, you can.®

## TABLE OF CONTENTS

---

(Armrest Hemiplegic) .....	137
(Armrest Azalea style) .....	138
(Armrest RIO, Flex II) .....	139
(T-Armrest detachable) .....	140
(Flip-up armrest) .....	141
(Mounting bracket Flip-up armrests) .....	142
(Skirt guard Flip-up armrest) .....	143
(Armrest for Firm Seat) .....	144
(Armrest, padded) .....	145
(Armrest, moulded) .....	146
(Armrest Ergonomic) .....	147
(Armrest for Max Seat) .....	148
(Armrest long) .....	149
(Armrest Hemiplegic) .....	137
(T-Armrest detachable) .....	140
(Armrest for Modulite Seat) .....	150
(T-Armrest with short/long Armpad) .....	151
(T-Armrest with padded Armpad) .....	152

### Sving-og Drivhjul

(Frontwheels 200x45) .....	153
(Frontwheels 230x60) .....	154

### (Drive Wheels)

(Drive Wheels) .....	155
----------------------	-----

### Drivenhet

(Motor =>31.07.2011) .....	156
(SSD-Motors) .....	157
(Swingarm SSD-Motor 18.04.2011=>) .....	159
(Hubs) .....	160

### Bremser

(Holdingbrake) .....	161
----------------------	-----

### Elektronikk

Elektronikk .....	162
(ACS Electronic) .....	163
(ACS Electronic Modules) .....	164
(Powermodules) .....	165
(Holder for Electronic ACS 2) .....	166
(ACT 2 for ACS 1, with Holder) .....	167
(ACT-Brackets) .....	168
(Lightboard) .....	169
(Lightboardholder Shark/ACS2) .....	170
(G-Trac-sensor) .....	171
(Holder for Gyro) .....	172
(ACS Remote & Brackets) .....	173
(Remote REM 24 SD with Brackets) .....	174
(Remote Bracket Standard) .....	175



Yes, you can.<sup>®</sup>

## TABLE OF CONTENTS

---

(Remote Bracket Swing away & height adjust.)	176
(Remote Bracket Attendant for Standard-/Flex2-/Max-Seat)	177
(Remote Bracket Attendant long for Firm Seat)	178
(Remote Bracket Attendant short for Firm Seat)	179
(Remote REM A & B with Brackets)	180
(Remote Bracket Standard)	175
(Remote Bracket Swing away & height adjust.)	181
(Remote Bracket Attendant for Standard-/Flex2-/Max-Seat)	177
(Remote Bracket Attendant long for Firm Seat)	178
(Remote Bracket Attendant short for Firm Seat)	182
(Remote REM 550 with Brackets)	183
(Remote Bracket Standard)	175
(Remote Bracket Swing away & height adjust.)	176
(Remote Bracket Attendant for Standard-/Flex2-/Max-Seat)	177
(Remote Bracket Attendant long for Firm Seat)	178
(Remote Bracket Attendant short for Firm Seat)	179
(Remote REM 34 with Brackets)	184
(Remote Bracket Standard)	185
(Remote Bracket Swing away & height adjust.)	186
(Remote G 90 A with Brackets)	187
(Remote Bracket Standard)	188
(Remote Bracket Swing away & height adjust.)	189
(Remote Bracket Attendant for Standard-/Flex2-/Max-Seat)	190
(Remote Bracket Attendant long for Firm Seat)	191
(Remote Bracket Attendant short for Firm Seat)	192
(Remote G91S with Brackets)	193
(Remote Bracket Standard)	194
(Centre control ACS REM41)	195
(Remote Small Chin Control with Brackets)	196
(Small Chincontrol)	197
(Framework for chincontrol)	198
(Remote Dual Control with Brackets)	199
(Remote Bracket Dual Control for Standard-/Flex2-/Max-Seat)	200
(Remote Bracket Dual Control for Modulite Seat)	201
(Remote Bracket Dual Control long for Firm-Seat)	202
(Remote Bracket Attendant short for Firm Seat)	182
(Remote IDC Dual Control with Brackets)	203
(ACS Cables & Accessories)	204
(ACS Buscable)	205
(T-Adapter ACS 2)	206
(Slow-Stop-Sensor & Holder)	207
(Programming Kits)	208
(Magnetkey for Remotes)	209
(Shark Electronic)	210
(Shark Electronic Modules)	211
(Powermodules)	165
(Lightboard)	169
(Lightboardholder Shark/ACS2)	170
(Shark Remote & Brackets)	212
(Remote Shark with Brackets)	213
(Remote Bracket Standard)	175
(Remote Bracket Swing away & height adjust.)	181



Yes, you can.®

## TABLE OF CONTENTS

---

(Remote Bracket Attendant for Standard-/Flex2-/Max-Seat)	177
(Remote Bracket Attendant long for Firm Seat)	178
(Remote Bracket Attendant short for Firm Seat)	182
(Remote Dual Control with Brackets)	214
(Remote Bracket Dual Control for Standard-/Flex2-/Max-Seat)	215
(Remote Bracket Dual Control for Modulite Seat)	216
(Remote Bracket Dual Control long for Firm-Seat)	202
(Remote Bracket Attendant short for Firm Seat)	182
(Shark Cables & Accessories)	217
(Buscable Shark)	218
(Programming Kits)	208
(R-Net Electronic)	219
(R-NET Electronic Modules)	220
(Powermodule R-Net)	221
(Powermodules)	165
(Halter R-Net)	222
(Holder ISM)	223
(Lightboard)	169
(Lightboardholder R-Net)	224
(R-NET Remote & Brackets)	225
(Remote R-Net LED JSMs with Brackets)	226
(Remote Bracket Standard)	227
(Remote Bracket Swing away & height adjust.)	228
(Remote Bracket Attendant for Standard-/Flex2-/Max-Seat)	229
(Remote Bracket Attendant long for Firm Seat)	230
(Remote Bracket Attendant short for Firm Seat)	231
(Centre control)	232
(Chincontrol R-Net)	233
(Remote Dual Control with Brackets)	234
(Remote Bracket Dual Control for Standard-/Flex2-/Seat)	235
(Remote Bracket Dual Control for Modulite Seat)	236
(Remote Bracket Dual Control long for Firm Seat)	237
(Remote Bracket Dual Control short for Firm Seat)	238
(R-NET Cables & Accessories)	239
(Buscable R-net)	240
(T-Adapter R-NET)	241
(Slow-Stop-Sensor & Holder)	207
(VR2 Electronic)	242
(VR2 Electronic Modules)	243
(Powermodules)	165
(Lightboard)	169
(VR2 Remote & Brackets)	244
(Remote VR2 with Brackets)	245
(Remote Bracket Standard)	227
(Remote Bracket Swing away & height adjust.)	228
(Remote Bracket Attendant for Standard-/Flex2-/Max-Seat)	229
(Remote Bracket Attendant long for Firm Seat)	230
(Remote Bracket Attendant short for Firm Seat)	231
(Remote Dual Control with Brackets)	246
(Remote Bracket Dual Control for Standard-/Flex2-/Seat)	235
(Remote Bracket Dual Control for Modulite Seat)	236
(Remote Bracket Dual Control long for Firm Seat)	237



Yes, you can.®

## TABLE OF CONTENTS

---

(Remote Bracket Dual Control short for Firm Seat) .....	238
---------------------------------------------------------	-----

### kabel og lader

(ACS Buscable) .....	205
(Battery Boxes) .....	247
(Batteries) .....	248
(Charger) .....	249
(Cables) .....	250
(Buscable R-Net) .....	251
(Battery cable & Plug) .....	252

### Sikkerhedsudstyr

(Light System (German homologation)) .....	253
(LED Light System) .....	254
(Reflective Sign) .....	255
(Kerb Rider) .....	256
(Kerb Rider (with parking position)) .....	257
(Lap Belt) .....	258
(Lap Belts (Modulite)) .....	259
(Rearview Mirror) .....	260

### Sete utstyr

(Trunk support Flex-Seat) .....	261
(Trunk support, multi adjustable) .....	262
(Trunk support Firm seat) .....	263
(Trunk support bracket (Modulite)) .....	264
(Headrest Curve 1, Firm-Seat) .....	265
(Headrest sidewise adjustable) .....	266
(Head- / Neckrest) .....	267
(Abduction Cushion) .....	268
(Headrest with cheek support) .....	269

### Annet Utstyr & Tilbehør

(Luggage Rack) .....	270
(Mounting Kit Luggage Rack) .....	271
(Cane holder (Std., Flex2, Firm, Max Seat)) .....	272
(Cane holder (Modulite: backrest frame)) .....	273
(Cane holder (Modulite: back posts man recliner)) .....	274
(Cane holder (Modulite: back posts pow recliner)) .....	275
(Fender for Drive Wheel) .....	276
(Tray) .....	277
(Hour Counter) .....	278
(KLICKfix® bag) .....	279
(KLICKfix® bracket) .....	280
(Toolkit) .....	281
(T-Nuts for Accessories Modulite Seat) .....	282



**(User Manuals Bora/Spectra XTR)**

CS ČESKY	FR FRANÇAIS	PL POLSKI
DA DANSK	HR HRVATSKI	PT PORTUGUÊS
DE DEUTSCH	HU MAGYAR	SK SLOVENČINA
EN ENGLISH	IT ITALIANO	SL SLOVENŠČINA
ES ESPAÑOL	LT LIETUVIŲ KALBA	SV SVENSKA
ET EESTI KEEL	NL NEDERLANDS	RO ROMÂNĂ
FI SUOMEN KIELI	NO NORSK	JA 日本語
BG БЪЛГАРСКИ	RU РУССКИЙ ЯЗЫК	LV LATVIAN
	HE עברית	

---

EN	SP1528224	(User Manual Bora / Spectra XTR English)	1
FR	SP1528225	(User Manual Bora / Spectra XTR French)	1
SV	SP1528226	(User Manual Bora / Spectra XTR Swedish)	1
PT	SP1528227	(User Manual Bora / Spectra XTR Portuguese)	1
IT	SP1528331	(User Manual Bora / Spectra XTR Italian)	1
FI	SP1529717	(User Manual Bora / Spectra XTR Finnish)	1
DA	SP1529719	(User Manual Bora / Spectra XTR Dansk)	1
NL	SP1529720	(User Manual Bora / Spectra XTR Dutch)	1
ES	SP1529947	(User Manual Bora / Spectra XTR Spanish)	1
CZ	SP1531142	(User Manual Bora / Spectra XTR Czech)	1
PL	SP1531143	(User Manual Bora / Spectra XTR Polish)	1
SL	SP1531144	(User Manual Bora / Spectra XTR Slovenian)	1
HR	SP1531145	(User Manual Bora / Spectra XTR Croatian)	1
LT	SP1531146	(User Manual Bora / Spectra XTR Lithuanian)	1
RU	SP1549079	(User Manual Bora / Spectra XTR Russian)	1
NO	SP1551034	BRUKSANVISNING BORA MANUAL NORSK	1
HU	SP1573154	(User Manual Bora / Spectra XTR Hungarian)	1

## (User Manuals REM550)

CS ČESKY	FR FRANÇAIS	PL POLSKI
DA DANSK	HR HRVATSKI	PT PORTUGUÊS
DE DEUTSCH	HU MAGYAR	SK SLOVENČINA
EN ENGLISH	IT ITALIANO	SL SLOVENŠČINA
ES ESPAÑOL	LT LIETUVIŲ KALBA	SV SVENSKA
ET EESTI KEEL	NL NEDERLANDS	RO ROMÂNĂ
FI SUOMEN KIELI	NO NORSK	JA 日本語
BG БЪЛГАРСКИ	RU РУССКИЙ ЯЗЫК	LV LATVIAN
	HE עברית	

---

DA	1528088	(User Manual REM550 Danish)	1
DE	1528089	(User Manual REM550 German)	1
EN	1528090	BRUKSANVISNING DX-REM550 MANUAL	1
ES	1528091	(User Manual REM550 Spanish)	1
FI	1528092	(User Manual REM550 Finsk)	1
FR	1528093	(User Manual REM550 French)	1
IT	1528094	(User Manual REM550 Italiano)	1
NL	1528095	(User Manual REM550 Dutch)	1
NO	1528096	BRUKSANVISNING DX REM550 MANUAL NORSK	1
PT	1528097	(User Manual REM550 Portuguese)	1
SV	1528098	(User Manual REM550 Swedish)	1
HR	SP1533279	(User Manual REM550 Croatian)	1
CS	SP1533280	(User Manual REM551 Czech)	1
ET	SP1533281	(User Manual REM552 Estonian)	1
HU	SP1533282	(User Manual REM553 Hungarian)	1
LT	SP1533283	(User Manual REM554 Lithuanian)	1
PL	SP1533284	(User Manual REM555 Polish)	1
SL	SP1533285	(User Manual REM556 Slovenian)	1



CS ČESKY	FR FRANÇAIS	PL POLSKI
DA DANSK	HR HRVATSKI	PT PORTUGUÊS
DE DEUTSCH	HU MAGYAR	SK SLOVENČINA
EN ENGLISH	IT ITALIANO	SL SLOVENŠČINA
ES ESPAÑOL	LT LIETUVIŲ KALBA	SV SVENSKA
ET EESTI KEEL	NL NEDERLANDS	RO ROMÂNĂ
FI SUOMEN KIELI	NO NORSK	JA 日本語
BG БЪЛГАРСКИ	RU РУССКИЙ ЯЗЫК	LV LATVIAN
	HE עברית	

---

DE	1436142	(User Manual REM24SD German)	1
FR	1436143	(User Manual REM24SD French)	1
EN	1436144	(User Manual REM24SD English)	1
SV	1436145	(User Manual REM24SD Swedish)	1
DA	1436146	(User Manual REM24SD Danish)	1
NO	1436147	(User Manual REM24SD Norwegian)	1
NL	1436148	(User Manual REM24SD Dutch)	1
ES	1436149	(User Manual REM24SD Spanish)	1
PT	1436150	(User Manual REM24SD Portuguese)	1
IT	1436385	(User Manual REM24SD Italian)	1
FI	1436386	(User Manual REM24SD Finnish)	1
HR	SP1533251	(User Manual REM24SD Croatian)	1
CS	SP1533252	(User Manual REM24SD Czech)	1
ET	SP1533253	(User Manual REM24SD Estonian)	1
HU	SP1533254	(User Manual REM24SD Hungarian)	1
LT	SP1533255	(User Manual REM24SD Lithuanian)	1
PL	SP1533256	(User Manual REM24SD Polish)	1
SL	SP1533257	(User Manual REM24SD Slovenian)	1

## (User Manuals REM A & REM B)

CS ČESKY	FR FRANÇAIS	PL POLSKI
DA DANSK	HR HRVATSKI	PT PORTUGUÊS
DE DEUTSCH	HU MAGYAR	SK SLOVENČINA
EN ENGLISH	IT ITALIANO	SL SLOVENŠČINA
ES ESPAÑOL	LT LIETUVIŲ KALBA	SV SVENSKA
ET EESTI KEEL	NL NEDERLANDS	RO ROMÂNĂ
FI SUOMEN KIELI	NO NORSK	JA 日本語
BG БЪЛГАРСКИ	RU РУССКИЙ ЯЗЫК	LV LATVIAN
	HE עברית	

---

DA	1529171	(User Manual REM A & B Dansk)	1
DE	1529172	(User Manual REM A & B German)	1
EN	1529173	(User Manual REM A & B English)	1
ES	1529174	(User Manual REM A & B Spanish)	1
FI	1529228	(User Manual REM A & B Finsk)	1
FR	1529229	(User Manual REM A & B French)	1
IT	1529230	(User Manual REM A & B Italian)	1
NL	1529231	(User Manual REM A & B Dutch)	1
NO	1529232	(User Manual REM A & B Norsk)	1
PT	1529233	(User Manual REM A & B Portuguese)	1
SV	1529234	(User Manual REM A & B Swedish)	1
HR	SP1533258	(User Manual REM A & B Croatian)	1
CS	SP1533259	(User Manual REM A & B Czech)	1
ET	SP1533265	(User Manual REM A & B Estonian)	1
HU	SP1533266	(User Manual REM A & B Hungarian)	1
LT	SP1533267	(User Manual REM A & B Lithuanian)	1
PL	SP1533268	(User Manual REM A & B Polish)	1
SL	SP1533269	(User Manual REM A & B Slovenian)	1
RO	SP1533270	(User Manual REM A & B Romanian)	1

CS ČESKY	FR FRANÇAIS	PL POLSKI
DA DANSK	HR HRVATSKI	PT PORTUGUÊS
DE DEUTSCH	HU MAGYAR	SK SLOVENČINA
EN ENGLISH	IT ITALIANO	SL SLOVENŠČINA
ES ESPAÑOL	LT LIETUVIŲ KALBA	SV SVENSKA
ET EESTI KEEL	NL NEDERLANDS	RO ROMÂNĂ
FI SUOMEN KIELI	NO NORSK	JA 日本語
BG БЪЛГАРСКИ	RU РУССКИЙ ЯЗЫК	LV LATVIAN
	HE עברית	

DE	1451346	(User Manual Shark II German)	1
FR	1451347	(User Manual Shark II French)	1
SV	1451348	(User Manual Shark II Swedish)	1
DA	1451349	(User Manual Shark II Danish)	1
NO	1451350	(User Manual Shark II Norwegian)	1
NL	1451351	(User Manual Shark II Dutch)	1
ES	1451352	(User Manual Shark II Spanish)	1
PT	1451353	(User Manual Shark II Portuguese)	1
IT	1451354	(User Manual Shark II Italian)	1
FI	1451355	(User Manual Shark II Finnish)	1
EN	1451356	(User Manual Shark II English)	1
SL	1451584	(User Manual Shark II Slovenian)	1
CS	SP1533297	(User Manual Shark2 CS)	1
ET	SP1533298	(User Manual Shark2 EE)	1
HU	SP1533299	(User Manual Shark2 HU)	1
LT	SP1533300	(User Manual Shark2 LT)	1
PL	SP1533301	(User Manual Shark2 PL)	1
BG	SP1546061	(User Manual Shark II Bulgarian)	1
RU	SP1570186	(User Manual Shark II Russian)	1
HR	SP1574028	(User Manual Shark II Croatian)	1
LV	SP1576075	(User Manual Shark II Latvian)	1

## (User Manuals IDC Dual Control)

CS ČESKY	FR FRANÇAIS	PL POLSKI
DA DANSK	HR HRVATSKI	PT PORTUGUÊS
DE DEUTSCH	HU MAGYAR	SK SLOVENČINA
EN ENGLISH	IT ITALIANO	SL SLOVENŠČINA
ES ESPAÑOL	LT LIETUVIŲ KALBA	SV SVENSKA
ET EESTI KEEL	NL NEDERLANDS	RO ROMÂNĂ
FI SUOMEN KIELI	NO NORSK	JA 日本語
BG БЪЛГАРСКИ	RU РУССКИЙ ЯЗЫК	LV LATVIAN
	HE עברית	

---

DE	1521480	(User Manual Dual-Control German)	1
EN	1521481	(User Manual Dual-Control English)	1
FR	1521482	(User Manual Dual-Control French)	1
NL	1521483	(User Manual Dual-Control Dutch)	1
NO	1521484	(User Manual Dual-Control Norwegian)	1
SV	1521485	(User Manual Dual-Control Swedish)	1
DA	1521486	(User Manual Dual-Control Danish)	1
IT	1522252	(User Manual Dual-Control Italian)	1
PT	1522254	(User Manual Dual-Control Portuguese)	1
ES	1522255	(User Manual Dual-Control Spanish)	1

**(User Manuals REM41 Tray/Centre Control)**

CS ČESKY	FR FRANÇAIS	PL POLSKI
DA DANSK	HR HRVATSKI	PT PORTUGUÊS
DE DEUTSCH	HU MAGYAR	SK SLOVENČINA
EN ENGLISH	IT ITALIANO	SL SLOVENŠČINA
ES ESPAÑOL	LT LIETUVIŲ KALBA	SV SVENSKA
ET EESTI KEEL	NL NEDERLANDS	RO ROMÂNĂ
FI SUOMEN KIELI	NO NORSK	JA 日本語
BG БЪЛГАРСКИ	RU РУССКИЙ ЯЗЫК	LV LATVIAN
	HE עברית	

---

DA	SP1538022	(User Manual REM41 Tray Control Danish)	1
EN	SP1538023	(User Manual REM41 Tray Control English)	1
ES	SP1538024	(User Manual REM41 Tray Control Spanish)	1
FR	SP1538025	(User Manual REM41 Tray Control French)	1
IT	SP1538026	(User Manual REM41 Tray Control Italian)	1
NL	SP1538027	(User Manual REM41 Tray Control Dutch)	1
NO	SP1538028	(User Manual REM41 Tray Control Norwegian)	1
PT	SP1538029	(User Manual REM41 Tray Control Portuguese)	1
SV	SP1538030	(User Manual REM41 Tray Control Swedish)	1
CS	SP1538031	(User Manual REM41 Tray Control Czech)	1
SL	SP1538032	(User Manual REM41 Tray Control Slovenian)	1
DE	SP1541402	(User Manual REM41 Tray Control German)	1

## (User Manual Proximity Head Remote DSL-105)

CS ČESKY	FR FRANÇAIS	PL POLSKI
DA DANSK	HR HRVATSKI	PT PORTUGUÊS
DE DEUTSCH	HU MAGYAR	SK SLOVENČINA
EN ENGLISH	IT ITALIANO	SL SLOVENŠČINA
ES ESPAÑOL	LT LIETUVIŲ KALBA	SV SVENSKA
ET EESTI KEEL	NL NEDERLANDS	RO ROMÂNĂ
FI SUOMEN KIELI	NO NORSK	JA 日本語
BG БЪЛГАРСКИ	RU РУССКИЙ ЯЗЫК	LV LATVIAN
	HE עברית	

---

EN	SP1555731	(User Manual Proximity Head Remote DSL-105 English)	1
DE	SP1555732	(User Manual Proximity Head Remote DSL-105 German)	1
FR	SP1555733	(User Manual Proximity Head Remote DSL-105 French)	1
DA	SP1555839	(User Manual Proximity Head Remote DSL-105 Danish)	1
ES	SP1555840	(User Manual Proximity Head Remote DSL-105 Spanish)	1
FI	SP1555841	(User Manual Proximity Head Remote DSL-105 Finnish)	1
IT	SP1555842	(User Manual Proximity Head Remote DSL-105 Italian)	1
NL	SP1555843	(User Manual Proximity Head Remote DSL-105 Dutch)	1
NO	SP1555844	(User Manual Proximity Head Remote DSL-105 Norwegian)	1
PT	SP1555845	(User Manual Proximity Head Remote DSL-105 Portuguese)	1
SV	SP1555846	(User Manual Proximity Head Remote DSL-105 Swedish)	1

**(User Manuals ACS Handprogrammer)**

CS ČESKY	FR FRANÇAIS	PL POLSKI
DA DANSK	HR HRVATSKI	PT PORTUGUÊS
DE DEUTSCH	HU MAGYAR	SK SLOVENČINA
EN ENGLISH	IT ITALIANO	SL SLOVENŠČINA
ES ESPAÑOL	LT LIETUVIŲ KALBA	SV SVENSKA
ET EESTI KEEL	NL NEDERLANDS	RO ROMÂNĂ
FI SUOMEN KIELI	NO NORSK	JA 日本語
BG БЪЛГАРСКИ	RU РУССКИЙ ЯЗЫК	LV LATVIAN
	HE עברית	


---

1	1419316	(User Manual Handheld Programmer German)	1
1	1419317	(User Manual Handheld Programmer English)	1



# (Frame)


1526532PAC.dwg

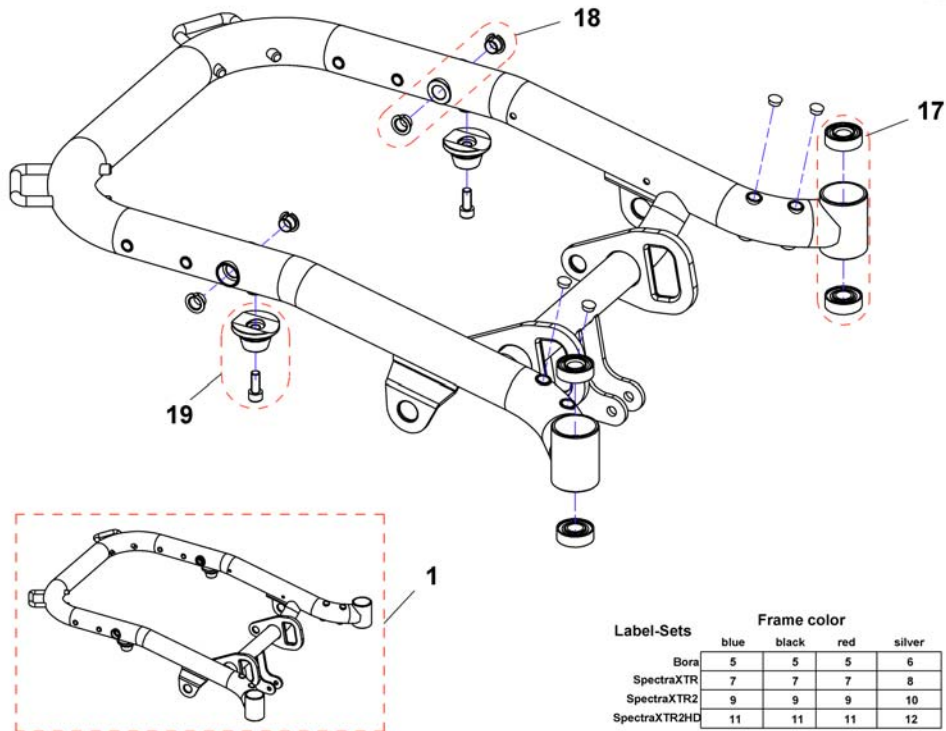

Label-Sets	Frame color			
	blue	black	red	silver
Bora	5	5	5	6
SpectraXTR	7	7	7	8
SpectraXTR2	9	9	9	10
SpectraXTR2HD	11	11	11	12

-	SP1552601	(Touch-up paint, Transparent Blue)	1
-	SP1553899	(Touch-up paint, black)	1
-	SP1552622	(Touch-up paint, Electric Red)	1
-	SP1552602	(Touch-up paint, Silver Metallic)	1
1	SP1526531	RAMME.BLA BORA	1
1	SP1526532	(Frame cpl., Black)	1
1	SP1526553	(Frame cpl., Electric Red)	1
1	SP1526554	(Frame cpl., Silver Metallic)	1
5	SP1528326	(Labelset Bora, white)	1
6	SP1529003	(Labelset Bora, black)	1
7	SP1528325	(Labelset Spectra XTR, white)	1
8	SP1529002	(Labelset Spectra XTR, black)	1
9	SP1550357	(Labelset Spectra XTR2, white)	1
10	SP1550356	(Labelset Spectra XTR2, black)	1
11	SP1552765	(Labelset Spectra XTR2 HD, white)	1
12	SP1552710	(Labelset Spectra XTR2 HD, black)	1
17	SP1528342	KULELAGERSETT SVINGHJUL BORA	1
18	SP1528343	LAGER RAMMERØR BORA	1
19	SP1528344	FJÆR RETAINER BORA	1



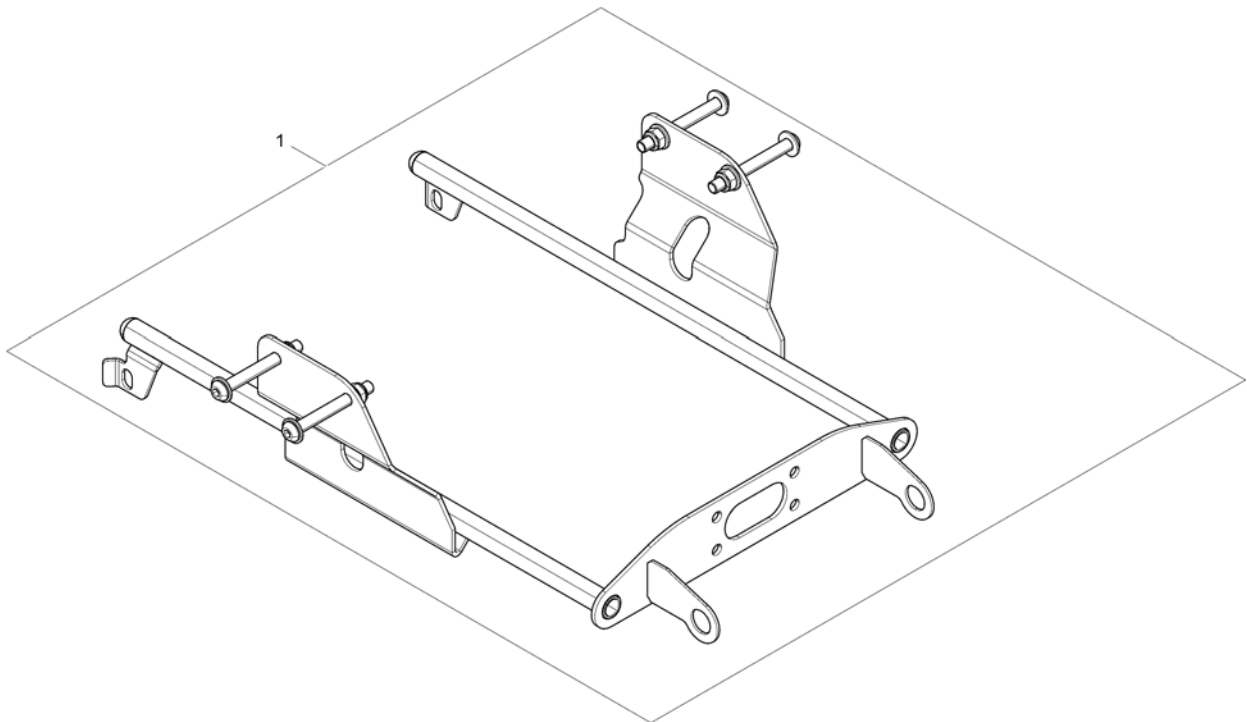


1 SP1526555

BATTERIBÆRER BORA


1

1527171X.svgz

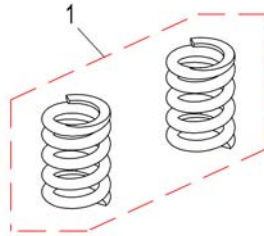

---

1	SP1531237	FJÆR HARD BORA	1
1	SP1527171	FJÆR NORMAL PAR BORA	1

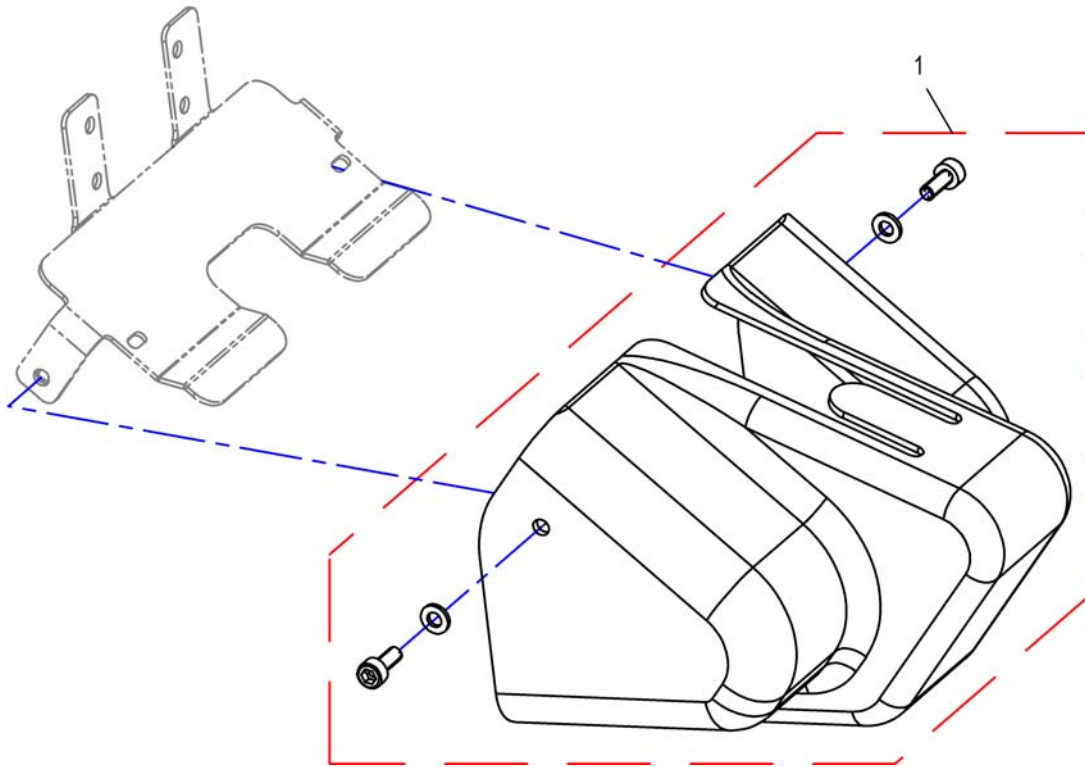


1 SP1530596

(Cover Lifter)

1

Invacare® Bora | Chassis  
**(Anti-Tipper)**

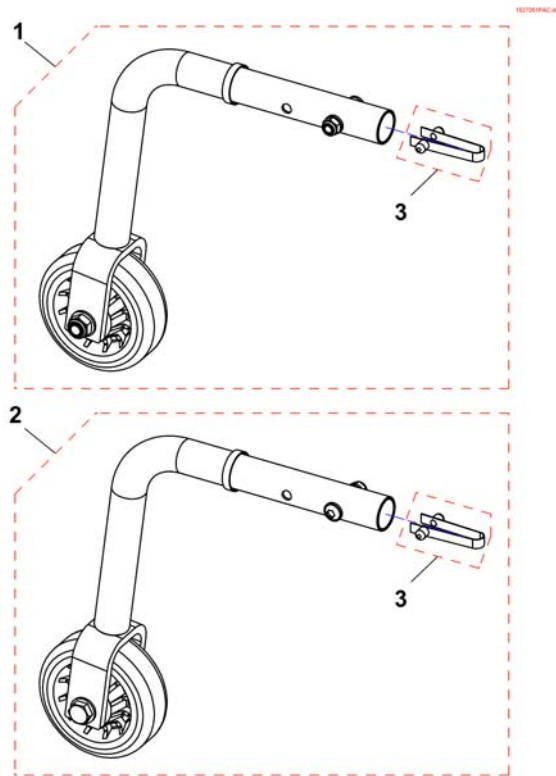

---

1	SP1533693	(Anti-Tipp-Wheel LH kompl.)	1
2	SP1533692	(Anti-Tipp-Wheel RH kompl.)	1
3	500023	LÄSFJÄDER, G63E	1

SFCBORA25 no





---


1	-	(Standard Seat)	1
2	-	(Flex2 Seat)	1
3	-	(Firm Seat)	1
4	-	(Max Seat)	1
5	-	(Modulite Seat)	1
6	-	(Leckey Seat (only UK))	1

Invacare® Bora | Sete  
**(Standard Seat)**

1571699.iss

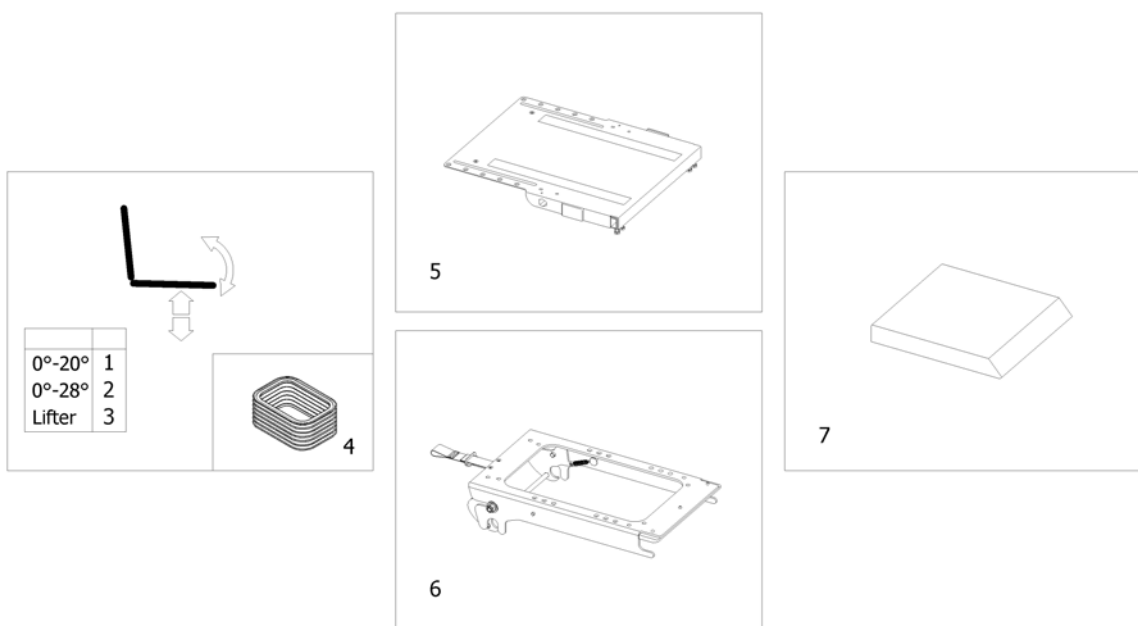


1	-	(Tilt 20°) [1002/1014]	1
2	-	(Tilt 28°) [1043]	1
3	-	(Lifter) [1021]	1
4	-	(Bellows) [1311]	1
5	-	(Seat Frame with Plate)	1
6	-	(Seat Frame Support)	1
7	-	(Seat Cushion) [1336/1901]	1

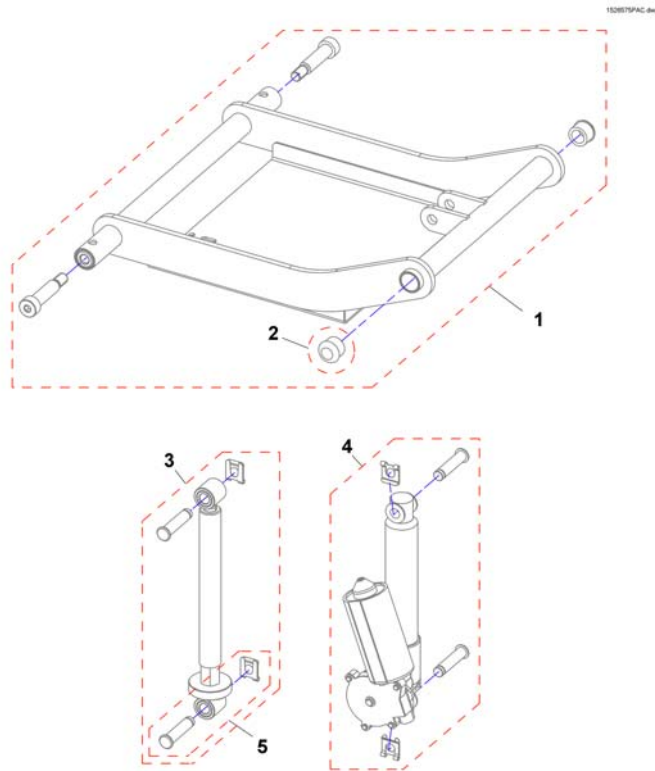

1	SP1526574	SETERAMMEADAPTER BORA		1
2	L01008	(Tube End Plug SI 22)		1
3	SP1526576	(Manual Adjustment, cpl.)		1
4	SP1526575	TILTMOTOR BORA	→ 30.09.2012	1
4	SP1560749	(Tilt actuator ACS, cpl.)	01.10.2012 →	1
4	SP1528265	(Tilt actuator Shark, cpl.)	→ 30.09.2012	1
4	SP1560774	(Tilt actuator Shark, cpl.)	01.10.2012 →	1
4	SP1527175	(Tilt actuator VR2 / R-Net, cpl.)	→ 30.09.2012	1
4	SP1560775	(Tilt actuator VR2 / R-Net, cpl.)	01.10.2012 →	1
5	SP1528345	(Mounting-Kit for actuators)		1


**(Tilt 28°)**

1545645.dwg

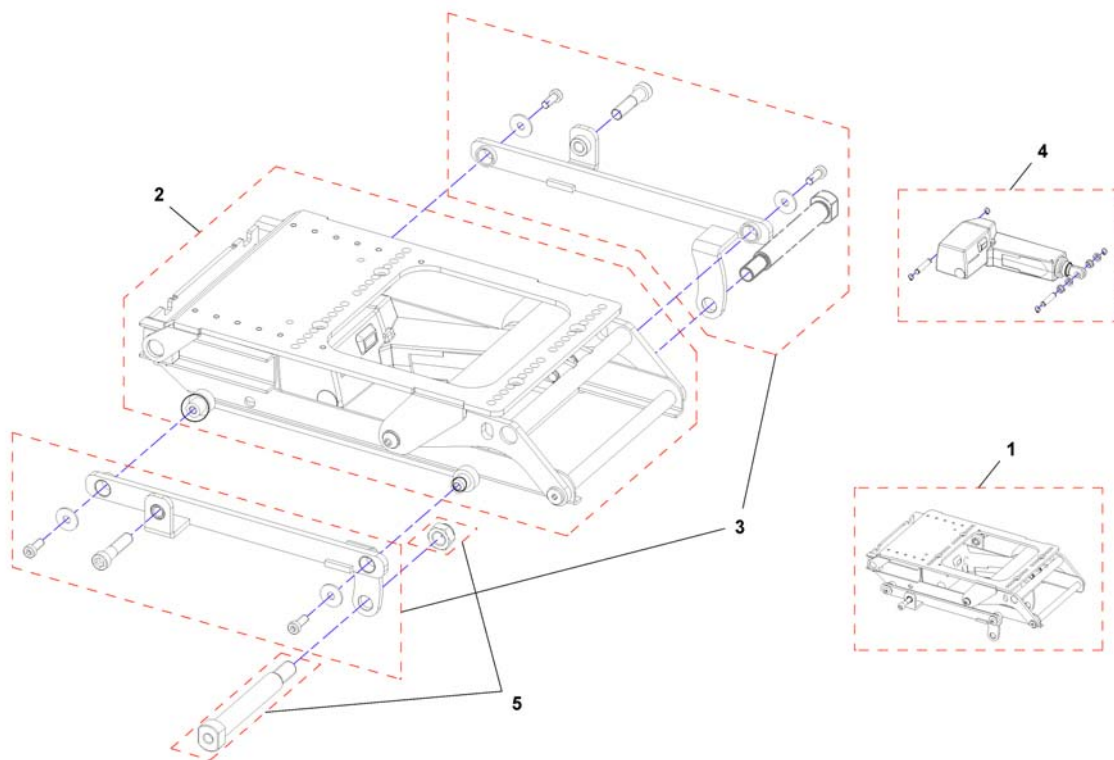

1	SP1545645	(Seat tilt cpl.)	1
2	SP1533063	TILTENHET 0-30° TDX	1
3	SP1550693	(Mounting brackets, cpl.)	1
4	SP1544888	(Actuator)	1
5	SP1529400	FESTEBOLTER SETELØFT BORA <i>for Bora only</i>	1




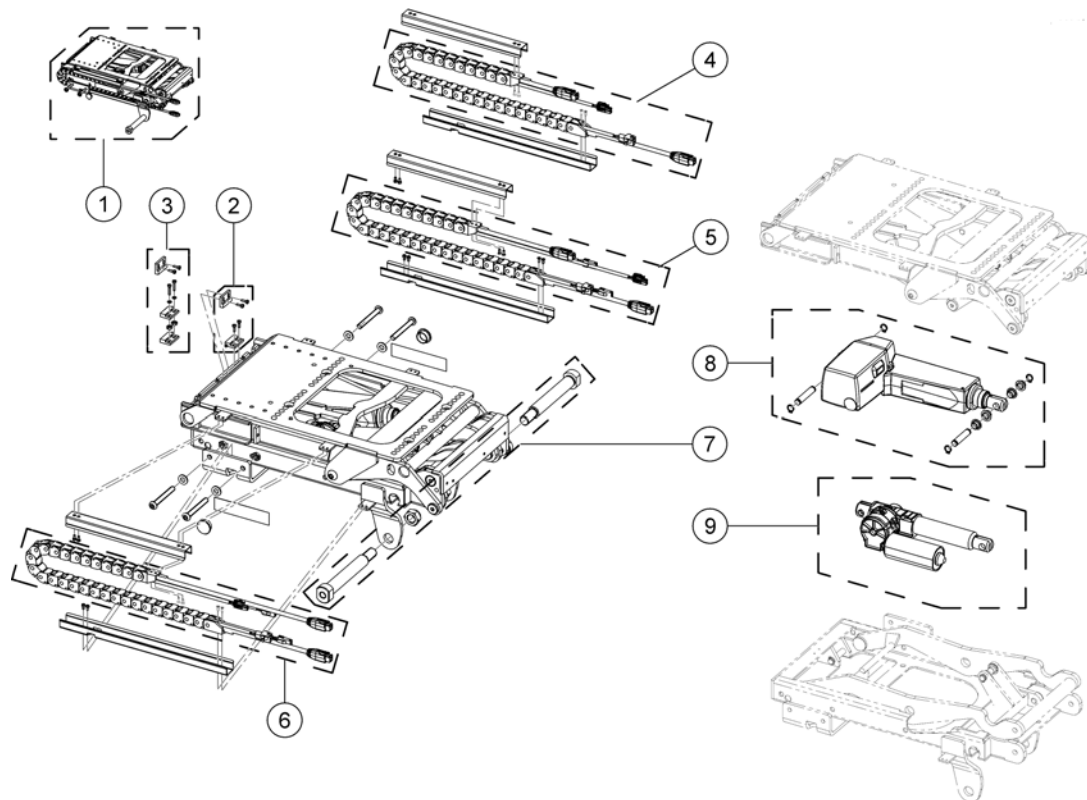

-	SP1533562	(Actuator extension cable, ACS2)	1
-	SP1536110	(Actuator extension cable, R-Net)	1
-	SP1571139	(Tilt actuator cable ACS2)	1
-	SP1571185	(Tilt actuator cable R-Net)	1
1	SP1531240	SETELØFT KPL BORA	1
1	SP1533981	(Lifter, R-Net)	1
2	SP1537899	(Speed reduction sensor, R-Net)	1
3	SP1537902	(Speed reduction sensor, ACS2)	1
4	SP1537396	(Energy chain for powered backrest and/or powered legrests, R-Net)	1
5	SP1534079	(Energy chain for lifter, ACS2)	1
6	SP1537475	(Energy chain for lifter, R-Net)	1
7	SP1529400	FESTEBOLTER SETELØFT BORA	1
8	SP1537887	(Lifter actuator, R-Net)	1
8	SP1533558	MOTOR SETELØFT BORA	1
9	SP1537888	(Tilt actuator, R-Net)	1
9	SP1533557	MOTOR TILT BORA	1

Invacare® Bora | Sete  
**(Bellows)**

1544177X.iso

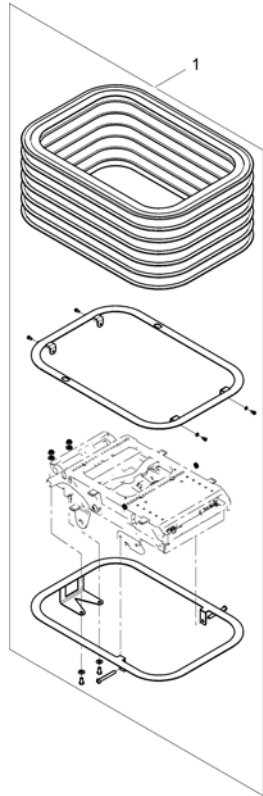


1 SP1544177

BESKYTTELSE SETEHEIS BORA

1


1571918.dwg

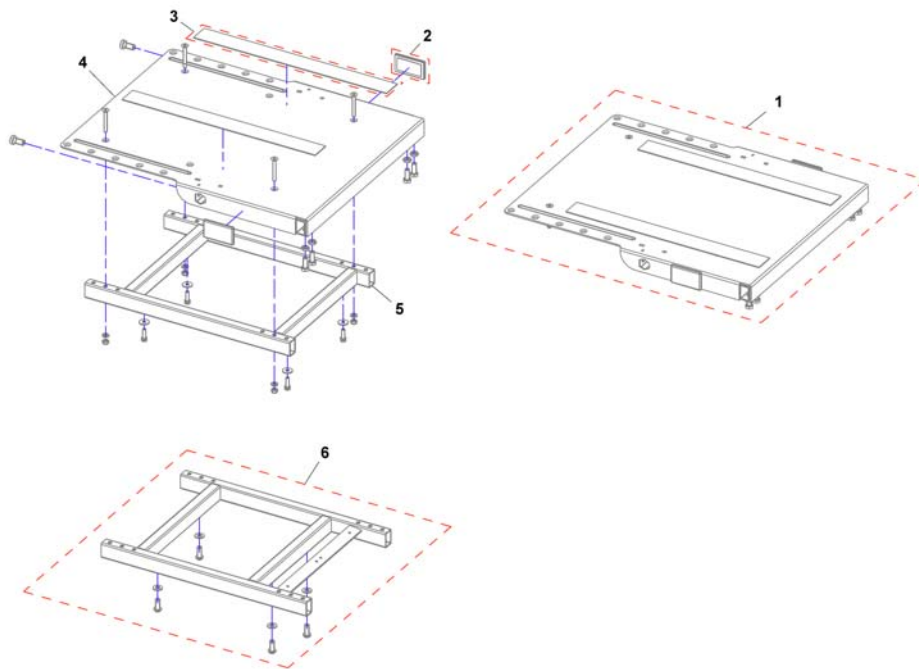


1	SP1526884	(Seat plate assembly) <i>without Lifter</i>	1
1	SP1531709	(Seat plate assembly for Lifter) <i>for Lifter</i>	1
2	P31298	REFLKES STORM	1
3	SPU21087	(Velcro)	1
4	1419999-9005	SETEPLATE HEL DRAGON	1
5	SP1523467	SETERAMME BORA <i>without lifter</i>	1
6	SP1532637	(Seat frame for Lifter) <i>with Lifter</i>	1

# (Seat Frame Support)

1524451PAC.dwg

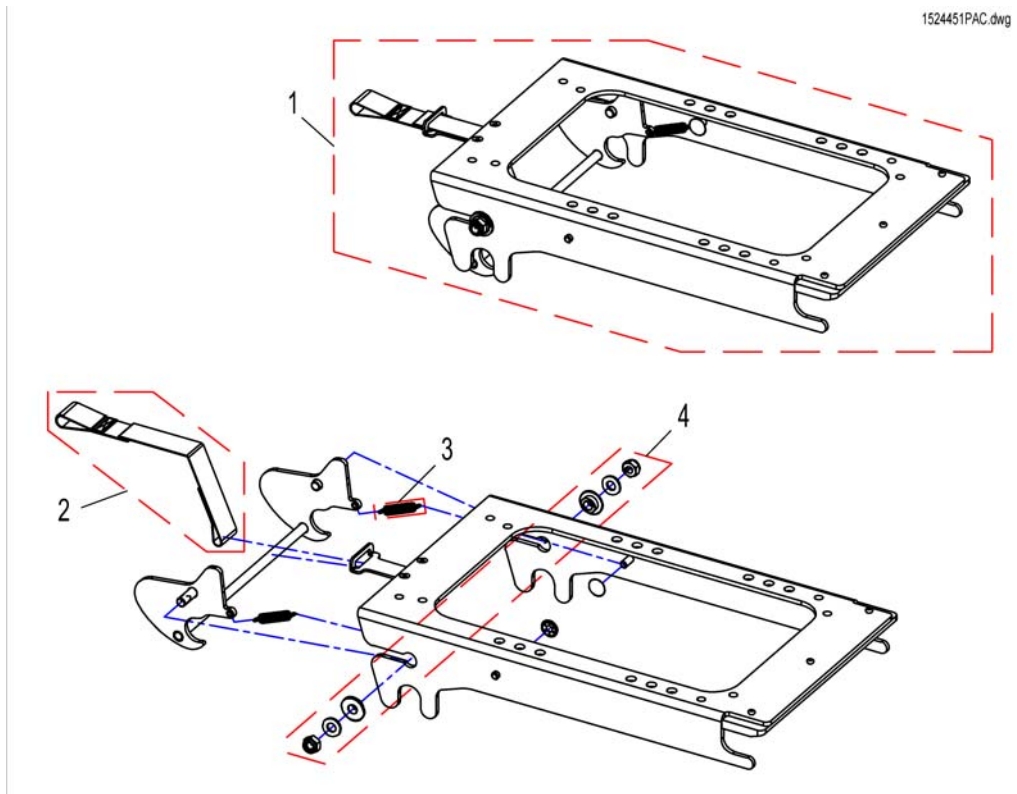


1	SP1524451	SETERAMME SUPPORT KPL BORA	1
1	SP1541790	(Seatframe support Leckey child seat (UK only))	1
2	SP1524455	(Strip)	1
3	SP1528340	(Pull-Spring)	1
4	SP1528341	(Bearing-Kit)	1

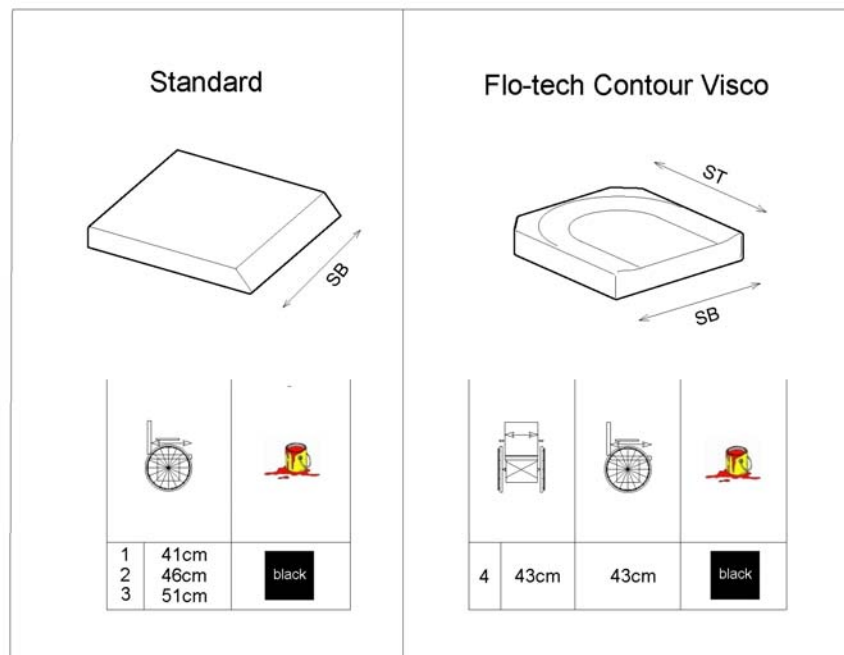

1	1420460	SETEPUTE SD41 STD DRAGON VERTIC [1336+0326]	1
2	1420461	SETEPUTE SD46 STD DRAGON VERTIC [1336+0327]	1
3	1420462	SETEPUTE SD51 STD DRAGON VERTIC [1336+0330]	1
4	1501113	(Seat Cushion Flo-tech Contour Visco SW 43cm x SD 43cm) [1901]	1


# (Flex2 Seat)

1571699.iss

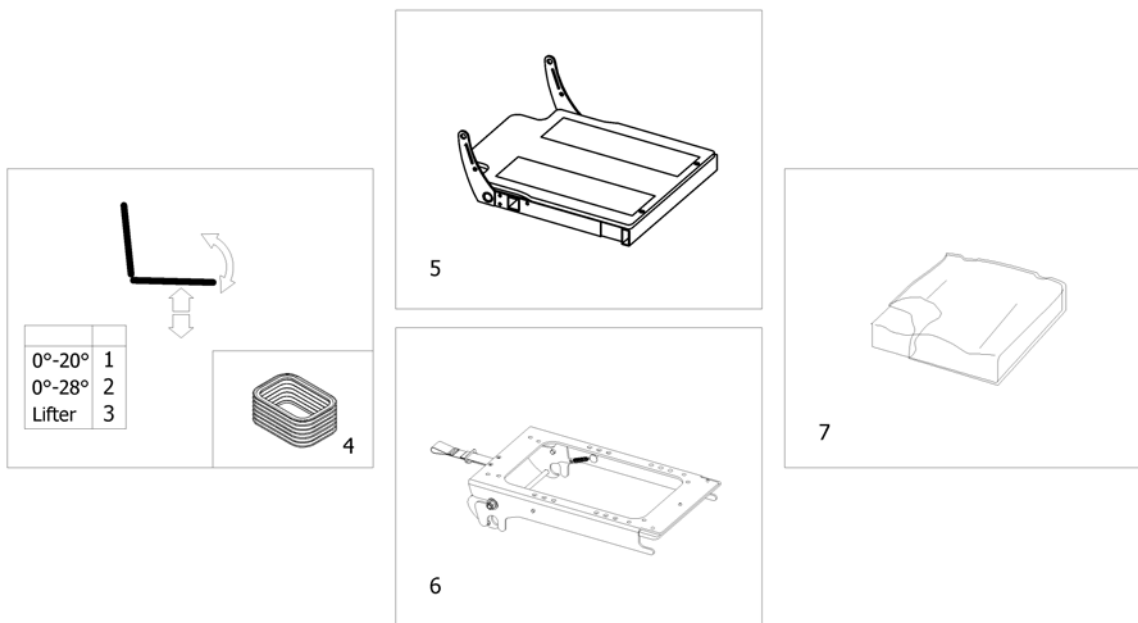


1	-	(Tilt 20°) [1002/1014]	1
2	-	(Tilt 28°) [1043]	1
3	-	(Lifter) [1021]	1
4	-	(Bellows) [1311]	1
5	-	(Seat Frame with Plates)	1
6	-	(Seat Frame Support)	1
7	-	(Seat cushion)	1

# (Seat Frame with Plates)

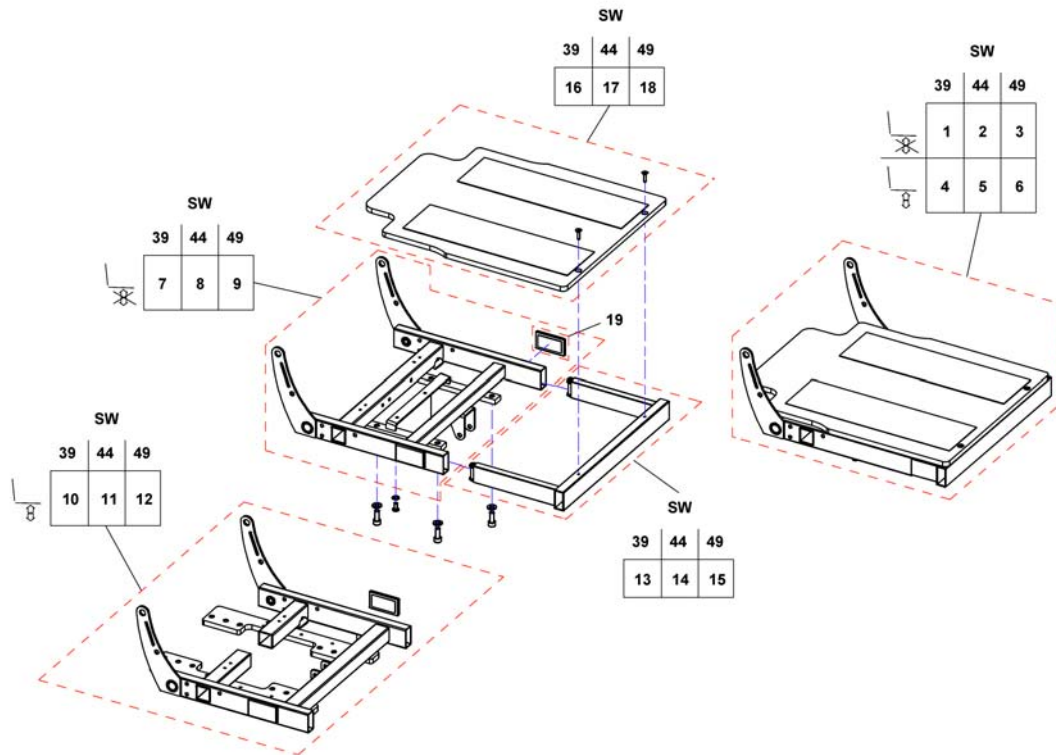


1	SP1526886	(Flex 2 Seat frame for SW 39)	1
2	SP1526887	RYGGRØRADAPTER SB44 BORA	1
3	SP1526888	RYGGRØRADAPTER SB49 BORA	1
4	SP1531262	(Flex2 Seat frame for SW 39)	1
5	SP1531263	SETERAMME FLEX2 SB44 BORA	1
6	SP1531264	(Flex2 Seat frame for SW 49)	1
7	SP1528422	SETERAMME SB39 BORA	1
8	SP1528423	SETERAMME SB44 BORA	1
9	SP1528424	SETERAMME SB49 BORA	1
10	SP1536658	SETERAMME M SETEHE SB39 BORA	1
11	SP1536659	SETERAMME M SETEHE SB44 BORA	1
12	SP1536660	SETERAMME SB49 BORA	1
13	SP1528416	FESTE BENST SB39 KPL BORA	1
14	SP1528417	FESTE BENST SB44 KPL BORA	1
15	SP1528418	FESTE BENST SB49 KPL BORA	1
16	SP1528419	SETEPLATE SB39 BORA	1
17	SP1528420	SETEPLATE SB44 BORA	1
18	SP1528421	SETEPLATE SB49 BORA	1
19	P31298	REFLKES STORM	1

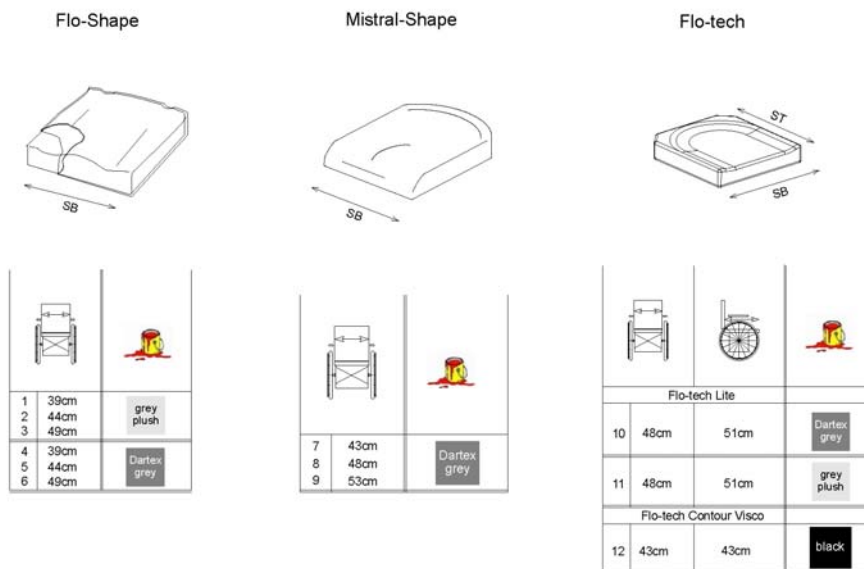


1	1508054	PUTE SETE FLO-SHAPE 355X535 TR 32 [1464+0289]	1
2	1508056	PUTE SETE FLO-SHAPE 405X535 TR 32 [1465+0289]	1
3	1508058	PUTE SETE FLO-SHAPE 455X535 TR 32 [1466+0289]	1
4	1508053	PUTE SETE FLO-SHAPE 355X535 TR 23 [1464+0278]	1
5	1508055	PUTE SETE FLO-SHAPE 405X535 TR 23 [1465+0278]	1
6	1508057	PUTE SETE FLO-SHAPE 455X535 TR 23 [1466+0278]	1
7	1435856	SETEPUTE 365/490 MISTRAL 2LAGS TR23 [1470]	1
8	1435857	SETEPUTE 415/490 MISTRAL 2LAGS TR23 [1321]	1
9	1435858	SETEPUTE 465/490 MISTRAL 2LAGS TR23 [1471]	1
10	SP1525259-0232	SETEPUTE SB48SD51 EASYAD STORM4 DARTEX [1978+0278]	1
11	SP1525259-0237	(Easy-Adapt cushion (Flo-tech Lite) SW 48 x SD 51cm Grey Plush) [1978+0289]	1
12	1501113	(Seat Cushion Flo-tech Contour Visco SW 43cm x SD 43cm) [1901]	1



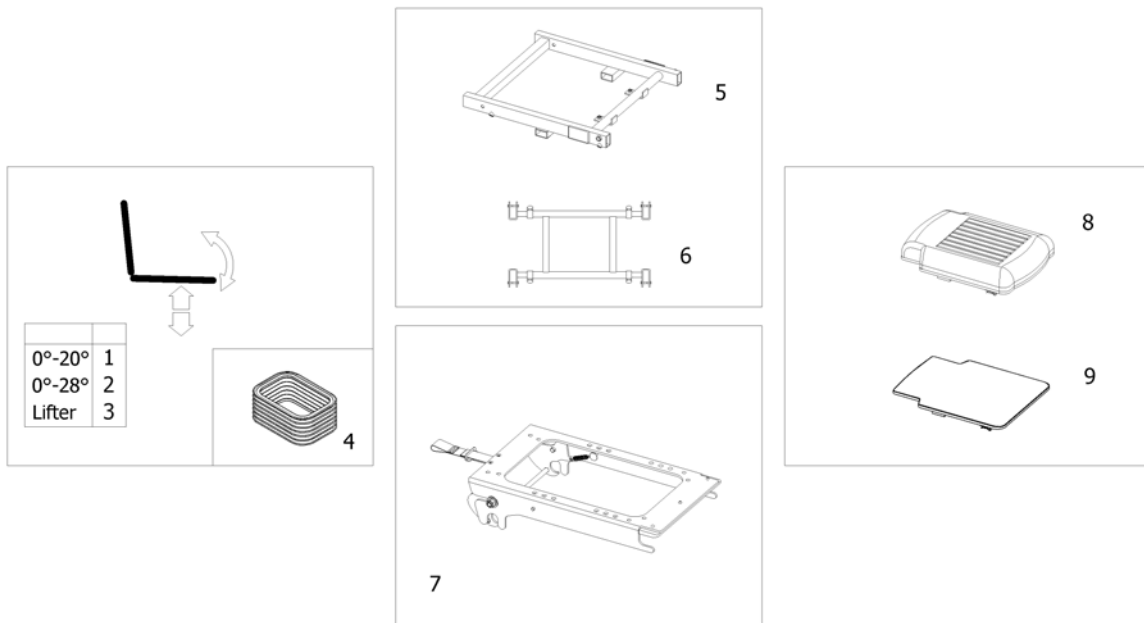


1	-	(Tilt 20°) [1002/1014]	1
2	-	(Tilt 28°) [1043]	1
3	-	(Lifter) [1021]	1
4	-	(Bellows) [1311]	1
5	-	(Seat Frame)	1
6	-	(RUIK-Seat Frame)	1
7	-	(Seat Frame Support)	1
8	-	(Seat Cushion)	1
9	-	(Seat Board)	1

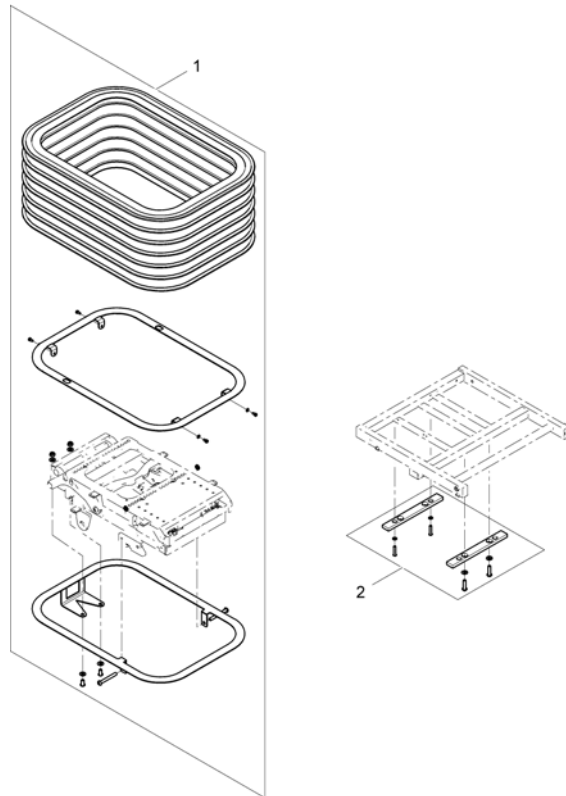


---

1	SP1544177	BESKYTTELSE SETEHEIS BORA	1
2	SP1544176	(Distance plates for bellows)	1

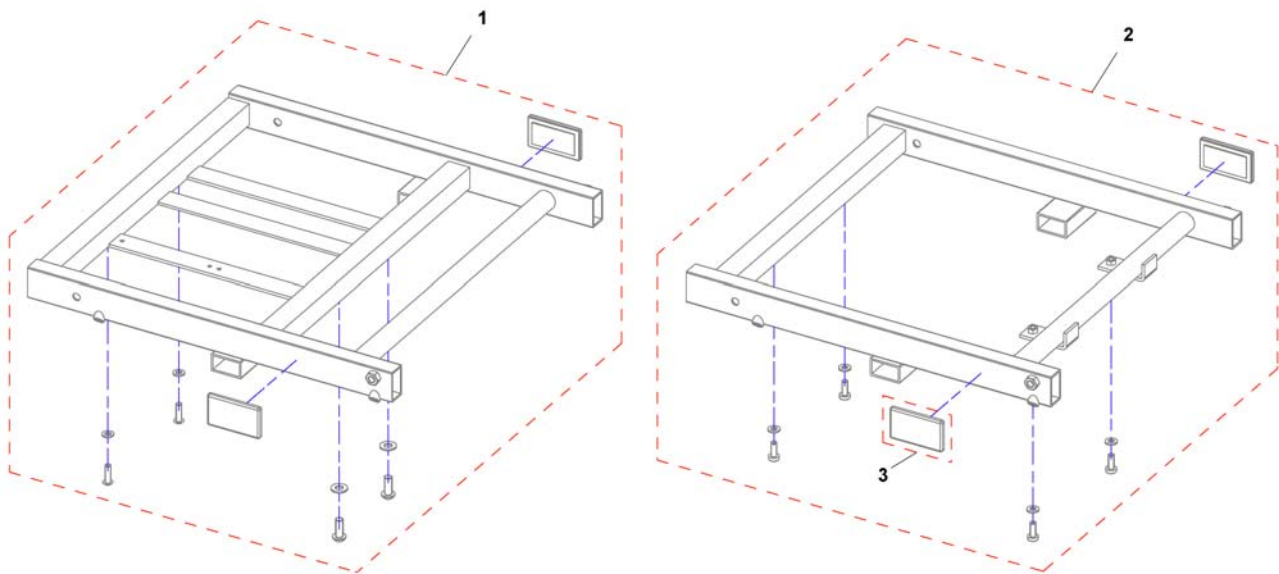

1	SP1531247	(Seat frame SW 33 with Lifter)	1
1	SP1531260	(Seat frame SW 38 with Lifter)	1
1	SP1531261	(Seat frame SW 43 with Lifter)	1
2	SP1527077	(Seat frame SW 33 without Lifter)	1
2	SP1527078	(Seat frame SW 38 without Lifter)	1
2	SP1527087	(Seat frame SW 43 without Lifter)	1
3	P31298	REFLKES STORM	1

# (RUIK-Seat Frame)

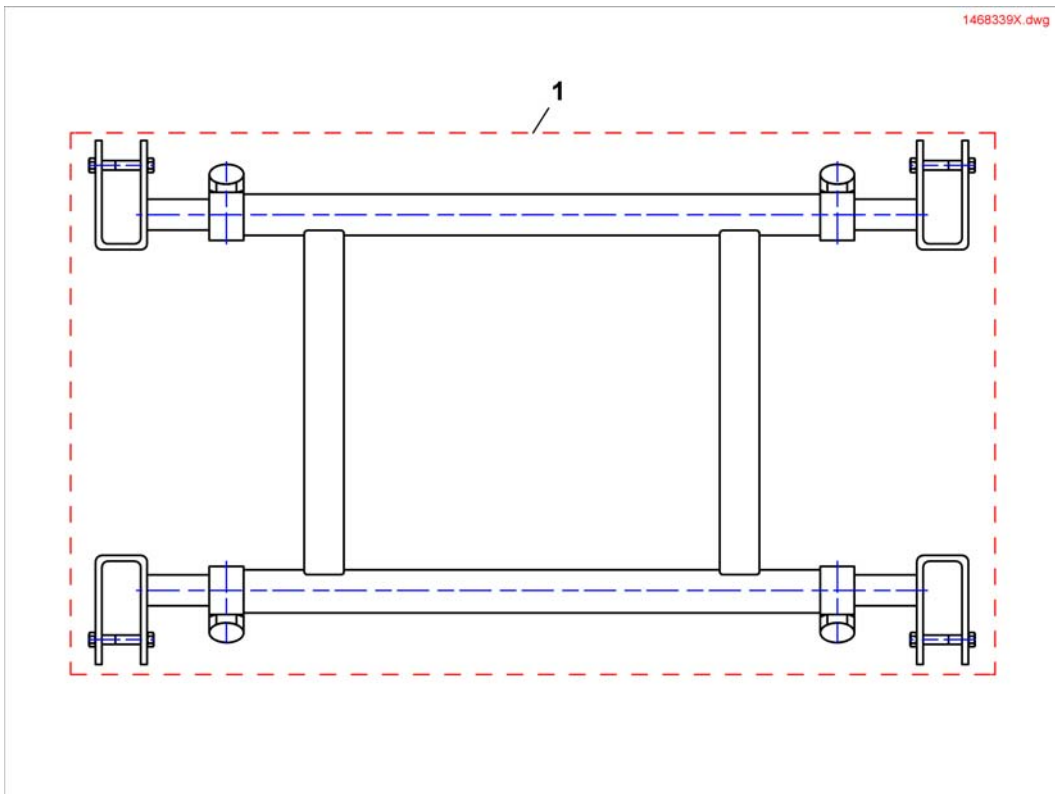


1 1468339

(RUIK-Seat Frame)

1


1511708/00

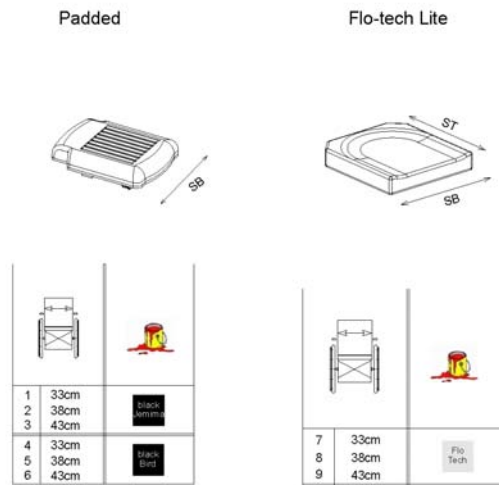

1	SP1497477	(Padded Seat SW 33, Black Jemima) [1411+0373+0286]	1
2	SP1497478	(Padded Seat SW 38, Black Jemima) [1411+0371+0286]	1
3	SP1497479	(Padded Seat SW 43, Black Jemima) [1411+0347+0286]	1
4	SP1497480	(Padded Seat SW 33, Black Bird) [1411+0373+0274]	1
5	SP1497481	(Padded Seat SW 38, Black Bird) [1411+0371+0274]	1
6	SP1497482	(Padded Seat SW 43, Black Bird) [1411+0347+0274]	1
7	SP1497500	(Seat cushion Flo Tech Lite SW 33cm) [1467]	1
8	SP1497501	(Seat cushion Flo Tech Lite SW 38cm) [1468]	1
9	SP1497539	(Seat cushion Flo Tech Lite SW 43cm) [1469]	1


# (Seat Board)

1422524PAC.dwg

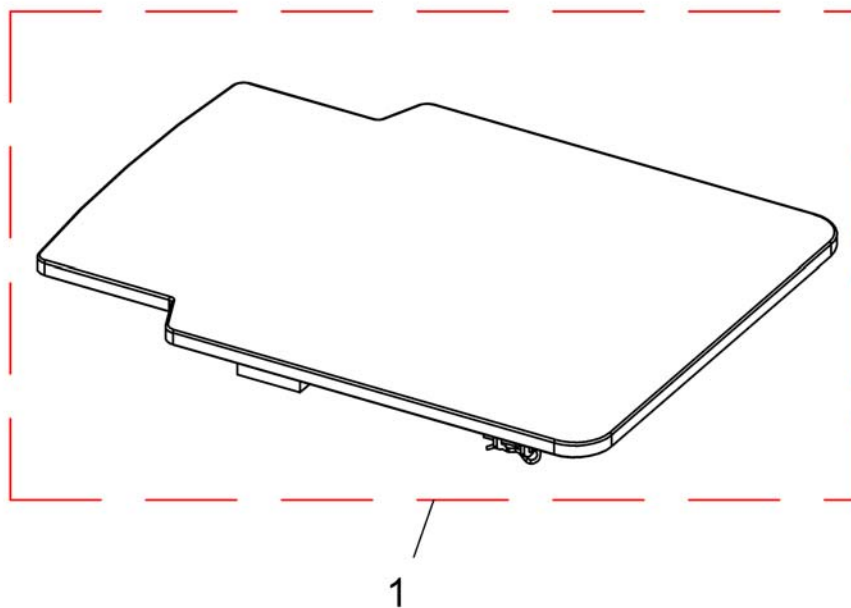

---


1	SP1497474	(Seat Board SW 33, Black Bird)	1
1	SP1497475	(Seat Board SW 38, Black Bird)	1
1	SP1497476	(Seat Board SW 43, Black Bird)	1
1	SP1543027	(Seat Board SW 33, Black Jemima)	1
1	SP1543028	(Seat Board SW38, Black Jemima)	1
1	SP1543029	(Seat Board SW 43, Black Jemima)	1

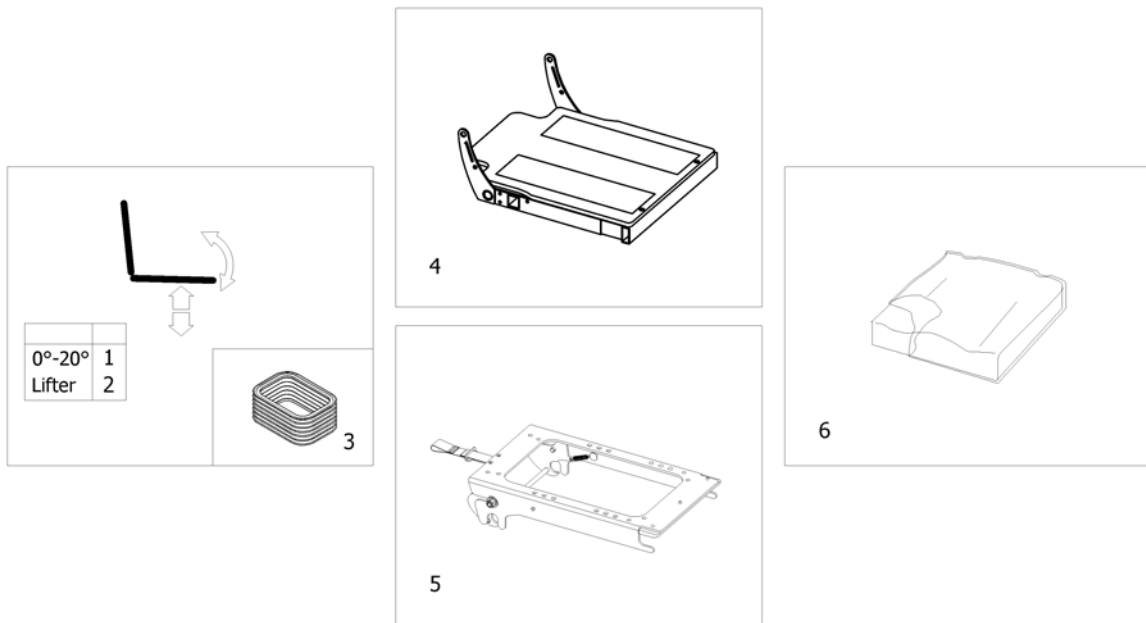

1	-	(Tilt 20°) [1002/1014]	1
2	-	(Lifter) [1021]	1
3	-	(Bellows) [1311]	1
4	-	(Seat Frame with Plates)	1
5	-	(Seat Frame Support)	1
6	-	(Seat Cushion Max)	1

## (Seat Frame with Plates)


15312826.dwg

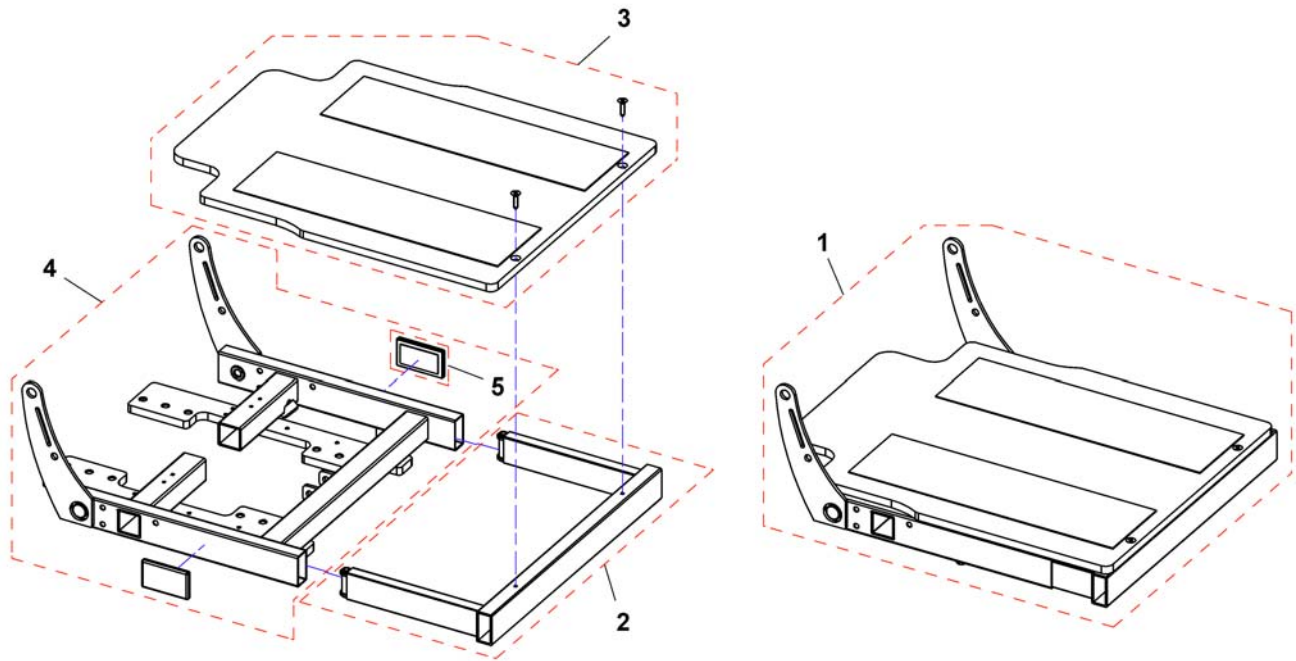

1	SP1551683	(Seat frame for 5)	1
2	SP1523723	(Legrest attachment SW 55)	1
3	SP1525083	(Seat plate SW 55)	1
4	SP1544672	(Seat frame SW 55)	1
5	P31298	REFLEKES STORM	1



1511723 (en)

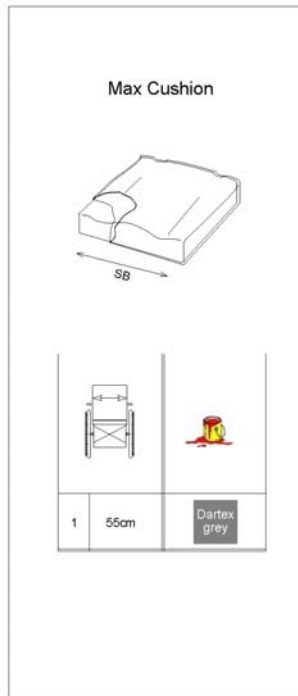


1 SP1536595

SETEPUTE SB55 SD65 MAX STORM4 MAX  
[1459]


1

Invacare® Bora | Sete  
**(Modulite Seat)**

1572995.iss

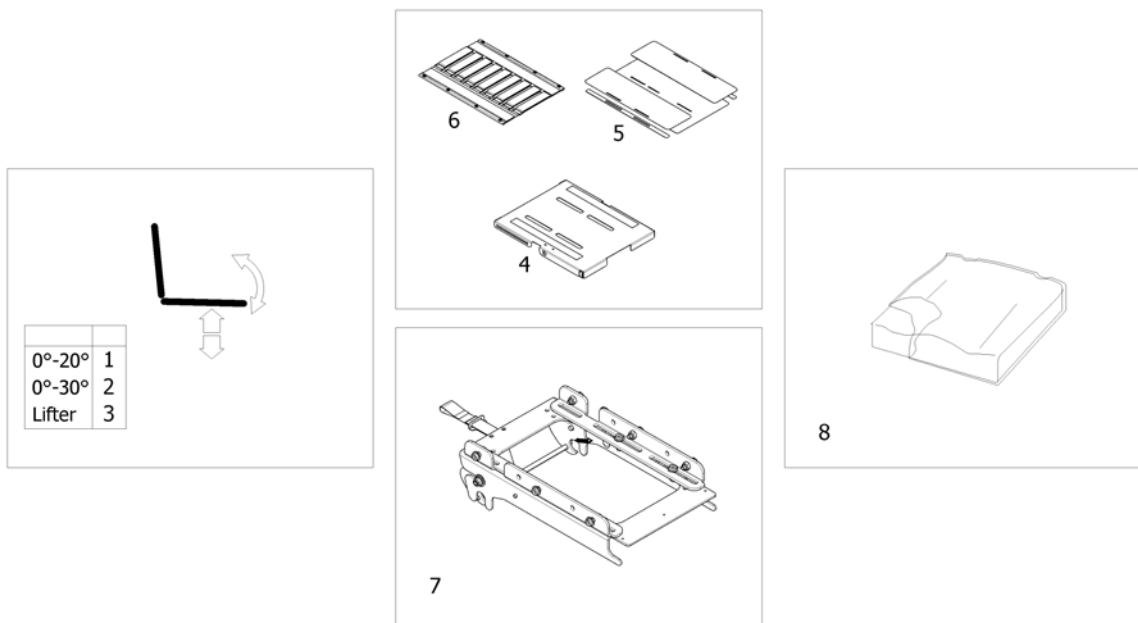

1	-	(Tilt 20°) [1091/1092]	1
2	-	(Tilt 30°) [1093]	1
3	-	(Lifter) [1094]	1
4	-	(One Piece Seat Plate) [2110]	1
5	-	(Telescopic Seat Frame and Plate) [2111]	1
6	-	(Telescopic Seat Frame and Sling Seat) [2112]	1
7	-	(Seat frame support)	1
8	-	(Seat Cushion)	1

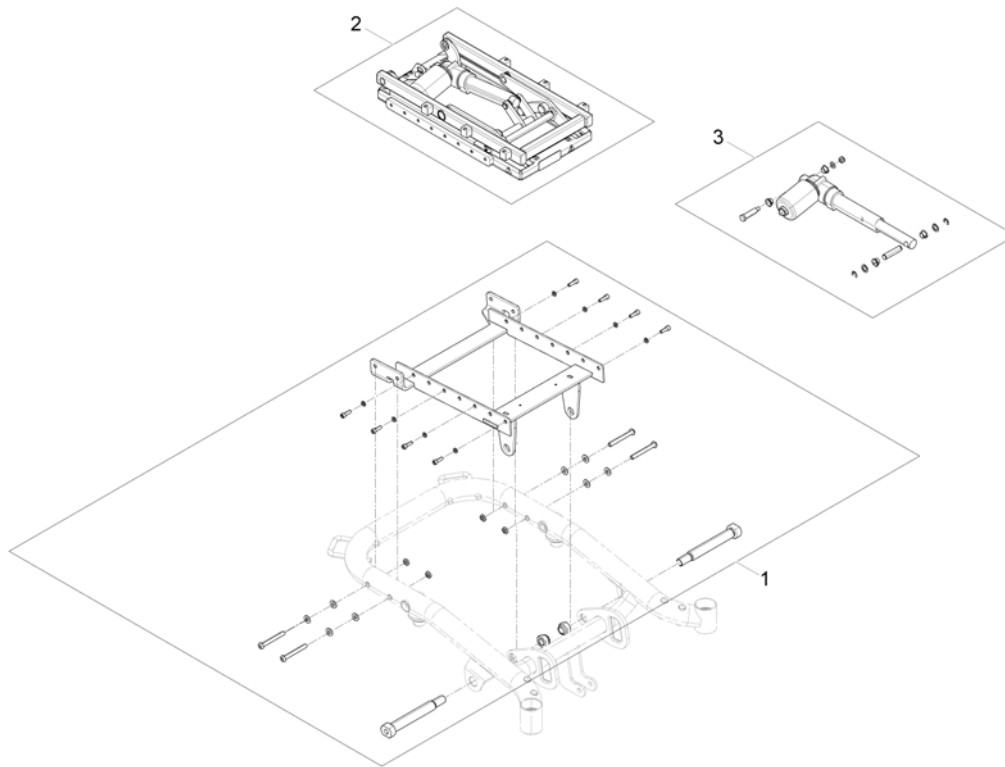


---

1	SP1568399	(Tilt bracket)	1
2	SP1558714	(Tilt unit)	1
3	SP1571809	(Actuator)	1

**(Lifter)**


157000 no

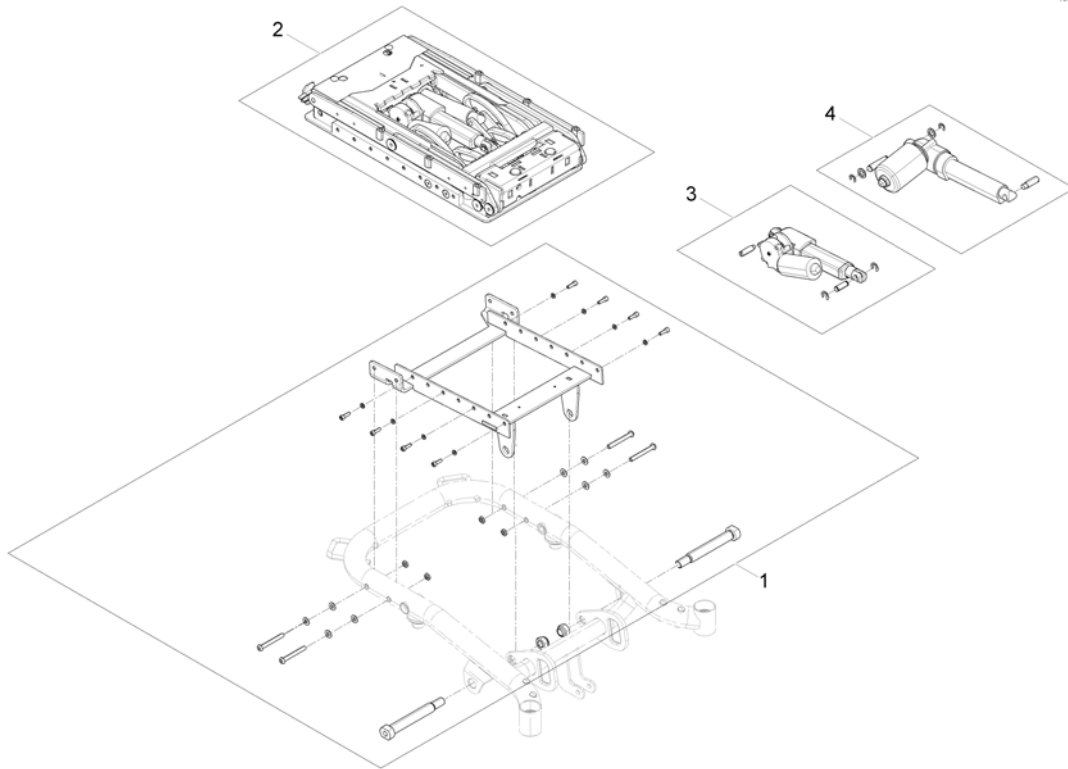


---

1	SP1568399	(Tilt bracket)	1
2	SP1558715	(Lifter- / Tilt unit)	1
3	SP1571822	(Tilt actuator)	1
4	SP1571823	(Lifter actuator)	1

1571064.000

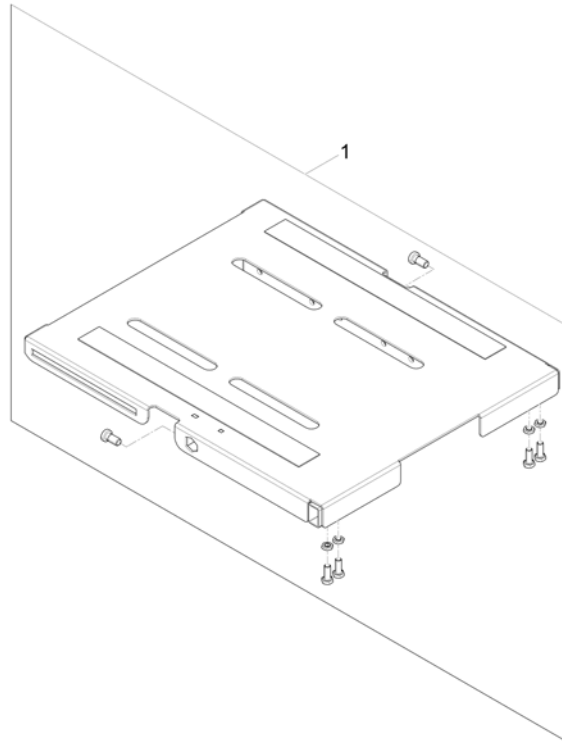


1 SP1571297

(One Piece Seat Plate)

1

# (Telescopic Seat Frame and Plate)

1572312 iso

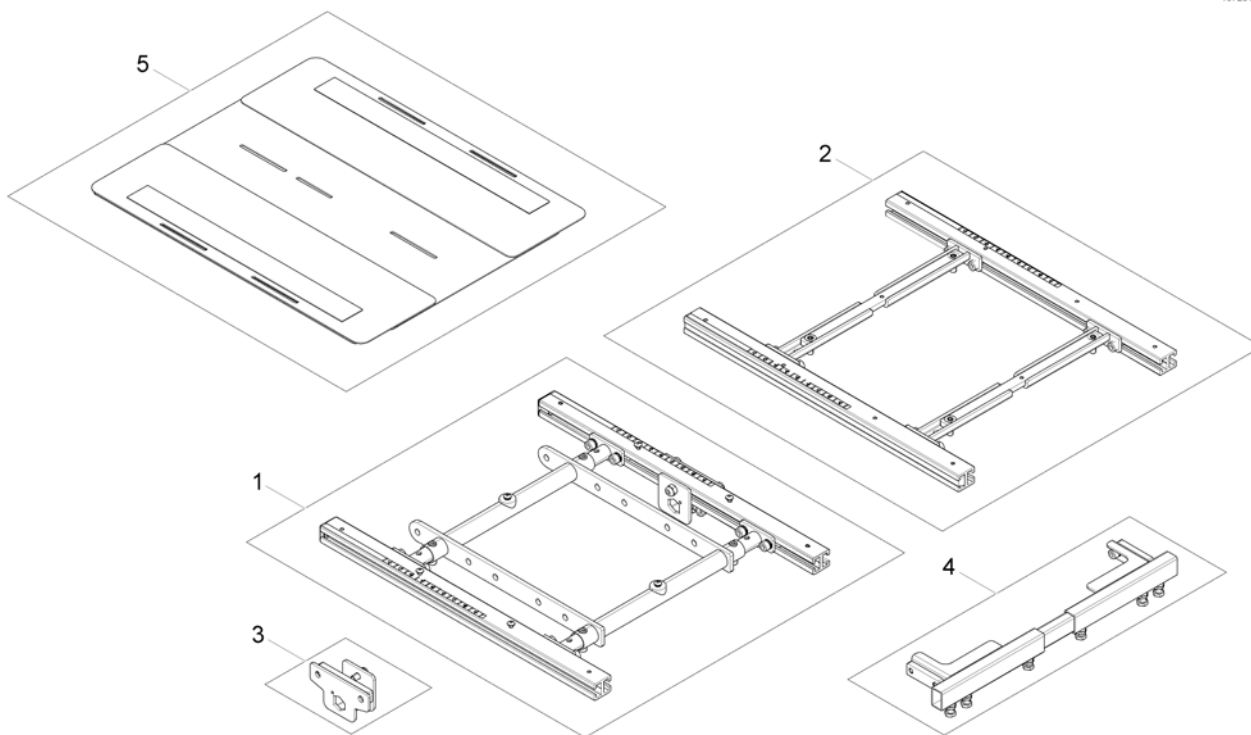



---

1	-	(Telescopic Seat Frame for Tilt 12° / 20°)	1
2	-	(Telescopic Seat Frame for Tilt 30° / Lifter)	1
3	-	(Armrest bracket)	1
4	-	(Legrest Support)	1
5	-	(Seat Plate)	1

# (Telescopic Seat Frame for Tilt 12° / 20°)


1572009 iso

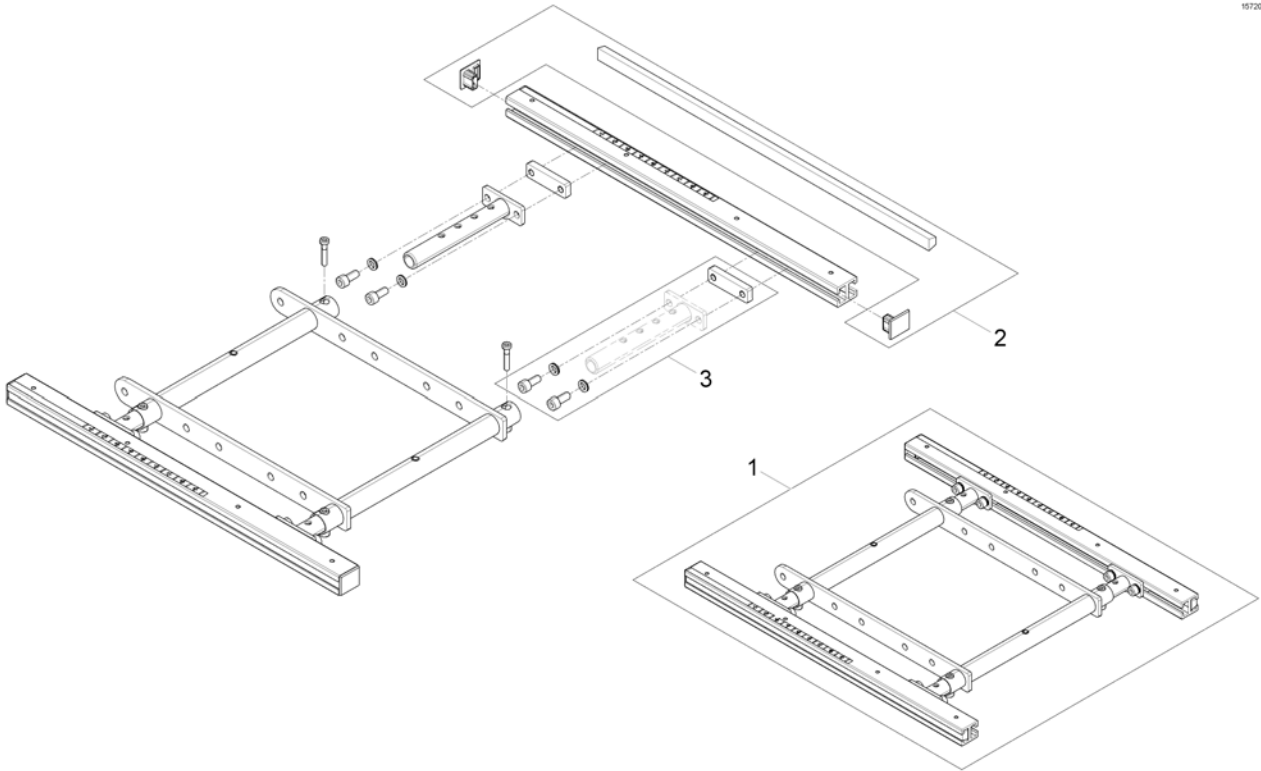

1	SP1572977	(Seatframe, cpl., pre set up to SW 43cm)	1
2	SP1571810	(Sealing strap with plugs)	1
3	SP1571811	(Groove stone with mounting parts)	1

# (Telescopic Seat Frame for Tilt 30° / Lifter)


1572004 no

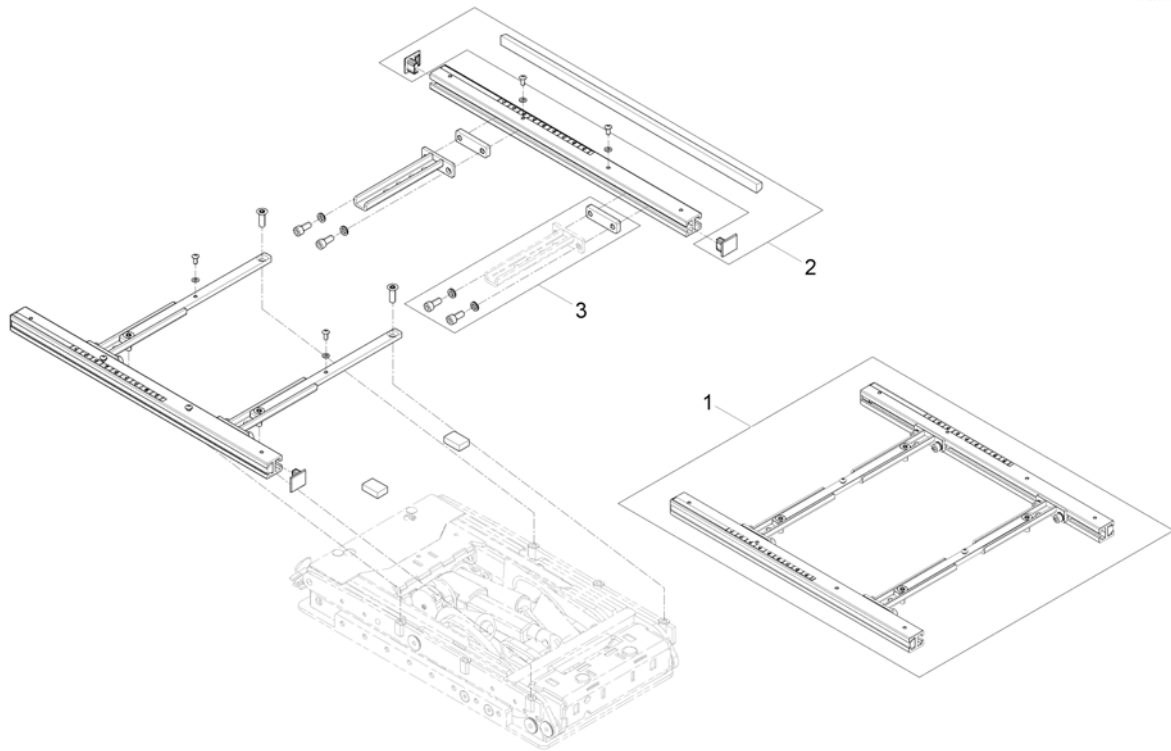


---

1	SP1572992	(Seatframe, cpl., pre set up to SW 43cm)	1
2	SP1571810	(Sealing strap with plugs)	1
3	SP1571811	(Groove stone with mounting parts)	1



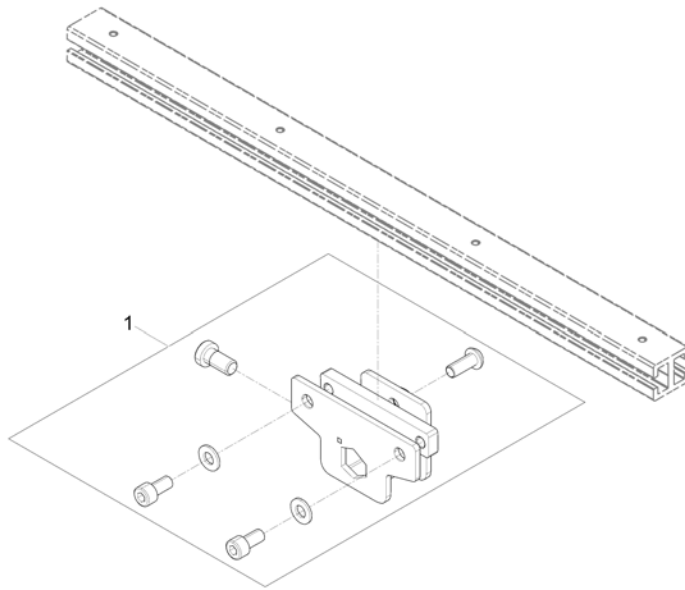

---


1 SP1570189

(Armrest bracket)


1

# (Legrest Support)

1572021.000

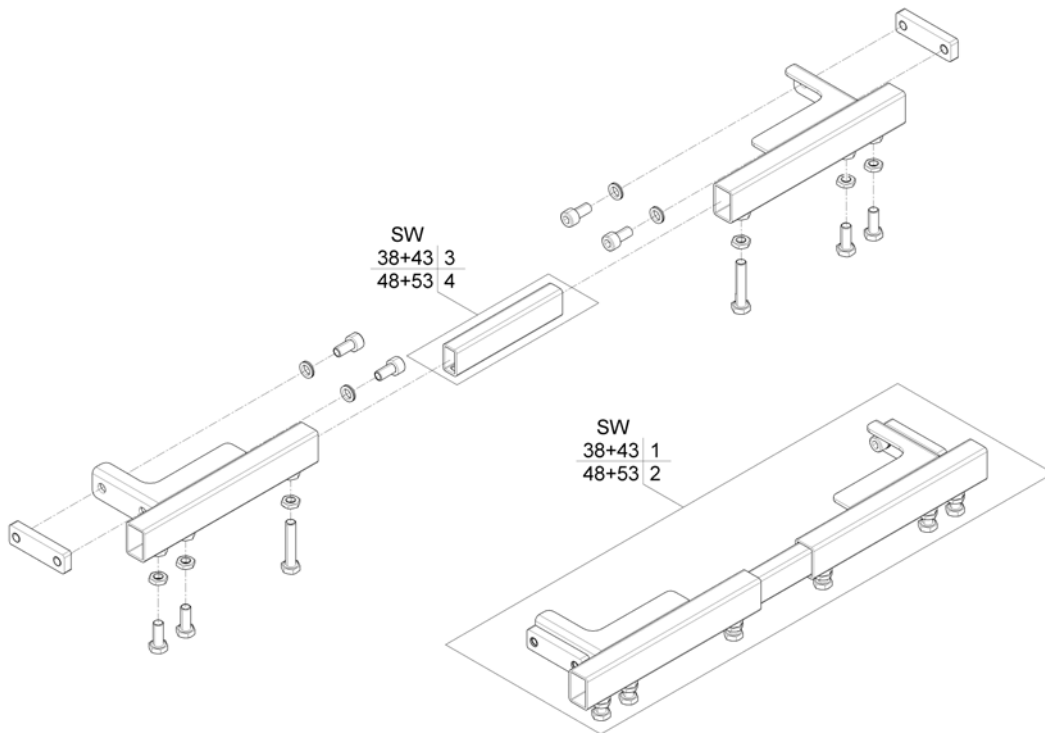

1	SP1558935	(Legrest Support SW38+43, cpl.)	1
2	SP1559294	(Legrest Support SW48+53, cpl.)	1
3	SP1558134	(Connection Tube SW38+43)	1
4	SP1558135	(Connection Tube SW48+53)	1

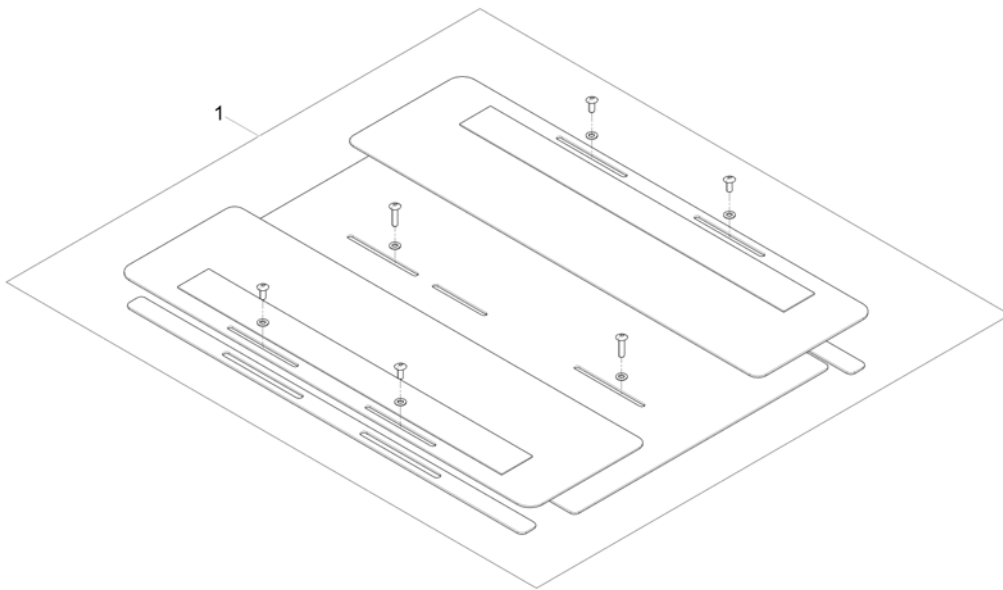


1 SP1572199

(Seat Plate, cpl.)

1

# (Telescopic Seat Frame and Sling Seat)


1572311.00

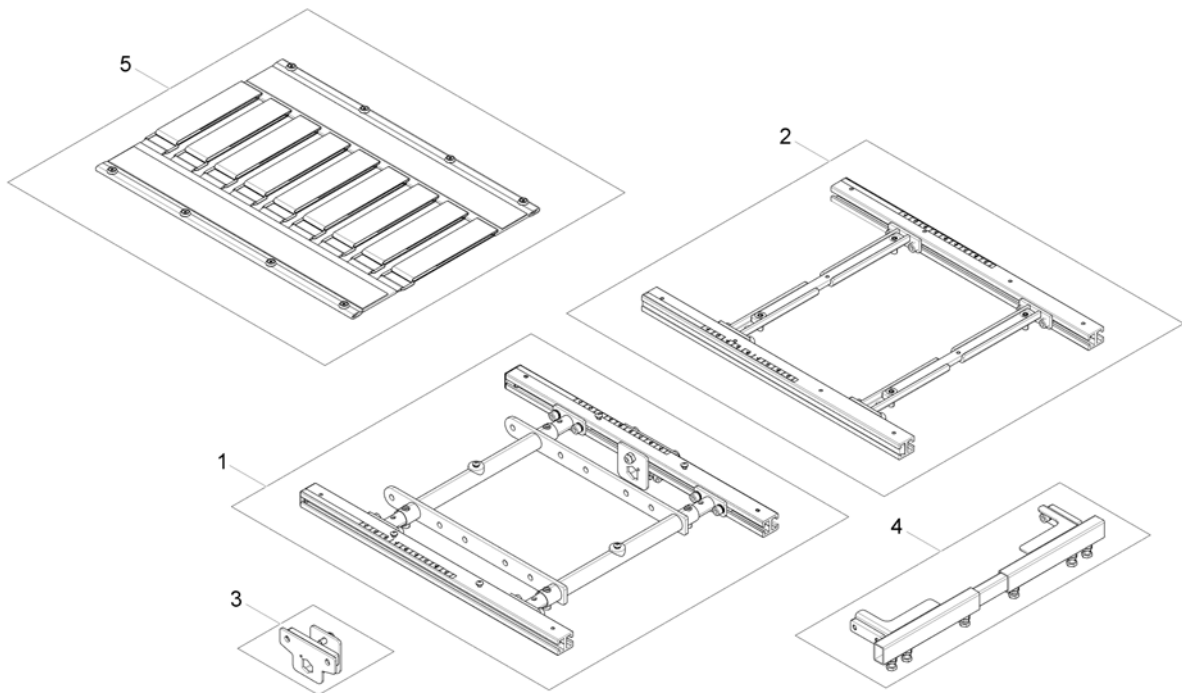

1	-	(Telescopic Seat Frame for Tilt 12° / 20°)	1
2	-	(Telescopic Seat Frame for Tilt 30° / Lifter)	1
3	-	(Armrest bracket)	1
4	-	(Legrest Support)	1
5	-	(Sling Seat)	1

# (Telescopic Seat Frame for Tilt 12° / 20°)


1571950 iso

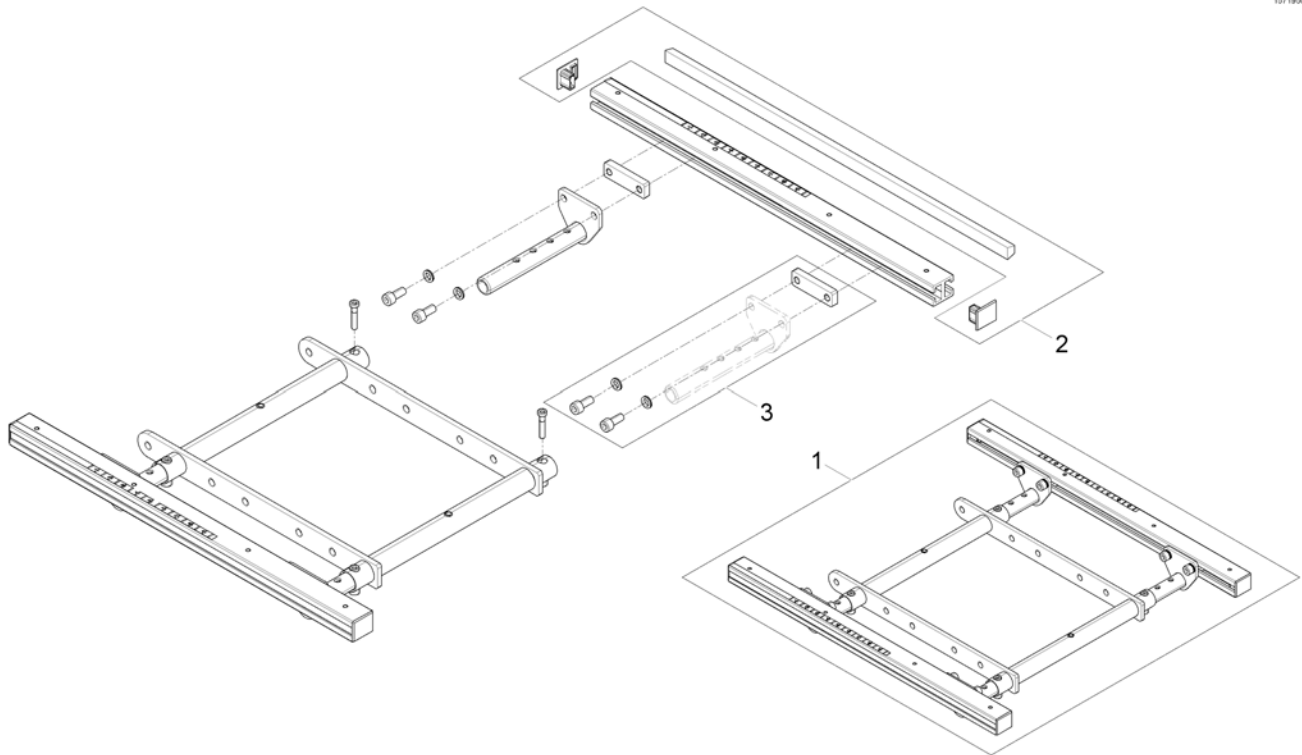


1	SP157298	(Seatframe, cpl., pre set up to SW 43cm)	1
2	SP1571810	(Sealing strap with plugs)	1
3	SP1571811	(Groove stone with mounting parts)	1

## (Telescopic Seat Frame for Tilt 30° / Lifter)

1571832 no

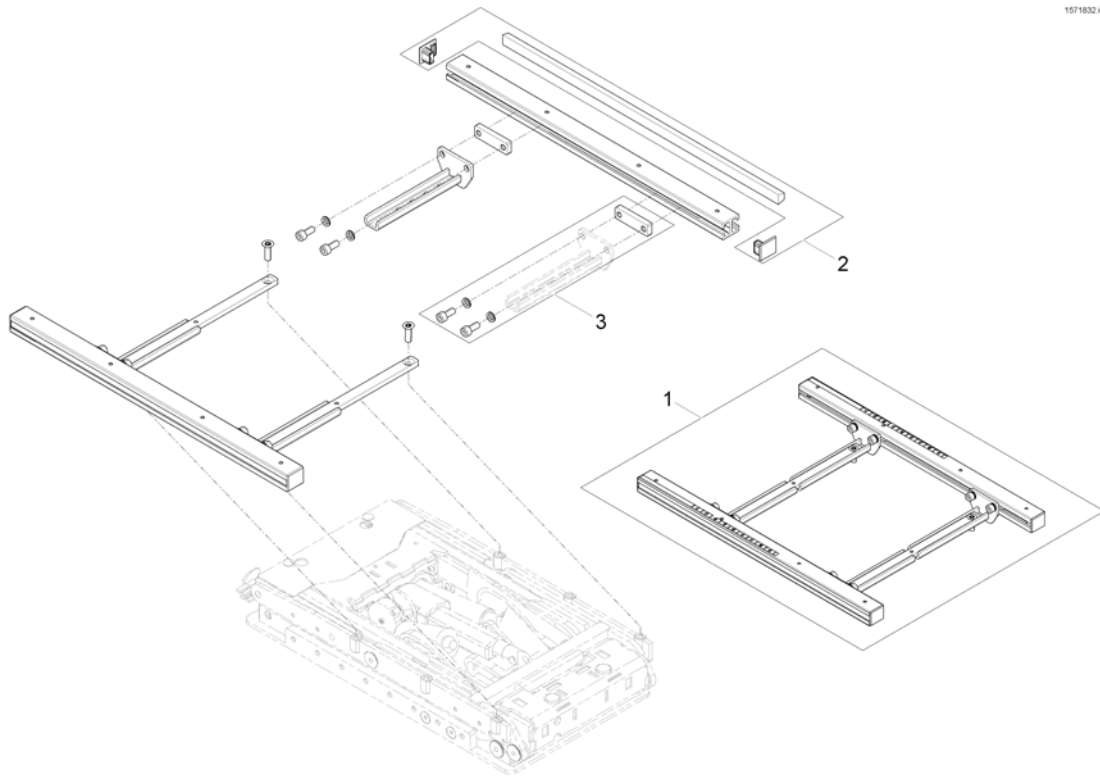


1	SP1572993	(Seatframe, cpl., pre set up to SW 43cm)	1
2	SP1571810	(Sealing strap with plugs)	1
3	SP1571811	(Groove stone with mounting parts)	1

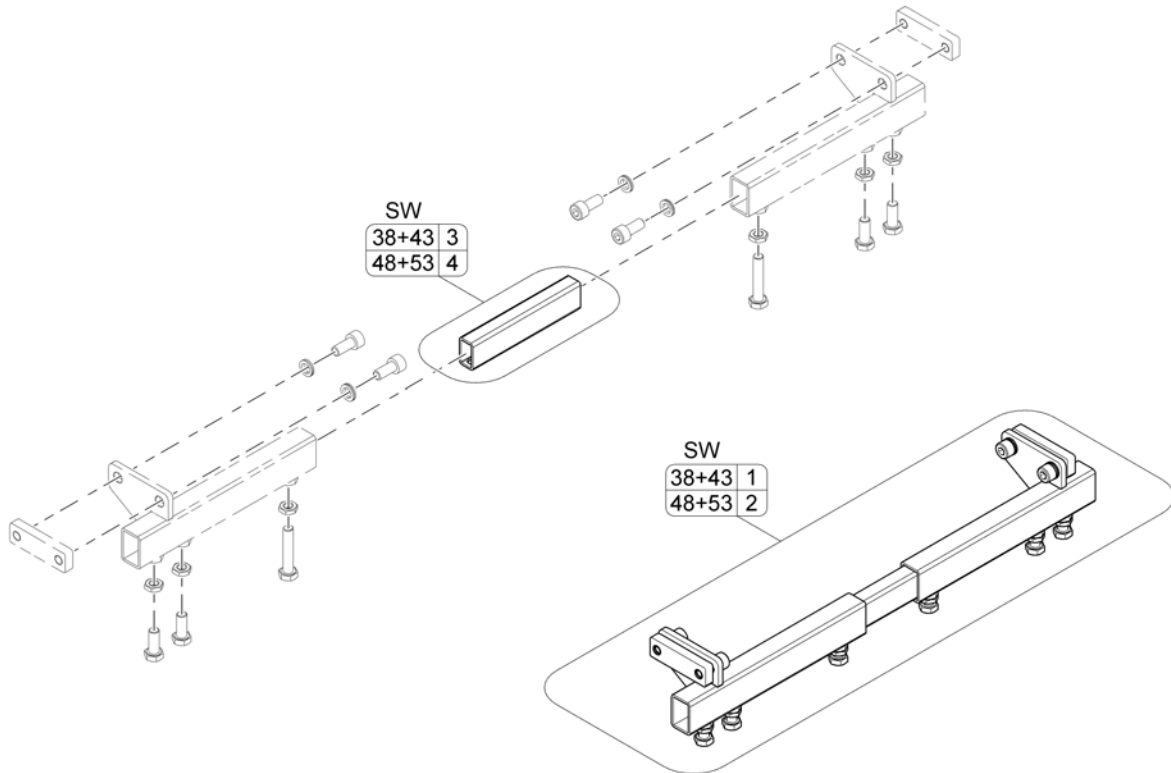

1	SP1559305	(Legrest Support SW38+43, cpl.)	1
2	SP1559307	(Legrest Support SW48+53, cpl.)	1
3	SP1558134	(Connection Tube SW38+43)	1
4	SP1558135	(Connection Tube SW48+53)	1

Invacare® Bora | Sete  
**(Sling Seat)**

151176290

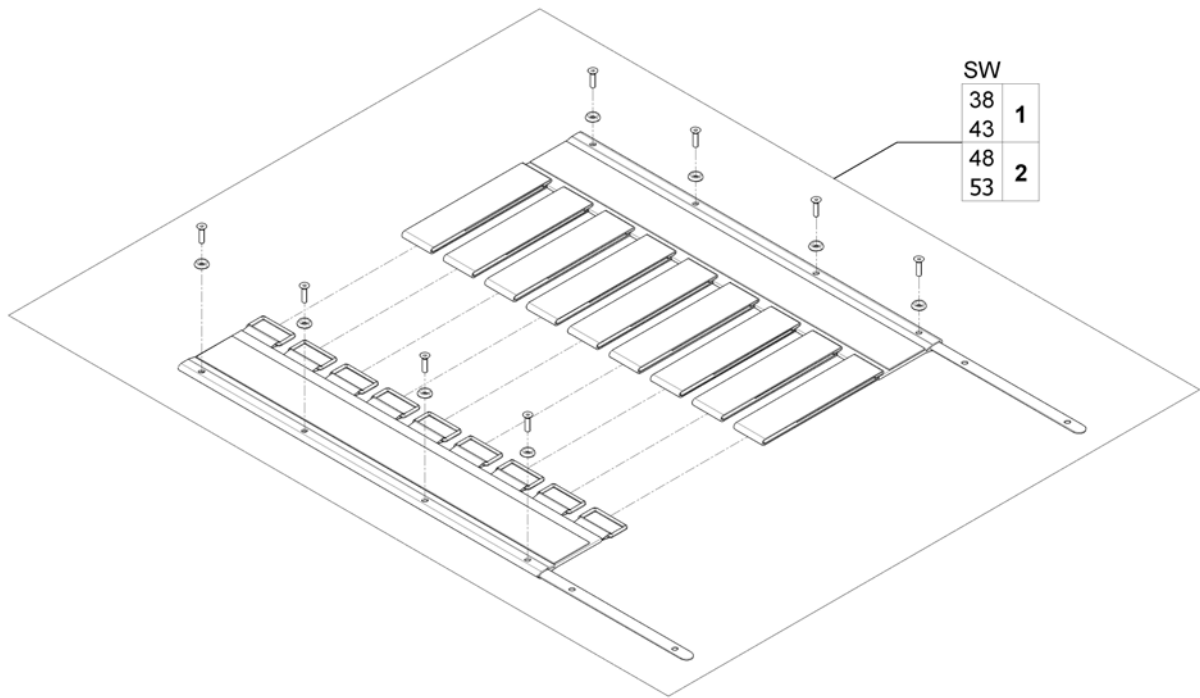


SW	
38	
43	1
48	
53	2

---


1	SP1557877	(Sling Seat)	1
2	SP1555911	(Sling Seat)	1



1572150 iso

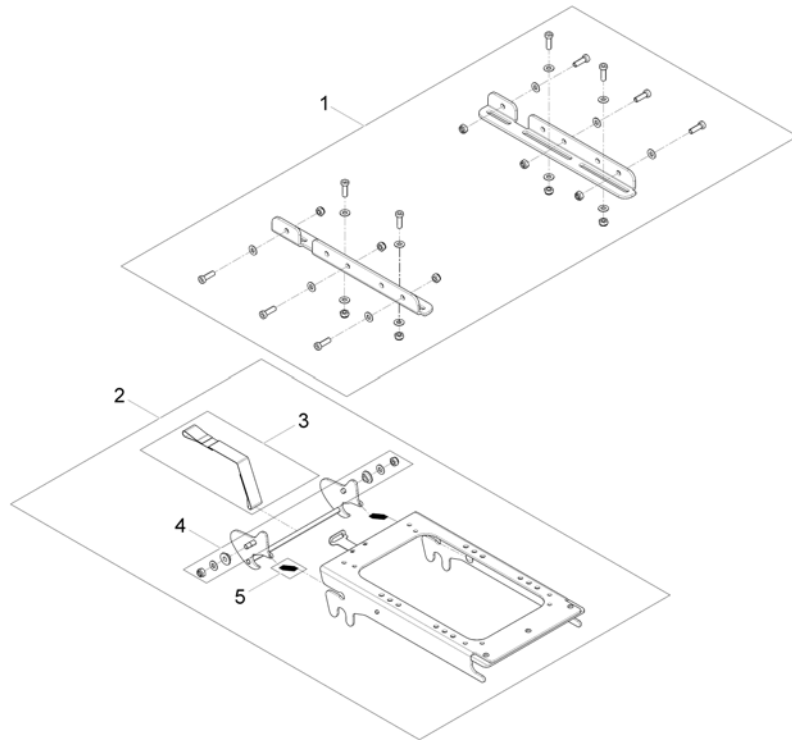

1	SP1563230	(Seatplate bracket)	1
2	SP1524451	SETERAMME SUPPORT KPL BORA	1
3	SP1524455	(Strip)	1
4	SP1528340	(Pull-Spring)	1
5	SP1528341	(Bearing-Kit)	1

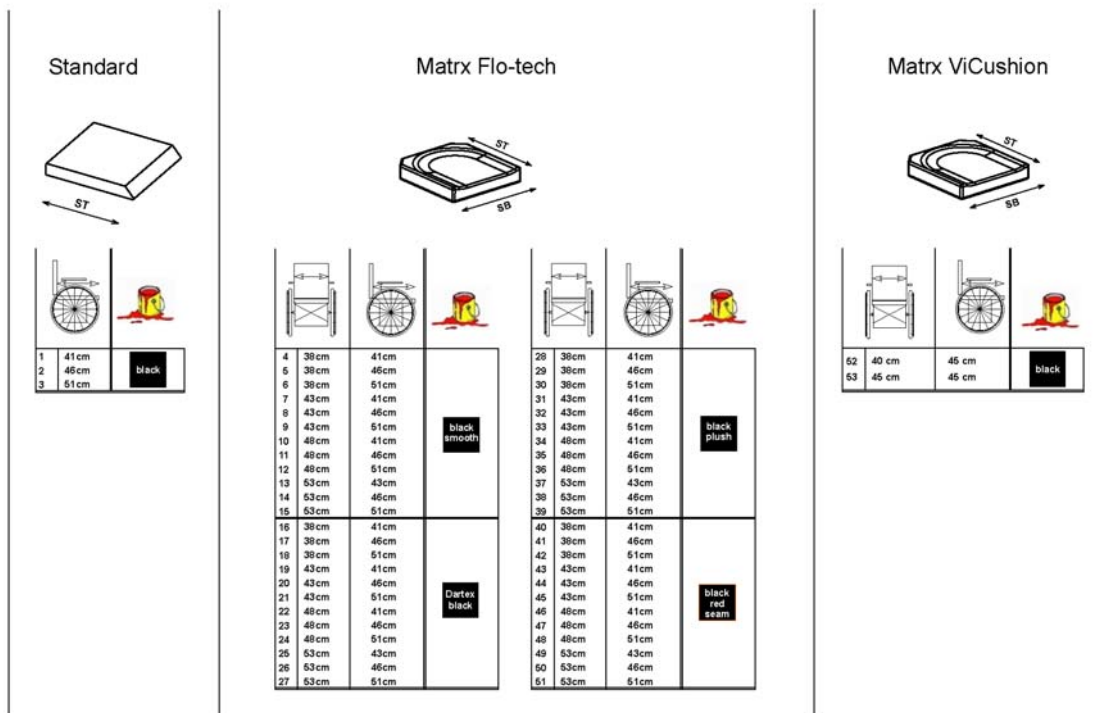


1	1420460	SETEPUTE SD41 STD DRAGON VERTIC [2141]	1
2	1420461	SETEPUTE SD46 STD DRAGON VERTIC [2141]	1
3	1420462	SETEPUTE SD51 STD DRAGON VERTIC [2141]	1
4	SPFLB15/16	SETEPUTE SB38 SD41 SORT STORM4 [2142+2151]	1
5	SPFLB15/18	SETEPUTE SB38 SD46 SORT STORM4 [2142+2151]	1
6	SPFLB15/20	SETEPUTE SB38 SD51 SORT STORM4 [2142+2151]	1
7	SPFLB17/16	SETEPUTE SB43 SD41 SORT STORM4 [2142+2151]	1
8	SPFLB17/18	SETEPUTE SB43 SD46 STD STORM4 [2142+2151]	1
9	SPFLB17/20	SETEPUTE SB43 SD51 SORT STORM4 [2142+2151]	1
10	SPFLB19/16	SETEPUTE SB48 SD41 SORT STORM4 [2142+2151]	1
11	SPFLB19/18	SETEPUTE SB48 SD46 SORT STORM4 [2142+2151]	1
12	SPFLB19/20	SETEPUTE SB48 SD51 SORT STORM4 [2142+2151]	1
13	SPFCB21/17	SETEPUTE SB53 SD43 SORT STORM4 [2142+2151]	1
14	SPFCB21/18	SETEPUTE SB53 SD46 SORT STORM4 [2142+2151]	1
15	SPFCB21/20	SETEPUTE SB53 SD51 SORT STORM4 [2142+2151]	1
16	SP1529546-TR26	(seat cushion Flo-tech Lite SW 38 x SD 41 Dartex black) [2142+2152]	1
17	SP1529547-TR26	(seat cushion Flo-tech Lite SW 38 x SD 46 Dartex black) [2142+2152]	1
18	SP1529548-TR26	(seat cushion Flo-tech Lite SW 38 x SD 51 Dartex black) [2142+2152]	1

19	SP1529549-TR26	(seat cushion Flo-tech Lite SW 43 x SD 41 Dartex black) [2142+2152]	1
20	SP1529550-TR26	(seat cushion Flo-tech Lite SW 43 x SD 46 Dartex black) [2142+2152]	1
21	SP1529551-TR26	(seat cushion Flo-tech Lite SW 43 x SD 51 Dartex black) [2142+2152]	1
22	SP1529552-TR26	(seat cushion Flo-tech Lite SW 48 x SD 41 Dartex black) [2142+2152]	1
23	SP1529553-TR26	(seat cushion Flo-tech Lite SW 48 x SD 46 Dartex black) [2142+2152]	1
24	SP1529554-TR26	(seat cushion Flo-tech Lite SW 48 x SD 51 Dartex black) [2142+2152]	1
25	SP1529555-TR26	(seat cushion Flo-tech Contour SW 53 x SD 43 Dartex black) [2142+2152]	1
26	SP1529556-TR26	(seat cushion Flo-tech Contour SW 53 x SD 46 Dartex black) [2142+2152]	1
27	SP1529570-TR26	(seat cushion Flo-tech Contour SW 53 x SD 51 Dartex black) [2142+2152]	1
28	SP1529546-TR35	(seat cushion Flo-tech Lite SW 38 x SD 41 black chenille fabric) [2142+2154]	1
29	SP1529547-TR35	(seat cushion Flo-tech Lite SW 38 x SD 46 black chenille fabric) [2142+2154]	1
30	SP1529548-TR35	(seat cushion Flo-tech Lite SW 38 x SD 51 black chenille fabric) [2142+2154]	1
31	SP1529549-TR35	(seat cushion Flo-tech Lite SW 43 x SD 41 black chenille fabric) [2142+2154]	1
32	SP1529550-TR35	(seat cushion Flo-tech Lite SW 43 x SD 46 black chenille fabric) [2142+2154]	1
33	SP1529551-TR35	(seat cushion Flo-tech Lite SW 43 x SD 51 black chenille fabric) [2142+2154]	1
34	SP1529552-TR35	(Seat cushion Flo-tech Lite SW 48 x SD 41 black chenille fabric) [2142+2154]	1
35	SP1529553-TR35	(seat cushion Flo-tech Lite SW 48 x SD 46 black chenille fabric) [2142+2154]	1
36	SP1529554-TR35	(seat cushion Flo-tech Lite SW 48 x SD 51 black chenille fabric) [2142+2154]	1
37	SP1529555-TR35	(seat cushion Flo-tech Contour SW 53 X SD 43 black chenille fabric) [2142+2154]	1
38	SP1529556-TR35	(seat cushion Flo-tech Contour SW 53 x SD 46 black chenille fabric) [2142+2154]	1
39	SP1529570-TR35	(seat cushion Flo-tech Contour SW 53 x SD 51 black chenille fabric) [2142+2154]	1
40	SP1529546	SETEPUTE EASY-A SB38SD41 SORT M ORANG SØM [2142+2153]	1
41	SP1529547	SETEPUTE EASY-A SB38SD46 SORT M ORANG SØM [2142+2153]	1
42	SP1529548	SETEPUTE EASY-A SB38SD51 SORT M ORANG SØM [2142+2153]	1
43	SP1529549	SETEPUTE EASY-A SB43SD41 SORT M ORANG SØM [2142+2153]	1
44	SP1529550	SETEPUTE EASY-A SB43SD46 SORT M ORANG SØM [2142+2153]	1
45	SP1529551	SETEPUTE EASY-A SB43SD51 SORT M ORANG SØM [2142+2153]	1
46	SP1529552	SETEPUTE EASY-A SB48SD41 SORT M ORANG SØM [2142+2153]	1
47	SP1529553	SETEPUTE EASY-A SB48SD46 SORT M ORANG SØM [2142+2153]	1
48	SP1529554	SETEPUTE EASY-A SB48SD51 SORT M ORANG SØM [2142+2153]	1
49	SP1529555	SETEPUTE EASY-A SB53SD43 SORT M ORANG SØM [2142+2153]	1
50	SP1529556	SETEPUTE EASY-A SB53SD46 SORT M ORANG SØM [2142+2153]	1

**(Seat Cushion)**

---

51	SP1529570	SETEPUTE EASY-A SB53 SD5 SORT M ORANG SØM <i>[2142+2153]</i>	1
52	SP1531760	(Seat Cushion MatrX Vi SW 40 x SD 45cm (16" x 18")) <i>[2144]</i>	1
53	SP1531761	(Seat Cushion MatrX Vi SW 45 x SD 45cm (18" x 18")) <i>[2144]</i>	1
54	SP1501113	SETEPUTE 43*43 TDX FLOTEC CONTUR VISCO <i>[2143]</i>	1

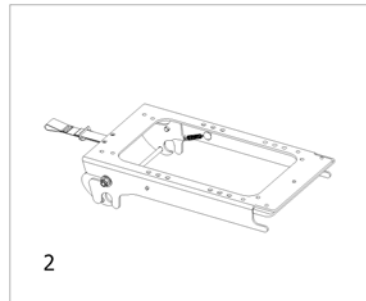

---

1	-	(Tilt 20°)	1
2	-	(Seat Frame Support)	1

# Rygg

SFCBORAZ25 no



---


1	-	(Backrest for Standard Seat)	1
2	-	(Backrest for Flex2 Seat)	1
3	-	(Backrest for Firm Seat)	1
4	-	(Backrest for Max Seat)	1
5	-	(Backrest for Modulite Seat)	1
6	-	(Push handle for Leckey Seat (UK only))	1

---

-	(Backrest Standard Seat, quick release) [0958]	1
-	(Backrest Standard Seat, settable) [0959]	1
-	(Backrest Standard Seat, Powered Recliner) [0947]	1

**(Backrest Standard Seat, quick release)**

1523521PAC.dwg

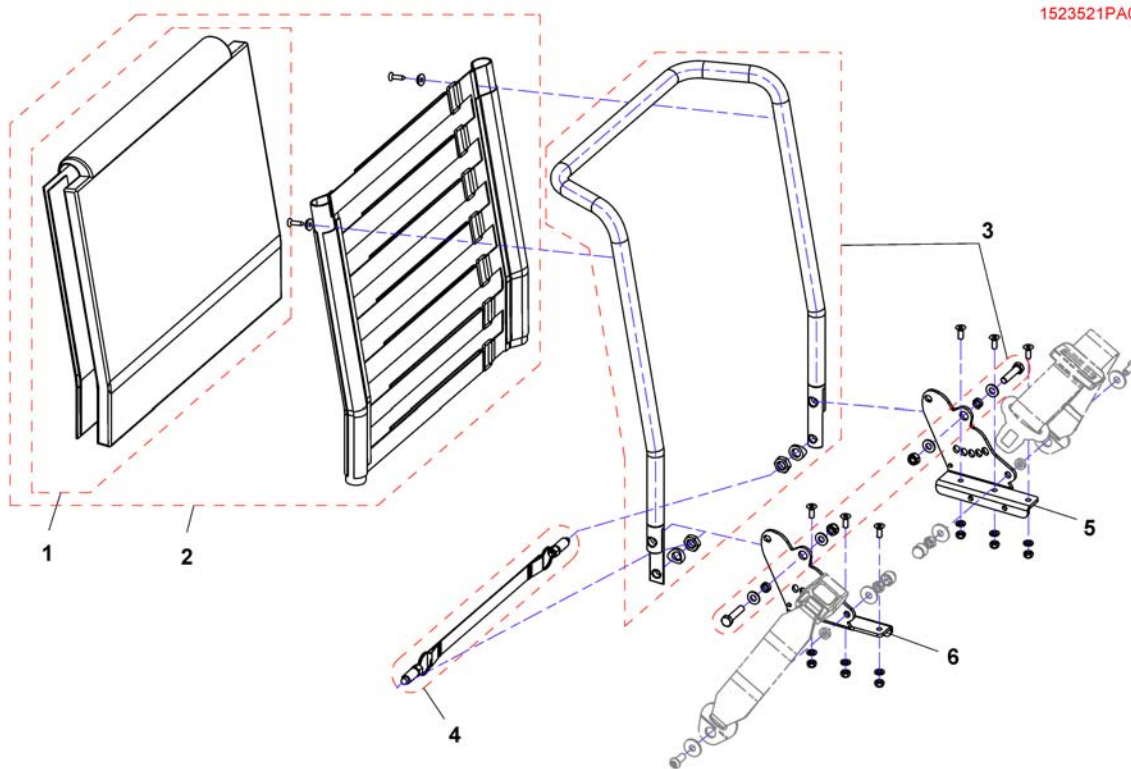


1	1420445	RYGGPUTE BORA	1
1	1420452	RYGGPUTE 54 CM BORA	1
2	1422062	VELCRO RYGG STANDARD SETE STORM3	1
2	1422063	VELCRO RYGG STANDARD SETE RH54 STORM3	1
3	SP1523520	RYGGRØR 480MM BORA	1
3	SP1527953	RYGGRØR 540MM BORA	1
4	SP1528346	(Snapper with Belt)	1
5	SP1523519	RYGGFESTE VE BORA	1
6	SP1523518	RYGGFESTE HØ BORA	1



**(Backrest Standard Seat, settable)**


1422061X.no

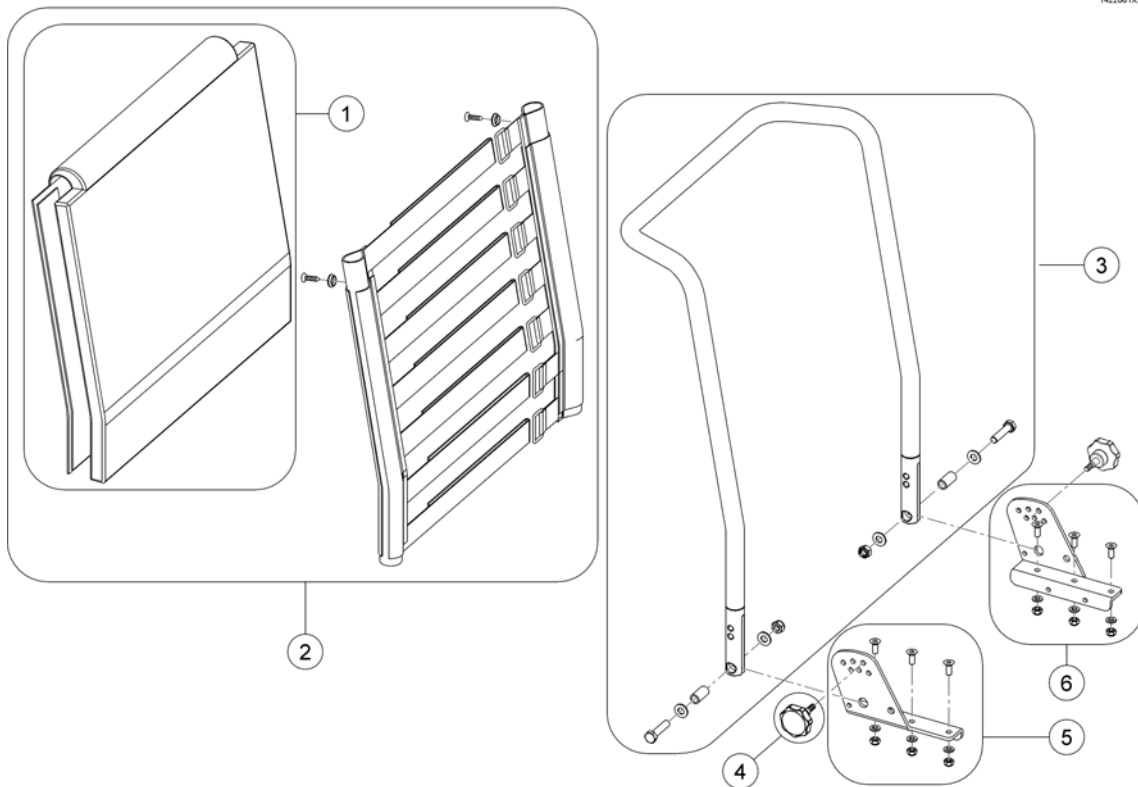


1	1420445	RYGGPUTE BORA	1
1	1420452	RYGGPUTE 54 CM BORA	1
2	1422062	VELCRO RYGG STANDARD SETE STORM3	1
2	1422063	VELCRO RYGG STANDARD SETE RH54 STORM3	1
3	SP1419996	RYGGGRAMME HØ 48CM STORM4	1
3	SP1420465	RYGGGRAMME HØ 54CM STORM4	1
4	SP1420444	FINGERSKRUE M 8 RYGGREG.DRAGON	1
5	SP1541567	(Backrest bracket LH)	1
6	SP1541568	(Backrest bracket RH)	1

# (Backrest Standard Seat, Powered Recliner)

1531008X.dwg

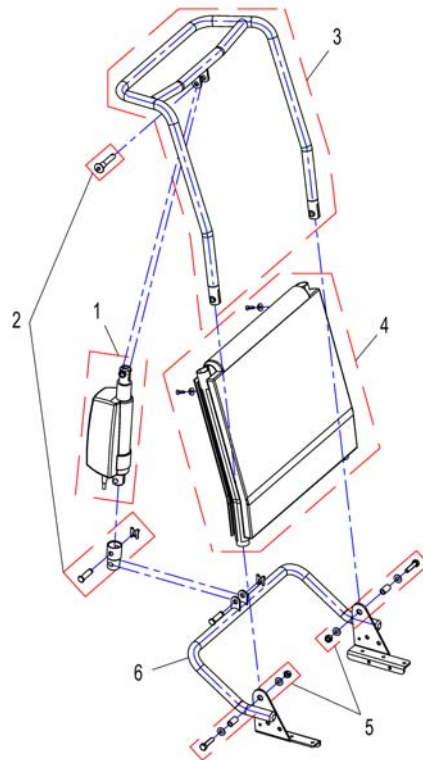


1	SP1533541	MOTOR RYGGREG BORA	1
1	SP1533542	(Actuator Backrest adjustment VR2 & R-NET)	1
2	1446700	MONTERINGSKIT ACTUATOR BORA	1
3	1442864-9005	(Backrest frame for 480 mm height (push handle), adjustable)	1
3	1442866-9005	(Backrest frame for 540 mm height (push handle), adjustable)	1
4	1422062	VELCRO RYGG STANDARD SETE STORM3	1
4	1422063	VELCRO RYGG STANDARD SETE RH54 STORM3	1
5	1446699	MONTERINGSSETT RYGGMOTOR BORA	1
6	1442845-9005	RYGGBØYE NEDRE BORA	1

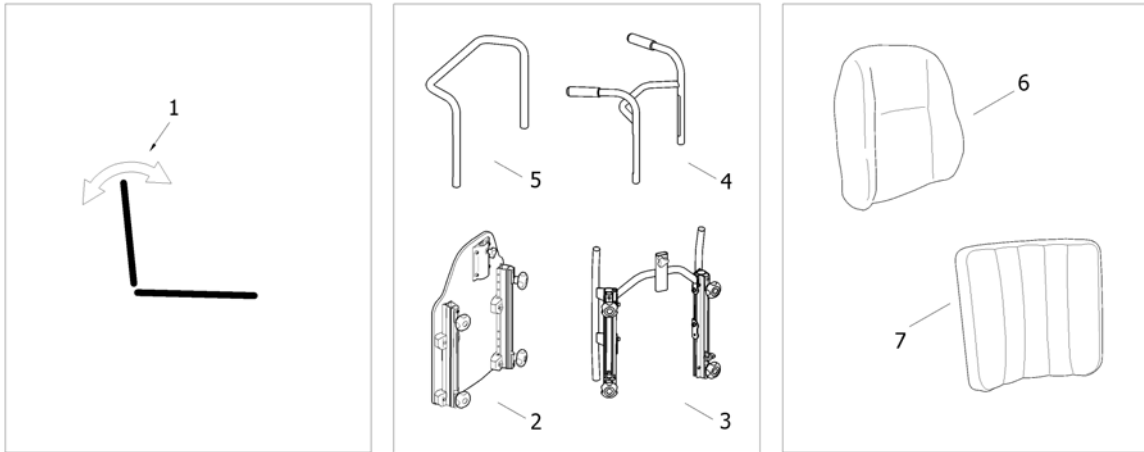


1	-	(Backrest Recliner)	1
2	-	(Backrest Flex2)	1
3	-	(Backrest frame Flex2, tension adjustable)	1
4	-	(Push handles Flex2) [1306]	1
5	-	(Push handles Flex2, attendant control) [1306]	1
6	-	(Backrest cushion)	1
7	-	(Tension adjustable)	1

# (Backrest Recliner)

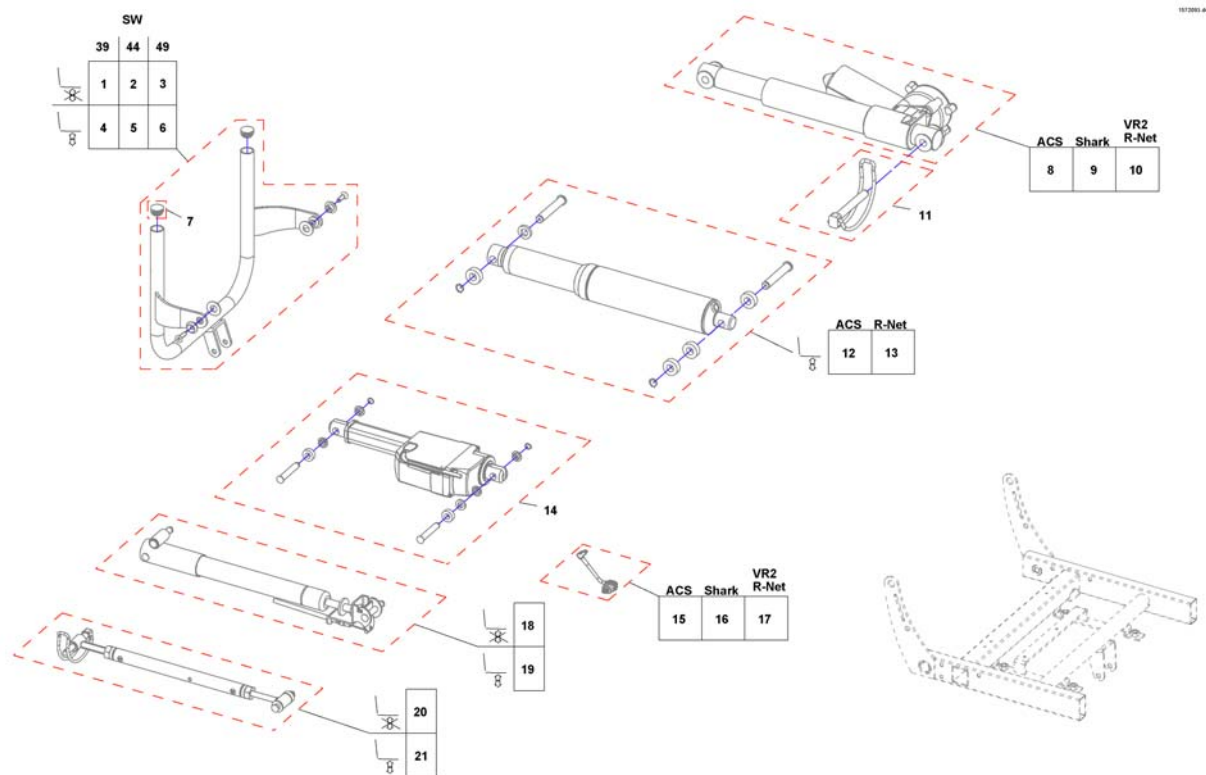


1	SP1526889	RYGGRØR MED FESTER SB39 BORA	1
2	SP1526890	RYGGRØR MED FESTER SB44 BORA	1
3	SP1526891	RYGGRØR MED FESTER SB49 BORA	1
4	SP1531265	(Backrest Support Flex2, SW 39 cm)	1
5	SP1531266	(Backrest Support Flex2, SW 44 cm)	1
6	SP1531267	(Backrest Support Flex2, SW 49 cm)	1
7	L01008	(Tube End Plug SI 22)	1
8	SP1526892	MOTOR RYGGREGULERING BORA	1
9	SP1528267	(Backrest adjustment, powered Shark)	1
10	SP1528264	(Backrest adjustment, powered VR2 / R-NET)	1
11	1439936	LÅSSPLINT 8X60 BORA	1
12	SP1531268	MOTOR RYGGREG BORA	1
13	SP1531269	(Backrest adjustment, powered R-NET)	1
14	SP1552547	(Backrest adjustment, powered, 30°)	1
15	SP1556449	(Cable for Actuator)	1
16	SP1556449	(Cable for Actuator)	1
17	SP1552297	(Cable for actuator)	1
18	SP1526893	GASSFJÆR RYGGREGULERING BORA	1
19	SP1552566	(Backrest adjustment, pneumatic)	1
20	SP1528085	(Backrest adjustment, manual)	1
21	SP1531270	RYGGREGULERING MAN BORA	1

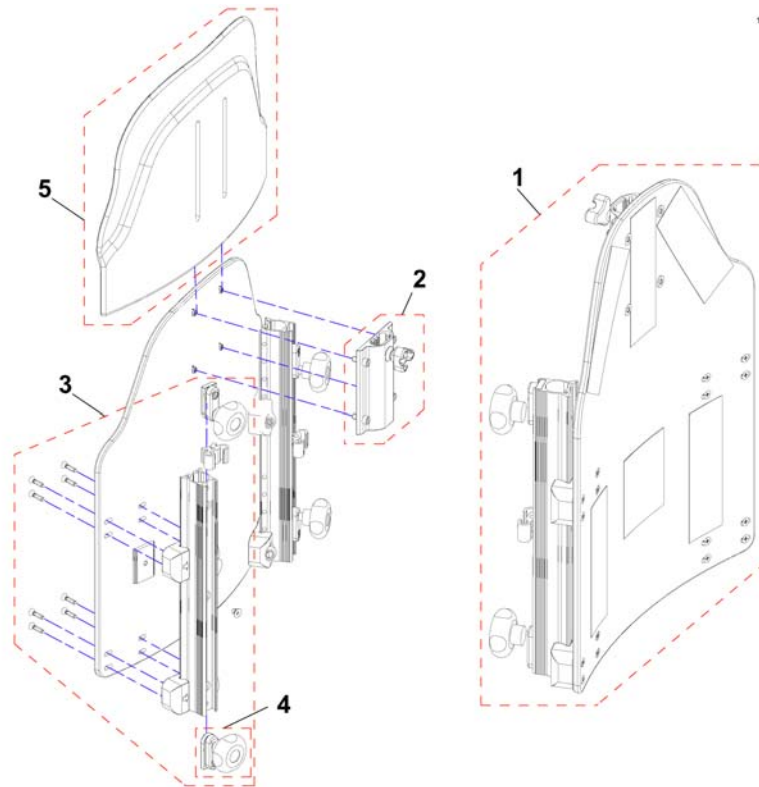


1	1440151	RYGG FLEX 2 292 SPI3 FAST BELLIS AZALEA SPI3	1
1	1440152	RYGGSKJOLD 34,2 KPL MED RYGGSKINNE	1
1	1440153	RYGGSKJOLD 39,2 KPL MED RYGGSKINNE	1
2	1446010	FESTE NAKKE HODESTØTTE KPL BELLIS COMFORT 2	1
3	1446008	RYGGRØR KPL HØ BELLIS NY MODELL	1
3	1446009	RYGGRØR KPL VE BELLIS NY MODELL	1
4	5253088	LÅSKLOSS KPL F RYGGRØR REA COMFORT	1
5	1440147	RYGGFORLENGER 29,0 KPL	1
5	1440148	RYGGFORLENGER 34,2 KPL	1
5	1440149	RYGGFORLENGER 39,2 KPL	1

**(Backrest frame Flex2, tension adjustable)**

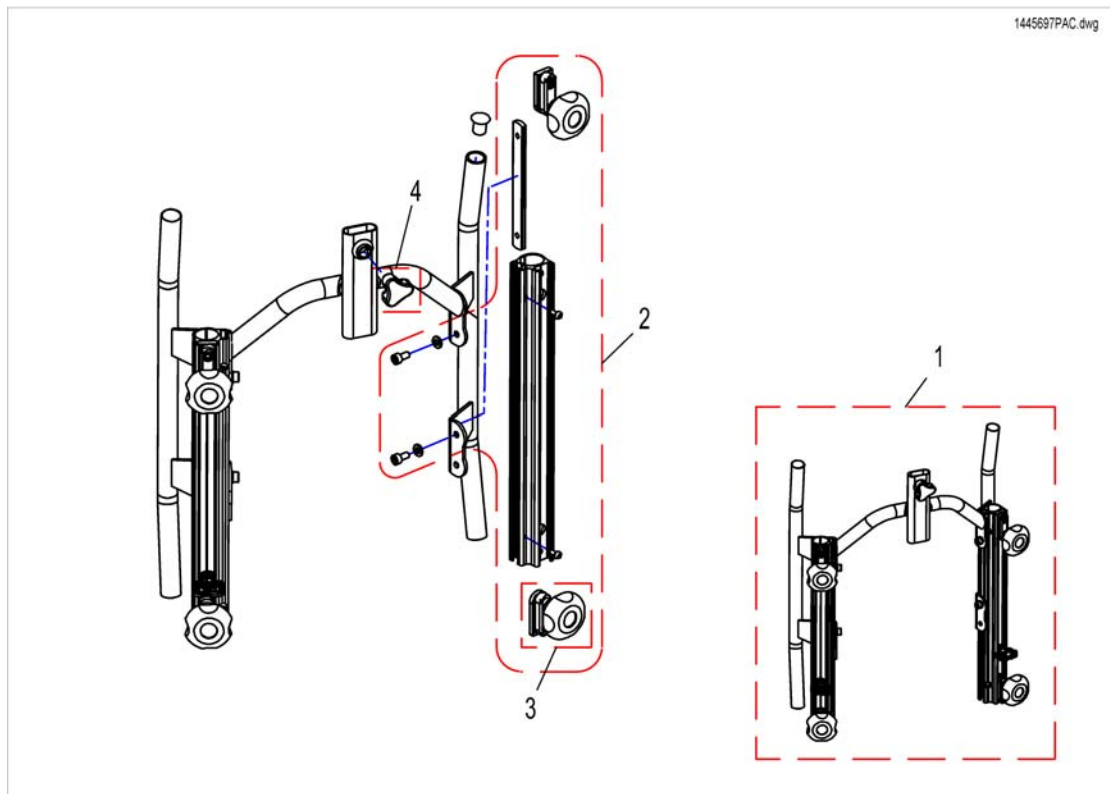

1	1445844-0509	RYGGRAMME KPL 292 BELLIS AZALEA	1
1	1445845-0509	RYGGRAMME KPL 342 BELLIS AZALEA	1
1	1445846-0509	RYGGRAMME KPL 392 BELLIS AZALEA	1
2	1445848	RYGGRØR TIL VELCRO KPL FLEX 3	1
3	5253088	LÅSKLOSS KPL F RYGGRØR REA COMFORT	1
4	1444175	RATT M12 STORM4 MAX / REA	1


1526957PAC.dwg

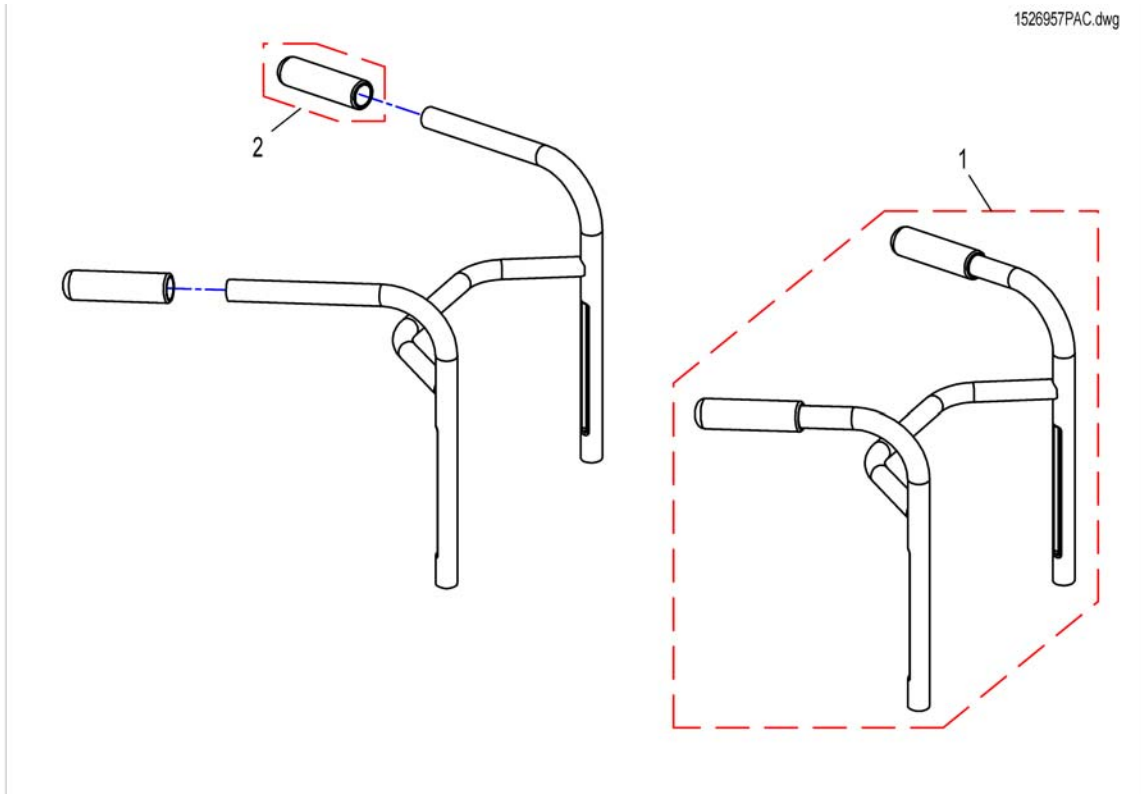

1	SP1526957	KJØREHÅNDTAK SB39 BORA	1
1	SP1526958	KJØREHÅNDTAK 44 BORA	1
1	SP1526959	KJØREHÅNDTAK 49 BORA	1
2	1435421	HÅNDTAK Ø22	1

# (Push handles Flex2, attendant control)

1534551X.dwg

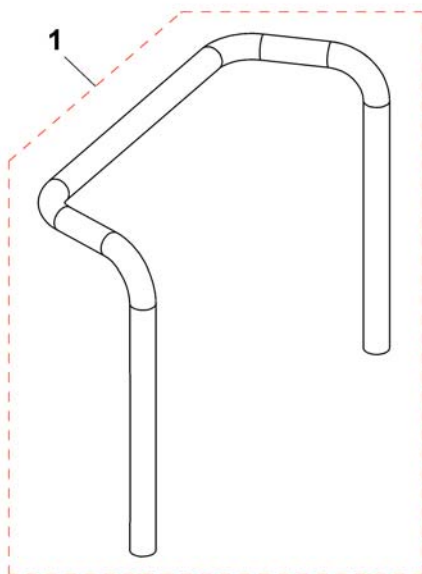



---

1	1425840-9005	KJØREBØYLE FLEX SB39 STORM3	1
1	1425839-9005	KJØREBØYLE FLEX SB43/48 STORM3	1
1	SP1534553	(PUSH HANDLE SB 49)	1



1526905PAC.dwg

**FLEX II**

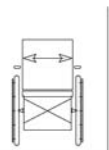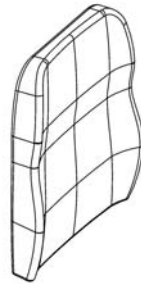


1	37cm	grey plush
2	40cm	
3	45cm	
4	49cm	
5	37cm	Dartex grey
6	40cm	
7	45cm	
8	49cm	


---

1	1444339	RYGGPUTE LAGUNA TR32 372X550 BELLIS	1
2	1435798	RYGGPUTE 41 LAGUNA TR32	1
3	1444340	RYGGPUTE LAGUNA 46 TR32	1
4	1444341	RYGGPUTE LAGUNA 50 TR32	1
5	1427118	RYGGPUTE SB 39 LAGUNA REA C BELLIS ASSIST TR23	1
6	1427119	RYGGPUTE LAGUNA S TR23	1
7	1427120	RYGGPUTE LAGUNA M TR23	1
8	1427121	RYGGPUTE LAGUNA L TR23	1

**(Tension adjustable)**

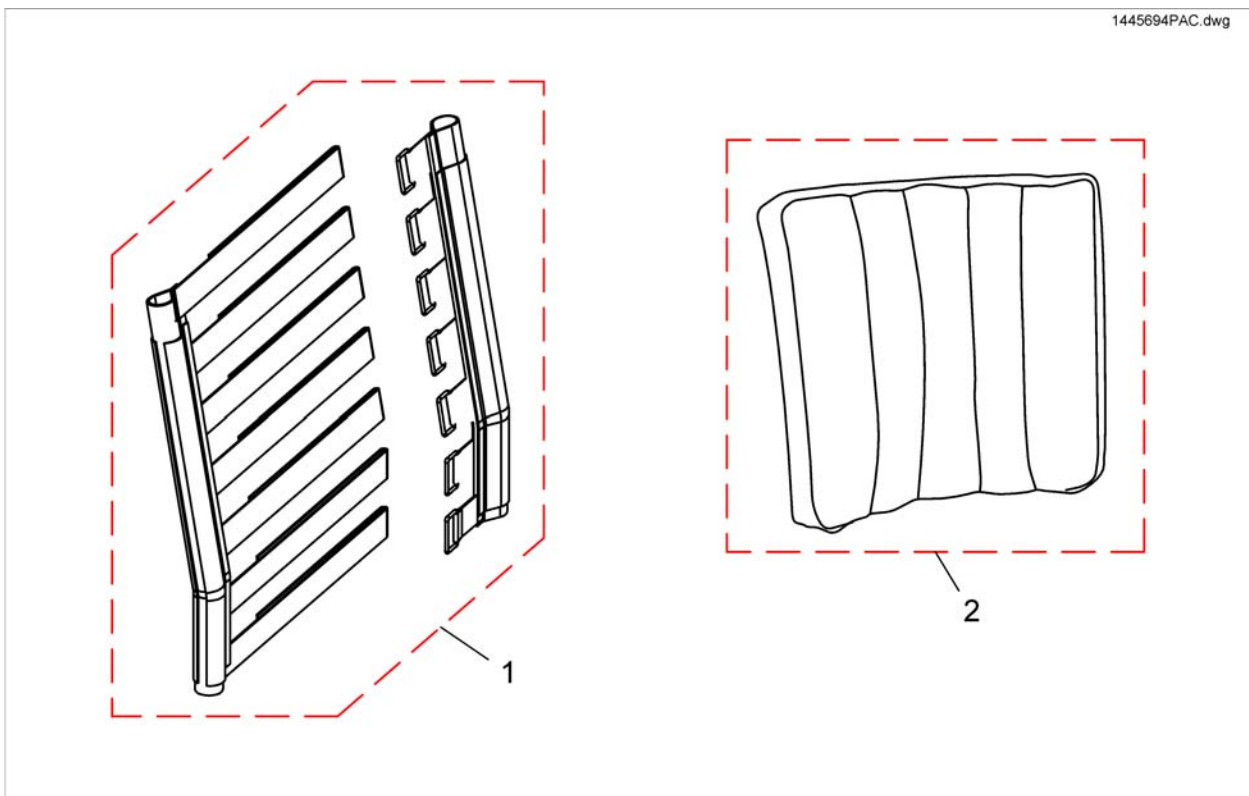


1	1445694	RYGGTREKK BORRELÅS STILL BAR RYGG BELLIS AZALEA	1
2	1444344	RYGGTREKK MED LATERAL XS STØTTE TR32 (450X460)	1
2	1444345	RYGGTREKK MED LATERAL S STØTTE TR32	1
2	1444346	TREKK SIDESTØTTE 52x50 TR32	1
2	1446512	RYGGTREKK THIN04 445 STI LLBAR RYGG BELLIS AZALEA	1
2	1446513	RYGGTREKK THIN04 465 STI LLBAR RYGG BELLIS AZALEA	1
2	1446522	RYGGTREKK THIN04 535 STI LLBAR RYGG BELLIS AZALEA	1
2	1508865	(Cover SW39, dartex grey)	1
2	1508866	(Cover SW44, dartex grey)	1
2	1508867	(Cover SW49, dartex grey)	1

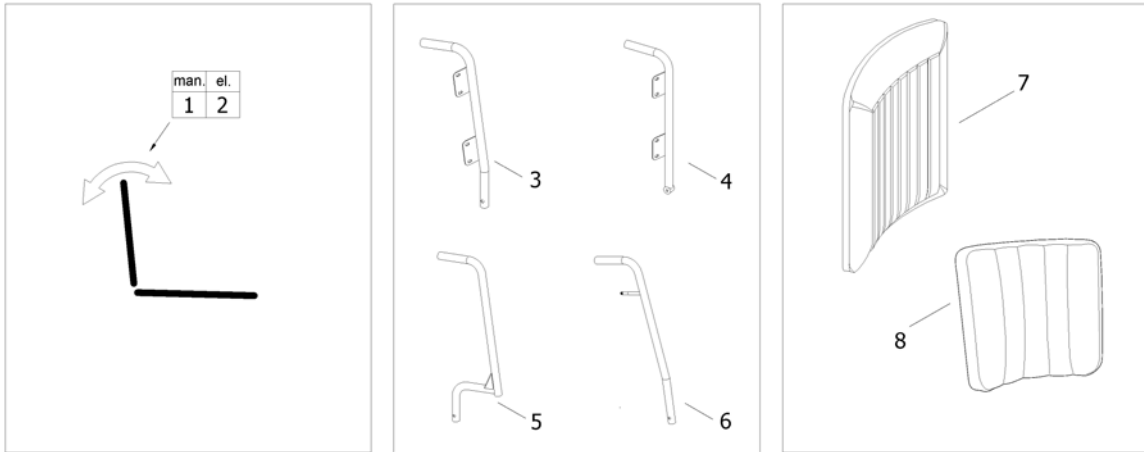

1	-	(Backrest Recliner, manual, Padded Back) [0958]	1
2	-	(Backrest Recliner, powered, Padded Back) [0947]	1
3	-	(Backposts for Padded Back, manual)	1
4	-	(Backposts for Padded Back, powered)	1
5	-	(Flangeless Backpost)	1
6	-	(Backposts for tension adjustable)	1
7	-	(Padded Back)	1
8	-	(Tension adjustable)	1


# (Backrest Recliner, manual, Padded Back)

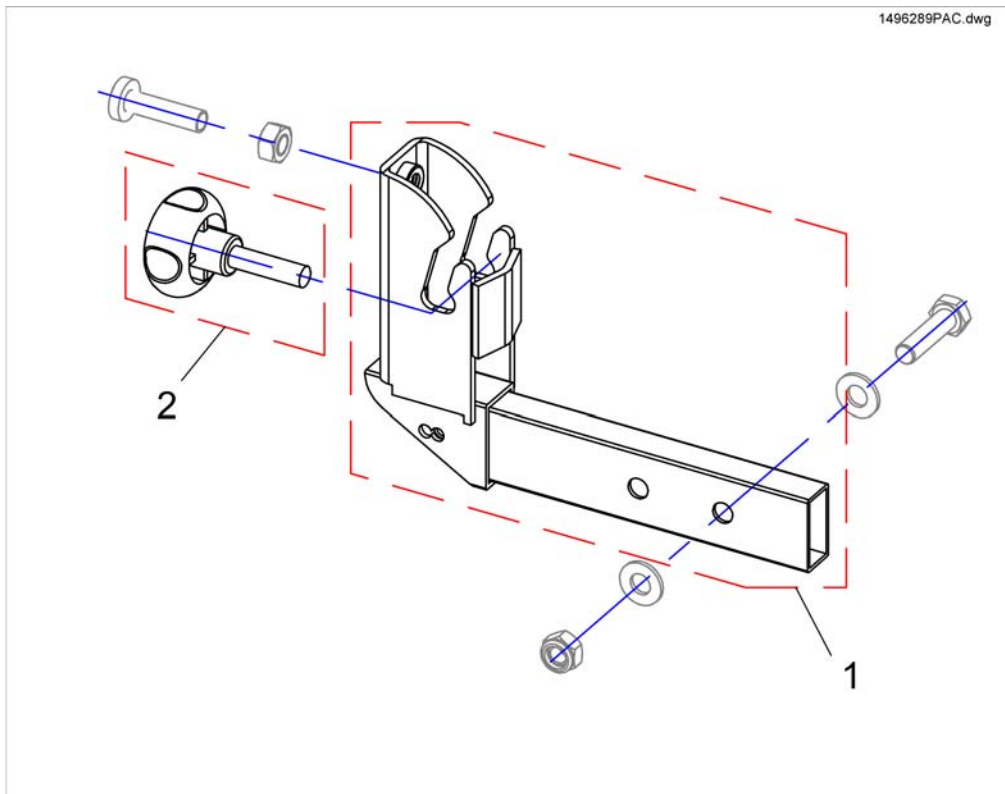

---


1	29000108	FESTE RYGG MAN REG BORA	1
2	90288161	HÅNDRATT RYGG MIRAGE 8X30MM	1

**(Backrest Recliner, powered, Padded Back)**

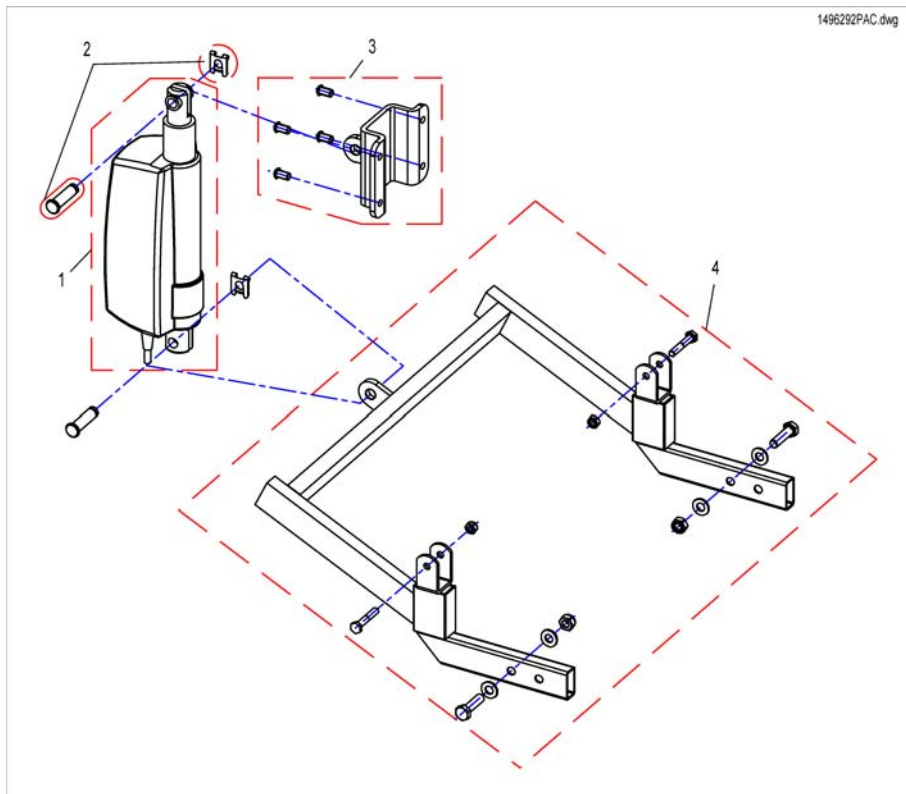


1	SP1558020	(Actuator (ACS, ACS2, Shark))	1
1	SP1558021	(Actuator (R-Net, VR2))	1
2	SP1528368	(Mounting-Kit)	1
3	SP1528381	ADAPTER MOTOR BORA	1
4	SP1528382	(Backpost Bracket SW33)	1
4	SP1528383	(Backpost Bracket SW38)	1
4	SP1528384	RAMMEFESTE RYGMOTOR BORA TDX	1

## (Backposts for Padded Back, manual)

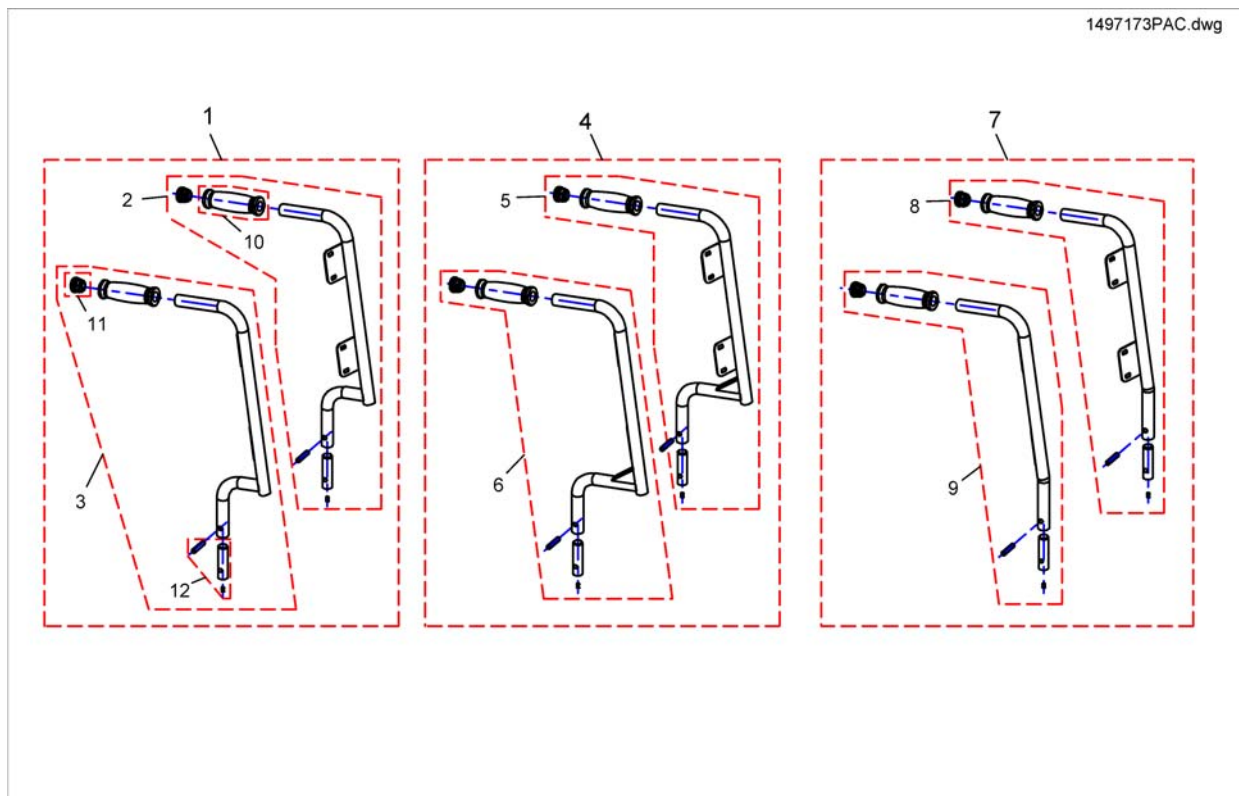


1	SP1497174	(Backposts LH/RH, SD 35/38cm)	1
2	SP1528347	(Backpost LH SD 35/38 cm)	1
3	SP1528348	RYGGRØR MED FESTER HØ BORA	1
4	SP1497173	(Backposts LH/RH, SD 30/33cm)	1
5	SP1528349	RYGGRØR MED FESTER VE BORA	1
6	SP1528350	RYGGRØR MED FESTER HØ BORA	1
7	SP1497175	(Backposts LH/RH, SD 44/48cm)	1
8	SP1528351	RYGGRØR MED FESTER HØ BORA	1
9	SP1528352	RYGGRØR MED FESTER VE BORA	1
10	1435421	HÅNDTAK Ø22	1
11	5816421	(Plug for Ø22 Tube)	1
12	SP1528353	(Mounting-Kit)	1

## (Backposts for Padded Back, powered)

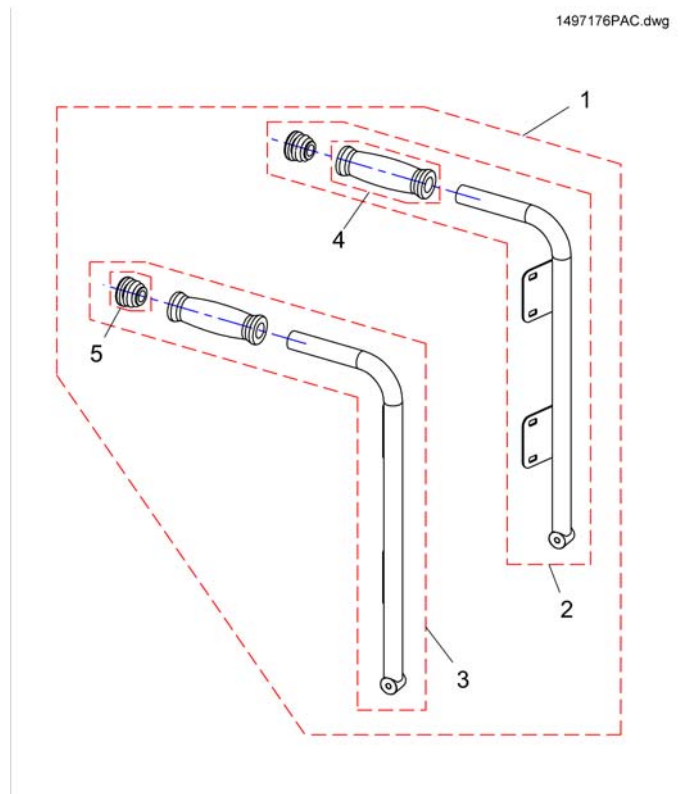

1	SP1497176	(Backposts LH/RH SD 44/48cm, powered)	1
2	SP1528366	(Backpost LH SD 44/48 cm)	1
3	SP1528367	(Backpost RH SD 44/48 cm)	1
4	1435421	HÅNDTAK Ø22	1
5	5816421	(Plug for Ø22 Tube)	1


# (Flangeless Backpost)

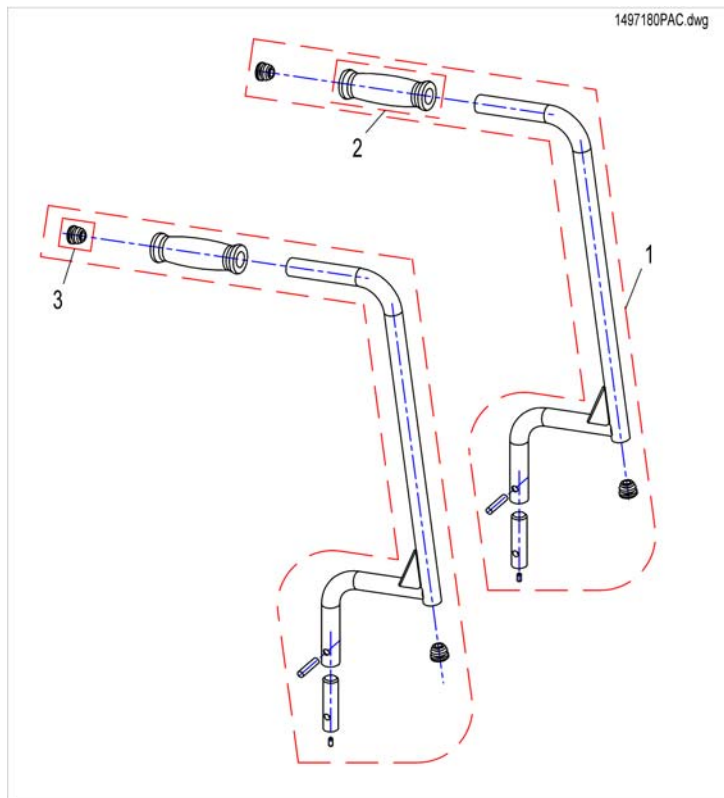

1	SP1528399	SETERØR SD30-33CM BORA	1
1	SP1528400	SETERØR KPL SD35-38CM BORA	1
1	SP1528401	SETERØR KPL SD44-48CM BORA	1
2	1435421	HÅNDTAK Ø22	1
3	L01019	(End Plug 7/8")	1




## (Backposts for tension adjustable)

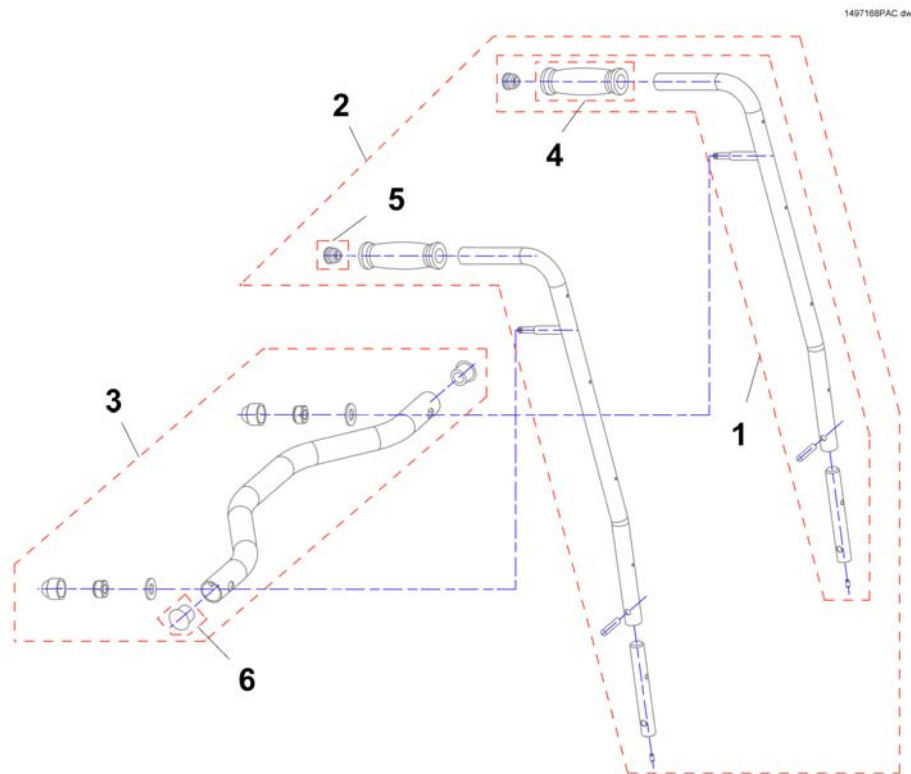


1	SP1528385	SETERØR KPL SD30-33CM BORA	1
1	SP1528386	SETERØR KPL SD35-38CM BORA	1
1	SP1528387	SETERØR KPL SD44-48CM BORA	1
2	SP1497166	SETERØR KPL SD30-33CM BORA	1
2	SP1497167	SETERØR KPL SD35-38CM BORA	1
2	SP1497168	SETERØR KPL SD44-48CM BORA	1
3	SP1497169	(Crossbar SW 33cm)	1
3	SP1497170	(Crossbar SW 38cm)	1
3	SP1497171	(Crossbar SW 43cm)	1
4	1435421	HÅNDTAK Ø22	1
5	L01019	(End Plug 7/8")	1
6	L01049	(TUBE END PLUG SL 18)	1

Invacare® Bora | Rygg  
**(Padded Back)**

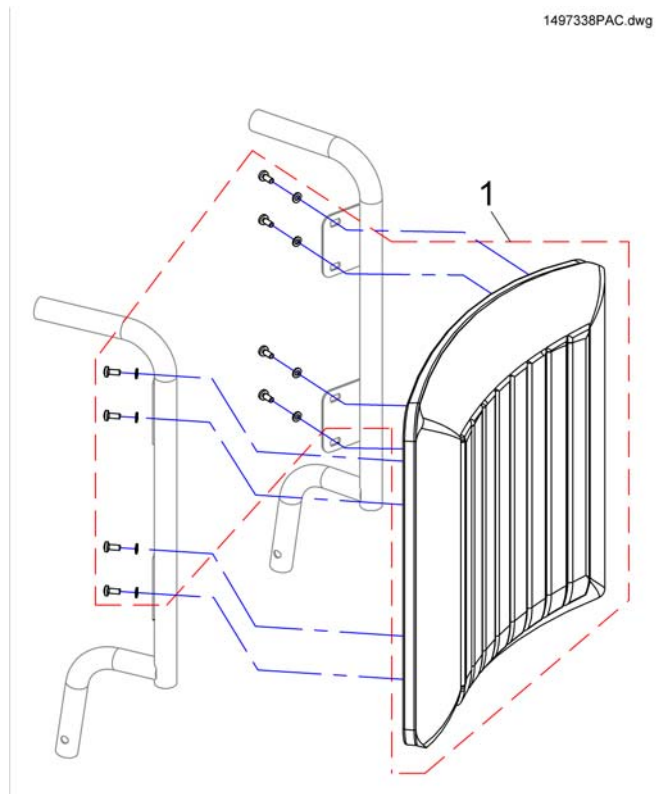

1	SP1497333	(Padded Back SW33 / H43, black Jemima)	1
1	SP1497335	(Padded Back SW38 / H43, black Jemima)	1
1	SP1497336	(Padded Back SW38 / H57, black Jemima)	1
1	SP1497337	(Padded Back SW43 / H43, black Jemima)	1
1	SP1497338	(Padded Back SW43 / H57, black Jemima)	1
1	SP1497339	(Padded Back SW33 / H43, black bird)	1
1	SP1497350	(Padded Back SW38 / H43, black bird)	1
1	SP1497351	(Padded Back SW38 / H57, black bird)	1
1	SP1497352	(Padded Back SW43 / H43, black bird)	1
1	SP1497362	(Padded Back SW43 / H57, black bird)	1

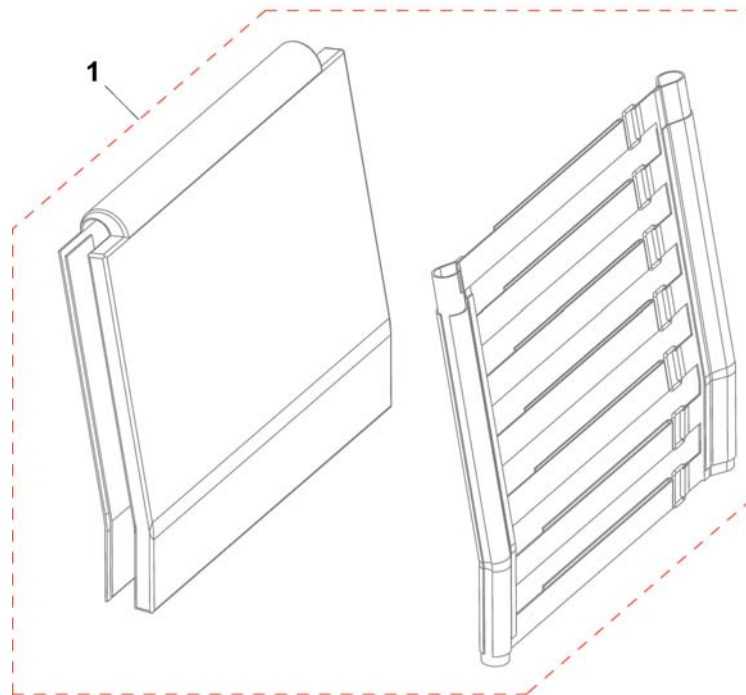


---

1	SP1496979	REGULERBAR RYGG 33 BORA	1
1	SP1497038	REGULERBAR RYGG 38 BORA	1
1	SP1497144	REGULERBAR RYGG 43 BORA	1

## (Backrest for Max Seat)

---


-	(Backrest Recliner)	1
-	(Backrest Max-Seat)	1

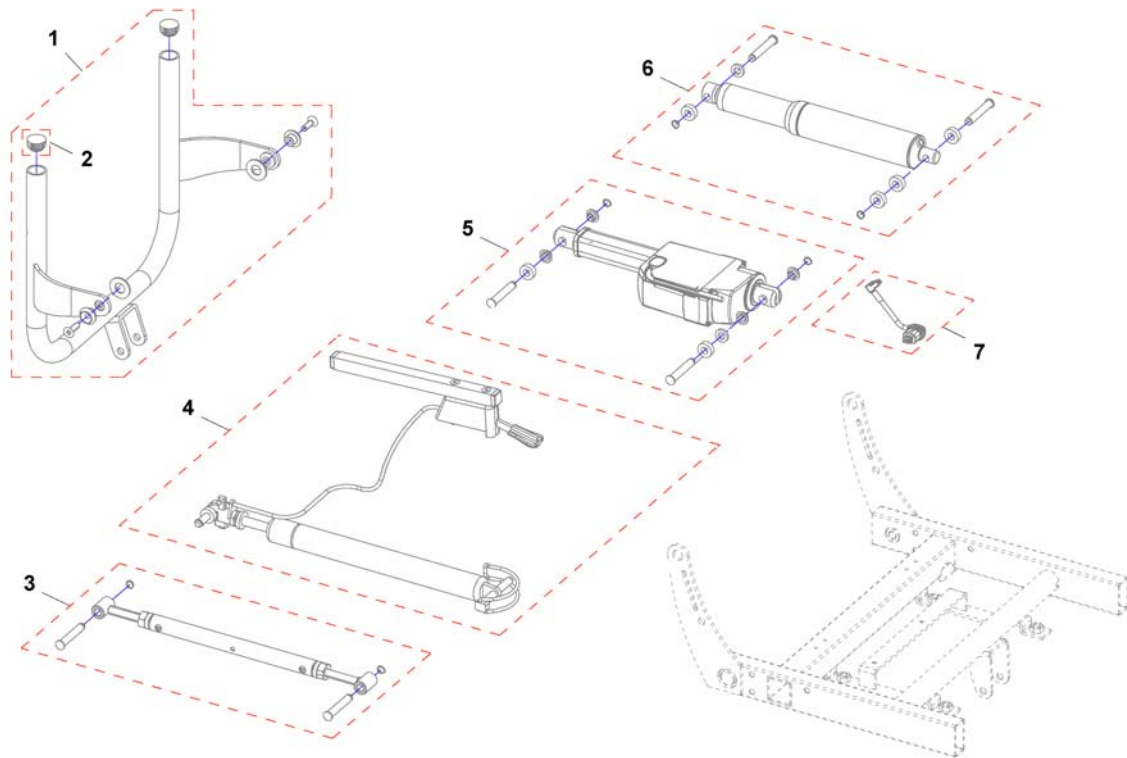


1	SP1551684	(Backrest Support Flex2, SW 55 cm (with and without lifter))	1
2	L01008	(Tube End Plug SI 22)	1
3	SP1531270	RYGGREGULERING MAN BORA [0959]	1
4	SP1552566	(Backrest adjustment, pneumatic)	1
5	SP1552547	(Backrest adjustment, powered, 30°) [0947]	1
6	SP1531268	MOTOR RYGGREG BORA [0947+ACS]	1
6	SP1531269	(Backrest adjustment, powered R-NET) [0947+R-NET]	1
7	SP1556449	(Cable for Actuator) [0947+ACS]	1
7	SP1556448	(Cable for actuator) [0947+R-NET]	1

# (Backrest Max-Seat)

1526824X.dwg

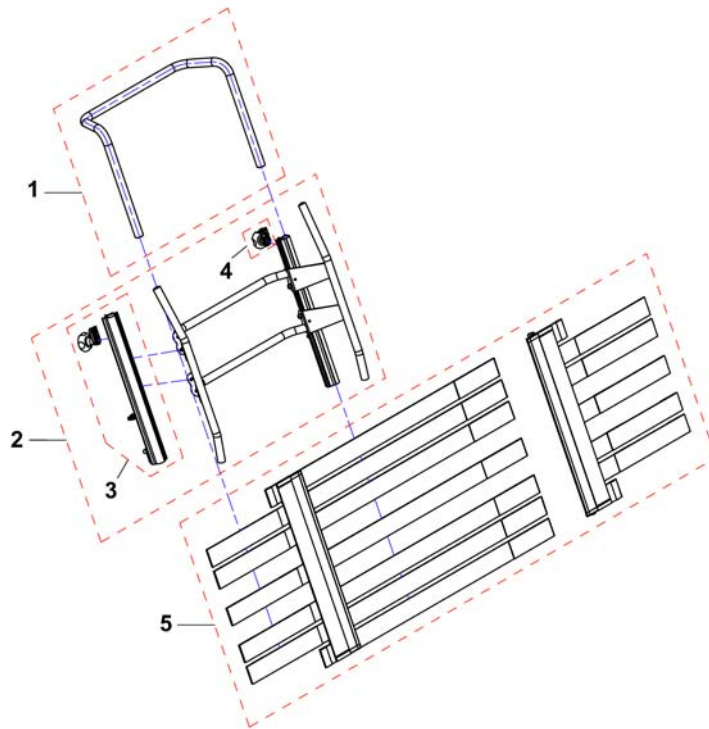


1	SP1543081	(Support tube, SW55)	1
2	SP1526824	(Backrest frame, cpl., SW55)	1
3	1445848	RYGGRØR TIL VELCRO KPL FLEX 3	1
4	5253088	LÅSKLOSS KPL F RYGGRØR REA COMFORT	1
5	SP1526308	(Backrest, tension adjustable, SW 55 - 61cm)	1
-	1526459	(Backrest upholstery SW 55 cm)	1

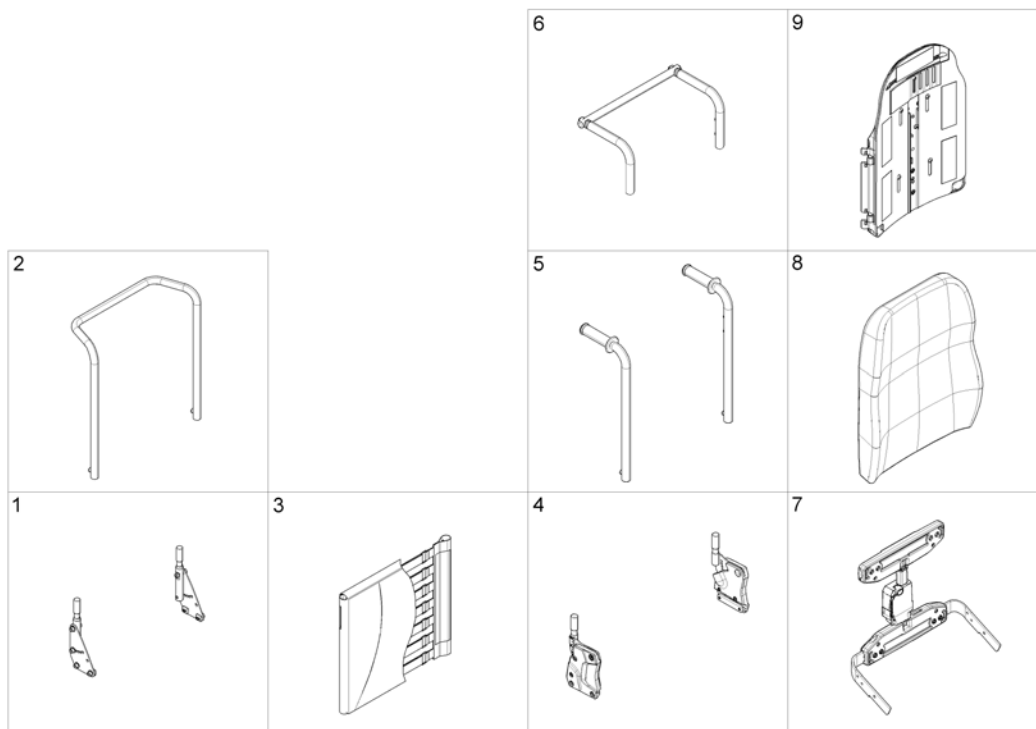

1	-	(Support for Backrest Frame) [0975+2161]	1
2	-	(Backrest Frame) [2161]	1
3	-	(Tension adjustable) [2171]	1
4	-	(Backrest support)	1
5	-	(Backposts) [2162/2163]	1
6	-	(Connection tube)	1
7	-	(Back Recliner / Stabilization bracket)	1
8	-	(Backrest Cushion)	1
9	-	(Flex3 adjustable Backrest Plate) [2172/2173]	1

# (Support for Backrest Frame)

1571533.no

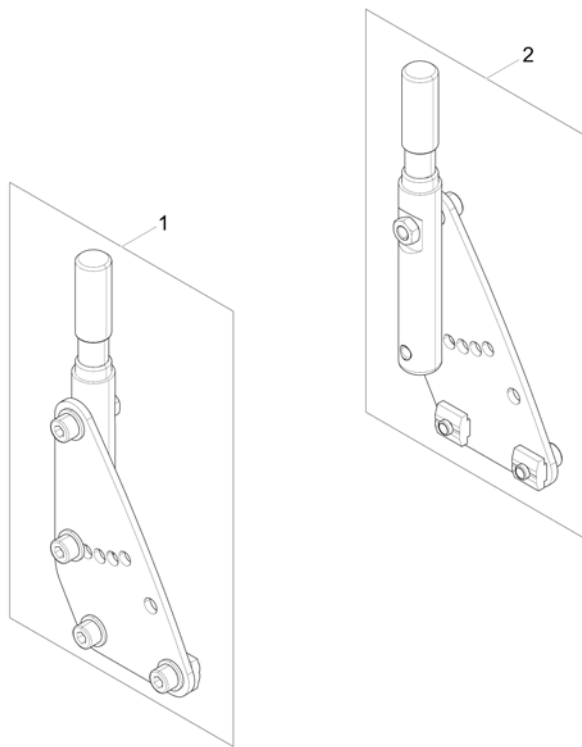


---

1	SP1571542	(Backrest Support RH)	1
2	SP1571553	(Backrest Support LH)	1



1571831.no

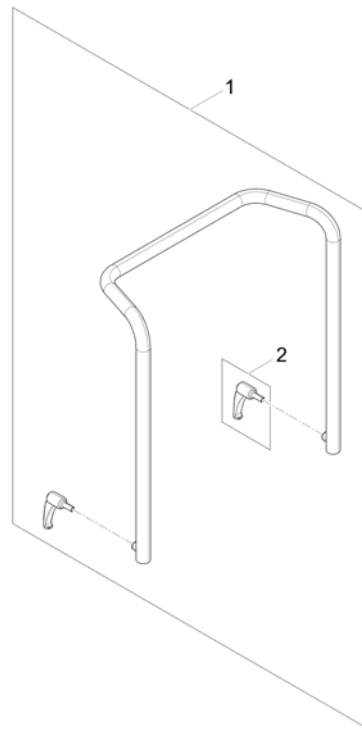


---

1	SP1567810	(Backrest Frame)	1
2	0021-70002	(Indexable lever 63mm M8x16)	1


**(Tension adjustable)**

1571829.no

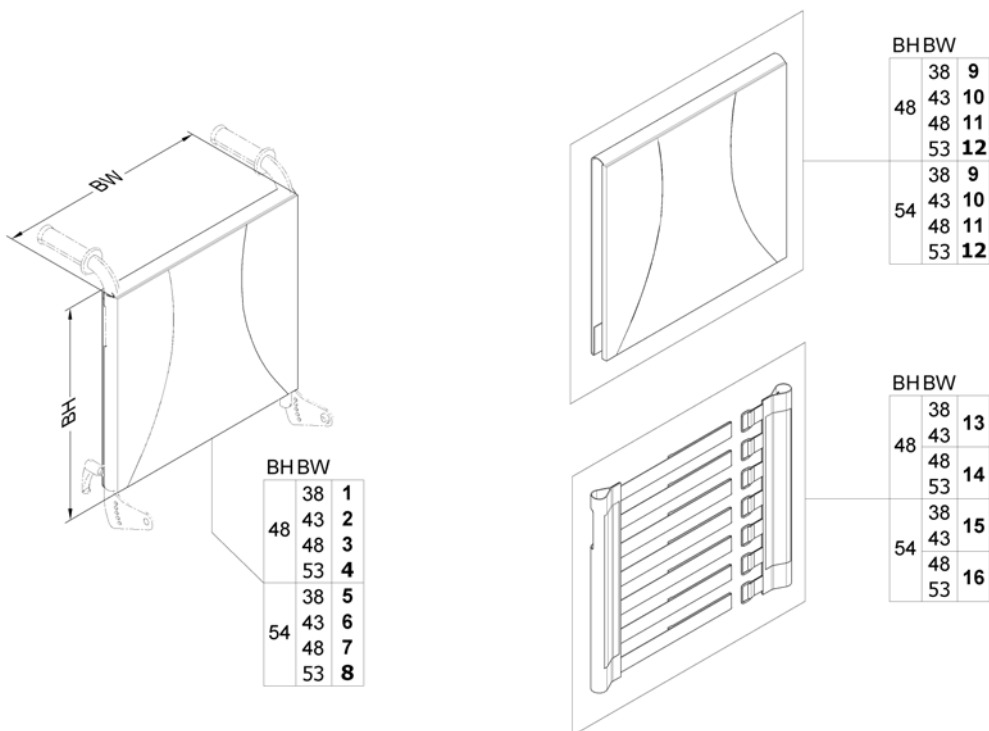


1	SP1567529	(Tension adjustable with Backrest upholstery)	1
2	SP1567465	(Tension adjustable with Backrest upholstery)	1
3	SP1567530	(Tension adjustable with Backrest upholstery)	1
4	SP1567531	(Tension adjustable with Backrest upholstery)	1
5	SP1567627	(Tension adjustable with Backrest upholstery)	1
6	SP1567746	(Tension adjustable with Backrest upholstery)	1
7	SP1567747	(Tension adjustable with Backrest upholstery)	1
8	SP1567748	(Tension adjustable with Backrest upholstery)	1
9	SP1563563	(Backrest upholstery)	1
10	SP1563653	(Backrest upholstery)	1
11	SP1563654	(Backrest upholstery)	1
12	SP1563655	(Backrest upholstery)	1
13	SP1571492	(Sling back)	1
14	SP1571493	(Sling back)	1
15	SP1571494	(Sling back)	1
16	SP1571510	(Sling back)	1

1572928.000

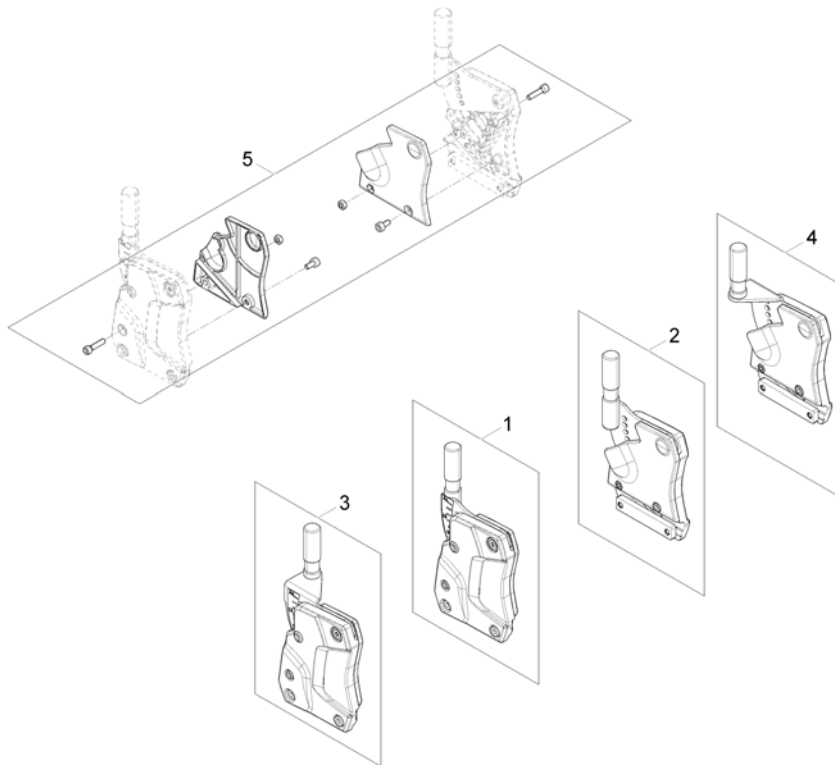


1	SP1571560	(Backrest support RH)	1
2	SP1571561	(Backrest support LH)	1
3	SP1571568	(Backrest support SW 53 RH)	1
4	SP1571569	(Backrest support SW 53 LH)	1
5	SP1571676	(Cover)	1

Invacare® Bora | Rygg  
**(Backposts)**

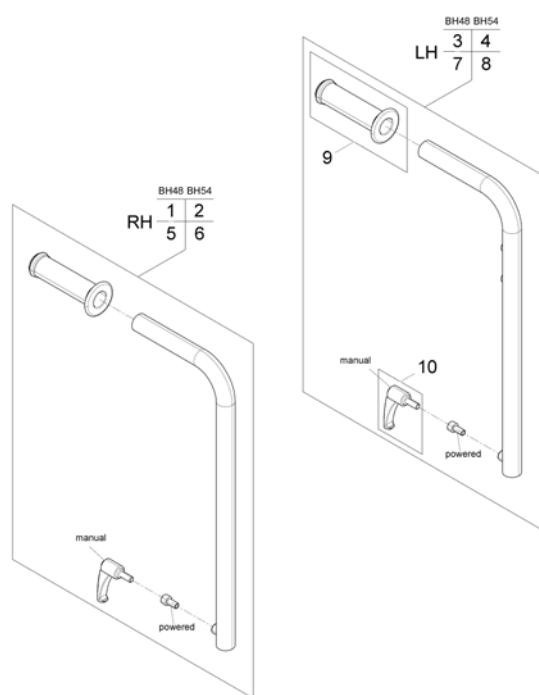

1	SP1563102	(Backpost, powered, height 48cm, RH)	1
2	SP1563103	(Backpost, powered, height 54cm, RH)	1
3	SP1560576	(Backpost, powered, height 48cm, LH)	1
4	SP1559578	(Backpost, powered, height 54cm, LH)	1
5	SP1572489	(Backpost, manual, height 48cm, RH)	1
6	SP1572490	(Backpost, manual, height 54cm, RH)	1
7	SP1572491	(Backpost, manual, height 48cm, LH)	1
8	SP1572492	(Backpost, manual, height 54cm, LH)	1
9	SP1563651	(Handles (pair))	1
10	0021-70002	(Indexable lever 63mm M8x16)	1

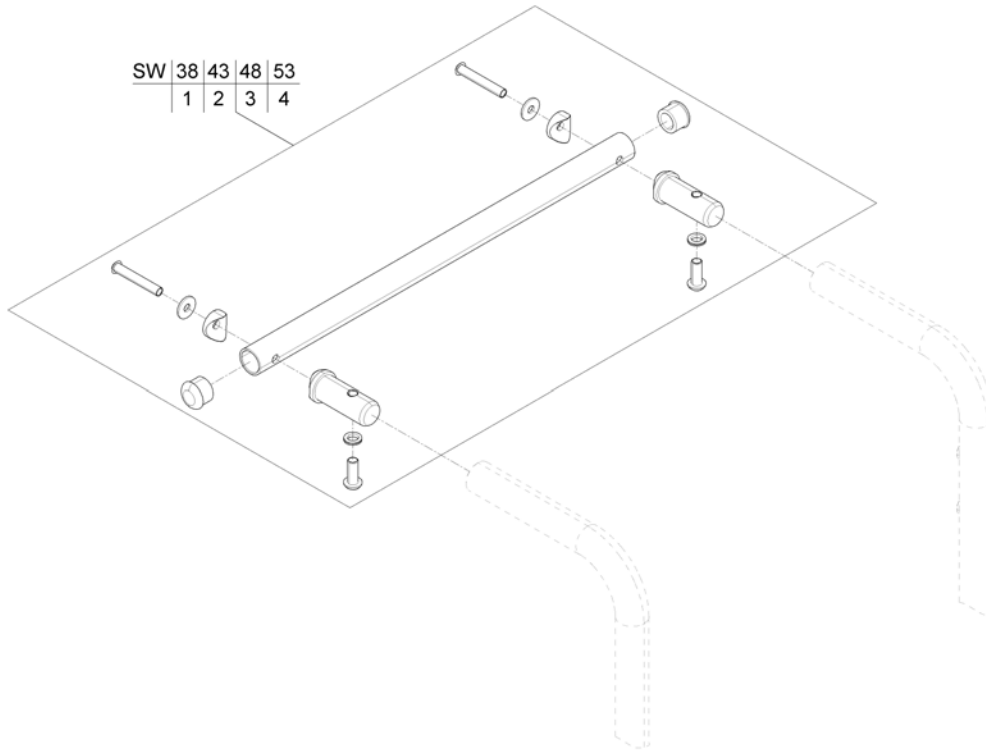


SW	38	43	48	53
	1	2	3	4

---


1	SP1570370	(Connection tube)	1
2	SP1570371	(Connection tube)	1
3	SP1570372	(Connection tube)	1
4	SP1570373	(Connection tube)	1

# (Back Recliner / Stabilization bracket)

1573111.no

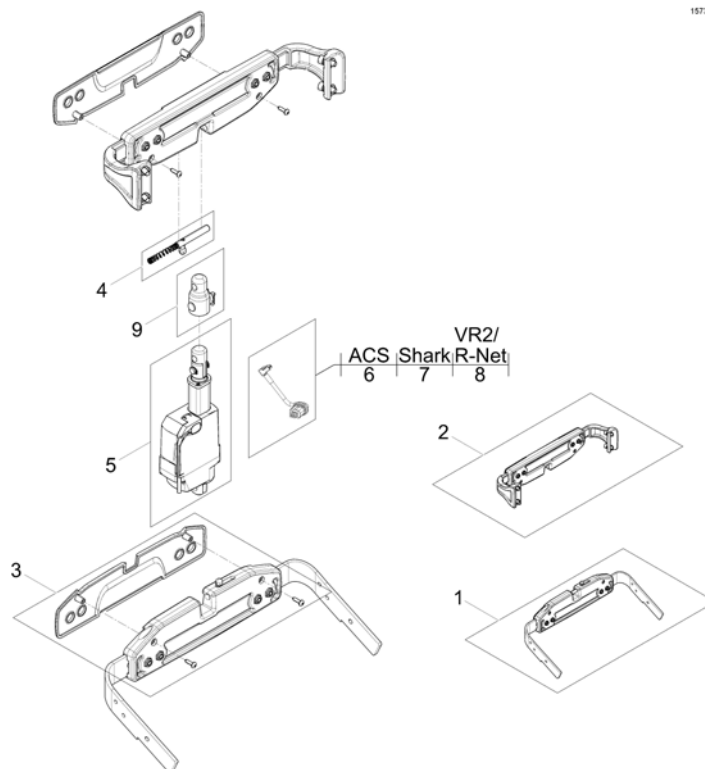

1	SP1567169	(Actuator bracket (pre set to SW 43cm))	1
2	SP1567168	(Stabilization bracket (pre set to SW 43cm))	1
3	SP1571702	(Cover)	1
4	SP1571704	(Locking bolt)	1
5	SP1557820	(Actuator)	1
6	SP1572166	(Actuator Cable)	1
7	SP1552297	(Cable for actuator)	1
8	SP1561401	(Cable for actuator)	1
9	SP1573002	(Safety adapter)	1

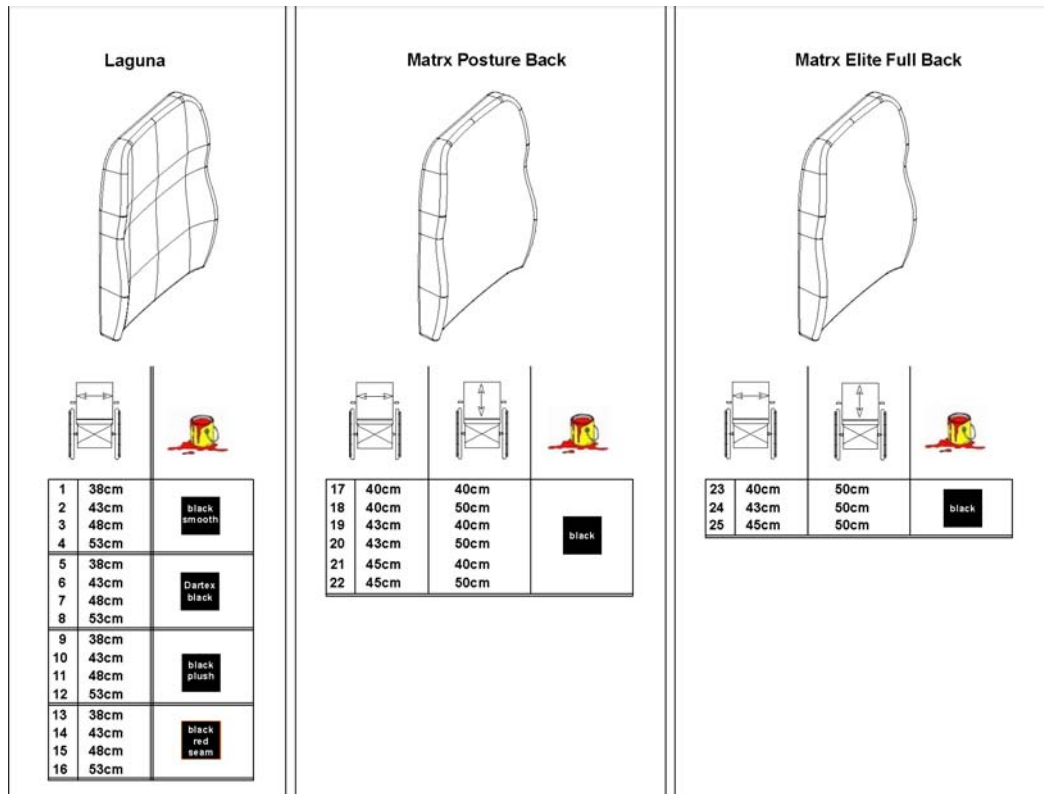

1	SP1529585	RYGGPUTE EASYADAPT SB38 STORM4 M SORT <i>[2173+2151]</i>	1
2	SP1529586	RYGGPUTE EASYADAPT SB43 STORM4 STD SORT <i>[2173+2151]</i>	1
3	SP1529587	RYGGPUTE EASYADAPT SB48 STORM4 SORT <i>[2173+2151]</i>	1
4	SP1529596	RYGGPUTE EASYADAPT SB53 STORM4 M SORT <i>[2173+2151]</i>	1
5	SP1529538-TR26	(Backrest cushion Laguna SW 38 Black Dartex) <i>[2173+2152]</i>	1
6	SP1529539-TR26	(Backrest cushion Laguna SW 43 Black Dartex) <i>[2173+2152]</i>	1
7	SP1529540-TR26	(Backrest cushion Laguna SW 48 Black Dartex) <i>[2173+2152]</i>	1
8	SP1529545-TR26	(Backrest cushion Laguna SW 53 Black Dartex) <i>[2173+2152]</i>	1
9	SP1529538-TR35	(Backrest cushion Laguna SW 38 Black Plush) <i>[2173+2154]</i>	1
10	SP1529539-TR35	(Backrest cushion Laguna SW 43 Black Plush) <i>[2173+2154]</i>	1
11	SP1529540-TR35	(Backrest cushion Laguna SW 48 Black Plush) <i>[2173+2154]</i>	1
12	SP1529545-TR35	(Backrest cushion Laguna SW 53 Black Plush) <i>[2173+2154]</i>	1
13	SP1529538	RYGGPUTE LAGUNA 37CM SORT M ORANG SØM <i>[2173+2153]</i>	1
14	SP1529539	RYGGPUTE LAGUNA 43CM SORT M ORANG SØM <i>[2173+2153]</i>	1
15	SP1529540	RYGGPUTE LAGUNA 48CM SORT M ORANG SØM <i>[2173+2153]</i>	1
16	SP1529545	RYGGPUTE LAGUNA 50CM SORT M ORANG SØM <i>[2173+2153]</i>	1
17	SP1531762	(Matrx "Posture Back" Back 40x40cm (16" x 16")) <i>[2174]</i>	1
18	SP1531763	(Matrx "Posture Back" Back 40x50cm (16" x 20")) <i>[2175]</i>	1


**(Backrest Cushion)**

---

19	SP1531792	(Matrx "Posture Back" Back 43x40cm (17" x 16")) <i>[2174]</i>	1
20	SP1531793	(Matrx "Posture Back" Back 43x50cm (17" x 20")) <i>[2175]</i>	1
21	SP1531822	(Matrx "Posture Back" Back 45x40cm (18" x 16")) <i>[2174]</i>	1
22	SP1531823	(Matrx "Posture Back" Back 45x50cm (18" x 20")) <i>[2175]</i>	1
23	SP1531824	(Matrx "Elite Full Back" Back 40x50cm (16" x 20")) <i>[2176]</i>	1
24	SP1531831	(Matrx "Elite Full Back" Back 43x50cm (17" x 20")) <i>[2176]</i>	1
25	SP1531832	(Matrx "Elite Full Back" Back 45x50cm (18" x 20")) <i>[2176]</i>	1



## (Flex3 adjustable Backrest Plate)

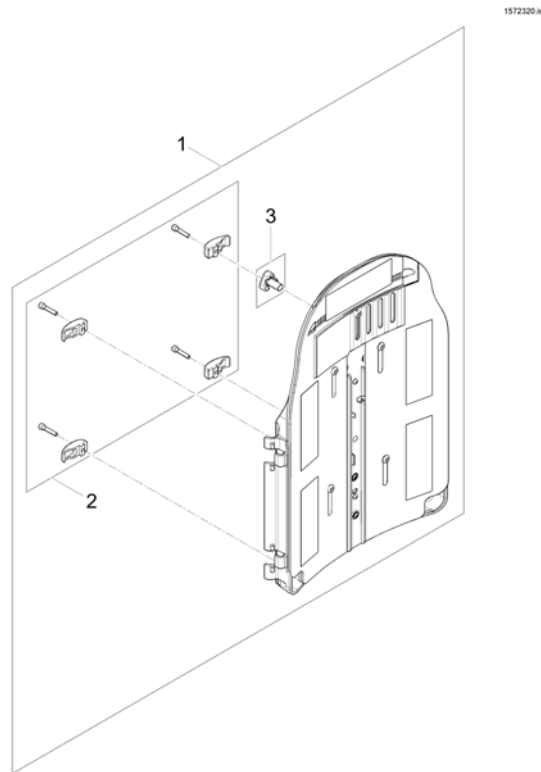



---

1	SP1568489	(Flex3 backrest plate)	1
2	SP1572321	(Clamps)	1
3	1497411	(HANDWHEEL M12 LONG NECK/HEADREST)	1

# (Push handle for Leckey Seat (UK only))

1544884Xiso

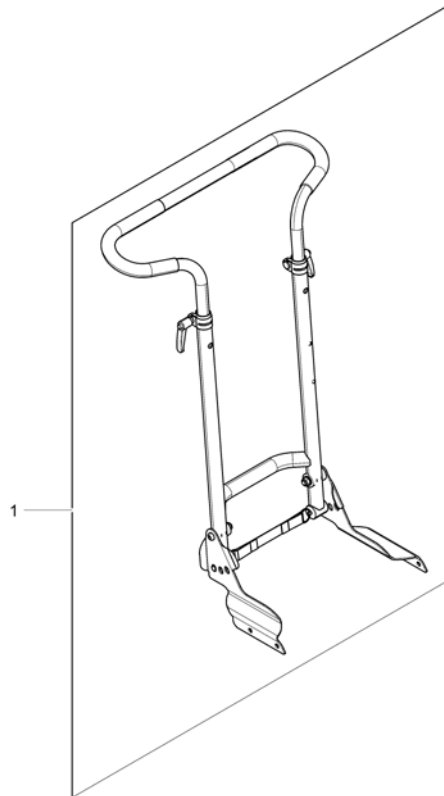


1 SP1544884

(Push handle Leckey seat (UK only))

1

**(Legrest Support, 80° & man. elevating, Standard-seat)**


1441526PAC.dwg

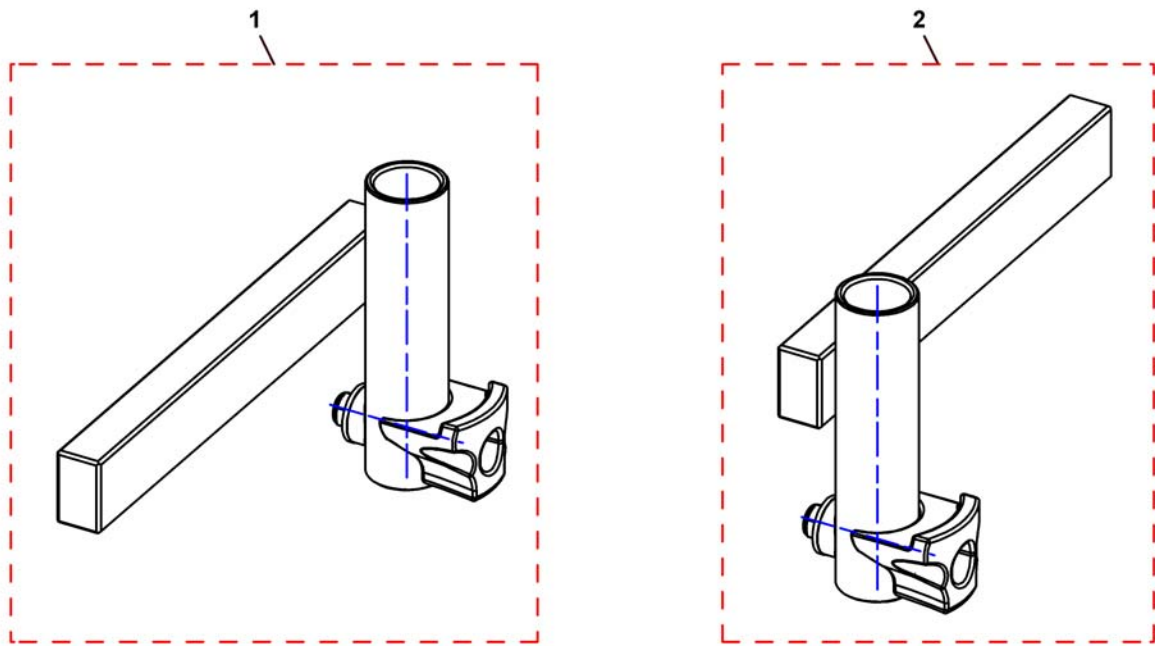

1	SP1531195	ADAPTER BENST KPL VE BORA <i>we recommend to exchange the hangers by pairs for products built before 01.01.12, because of new design.</i>	1
2	SP1531196	ADAPTER BENST KPL HØ BORA <i>we recommend to exchange the hangers by pairs for products built before 01.01.12, because of new design.</i>	1

## (Legrest Support, 80° & man. elevating, Flex2-/Max-Seat)

1527089PAC.dwg

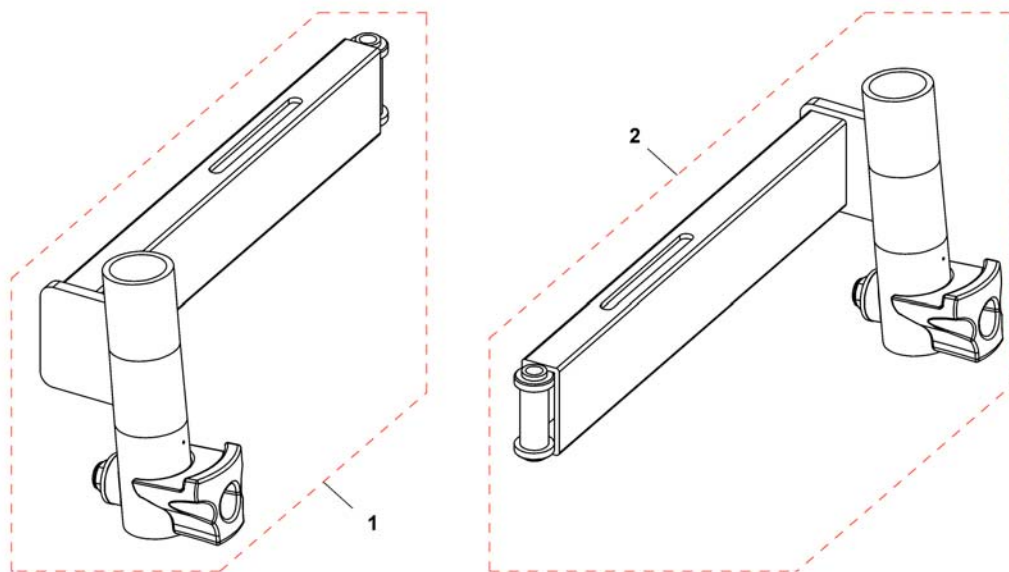


---

1	SP1533502	<b>BENST OVERD FLEX HØ BORA</b> <i>we recommend to exchange the hangers by pairs for products built before 01.01.12, because of new design.</i>	1
2	SP1533503	<b>BENST FESTEFLEX 2 VE BORA</b> <i>we recommend to exchange the hangers by pairs for products built before 01.01.12, because of new design.</i>	1

# (Legrest Support, 80° & man. elevating, Firm-Seat)

1528061PAC.dwg

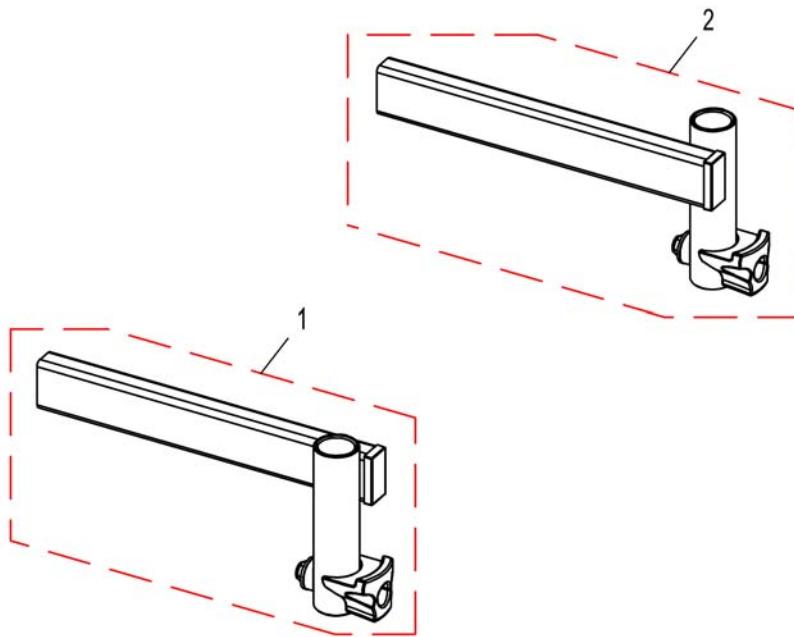


---

1	SP1528061	(Legrest Support for 80° Legrest RH)	1
2	SP1528062	(Legrest Support for 80° Legrest LH)	1

# (Legrest Support, 80° & man. elevating, Modulite Seat)

1441528PAC.dwg

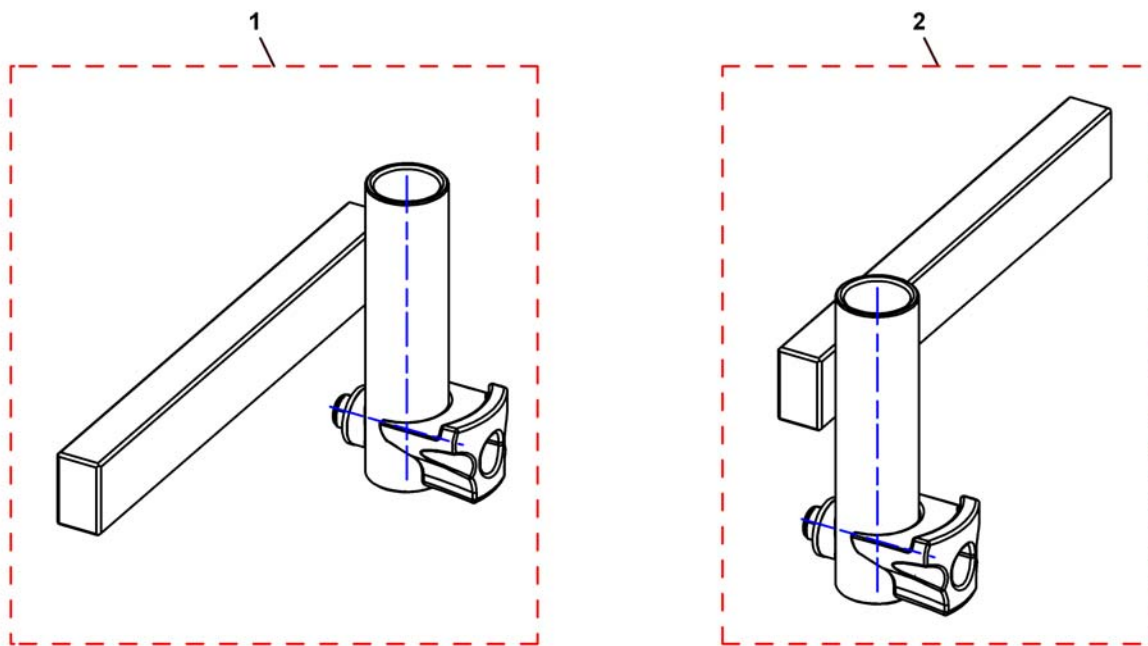


---

1	SP1531195	ADAPTER BENST KPL VE BORA	1
2	SP1531196	ADAPTER BENST KPL HØ BORA	1


# (Legrest Support, Vari series, Standard-Seat)

1562379.dwg

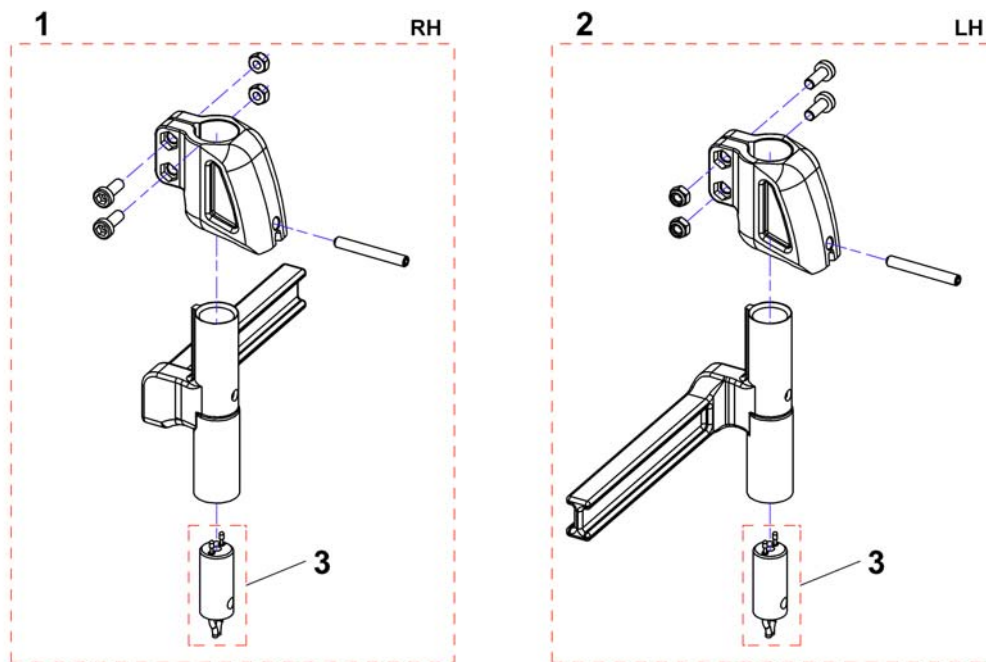

1	SP1449732	BENSTØTTEHOLDER MAN HØ 5CM STORM3 TO TY 010705	1
1	SP1529949	BENSTHOLDER 5CM HØ TDX <i>for ACS</i>	1
1	SP1534878	(Footrest/Legrest receptacle, powered, RH, 5cm) <i>for R-Net without Lifter</i>	1
1	SP1537397	(Footrest/Legrest receptacle, powered, RH, 5cm) <i>for R-Net with Lifter</i>	1
2	SP1449733	BENSTØTTEHOLDER MA VE 5C STORM3 OG 4 TO TY 010705	1
2	SP1529951	BENSTHOLD EL 5CM VE TDX <i>for ACS</i>	1
2	SP1534879	(Footrest/Legrest receptacle, powered, LH, 5cm) <i>for R-Net without Lifter</i>	1
2	SP1537398	(Footrest/Legrest receptacle, powered, LH, 5cm) <i>for R-Net with Lifter</i>	1
3	SP1527775	(Power Supply Cable Legrest (650mm)) <i>for ACS</i>	1
3	SP1533694	(Power Supply Cable Legrest (850mm)) <i>for R-Net without Lifter</i>	1
3	1450232	KABEL EL BENST KORT STORM3 <i>for R-Net with Lifter</i>	1

# (Legrest Support, Vari series, Flex2-/Max-Seat)

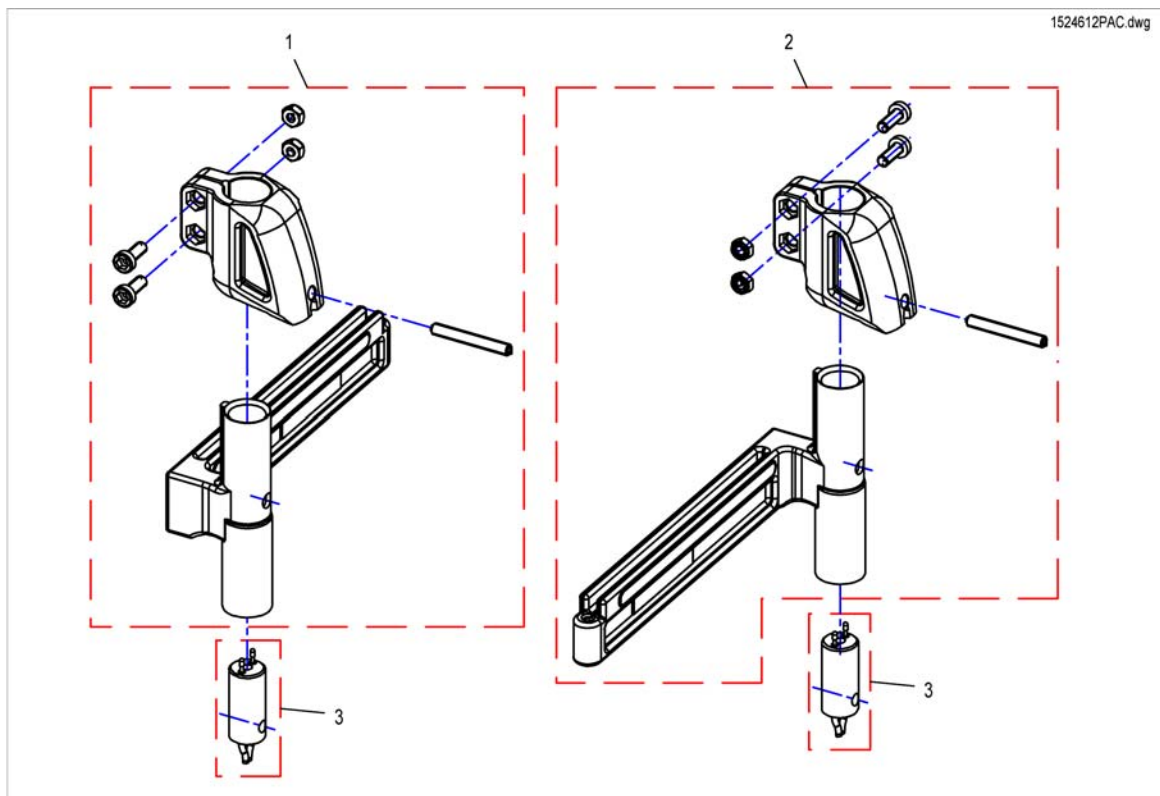


1	SP1526910	FESTE BENST MAN BORA <i>Flex2 &amp; MAX</i>	1
1	SP1524612	HOLDER BENST EL ADE HØ BORA <i>for ACS, Flex2</i>	1
1	SP1555995	(Footrest/Legrest receptacle, powered, RH, 5cm) <i>for ACS, MAX</i>	1
1	SP1534880	(Footrest/Legrest receptacle, powered, RH, 5cm) <i>for R-Net, Flex2 &amp; MAX without Lifter</i>	1
1	SP1537399	(Footrest/Legrest receptacle, powered, RH, 5cm) <i>for R-Net, Flex2 &amp; MAX with Lifter</i>	1
2	SP1526909	FESTE BENST MAN BORA <i>Flex2 &amp; MAX</i>	1
2	SP1524613	HOLDER BENST EL ADE VE BORA <i>for ACS, Flex2</i>	1
2	SP1555996	(Footrest/Legrest receptacle, powered, LH, 5cm) <i>for ACS, MAX</i>	1
2	SP1534881	(Footrest/Legrest receptacle, powered, LH, 5cm) <i>for R-Net, Flex2 &amp; MAX without Lifter</i>	1
2	SP1537400	(Footrest/Legrest receptacle, powered, LH, 5cm) <i>for R-Net, Flex2 &amp; MAX with Lifter</i>	1
3	SP1527775	(Power Supply Cable Legrest (650mm)) <i>for ACS, Flex2 RH</i>	1
3	SP1530795	KABEL I HOLDER F BENST STORM4 BORA TDX <i>for ACS, Max-Seat RH &amp; LH / Flex2 LH</i>	1
3	SP1533694	(Power Supply Cable Legrest (850mm)) <i>for R-Net without Lifter, RH &amp; LH</i>	1
3	1450232	KABEL EL BENST KORT STORM3 <i>for R-Net with Lifter, RH &amp; LH</i>	1



## (Legrest Support, Vari series, Firm-Seat)


1524616PAC.dwg

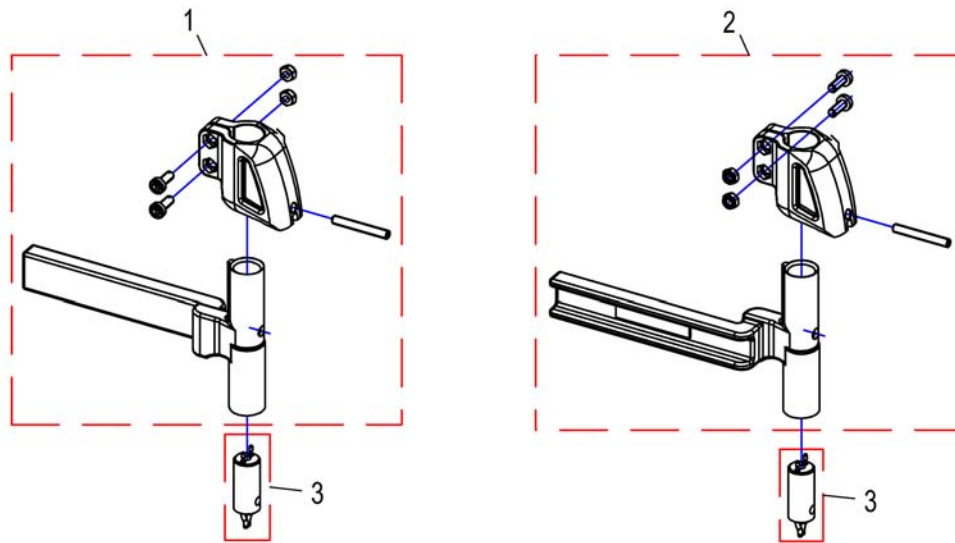

1	SP1526912	(Footrest/Legrest receptacle, manuell, RH, 5cm)	1
1	SP1524616	(Footrest/Legrest receptacle, powered, RH, 5cm) <i>for ACS</i>	1
1	SP1534947	(Footrest/Legrest receptacle, powered, RH, 5cm) <i>for R-Net without Lifter</i>	1
1	SP1537401	(Footrest/Legrest receptacle, powered, RH, 5cm) <i>for R-Net with Lifter</i>	1
2	SP1526911	(Footrest/Legrest receptacle, manuell, LH, 5cm)	1
2	SP1524617	(Footrest/Legrest receptacle, powered, LH, 5cm) <i>for ACS</i>	1
2	SP1534950	(Footrest/Legrest receptacle, powered, LH, 5cm) <i>for R-Net without Lifter</i>	1
2	SP1537402	(Footrest/Legrest receptacle, powered, LH, 5cm) <i>for R-Net with Lifter</i>	1
3	SP1527775	(Power Supply Cable Legrest (650mm)) <i>for ACS</i>	1
3	SP1533694	(Power Supply Cable Legrest (850mm)) <i>for R-Net without Lifter</i>	1
3	1450232	KABEL EL BENST KORT STORM3 <i>for R-Net with Lifter</i>	1


## (Legrest Support, Vari series, Modulite-Seat)

1562379.dwg

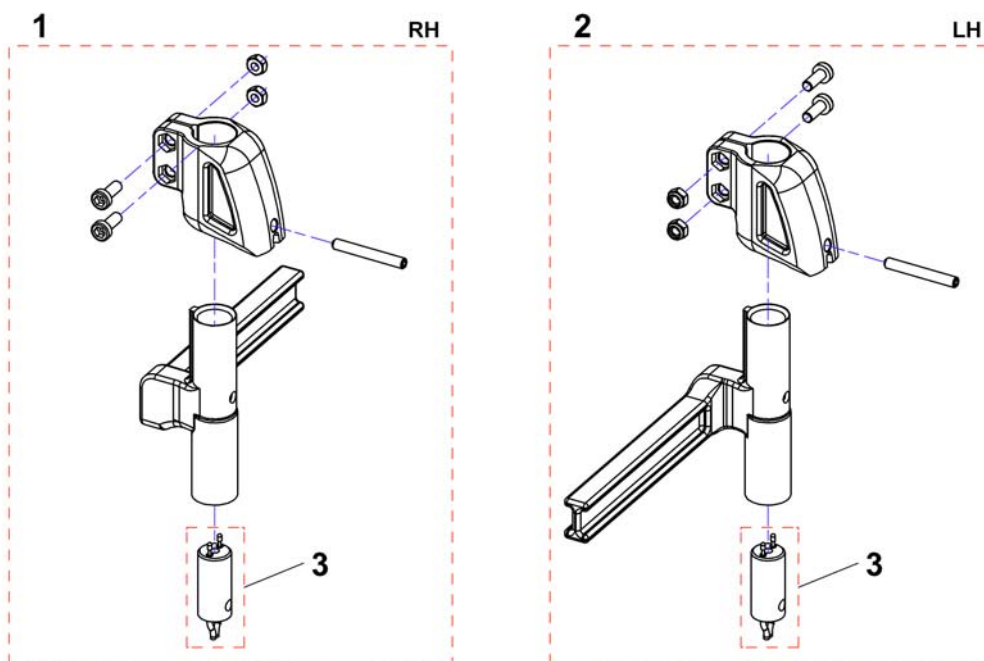

1	SP1449732	BENSTØTTEHOLDER MAN HØ 5CM STORM3 TO TY 010705	1
1	SP1529949	BENSTHOLDER 5CM HØ TDX <i>for ACS</i>	1
1	SP1534878	(Footrest/Legrest receptacle, powered, RH, 5cm) <i>for R-Net without Lifter</i>	1
1	SP1537397	(Footrest/Legrest receptacle, powered, RH, 5cm) <i>for R-Net with Lifter</i>	1
2	SP1449733	BENSTØTTEHOLDER MA VE 5C STORM3 OG 4 TO TY 010705	1
2	SP1529951	BENSTHOLD EL 5CM VE TDX <i>for ACS</i>	1
2	SP1534879	(Footrest/Legrest receptacle, powered, LH, 5cm) <i>for R-Net without Lifter</i>	1
2	SP1537398	(Footrest/Legrest receptacle, powered, LH, 5cm) <i>for R-Net with Lifter</i>	1
3	SP1527775	(Power Supply Cable Legrest (650mm)) <i>for ACS</i>	1
3	SP1533694	(Power Supply Cable Legrest (850mm)) <i>for R-Net without Lifter</i>	1
3	1450232	KABEL EL BENST KORT STORM3 <i>for R-Net with Lifter</i>	1

Invacare® Bora | Benstøtter og holdere  
**(Legrest Support, 70°/90°, Firm-Seat)**

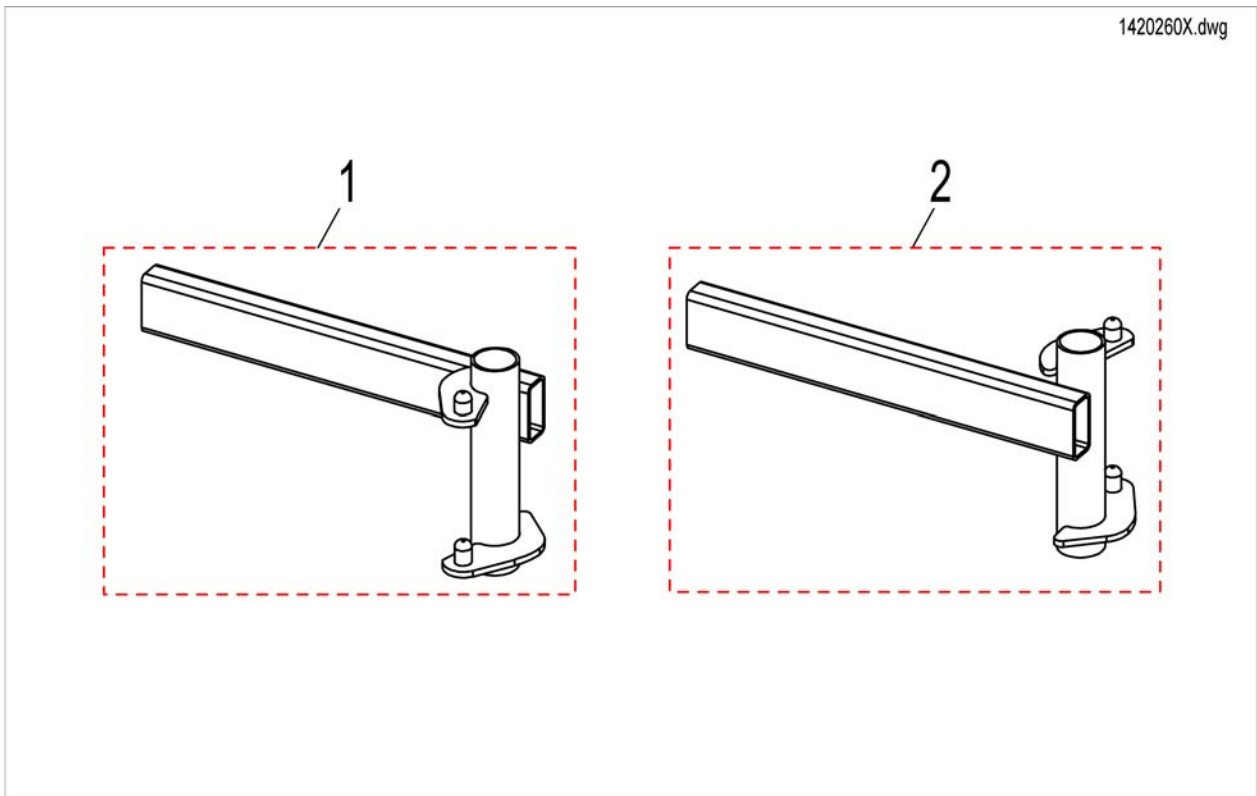


---


1	SP1420260	(Legrest Support for Hanger 70°/90° RH)	1
2	SP1420261	(Legrest Support for Hanger 70°/90° LH)	1

**(Legrest Support, angle adj., Firm-Seat)**

1527122PAC.dwg

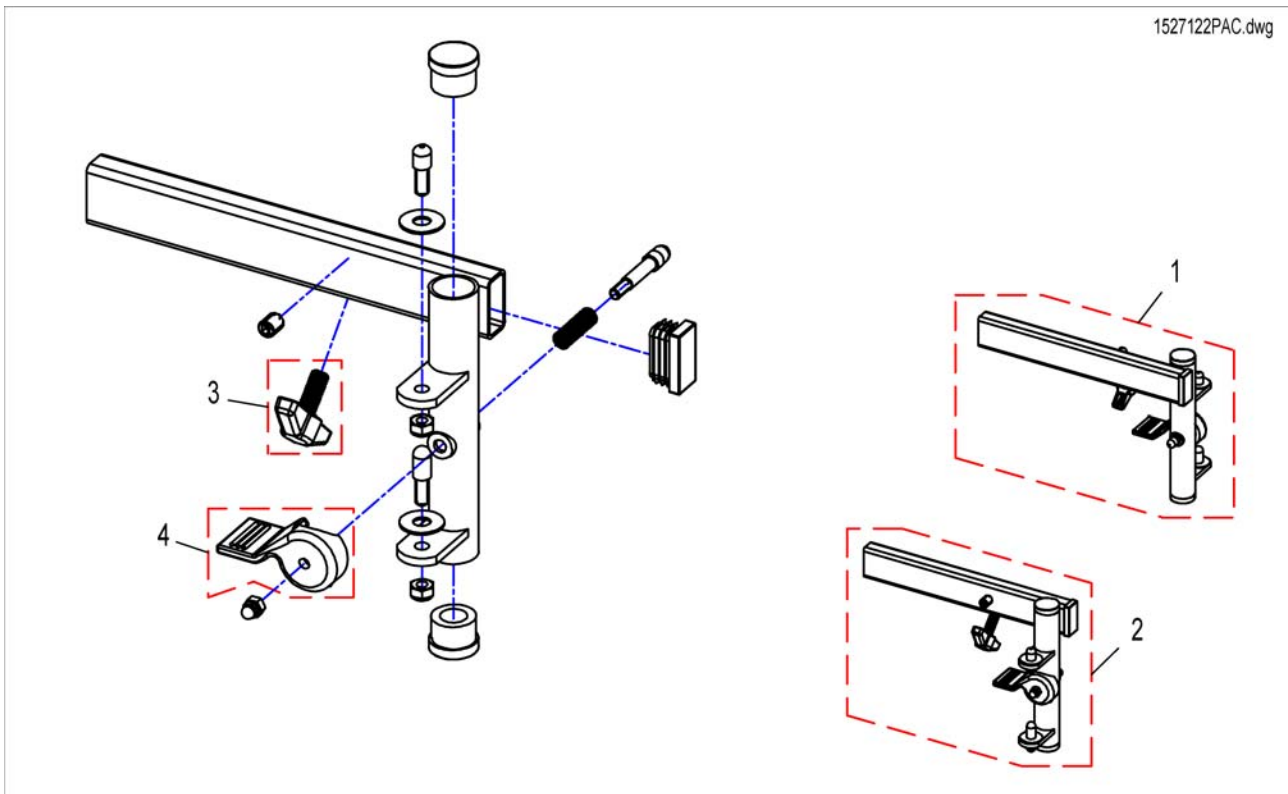

1	SP1527124	(Legrest support for 80° Legrest LH)	1
2	SP1527122	(Legrest Support for 80° Legrest RH)	1
3	256099	(Wing Screw M8 X 20 639)	1
4	5006757	UTLØSERHENDEL VENSTRE	1
4	5006756	UTLØSERHENDEL BENST HØYRE	1

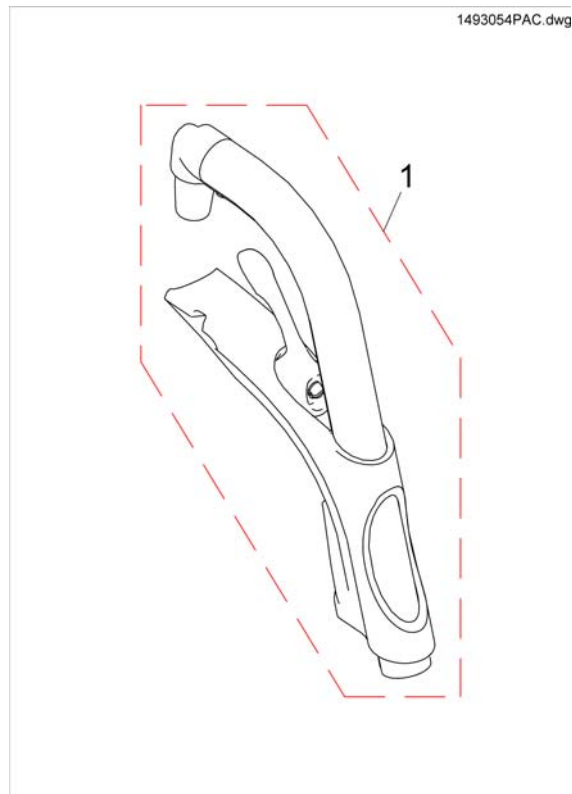


1 SP1493055


(Hanger 80°, Fixed)

1

**(Hanger manually height adjustable)**

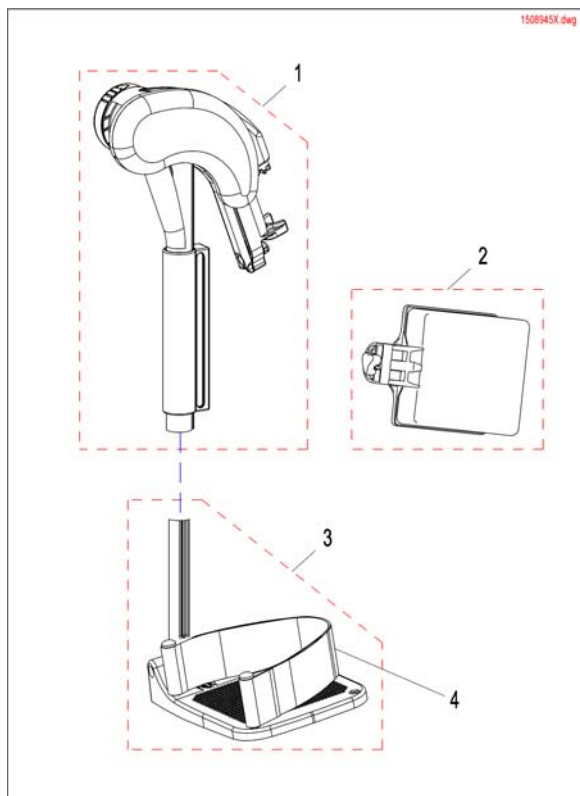


1	1513006	BENST OVERDEL VREG HØ M/ KNEPUTE ALU 10 2008	1
1	1513007	BENST OVERDEL VREG VE M/ KNEPUTE ALU 10 2008	1
2	1516314	LEGGSTØTTE 19.16,5X17X8 TR26 KPL	1
2	1518532	LEGGSTØTTE KPL 19" FOCUS,SPIREA	1
3	SP1442853	BEMSTØTTE NEDRE VINKELJ DRAGON HØ	1
3	SP1442854	BEMSTØTTE NEDRE VINKELJ DRAGON	1
4	1430402	HÆLBÅNDSATS BORA	1

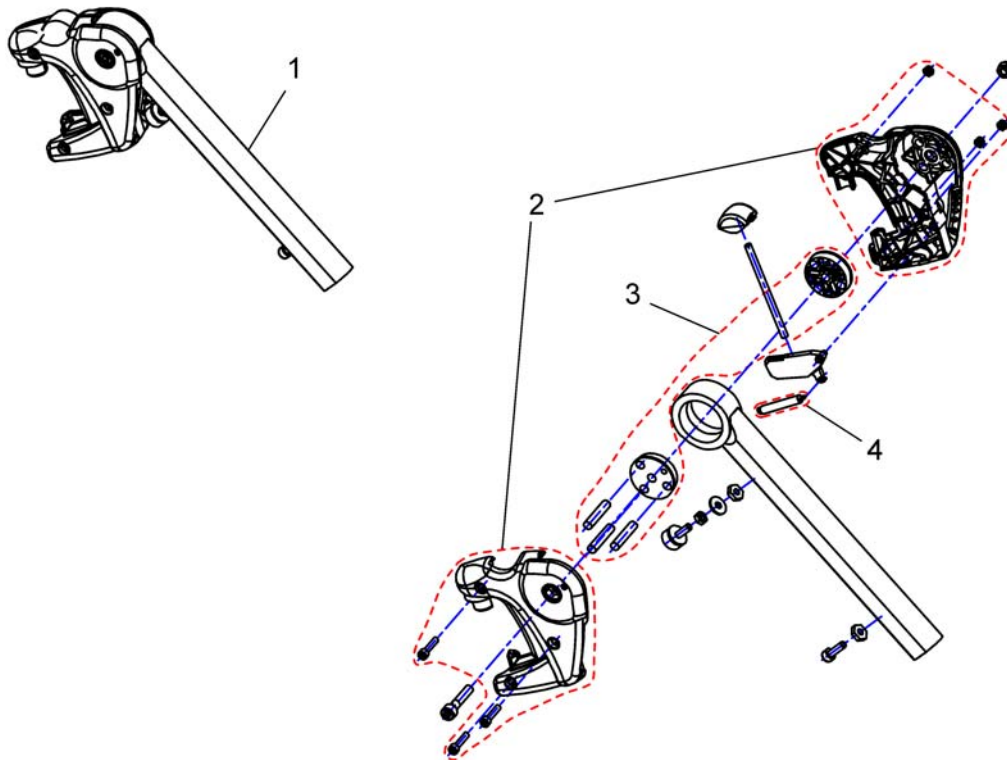

1	SP1449347	BENSTØTTE OVERDEL VARI F STORM TDX BORA VE	1
1	SP1449346	BENSTØTTE OVERDEL VARI F STORM TDX BORA HØ	1
2	SP1434955	(Cover Legrest cpl.)	1
3	1434956	(Mechanic)	1
4	SPZ173178	FJÆR STORM4 BORA	1

**(Vari A Hanger RH/LH 07.2005 ==>)**

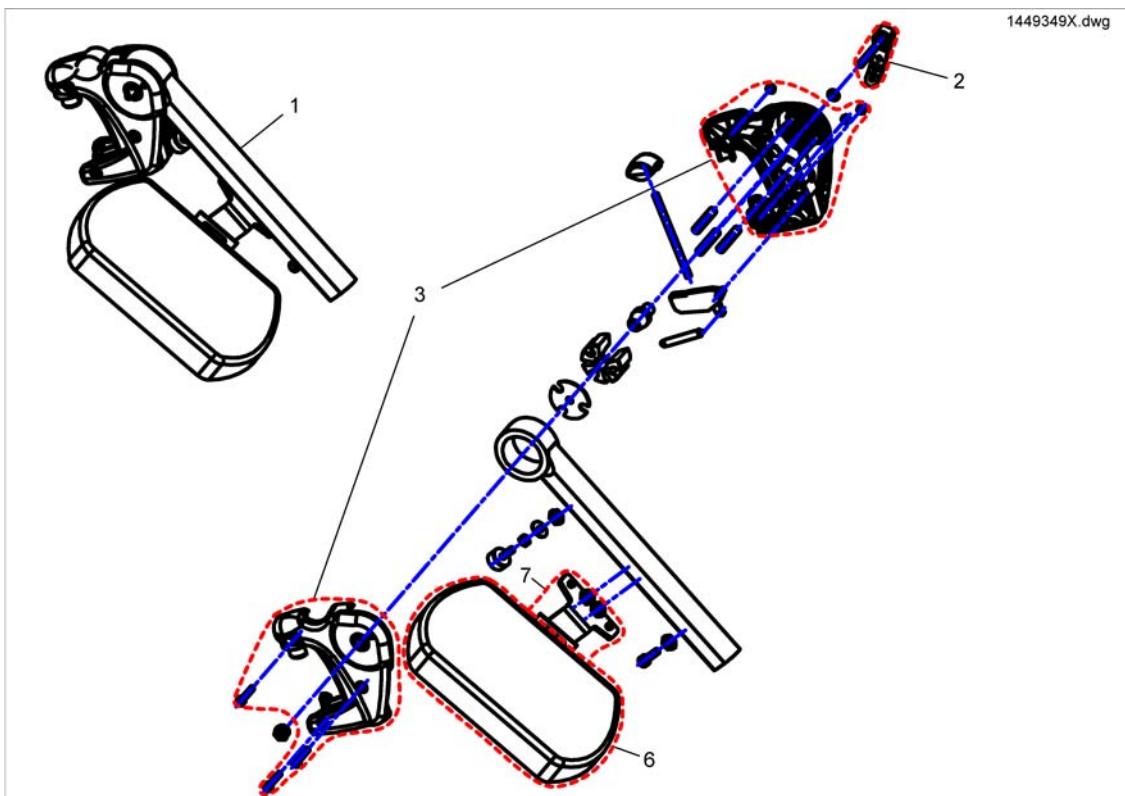


-	SP1527820	(Rework-Kit VariA-Legrest Version 1 ==> Version 2)	→ 01.04.2009	1
1	SP1449348	BENST OVERD MAN VREG VE BORA		1
1	SP1449349	BENST OVERD MAN VREG VE BORA		1
2	SPL32906	(Excentric Clamping Lever)		1
3	SP1434955	(Cover Legrest cpl.)		1
6	6301333	LEGGPLATE EL OG MAN VREG BENST STORM/TWISTER		1
7	SP1450447	LEGGPLATE KPL VE NY MAI 05		1
7	SP1450446	LEGGPLATE KPL HØ NY MAI 05		1



**(ADE Hanger RH/LH Electr. 07.2005 ==>)**


1449407X.dwg

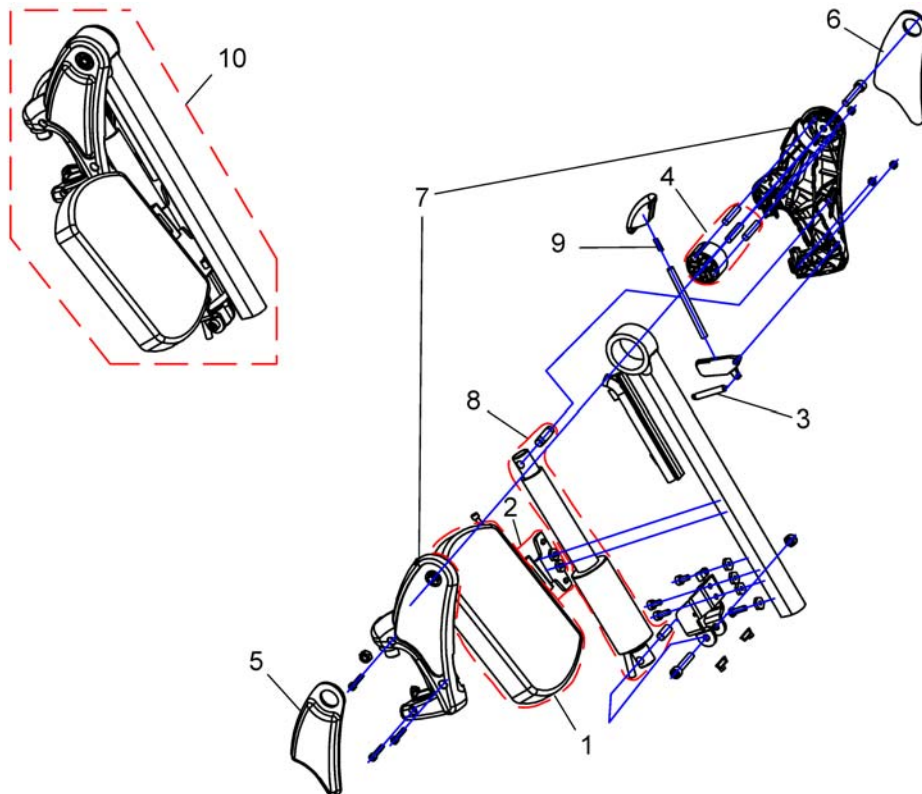


1	6301333	LEGGPLATE EL OG MAN VREG BENST STORM/TWISTER	1
2	SP1450447	LEGGPLATE KPL VE NY MAI 05	1
2	SP1450446	LEGGPLATE KPL HØ NY MAI 05	1
3	SPZ173178	FJÆR STORM4 BORA	1
4	SPZ174134	(Floating Cylinder With Pin)	1
5	SP1450448	KNEPOLSTER BENSTG EL HØ 07.2005->	1
6	SP1450410	KNEPOLSTER BENST EL VE 07.2005->	1
7	SP1449293	BENST HOLDER OVERDEL PAR EL VREG 07.2005->	1
8	SPZ151006	MOTOR EL BENST STORM4	1
9	SPN12196	FJÆR STORM4 BORA	1
10	SP1449408	BENST OVERDEL ADP HØ STORM4	1
10	SP1449407	BENST OVERDEL ADP VE STORM4	1

## (Hanger 70 Deg, Standard)

1497862.dwg

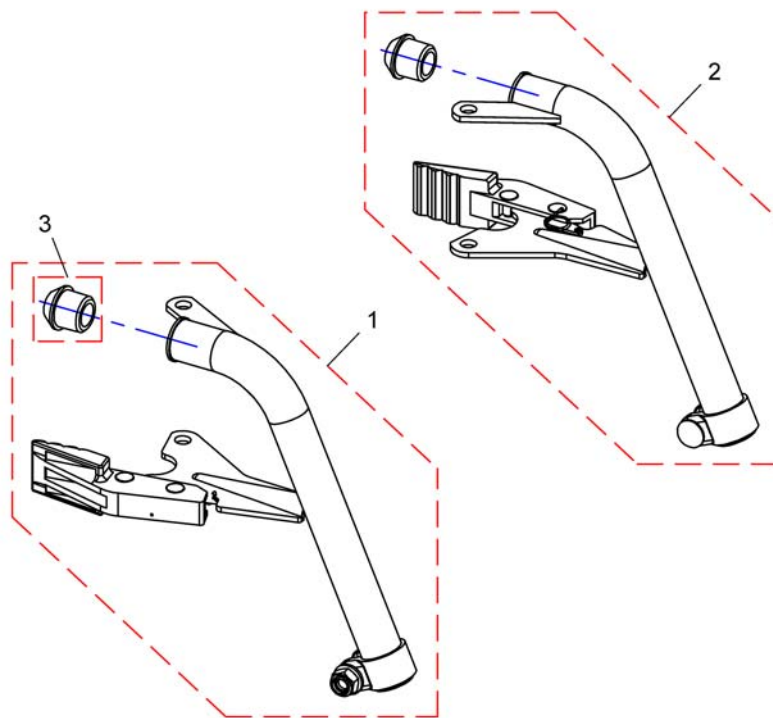


1	SP1497862	(Hanger 70 Deg, Standard, LH cpl.)	1
2	SP1497827	(Hanger 70 Deg, Standard, RH cpl.)	1
3	5007790	(Rubber pin)	1

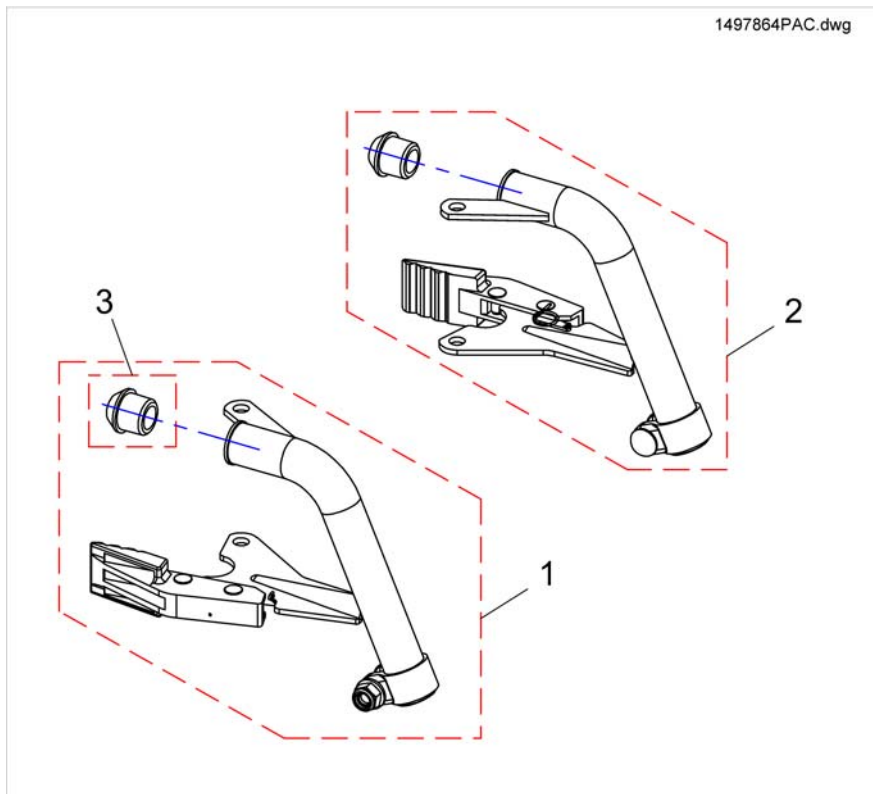


---

1	SP1497864	(Hanger 70 Deg, Junior, LH cpl.)	1
2	SP1497863	(Hanger 70 Deg, Junior, RH cpl.)	1
3	5007790	(Rubber pin)	1

## (Hanger 90°, Firm-Seat)

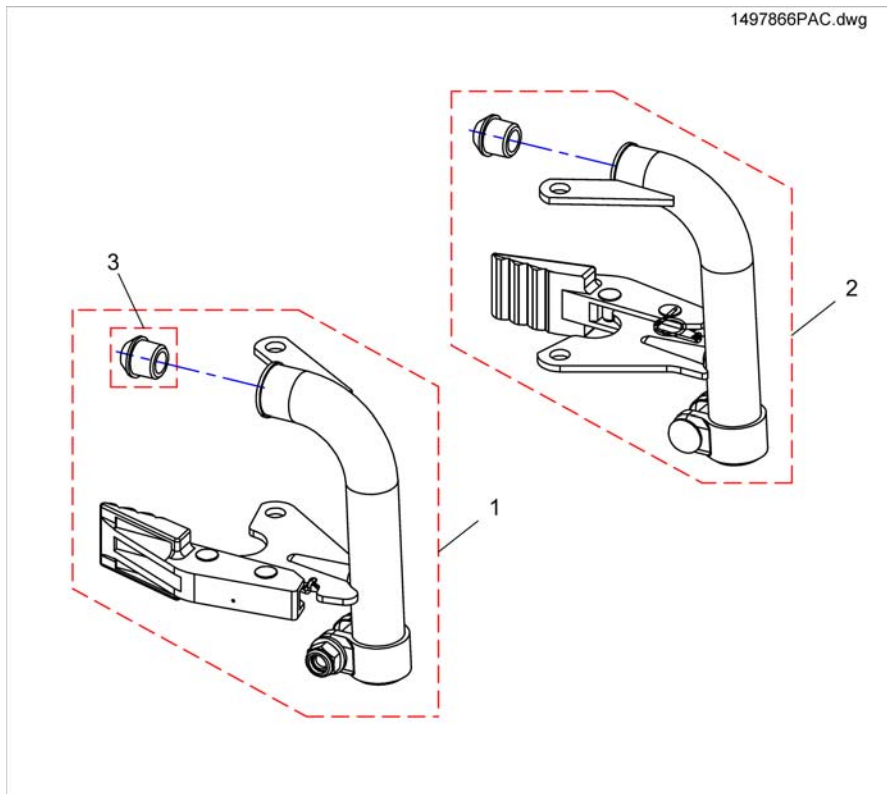


1	SP1497866	(Hanger 90 Deg, LH cpl.)	1
2	SP1497865	(Hanger 90 Deg, RH cpl.)	1
3	5007790	(Rubber pin)	1

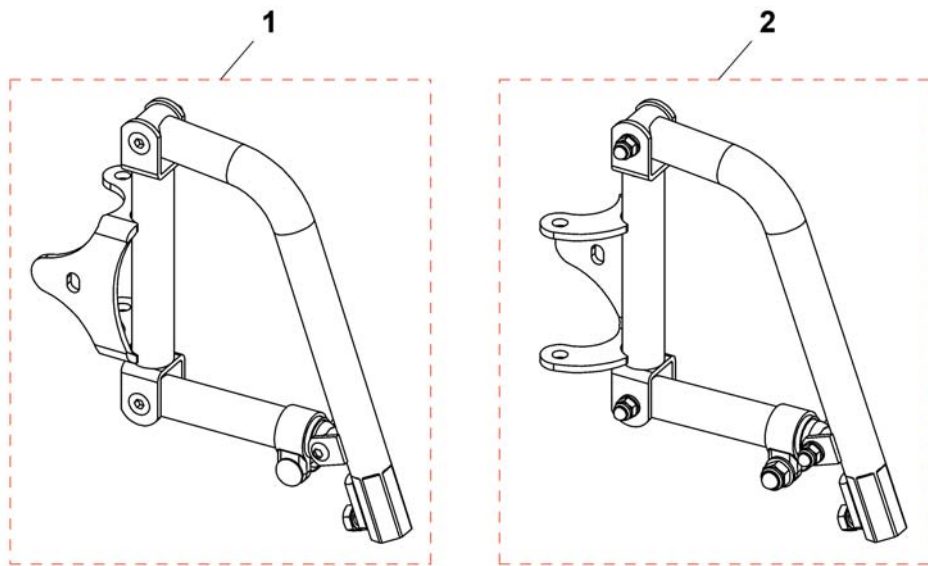

---

1	1488704	(Hanger Angle Adjustment, LH, cpl.)	1
2	1492519	(Hanger Angle Adjustment, RH, cpl.)	1

# (Legrest 80°)

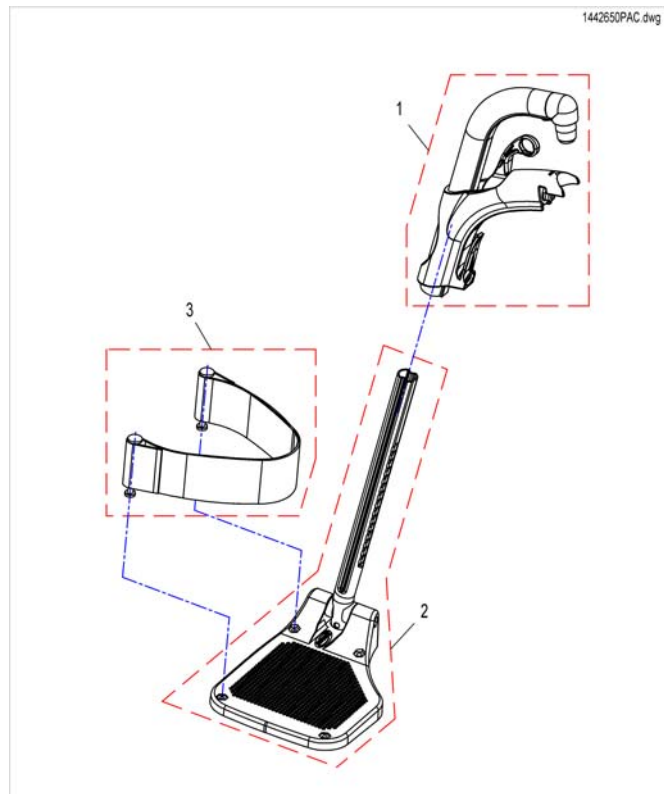


---

1	SP1493055	(Hanger 80°, Fixed)	1
2	1428363	FOTPLATE 21 FAST LANG VE ALU2003	1
2	1428364	FOTPLATE 21 FAST LANG HØ ALU2003	1
3	1430402	HÆLBÅNDSATS BORA	1

**(Footplate, depth-, angledadjustable, 80°)**


1428654X.iso

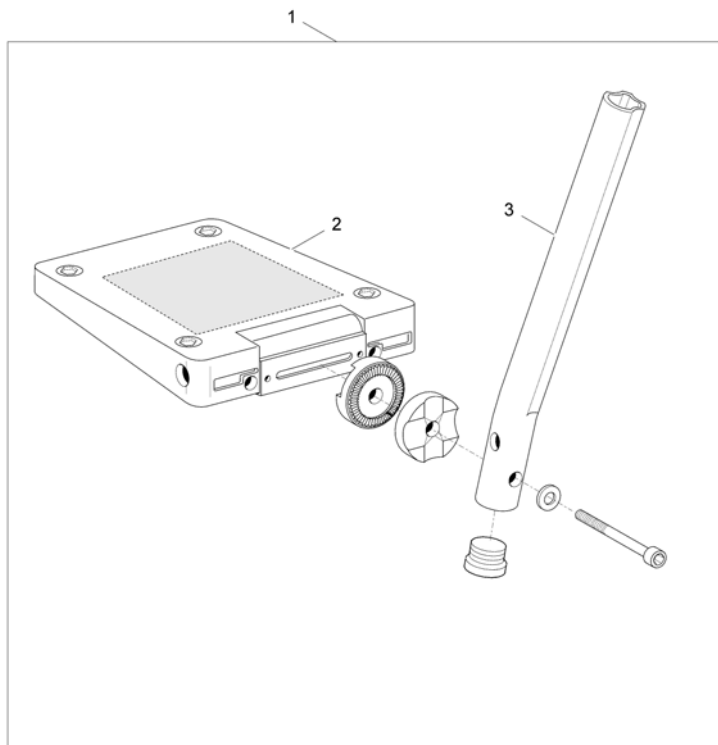

1	1428660	RØR NEDRE DEL VENSTRE V.REG ACTION 3	1
1	1428654	RØR NEDRE DEL HØYRE V.REG ACTION 3	1
2	5106594	FOTPLATE 48 BELLIS AZAL/FOC 166	1
3	1434561-0010	INNERRØR BENSTØTTE KORTE AZALEA/BELLIS	1

**(Footrest, fixed, 70°)**


146704PAC.dwg

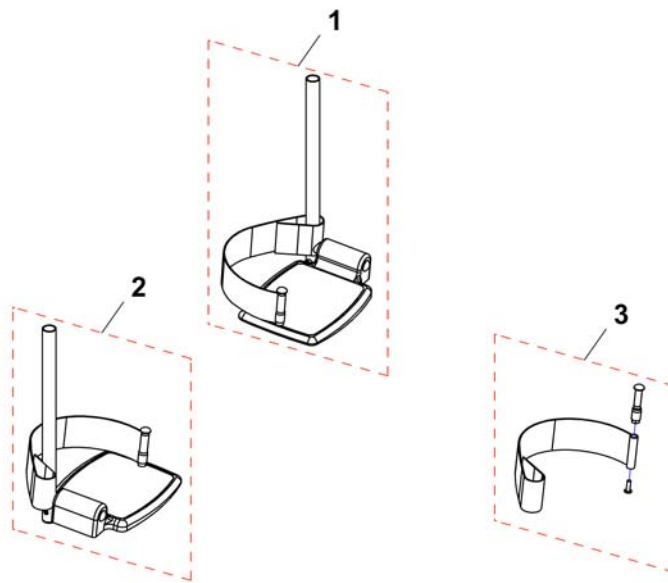

1	1416153	(F/Rest Ext Mini LH C/W)	1
1	1422339	(F/Rest Ext Mini LH C/W 7.5 Alum Footplates)	1
1	26003109	FOTPLATE VE SB 46 MOOVER SPECTRA	1
1	26003111	(F/Rest Ext Blk LH Narr)	1
2	1416152	(F/Rest Ext Mini RH C/W)	1
2	1422338	(F/Rest Ext Mini RH C/W 7.5 Alum Footplates)	1
2	26003108	FOTPLATE HØ SB 46 MOOVER SPECTRA	1
2	26003110	(F/Rest Ext Blk RH Narr)	1
3	SP1551628	(Heel strap, cpl.)	1




1497928PAC.dwg

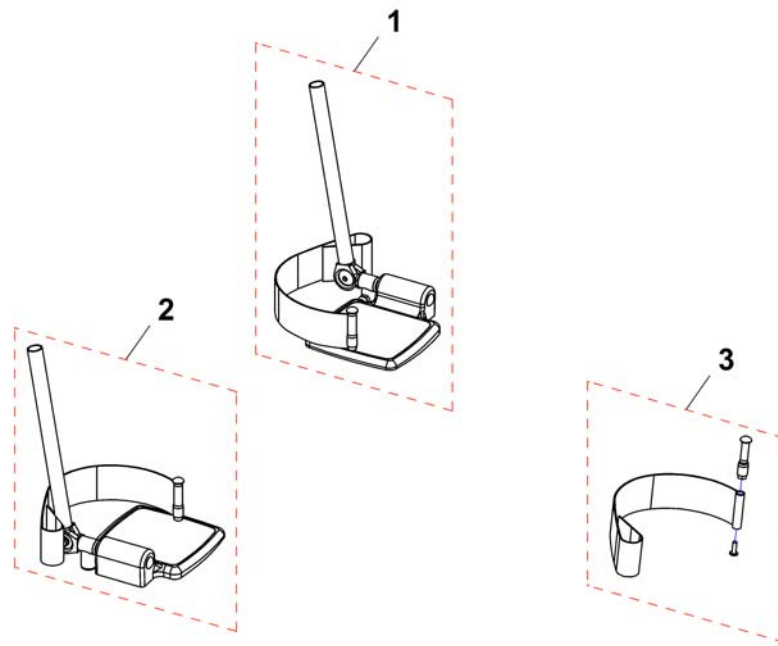

1	1425957	(Adj Angle F/Rest Kit LH C/W 6.5 Alum Footplate)	1
1	1425959	(Adj Angle F/Rest Kit LH C/W 7.5 Alum Footplate)	1
1	26002975	(Adj F/Rest Ext Narrow LH)	1
1	26003009	FOTPLATE HØ BORA	1
2	1425956	(Adj Angle F/Rest Kit RH C/W 6.5 Alum Footplate)	1
2	1425958	(Adj Angle F/Rest Kit RH C/W 7.5 Alum Footplate)	1
2	26002974	(Adj F/Rest Ext Narrow RH)	1
2	26003008	FOTPLATE KPL VE BORA	1
3	SP1551628	(Heel strap, cpl.)	1


**(Footrest, depth-, angledjustable, 70°)**

1497993PAC.iso

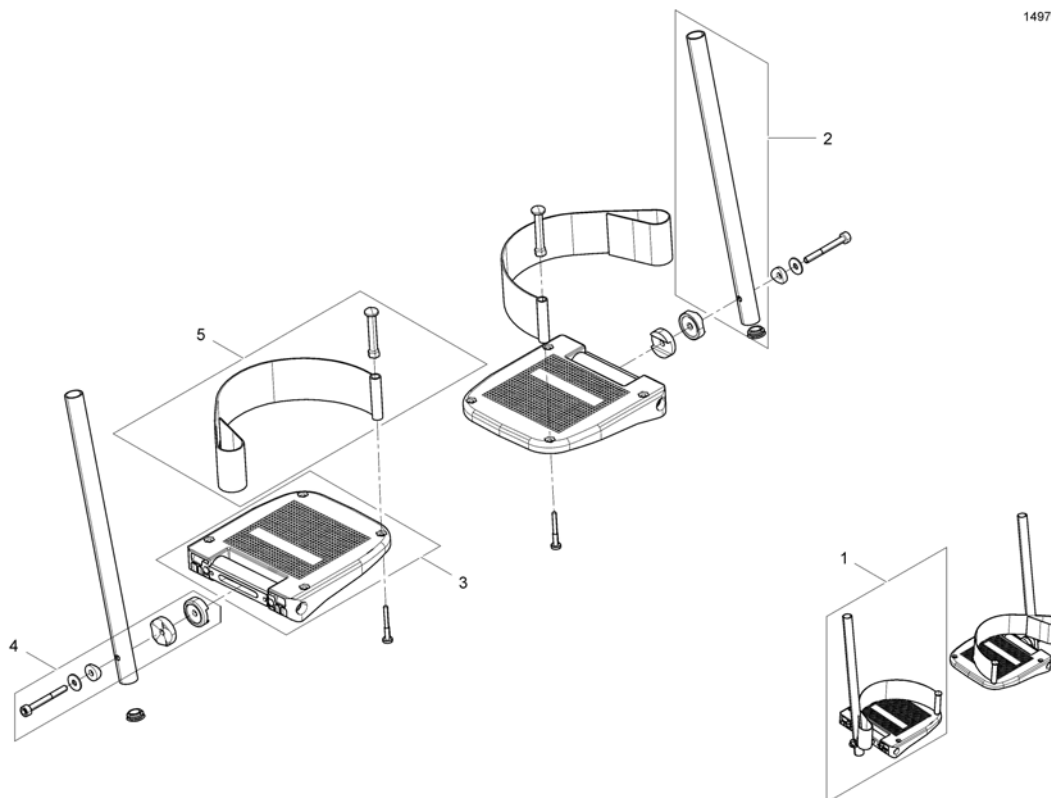


1	SP1545298	(Footplate, depth- and angledjustable, long insert tube (330mm) SW39, RH/LH, cpl.)	1
1	SP1545299	(Footplate, depth- and angledjustable, long insert tube (330mm) SW43, RH/LH, cpl.)	1
1	SP1545300	(Footplate, depth- and angledjustable, long insert tube (330mm) SW46, RH/LH, cpl.)	1
1	SP1545343	(Footplate, depth- and angledjustable, short insert tube (150mm) SW39, RH/LH, cpl.)	1
1	SP1545344	(Footplate, depth- and angledjustable, short insert tube (150mm) SW43, RH/LH, cpl.)	1
1	SP1545345	(Footplate, depth- and angledjustable, short insert tube (150mm) SW46, RH/LH, cpl.)	1
2	1422289-0205	INNERRØR LANG BORA	1
2	1422288-0205	INNERRØR KORT BORA	1
3	5106371	FOTPLATER ADAPT SB36	1
3	5106374	FOTBRETT KPL BR: 43 ADAPT 9 GR	1
3	5106375	FOTBRETT KPL BR: 46 ADAPT 9 GR	1
4	SP1528406	(Mounting-Kit)	1
5	SP1423202	HÆLBÅND BENSTØTTE STORM/TWISTER	1

**(Footplate with support tube angle adjust. RH/LH)**

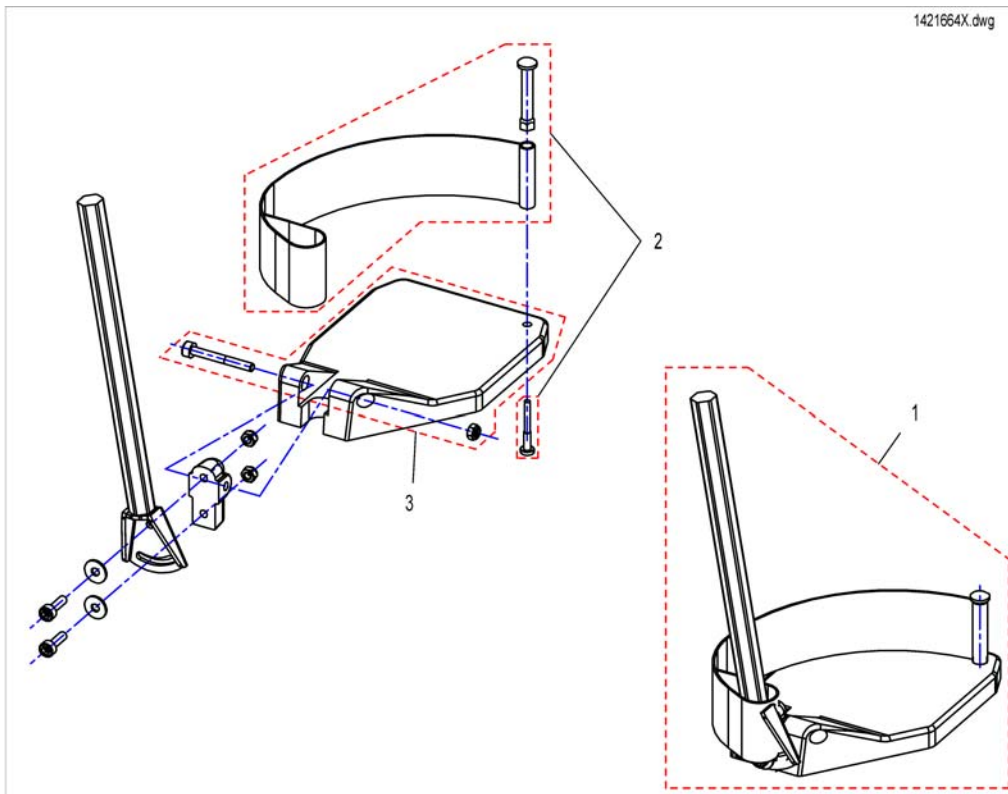


1	SP1421664	BENSTØTTEUNDERDEL HØ VINKELREG. STORM	1
1	SP1421665	BENSTØTTEUNDERDEL VE VINKELREG. STORM	1
2	SPB273011	HÆLBÅND BORA TDX	1
3	SPZ173217	(Footplate)	1
3	SPZ173218	(Footplate)	1

**(Footrests 80° & man. elevating)**

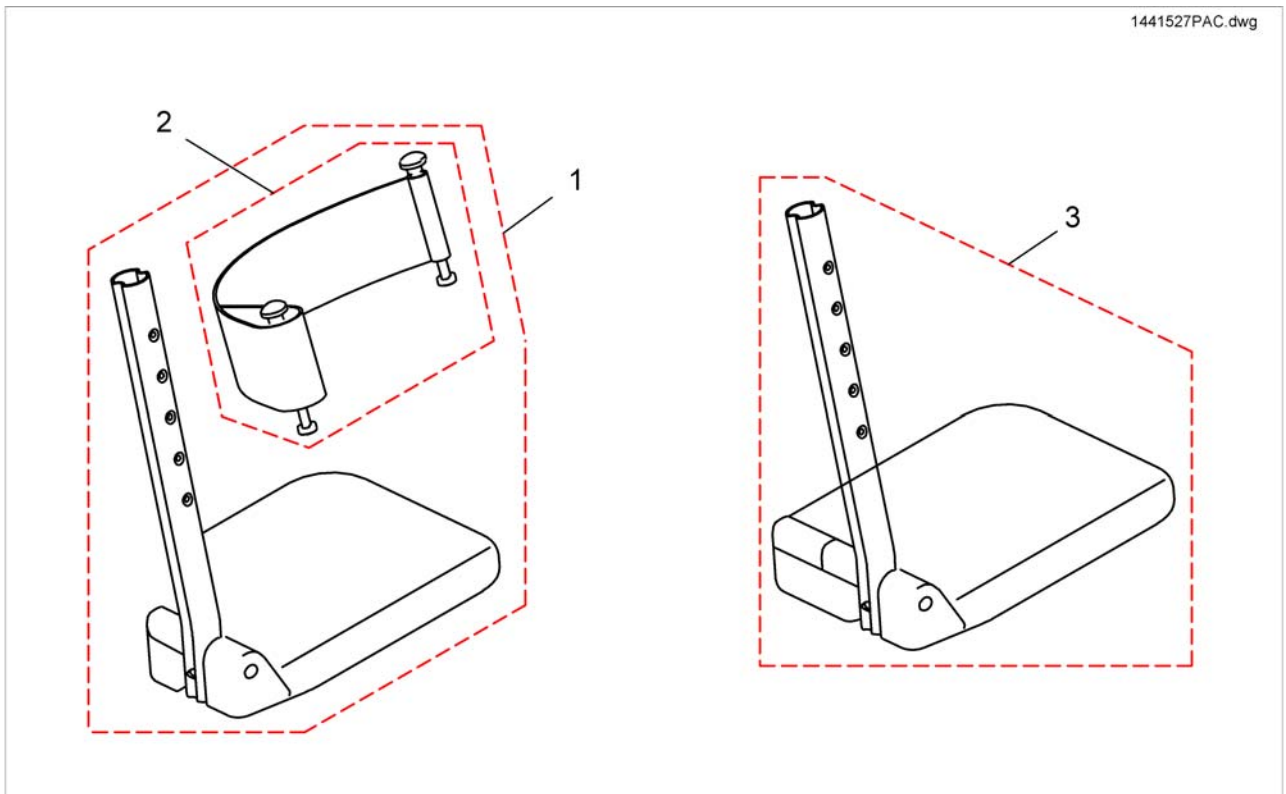


1	SP1442851	BENSTØTTEUNDERDEL HØ DRAGON	1
1	SP1442852	BENSTØTTEUNDERDEL VE DRAGON	1
2	1430402	HÆLBÅNDSATS BORA	1
3	1428367	FOTPLATE 24 FAST LANG VE ALU2003	1
3	1428368	FOTPLATE 24 FAST LANG HØ ALU2003	1

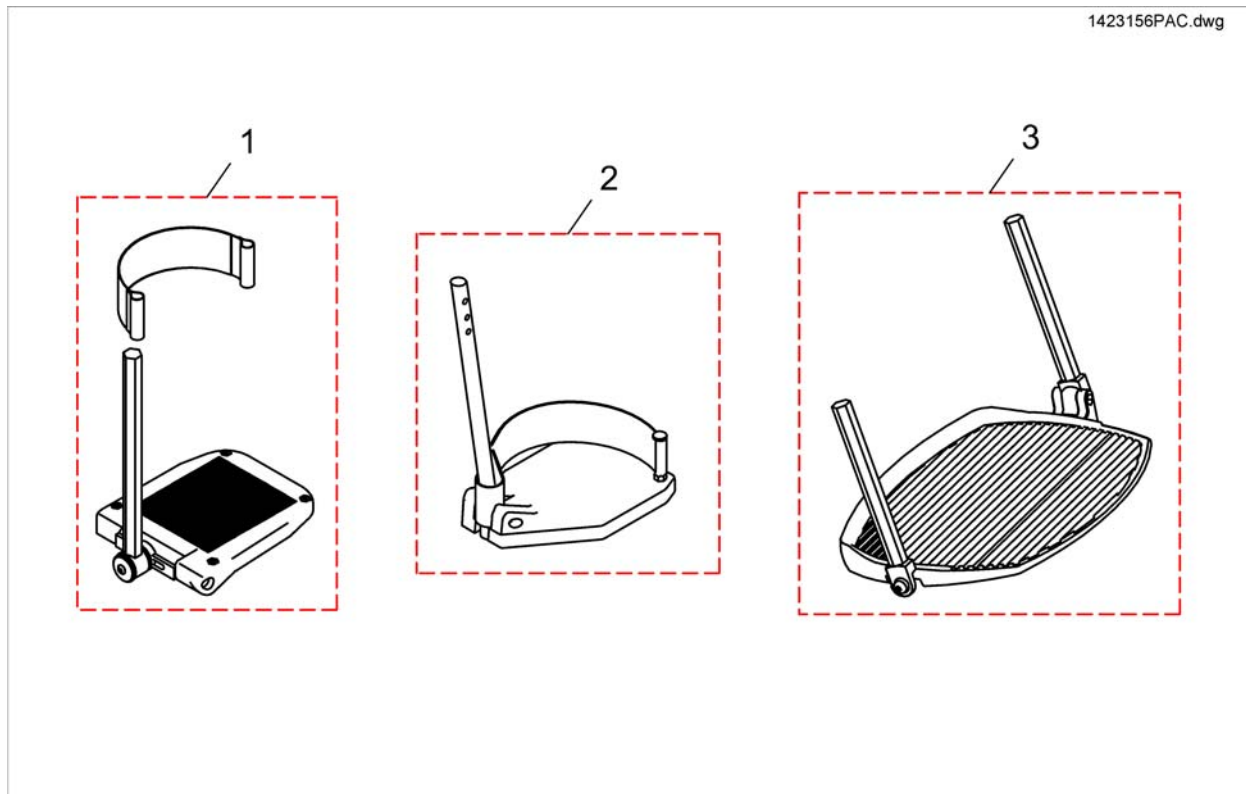

1	SP1419344	FOTPLATE VE VREG OG DREG KPL MED INNERRØR	1
1	SP1419345	FOTPLATE HØ VREG OG DREG KPL MED INNERRØR	1
2	SP1421662	BENSTØTTEUNDERDEL HØ STORM3	1
2	SP1421663	BENSTØTTEUNDERDEL VE STORM3	1
3	SP1423156	FOTPLATE HEL KPL STORM3	1


**(Flip-up footrests Vari series 01.08.10 ==>)**

1529644X.iso

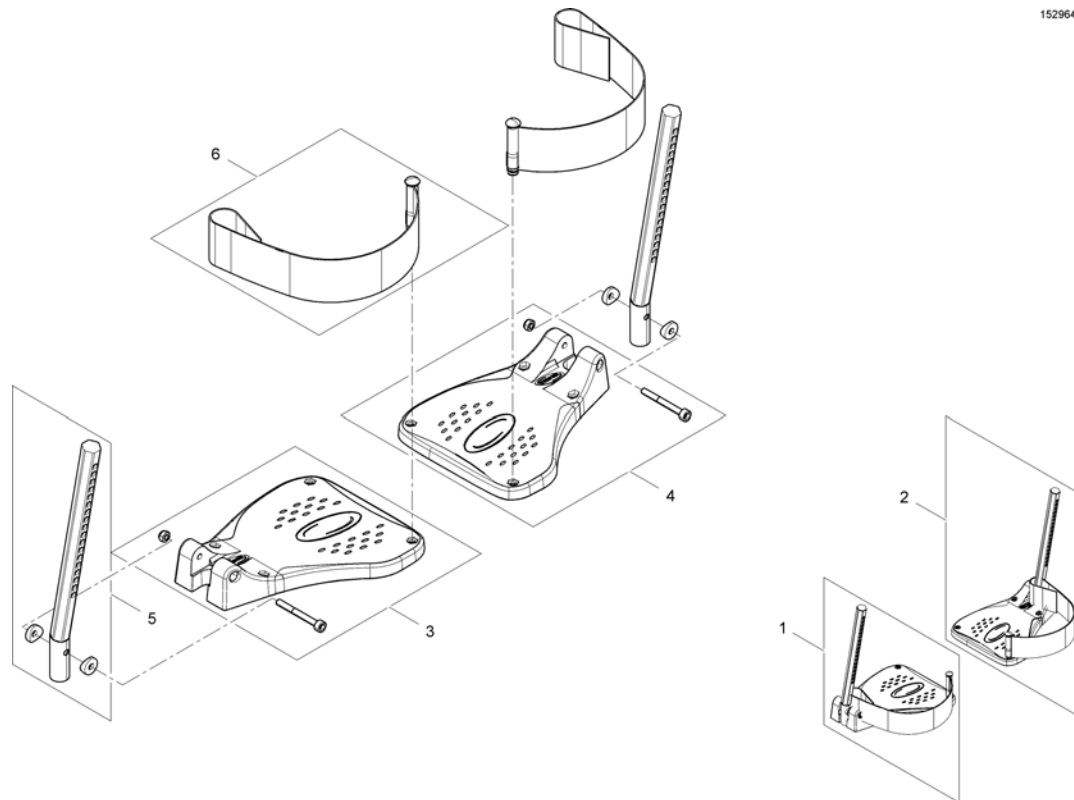

1	SP1541575	FORPLATE OPPFELLBAR KPL VENSTRE	1
2	SP1541574	FOTPLATE OPPFELLBAR KPL HØYRE	1
3	SP1529645	FOTPLATE KLP SB43-45 VE STORM4	1
4	SP1529644	FOTPLATE KLP SB43-45 HØ STORM4	1
5	SP1543045	(Plug-in tube)	1
6	SP1442467	LEGBÅND HELKAPPE PAR MIRAGE, STORM4, TDX	1

**(Angle adjustable footrests Vari series 01.08.10 ==>)**

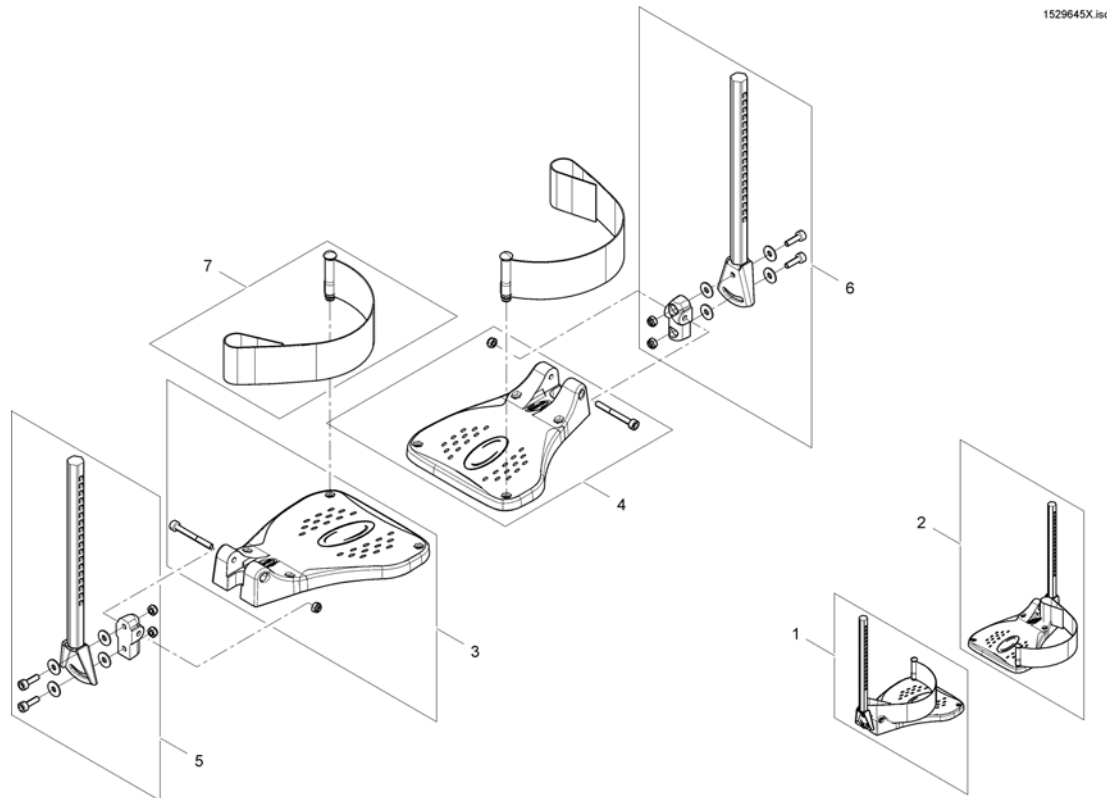

1529645X.iso

1	SP1541580	(Footplate flip up, angle adjustable, LH, cpl.)	1
2	SP1541579	(Footplate flip up, angle adjustable, RH, cpl.)	1
3	SP1529645	FOTPLATE KLP SB43-45 VE STORM4	1
4	SP1529644	FOTPLATE KLP SB43-45 HØ STORM4	1
5	SP1423204	(FOOT PLATE TUBE, ANGLE ADJUSTMENT LH CPL.)	1
6	SP1423203	(FOOT PLATE TUBE, ANGLE ADJUSTMENT RH CPL.)	1
7	SP1442467	LEGGBÅND HELKAPPE PAR MIRAGE, STORM4, TDX	1


## Sidegring



---

1	-	(Armrest for Standard Seat)	1
2	-	(Armrest for Flex2 Seat)	1
3	-	(Armrest for Firm Seat)	1
4	-	(Armrest for Max Seat)	1
5	-	(Armrest for Modulite Seat)	1



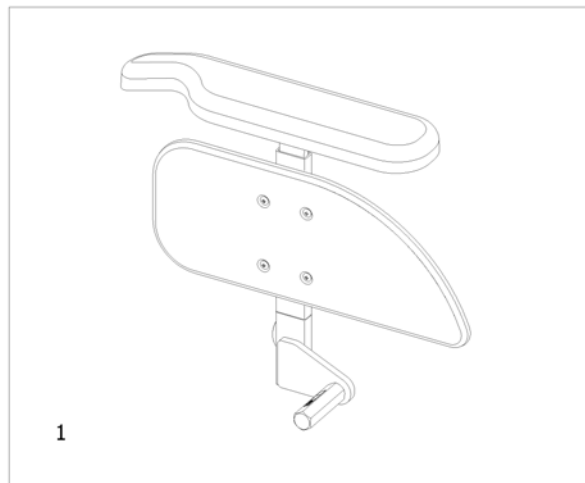


1 -


(T-Armrest, detachable, with Armpad, for Standard Seat 2008-08-01 ==>)  
[0606+0647/0649]

1

**(T-Armrest, detachable, with Armpad, for Standard Seat 2008-08-01 ==>)**

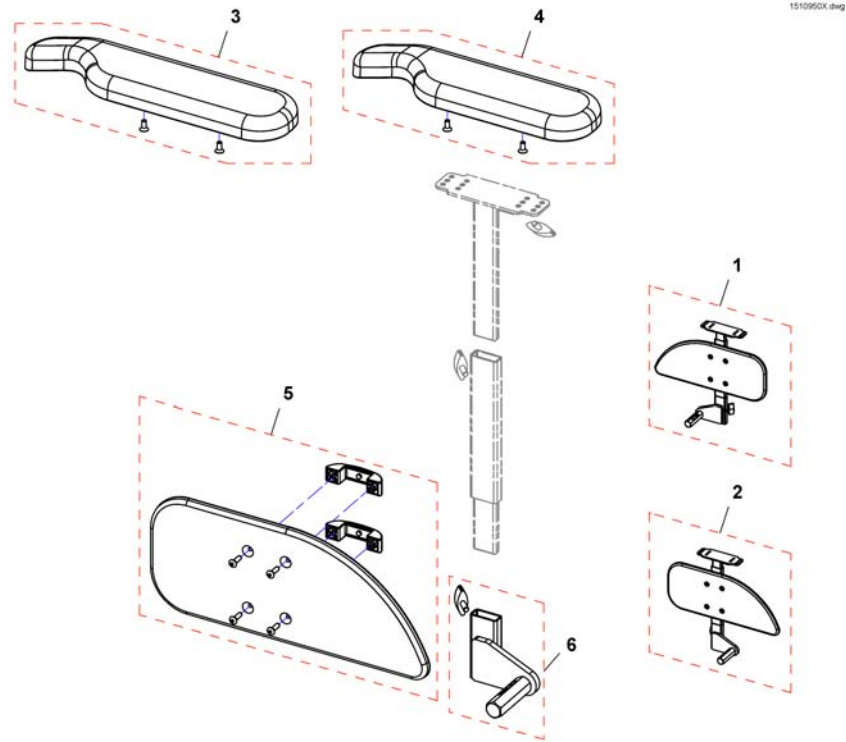


1	SP1510935	(Armrest, without armpad, RH)	1
2	SP1510950	(Armrest, without armpad, LH)	1
3	SP1528846	ARMLENS POLSTER VE LANG STORM TDX BORA	1
3	SP1528845	ARMLENS POLSTER HØ LANG STORM4 TDX BORA	1
4	SP1443004	ARMLENS POLSTER VE STORM STORM TDX BORA	1
4	SP1449043	ARMLENS POLSTER HØ STORM STORM TDX BORA	1
5	SP1541505	(Skirt guard, RH)	1
5	SP1541506	(Skirt guard, LH)	1
6	SP1541510	(Bracket for T-arm, RH)	1
6	SP1541511	(Bracket for T-arm, LH)	1

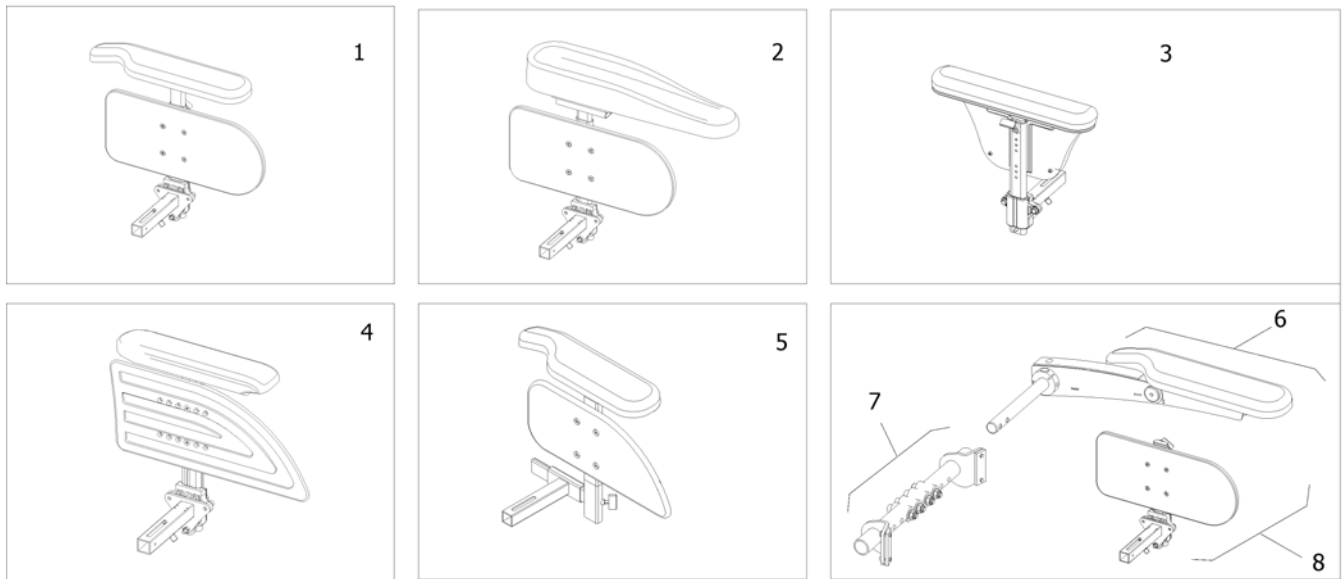


1	-	(Armrest long) [0670]	1
2	-	(Armrest Hemiplegic) [0670+0671/0672]	1
3	-	(Armrest Azalea style) [0625]	1
4	-	(Armrest RIO, Flex II) [0624]	1
5	-	(T-Armrest detachable) [0606]	1
6	-	(Flip-up armrest) [0621]	1
7	-	(Mounting bracket Flip-up armrests) [0621]	1
8	-	(Skirt guard Flip-up armrest) [0623]	1

**(Armrest long)**

1549087X.dwg

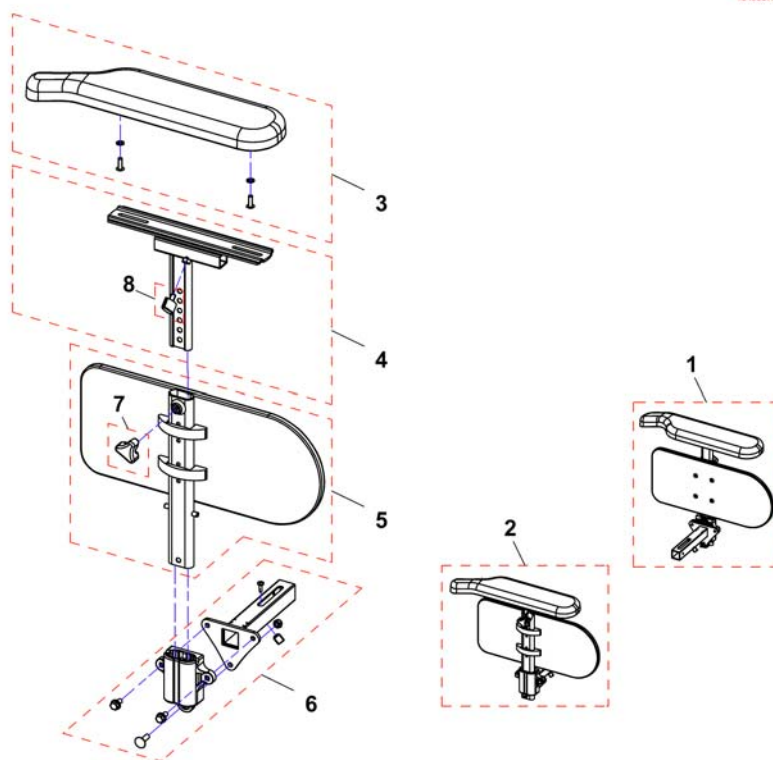

1	SP1549088	ARMLEN LANGT VE BORA	1
2	SP1549087	ARMLEN LANGT HØ BORA	1
3	SP1528846	ARMLENSPOLSTER VE LANG STORM TDX BORA	1
3	SP1528845	ARMLENSPOLSTER HØ LANG STORM4 TDX BORA	1
4	SP1433317	(T-Arm, LH)	1
4	SP1433316	(T-Arm, RH)	1
5	1415521	SIDESTØTTE(KLESBESK.)VE >010507 KPL	1
5	1415520	SIDESTØTTE (KLESBESK.)HØ >010507 KPL	1
6	SP1528816	ARMLINFESTE BORA	1
7	1434623	LÅSERATT 75MM STORM4	1
8	L31001	VINGSKRUV M 6X10 GARANT G24 S	1

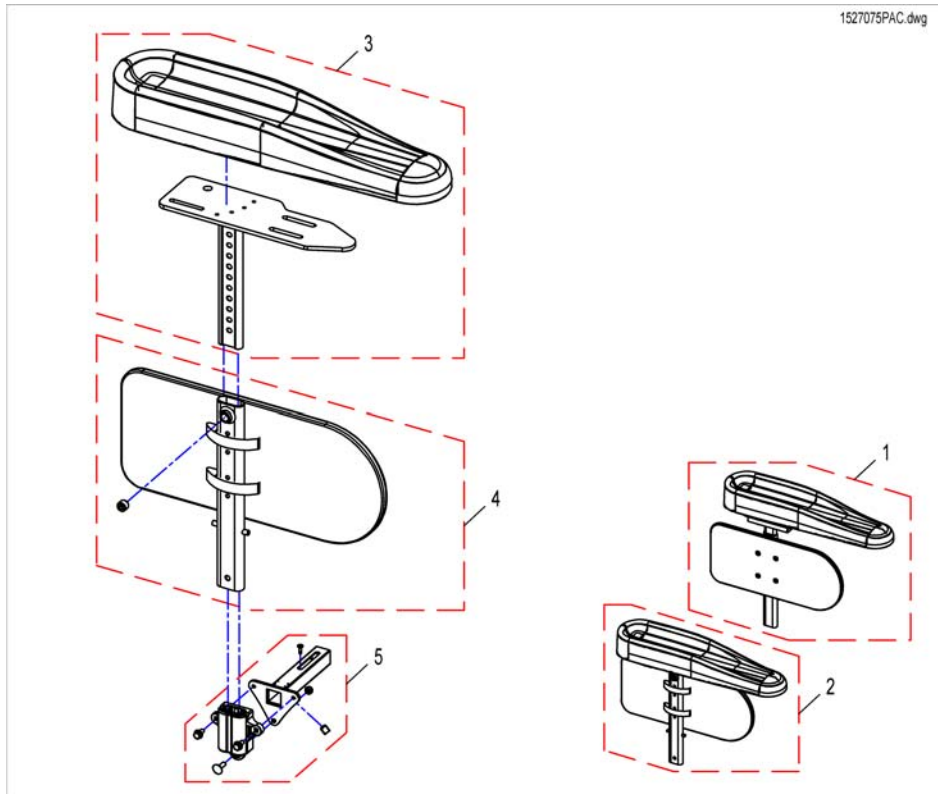

1	SP152706	ARMLNSPOLSTER HEMI VE BORA UTSVINGBART	1
2	SP152705	ARMLNSPOLSTER HEMI HØ BORA UTSVINGBART	1
3	1493219	ARMLN HEMI KPL HØ HEMI	1
3	1493220	ARMLNS HEMI KPL VE HEMI	1
4	1415520	SIDESTØTTE (KLESBESK.)HØ >010507 KPL	1
4	1415521	SIDESTØTTE(KLESBESK.)VE >010507 KPL	1
5	SP1528816	ARMLNFESTE BORA	1


# (Armrest Azalea style)

1527196PAC.dwg



---


1	SP1527196	ARMLÉN KPL HØ VE BORA	1
2	L31001	VINGSKRUV M 6X10 GARANT G24 S	1
3	SP1528816	ARMLÉNFESTE BORA	1

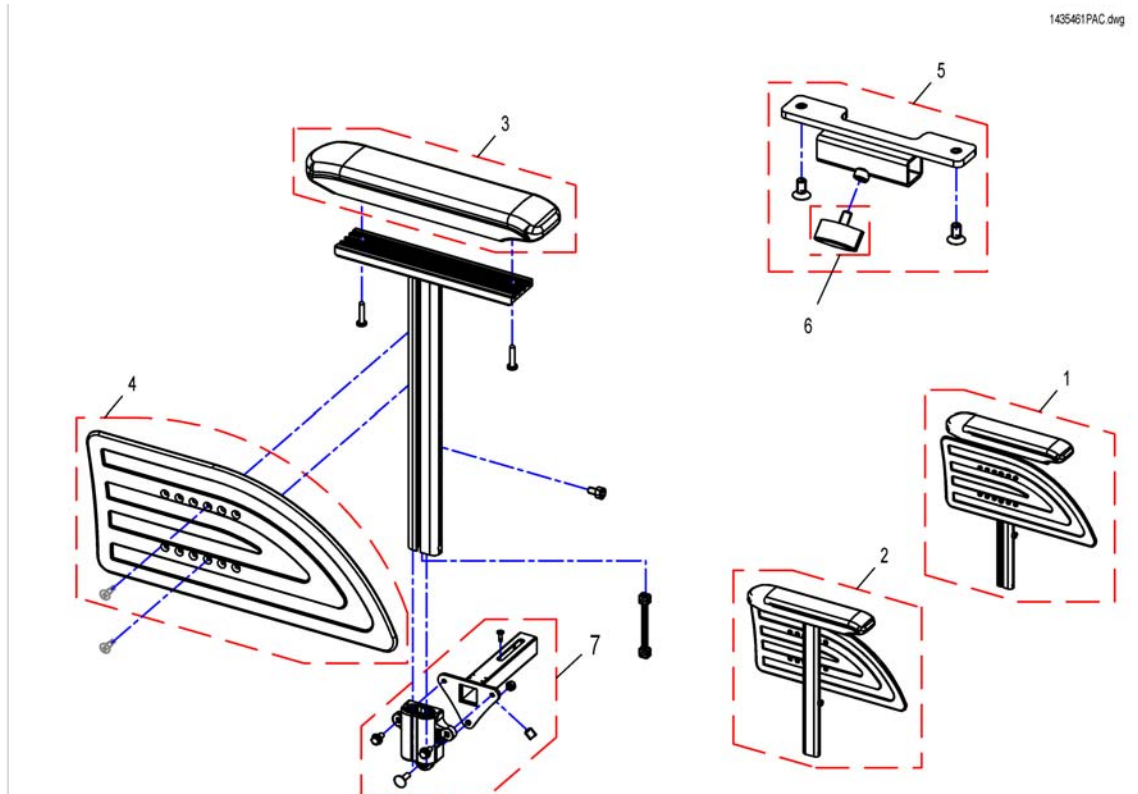

1	1435461	ARMLÉN HREG RIO VE KORT	1
2	1435460	ARMLÉN HREG RIO HØ KORT	1
3	1434676	ARMLÉNSPOLSTER KORT FOCUS/SPIREA3/ACTION3	1
4	1434725	KLESBESKYTTER TIL ARMLÉN RIO	1
5	SP1528327	FESTE STYREBOKS ARMLÉN BORA	1
6	L31001	VINGSKRUV M 6X10 GARANT G24 S	1
7	SP1528816	ARMLÉNFESTE BORA	1


# (T-Armrest detachable)

1559709 iso

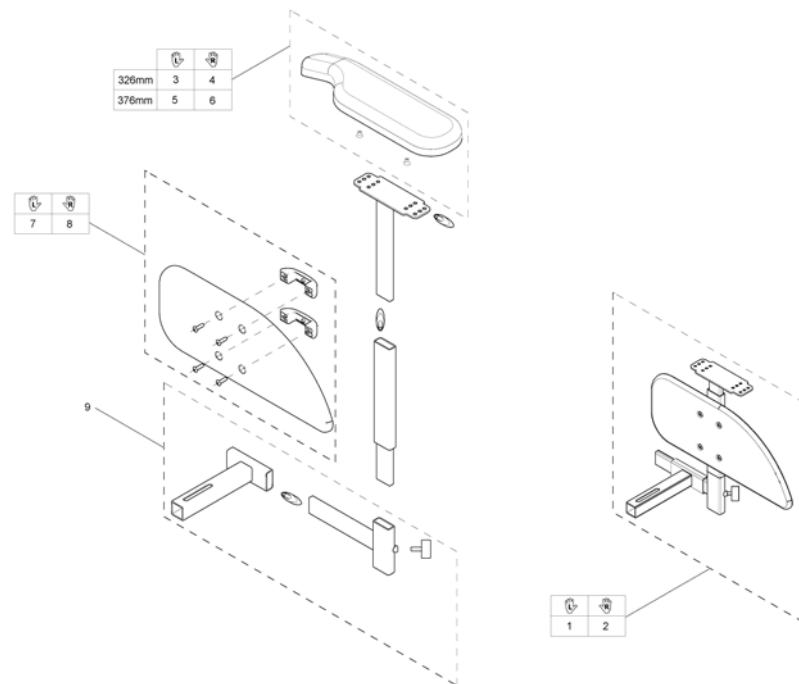

1	SP1556920	ARMLÉN LANG U POLSTER VE BORA	1
2	SP1556925	ARMLÉN LANG HØ U POLSTER BORA	1
3	SP1443004	ARMLÉN POLSTER VE STORM STORM TDX BORA	1
4	SP1449043	ARMLÉN POLSTER HØ STORM STORM TDX BORA	1
5	SP1528846	ARMLÉN POLSTER VE LANG STORM TDX BORA	1
6	SP1528845	ARMLÉN POLSTER HØ LANG STORM4 TDX BORA	1
7	SP1541506	(Skirt guard, LH)	1
8	SP1541505	(Skirt guard, RH)	1
9	SP1559710	(Receiver for armrest)	1




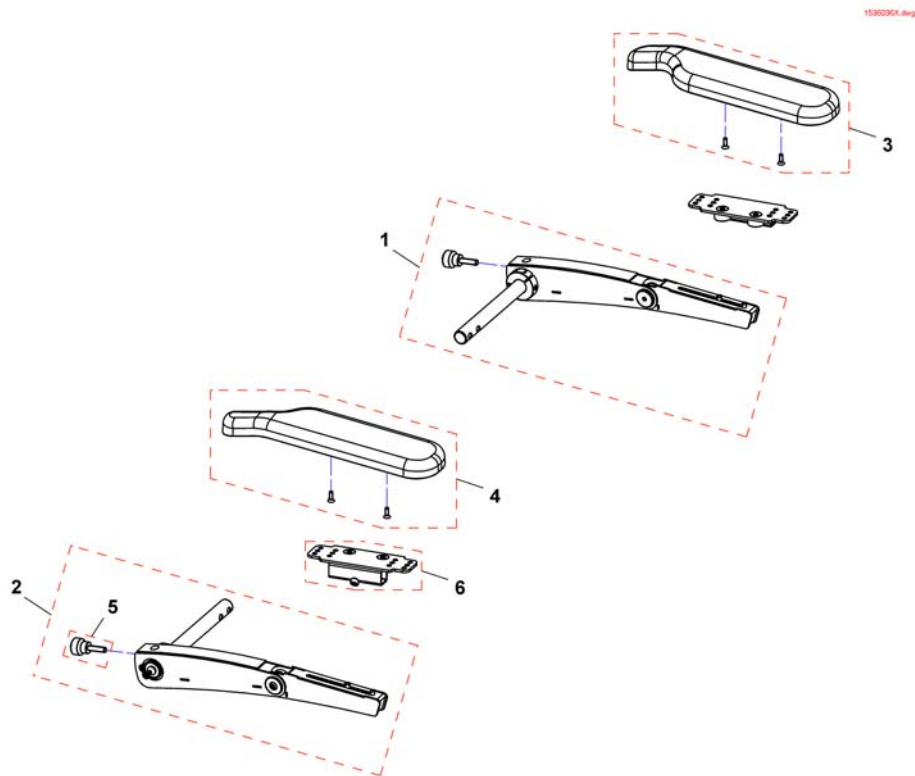

1	SP1536100	ARMLÉN LANG VE OPPFELLB STORM4	1
2	SP1536099	ARMLÉN LANG HØ OPPFELLB STORM4	1
3	SP1528846	ARMLÉNSPOLSTER VE LANG STORM TDX BORA	1
4	SP1528845	ARMLÉNSPOLSTER HØ LANG STORM4 TDX BORA	1
5	SP1535504	(Knurled screw M8x30)	1
6	SP1535244	FESTE ARMLÉN RYGGMONT STORM4	1

# (Mounting bracket Flip-up armrests)


1548568.png

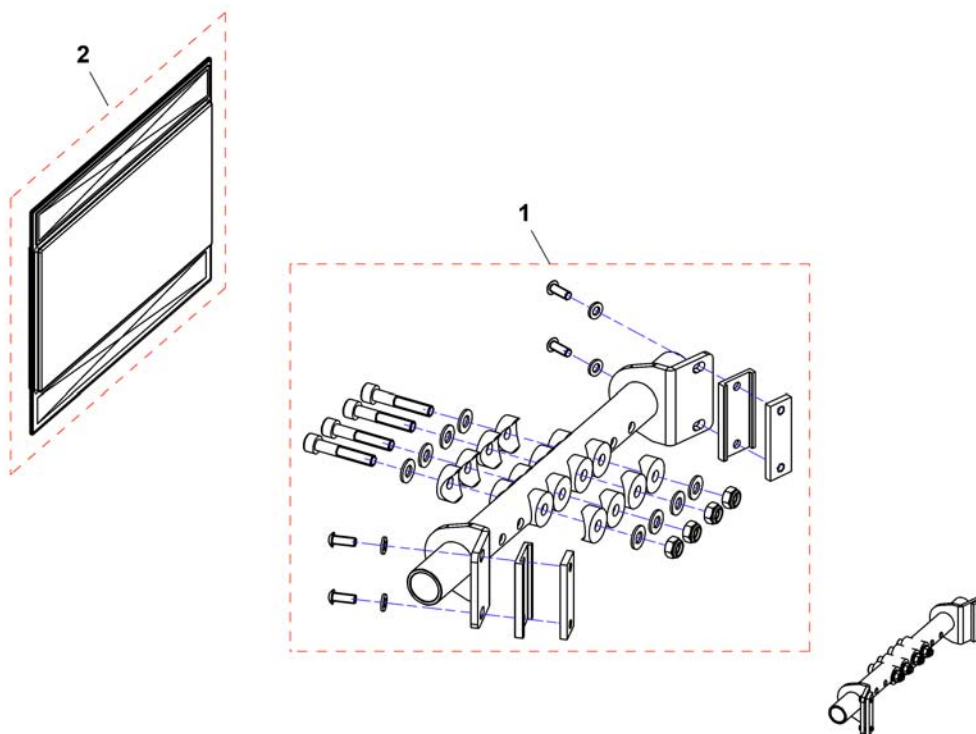

1	SP1548568	FESTER OPPFEL ARML SB39 BORA	1
1	SP1548569	FESTER OPPFEL ARML SB44 BORA	1
1	SP1548631	FESTER OPPFEL ARML SB49 BORA	1
2	SP1550410	(Protection cover)	1

Invacare® Bora | Sidegring  
**(Skirt guard Flip-up armrest)**


1549094.png

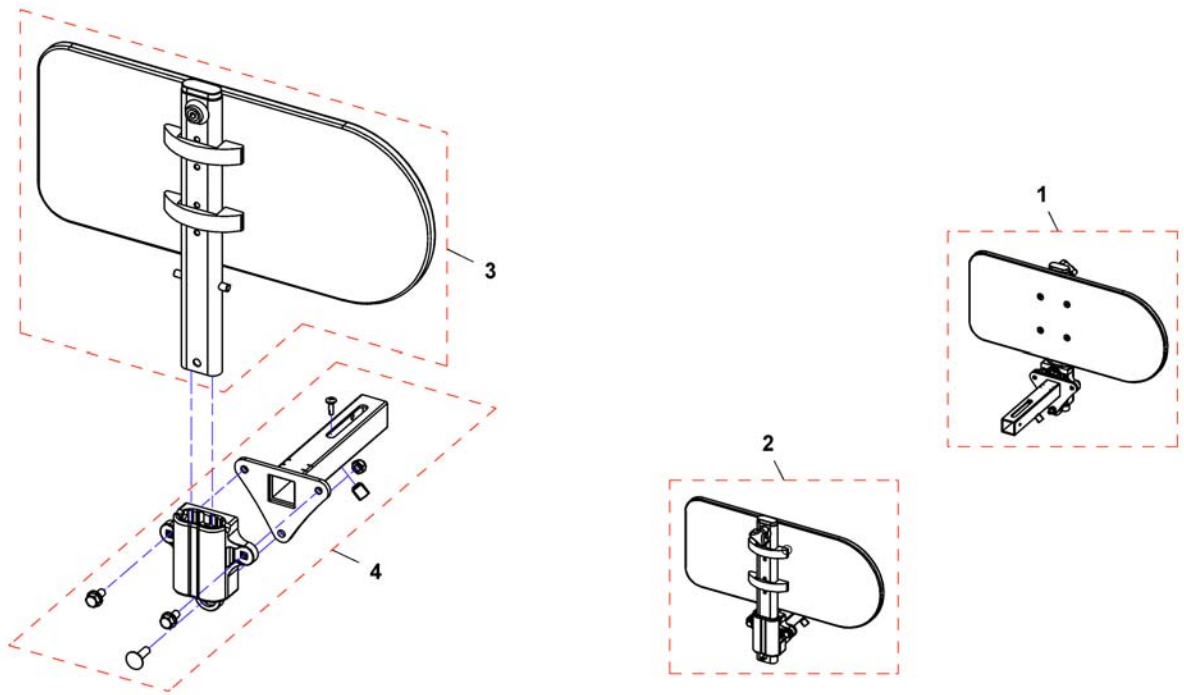

1	SP1549095	KLESBESKYTTER VE OPPFARM BORA	1
2	SP1549094	KLESBESKYTTER HØ OPPFARM BORA	1
3	1415521	SIDESTØTTE(KLESBESK.)VE >010507 KPL	1
3	1415520	SIDESTØTTE (KLESBESK.)HØ >010507 KPL	1
4	SP1528816	ARMLNFESTE BORA	1


# (Armrest for Firm Seat)

1572568.iso

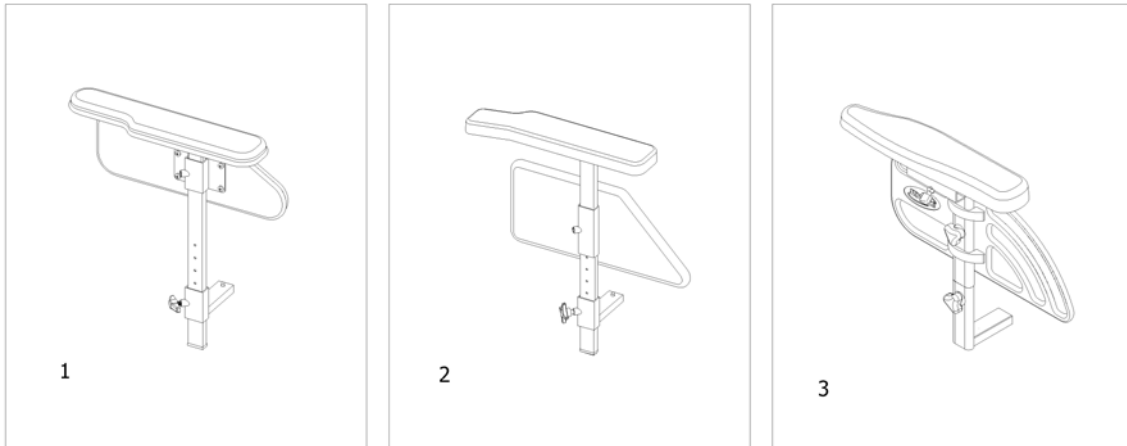


---


1	-	(Armrest, padded) <i>[0674]</i>	1
2	-	(Armrest, moulded) <i>[0675]</i>	1
3	-	(Armrest Ergonomic) <i>[0676]</i>	1

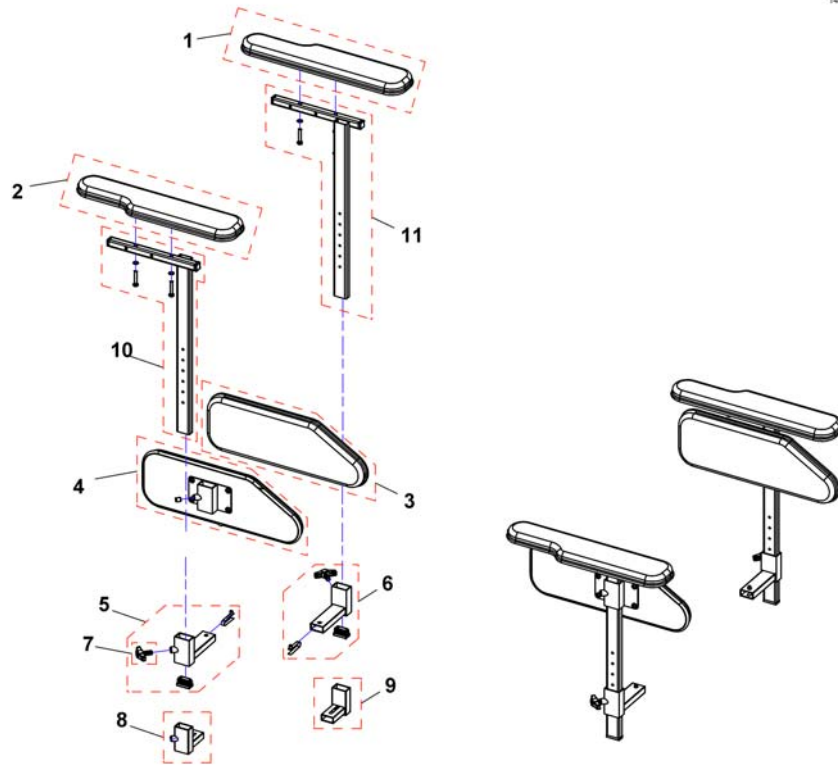


1	23008731BKB	(Armpad LH SW 38/43, Black Bird)	1
1	23008731BKJ	(Armpad LH SW 38/43, Black Jemina)	1
1	SP1422421	(Armpad LH SW33/38, Black Jemina)	1
2	23008730BKB	(Armpad RH SW 38/43, Black Bird)	1
2	23008730BKJ	(Armpad RH SW 38/43, Black Jemima)	1
2	SP1422420	(Armpad RH SW 33/38, Black Jemina)	1
3	SP26003225BKJ	(Padded Skirt Guard LH SW 38/43, Black Jemima)	1
3	26003225BKB	(Padded Skirt Guard LH SW 38/43, Black Bird)	1
3	SP26003223BKJ	(Padded Skirt Guard LH SW 33, Black Jemima)	1
3	SP26003223BKB	(Padded Skirt Guard LH SW 33, Black Bird)	1
4	SP26003224BKJ	(Padded Skirt Guard RH SW 38/43, Black Jemima)	1
4	26003224BKB	(Padded Skirt Guard RH SW 38/43, Black Bird)	1
4	SP26003222BKJ	(Padded Skirt Guard RH SW 33, Black Jemima)	1
4	SP26003222BKB	(Padded Skirt Guard RH SW 33, Black Bird)	1
5	SP1534391	HOLDER ARMLEN HØ BORA <i>Use for Seatwidth 33cm</i>	1
6	SP1534392	HOLDER ARMLEN VE BORA <i>Use for Seatwidth 33cm</i>	1
7	256099	(Wing Screw M8 X 20 639)	1
8	26003706	HOLDER ARMLEN HØ BORA	1
9	26003707	HOLDER ARMLEN VE BORA	1
10	SP1541558	(Armpad holder RH, for SW 38/43)	1
11	SP1541559	(Armpad holder LH, for SW 38/43)	1
10	SP1543097	(Armpad holder RH, for SW 33)	1
11	SP1543096	(Armpad holder LH, for SW 33)	1

**(Armrest, moulded)**

1497551PAC.dwg

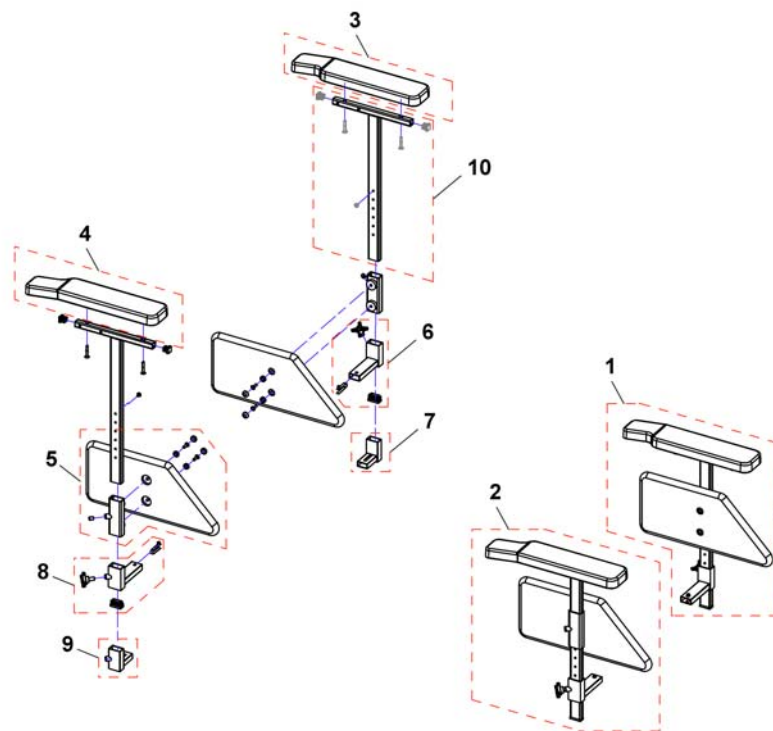

1	1498348	(Armrest moulded LH, cpl.)	1
1	1498396	ARMLÉN SB33-38 KPL VE BORA	1
2	1498347	(Armrest moulded RH, cpl.)	1
2	1498395	(Armrest moulded RH, for SW 33/38 cpl.)	1
3	5008438	ARMLÉNSPOLSTER VE G63	1
4	5008437	(Armpad RH)	1
5	SP1528410	FESTE KLESBESK NEDRE VE BORA	1
5	SP1528411	FESTE KLESBESK NEDRE HØ BORA	1
8	26003706	HOLDER ARMLÉN HØ BORA	1
9	SP1534391	HOLDER ARMLÉN HØ BORA <i>Use for Seatwidth 33cm</i>	1
6	26003707	HOLDER ARMLÉN VE BORA	1
7	SP1534392	HOLDER ARMLÉN VE BORA <i>Use for Seatwidth 33cm</i>	1
10	SP26004119	(Armpad holder RH/LH, for SW38/43)	1
10	SP1543095	(Armpad holder RH/LH, for SW 33)	1

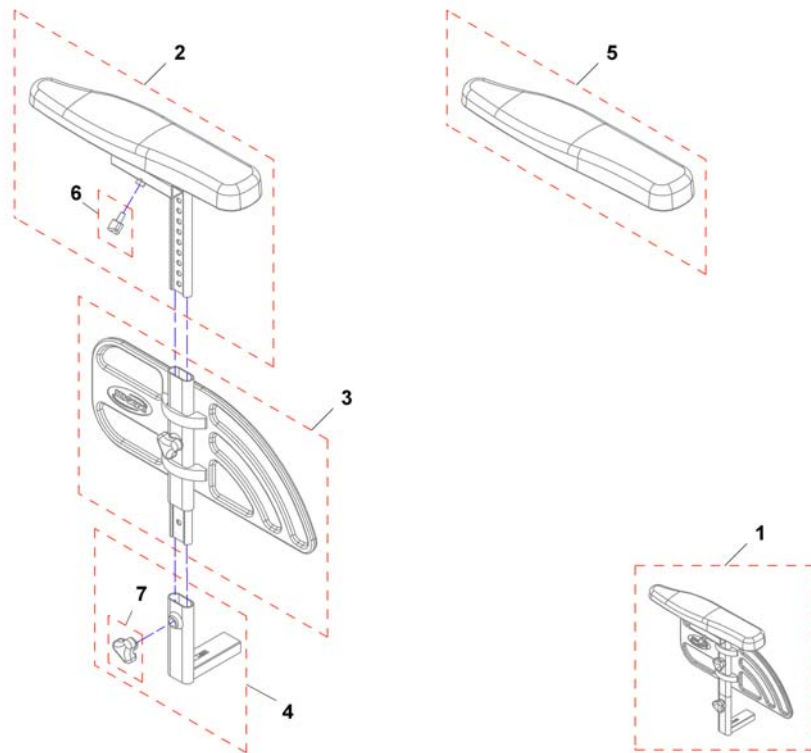

1	SP1441722	ARMLÉN ERGONOMISK MIRAGE	1
1	SP1441723	(Armrest LH, cpl.)	1
2	SP1441607	ARMLÉN TOP HØ ERGOMEDIC MIRAGE STD	1
2	SP1441608	ARMLÉN TOP VE ERGOMEDIC MIRAGE STD.	1
3	SP1436962	KLESBESKYT KPL MIRAGE HØ MIRAGE	1
3	SP1436963	KLESBESKYT KPL MIRAGE VE MIRAGE	1
4	1485103	ARMLÉN SFESTE LANG BORA	1
5	1433327	ARMLÉN SPOLSTER HØ MIRAGE	1
5	1433328	ARMLÉN SPOLSTER VE MIRAGE	1
6	L31001	VINGSKRUV M 6X10 GARANT G24 S	1
7	1434623	LÅSERATT 75MM STORM4	1

# (Armrest for Max Seat)

1571916.no




---

1	-	(Armrest long) [0670+0649]	1
2	-	(Armrest Hemiplegic) [0670+0671/0672]	1
3	-	(T-Armrest detachable) [0606+0649]	1





1552491X.dwg

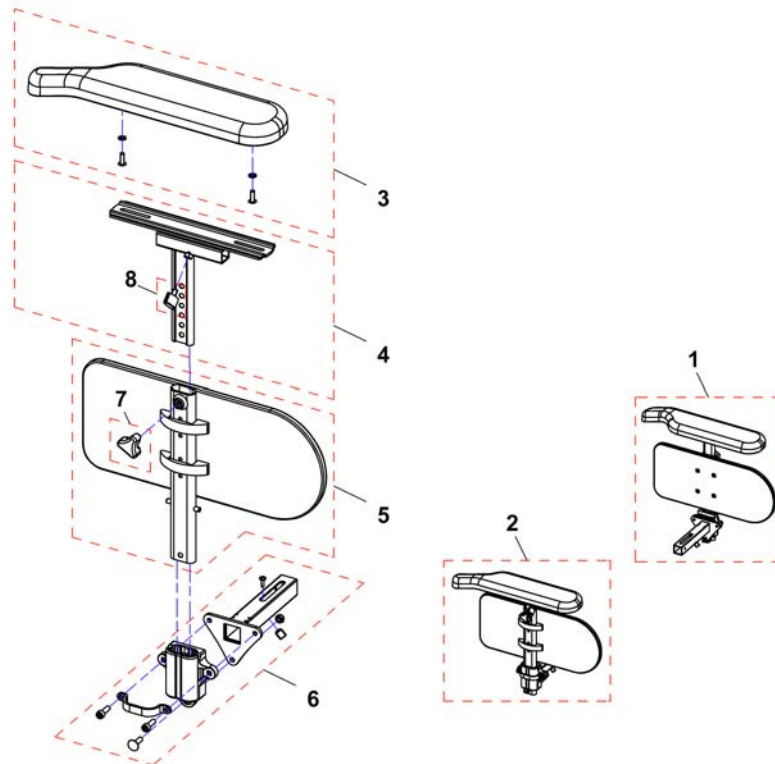

1	SP1552491	(Armrest compl. LH)	1
2	SP1552492	(Armrest compl. RH)	1
3	SP1528846	ARMLENSPOLSTER VE LANG STORM TDX BORA	1
3	SP1528845	ARMLENSPOLSTER HØ LANG STORM4 TDX BORA	1
4	SP1433317	(T-Arm, LH)	1
4	SP1433316	(T-Arm, RH)	1
5	1415521	SIDESTØTTE(KLESBESK.)VE >010507 KPL	1
5	1415520	SIDESTØTTE (KLESBESK.)HØ >010507 KPL	1
6	SP1525035	(Reinforced bracket HD armrest RH/LH)	1

## (Armrest for Modulite Seat)

1571738.no

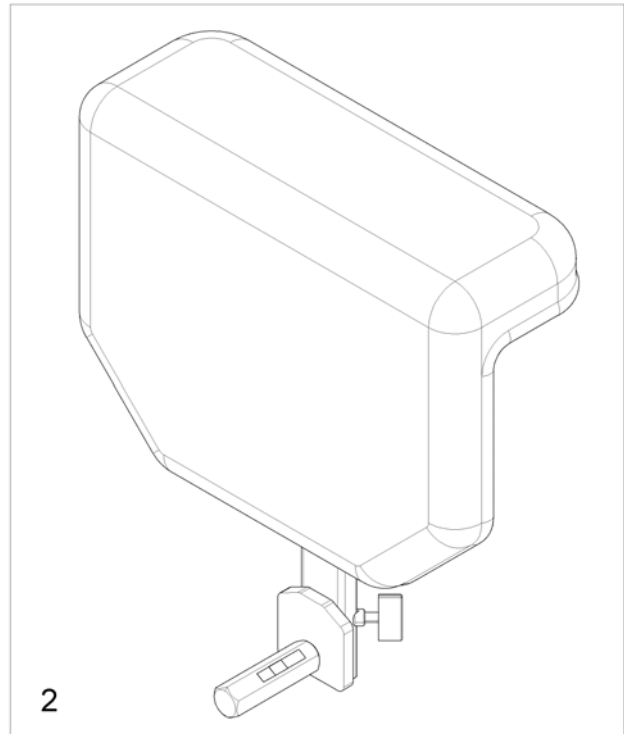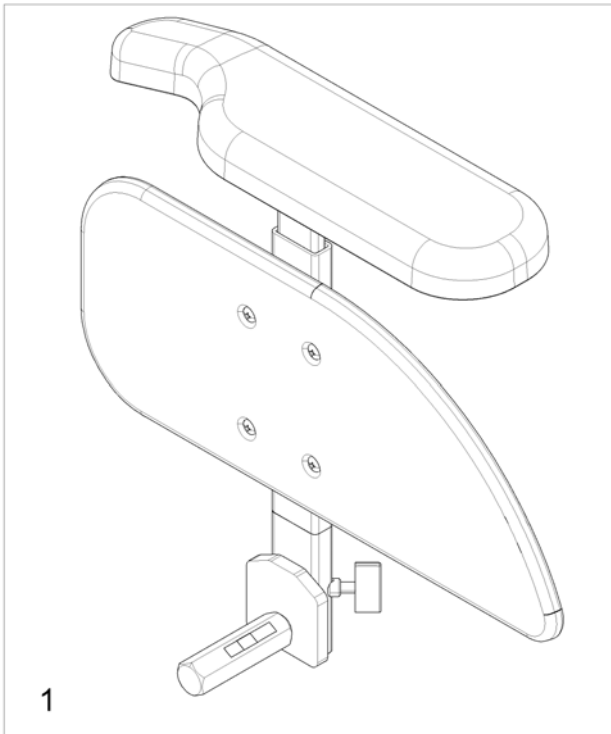


---

1	-	(T-Armrest with short/long Armpad) [2191+2195/2196]	1
2	-	(T-Armrest with padded Armpad) [2191+2198]	1

# (T-Armrest with short/long Armpad)


1572570 iso

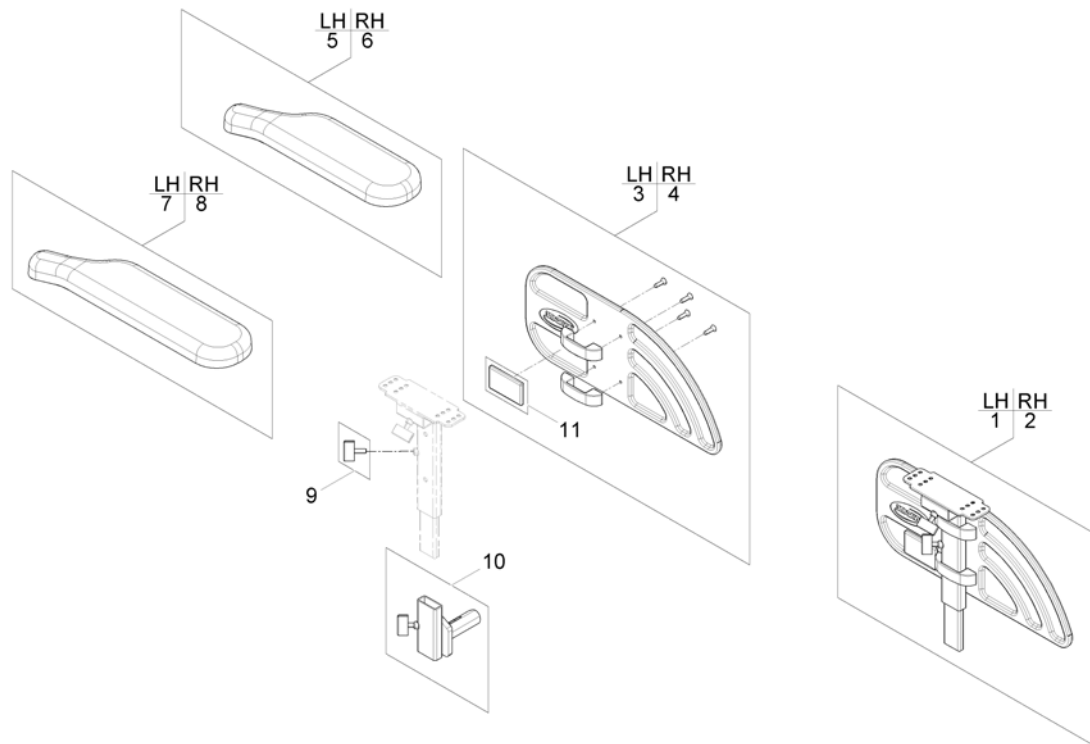


1	SP1570318	(Siderest LH)	1
2	SP1570368	(Siderest RH)	1
3	SP1572940	(Skirt guard LH)	1
4	SP1572948	(Skirt guard RH)	1
5	SP1443004	ARMLENSPOLSTER VE STORM STORM TDX BORA	1
6	SP1449043	ARMLENSPOLSTER HØ STORM STORM TDX BORA	1
7	SP1528846	ARMLENSPOLSTER VE LANG STORM TDX BORA	1
8	SP1528845	ARMLENSPOLSTER HØ LANG STORM4 TDX BORA	1
9	SPL10024	(Wing ScREW M06x10)	1
10	SP1567749	(Siderest bracket RH/LH)	1
11	SPP31298	(Reflector, yellow)	1

# (T-Armrest with padded Armpad)

1573214.00

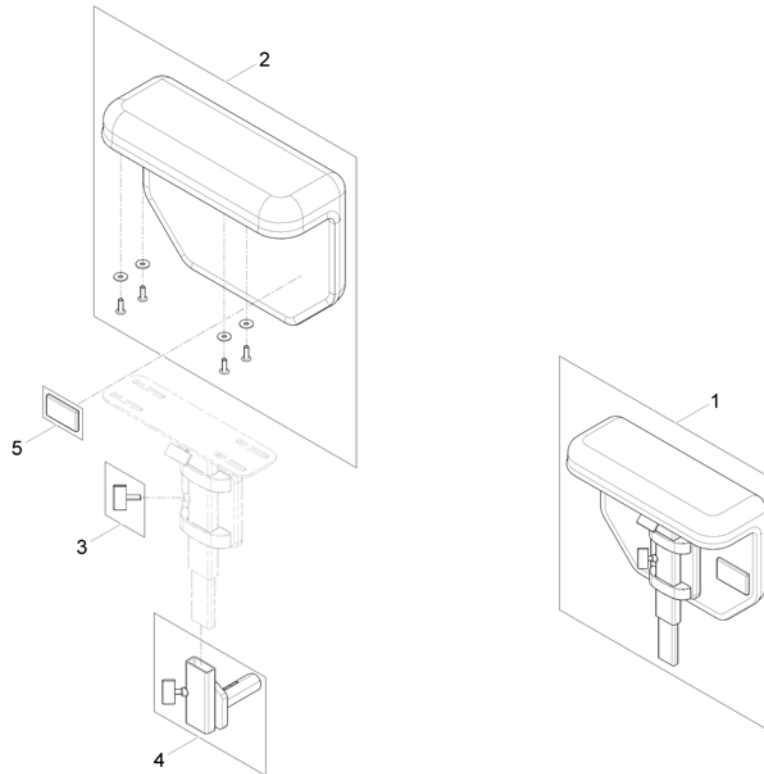


1	SP1571987	(Armrest RH/LH)	1
2	SP1571962	(Padded Armpad RH/LH with integrated cloth guard)	1
3	SPL10024	(Wing ScREW M06x10)	1
4	SP1567749	(Siderest bracket RH/LH)	1
5	SPP31298	(Reflector, yellow)	1

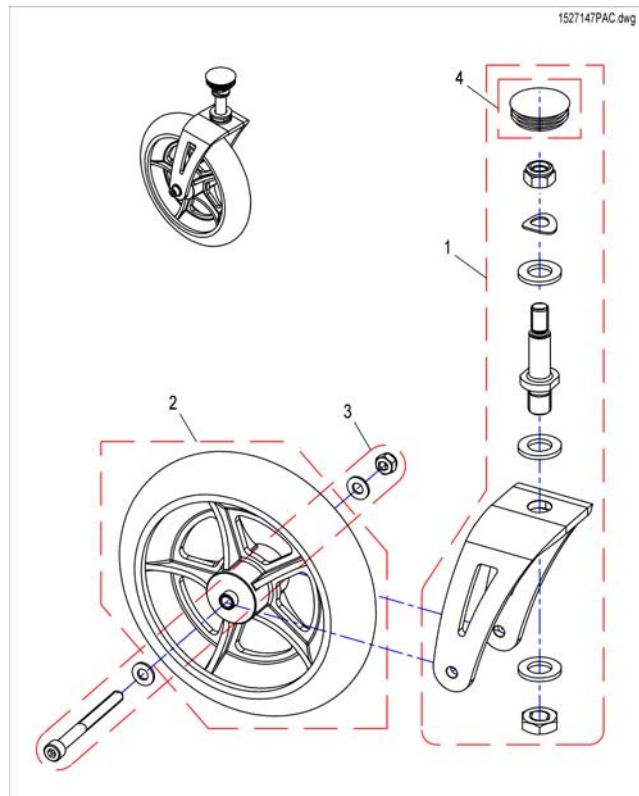


1	SP1527848	GAFFEL BORA	1
2	1418196	SVINGHJUL MEDIUM HARD 200*45(8")BORA BELLIS	1
3	SP1527849	FESTE GAFFEL BORA	1
4	1421687	PROPP GAFFEL SORT 42MM STORM4	1

Invacare® Bora | Sving-og Drivhjul  
**(Frontwheels 230x60)**

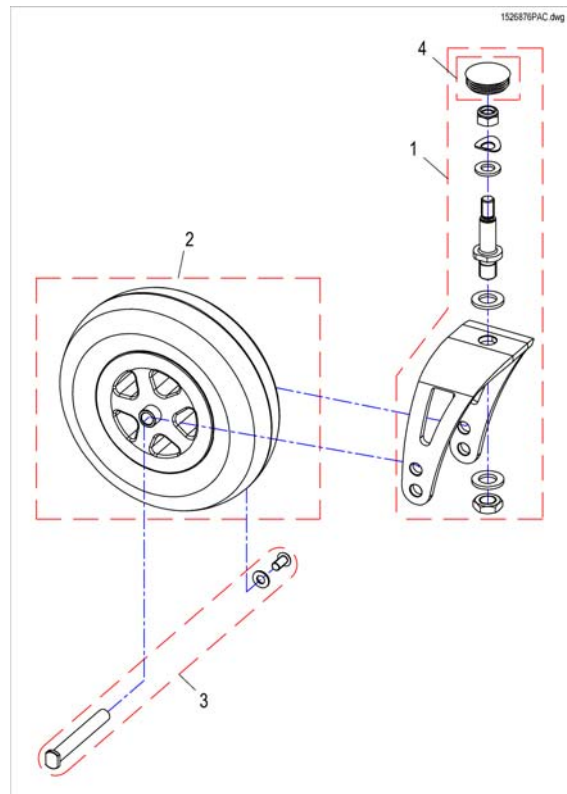


1	SP1527839	GAFFEL BORA	1
2	SP1478768	HJUL 2,8/2,5X4 BORA	1
2	1483962	HJUL LUFT 2,8/2,5X4 BORA	1
2	1513654	(Tyre 230x60 (C178-II NHS 2,80/2,50))	1
2	1513655	(Tube 230x60 (2,80/2,50-4 C178-II))	1
2	1513656	(Tyre 230x60, puncture proof (C178-II Grey NHS 2,80/2,50-4))	1
3	1442510	HJULAKSEL 8X85 M SKIVER OG MUTTER SVINGHJ.DRAGON	1
4	1421687	PROPP GAFFEL SORT 42MM STORM4	1

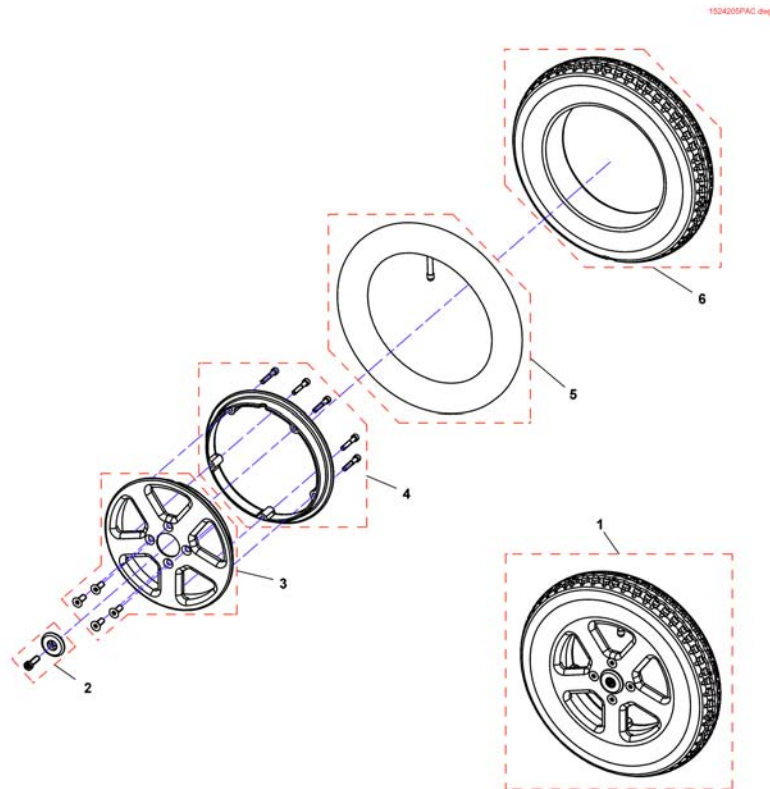


1	SP1544263	HJUL SORT KOMPAKT KPL BORA	1
1	SP1544264	HJUL SØLV KOMPACT KPL BORA	1
1	SP1544265	HJUL SORT LUFT KPL BORA BORA	1
1	SP1544266	HJUL SØLV LUFT KPL BORA BORA	1
2	SP1527853	NAVBRISKE SØLV BORA	1
2	SP1527854	NAVBRISKE SORT BORA	1
3	SP1527855	FELG SØLV BORA	1
3	SP1527856	(Outer Rim (black))	1
4	SP1527857	FELG INDRE SØLV BORA	1
4	SP1527858	FELG INDRE SORT BORA	1
5	M12004	SLANGE 12 1/4' X 2 1/4' DRIVHJUL TORNADO/TWISTER	1
6	23009262	DEKK KOMPACT 12,5" DRAGON	1
6	M11005	DEKK 12 1/2 X 2 1/4 LUFT GRÅ TWISTER	1

**(Motor =>31.07.2011)**

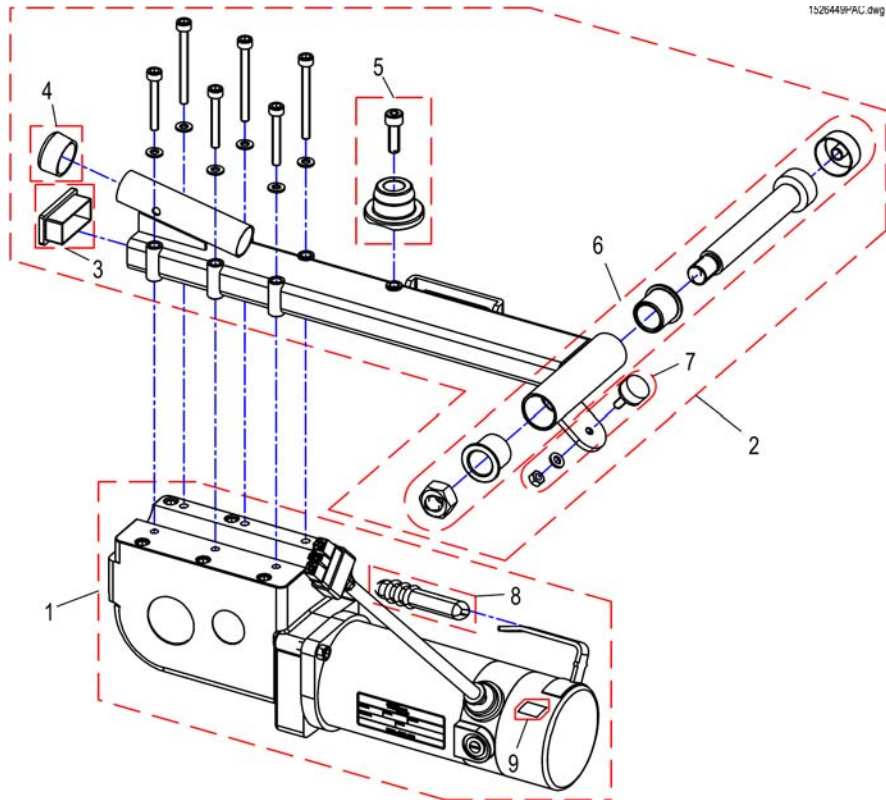

1	SP1527812	DRIVENHET 6KMT VE BORA BRUK SP1524195	1
1	SP1527813	DRIVENHET 6KMT HØ BORA BRUK SP1524194	1
1	SP1527814	DRIVENHET 10KMT VE BORA	1
1	SP1527815	DRIVENHET 10KMT HØ BORA	1
1	SP1536443	(Drive, 6km/h, R-Net, LH)	1
1	SP1536442	(Drive, 6km/h, R-Net, RH)	1
1	SP1536505	(Drive, 10km/h, R-Net, LH)	1
1	SP1536476	(Drive, 10km/h, R-Net, RH)	1
1	SP1536507	(Drive, 6km/h, VR2, LH)	1
1	SP1536506	(Drive, 6km/h, VR2, RH)	1
1	SP1536509	(Drive, 10km/h, VR2, LH)	1
1	SP1536508	(Drive, 10km/h, VR2, RH)	1
2	SP1530290	(Swing Arm LH)	1
2	SP1530289	(Swing Arm RH)	1
3	SP1524459	PLUGG FIRKANT BORA	1
4	SP500017	(Cap (round))	1
5	SP1527818	FJÆR SUPPORT BORA TDX	1
6	SP1527819	FESTEBOLT DRIVMOTOR BORA	1
7	SP1440366	(rubber buffer)	1
8	SP1101725	PROPP FOR FRIKPL HENDEL STORM4	1
9	SP1551712	(Labelset "Push/Drive")	1
10	SP1540358	(Kit, brushes with caps, 2-pole (Set of 2))	1



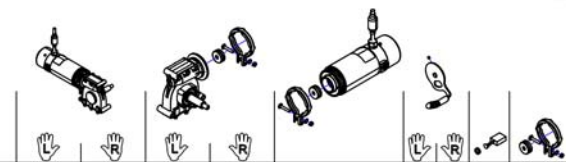

Model	Speed	Gear	ACS/ Shark		VR2		R-Net		FDX		Kite		Kite HD		Storm/ Storm X-glore		TDX-SP		TDX-SP Narrow	
			L	R	L	R	L	R	L	R	L	R	L	R	L	R	L	R	L	R
Bora/ SpectraXTR	6km/h	c	1	2	33	34	37	48	47	45	35	ACS/ Shark	54	55	33	34	39	49	50	46
		VR2	3	4	33	34	43	48	47	45			56	57	31	32	38	49	50	46
		R-Net	5	6	33	34	41	48	47	45			58	59	33	34	62	49	50	46
	10km/h	d	7	8	31	32	36	48	47	45		60	61	31	32	63	49	50	46	
		VR2	9	10	31	32	40	48	47	45		13	14	33	34	37	48	47	45	
		R-Net	11	12	31	32	42	48	47	45		15	16	31	32	38	53	53	46	
BoraHD/ Spectra XTR2HD	6km/h	ACS/ Shark	17	18	33	34	37	48	47	45	ACS/ Shark	21	22	33	34	39	49	50	46	
	10km/h	ACS/ Shark	19	20	31	32	36	48	47	45		23	24	31	32	44	49	50	46	
FDX	6km/h	ACS/ Shark	25	26	34	33	39	51	52	46	ACS/ Shark	25	26	34	33	44	51	52	46	
	10km/h	ACS/ Shark	27	28	32	31	44	51	52	46		ACS/ Shark	29	30	34	33	44	51	52	46
Kite	6km/h	ACS/ Shark	27	28	32	31	44	51	52	46	ACS/ Shark		25	26	34	33	44	51	52	46
	10km/h	ACS/ Shark	25	26	34	33	39	51	52	46			ACS/ Shark	27	28	32	31	44	51	52
Kite HD	6km/h	ACS/ Shark	25	26	34	33	39	51	52	46	ACS/ Shark	25		26	34	33	44	51	52	46
	10km/h	ACS/ Shark	27	28	32	31	44	51	52	46		ACS/ Shark	29	30	34	33	44	51	52	46
Storm/ Storm X-glore	6km/h	ACS/ Shark	27	28	32	31	44	51	52	46	ACS/ Shark		25	26	34	33	44	51	52	46
	10km/h	ACS/ Shark	27	28	32	31	44	51	52	46			ACS/ Shark	27	28	32	31	44	51	52
	6km/h	ACS/ Shark	25	26	34	33	39	51	52	46		ACS/ Shark		27	28	32	31	44	51	52
TDX-SP	6km/h	ACS/ Shark	25	26	34	33	39	51	52	46	ACS/ Shark		25	26	34	33	44	51	52	46
	8km/h	ACS/ Shark	27	28	32	31	44	51	52	46		ACS/ Shark	25	26	34	33	44	51	52	46
	10km/h	ACS/ Shark	27	28	32	31	44	51	52	46			ACS/ Shark	25	26	34	33	44	51	52
TDX-SP Narrow	6km/h	ACS/ Shark	25	26	34	33	39	51	52	46	ACS/ Shark	25		26	34	33	44	51	52	46
	10km/h	ACS/ Shark	27	28	32	31	44	51	52	46										

a) used from 28.09.11 / b) used from 07.09.11 / c) used from 18.04.11 / d) used from 01.08.11


-	N23004	KIL KRYSS DRIVAXEL (MO	1
1	SP1524195	DRIVENHET 6KMT VE BORA	1
2	SP1524194	DRIVENHET 6KMT HØ BORA	1
3	SP1541408	(Propulsion, cpl., Gear box i = 23:1, Motor, 2-pole, 250W, slow, 600mm cable, VR2)	1
4	SP1541410	(Propulsion, cpl., Gear box i = 23:1, Motor, 2-pole, 250W, slow, 600mm cable, VR2)	1
5	SP1541409	(Propulsion, cpl., Gear box i = 23:1, Motor, 2-pole, 250W, slow, 600mm cable, R-Net)	1
6	SP1541411	(Propulsion, cpl., Gear box i = 23:1, Motor, 2-pole, 250W, slow, 600mm cable, R-Net)	1
7	SP1524197	(Propulsion, cpl., Gear box i = 18:1, Motor, 2-pole, 250W, fast, 600mm cable)	1
8	SP1524196	(Propulsion, cpl., Gear box i = 18:1, Motor, 2-pole, 250W, fast, 600mm cable)	1
9	SP1541404	(Propulsion, cpl., Gear box i = 18:1, Motor, 2-pole, 250W, fast, 600mm cable, VR2)	1
10	SP1541406	(Propulsion, cpl., Gear box i = 18:1, Motor, 2-pole, 250W, fast, 600mm cable, VR2)	1
11	SP1541405	(Propulsion, cpl., Gear box i = 18:1, Motor, 2-pole, 250W, fast, 600mm cable, R-Net)	1
12	SP1541407	(Propulsion, cpl., Gear box i = 18:1, Motor, 2-pole, 250W, fast, 600mm cable, R-Net)	1
13	SP1168983	(Propulsion, cpl., Gear box i = 23:1, Motor, 2-pole, 250W, slow, 600mm cable)	1
14	SP1168984	(Propulsion, cpl., Gear box i = 23:1, Motor, 2-pole, 250W, slow, 600mm cable)	1
15	SP1168989	(Propulsion, cpl., Gear box i = 18:1, Motor, 4-pole, 340W, fast, 760mm cable)	1
16	SP1168988	(Propulsion, cpl., Gear box i = 18:1, Motor, 4-pole, 340W, fast, 760mm cable)	1
17	SP1568147	(Propulsion, 2-Pol, 230W, i = 23:1, 6km/h, ACS, LH, cpl.)	1
18	SP1568148	(Propulsion, 2-Pol, 230W, i = 23:1, 6km/h, ACS, RH, cpl.)	1
19	SP1568149	(Propulsion, 2-Pol, 230W, i = 18:1, 10km/h, ACS, LH, cpl.)	1
20	SP1568150	(Propulsion, 2-Pol, 230W, i = 18:1, 10km/h, ACS, RH, cpl.)	1

**(SSD-Motors)**

21	SP1568151	(Propulsion, 4-Pol, 340W, i = 23:1, 6km/h, ACS, LH, cpl.)	1
22	SP1568152	(Propulsion, 4-Pol, 340W, i = 23:1, 6km/h, ACS, RH, cpl.)	1
23	SP1568153	(Propulsion, 4-Pol, 340W, i = 18:1, 10km/h, ACS, LH, cpl.)	1
24	SP1568208	(Propulsion, 4-Pol, 340W, i = 18:1, 10km/h, ACS, RH, cpl.)	1
25	SP1536022	(Propulsion, cpl., Gear box i = 23:1, Motor, 4-pole, 340W, slow, 760mm cable)	1
26	SP1536021	(Propulsion, cpl., Gear box i = 23:1, Motor, 4-pole, 340W, slow, 760mm cable)	1
27	SP1541366	DRIVMOTOR KPL VE STORM4 SSD	1
28	SP1541365	DRIVMOTOR KPL HØ STORM4 SSD	1
29	SP1170307	DRIVMOTOR KPL VE TDX TDX SSD	1
30	SP1170306	DRIVMOTOR KPL HØ TDX TDX SSD	1
31	SP1163413	(Gear box i = 18:1)	1
32	SP1163412	(Gear box i = 18:1)	1
33	SP1163415	(Gear box i = 23:1)	1
34	SP1163414	(Gear box i = 23:1)	1
35	SP1542953	(Clamp band with clutch)	1
36	SP1163099	(Motor, 2-pole, 250W, fast, 600mm cable)	1
37	SP1164046	(Motor, 2-pole, 250W, slow, 600mm cable)	1
38	SP1165863	(Motor, 4-pole, 340W, fast, 760mm cable)	1
39	SP1165865	(Motor, 4-pole, 340W, slow, 760mm cable)	1
40	SP1544840	(Motor, 2-pole, 250W, fast, 600mm cable, VR2)	1
41	SP1544796	(Motor, 2-pole, 250W, slow, 600mm cable, R-Net)	1
42	SP1544841	(Motor, 2-pole, 250W, fast, 600mm cable, R-Net)	1
43	SP1544795	(Motor, 2-pole, 250W, slow, 600mm cable, VR2)	1
44	SP1165863	(Motor, 4-pole, 340W, fast, 760mm cable)	1
45	SP1540358	(Kit, brushes with caps, 2-pole (Set of 2))	1
46	SP1522967	BØRSTER 4 STK KIT TDX STORM 4	1
47	SP1168669	(Declutching lever)	1
48	SP1168668	(Declutching lever)	1
49	SP1548203	(Declutching lever)	1
50	SP1548137	(Declutching lever)	1
51	SP1541560	(Declutching lever)	1
52	SP1541561	(Declutching lever)	1
53	SP1169005	(Declutching lever)	1
54	SP1552782	(Propulsion, cpl., Gear box i = 23:1, Motor, 4-pole, 340W, slow, 760mm cable)	1
55	SP1552783	(Propulsion, cpl., Gear box i = 23:1, Motor, 4-pole, 340W, slow, 760mm cable)	1
56	SP1552872	(Propulsion, cpl., Gear box i = 18:1, Motor, 4-pole, 340W, fast, 760mm cable)	1
57	SP1552873	(Propulsion, cpl., Gear box i = 18:1, Motor, 4-pole, 340W, fast, 760mm cable)	1
58	SP1552832	(Propulsion, cpl., Gear box i = 23:1, Motor, 4-pole, 340W, slow, 760mm cable, R-Net)	1
59	SP1552833	(Propulsion, cpl., Gear box i = 23:1, Motor, 4-pole, 340W, slow, 760mm cable, R-Net)	1
60	SP1552836	(Propulsion, cpl., Gear box i = 18:1, Motor, 4-pole, 340W, fast, 760mm cable, R-Net)	1
61	SP1552837	(Propulsion, cpl., Gear box i = 18:1, Motor, 4-pole, 340W, fast, 760mm cable, R-Net)	1
62	SP1562498	(Motor, 4-Pol, 340W, slow, 760mm cable, R-Net)	1
63	SP1562499	(Motor, 4-Pol, 340W, fast, 760mm cable, R-Net)	1

**(Swingarm SSD-Motor 18.04.2011=>)**


1526344.dwg

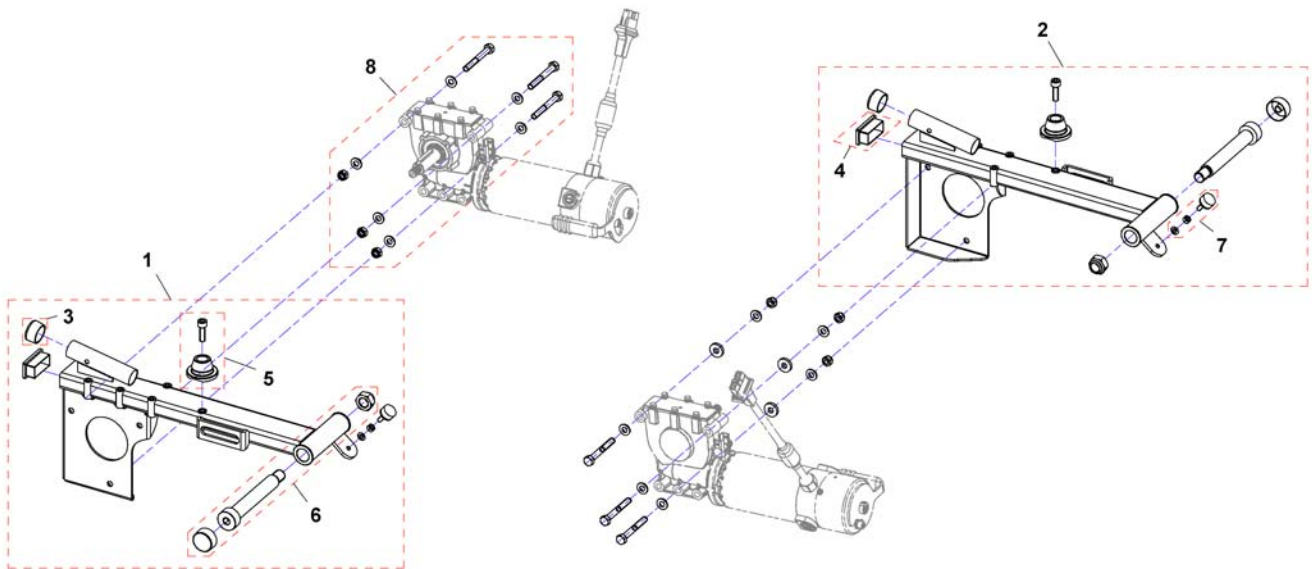

1	SP1526344	(Swingarm, cpl. RH)	1
2	SP1526345	(Swingarm, cpl. LH)	1
3	SP500017	(Cap (round))	1
4	SP1524459	PLUGG FIRKANT BORA	1
5	SP1527818	FJÆR SUPPORT BORA TDX	1
6	SP1527819	FESTEBOLT DRIVMOTOR BORA	1
7	SP1440366	(rubber buffer)	1
8	SP1548781	(Mounting-Kit SSD-Motor)	1

# (Hubs)


1168543X.dwg

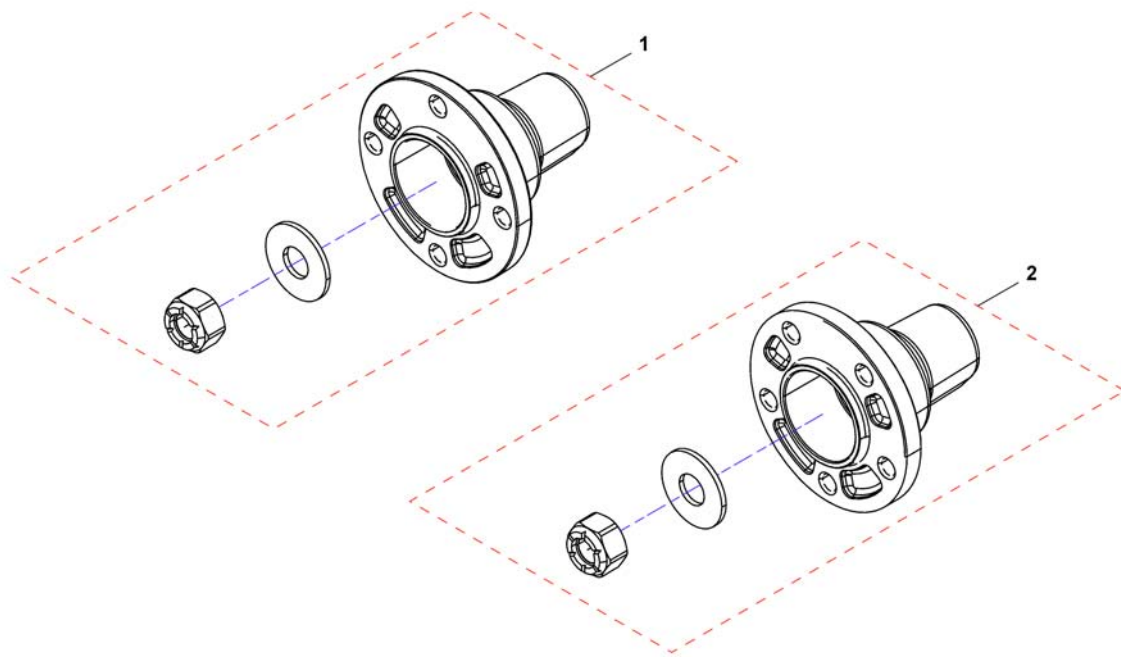

---

1	SP1542993	(Hub, 4-hole)	1
2	SP1542992	(Hub, 5-hole)	1

1527950PAC.dwg

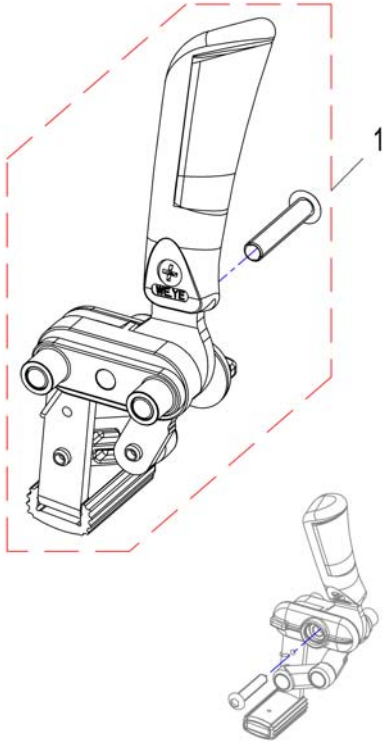


---

1	SP1527950	(Brake, LH)	1
1	SP1527949	(Brake, RH)	1

# Elektronikk

SPCBORATO.iss

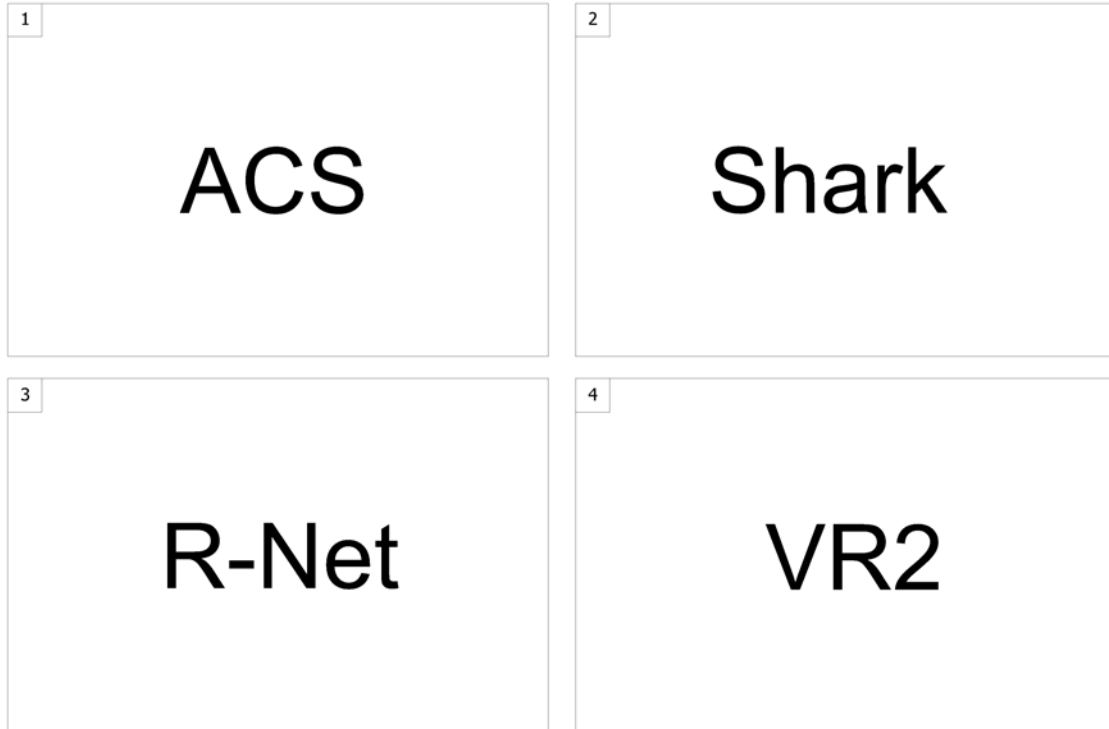

---

1	-	(ACS Electronic)	1
2	-	(Shark Electronic)	1
3	-	(R-Net Electronic)	1
4	-	(VR2 Electronic)	1

1500784.no

1  
**Modules**

2  
**Remote  
& Brackets**

3  
**Cables &  
Accessories**

---

1	-	(ACS Electronic Modules)	1
2	-	(ACS Remote & Brackets)	1
3	-	(ACS Cables & Accessories)	1

## (ACS Electronic Modules)

---

-	(Powermodules)	1
-	(Holder for Electronic ACS 2)	1
-	(ACT 2 for ACS 1, with Holder)	1
-	(ACT-Brackets)	1
-	(Lightboard)	1
-	(Lightboardholder Shark/ACS2)	1
-	(G-Trac-sensor)	1
-	(Holder for Gyro)	1



Powermodule.xls


	Dynamics											Penny & Giles					
	Shark			ACS					ACS 2			Pilot +	VR2		R-net		
	DK-PMB01	DK-PMB21	DK-PMC28	DX-PMB-AS	DX-PMB-2	DX-PMB2-SAS	DX-GB-AS	DX-GB-ASG	DX-GB-AS II	DX2-PMA70	DX2-PMA70L	DX2-PMA90L	DX2-PMA90LG	Pilot+ 50-xM 50A	VR2 60	VR2 60-A2	R-net EL PM90
Bora/SpectraXTR	2	3								12	13	14	15		25	26	27
Dragon	2	3			28												
Dragon Vertic	2	3			29												
FDX																	
G40						7				12	13	14	15				
G50						7											
Mirage	2	3												24	25	26	
Mistral3	2			5													
Pronto M61	2	3															
Spectra Blitz	2	3		5										24	25	26	27
Spectra Comfort	2	3		5						12	13			24	25	26	27
Spectra Plus	2	3		5						12	13			24	25	26	27
Spectra Seat Riser	2	3		5										24	25	26	27
Storm <sup>3</sup>	2	3		5	30												
Storm <sup>3</sup> GB							9										
Storm <sup>3</sup> Vertic					31												
Storm4												14	15				
Storm4 MAX												14	15				
Stream												14	15				
TDX-SP			4														
Tiger		3	4				28			12	13						
Topan	2	3					28										
Tornado	2	3					NLA										
Typhoon							NLA										
Typhoon GB								9									
Typhoon II								9									
Zephyr									10								

NLA = No Longer Available

-	SP1555702	(Gasket kit for ACS2, Powermodule and ACT)	1
2	1433332	POWERMODUL DK-PMB2 SHARK MIRAGE	1
3	1448744	POWERMODUL SHARK MED EL FUNK DX-PMB21 ACSP3506	1
4	1136898	(Powermodule, Shark, 90A, 2 actuators, DK-PMC28)	1
5	6001423	POWERMODUL 60 AH DX-PMB-AS	1
7	1451448	(Powermodule, ACS, 160A, DX-PMB2-SAS)	1
9	1430150	POWERMODUL GB MOTORER	1
10	AT.1496444	(Powermodule for GB-motors, ACS, 100A, (only Zephyr), DX-GB-ASG)	1
12	SP1528673	POWERMODUL BORA INNE BORA U/LYS DX-PMA70	1
13	SP1523433	POWERMODUL BORA EXTRA BORA M LYS DX-PMA70L	1
14	SP1524244	POWERMODUL DX2 STORM4	1
15	SP1537975	POWERMODUL ACS2 FOR GYRO ACS2	1
24	30000265	(Powermodule, Pilot +, 50A, Pilot+ 50-xM)	1
25	SP1504056	(Powermodule, VR2, 60A, only drive, VR2 60)	1
26	SP1510131	(Powermodule, VR2, 60A, 2 actuators, VR2 60-A2)	1
27	SP1532445	POWERMODUL BORA STORM4	1
28	SP1570293	(Upgrade-Kit ACS 80A ==> ACS2)	1
29	SP1570367	(Upgrade-Kit DragonVertic ACS 80A ==> ACS2)	1
30	SP1570281	POWERMODUL KIT PMB =>DX2 ERSTATTER DX-PMB	1
30	SP1570290	(Upgrade-Kit ACS 80A ==> ACS2, Storm <sup>3</sup> , only light, lifter)	1
31	SP1570369	(Upgrade-Kit ACS 80A ==> ACS2, Storm <sup>3</sup> Vertic)	1

## (Holder for Electronic ACS 2)


1530726X.dwg

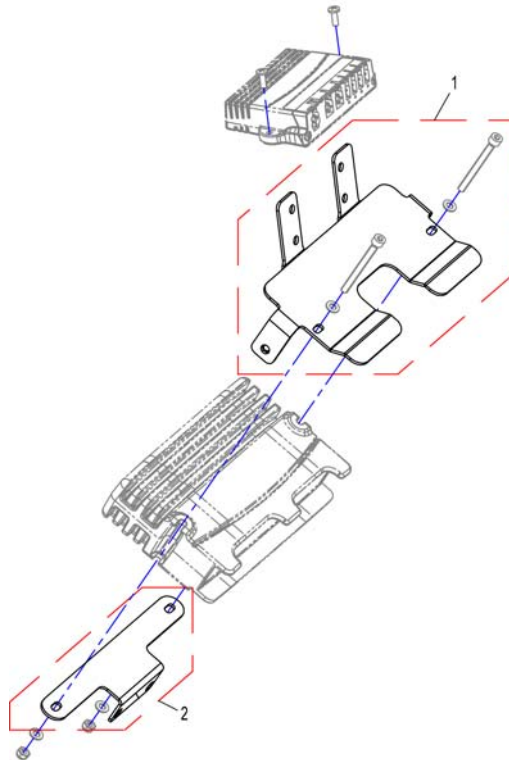

---


1	SP1530583	HOLDER ELEKTRONIKK BORA	1
2	SP1530584	HOLDER POWERMODULE BORA	1

1527842PAC.dwg

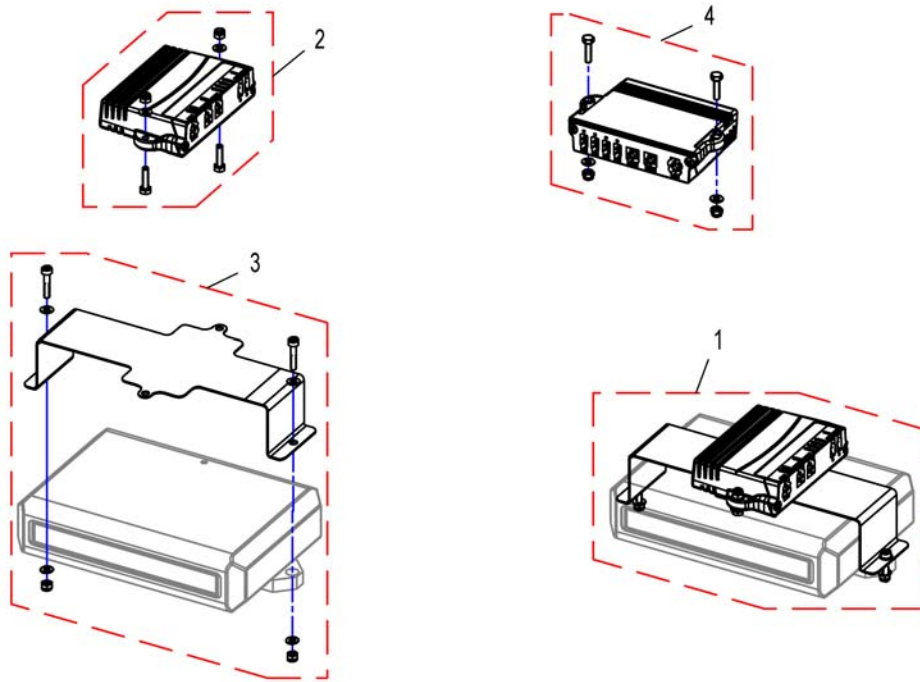


1	SP1527842	(ACT2 for ACS1, kmpl. with Holder)	1
2	SP1528542	ACTUATORMODUL ACT2 DX2 STORM4 TDX BORA	1
3	SP1528543	HOLDER ACT2 BORA	1
4	SP1527170	(ACT4, cpl.)	1

# (ACT-Brackets)

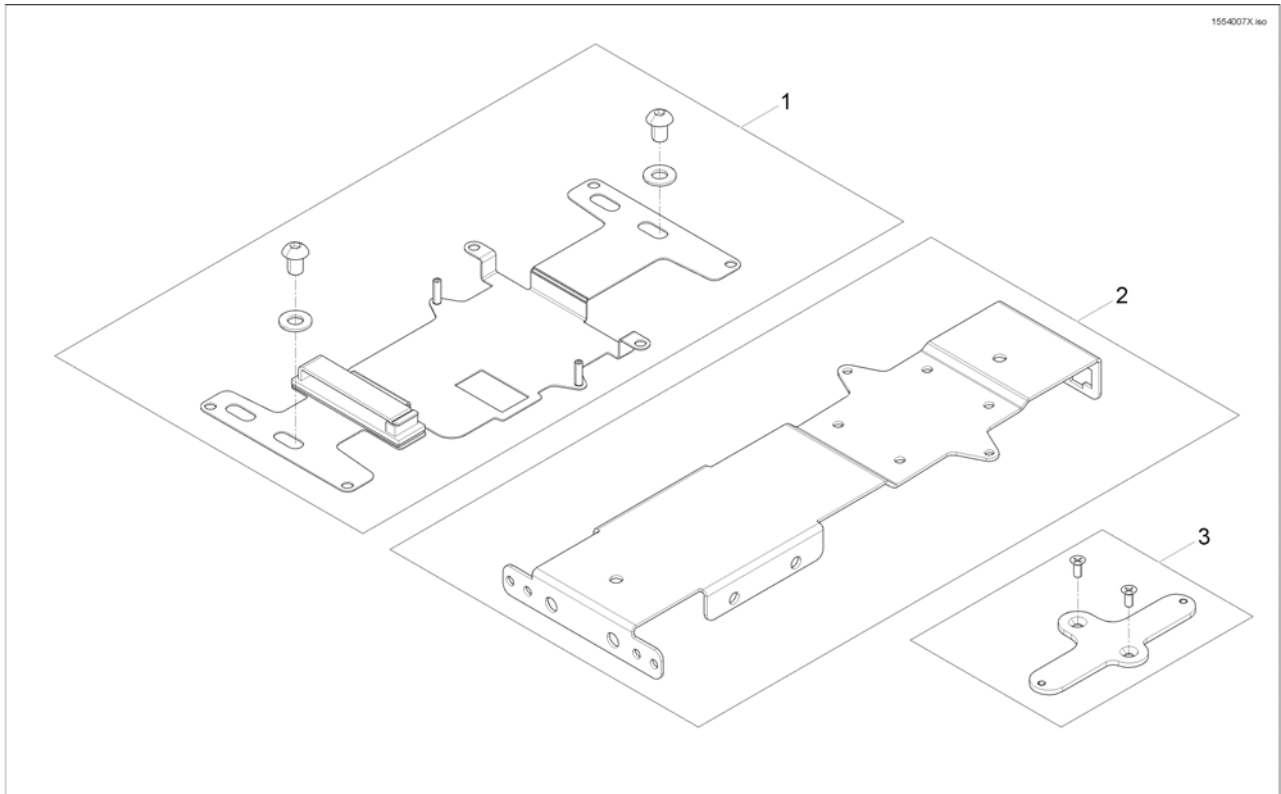

---

1	SP1569960	(ACT bracket)	1
2	SP1564650	(Multi support bracket)	1
3	SP1554007	(ACT cross bracket)	1

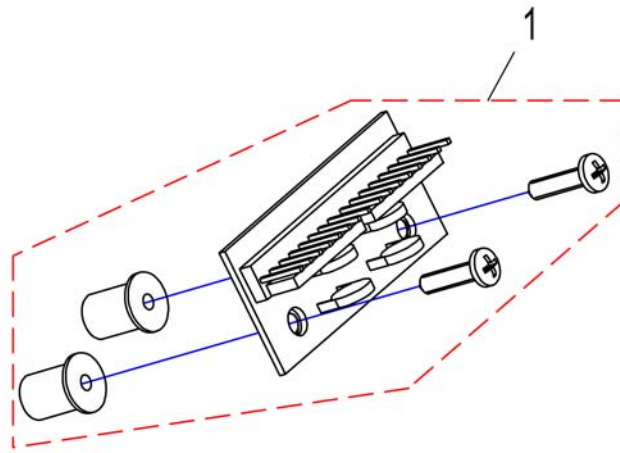


1 SP1528550

(Lightboard for all elektronikk systems)

1


# (Lightboardholder Shark/ACS2)

1527841PAC.dwg

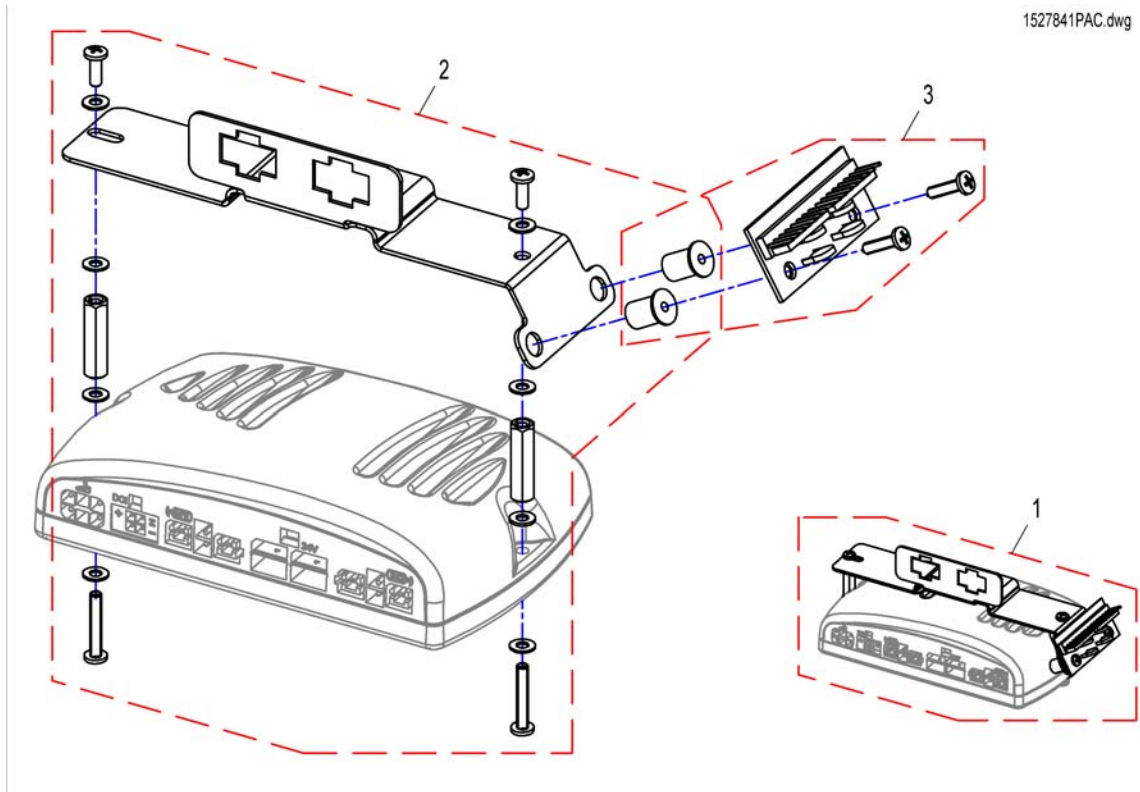

1	SP1527841	LYSFORDELER BORA	1
2	SP1528412	HOLDER LYSMODUL BORA	1
3	SP1528550	(Lightboard for all elektronik systems)	1


1537848X.dwg

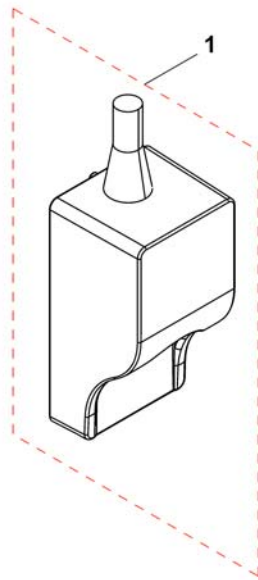

---

1 SP1541535 (G-Trac)

1

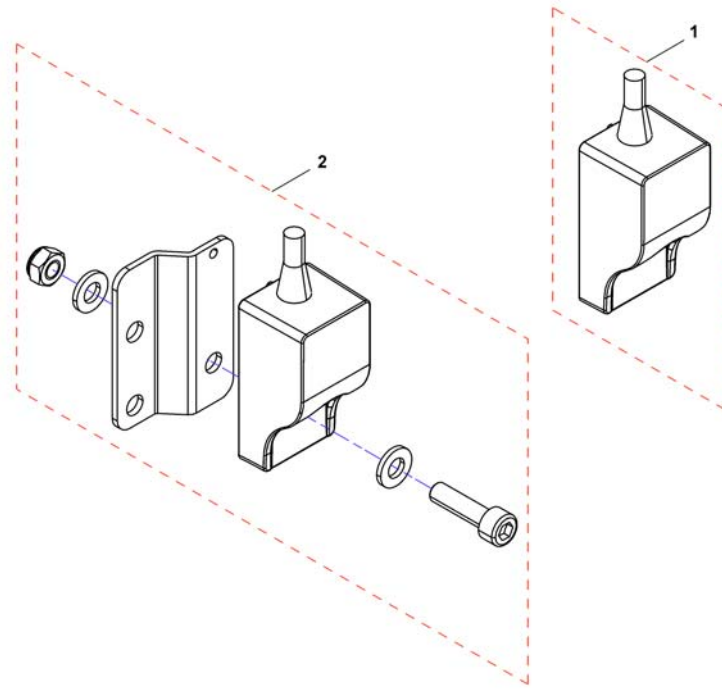


1 SP1541724

(Holder Gyro)

1




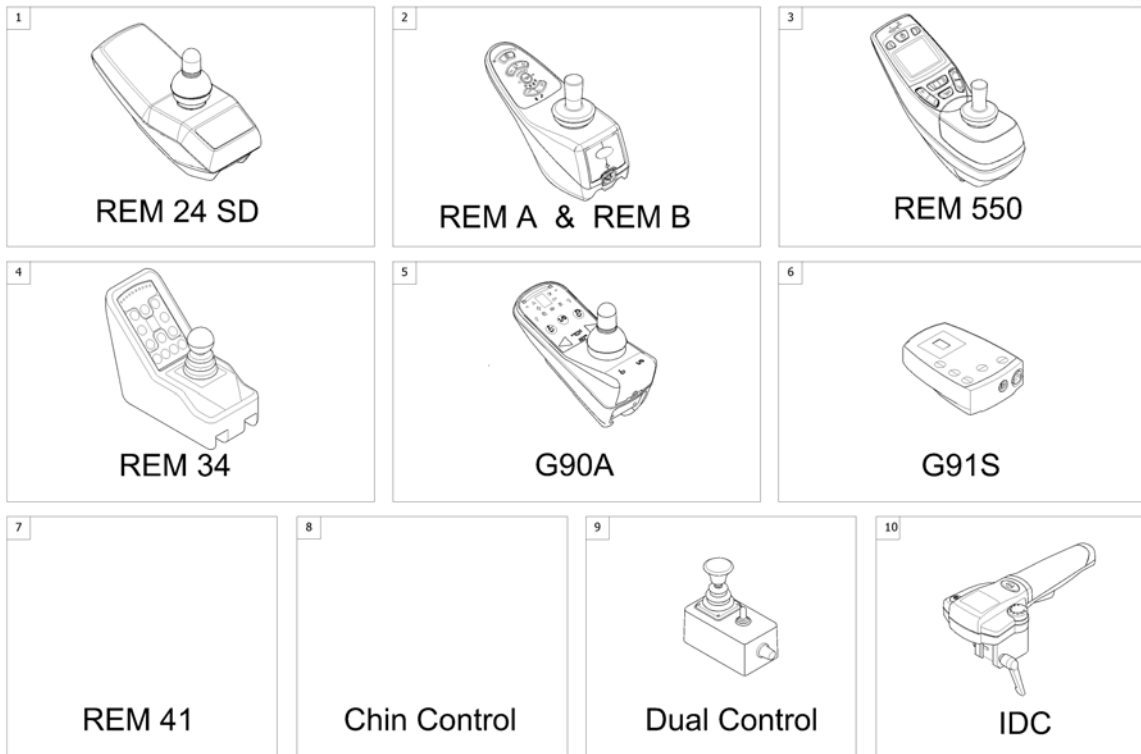


1	-	(Remote REM 24 SD with Brackets) [1601/1602]	1
2	-	(Remote REM A & B with Brackets) [1638/1639/1643/1644]	1
3	-	(Remote REM 550 with Brackets) [1617/1618]	1
4	-	(Remote REM 34 with Brackets) [1611]	1
5	-	(Remote G 90 A with Brackets) [1614/1616]	1
6	-	(Remote G91S with Brackets) [1615]	1
7	-	(Centre control ACS REM41) [1607/1608]	1
8	-	(Remote Small Chin Control with Brackets) [1609]	1
9	-	(Remote Dual Control with Brackets) [1291/1292]	1
10	-	(Remote IDC Dual Control with Brackets) [1645]	1

## (Remote REM 24 SD with Brackets)

1560836 iso

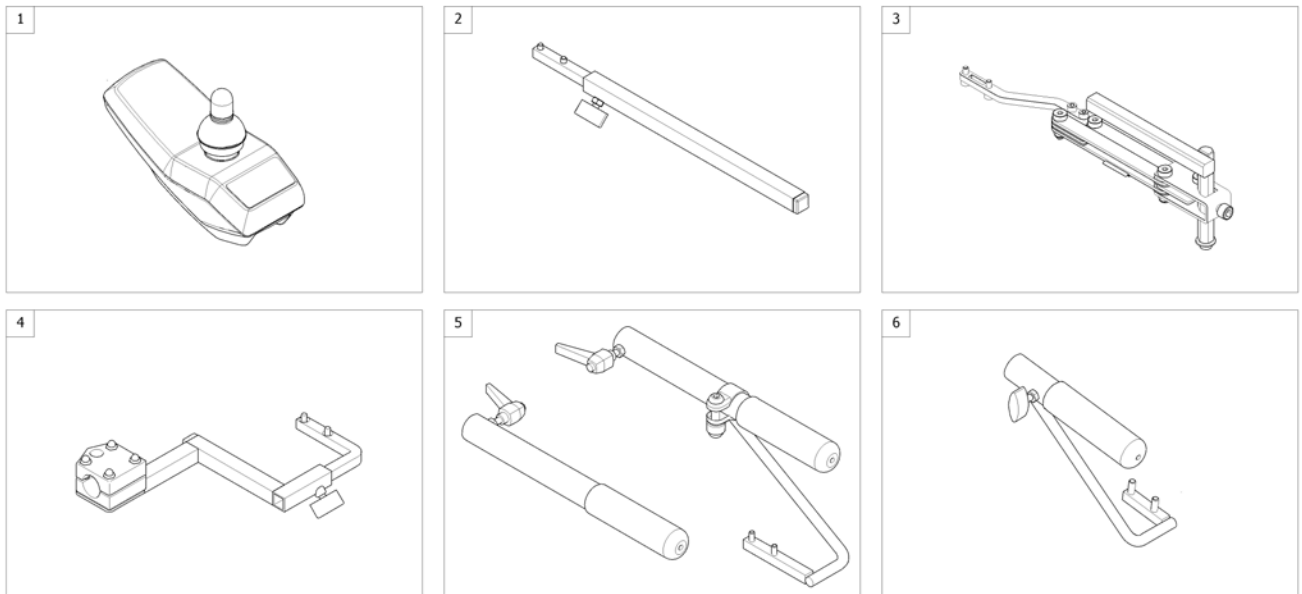

1	1436541	(Remote REM24SD) [1601/1602]	1
2	-	(Remote Bracket Standard) [1601+1207/1208]	1
3	-	(Remote Bracket Swing away & height adjust.) [1601+1707/1708]	1
4	-	(Remote Bracket Attendant for Standard-/Flex2-/Max-Seat) [1602+1207/1208]	1
5	-	(Remote Bracket Attendant long for Firm Seat) [1602+1207/1208]	1
6	-	(Remote Bracket Attendant short for Firm Seat) [1602+1207/1208]	1

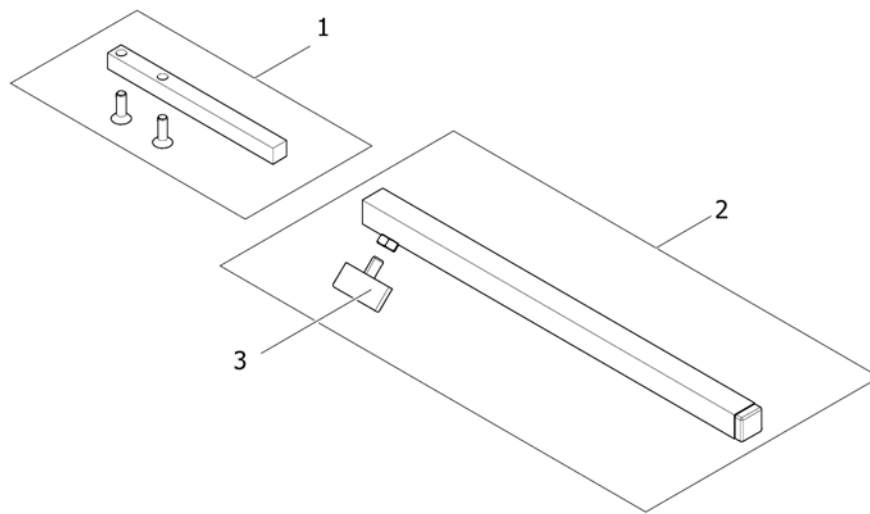


---

1	SP1436542	HOLDER DX-REM 24 SD RETT STORM SPORT/3 OPPFL.ARML	1
2	SPS183040	STYREBOKSHOLDER STD KPL STORM3 G40	1
3	L31001	VINGSKRUV M 6X10 GARANT G24 S	1

# (Remote Bracket Swing away & height adjust.)

1559852.iso

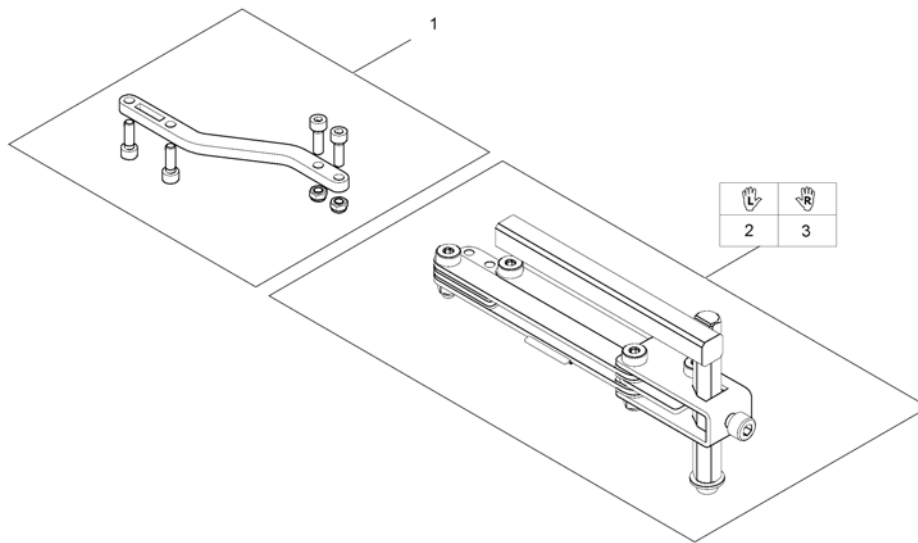



---

1	SP1498222	FESTE FOR STYREBOKST TIL SWING AWAY	1
2	SP1496249	SWING AWAY VE NY MOD2008 KPL	1
3	SP1496248	SWING AWAY HØ NY MOD2008 KPL	1

# (Remote Bracket Attendant for Standard-/Flex2-/Max-Seat)


1560837.no

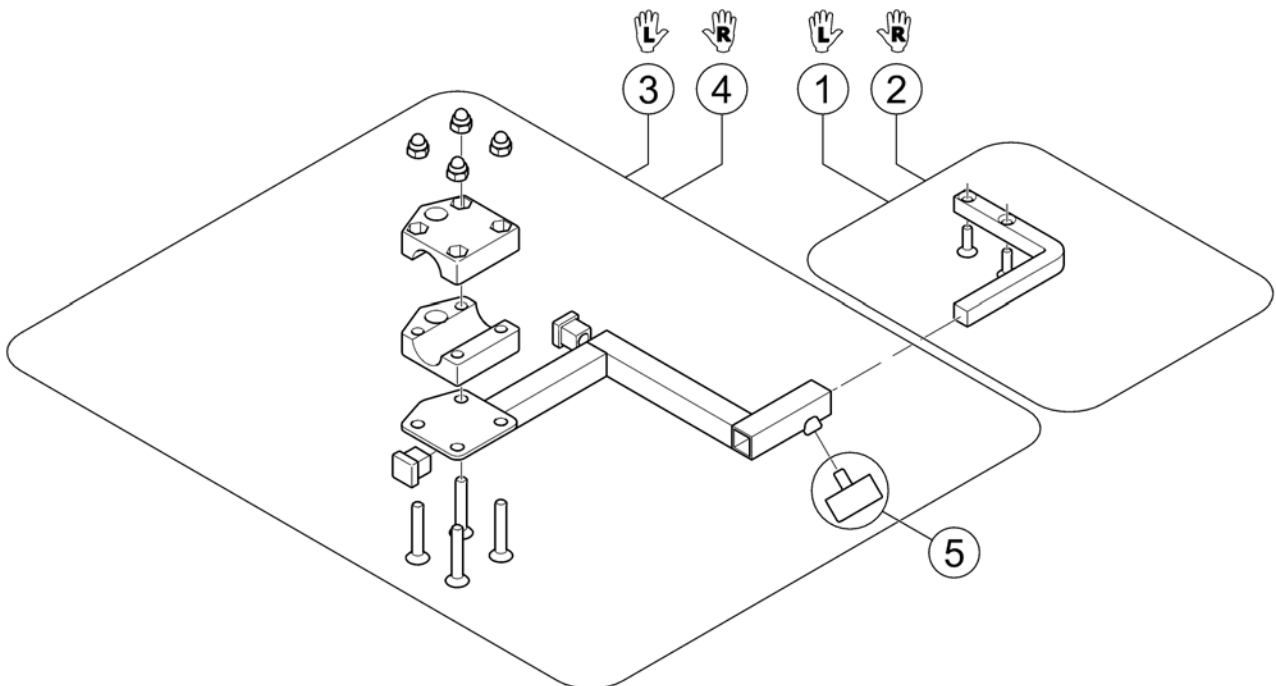

1	1422806	(Remote Support LH)	1
2	1422810	(Remote Support RH)	1
3	SP1422937	(Bracket for attendant control LH)	1
4	SP1422936	(Bracket for attendant control RH)	1
5	L31001	VINGSKRUV M 6X10 GARANT G24 S	1

## (Remote Bracket Attendant long for Firm Seat)


1559854.iso

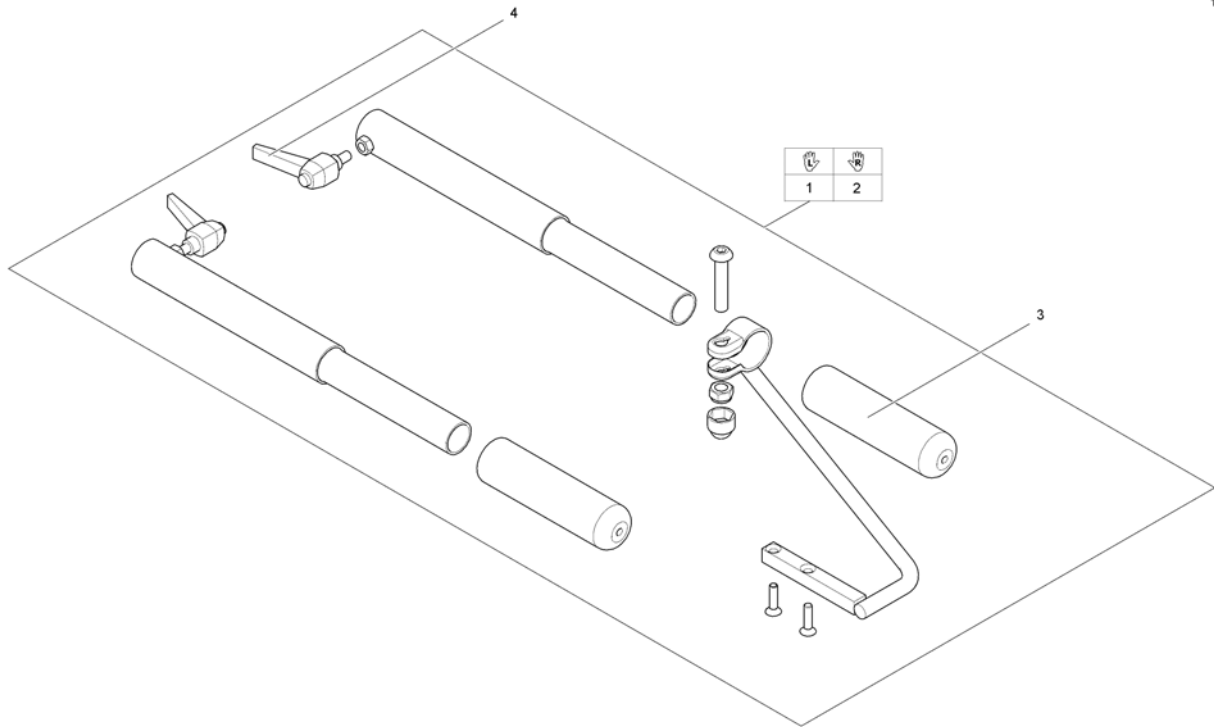

1	SP1511739	(Remoteholder for Attendant Control ACS & Shark LH)	1
2	SP1511766	(Remoteholder for Attendant Control ACS & Shark RH)	1
3	1435421	HÅNDTAK Ø22	1
4	L32902	LÅSESKRUE MED HENDEL ARMLEN M6X10 STORM/G40	1

# (Remote Bracket Attendant short for Firm Seat)


1559853.iso

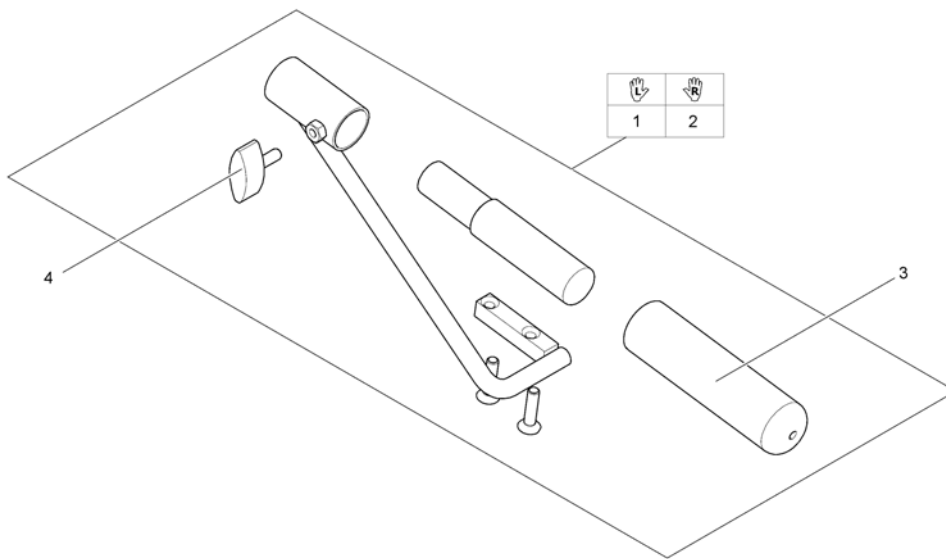

1	SP1544050	(Remote Bracket Attendant short for Firm Seat LH)	1
2	SP1544049	(Remote Bracket Attendant short for Firm Seat RH)	1
3	1435421	HÅNDTAK Ø22	1
4	L10024	SKRUE M6X16	1

## (Remote REM A & B with Brackets)

1560839 iso

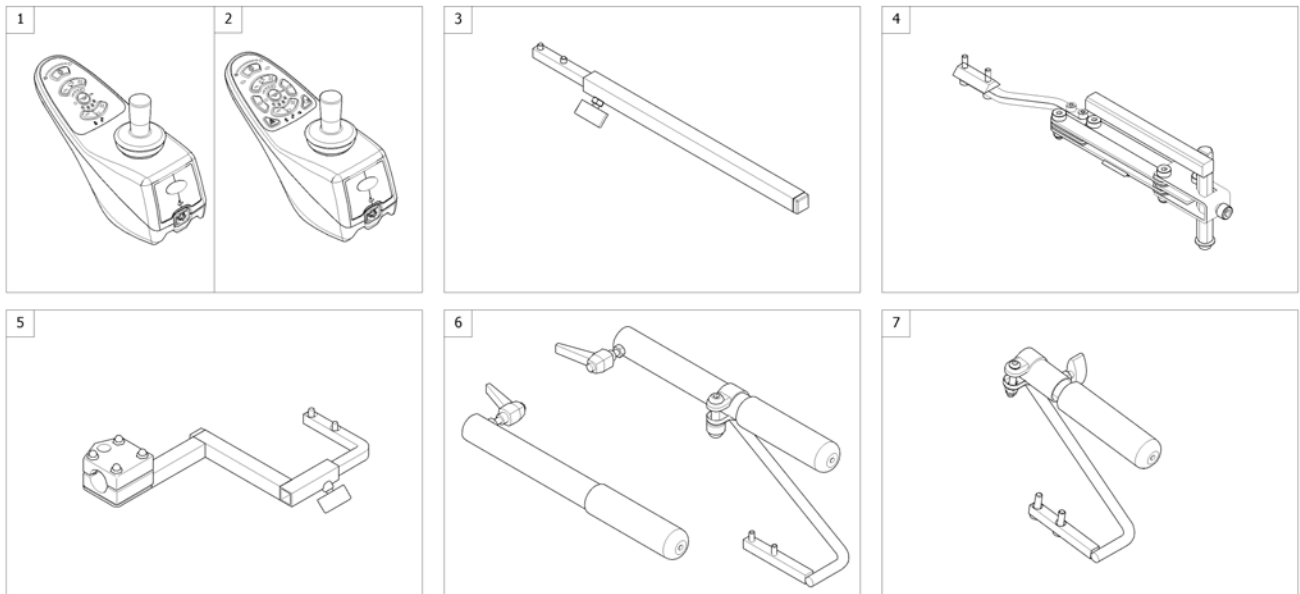


1	SP1523440	(Remote "REM A") [1638/1643]	1
2	SP1523445	(Remote "REM B") [1639/1644]	1
3	-	(Remote Bracket Standard) [1638/1639+1207/1208]	1
4	-	(Remote Bracket Swing away & height adjust.) [1638/1639+1707/1708]	1
5	-	(Remote Bracket Attendant for Standard-/Flex2-/Max-Seat) [1643/1644+1207/1208]	1
6	-	(Remote Bracket Attendant long for Firm Seat) [1643/1644+1207/1208]	1
7	-	(Remote Bracket Attendant short for Firm Seat) [1643/1644+1207/1208]	1



# (Remote Bracket Swing away & height adjust.)


1559857.iso

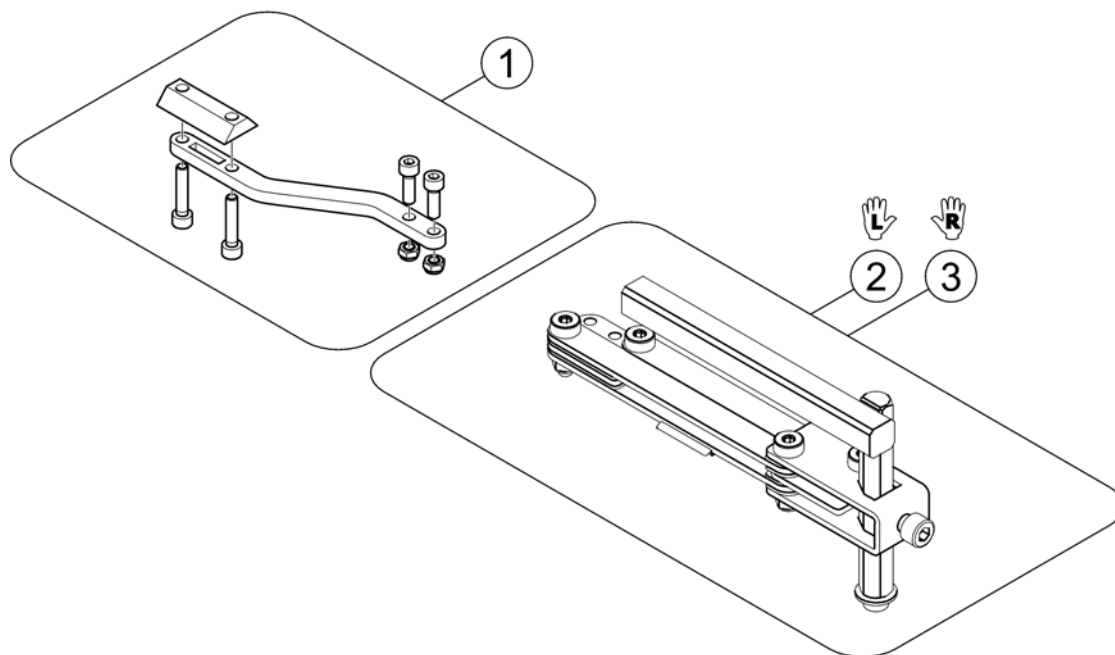

1	SP1536180	HOLDER STYREBOKS SHARK/REM A/REM B	1
2	SP1496249	SWING AWAY VE NY MOD2008 KPL	1
3	SP1496248	SWING AWAY HØ NY MOD2008 KPL	1


## (Remote Bracket Attendant short for Firm Seat)

1559858.iso

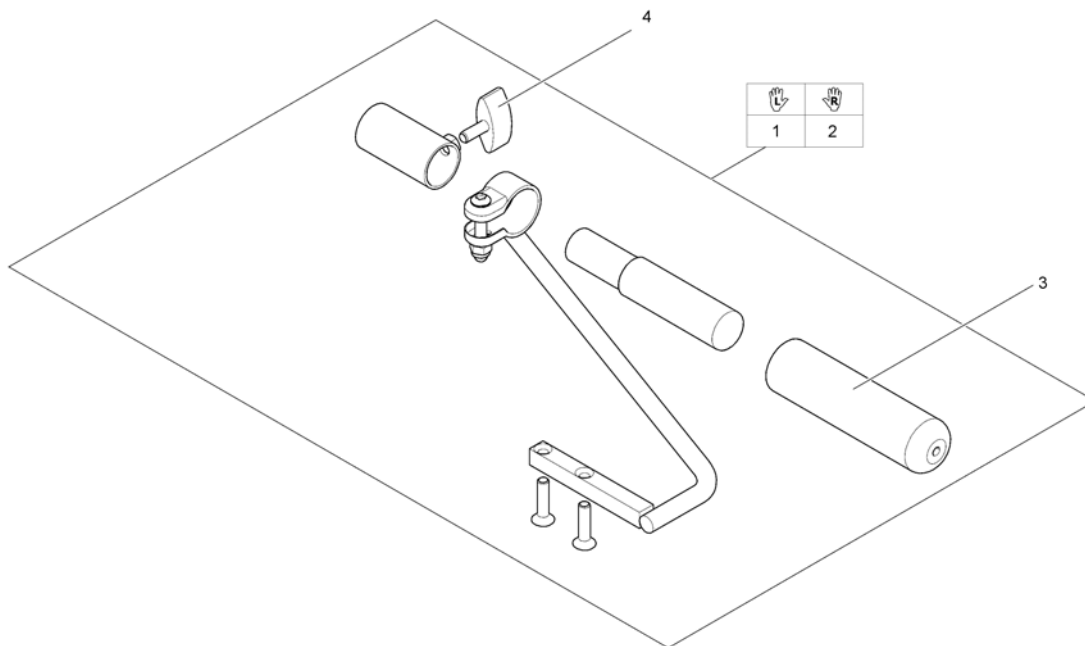

1	SP1541705	(Remote Bracket Attendant LH short)	1
2	SP1541706	(Remote Bracket Attendant RH short)	1
3	1435421	HÅNDTAK Ø22	1
4	L10024	SKRUE M6X16	1



1560840 iso

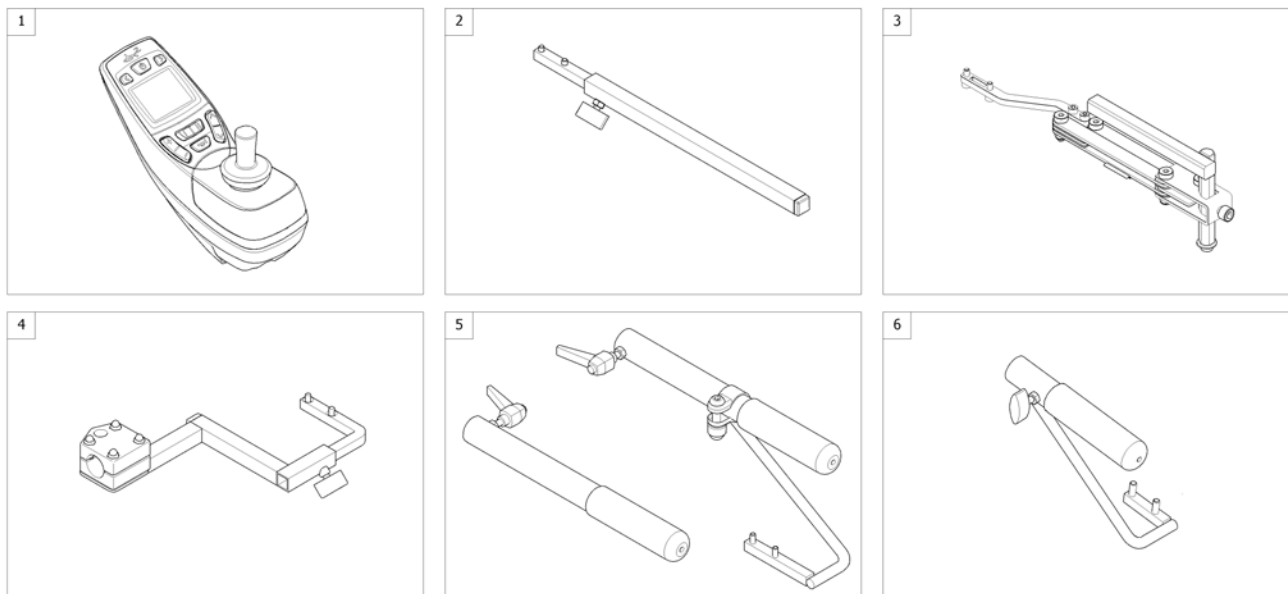


1	SP1523444	STYREBOKS REM550 STORM4 TDX BORA [1617/1618]	1
2	-	(Remote Bracket Standard) [1617+1207/1208]	1
3	-	(Remote Bracket Swing away & height adjust.) [1617+1707/1708]	1
4	-	(Remote Bracket Attendant for Standard-/Flex2-/Max-Seat) [1618+1207/1208]	1
5	-	(Remote Bracket Attendant long for Firm Seat) [1618+1207/1208]	1
6	-	(Remote Bracket Attendant short for Firm Seat) [1618+1207/1208]	1

# (Remote REM 34 with Brackets)

1560846 iso

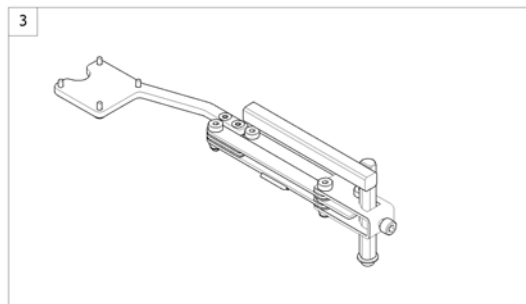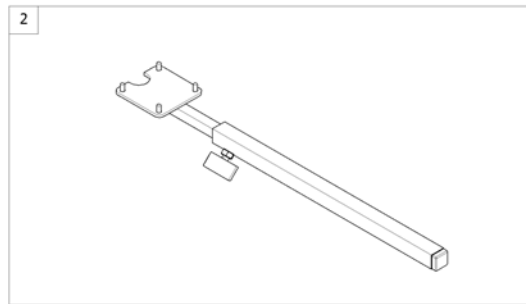

1	SP1440677	(Remote REM 34) [1611]	1
2	-	(Remote Bracket Standard) [1611+1207/1208]	1
3	-	(Remote Bracket Swing away & height adjust.) [1611+1707/1708]	1

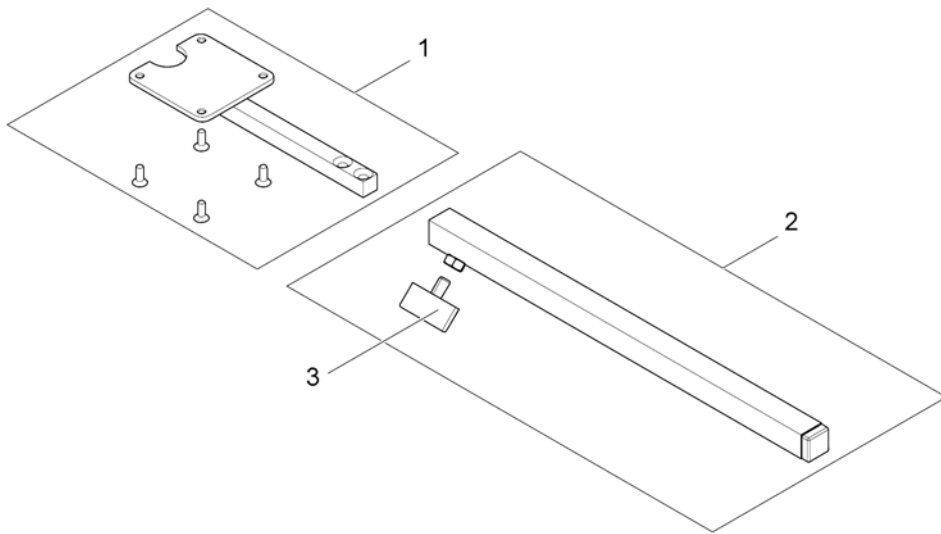


---


1	SP1434921	HOLDER FOR REM 34 STORM4	1
2	SPS183040	STYREBOKSHOLDER STD KPL STORM3 G40	1
3	L31001	VINGSKRUV M 6X10 GARANT G24 S	1

# (Remote Bracket Swing away & height adjust.)

1560845.iso

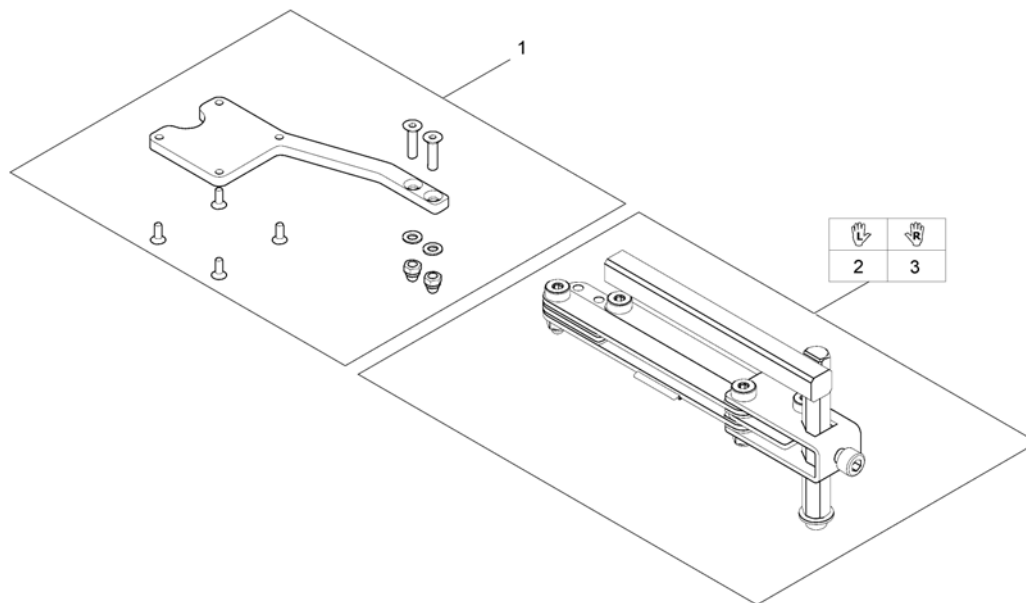


1	SP1527125	FESTE FOR REM34 STORM4	1
2	SP1496249	SWING AWAY VE NY MOD2008 KPL	1
3	SP1496248	SWING AWAY HØ NY MOD2008 KPL	1

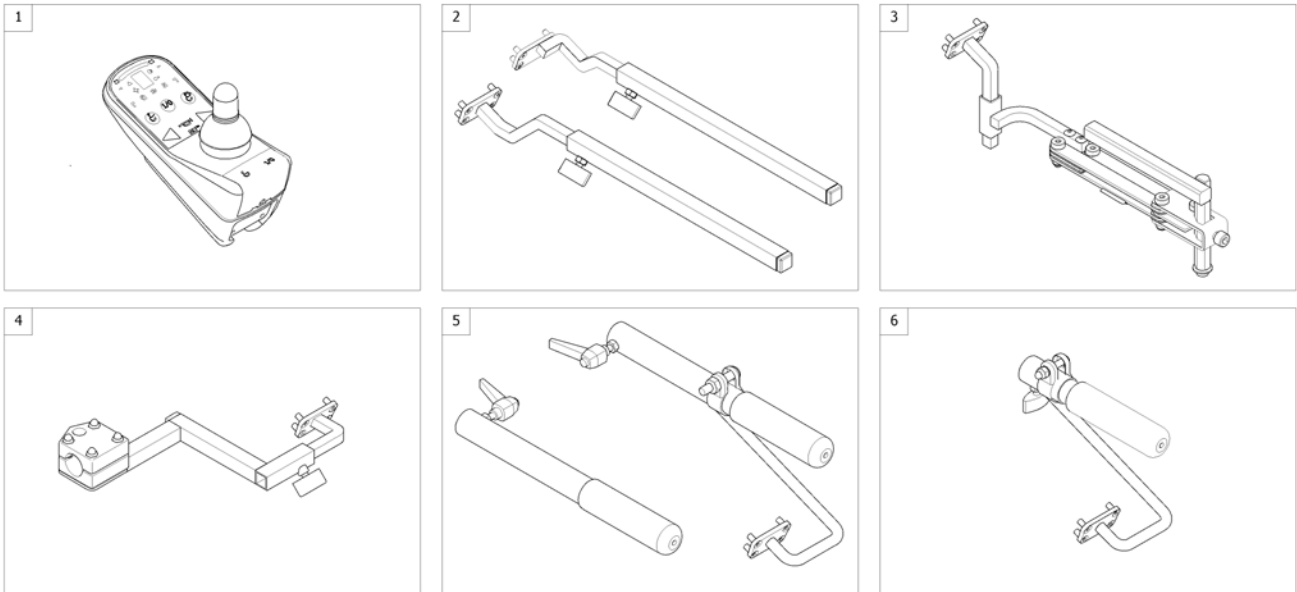

1	SP1504720	(Remote G90A) [1614/1616]	1
2	-	(Remote Bracket Standard) [1614+1207/1208]	1
3	-	(Remote Bracket Swing away & height adjust.) [1614+1707/1708]	1
4	-	(Remote Bracket Attendant for Standard-/Flex2-/Max-Seat) [1616+1207/1208]	1
5	-	(Remote Bracket Attendant long for Firm Seat) [1616+1207/1208]	1
6	-	(Remote Bracket Attendant short for Firm Seat) [1616+1207/1208]	1

# (Remote Bracket Standard)

1560842.iso

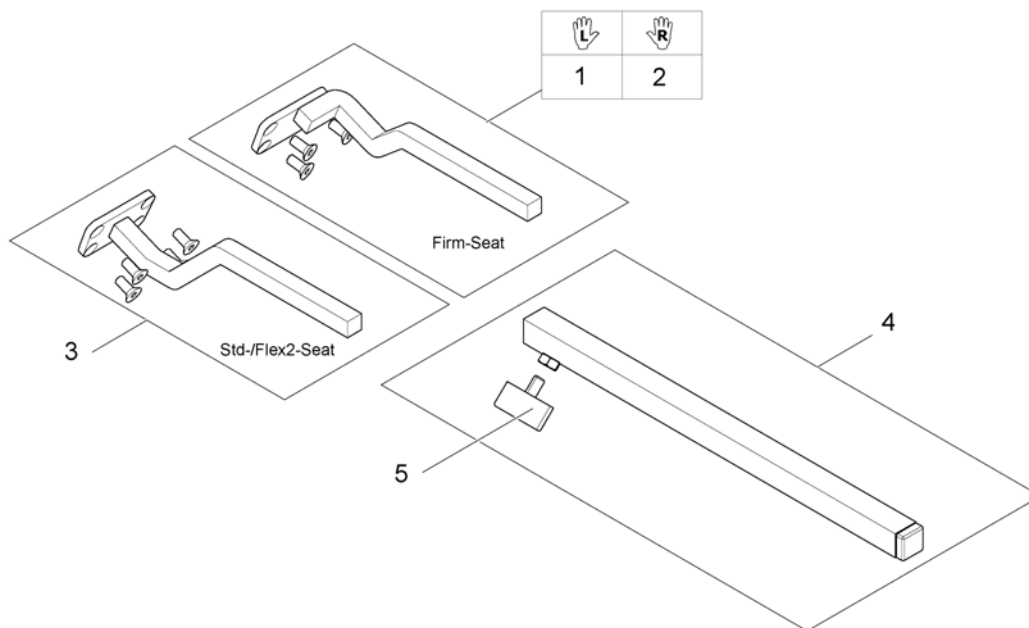


1	SP1518450	HOLDER FOR REMG90 STORM4	1
2	SP1518449	STYREBOKSHOLDER G90 BORA STORM4 TDX	1
3	SP1504822	(Remote Support Standard)	1
4	SPS183040	STYREBOKSHOLDER STD KPL STORM3 G40	1
5	L31001	VINGSKRUV M 6X10 GARANT G24 S	1



# (Remote Bracket Swing away & height adjust.)


1560310.iso

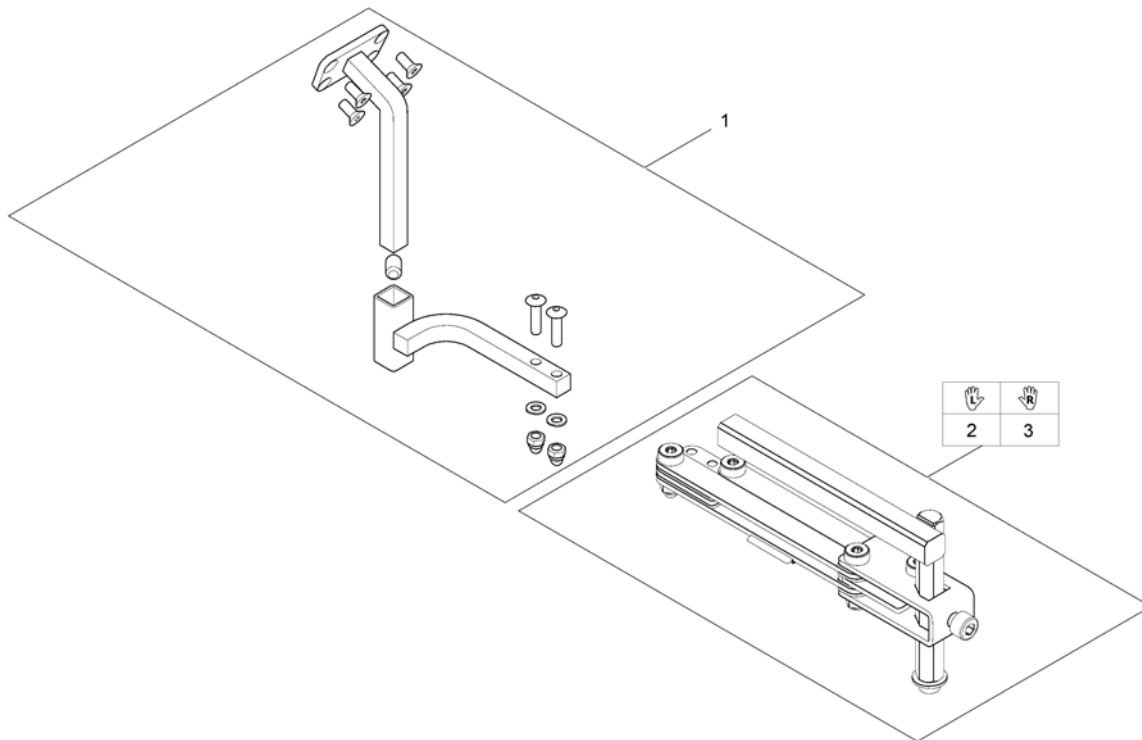

1	SP1517352	FESTE FOR G90 SWINGAWAY STORM4 TDX BORA	1
2	SP1496249	SWING AWAY VE NY MOD2008 KPL	1
3	SP1496248	SWING AWAY HØ NY MOD2008 KPL	1

# (Remote Bracket Attendant for Standard-/Flex2-/Max-Seat)


1560853.iso

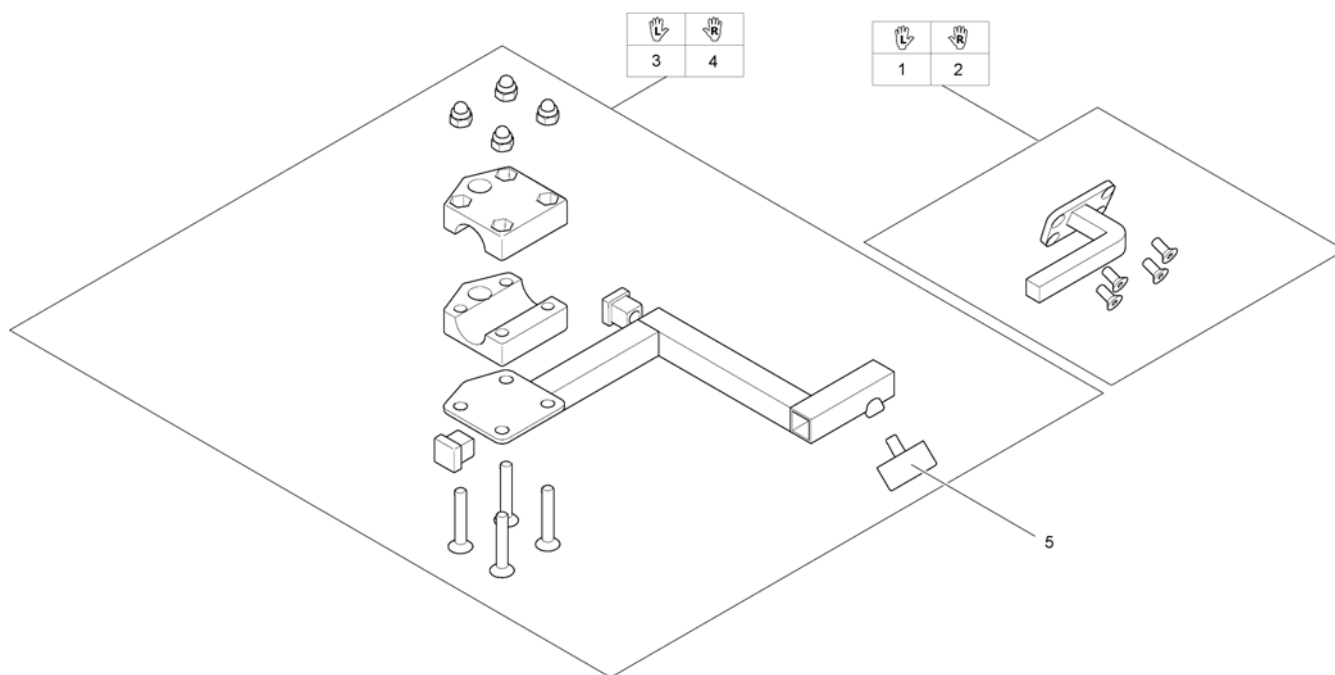

1	SP1509435	(Remote Support attendant control G90A, LH)	1
2	SP1509439	(Remote Support attendant control G90A, RH)	1
3	SP1422937	(Bracket for attendant control LH)	1
4	SP1422936	(Bracket for attendant control RH)	1
5	L31001	VINGSKRUV M 6X10 GARANT G24 S	1

**(Remote Bracket Attendant long for Firm Seat)**


1560312.iso

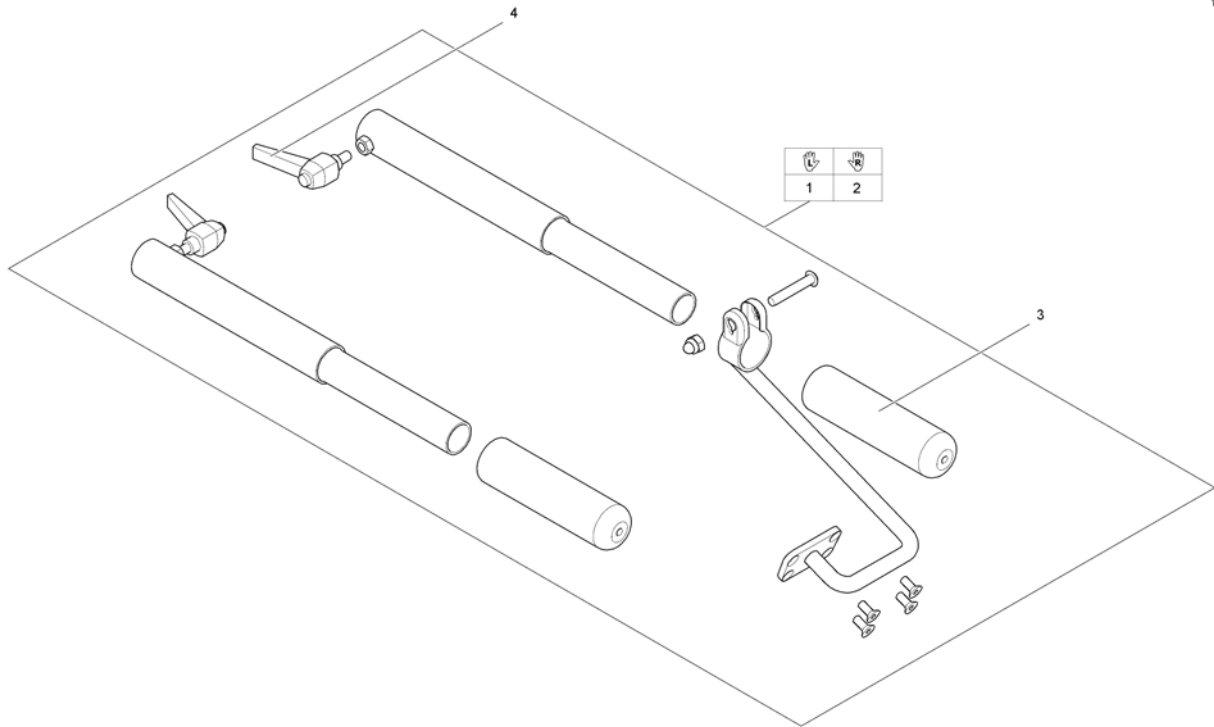



1	SP1509702	(Holder Attendant Control G90A LH)	1
2	SP1509696	(Holder Attendant Control G90A RH)	1
3	1435421	HÅNDTAK Ø22	1
4	L32902	LÅSESKRUE MED HENDEL ARMLEN M6X10 STORM/G40	1

## (Remote Bracket Attendant short for Firm Seat)

1560317.iso

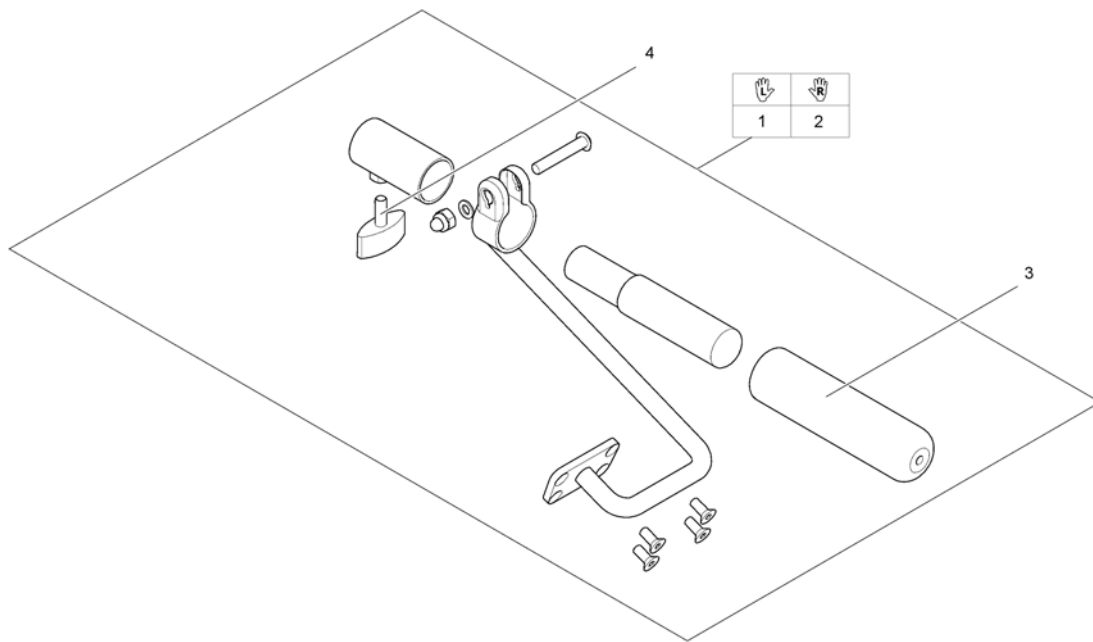

1	SP1541699	(Holder Attendant Control LH short)	1
2	SP1541570	(Holder Attendant Control RH short)	1
3	1435421	HÅNDTAK Ø22	1
4	L10024	SKRUE M6X16	1

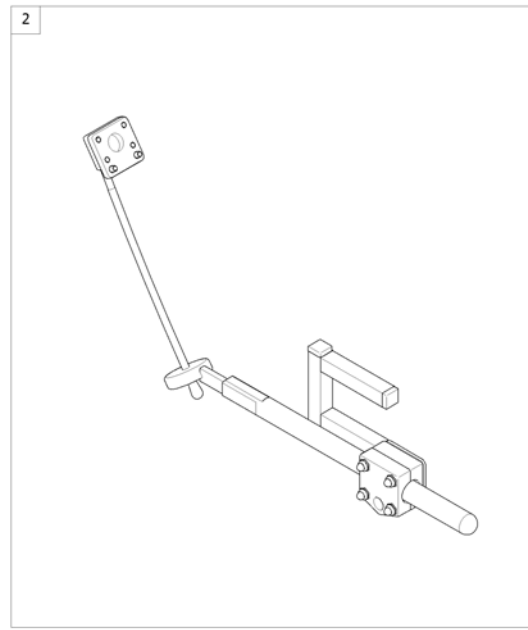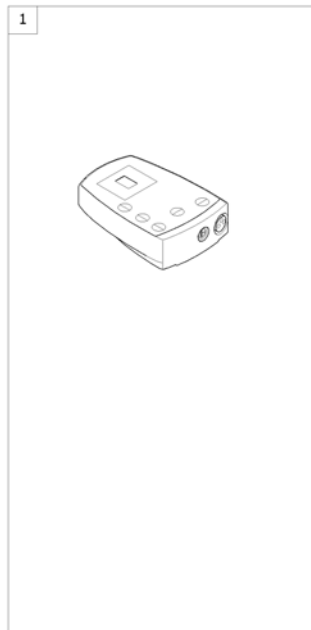


---

1	SP1505187	(Remote Control G91S) [1615]	1
2	-	(Remote Bracket Standard) [1615+1207/1208]	1


# (Remote Bracket Standard)

1560654.iso

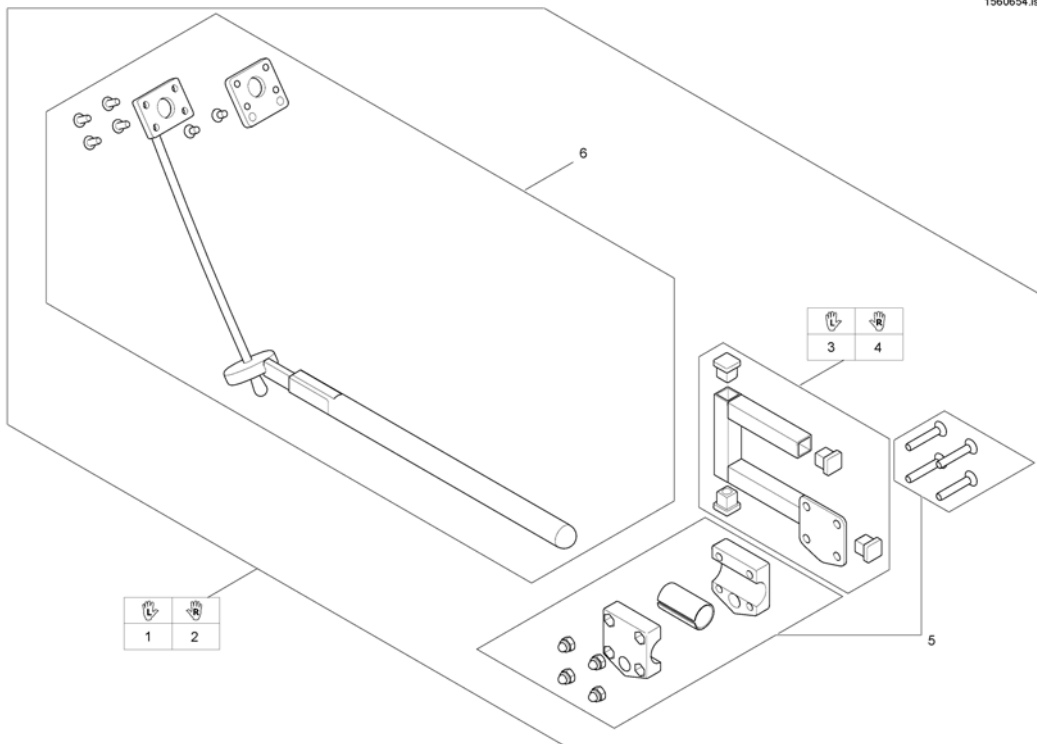

1	SP1509718	HOLDER FOR DX-REMG91 VE JUSTERBAR HØYDE/DYBDE	1
2	SP1509690	HOLDER FOR DX-REMG90 HØ JUSTERBAR HØYDE/DYBDE	1
3	SP1509716	(Adapter G91 LH)	1
4	SP1509711	(Adapter G91 RH)	1
5	SP1527503	(Gripper clamp cpl.)	1
6	SP1527504	(Swan neck remote bracket with adapter plate)	1

F383057.dwg

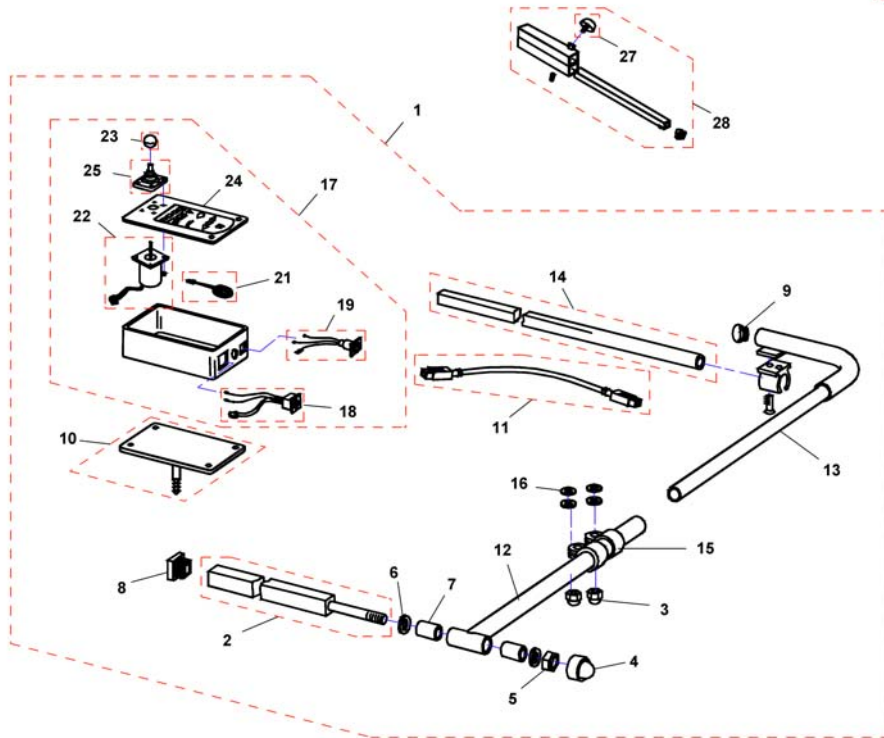


-	Q99081	(User Manual Table- & Middle Control REM41-S German)	1
1	SP1530588	DX-REM41 MED SENTERFESTE TDX	1
1	SP1530589	(CENTRAL STEERING CPL.; SWING-AWAY LH, REM41-D)	1
2	Z177024.00	FESTE F MITTSTYRNING,	1
3	F60208	(Cap nut DIN1587-M08 zc)	1
4	L01037	(Sks Nut Cap M12)	1
5	F20106	MUTTER SEKSKANT SELVLÅSENDE G40	1
6	G00104	SKIVE 13X24X2,5 G40	1
7	Z17032A	(Bush Rd. 19 X 3.5 X 25)	1
8	L01055	ENDESTYKKE FESTEBR.	1
9	L01008	(Tube End Plug SI 22)	1
10	Z183216.P40	FESTE F MANBOX MITTST,	1
11	1559862	(Bus cable ACS, 1700mm)	1
12	Z183089.00	BREDD JUST ARM MITTST,	1
13	Z183086.00	ÄNDRÖR HÖ, STORM	1
13	Z183092.00	ÄNDRÖR VÄ, STORM	1
14	H281005A.00	(MIRROR SUPPORT RAIL; CHROME PLATED)	1
15	5006720	KLEMME 22MM TDX	1
16	G00106	SKIVE M8 DRIVHJUL MIRAGE	1
17	SP1527930	BORDSTYRING DX-REM41-D DOLPHIN MASTER	1
18	P41108	KONTAKT BUS KABEL DX-REM41	1
19	P41106	(CHARGER SOCKET)	1
21	P30493	(HORN)	1
22	GSM55120	(ACS-remote WITH 180MM CABLE)	1
23	L36999	(KNOB FOR remote 32 MM, HOLE 5.8 MM)	1
24	P30907	MANØVERBOKS FOLIE DX-REM34	1
25	Z183001A	JOYSTICK GUMMI ACS	1
26	Z183289	SENTERSTYRT STØTTE RØR KPL	1
27	L31001	VINGSKRUV M 6X10 GARANT G24 S	1
28	Z183289	SENTERSTYRT STØTTE RØR KPL	1

## (Remote Small Chin Control with Brackets)

---


-	(Small Chincontrol)	1
-	(Framework for chincontrol)	1



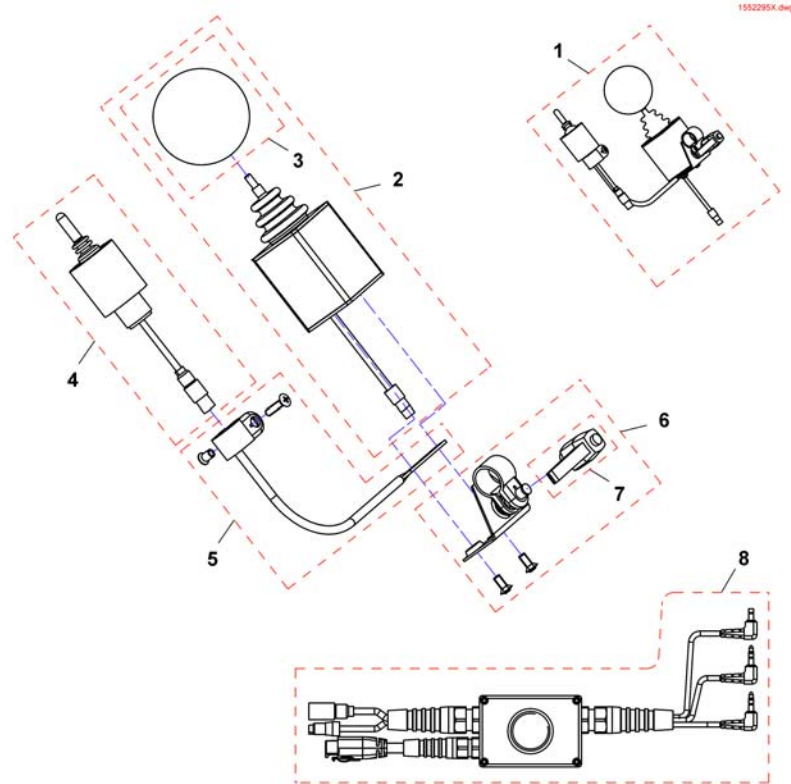

1	1448705	\$HAKESTYRING KPL MONT EKSTRA JOYSTICK	1
1	1421770	(Chin control G80/G90, cpl.)	1
1	SP1544255	(Chin control G91, cpl.)	1
2	Z183282	(Joystick, kmpl.)	1
3	Z183265	BRUK GJS26826 DYNAMICS	1
4	1442943	(Switchjoystick REM24, kmpl.)	1
4	Z183283	(Switchjoystick G80/G90, cpl.)	1
4	SP1544250	(Switchjoystick G91, kmpl.)	1
5	SP1552294	(Switchjoystick bracket)	1
6	SP1552295	(Joystick clamp)	1
7	L32901	SPAK M8 STORM4	1
8	SP1442944	(Harness REM24)	1
8	Z183275	(Harness G80/G90)	1
8	SP1541715	(Harness G91)	1


# (Framework for chincontrol)

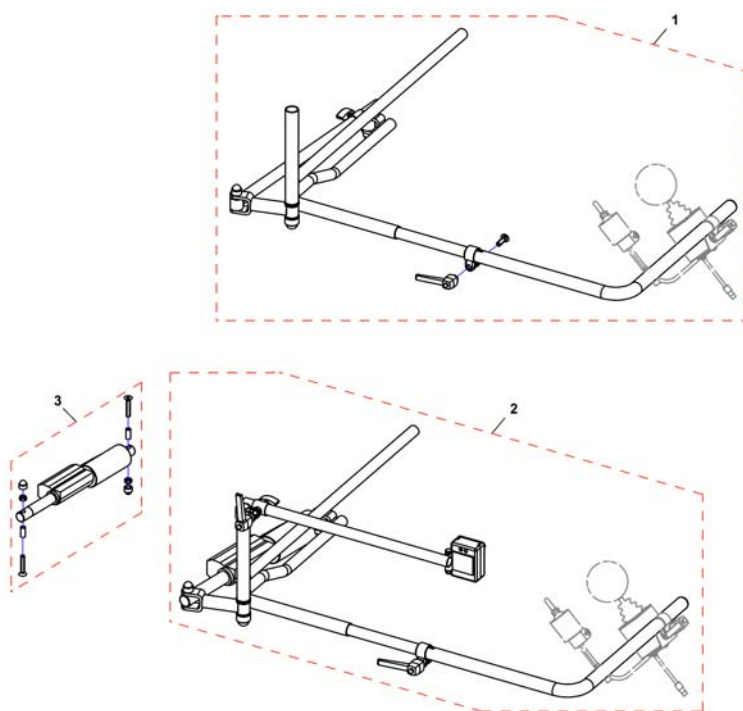

-	Z167994	(CHIN CONTROL SUPPORT . CPL.) <i>not shown</i>	1
1	SP1422943	(Framework for chin control, manual, horizontal swing away LH)	1
1	SP1422942	(Framework for chin control, manual, horizontal swing away RH)	1
2	SP1422945	HOLDER HAKEBETJ EL HORIS SVINGAWAY VE KPL	1
2	SP1552336	(Framework for chincontrol, powered, horizontal swing away RH)	1
5	SP1422946	(Framework for chincontrol, powered, vertikal swing away) <i>not shown</i>	4
3	SP1552337	(Actuator for horizontal swing away chin control, cpl.)	1
3	SP1552418	(Gestell für Kinnbedienung, elektrisch, vertikal abschwenkbar) <i>not shown</i>	1

# (Remote Dual Control with Brackets)


1560852 iso

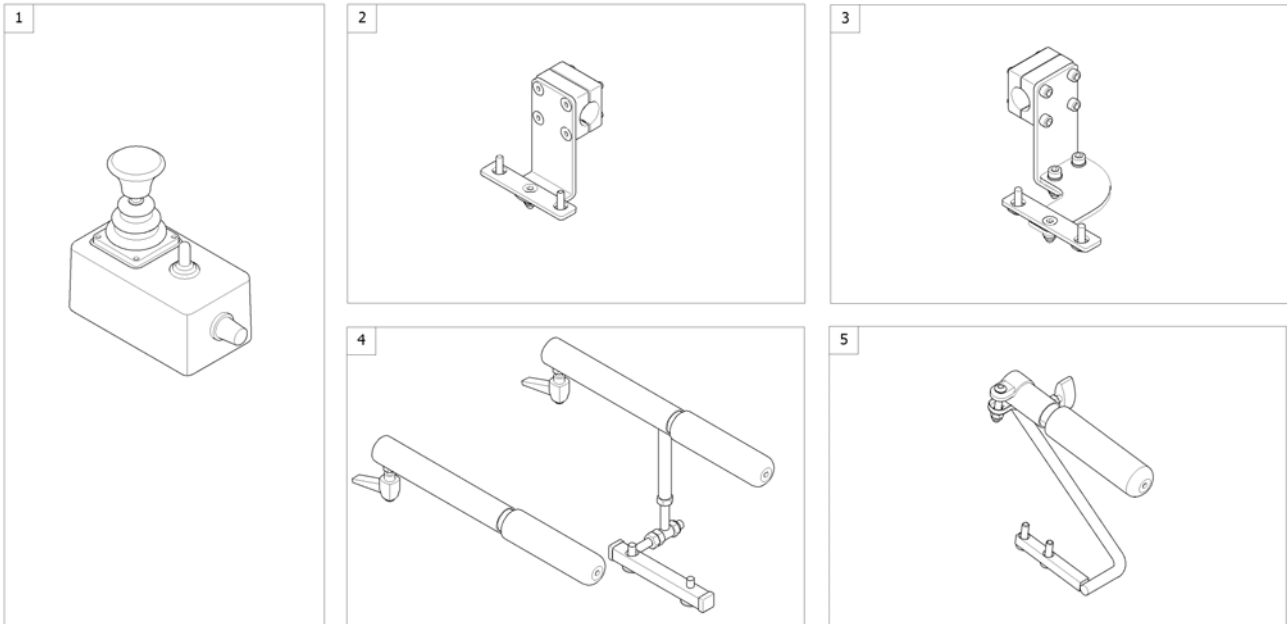

1	6001425	(ACS dual control) [1291/1292]	1
2	-	(Remote Bracket Dual Control for Standard-/Flex2-/Max-Seat)	1
3	-	(Remote Bracket Dual Control for Modulite Seat)	1
4	-	(Remote Bracket Dual Control long for Firm-Seat)	1
5	-	(Remote Bracket Attendant short for Firm Seat)	1

# (Remote Bracket Dual Control for Standard-/Flex2-/Max-Seat)

1560885.iso

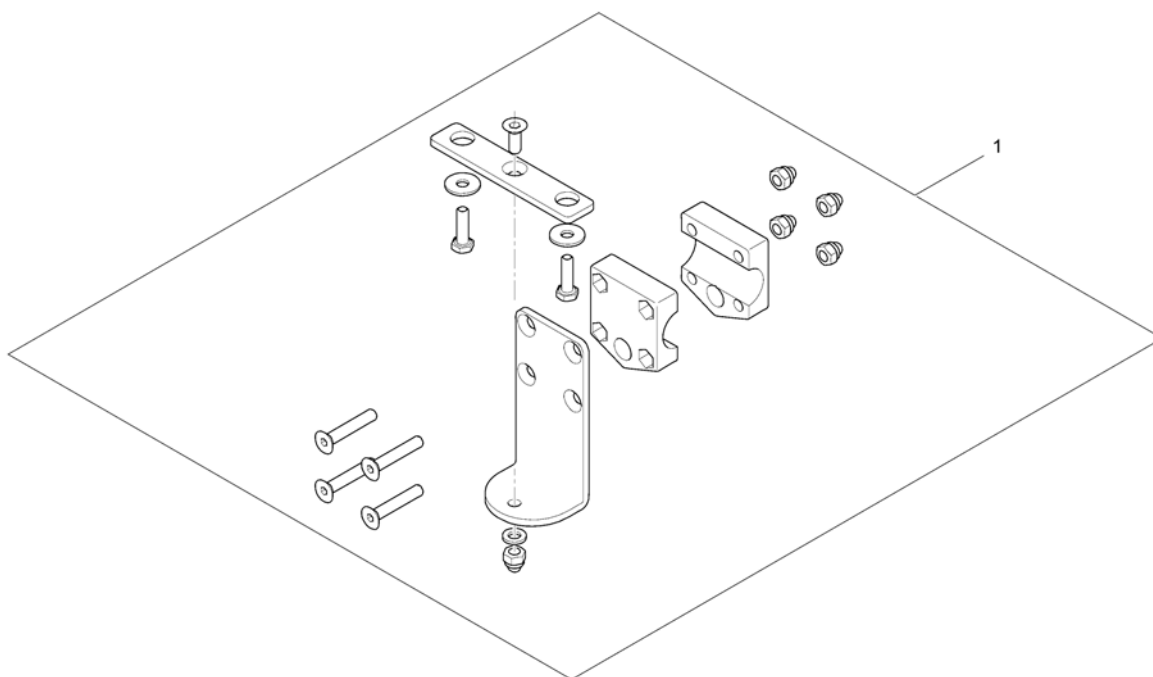


1 SP1422818

FESTEBRAKETT LEDSAGERSTY RYGGMONT ENEKL

1

# (Remote Bracket Dual Control for Modulite Seat)


1574291 iso

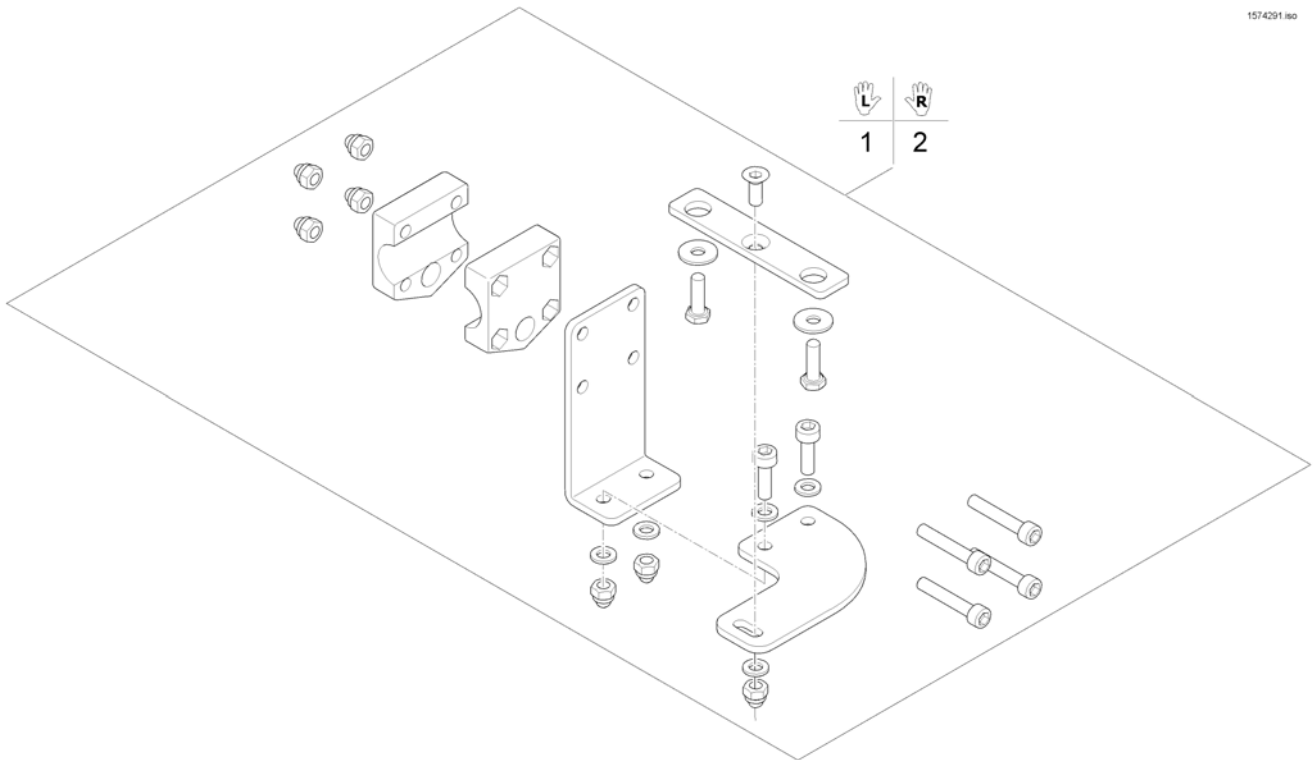

1	SP1571799	(Remote Bracket Dual Control LH)	1
2	SP1571800	(Remote Bracket Dual Control RH)	1

# (Remote Bracket Dual Control long for Firm-Seat)


1560884.iso

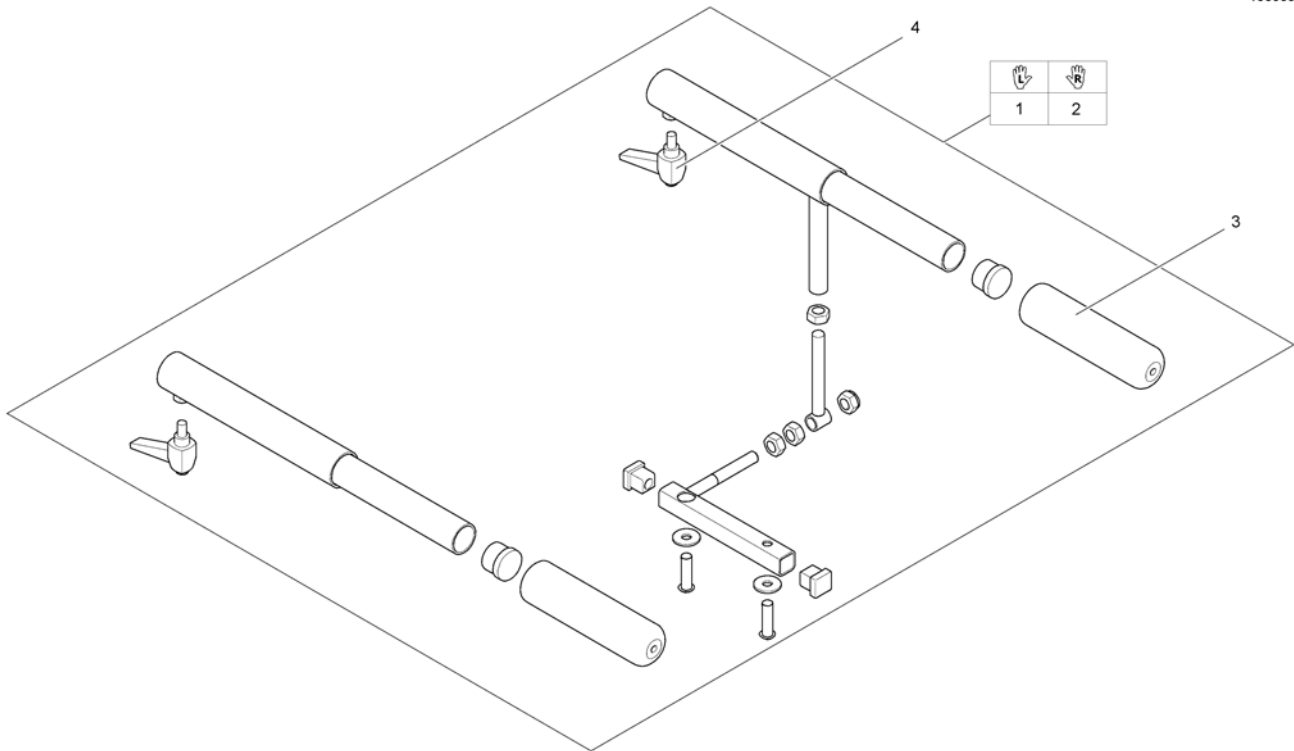

1	SP1511703	(Assembly Kit Dual Control, ACS & Shark, LH (Firm-Seat))	1
2	SP1511704	(Assembly Kit Dual Control, ACS & Shark, RH (Firm-Seat))	1
3	1435421	HÅNDTAK Ø22	1
4	SPL32903	(LOCKING SCREW M6X2 (X2))	1

# (Remote IDC Dual Control with Brackets)

1509875X.iso

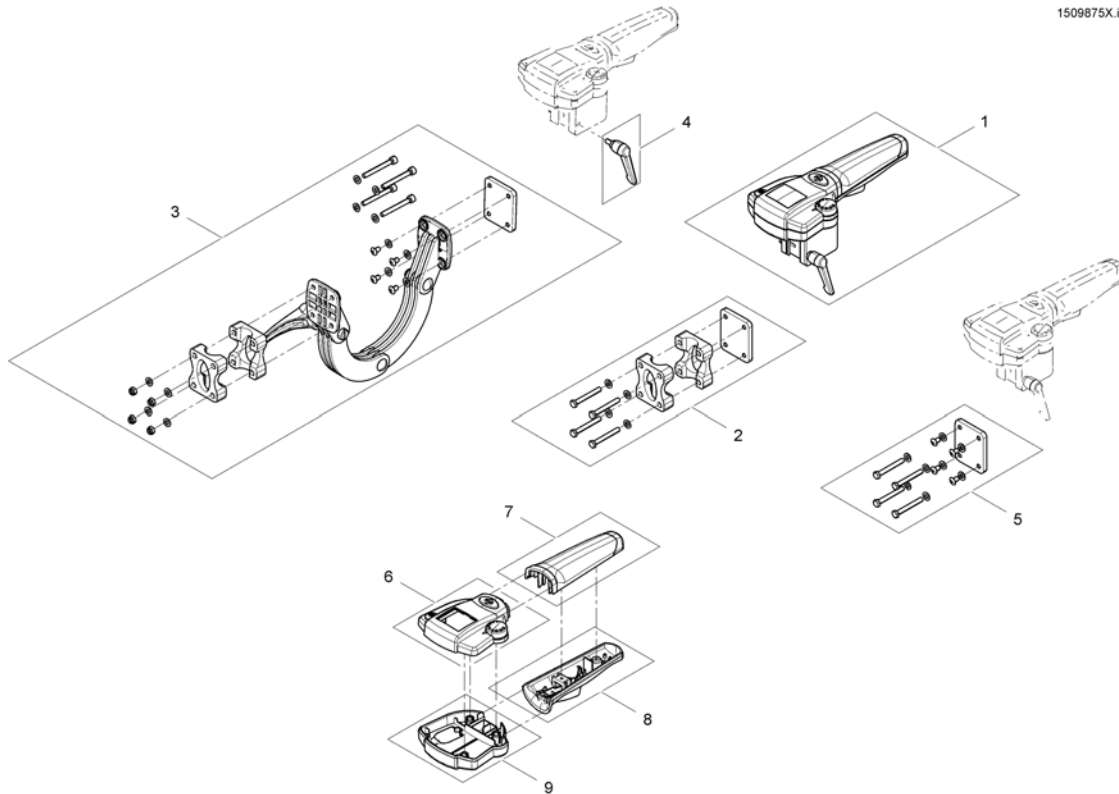

1	SP1509875	LEDSAGERSTYRING INTUITIV ALLE STOLER MED ACS <i>For chairs built before 01/02/10 please also order SP1537148</i>	1
2	SP1522310	KLEMME RYGGRØR IDC FESTE INTUITIV LEDESAGERSTYRING	1
3	SP1521488	HOLDER INTUITIV LEDESAGER RYGGER MED RUNDT RØR	1
4	SP1510135	(Clamp lever M5x10)	1
5	SP1537148	(Supportplate IDC for chairs before 01.02.10)	1
6	SP1510990	DEKSEL TIL IDC INT LEDESAGERST ACS	1
7	SP1511060	(Handhold upper part)	1
8	SP1511099	(Handhold lower part)	1
9	SP1510470	DEKSEL INT LEDESAGERST ACS NEDRE DEKSEL	1


## (ACS Cables & Accessories)

---

-	(ACS Buscable)	1
-	(T-Adapter ACS 2)	1
-	(Slow-Stop-Sensor & Holder)	1
-	(Programming Kits)	1
-	(Magnetkey for Remotes)	1




ACSBUSKABEL.iso

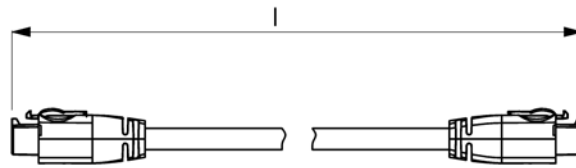

Position	length l [mm]
1	300
2	500
3	700
4	1100
5	1700
6	2000
7	2500
8	2700
9	1060

1	GSM63003	KABEL BUS 0,3M ACS,	1
2	SPGSM63005	(Bus cable ACS, 500mm) <i>Replace GSM63004</i>	1
3	GSM63007	(Bus cable ACS 700mm)	1
4	GSM63011	KABEL-BUS 1,1M ACS, ST utgått bruk 63010	1
5	1559862	(Bus cable ACS, 1700mm) <i>Replace GSM63016</i>	1
6	GSM63020	DX BUS CABLE 2.0M STORM <i>Replace 1419386</i>	1
7	1440424	(Bus cable ACS, 2500mm)	1
8	SP1559873	(Bus cable ACS, 2700mm) <i>Replace GSM63026</i>	1
9	SP1550402	(Extension cable ACS, 1060mm)	1

Invacare® Bora | Elektronikk  
**(T-Adapter ACS 2)**

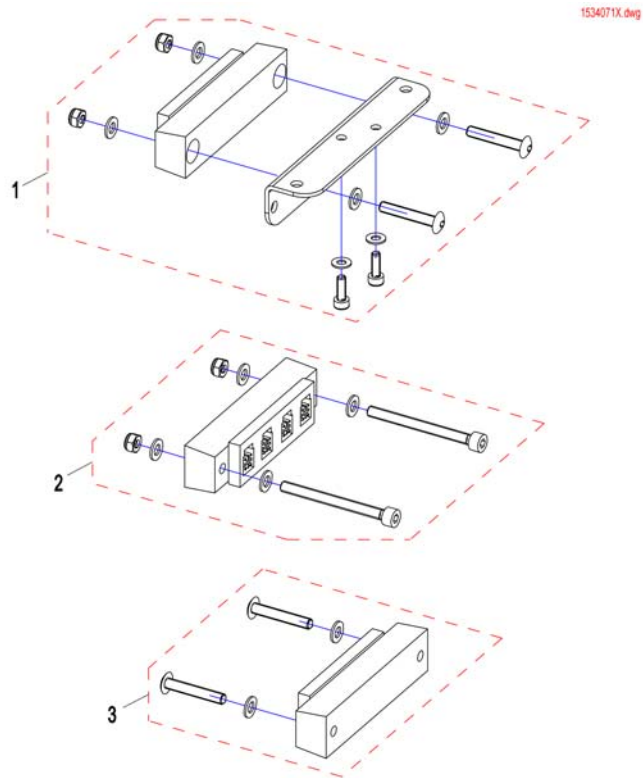



---


1	SP1536863	(T-Adapter ACS 2 for Flex2 with lifter)	1
2	SP1534948	(T-Adapter ACS 2 for Firmseat with / without lifter and Flex2 without lifter)	1
3	SP1534072	(T-Adapter ACS 2 for Standardseat with / without lifter)	1

# (Slow-Stop-Sensor & Holder)

1533662X.dwg

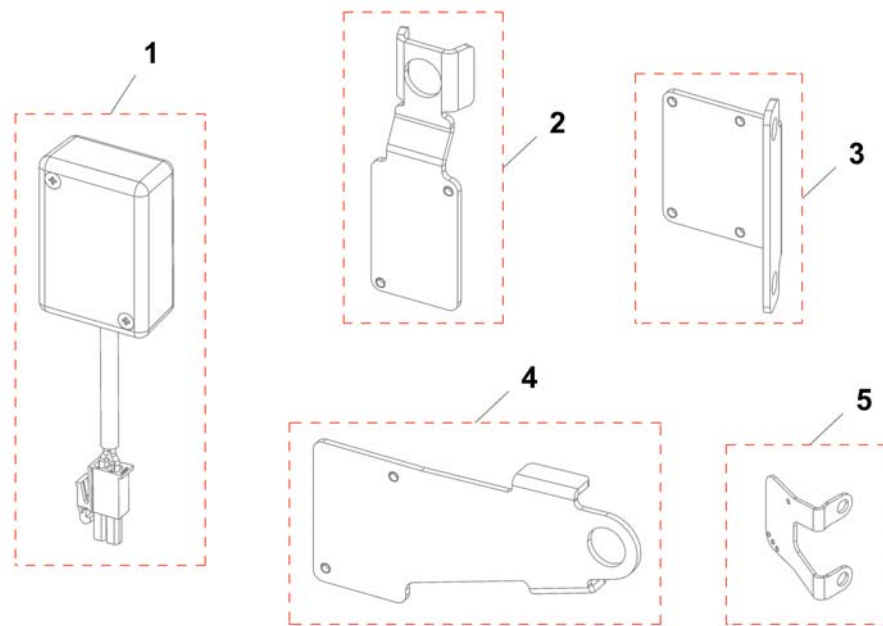


1	SP1533662	(Slow-stop-sensor, ACS2, cpl.)	1
1	SP1572175	(Slow-stop-sensor, ACS2, Modulite, cpl.)	1
1	SP1537477	(Slow-stop-sensor, R-Net, cpl.)	1
1	SP1571202	(Slow-stop-sensor, R-Net, Modulite, cpl.)	1
2	SP1533665	FESTE STD SETE KPL BORA	1
3	SP1533661	FESTE FAST SETE BORA	1
4	SP1534049	FESTE FLEXII SETE BORA	1
5	SP1563880	(Holder slow-stop-sensor Modulite)	1

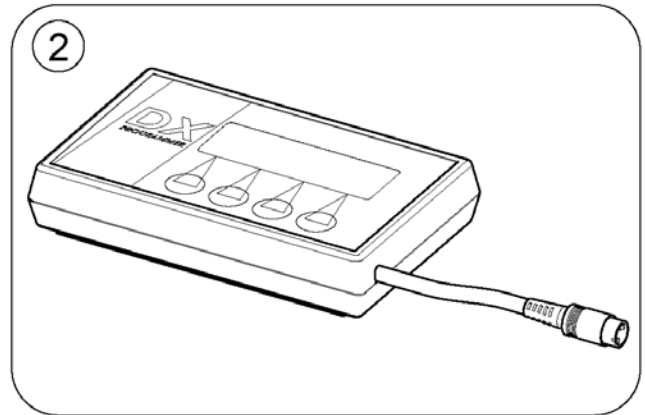

---




1	1449280	(USW wizard dongle kit, EDL-USW, cpl.)	1
2	6001450	(Hand Programmer, Instrucions In French, German, Spanish)	1

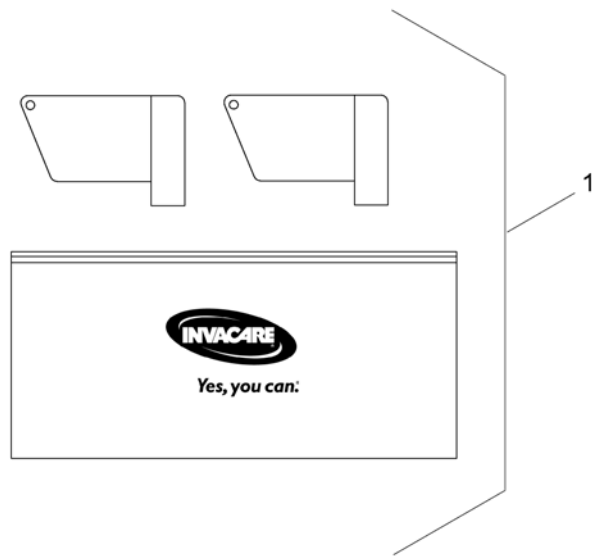

1 SP1419695

(Magnet key with bag)

1

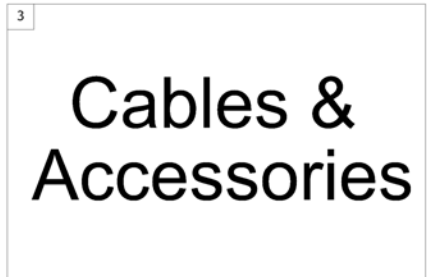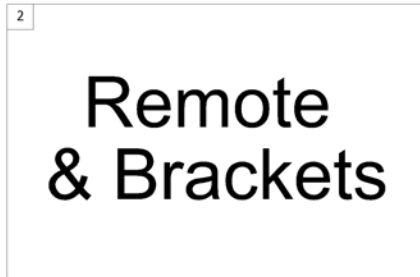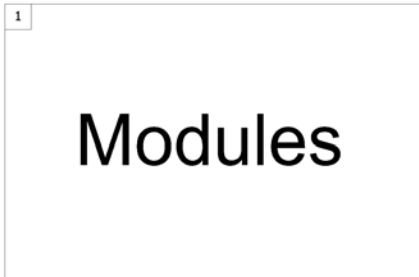

---


1	-	(Shark Electronic Modules)	1
2	-	(Shark Remote & Brackets)	1
3	-	(Shark Cables & Accessories)	1

---

-	(Powermodules)	1
-	(Lightboard)	1
-	(Lightboardholder Shark/ACS2)	1

# (Shark Remote & Brackets)

156025 no

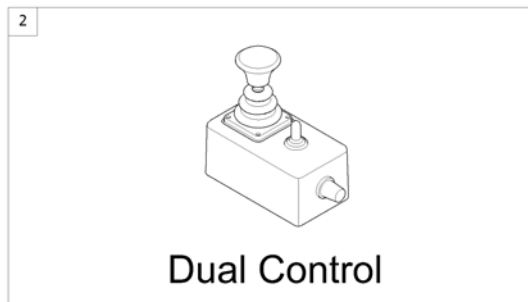


---

1	-	(Remote Shark with Brackets) <i>[1801/1802]</i>	1
2	-	(Remote Dual Control with Brackets) <i>[1891/1892]</i>	1




150021-00

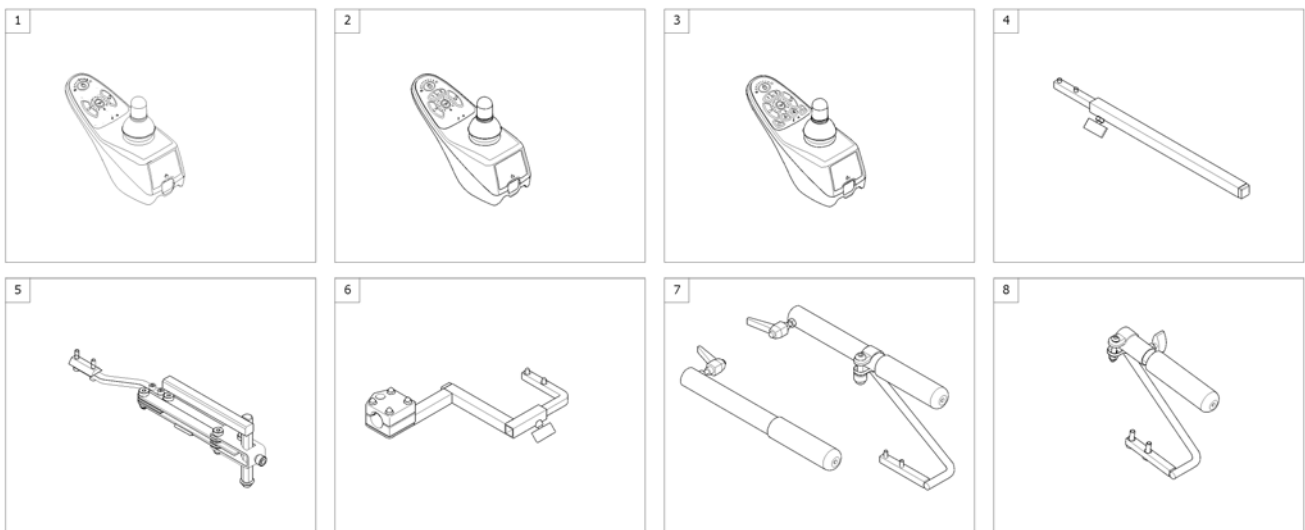

1	1492567	STYREBOKS SHARK DKREMD01 DRAGON <i>[1801/1802]</i>	1
2	1449177	STYREBOKS SHARK II DX-REMD11 DRAGON <i>[1801/1802]</i>	1
3	1484904	STYREBOKS SHARK II DX-REMD11 LYS OG EL DRAG <i>[1801/1802]</i>	1
4	-	(Remote Bracket Standard) <i>[1801+1207/1208]</i>	1
5	-	(Remote Bracket Swing away & height adjust.) <i>[1801+1707/1708]</i>	1
6	-	(Remote Bracket Attendant for Standard-/Flex2-/Max-Seat) <i>[1802+1207/1208]</i>	1
7	-	(Remote Bracket Attendant long for Firm Seat) <i>[1802+1207/1208]</i>	1
8	-	(Remote Bracket Attendant short for Firm Seat) <i>[1802+1207/1208]</i>	1

## (Remote Dual Control with Brackets)


1561035.no

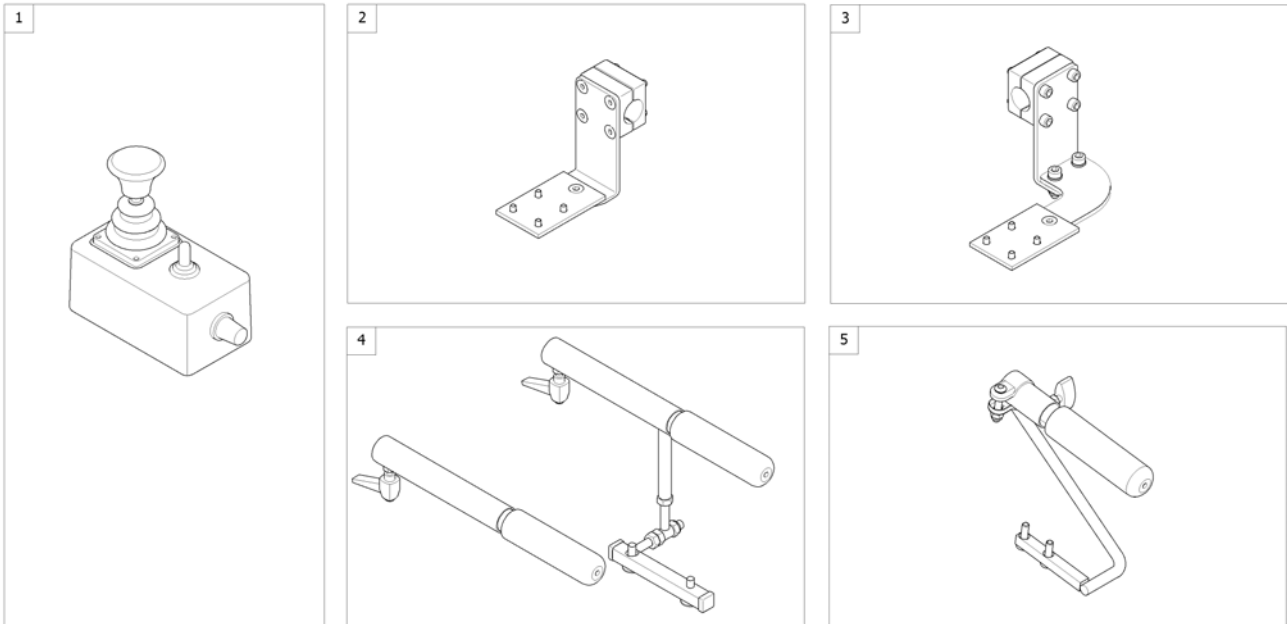

1	1444717	(Shark Dual Control, Switchable) [1891/1892]	1
2	-	(Remote Bracket Dual Control for Standard-/Flex2-/Max-Seat)	1
3	-	(Remote Bracket Dual Control for Modulite Seat)	1
4	-	(Remote Bracket Dual Control long for Firm-Seat)	1
5	-	(Remote Bracket Attendant short for Firm Seat)	1

# (Remote Bracket Dual Control for Standard-/Flex2-/Max-Seat)

1560883.iso

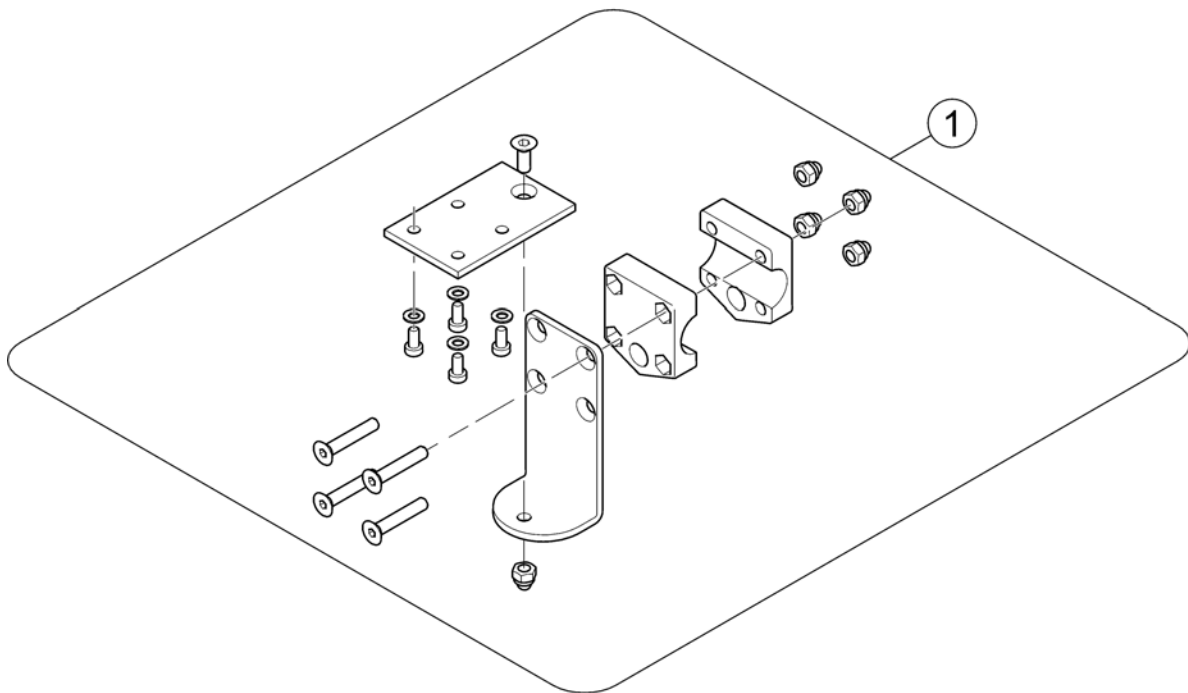


1 1450307

(Remote Bracket Attendent Control Shark)

1

# (Remote Bracket Dual Control for Modulite Seat)


1574296 iso

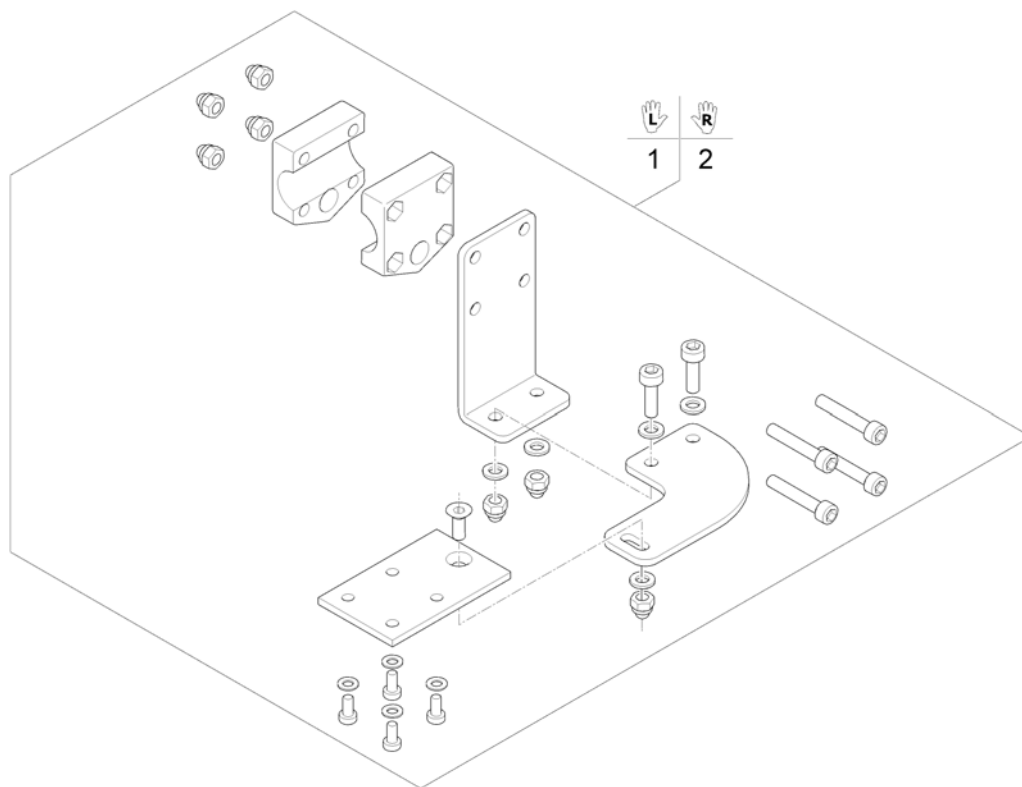

1	SP1571824	(Remote Bracket Dual Control LH)	1
2	SP1571825	(Remote Bracket Dual Control LH)	1

---

-	(Buscable Shark)	1
-	(Programming Kits)	1

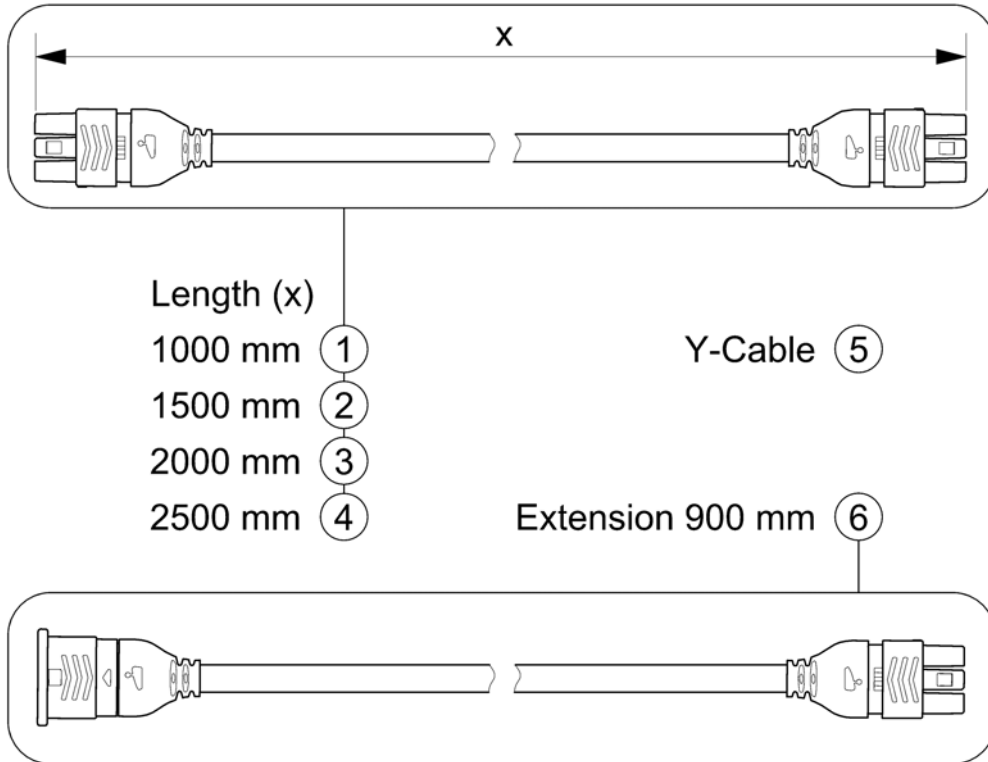

1	1448922	(Buscable Shark 1000mm)	1
2	1450313	BUSKABEL 1500 MM SHARK	1
3	1484911	(Buscable Shark 2000mm)	1
4	SP1510073	(Buscable Shark 2500mm)	1
5	1450314	BUSKABEL Y SHARK	1
6	1116404	(Extension Cable Shark 900mm)	1

1560022.no

1  
**Modules**

2  
**Remote  
& Brackets**

3  
**Cables &  
Accessories**

---


1	-	(R-NET Electronic Modules)	1
2	-	(R-NET Remote & Brackets)	1
3	-	(R-NET Cables & Accessories)	1

## (R-NET Electronic Modules)

---


-	(Powermodule R-Net)	1
-	(Powermodules)	1
-	(Halter R-Net)	1
-	(Holder ISM)	1
-	(Lightboard)	1
-	(Lightboardholder R-Net)	1



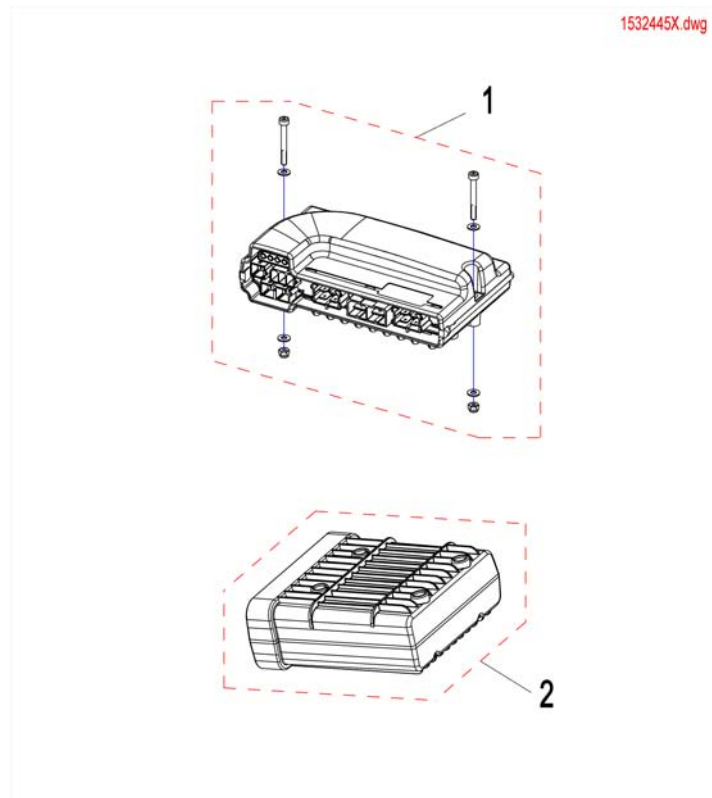


---

1	SP1532445	POWERMODUL BORA STORM4	1
2	SP1532446	(Seating/Lightning Module R-NET ISM-6L PGDT)	1

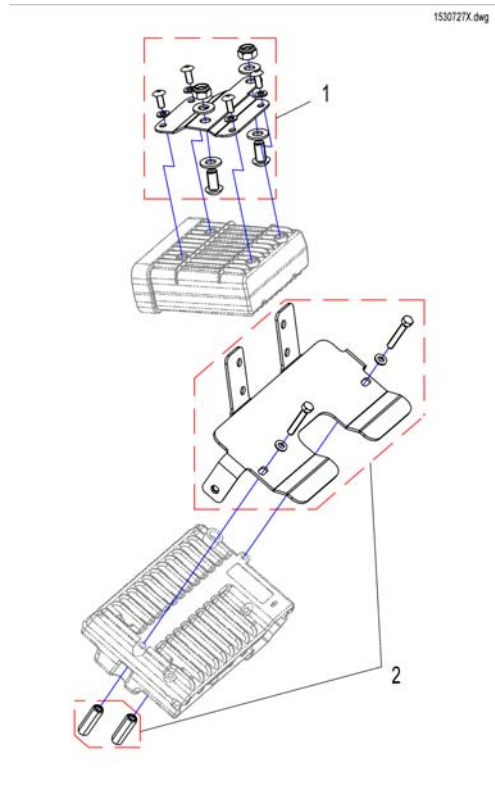

---


1	SP1536677	(Holder Seat-/Lightmodule (ISM))	1
2	SP1536740	(Holder Powermodule)	1

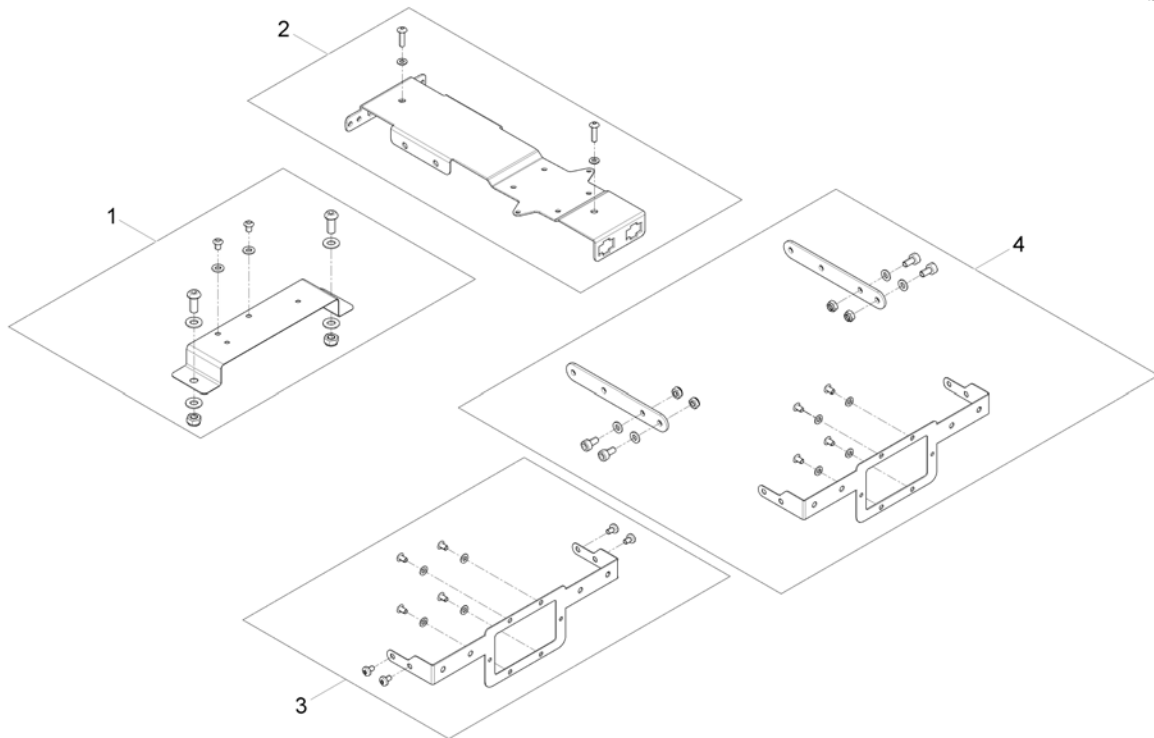

1	SP1528232	FESTE FAST OG STD SETE BORA	1
2	SP1569979	(Multi support bracket)	1
3	SP1569959	(ISM bracket)	1
4	SP1571925	(Support bracket ISM-Module lifter)	1

# (Lightboardholder R-Net)



1533410X.dwg

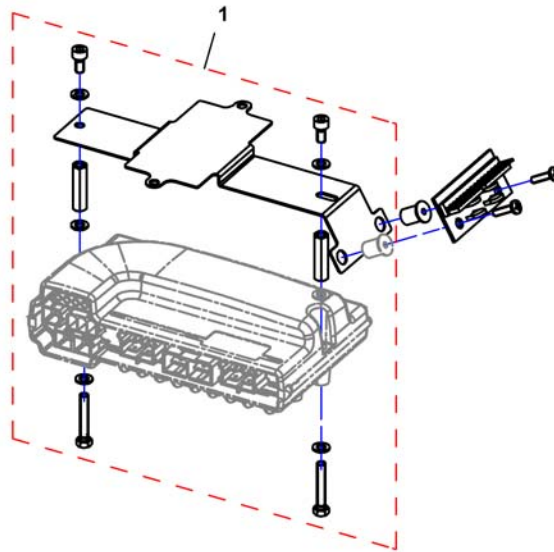

1 SP1537916

(Holder for Lightboard)

1

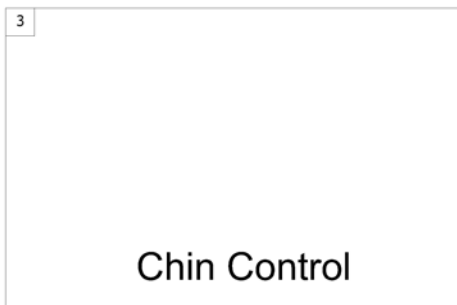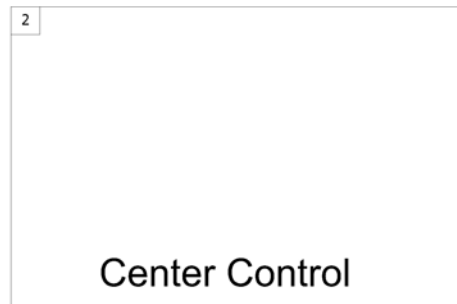



---


1	-	(Remote R-Net LED JSMs with Brackets) [1503/1504]	1
2	-	(Centre control) [1509]	1
3	-	(Chincontrol R-Net) [1511]	1
4	-	(Remote Dual Control with Brackets) [1595]	1

## (Remote R-Net LED JSMs with Brackets)

1561251-00

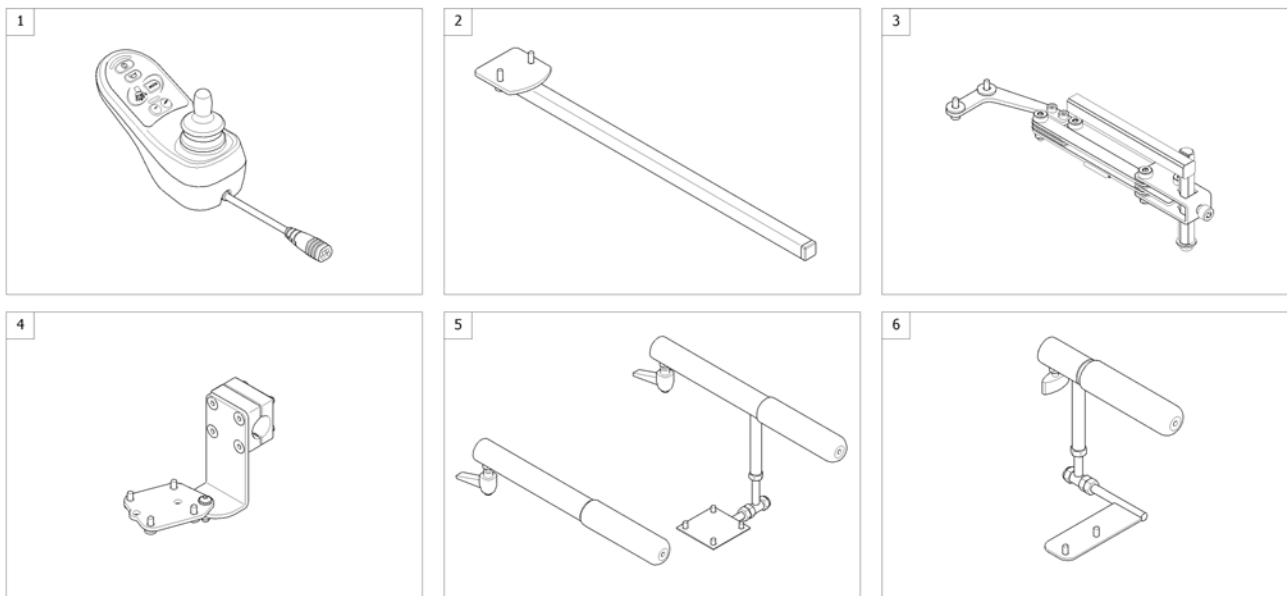

1	SP1531273	(Remote R-Net JSM LED-L") [1503/1504]	1
2	-	(Remote Bracket Standard) [1503+1207/1208]	1
3	-	(Remote Bracket Swing away & height adjust.) [1503+1707/1708]	1
4	-	(Remote Bracket Attendant for Standard-/Flex2-/Max-Seat) [1504+1207/1208]	1
5	-	(Remote Bracket Attendant long for Firm Seat) [1504+1207/1208]	1
6	-	(Remote Bracket Attendant short for Firm Seat) [1504+1207/1208]	1

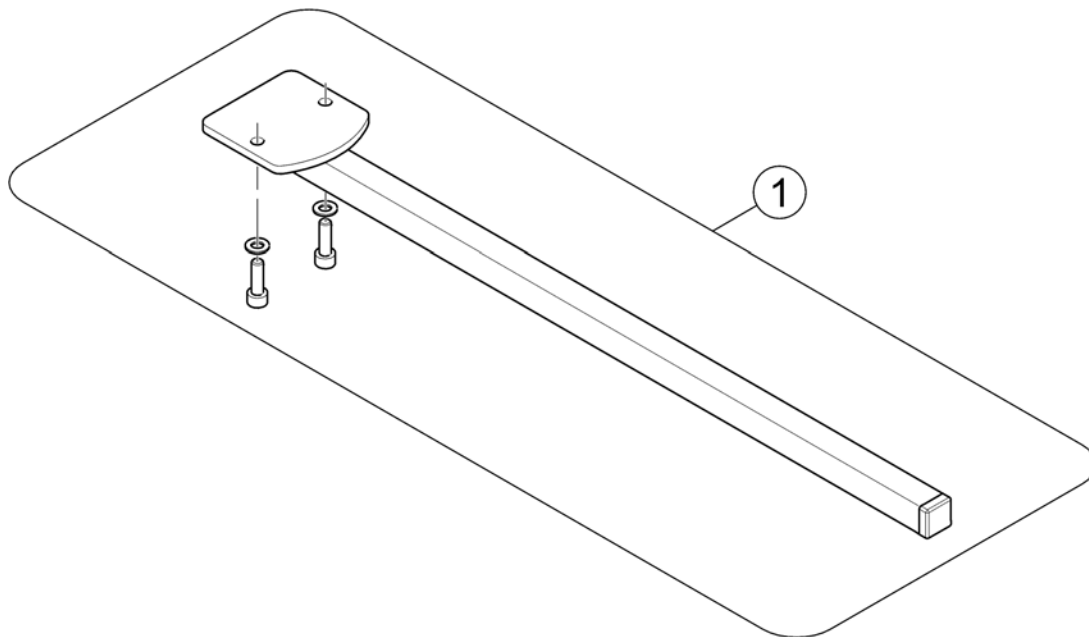


1 SP1527190 (Remote Bracket)

1

**(Remote Bracket Swing away & height adjust.)**

1560572.iso

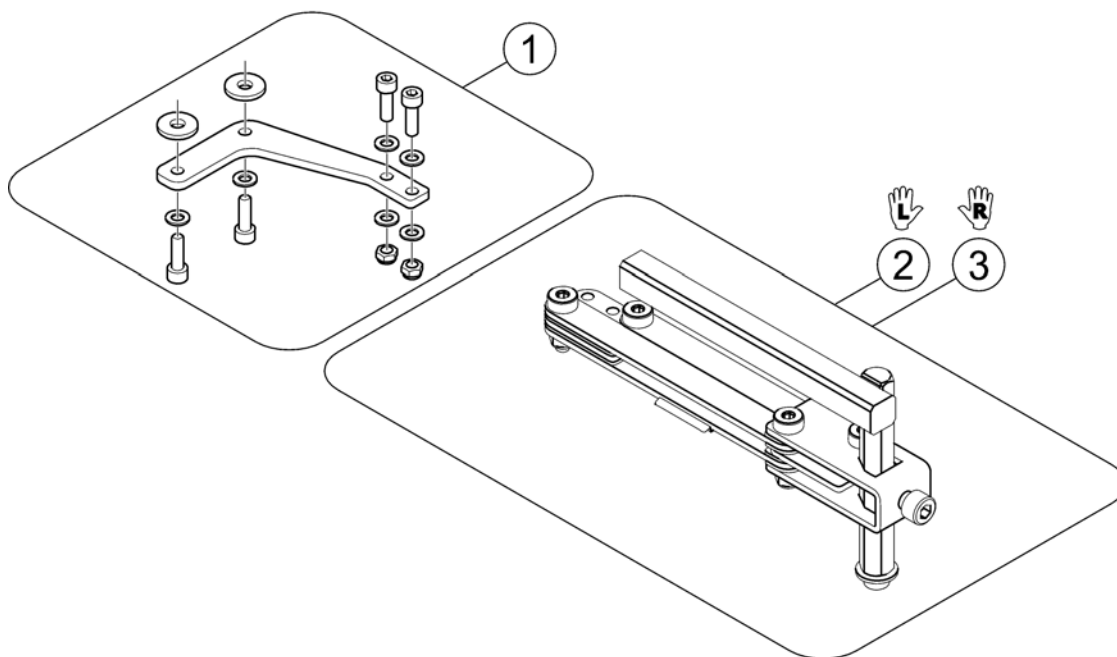


1	SP1528313	(Remote Support for Bracket height adjustable and swing away)	1
2	SP1496249	SWING AWAY VE NY MOD2008 KPL	1
3	SP1496248	SWING AWAY HØ NY MOD2008 KPL	1



# (Remote Bracket Attendant for Standard-/Flex2-/Max-Seat)

1561250.iso

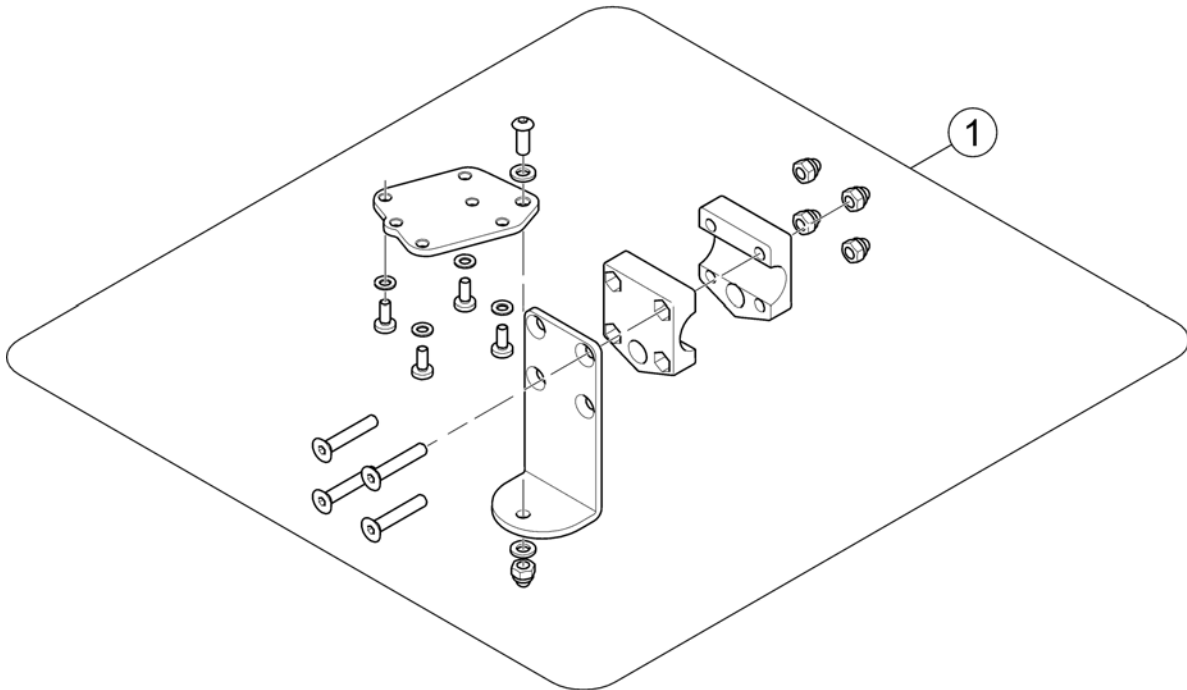


1 SP1531713

(Remote Bracket)

1

## (Remote Bracket Attendant long for Firm Seat)


1560574.iso

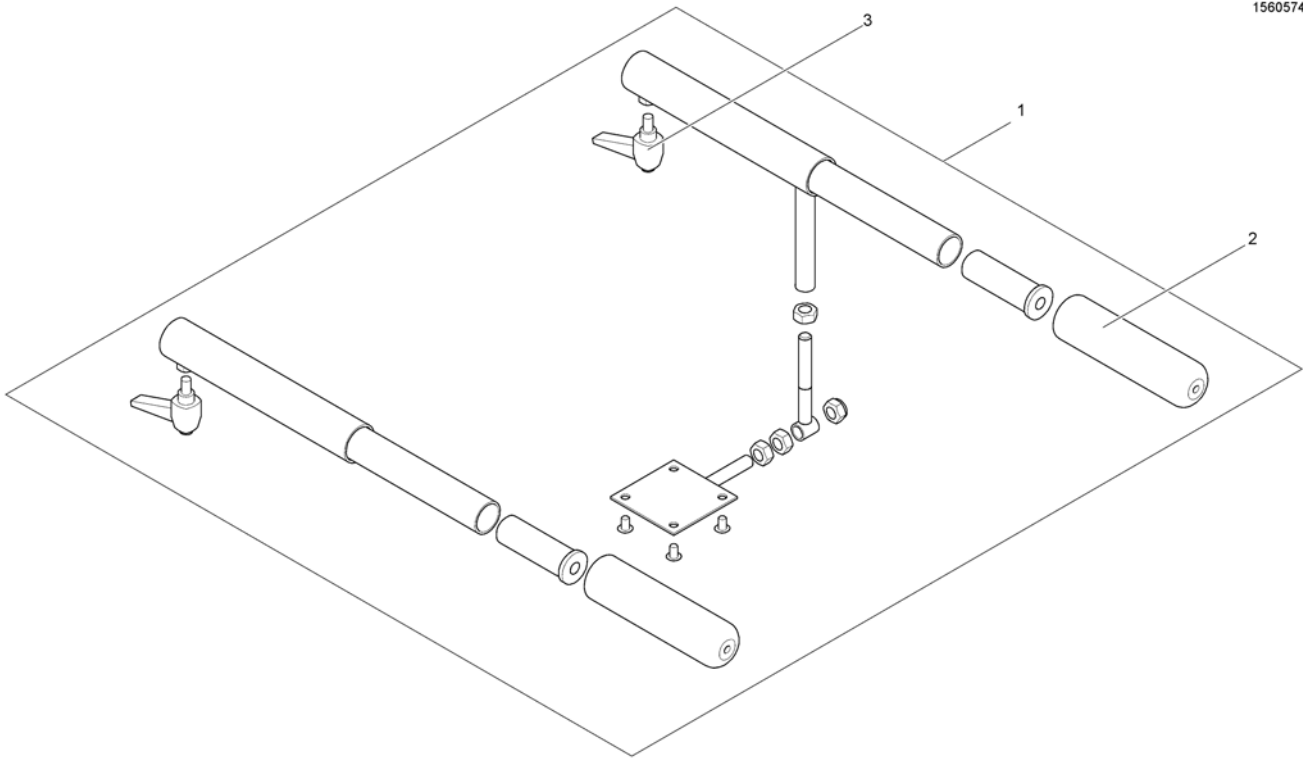


1	SP1531714	(Holder attendantcontrol R-Net)	1
2	1435421	HÅNDTAK Ø22	1
3	L32902	LÅSEKRUE MED HENDEL ARMLEN M6X10 STORM/G40	1

# (Remote Bracket Attendant short for Firm Seat)

1560573.iso

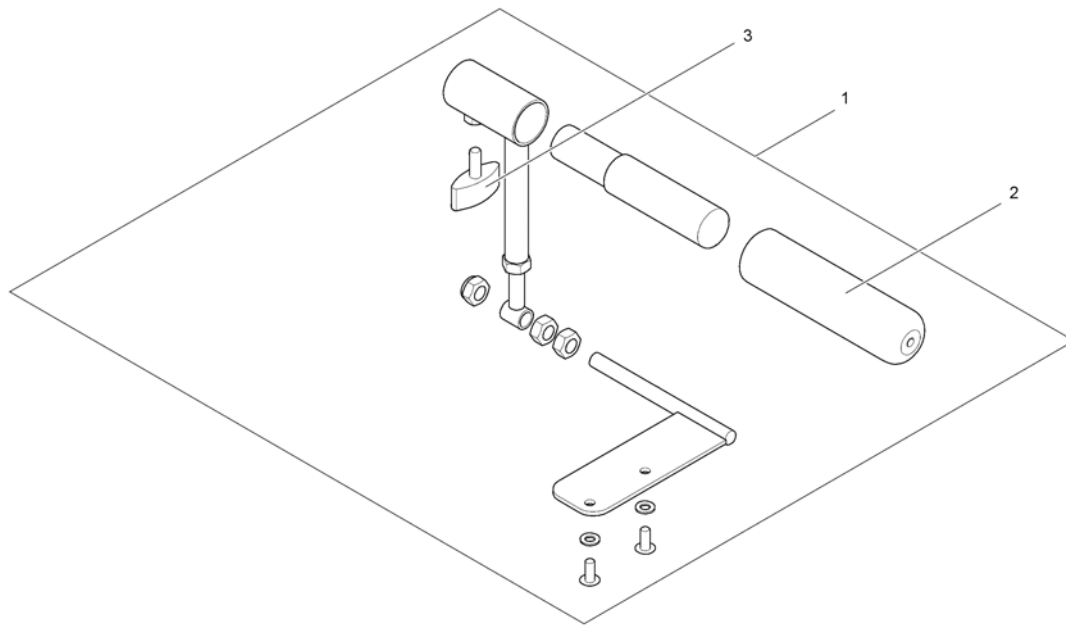


1	SP1543164	(Remote Bracket Dual Control short for Firm seat)	1
2	1435421	HÅNDTAK Ø22	1
3	L10024	SKRUE M6X16	1

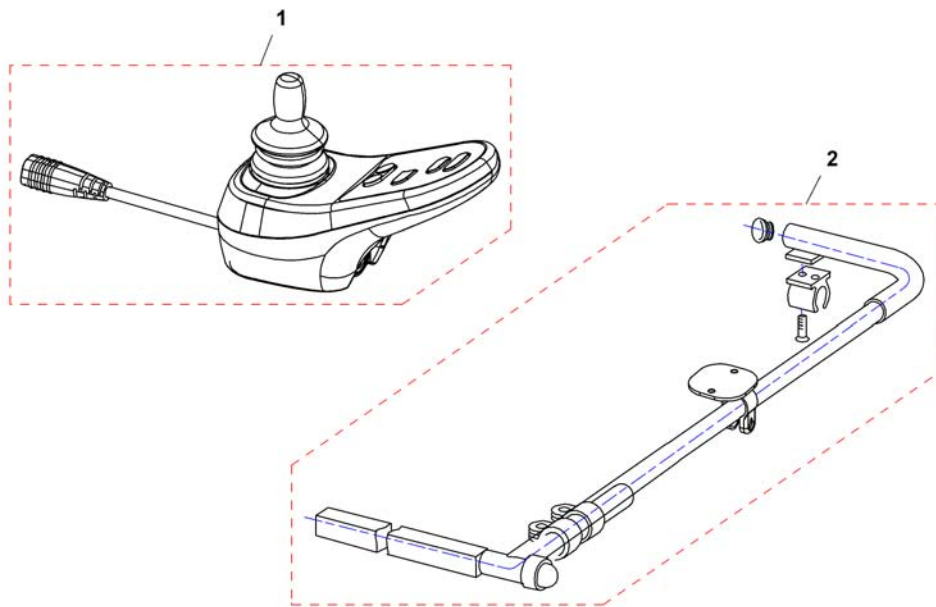

---

1	SP1531273	(Remote R-Net JSM LED-L")	1
2	SP1531278	(Holder middlecontrol)	1

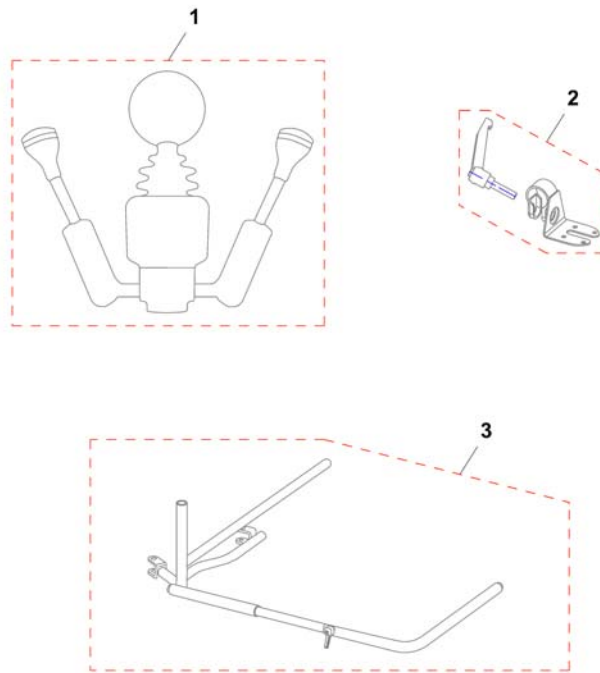



---

-	1422947	(ASSEMBLY KIT F. CHIN CONTROL HOLDER ,STANDARDSEAT ,CPL.)	1
-	SP1533386	(ASSEMBLY KIT F. CHIN CONTROL HOLDER ,FIRMSEAT ,CPL.)	1
1	SP1534499	(Chincontrol R-Net Chin")	1
2	SP1537939	(Holder chincontrol)	1
3	SP1422942	(Framework for chin control, manual, horizontal swing away RH)	1
3	SP1422943	(Framework for chin control, manual, horizontal swing away LH)	1

## (Remote Dual Control with Brackets)


1561036 iso

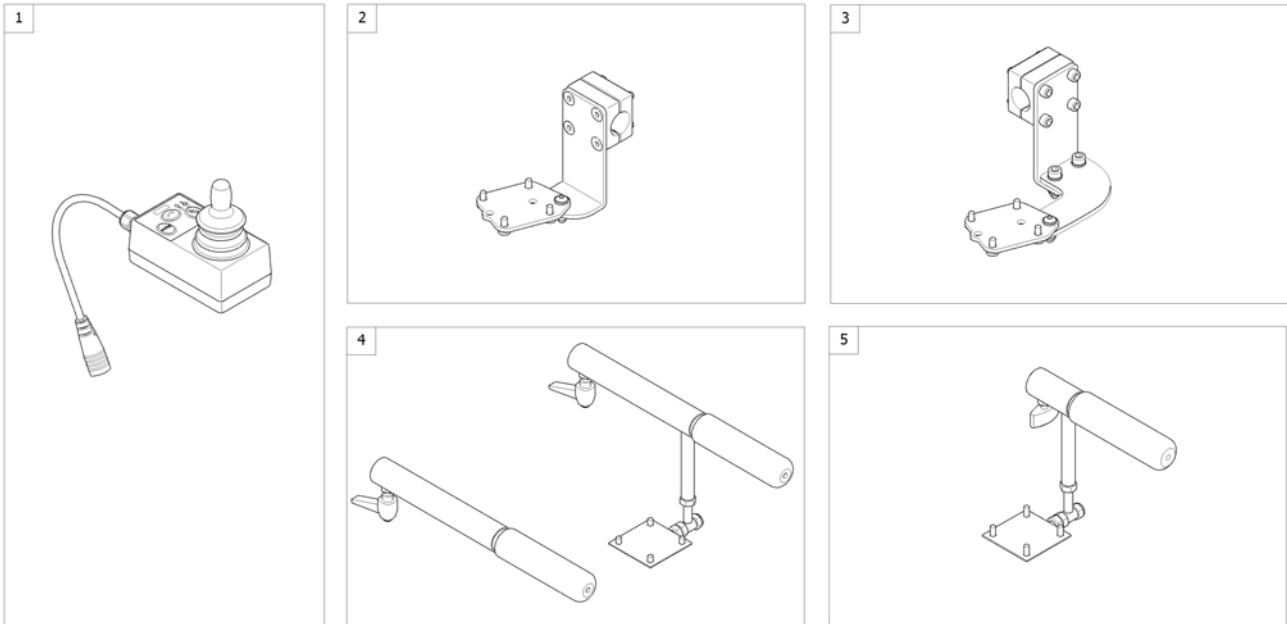

1	SP1532447	(Remote Dualcontrol R-Net Attendant") [1595]	1
2	-	(Remote Bracket Dual Control for Standard-/Flex2-Seat)	1
3	-	(Remote Bracket Dual Control for Modulite Seat)	1
4	-	(Remote Bracket Dual Control long for Firm Seat)	1
5	-	(Remote Bracket Dual Control short for Firm Seat)	1

# (Remote Bracket Dual Control for Standard-/Flex2-Seat)

1561352.iso

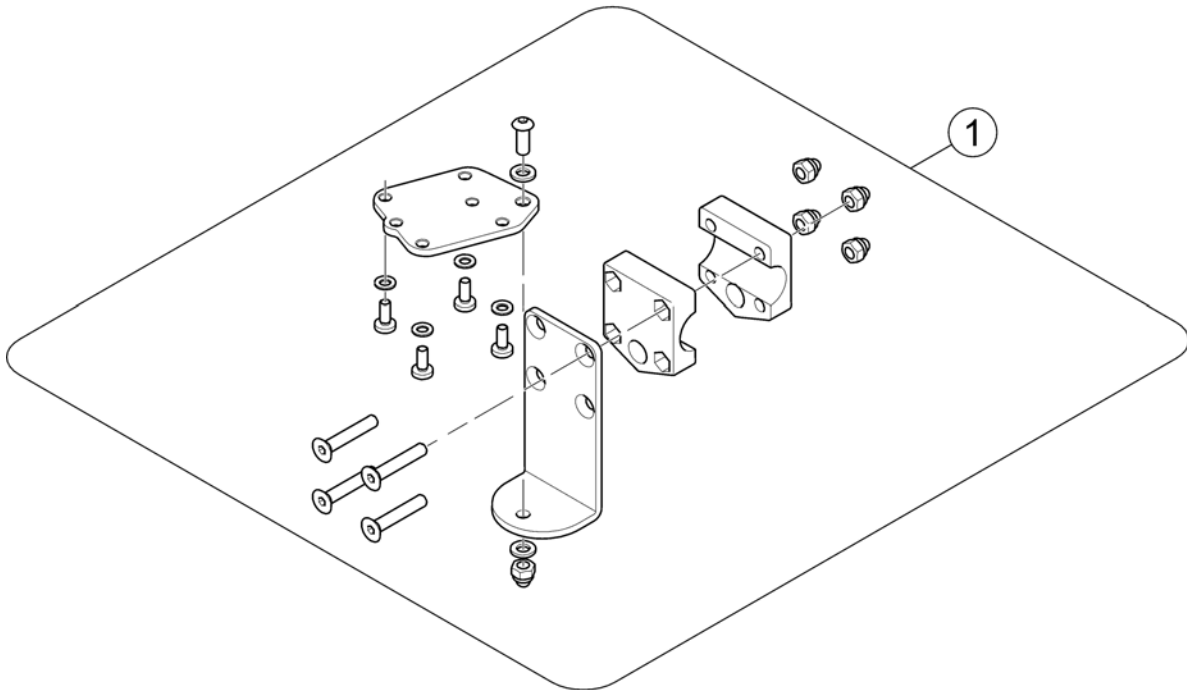


1 SP1531713

(Remote Bracket)

1

# (Remote Bracket Dual Control for Modulite Seat)

1574297.iso

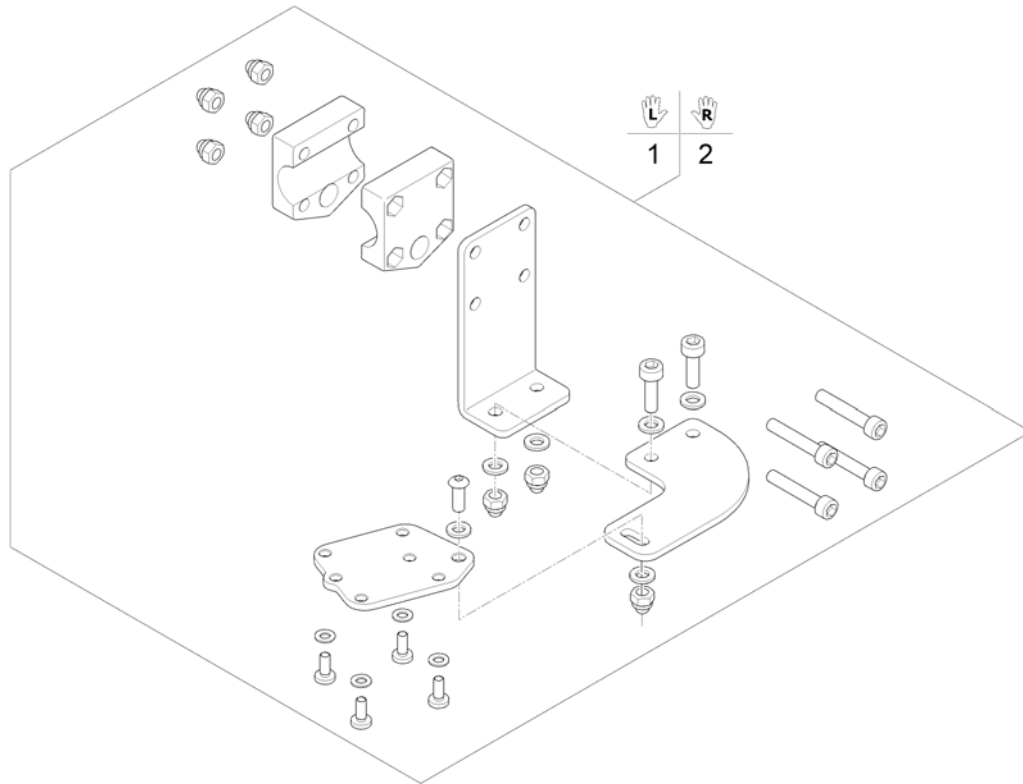


1	SP1571801	(Remote Bracket Dual Control LH)	1
2	SP1571802	(Remote Bracket Dual Control RH)	1



**(Remote Bracket Dual Control long for Firm Seat)**


1560542.iso

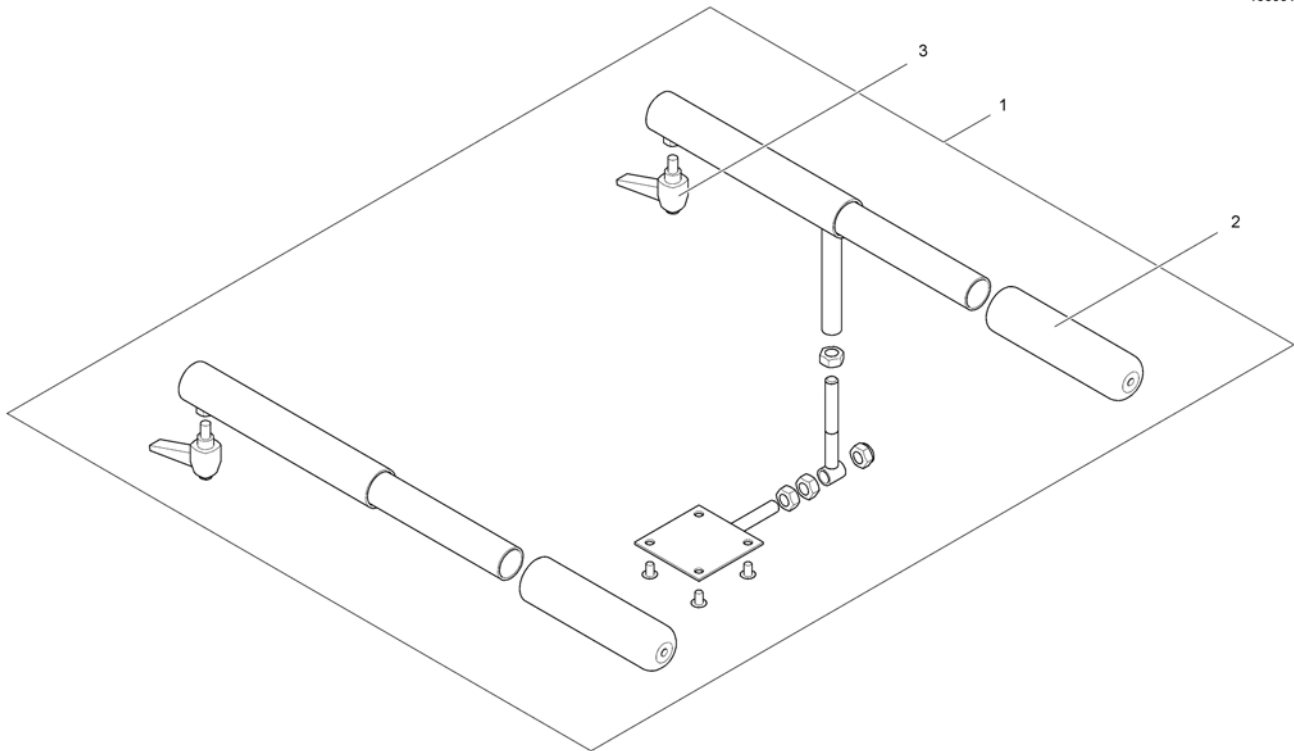

1	SP1531737	(Remote Bracket Dual Control long)	1
2	1435421	HÅNDTAK Ø22	1
3	L32902	LÅSESKRUE MED HENDEL ARMLEN M6X10 STORM/G40	1

## (Remote Bracket Dual Control short for Firm Seat)

1560541.iso

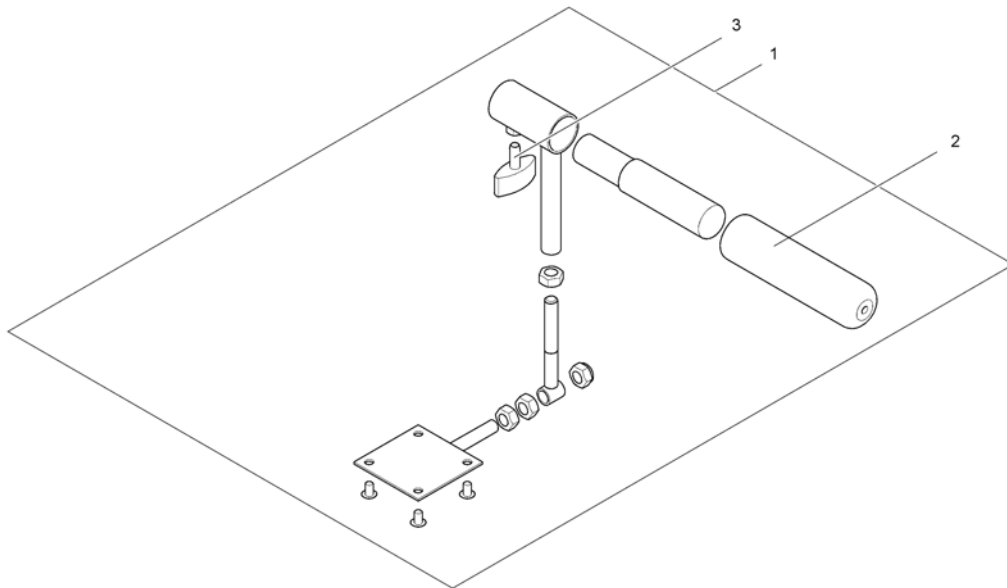


---

1	SP1541569	(Remote Bracket Dual Control short)	1
2	1435421	HÅNDTAK Ø22	1
3	L10024	SKRUE M6X16	1

---

-	(Buscable R-net)	1
-	(T-Adapter R-NET)	1
-	(Slow-Stop-Sensor & Holder)	1


**R-net Buscable**

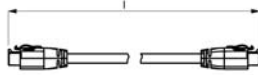

Position	length l [mm]
1	500
2	1000
3	1500
4	2000

---

1	SP1533675	(Buscable R-net 500mm)	1
2	SP1533676	(Buscable R-net 1000mm)	1
3	SP1533677	(Buscable R-net 1500mm)	1
4	SP1533678	(Buscable R-net 2000mm)	1

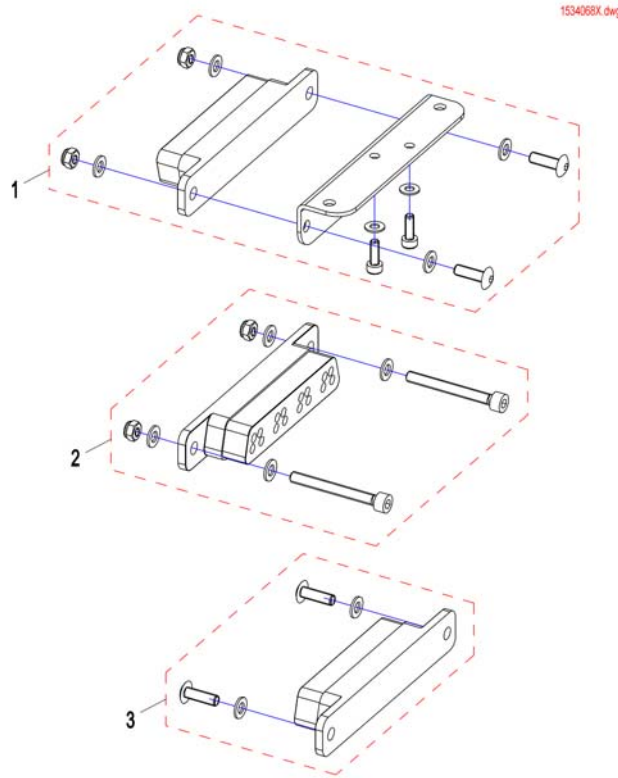

---

1	SP1536923	(T-Adapter R-NET for Flex2 with lifter)	1
2	SP1534949	(T-Adapter R-NET for Firmseat with / without lifter, and Flex2 without lifter)	1
3	SP1534064	(T-Adapter R-NET for Standardseat with / without lifter)	1



---

1	-	(VR2 Electronic Modules)	1
2	-	(VR2 Remote & Brackets)	1

---

-	(Powermodules)	1
-	(Lightboard)	1

## (VR2 Remote & Brackets)

1561182 no




---

1	-	(Remote VR2 with Brackets) <i>[1552/1554]</i>	1
2	-	(Remote Dual Control with Brackets) <i>[1594]</i>	1




1561252/90

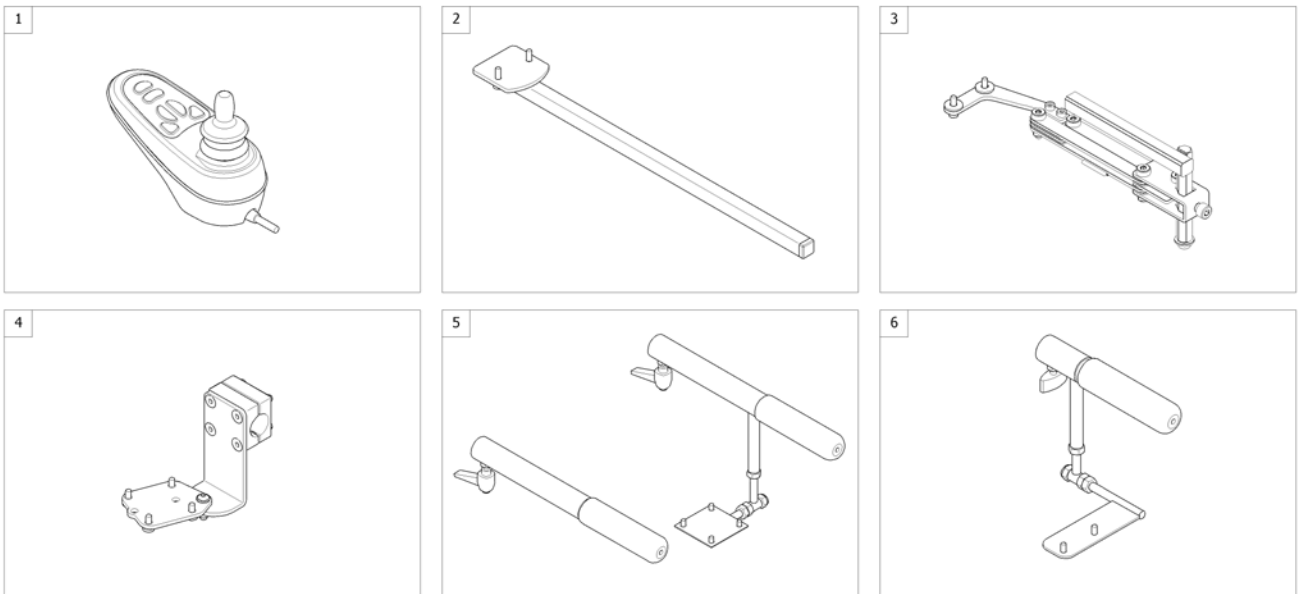


1	SP1510130	(VR2 Remote (2 actuators)) [1552/1554]	1
2	-	(Remote Bracket Standard) [1552+1207/1208]	1
3	-	(Remote Bracket Swing away & height adjust.) [1552+1707/1708]	1
4	-	(Remote Bracket Attendant for Standard-/Flex2-/Max-Seat) [1554+1207/1208]	1
5	-	(Remote Bracket Attendant long for Firm Seat) [1554+1207/1208]	1
6	-	(Remote Bracket Attendant short for Firm Seat) [1554+1207/1208]	1

## (Remote Dual Control with Brackets)

1560843 iso

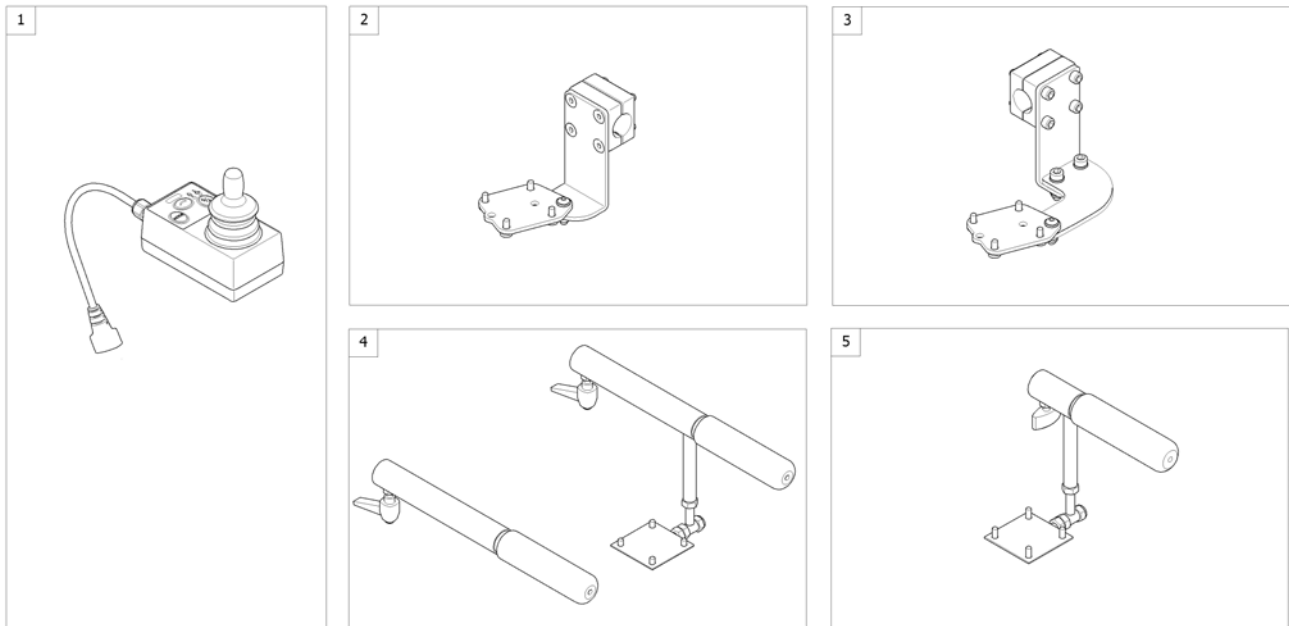

1	SP1531736	(Remote Dual Control (incl. Y-cable)) [1594]	1
2	-	(Remote Bracket Dual Control for Standard-/Flex2-Seat) [1594+1207/1208]	1
3	-	(Remote Bracket Dual Control for Modulite Seat)	1
4	-	(Remote Bracket Dual Control long for Firm Seat) [1594+1207/1208]	1
5	-	(Remote Bracket Dual Control short for Firm Seat) [1594+1207/1208]	1

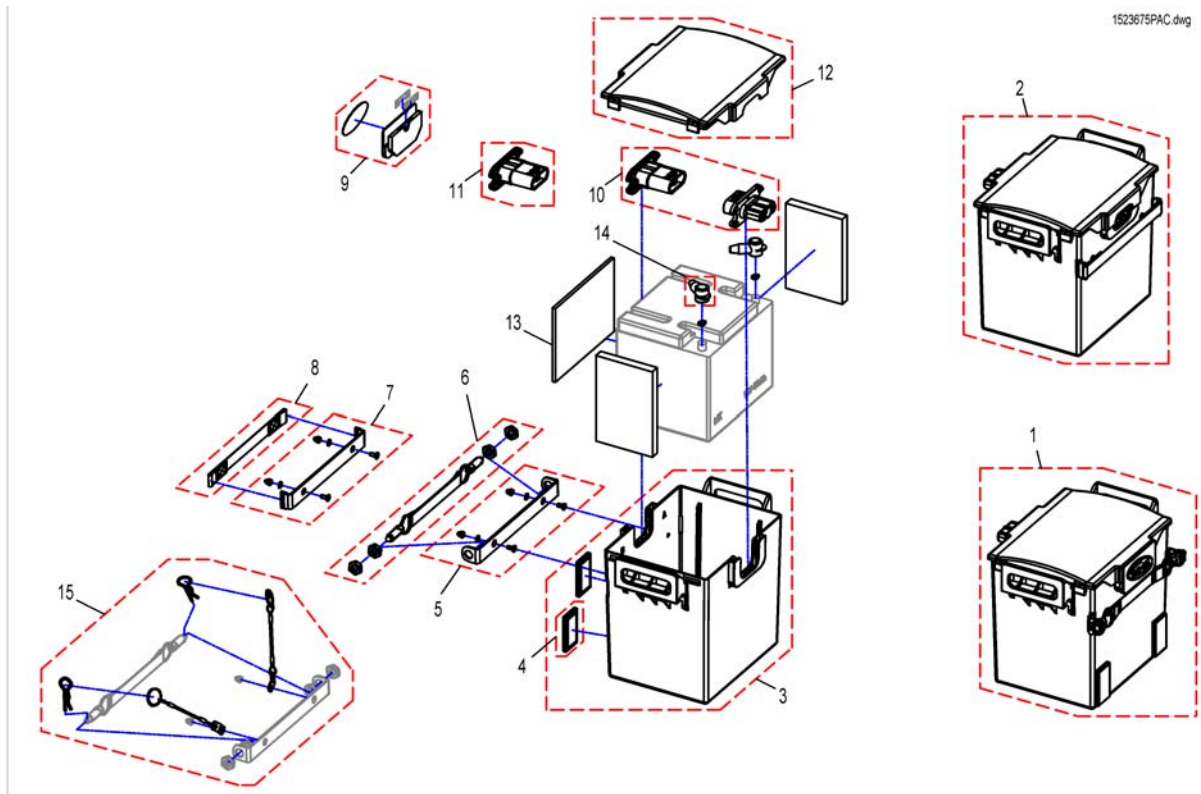


1	SP1523726	BATTERIBOKS KPL BORA	1
1	SP1543058	(Rear Battery Box cpl. (without Battery))	1
2	SP1523675	BATTERIBOKS FORAN BORA	1
2	SP1543059	(Front Battery Box cpl. (without Battery))	1
3	SP1527862	BATTERIBOKS NEDRE BORA	1
4	5807542	REFLEX BORA	1
5	SP1527863	(Bracket Battery Close)	1
6	SP1527864	(Battery Close)	1
7	SP1527865	(Bracket Handle Support)	1
8	SP1523732	(Strip Battery Box)	1
9	SP1527866	DEKKPLATE BORA	1
10	SP1523673	BATTERIKABEL M/PLUGGER	1
11	SP1534152	(Batterycable & Plug, Rear)	1
12	SP1523650	BATTERIBOKS DEKSEL BORA	1
13	SP1527867	(Battery Insert Set (1 Battery Box))	1
14	P52996	POLSKO DEKSEL	1
15	SP1531198	KRÆSJSIKRING BATTERIBOKS BORA	1

# (Batteries)

	Capacity [Ah]															
	1,2	12	30	31	32	36	38	40	40	50	50	50,9	60	73,6	75	97,6
Bora/SpectraXTR								8		10						
Dragon										10			13			
Dragon Vertic													13			
FDX													13			
G50													13	14		16
Kite													13	14		
Mirage				4				8								
Mistral3				4									12			
P9000XDT					5											
Pronto M41					5											
Pronto M61					5											
Spectra Blitz								8					12			
Spectra Comfort								7								
Spectra Plus								7								
Spectra Seat Riser								7								
Storm3										10			13	14		
Storm3 True Track	1														14	
Storm3 Vertic															14	
Storm4															14	
Storm4 Max															14	
Stream										10			13			
TDX-SP														14		
TDX-SP Narrow													12			
Tiger										10			13			
Topan													13			
Tornado								8								
Typhoon													13			
Typhoon GB	1												13			
Typhoon GB	1												13			
Zephyr	1												13			
Auriga			3					8		10						
Comet														14	15	
Leo						6			9							
Lynx		2														
Meteor														14		
Orion												11				
length [mm]	97	152	197	196	211	196	210	197	198	198	198	238	259	260	260	329
width [mm]	49	99	132	132	132	130	175	168	165	166	165	140	169	171	168	171
height [mm]	51	102	180	183	183	179	175	175	171	176	170	235	178	210	223	238
weight [kg]	0,6	3,9	11,1	10,5	11	12	14,6	14,4	14,2	14	16	17,1	19,3	24,3	26	32,5



i	-	(Change batteries only pairwise and from the same type!)	1
1	1440386	BATTERY 1,2AH 12V FRIKPL.THYPOON	1
2	1506493	(Battery 12V/12Ah C20 AGM)	1
3	P50021	(Battery, 12V/30Ah C20, GEL)	1
4	P50029	BATTERY 30AH MK12-31 TROJAN	1
5	SP1521459	(Battery, 12V/32Ah C20, GEL)	1
6	1505498	(Battery, 12V/36Ah C20, AGM)	1
7	P50028	(Battery, 12V/38Ah C20, GEL (UK & DK only))	1
8	PG50988	BATTERY 40AH MK12-40 TROJAN	1
9	SP1550692	(Battery, 12V/40Ah C20, AGM)	1
10	SP1522107	(Battery, 12V/50Ah C20, AGM)	1
11	SP1521695	(Battery, 12V/50Ah C20 AGM)	1
12	PG50989	BATTERY 50AH MK12-50 TROJAN	1
13	1432944	BATTERY 60AH MK12-60 TROJAN	1
14	P50016	BATTERY 70AH MK12-73 TROJAN	1
15	SP1522026	(Battery, 12V/75Ah C20, AGM)	1
16	1478675	BATTERY 97AH MK12-97 TROJAN	1

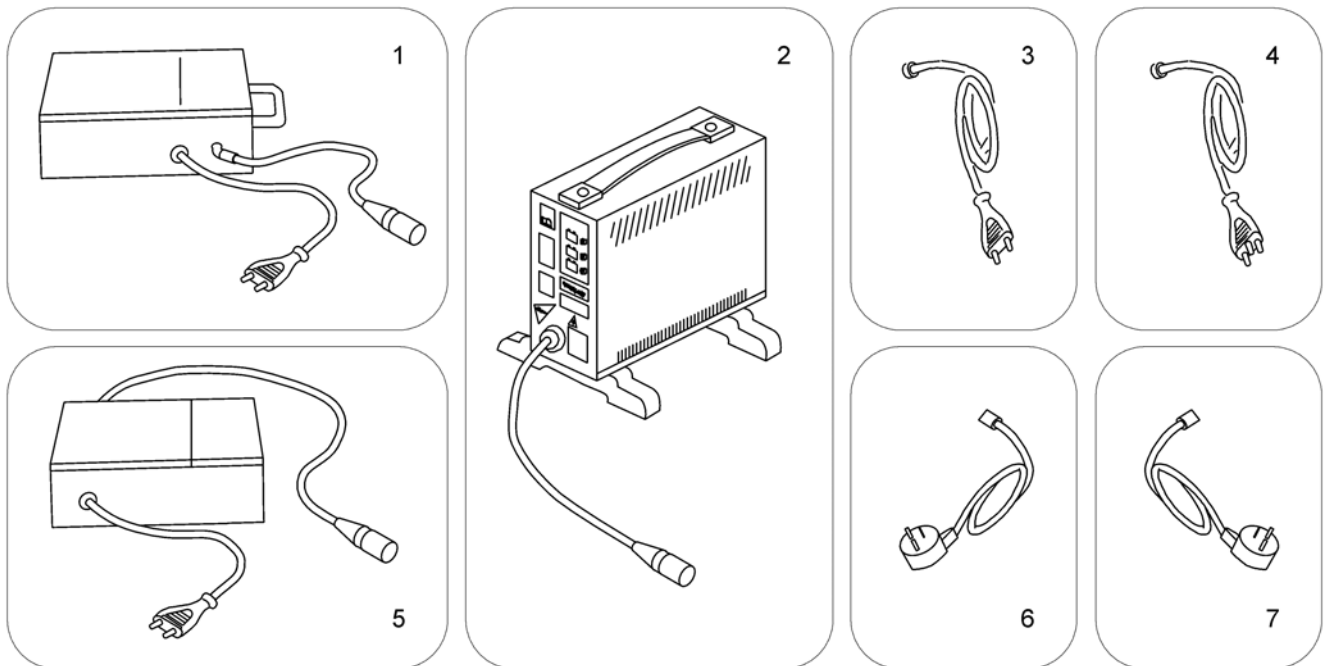

1	5813987	(Charger 8A, Gi Popular (Sweden only))	1
2	SPP51981	(Charger 8A for Batteries up to 73Ah (without power cord))	1
3	P21079	(Power Cord, Continental Shockproof Plug)	1
4	1443841	(Power Cord, Swiss)	1
5	1510201	(Charger 10A, Powercharge Classic (G50), for Batteries from 97AH (with power cord))	1
5	SP1545511	LADER LADAC/DEFA 701239 MED PLUGG	1
6	P21080	(Power Cord, UK)	1
7	SP1553849	(Power Cord, Israel)	1

## (Cables)

GSM63003PAC.dwg

### Shark-Buskabel

Länge / lenght (mm)	Art.-Nr.	Pos.
1000	1448922	1

### Kabel für Licht- und Aktuatoremodule (CLAM) Cable for Light- and Actuator modules (CLAM)

Licht / Light	Sitzwinkelverstellung Seatangleadjustment	Rückenverstellung Backrestadjustment	Sitzhöhenverstellung Seathightadjustment	Beinstützenverstellung Legrestadjustment	Art.-Nr.	Pos.
x	-		x	x	1488951	2


---

1	1448922	(Buscable Shark 1000mm)	1
2	1488951	KABELSETT KPL FOR LYS OG AKTUATOR DRAGON	1



---

-	SP1533675	(Buscable R-net 500mm)	1
-	SP1533676	(Buscable R-net 1000mm)	1
-	SP1533677	(Buscable R-net 1500mm)	1
-	SP1533678	(Buscable R-net 2000mm)	1

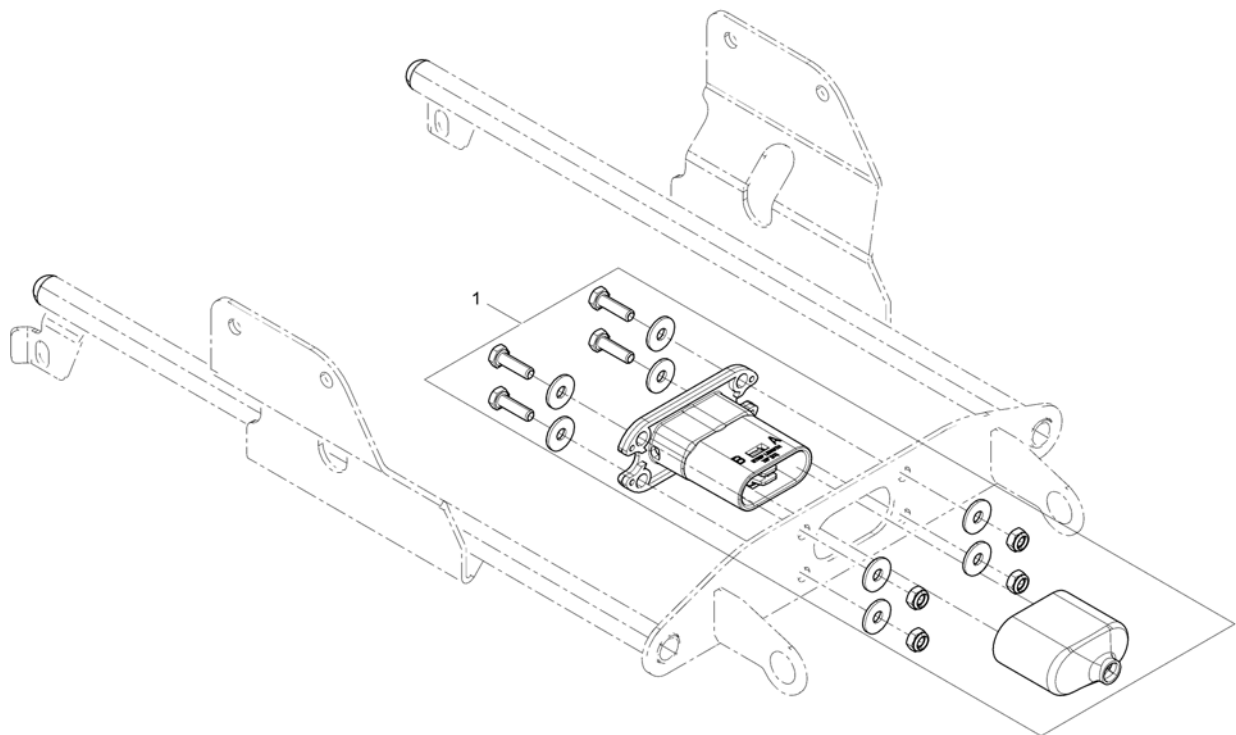


---


1	SP1528270	(Batterycable & Plug (ACS1))	1
1	SP1526803	(Batterycable & Plug (ACS2/Shark))	1
1	SP1526804	(Batterycable & Plug (VR2))	1
1	SP1533701	(Batterycable & Plug (R-Net))	1



## (Light System (German homologation))

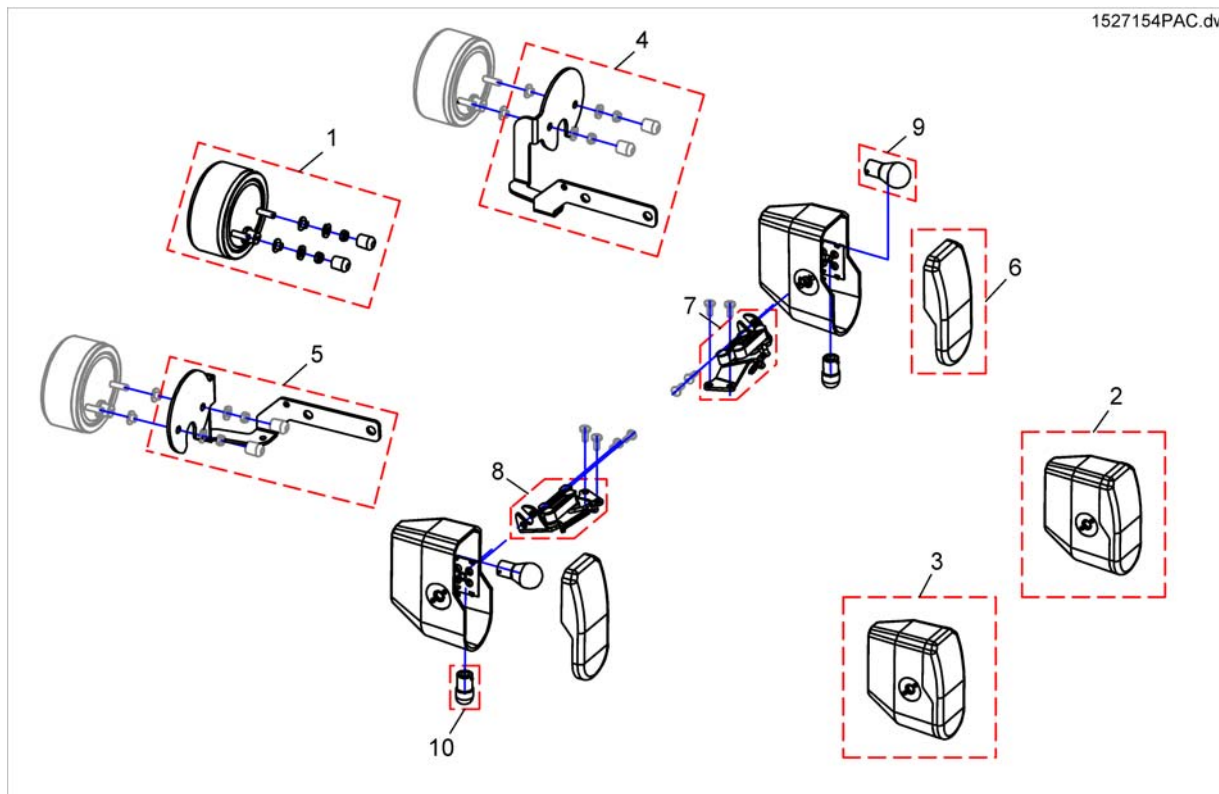


1	SP1529532	BAKLYS VE BORA	1
1	SP1529479	BAKLYS HØ BORA	1
2	1442735	FRONTLYS VE KPL DRAGON <i>In case of retrofitting lights, please also order the SP1528415</i>	1
3	1442734	FRONTLYS HØ KPL TYPHOON DRAGON <i>In case of retrofitting lights, please also order the SP1528415</i>	1
4	SP1527092	(Rearlight bracket, LH)	1
5	SP1527091	(Rearlight bracket, RH)	1
6	1433737	GLASS FRONTLYS STORM 3	1
7	1428081	SVINGMEKANISME VE LYS FORAN STORM3 DRAGON	1
8	1428082	SVINGMEKANISME HØ LYS FORAN STORM3 DRAGON	1
9	SP1419271	(Incandescent Lamp 24V/21W yellow)	1
10	SPP31108	(Incandescent Lamp 24V/10W BA15s)	1

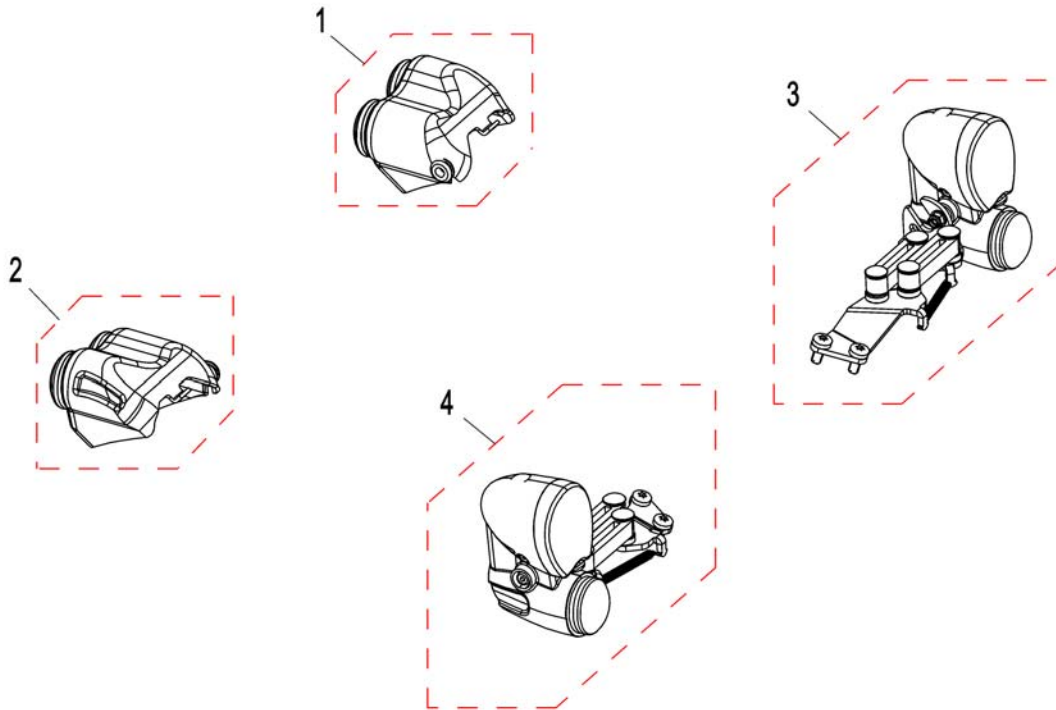

If retrofitting light, or for chairs befor CW. 06/2010, fenders are needed.

---


1	SP1531503	BAKLYS VE BORA	1
2	SP1531502	BAKLYS LED HØ BORA	1
3	SP1531276	LYS FORAN LED VE BORA	1
4	SP1531650	LYS FORAN HØ BORA	1

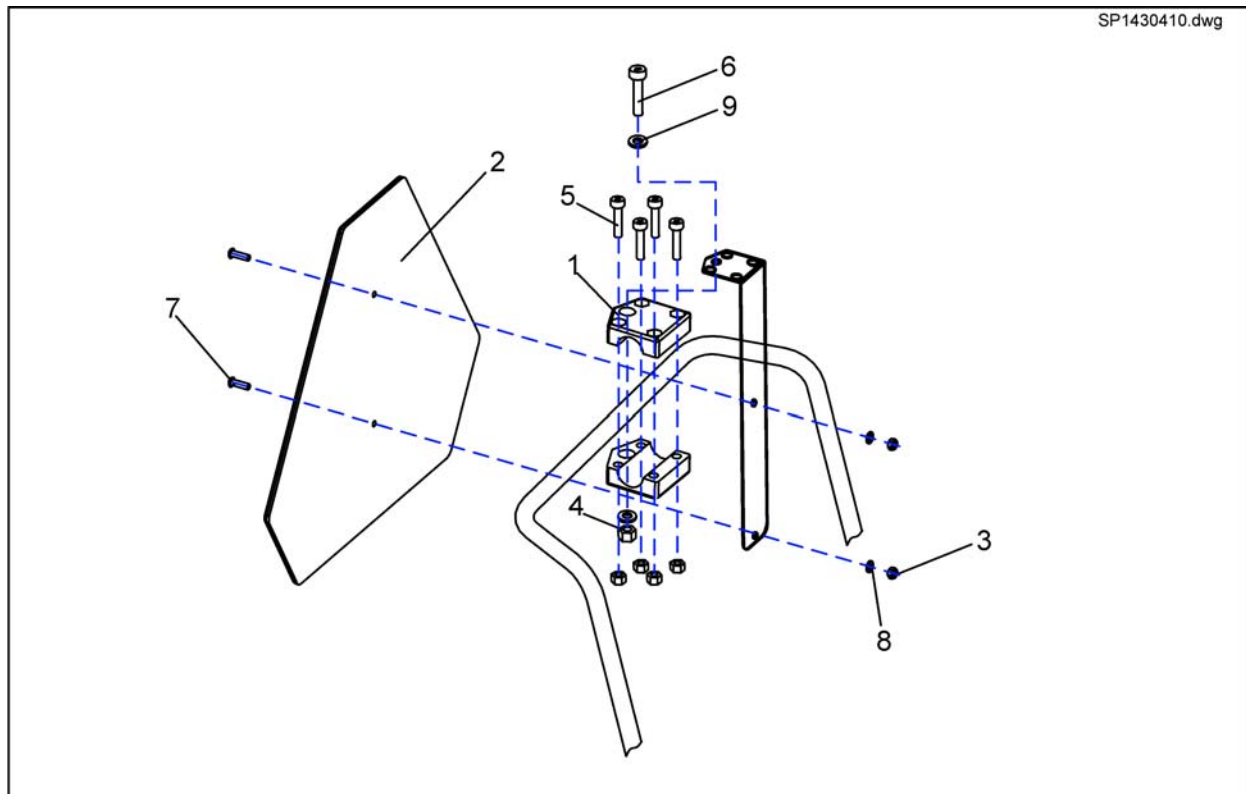

1-10	SP1430410	(Rear markig sign ECE-R 69 cpl.)	1
1	Z12008A.P40	(Gripping clip with hexagon hole, RAL 9005 black)	1
2	1430834	(Rear Marking Sign Ece-R 69)	1
3	F20109	MUTTER 6 KT M6	1
4	F00002	SEKSKANTMUTTER M8	1
5	D50004	(Cylinder bolt DIN912-M06x30-8.8 bk)	1
6	D00102	SEKSKANT SKRUE M8X45	1
7	D90187	(Lens head screw DIN7380-M06x20-10.9 bk)	1
8	G00115	SKIVE 6,4X12,5X1,6 MISTRAL	1
9	G00107	STOPPSKIVE 8,4X17X1,6	1
10	1430418-9005	(FIXING BRACKET FOR REAR SIGN/RAL 9005)	1

# (Kerb Rider)


1527146PAC.dwg

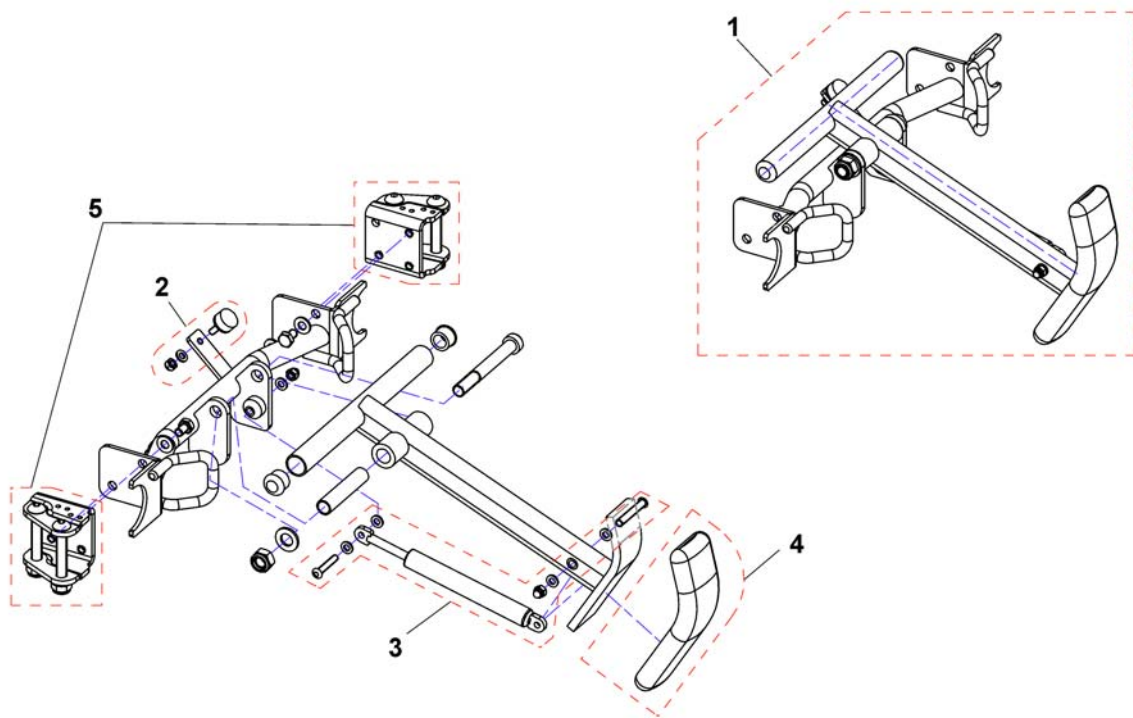

1	SP1527146	(Kerb Rider)	1
2	SP1440366	(rubber buffer)	1
3	SP1528414	GASSFJÆR BORA	1
4	B11037B	GUMMIFOT T KANTKLÄTTRA	1
5	SP1528415	FESTEBRAKETT KANTKLATRER BORA	1


# (Kerb Rider (with parking position))

1573593.iso

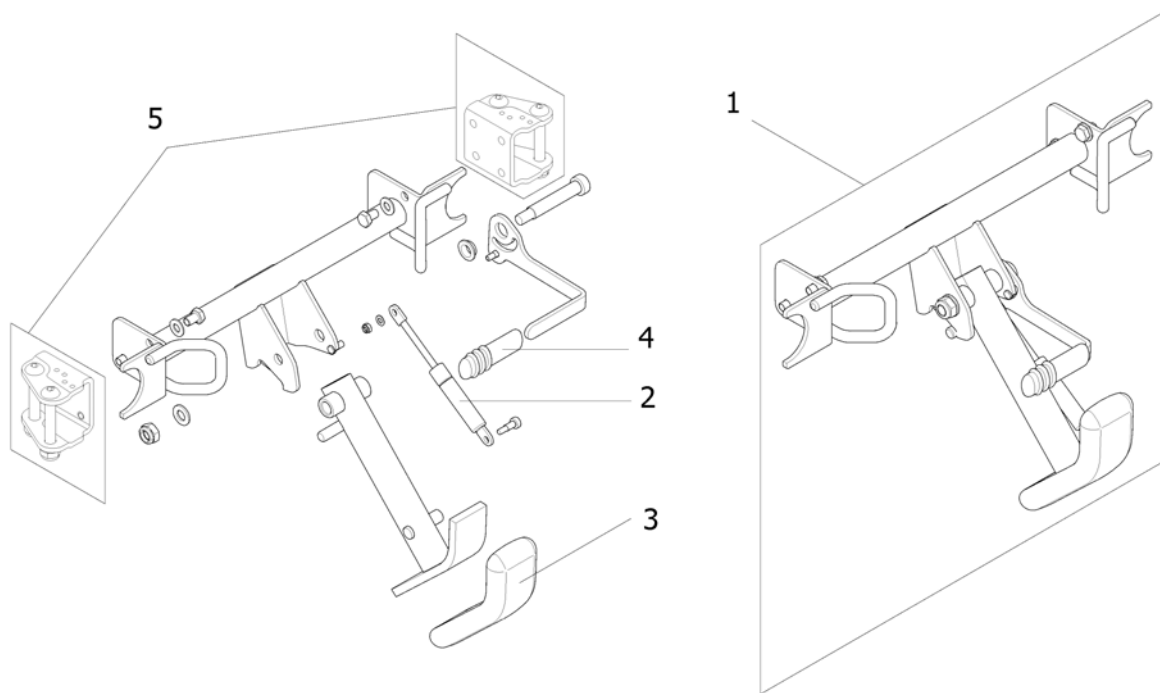


1	SP1568291	(Kerb Rider (with park position))	1
2	SP1559722	(Gas Spring)	1
3	B11037B	GUMMIFOT T KANTKLÄTTRA	1
4	SP1101725	PROPP FOR FRIKPL HENDEL STORM4	1
5	SP1528415	FESTEBRAKETT KANTKLATRER BORA	1

## (Lap Belt)

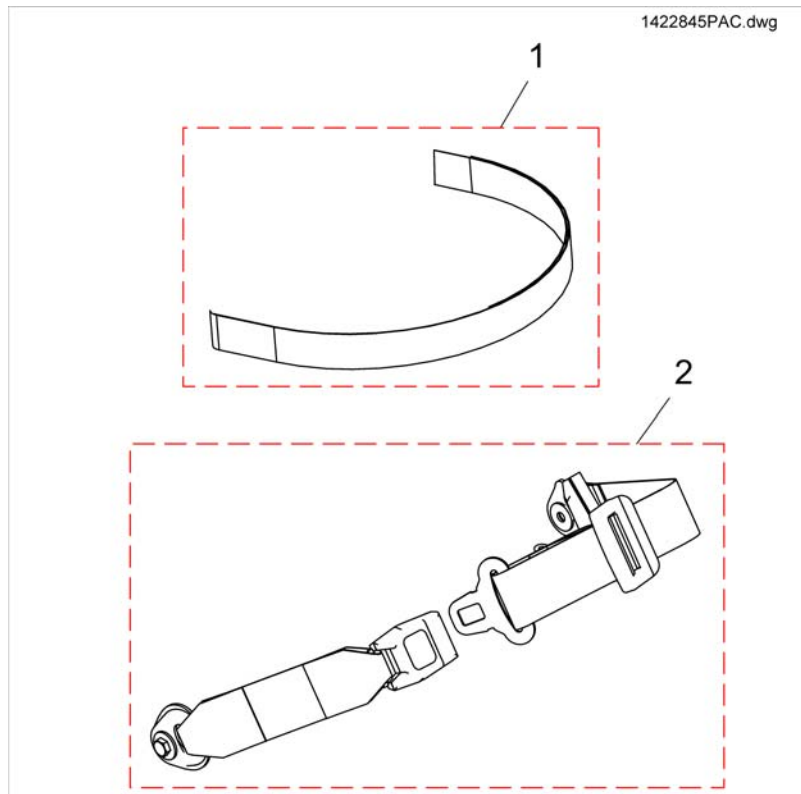

---


1	1422869	HOFTEBELTE BORA EXTRA BORA INNE	1
2	1422845	HOFTEBELTE MED SNAP LÅS STORM 3/ DRAGON	1
2	1551067	HOFTEBELTE, LANG MAX	1

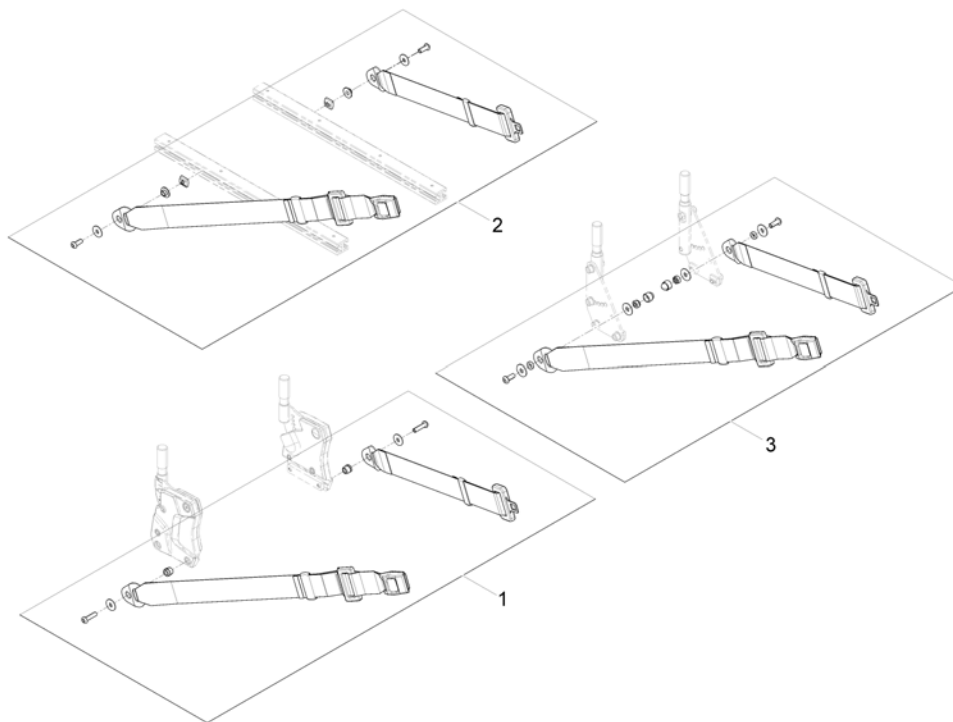

---


1	SP1570381	(Lap Belt (fix mounting on Backrest Support), cpl.)	1
2	SP1571183	(Lap Belt (flexible mounting in side rails), cpl.)	1
3	1422845	HOFTEBELTE MED SNAP LÅS STORM 3/ DRAGON	1

# (Rearview Mirror)

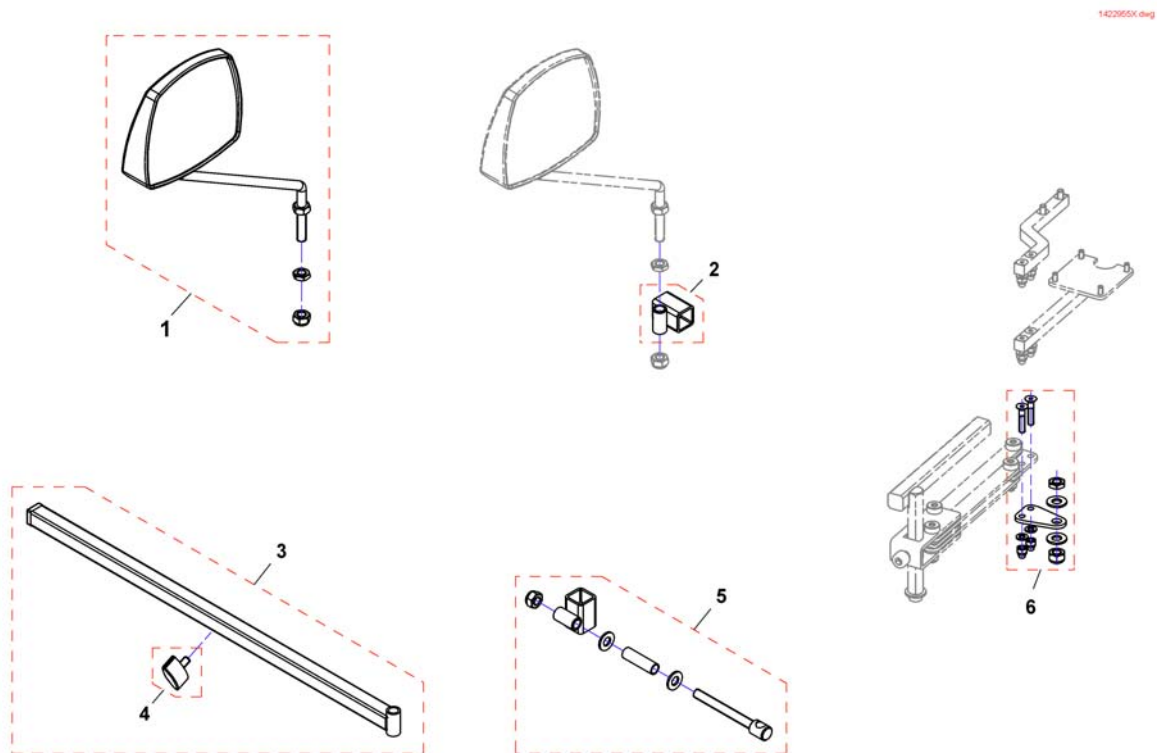

1	SP1422955	SPEIL KPL KUN SPEIL STORM 3	1
2	F276005.00	HOLDER SPEIL DRAGON STORM4 HUSK SP1527151	1
3	SP1423435	HOLDER SPEIL STORM3	1
4	L31001	VINGSKRUV M 6X10 GARANT G24 S	1
5	SP1422957	SPEIL FESTEBRACKETT STORM4 PÅ SWINGAWAY	1
6	SP1527151	FESTE FOR SPEILFESTE STORM4 TDX BORA	1



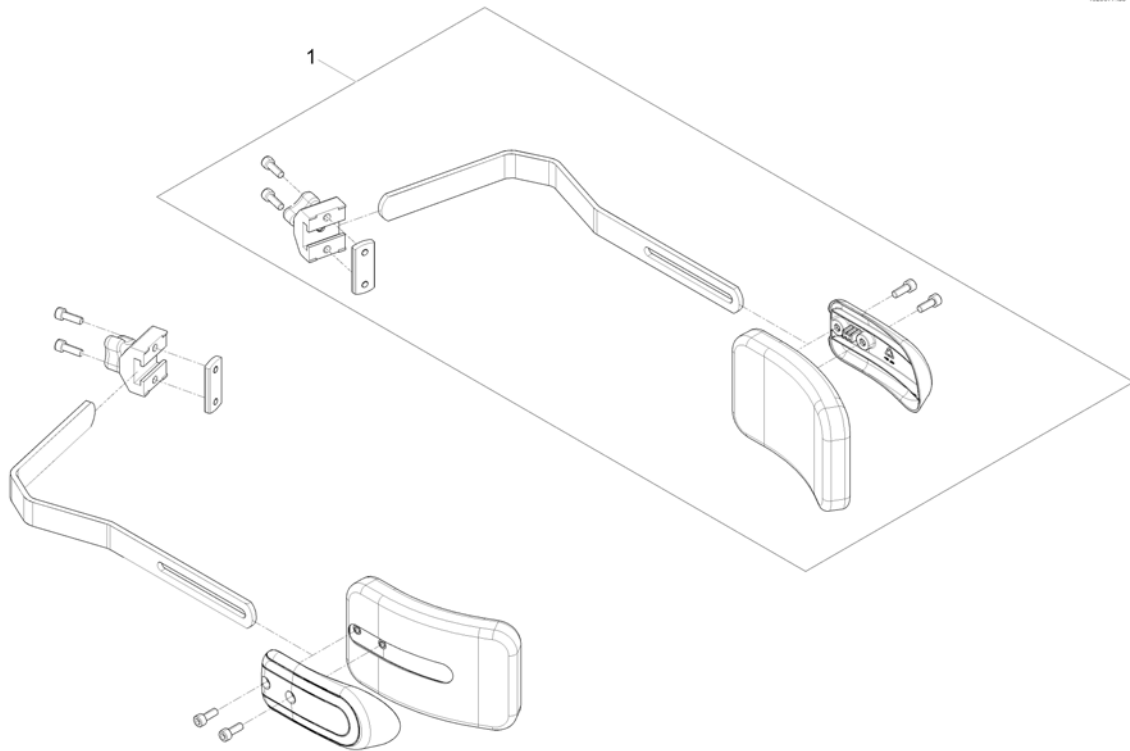


1 SP1561841

SIDESTØTTE FAST HØVE TDX

1

## (Trunk support, multi adjustable)


1502647.no

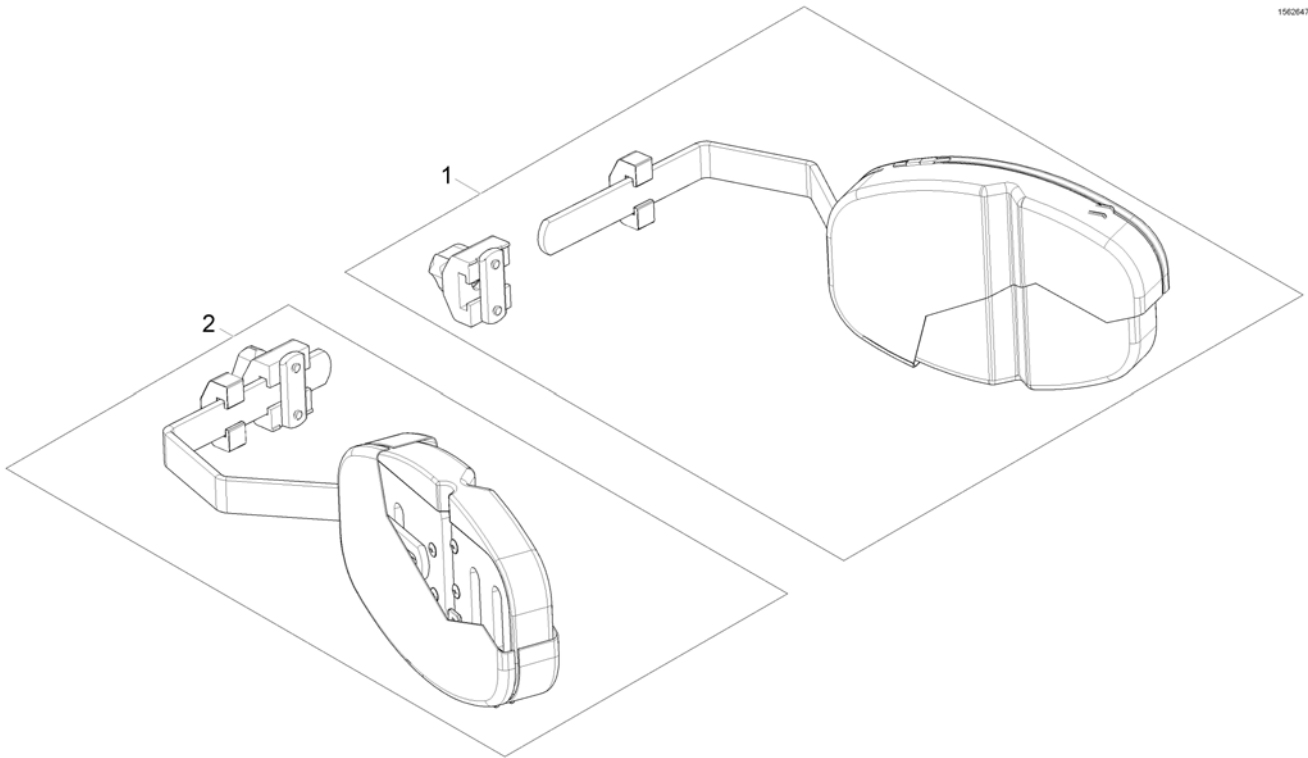

---

1	1448692	SIDESTØTTE VE TR26 MULTI REGULERBAR	1
2	1448693	SIDESTØTTE HØ TR26 MULTI REGULERBAR	1

1502718.jpg

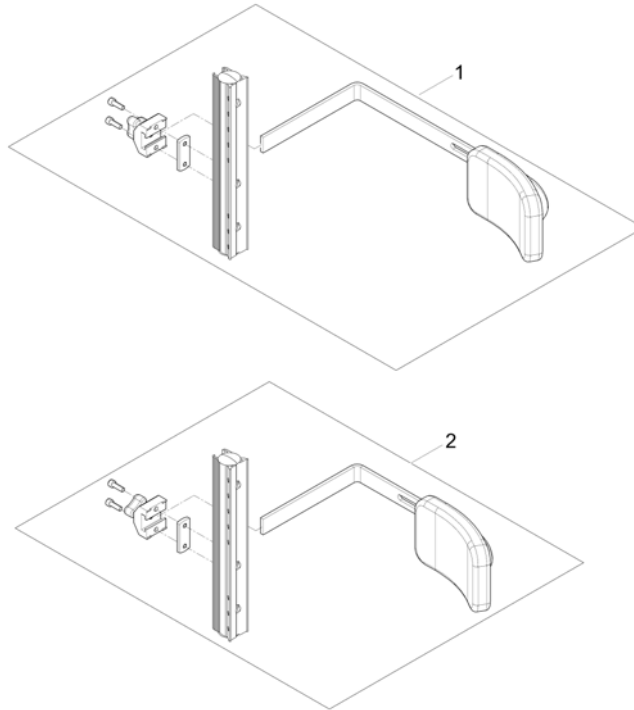


---

1	SP1561819	(Trunk support, long, cpl.)	1
2	SP1561818	(Trunk support, short, cpl.)	1

# (Trunk support bracket (Modulite))


1572155.iss

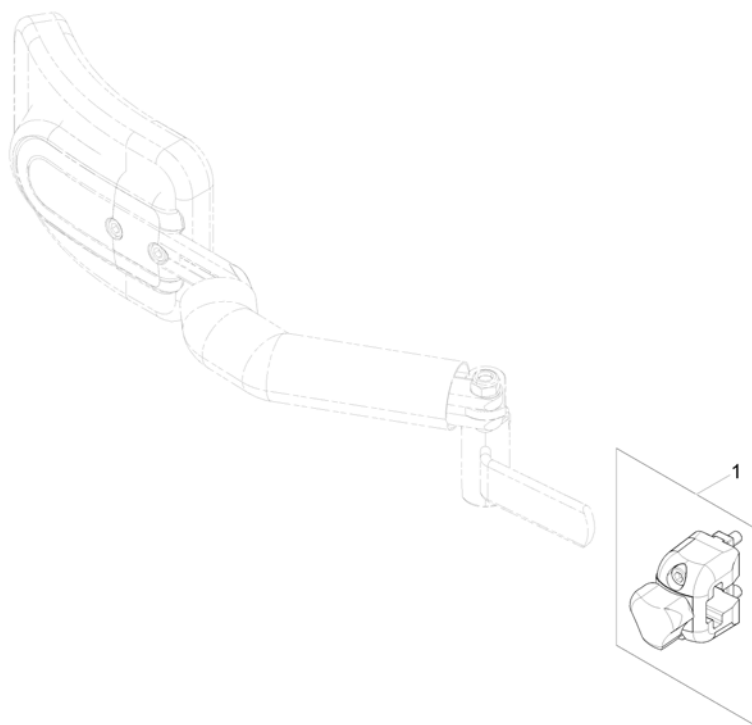

1 1543574


SIDESTØTTEFESTE KPL LAGUNA 2, CLEMATIS

1

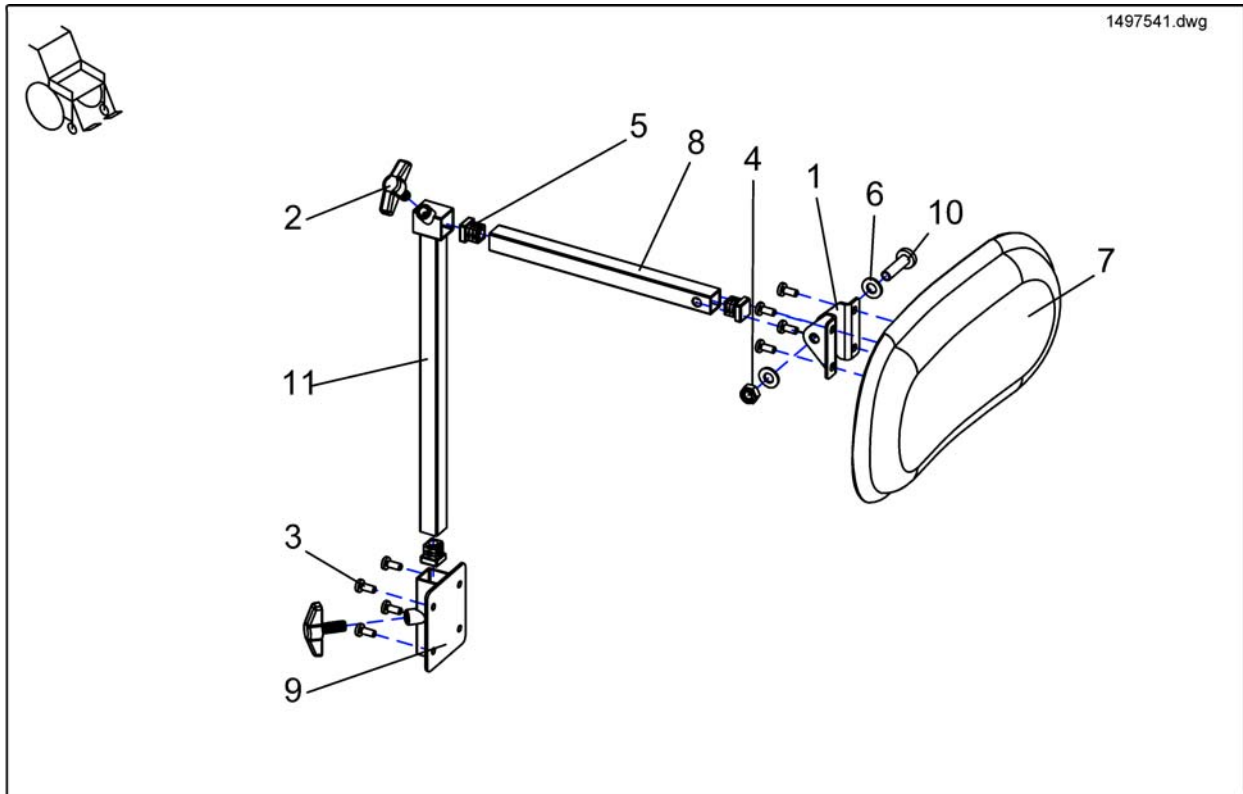
**(Headrest Curve 1, Firm-Seat)**

1-11	1450781	TREKK HODESTØTT SORT BORA	1
1-11	SP1497541	(Headrest Curve 1, Jemima, cpl.)	1
1-11	SP1497542	(Headrest Curve 1, black bird, cpl.)	1
1	26002350-9005	(Holder)	2
2	256099	(Wing Screw M8 X 20 639)	2
3	D30197	(Lens head screw DIN7985-M05x12-4.8 zc)	8
4	F20105	(Hexagon Nut M8 - DIN985 zc)	1
5	L01055	ENDESTYKKE FESTEBR.	3
6	G00106	SKIVE M8 DRIVHJUL MIRAGE	2
7	SP26002210BKB	(Headrest Curve 1, Black Bird)	1
7	26002210BKJ	(Headrest Curve 1, Black Jemima)	1
8	1422193-9005	(Horiz. Tube H/Rest Supp.200mm)	1
9	26002351	(Tube Headrest)	1
10	00007840	(Lens head screw DIN7985-M08x30-4.8 zc)	1
11	26002348-0205	(Tube)	1

**(Headrest sidewise adjustable)**

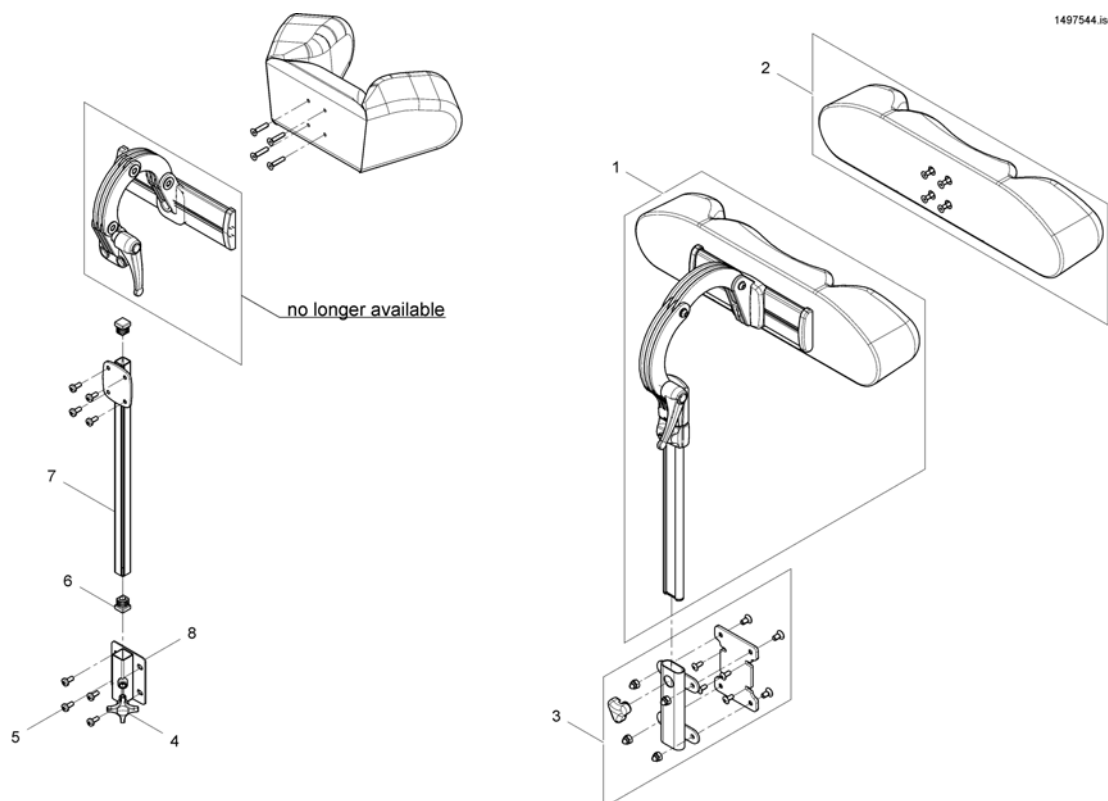


1	1446559	HODESTØTTE MULTI KPL TR26 MED SIDEFORFLYTNING	1
2	1422865	NAKKEPUTE MULTI SORT CONTUR STORM3	1
2	1422866	(HEADREST PAD STD./CONTOUR GREY)	1
2	SP1522214	(HEADREST PAD STD./CONTOUR BLACK PLUSH)	1
3	SP1544680	(MOUNTING-KIT)	1
4	256099	(Wing Screw M8 X 20 639)	1
5	D30197	(Lens head screw DIN7985-M05x12-4.8 zc)	1
6	L01055	ENDESTYKKE FESTEBR.	1
7	1451639-0205	(Headrest REA Angle Adjustment)	1
8	26002351	(Tube Headrest)	1

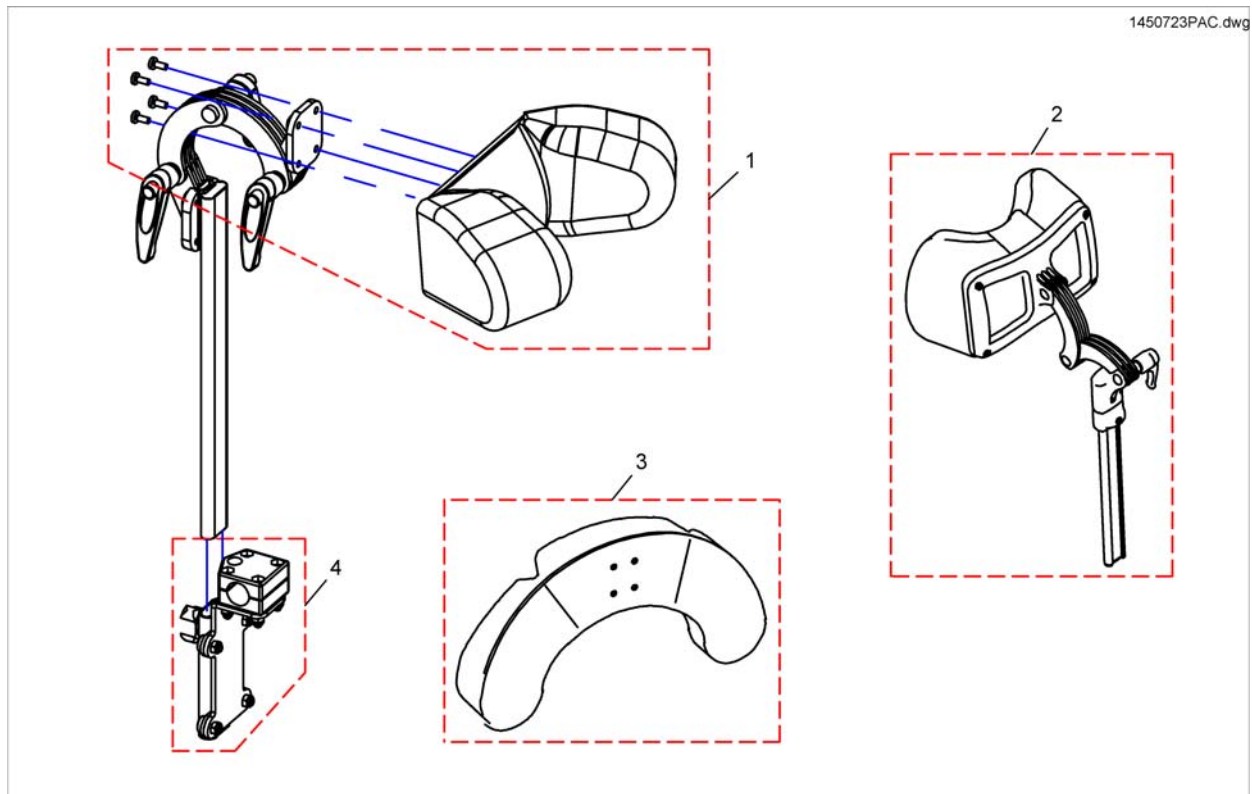

1	1487404	HODESTØTTEPUTE KPL BORA	1
2	SP1527050	(Neckrest, depth adjustable)	1
3	1444304	PUTE HAKESTØTTE BELLIS/A ZALEA	1
4	SP1422914	HOLDER HODESTØTTE RYGG STD RYGG STORM3 DRAGON	1

# (Abduction Cushion)

1526960PAC.dwg

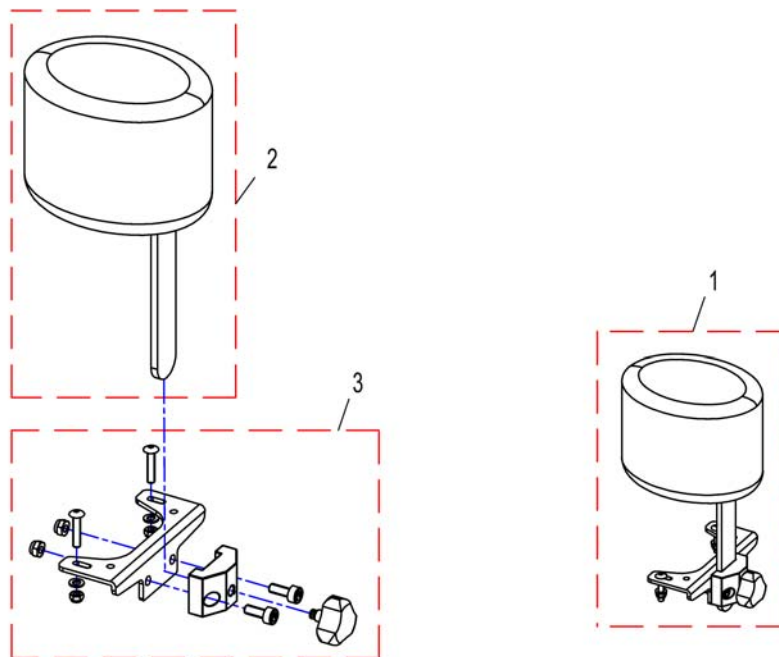


1	SP1526960	ABDUKSJONSKLOSS DARTEX BORA	1
2	1444172	ADB KLOSS KPL TR26 ASSIST	1
3	1493099	BRUK SP1450464 LOSS AZALEA	1



## (Headrest with cheek support)


1446560X.dwg

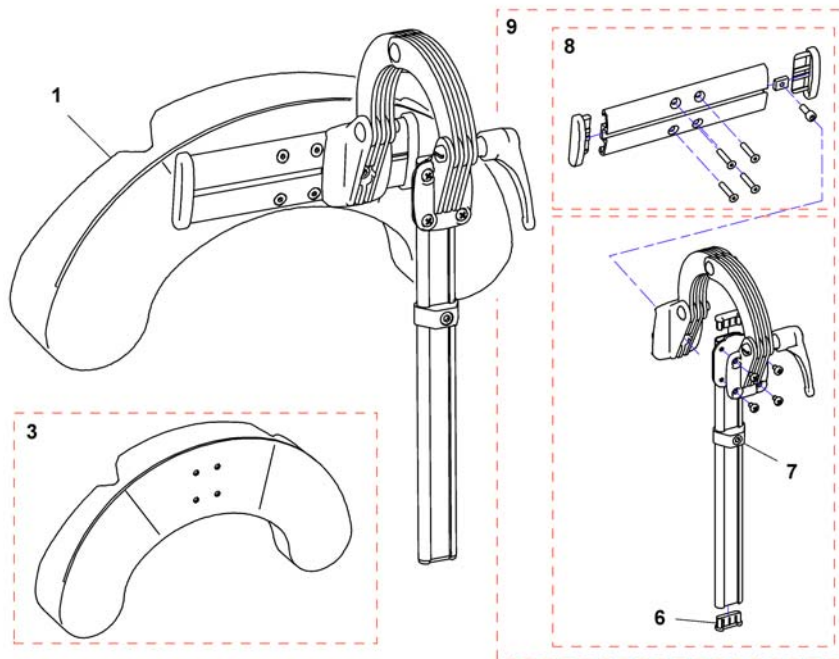

1	1446560	HODE/SKINN STØTTE KPL TR26 MED SIDEFORFLYTNING	1
1	SP1541658	(Head/cheek support TR26 cpl. sidewise adjustable EasyAdapt)	1
3	1444304	PUTE HAKESTØTTE BELLIS/A ZALEA	1
5	SP1529572	(Headrest holder for EasyAdapt)	1
6	5716497	HYLSE MESSINGSKRUE FOR LOMME REA	1
7	1448661	STOPPER FOR HREG NAKKE OG HODEST REA M SKRUE	1
8	1445838	SKINNE HODESTØTTE KPL BE LLIS AZAL FOC SPI3	1
9	1485538	FESTE HODESTØTTE STORM4	1

## (Luggage Rack)


1551407X.dwg

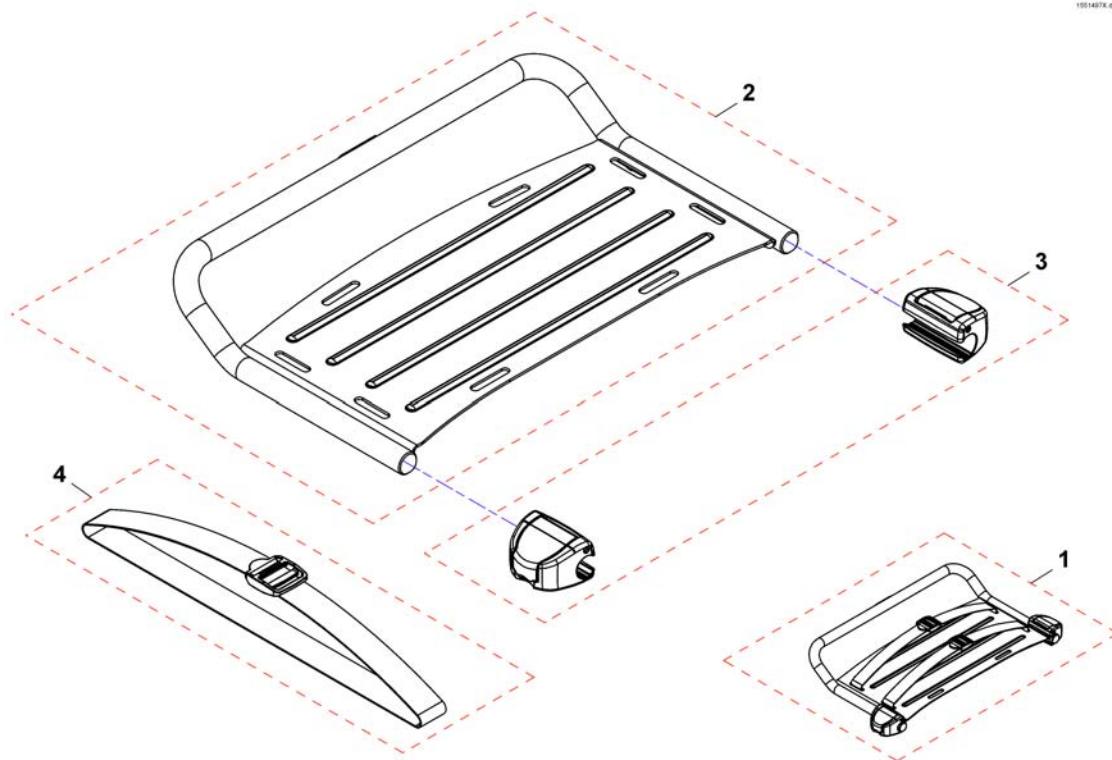

Only for chairs without lights, or with LED lights.

1	SP1551619	BAGASJEBÆRER STORM4 (BE- STILL FESTE, SP1546314)	1
2	SP1551497	(Luggage rack)	1
3	SP1556240	(Luggage rack bracket)	1
4	F384020	MONT SATS BATTERI BOKS,	1

156243X.dwg

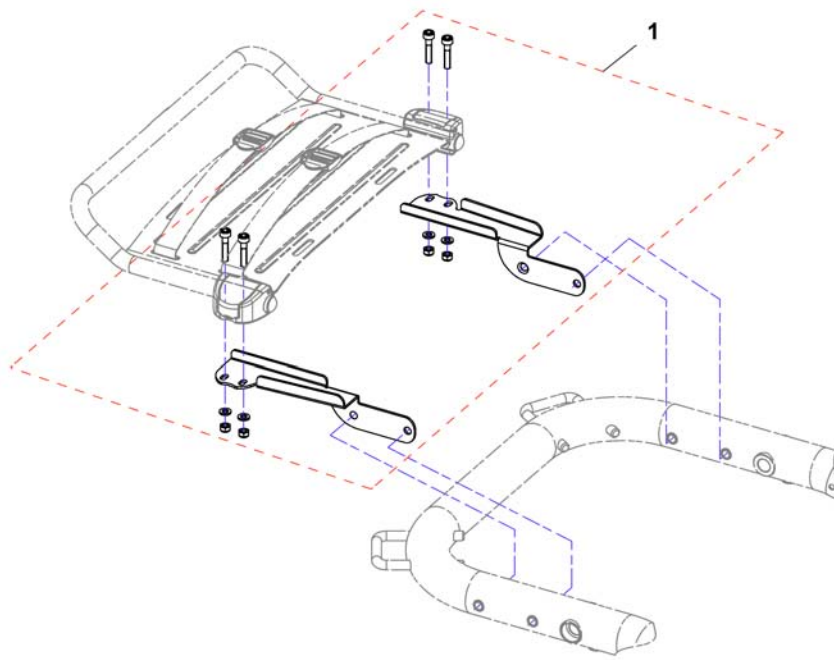


1 SP1556243

(mounting kit luggage rack)

1


# (Cane holder (Std., Flex2, Firm, Max Seat))

1567253.iso

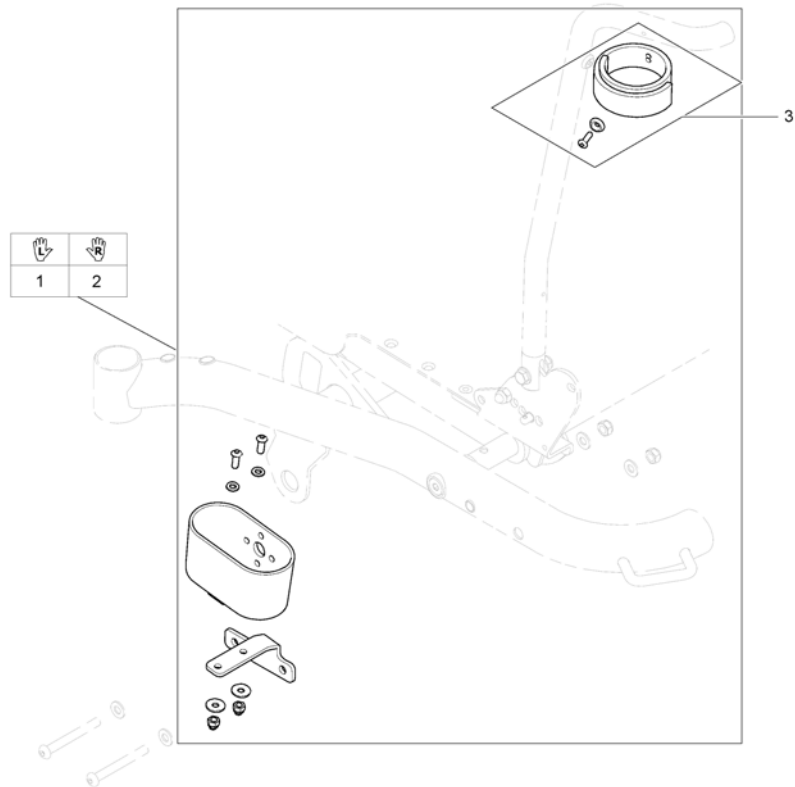


1	SP1528488	KRYKKEHOLDER VE BORA	1
2	SP1531230	KRYKKEHOLDER HØ BORA	1
3	SP1420493	(Belt for caneholder)	1

## (Cane holder (Modulite: backrest frame))

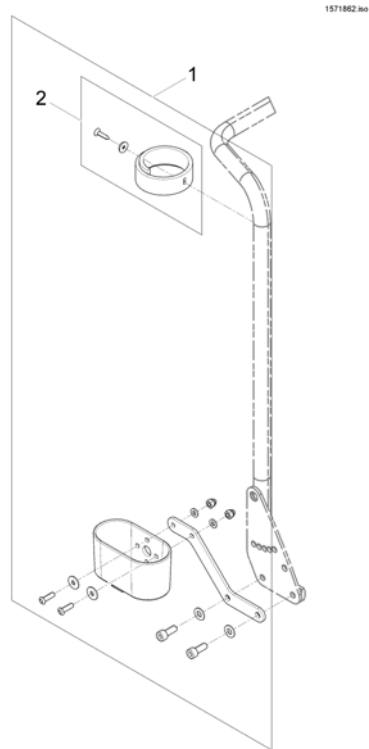

1	SP1570396	(Cane holder, standard backrest)	1
2	SP1420493	(Belt for caneholder)	1

## (Cane holder (Modulite: back posts man recliner))

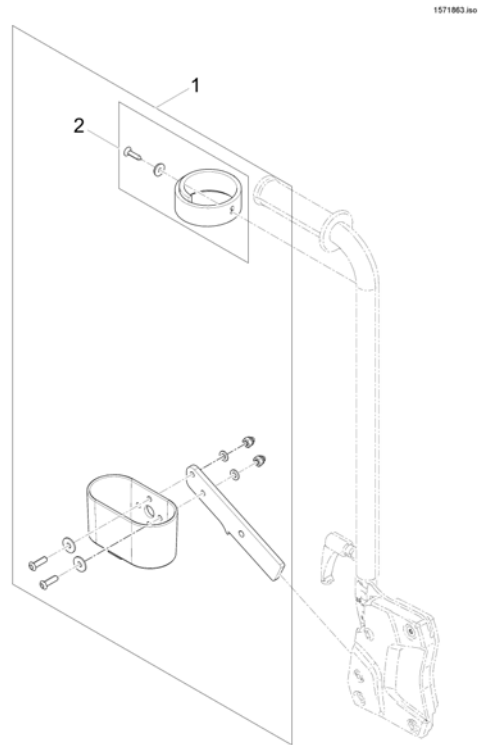


---

1	SP1570443	(Cane holder, manual backrest)	1
2	SP1420493	(Belt for caneholder)	1


# (Cane holder (Modulite: back posts pow recliner))

1571837.no

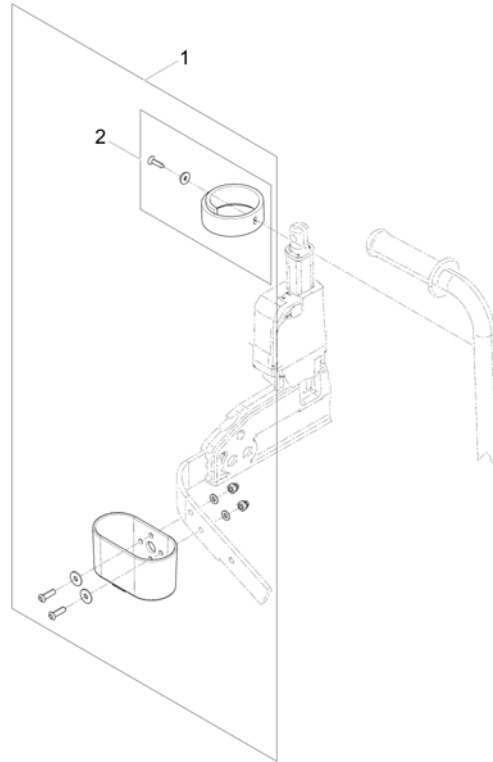


---





1	SP1570382	(Cane holder, powered backrest)	1
2	SP1420493	(Belt for caneholder)	1

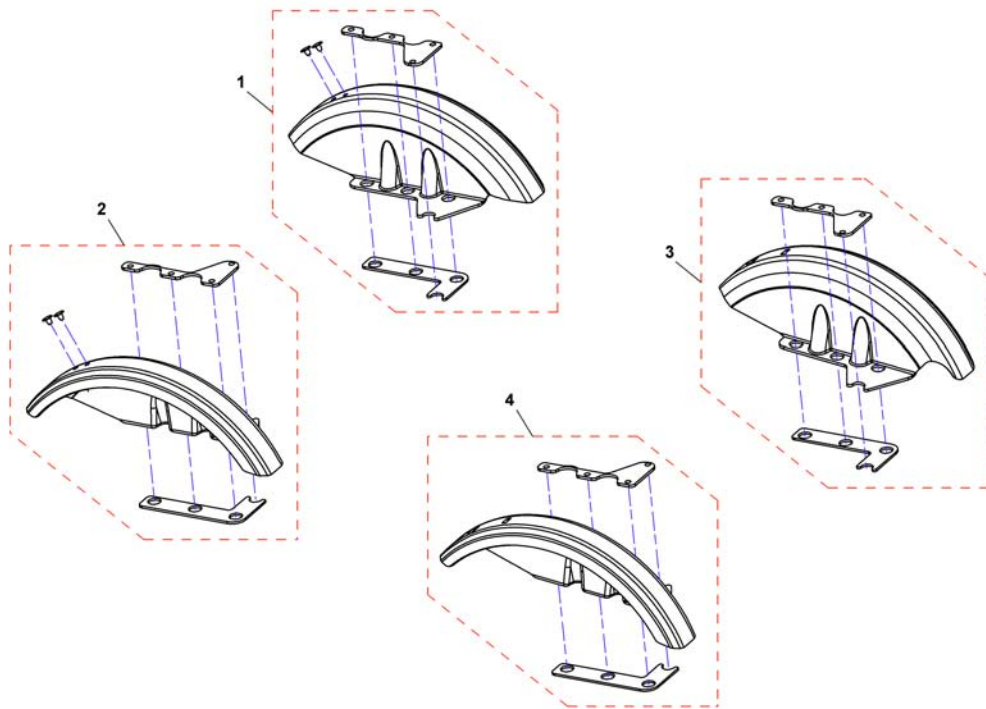


---

1	SP1536858	SKJERM BAK VE BORA U/LYS BORA	1
2	SP1536859	SKJERM HØ BAK BORA U/LYS BORA	1
3	SP1532919	HJULSKJERM VE BORA	1
4	SP1532918	HJULSKJERM HØ BORA	1



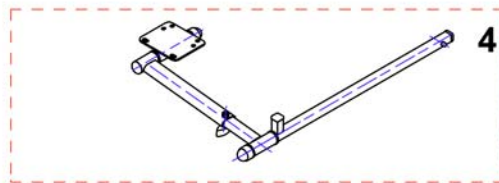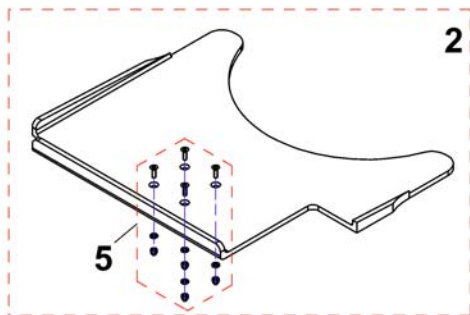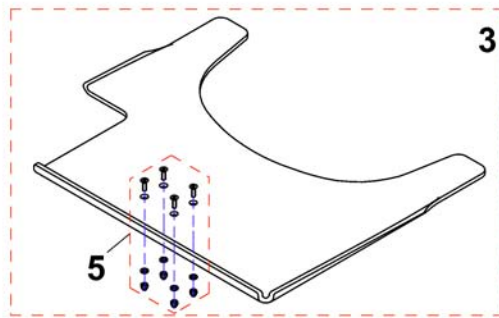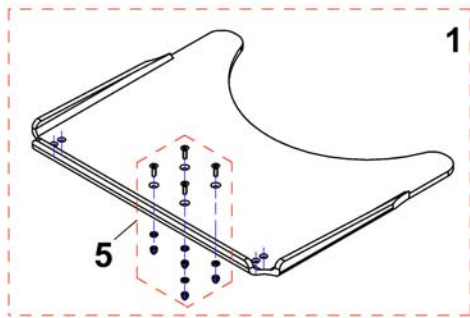


1	SP1535050	BORD 540MM SB38-48 UTEN UTSKJÆRING STORM4	1
1	SP1535010	BORD 620MM SB38-48 UTEN UTSKJÆRING STORM4	1
1	SP1535011	BORD 660MM SB44-53 UTEN UTSKJÆRING STORM4	1
2	SP1535009	BORD 540MM SB33-43 STORM4 UTSKJÆRING VE	1
2	SP1534974	BORD 620 MM SB38-48 UTSK VE STORM 4 UTEN FESTE	1
2	SP1534975	BORD 660MM SB38-48 UT VE STORM4 UTSKJÆRING VE	1
3	SP1535051	BORD 540MM SB33-43 STORM4 UTSKJÆRING HØ	1
3	SP1535012	BORD 620 MM SB38-48 UTSK HØ STORM 4 UTEN FESTE	1
3	SP1535049	BORD 660MM SB 44-53 UTSK HØ SIDE UTEN FESTE	1
4	SPZ177011	HOLDER FOR BORD SVINGBAR KPL STORM3/STORM4	1
5	Z177025	FESTE KIT BOR STORM4	1

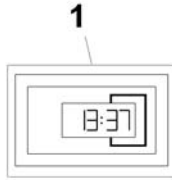

---

1 SP1530275 (Hour counter)

1

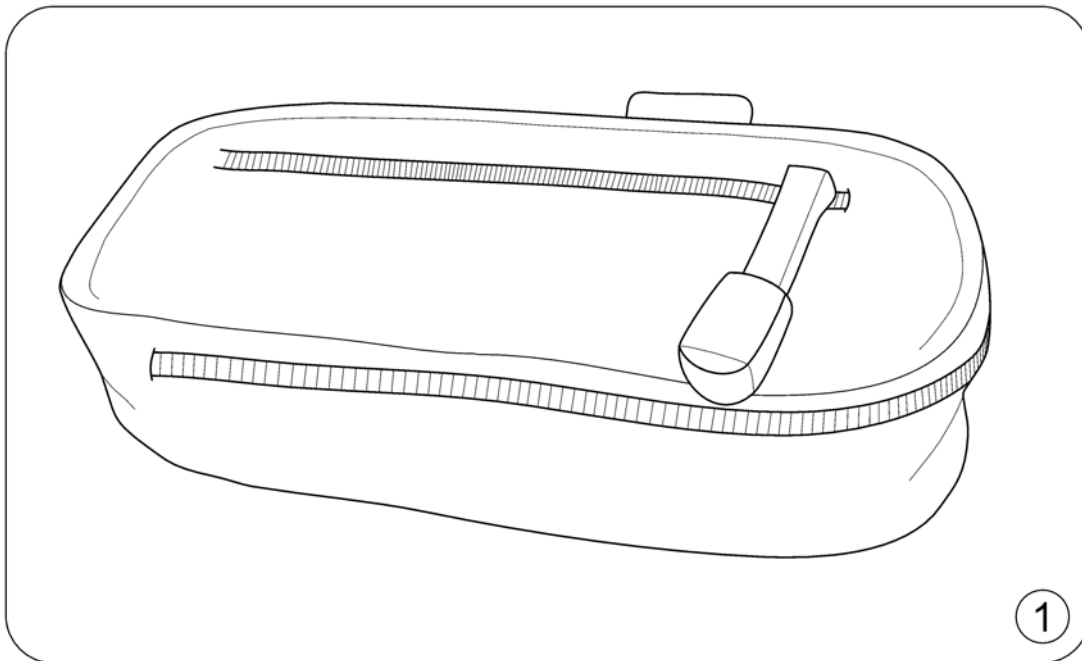


1 SP1557532

ARMLENSVESKE KLICK FIX STORM4 TDX KLICK FIX

1

**(KLICKfix® bracket)**


1571871 iso

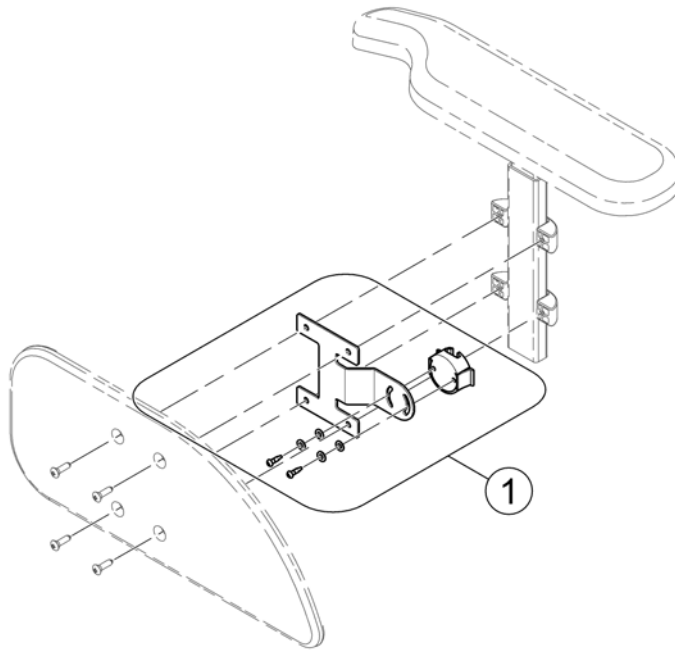

1 SP1571678

(KLICKfix® bracket)

1

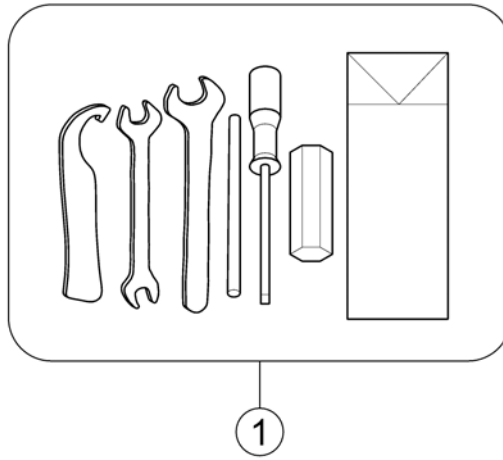

---


1 SP1422868

VERKTØY SETT

1

# (T-Nuts for Accessories Modulite Seat)

1574067.iso

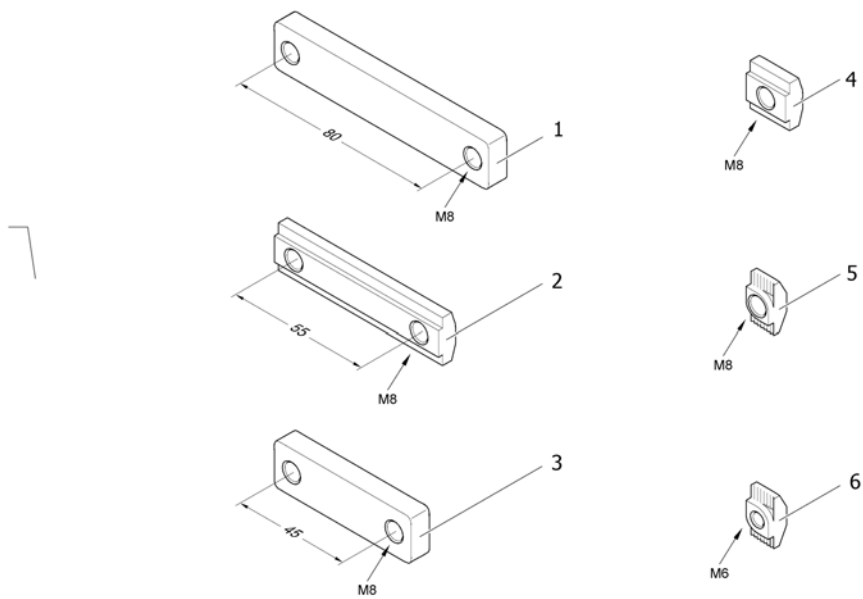

1	SP1556401	(T-Nut, hole distance 80mm, 2x M8)	1
2	SP1572444	(T-Nut, hole distance 55mm, 2x M8)	1
3	SP1555168	(T-Nut, hole distance 45mm, 2x M8)	1
4	SP1557341	(T-Nut, 1x M8)	1
5	SP1573158	(T-Nut, 1x M8)	1
6	SP1573721	(T-Nut, 1x M6)	1





Yes, you can.®



---

**Belgium & Luxemburg:**

Invacare nv, Autobaan 22, B-8210 Loppem  
Tel: (32) (0)50 83 10 10, Fax: (32) (0)50 83 10 11  
belgium@invacare.com / www.invacare.be

**Denmark:**

Invacare A/S, Sdr. Ringvej 37, DK-2605 Brøndby  
Tel: (45) (0)36 90 00 00, Fax: (45) (0)36 90 00 01  
denmark@invacare.com / www.invacare.dk

**Deutschland:**

Invacare Aquatec GmbH, Alemannenstraße 10, D-88316 Isny  
Tel: (49) (0)75 62 7 00 0, Fax: (49) (0)75 62 7 00 66  
info@invacare-aquatec.com / www.invacare-aquatec.de

**European Distributor Organisation:**

Invacare, Kleiststraße 49, D-32457 Porta Westfalica  
Tel: (49) (0)57 31 754 540, Fax: (49) (0)57 31 754 541  
edo@invacare.com / www.invacare.eu.com

**España:**

Invacare SA, c/Areny s/n, Polígon Industrial de Celrà, E-17460 Celrà (Girona)  
Tel: (34) (0)972 49 32 00, Fax: (34) (0)972 49 32 20  
contactsp@invacare.com / www.invacare.es

**France:**

Invacare Poirier SAS, Route de St Roch, F-37230 Fondettes  
Tel: (33) (0)2 47 62 64 66, Fax: (33) (0)2 47 42 12 24  
contactfr@invacare.com / www.invacare.fr

**Ireland:**

Invacare Ireland Ltd, Unit 5 Seatown Business Campus, Seatown Road, Swords, County Dublin - Ireland  
Tel: (353) 1 810 7084, Fax: (353) 1 810 7085  
ireland@invacare.com / www.invacare.ie

**Italia:**

Invacare Mecc San s.r.l., Via dei Pini 62, I-36016 Thiene (VI)  
Tel: (39) 0445 38 00 59, Fax: (39) 0445 38 00 34  
italia@invacare.com / www.invacare.it

**Nederland:**

Invacare BV, Celsiusstraat 46, NL-6716 BZ Ede  
Tel: (31) (0)318 695 757, Fax: (31) (0)318 695 758  
nederland@invacare.com, csede@invacare.com / www.invacare.nl

**Norge:**

Invacare AS, Grensesvingen 9, Postboks 6230, Etterstad, N-0603 Oslo  
Tel: (47) (0)22 57 95 00, Fax: (47) (0)22 57 95 01  
norway@invacare.com, island@invacare.com / www.invacare.no

**Österreich:**

Invacare Austria GmbH, Herzog Odilostrasse 101, A-5310 Mondsee  
Tel: (43) 6232 5535 0, Fax: (43) 6232 5535 4  
info-austria@invacare.com / www.invacare.at

**Portugal:**

Invacare Lda, Rua Estrada Velha, 949, P-4465-784 Leça do Balio  
Tel: (351) (0)225 1059 46/47, Fax: (351) (0)225 1057 39  
portugal@invacare.com / www.invacare.pt

**Sverige & Suomi:**

Invacare AB, Fagerstagatan 9, S-163 91 Spånga  
Tel: (46) (0)8 761 70 90, Fax: (46) (0)8 761 81 08  
sweden@invacare.com, finland@invacare.com / www.invacare.se

**Switzerland:**

Invacare AG, Benkenstrasse 260, CH-4108 Witterswil  
Tel: (41) (0)61 487 70 80, Fax: (41) (0)61 487 70 81  
switzerland@invacare.com / www.invacare.ch

**United Kingdom:**

Invacare Limited, Pencoed Technology Park, Pencoed, Bridgend CF35 5AQ  
Switchboard Tel: (44) (0)1656 776200, Fax: (44) (0)1656 776201  
Customer services Tel: (44) (0)1656 776222, Fax: (44) (0)1656 776220  
uk@invacare.com / www.invacare.co.uk