
Yes, you can.[®]

*Invacare[®] **Stretcher[™] sling***


***User's manual
Brugermanual
Brukerveiledning
Bruksanvisning***


Stretcher™ sling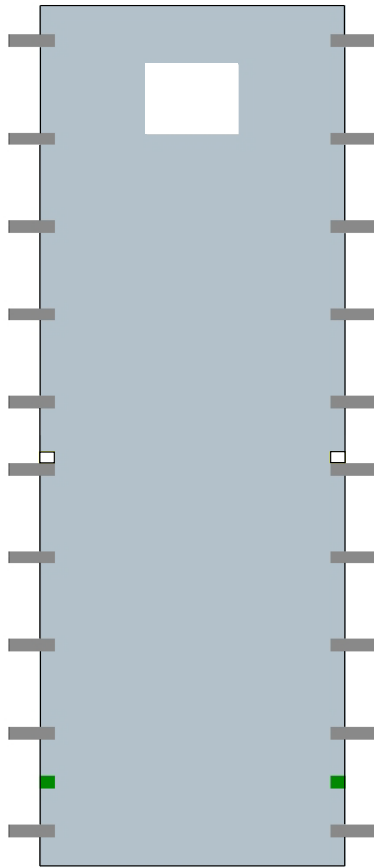


1.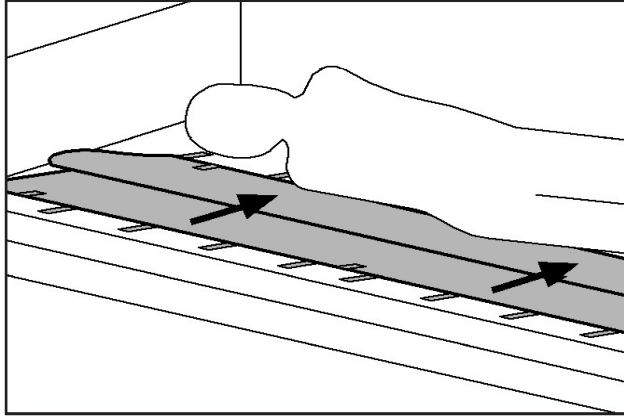


2.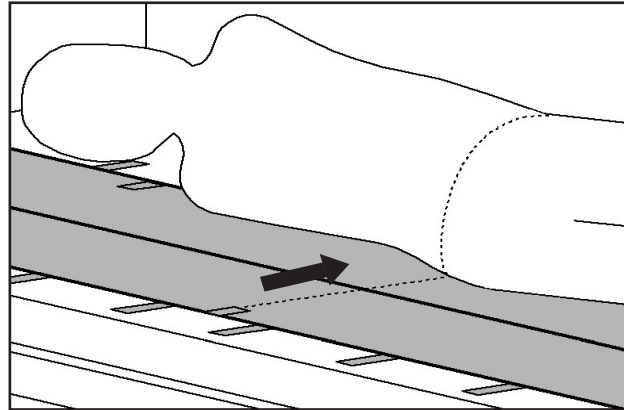

3.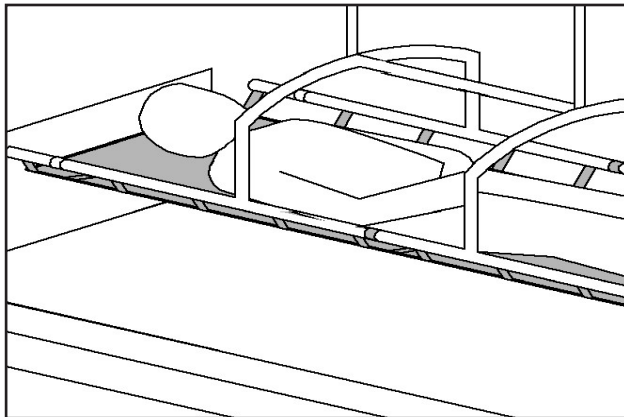

4.

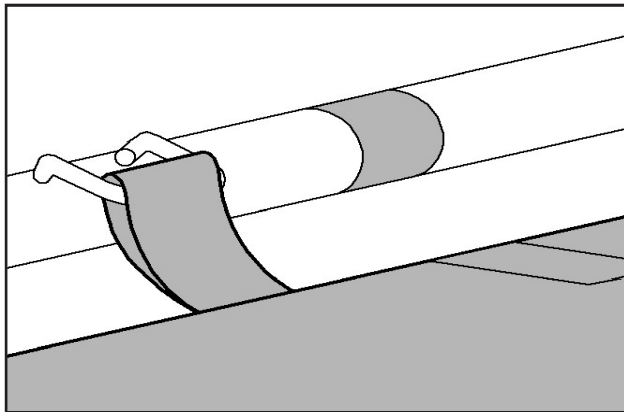Quality Declaration

Congratulations with your new hoist Invacare® **Stretcher™ sling** from Invacare® EC-Høng A/S.

Your new hoist is **CE**-marked in accordance with directive 93/42/EEC concerning medical devices.

Invacare® **Stretcher™ sling** is developed and constructed with consideration for the user and others handling or assisting with lifting.

Invacare® **Stretcher™ sling** is supervised and quality controlled throughout the entire production process, and the finished hoist is inspected by our finished goods control.

Please read the entire user's manual before using the hoist.

Invacare® EC-Høng A/S is certified according to ISO 9001 and ISO 13485.

For and on behalf of Invacare® EC-Høng A/S:

Technical Manager
Brian Mundeling

Invacare[®] Stretcher[™] sling

| | |
|-----------------------|-------|
| USER'S MANUAL (GB) | 8-9 |
| BRUGSANVISNING (DK) | 10-11 |
| BRUKERVEILEDNING (NO) | 12-13 |
| BRUKSANVISNING (SE) | 14-15 |

Stretcher™ sling

Thank you for purchasing an Invacare® Stretcher™ sling.

The following user guide must be read and adhered to, or the warranty may be void. Please check your sling on delivery – if the sling is not in perfect condition, please contact Invacare® on phone no. +45 3690 0000.

The Invacare® Stretcher™ sling provides maximum support to body and head. The sling must be applied in a lying position and must only be used for transfer in lying position.

The Stretcher™ is especially suitable for clients who have problems with back-, neck- or hip or somehow are in pain. The Invacare® Stretcher sling can be used together with the Stretcher™ for Robin.

Sling application – lying in bed or on floor

Roll the sling to mid-way and roll the client to one side to position the rolled up sling under the lifted side.

Picture 1 and 2.

Ensure that the middle of the sling is placed along the spine. Roll the client to the other side, unroll the sling and roll the client back into position. The client is now lying on the sling. **Picture 3.**

Note: To ensure a comfortable and secure lift it is important, that the client is in a horizontal position during the lift. The easiest way is to place the clients hip joint by the yellow mark on the sling and the feet against the label. If the clients position is not horizontal, the client must be moved on the sling:

- If the client is lying too high towards the head of the sling, place the yellow mark a little below the hip joint.
- If the client is lying too low towards the foot of the sling, place the yellow mark a little above the hip joint.

Lower the Stretcher™ over the client, and ensure that the green and the yellow mark on the stretcher are opposite the green and yellow marks on the sling.

Attach the loops. **Picture 4.**

Attach loops to the Stretcher™

Attach all loops on both sides of the Stretcher™ . Invacare® recommend a professional risk assessment before using the lift.

The Stretcher™ sling can only be used together with the Stretcher™ for the Robin hoist.

Only use the Stretcher™ for Robin™ and Robin Mover™ ceiling hoists.

Always check that all the straps are properly attached to the Stretcher™.

Invacare® recommend a professional risk assessment before using lifting equipment. Always choose design and size of the sling according to the client's weight, size and physical ability.

Slings should be checked regularly for wear and damage* — NEVER use a damaged sling.
(*In accordance with each country's regulations).

If you are in any doubt, please do not attempt to use the sling and seek the advice of a professional or your supplier.

Permissible max. load: 200 kg.

Symbols:  = 200 kg     

Safe Working Load, consult the Users Manual, do not exceed 95°C washing temperature, do not bleach, do not tumble dry, do not iron.

All Invacare® slings are guaranteed for one year.

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INVACARE Australia Pty Ltd
MOBITEC MOBILITÄTSHILFEN GmbH
INVACARE EC-Høng A/S