User Manual

DEALER: This manual MUST be given to the user of the product.

USER: BEFORE using this product, read this manual and save for future reference.

Yes, you can.

Drop Arm Commode Model H340

EN	User Manual	1
DE	Benutzerhandbuch	8
FR	Manuel d'utilisation	15
ES	Manual del usuario	23
PT	Manual de Utilização	31
NL	Gebruikershandleiding	38
IT	Manuale d'uso	45

I General

I.1 Symbols

Signal words are used in this manual and apply to hazards or unsafe practices which could result in personal injury or property damage. See the information below for definitions of the signal words.

WARNING

Warning indicates a potentially hazardous situation which, if not avoided, could result in death or serious injury.

CAUTION

Caution indicates a potentially hazardous situation which, if not avoided, may result in property damage or minor injury or both.

This product complies with Directive 93/42/EEC concerning medical devices.


Gives useful tips, recommendations and information for efficient, trouble-free use.

I.2 Life

The expected life of the product listed in this manual when used in accordance with the safety instructions, maintenance intervals and care instructions is five years. Maintenance and inspection should be done at least every six months.

I.3 Product Labeling

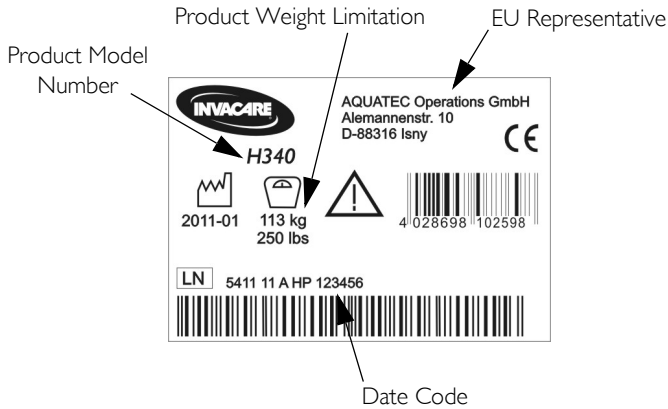I.4 Weight Limitation

WARNING

The Commode listed has a weight limitation of 250 lbs (113 kg). Consult a qualified technician if assistance is required in selecting the appropriate Invacare commode for you.

2 Safety

Check ALL parts for shipping damage. If shipping damage is noted, DO NOT use. Contact carrier/dealer for further instruction.

2.1 Intended Use

WARNING

This product is not intended for use in the shower.

This product is intended to be used solely as a commode. All other use can cause risk of injury.

2.2 General Warnings

WARNING

DO NOT USE THIS PRODUCT OR ANY AVAILABLE OPTIONAL EQUIPMENT WITHOUT FIRST COMPLETELY READING AND UNDERSTANDING THESE INSTRUCTIONS AND ANY ADDITIONAL INSTRUCTIONAL MATERIAL SUCH AS OWNER'S MANUALS, SERVICE MANUALS OR INSTRUCTION SHEETS SUPPLIED WITH THIS PRODUCT OR OPTIONAL EQUIPMENT. IF YOU ARE UNABLE TO UNDERSTAND THE WARNINGS, CAUTIONS OR INSTRUCTIONS, CONTACT A HEALTHCARE PROFESSIONAL, DEALER OR TECHNICAL PERSONNEL BEFORE ATTEMPTING TO USE THIS EQUIPMENT - OTHERWISE, INJURY OR DAMAGE MAY OCCUR.

ACCESSORIES WARNING

Invacare products are specifically designed and manufactured for use in conjunction with Invacare accessories. Accessories designed by other manufacturers have not been tested by Invacare and are not recommended for use with Invacare products.

WARNING

Snap buttons **MUST** be fully engaged and aligned in the proper height adjustment hole of each leg before use. This ensures that the legs are securely locked and an even height adjustment is achieved.

Make certain all screws, nuts and/or bolts are tight at all times.

Inspect rubber tips on the leg extensions for rips, wear or if they are missing. Replace them immediately if any of these conditions exist.

Leg extensions with rubber tips **MUST** be in contact with the floor at all times.

Toilet seat **MUST** be down before attempting to sit on the commode. Pail holders are not constructed to support the weight of an individual, only the pail and its contents.


Use the arm supports only for assistance. **DO NOT** attempt to use the arms to support full body weight.

Arms should be unlatched and dropped to assist in transfer onto/off the device.

Users with limited physical capabilities should be supervised or assisted when using the commode.

3 Usage

3.1 Installing the Pail

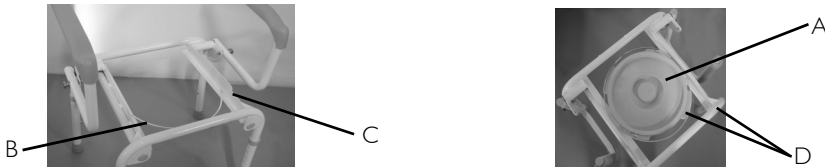1. Install the pail (A) onto the pail holder (B) in the commode frame (C).
2. Turn the pail so the tabs (D) keep the pail on the pailholder.

3.2 Installing/Removing the Commode Seat and Lid



CAUTION: Risk of Product Damage.

Cold seat clamps may break when removing/installed onto commode back frame rail.

- Before removing/installing seat and lid, allow seat/ring to reach room temperature if exposed to cold.


Always remove the lid before removing the seat.

Pull up on the back corner of the lid or seat when removing.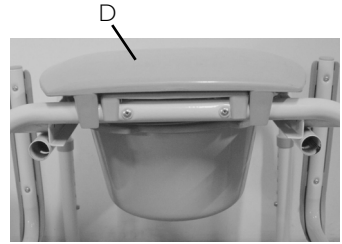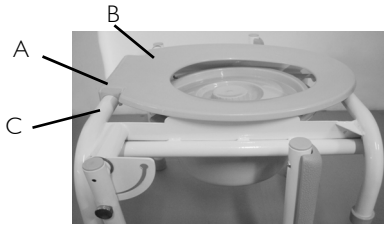



1. Align the seat clamps (A) of the toilet seat (B) onto the back frame rail (C).
2. Push down on the seat until it clamps firmly into place.
3. Repeat STEPS 1-2 to install the lid (D).
4. Reverse STEPS 1-3 to remove the lid and seat.

3.3 Installing the Backrest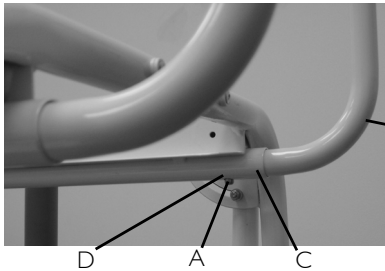


1. Press in and hold the snap buttons (A) on the backrest (B).
2. While holding the snap buttons, insert the backrest into the backrest tubes (C).
3. Push the backrest in until the snap buttons are fully engaged into the holes (D) in the backrest tubes. There will be an audible click.

3.4 Adjusting the Leg Height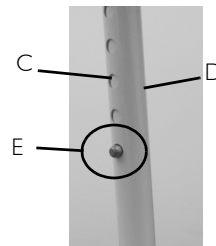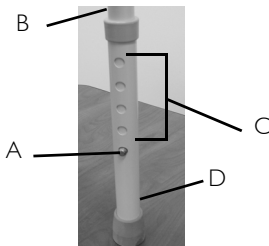

1. Align the snap button (A) of each leg frame (B) with the appropriate adjustment hole (C) in the leg extension (D).
2. Make certain that the snap buttons are fully engaged (E) into the adjustment hole of each leg extension. There will be an audible click.
3. Make sure all leg extensions are at the same height before using.

3.5 Operating the Drop Arms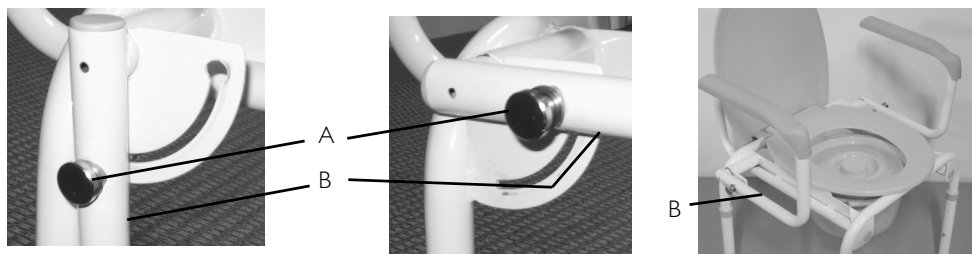

1. Press and hold the button (A) on the armrest (B).
2. While holding the button, move the armrest all the way up or down.
3. Let go of the button and make sure armrests are locked in position.

4 Care and Maintenance


4.1 Cleaning and Lubrication

Cleaning

CAUTION:

Risk of Product Damage.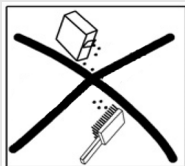

- DO NOT use an abrasive detergent, cleaner or cloth to clean or dry the commode.
- Regular cleaning and inspection is recommended.

1. Clean with a damp cloth and non-abrasive cleanser.

Lubrication

To make raising or lowering the seat/lid easier, you may lubricate the commode back frame rail with a very small amount of petroleum jelly or furniture polish.

5 After Use

5.1 Disposal

When the product can no longer be used, the product, accessories and the packaging materials should be sorted correctly. Please be environmentally responsible and recycle this product through your recycling facility at its end of life.

5.2 Reuse

WARNING: Risk of Injury

Always have a dealer inspect the product for damage before transferring the product to a different user. If any damage is found, DO NOT use the product. Otherwise injury may occur.

6 Limited Warranty - North America

PLEASE NOTE: THE WARRANTY BELOW HAS BEEN DRAFTED TO COMPLY WITH FEDERAL LAW APPLICABLE TO PRODUCTS MANUFACTURED AFTER JULY 4, 1975.

This warranty is extended only to the original purchaser/user of our products.

This warranty gives you specific legal rights and you may also have other legal rights which vary from state to state.

Invacare warrants this product to be free from defects in materials and workmanship for five years for the original purchaser. If within such warranty period any such product shall be proven to be defective, such product shall be repaired or replaced, at Invacare's option. This warranty does not include any labor or shipping charges incurred in replacement part installation or repair of any such product. Invacare's sole obligation and your exclusive remedy under this warranty shall be limited to such repair and/or replacement.

For warranty service, please contact the dealer from whom you purchased your Invacare product. In the event you do not receive satisfactory warranty service, please write directly to Invacare at the address on the back cover. Provide dealer's name, address, model number, the date of purchase, indicate nature of the defect and, if the product is serialized, indicate the serial number.

Invacare Corporation will issue a return authorization. The defective unit or parts must be returned for warranty inspection using the serial number, when applicable, as identification within thirty (30) days of return authorization date. DO NOT return products to our factory without our prior consent. C.O.D. shipments will be refused; please prepay shipping charges.

LIMITATIONS AND EXCLUSIONS: THE WARRANTY SHALL NOT APPLY TO PROBLEMS ARISING FROM NORMAL WEAR OR FAILURE TO ADHERE TO THE ENCLOSED INSTRUCTIONS. IN ADDITION, THE FOREGOING WARRANTY SHALL NOT APPLY TO SERIAL NUMBERED PRODUCTS IF THE SERIAL NUMBER HAS BEEN REMOVED OR DEFACED; PRODUCTS SUBJECTED TO NEGLIGENCE, ACCIDENT, IMPROPER OPERATION, MAINTENANCE OR STORAGE; OR PRODUCTS MODIFIED WITHOUT INVACARE'S EXPRESS WRITTEN CONSENT INCLUDING, BUT NOT LIMITED TO: MODIFICATION THROUGH THE USE OF UNAUTHORIZED PARTS OR ATTACHMENTS; PRODUCTS DAMAGED BY REASON OF REPAIRS MADE TO ANY COMPONENT WITHOUT THE SPECIFIC CONSENT OF INVACARE; PRODUCTS DAMAGED BY CIRCUMSTANCES BEYOND INVACARE'S CONTROL; PRODUCTS REPAIRED BY ANYONE OTHER THAN AN INVACARE DEALER, SUCH EVALUATION SHALL BE SOLELY DETERMINED BY INVACARE.

THE FOREGOING WARRANTY IS EXCLUSIVE AND IN LIEU OF ALL OTHER EXPRESS WARRANTIES, IF ANY, INCLUDING THE IMPLIED WARRANTIES OF MERCHANTABILITY AND FITNESS FOR A PARTICULAR PURPOSE. IT SHALL NOT EXTEND BEYOND THE DURATION OF THE EXPRESSED WARRANTY PROVIDED HEREIN AND THE REMEDY FOR VIOLATIONS OF ANY IMPLIED WARRANTY SHALL BE LIMITED TO REPAIR OR REPLACEMENT OF THE DEFECTIVE PRODUCT PURSUANT TO THE TERMS CONTAINED HEREIN. INVACARE SHALL NOT BE LIABLE FOR ANY CONSEQUENTIAL OR INCIDENTAL DAMAGES WHATSOEVER. THIS WARRANTY SHALL BE EXTENDED TO COMPLY WITH STATE/PROVINCIAL LAWS AND REQUIREMENTS.

7 Customer Service and Warranty Information

United Kingdom	Invacare Limited, Pencoed Technology Park, Pencoed, Bridgend CF35 5HZ Tel: (44) (0) 1656 776 222, Fax: (44) (0) 1656 776 220 www.invacare.co.uk, UK@invacare.com
Ireland	Invacare Ireland Ltd, Unit 5 Seatown Business Campus, Seatown Road, Swords, County Dublin - Ireland Tel: (353) 1 810 7084, Fax: (353) 1 810 7085 www.invacare.co.uk, ireland@invacare.com
France	Invacare France Opération, Route de Saint Roch, 37230 Fondettes Tel: (33) (0) 2 47 62 64 66, Fax: (33) (0) 2 47 42 12 24 www.invacare.fr, contactfr@invacare.com
Belgium & Luxembourg	Invacare nv, Autobaan 22, B-8210 Loppem, Brügge Tel: (32) (0) 50 83 10 10, Fax: (32) (0) 50 83 10 11 www.invacare.be, belgium@invacare.com
Netherlands	Invacare B. V., Celsiusstraat 46, NL-6716 BZ Ede, The Netherlands Tel: (31) (0) 318 69 57 57, Fax: (31) (0) 318 69 57 58 www.invacare.nl, nederland@invacare.com
Switzerland	Invacare AG, Benkenstrasse 260, CH-4108 Witterswil Tel: (41) (0) 61 487 70 80, Fax: (41) (0) 61 487 70 81 switzerland@invacare.com
Deutschland	Invacare Aquatec GmbH, Alemannenstraße 10, D-88316 Isny Tel: (49) (0) 75 62 7 00 0, Fax: (49) (0) 75 62 7 00 66 www.invacare-aquatec.de, info@invacare-aquatec.com
Portugal	Invacare Lda, Rua Estrada Velha, 949, P-4465-784, Leça do Balio Tel.: (351) (0)225 1059 46/47, Fax: (351) (0)225 1057 39 portugal@invacare.com
Italia	Invacare Mecc San s.r.l., Via dei Pini 62, I-3016 Thiene (VI) Tel: (39) 0445 38 00 59, Fax: (39) 0445 38 00 34 www.invacare.it, italia@invacare.com
España	Invacare SA, c/Areny s/n, Polígon Industrial de Celrà, E-17460 Celrà (Girona) Tel: (34) (0) 972 49 32 00, Fax: (34) (0) 972 49 32 20 www.invacare.es, contactsp@invacare.com
Canada	Invacare Canada, Inc., 570 Matheson Blvd. E. Unit 8, Mississauga, Ontario, L4Z 4G4 Tel: (905) 890-8300, Fax: (905) 890-5244 www.invacare.ca
USA	Invacare Corporation, One Invacare Way, Elyria, Ohio 44036-2125 Tel: (440) 329-6000, (800) 333-6900 www.invacare.com

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© 2011 Invacare Corporation. All rights reserved.
from Invacare. Trademarks are identified by TM and [®].
All trademarks are owned by or licensed to Invacare
Invacare Corporation www.invacare.com
Corporation or its subsidiaries unless otherwise noted.